AUTESTER-E 30 DRY
AUTESTER-E 75 DRY
AUTESTER-E 140 DRY

4001414
4001415
4001416

**AUTOCLAVES AUTOMÁTICAS PARA ESTERILIZACIÓN
A VAPOR CON SISTEMA PREVACUUM
AUTOMATIC AUTOCLAVES FOR STEAM STERILIZATION**

INFORMACIÓN GENERAL

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GENERAL INFORMATION

- 1) *Handle the parcel with care. Unpack and check that the contents coincide with the packing-list. If any part is damaged or missing, please advise the distributor immediately.*
- 2) *Do not install or use the equipment without reading this handbook before.*
- 3) *This handbook must always be attached to the equipment and it must be available for all users.*
- 4) *If you have any doubts or enquiries, please contact with your supplier or J.P. Selecta's technical service.*
- 5) **IMPORTANT! J.P. SELECTA WILL NOT ACCEPT ANY APPARATUS TO BE REPAIRED IF IT IS NOT DULY CLEANED.**
- 6) *If any modification, elimination or lacking in maintenance of any device of the equipment by the user transgress the directive 89/655/CEE, the manufacturer is not responsible for the damage that can occur.*
- 7) *Do not use the apparatus with liquids which can give off vapours capable of making explosive mixtures.*

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CONTENTS

GENERAL INFORMATION
CONTENTS
PACKING LISTS
ACCESSORIES
TECHNICAL FEATURES
EQUIPMENT DESCRIPTION
INSTALLATION
OPERATION
MAINTENANCE
GUARANTEE
ELECTRICAL DIAGRAM
SPARE PARTS
"EC" CONFORMITY DECLARATION

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The standard equipment consist of the following components:

Descripción	Description	Código/Code		
Autester-E - DRY	Autester-E - DRY	4001414	4001415	4001416
Cubre resistencia	Shelf	5601	5600	5603
Porta gomas	Carry Rubbers	34100		
Manual de instrucciones	Instruction manual	80121		

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* **Thermal printer** with a continuous paper roll indicating temperature, pressure, time and operation mode. **Code 4001220.**

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* **Temperature probe Pt100** for reading the internal temperature of the chamber. A terminal permits the Pt100 to be connected to a printer or temperature measuring device (which are not subministrated with the probe). **Part No. 4001219.**

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* **Perforated AISI 316 stainless steel baskets:**

Código cestillo / Basket code	Medidas cm. / Dimensions cm.		Código autoclave / For autoclave code	Capacidad cestillos / Number of baskets
	Ø	Alto / Depth		
1000495	28	20	4001414	2
1000496	38	28	4001415	2
1000780	48	22	4001416	3

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Voltage supply 115-230V 50/60 Hz according to the characteristics plate indications.

Código / Code	Secado / Drying	Capacidad l. / Capacity l.	Medidas útiles cm. / Chamber dimensions cm.		Medidas exteriores cm. / Overall dimensions cm.			Rango °C / Range °C	Consumo W. / Consumption W.	Consumo secado W. / Drying consumption W.	Peso Kg. / Weight Kg.
			Ø	Alto / Height	Alto / Height	Ancho / Length	Fondo / Width				
4001414	SI / YES	30	30	40	100	59	48	105°C a / to 134°C	2800	1350	87
4001415		75	40	60	109	70	59		4300	2250	147
4000779		140	50	70	129	89	82		7500	4750	249

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EQUIPMENT DESCRIPTION

The Autester-E are a family of **saturated with steam vapour** autoclaves ruled by a microcontroller with digital reading of the temperature which checks:

1. Prevacuum function: Evacuation of the air in the chamber at the beginning of the sterilisation cycle.
2. Safety door closed indicator.
3. Automatic water filled of the chamber.
4. Temperature, pressure and time control.

EQUIPMENT:

- * Safety system:
SAFETY VALVE.
SAFETY PRESSURE SWITCH. SAFETY
THERMOSTAT.
SAFETY DOOR CLOSED INDICATOR.
THERMAL PROTECTION
- * Boiler, tank and cover in 18/8/2 stainless steel.
- * Outer casing in stainless steel.
- * Inner drainage filter.
- * Air filter.
- * Manual steam drainage valve.
- * Select filling valve (external / tank).
- * Drain valve selector (external / tank).
- * Condensed output nipple (vacuum pump).
- * Tap water nipple.
- * Output water nipple of external tank.

FEATURES:

Sterilization temperature from 105°C to 134°C.
Sterilization time from 1 minute up to 99 minutes.
Drying time up to 60 minutes.

Three types of function:

-SOLIDS
-LIQUIDS
-SOLIDS+DRYING

There are 10 functioning programmes.

The programmes 1 to 9 can be recorded depending on the user needs.

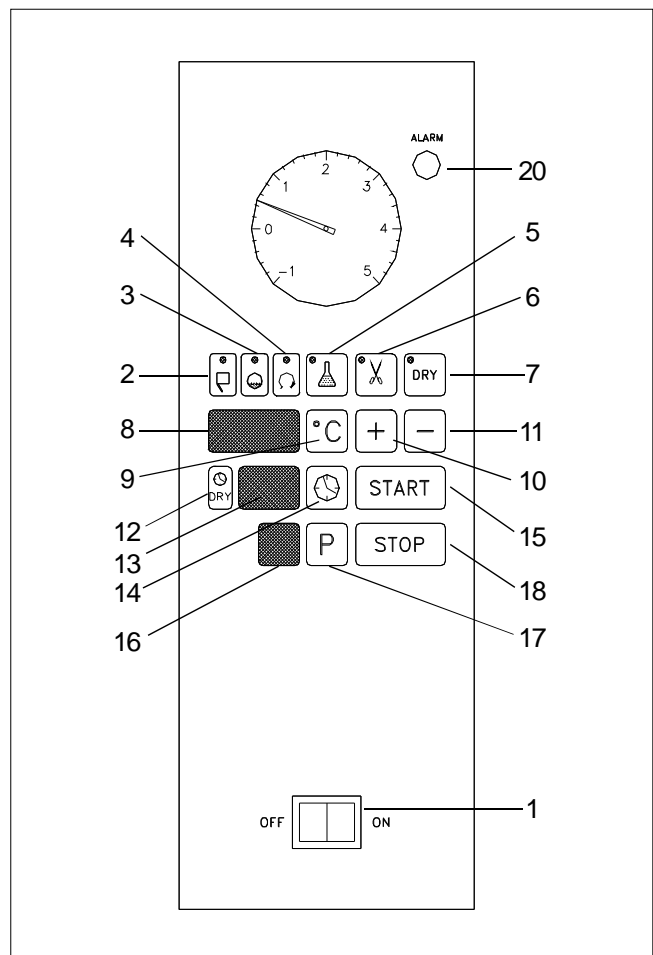
The programme 0 is free and cannot be saved.

When starting the machine for the first time, the default values of the different parameters are the ones indicated in the following table:

Programa nº Program nº	Modo Mode	Temperatura / Presión Temperature / pressure	Tiempo de esterilización Sterilization time	Tiempo de secado Drying time
1	Solid / Solid	121°C / 1bar	30'	-
2	Solid / Solid	126°C / 1.33 bar	20'	-
3	Solid / Solid	134C / 2 bar	18'	-
4	Solid / Solid	134°C / 2 bar	10'	-
5	Líquido / Liquid	121°C / 1bar	30'	-
6	Sólido+Secado / Solid+Dry	121°C / 1bar	30'	35'
7	Sólido+Secado / Solid+Dry	126°C / 1.33 bar	20'	35'
8	Sólido+Secado / Solid+Dry	134°C / 2 bar	18'	35'
9	Sólido+Secado / Solid+Dry	134°C / 2 bar	10'	35'
0	Libre / Free			

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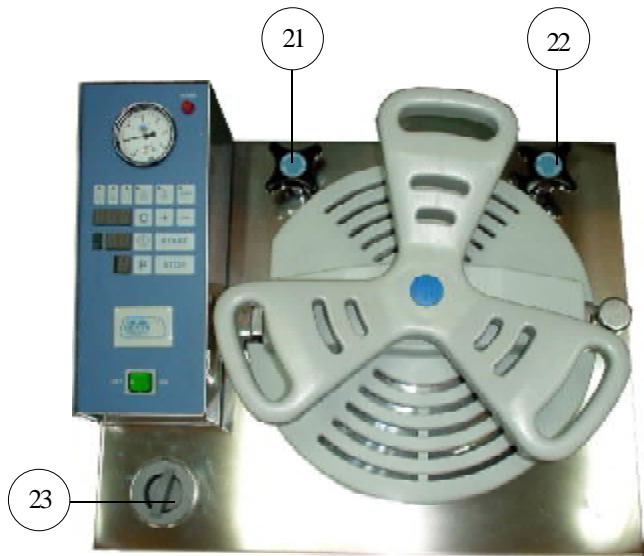
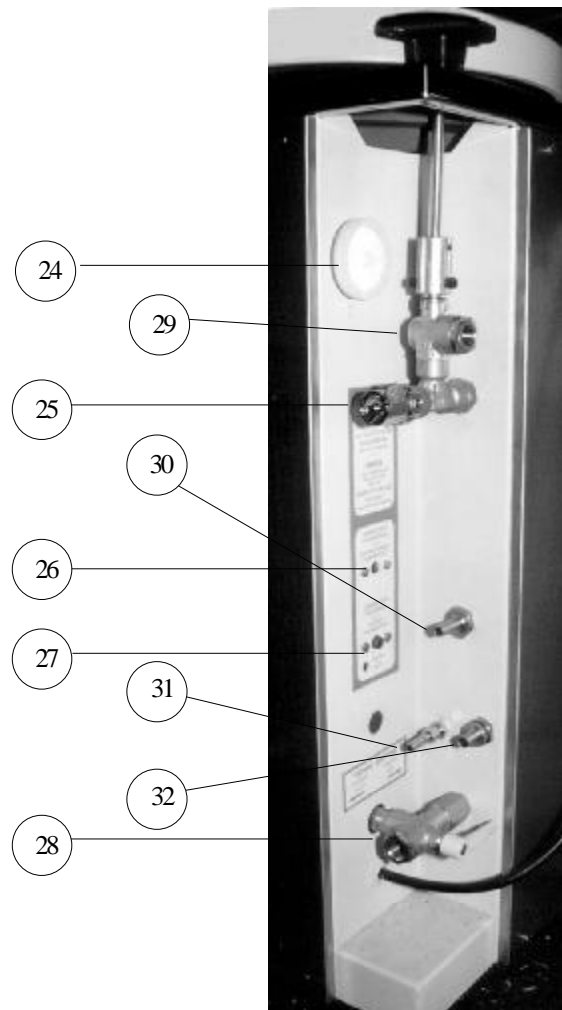
INSTALLATION

- 1) Place the autoclave near the mains connection according to the machine consumption.
- 2) The autoclave should be placed on a flat, horizontal and stable surface, leaving a space of about 10 cm at back and on each sides.
- 3) Using the stops provided in the lower front part of the apparatus, the Autoclave is immobilised and levelled with the floor.

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- 4) CHOOSE THE DRAINING SYSTEM with the drainage selector valve (28). **If you wish to recover the water**, point the knob towards the inside.

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If you wish to evacuate the water, at the end of the sterilization process, point the knob towards the outside. Fix a tube in the water/steam exit nipple of the machine, and also fix the other end to a drain or deposit without obstructing the flow. **NEVER PRESS THIS TUBE BY HAND.**

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Do not place the knob valve in its central position, because the evacuation process will be cancelled.

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5) Select the filling mode of the autoclave with the valve (21).

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TANK mode: The autoclave fills with the water from the internal tank.

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The fill of the internal tank can be done with the water reservoir hole (23) until the water reaches the level of the filling tube.

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With the tank drainage (32), the internal tank can be emptied. Place a hose and uncoil it.

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External mode: The autoclave fills from an external mains from the water inlet (30). **The pressure should never be greater than 2 bar.**

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Do not place the knob of the filling valve in its central position, because the filling process will be cancelled.

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Always that the autoclave is filled from an external main, is necessary to empty the autoclave to the external main.

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7) *Place a tube on the condensation outlet (31) to get the condensations produced during the phases were the drying pump works. Do not place the recipient in a higher level than the outlet.*

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8) *The tank of autoclave can empty if the working pressure is between 1 and 2 bar. For lower pressures, the tank will not empty totally.*

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
9) **Drying phase cannot be done when there is water in tank.**

10) **AUTOCLAVE WITH CHART RECORDER**

- A Place the chart recorder on the left side of the control panel and fasten it with two screws, supplied for that purpose.
- B Plug the 5 pin connector in the socket located at the rear of the control panel.
- C Make sure that the chart paper is properly loaded. Follow the included chart recorder instructions.
- D When the autoclave is started with the switch (1), the chart recorder starts automatically.

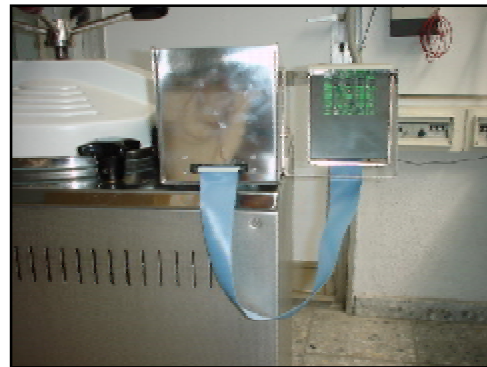If the printer clock configuration has to be modified, do as follows:

1. Select program 0
2. Press simultaneously the temperature button (9) and the time one (14) for at least 10 seconds and the indicator lamps of DOOR (2), WATER (3) and END (4) will lit up.
The display (8) will show: C-1.

A) Year configuration (C-1):
The display (13) will show two numbers that are the last numbers of the year.
Press (10) or (11) to select the desired numbers of the year.
Confirm them with STOP (18).
The display (8) will show: C-2.

B) Month configuration (C-2):
The display (13) will show two numbers that are: 01 January, 02 February up to 12 December.
Press (10) or (11) to select the desired month number.
Confirm them with STOP (18).
The display (8) will show: C-3.

C) Day of the month configuration (C-3):
The display (13) will show two numbers that are the day of the month.
Press (10) or (11) to select the day of the month.
Confirm them with STOP (18).
The display (8) will show: C-4

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D) Day of the week configuration (C-4):

The display (13) will show a number that is:

0= Sunday	4= Thursday
1= Monday	5= Friday
2= Tuesday	6= Saturday
3= Wednesday	

Press (10) or (11) to select the desired day of the week.

Confirm it with STOP (18).

The display (8) will show: C-5.

B) Hour configuration (C-5):

The display (13) will show two numbers that are: 00h up to 23h.

Press (10) or (11) to select the desired hour.

Confirm it with STOP (18).

The display (8) will show: C-6.

C) Minutes configuration (C-6):

The display (13) will show two numbers that are: 00 up to 59 minutes.

Press (10) or (11) to select the minutes.

Confirm them with STOP (18).

The indicator lamps of DOOR (2), WATER (3) and END (4) will lit off and come back to program 0.

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CAUTION! The steam drainage valve (29) and the drainage valve (28) are a PRESSURE and HIGH TEMPERATURE. Require the necessary precautions to avoid SERIOUS BURNS.

NEVER PRESS THE TUBE PLACED ON THE DRAINAGE VALVE (28) BY HAND. SERIOUS BURNS CAN HAPPEN.

Do not install the autoclave in zones where INFLAMMABLE LIQUIDS ARE STORED OR ZONES WITH SPECIAL PROTECTION.

Before opening the cover ALWAYS check that there is NO PRESSURE. OPEN STEAM VALVE. The pressure gauge indicator installed in the control panel must read 0 bar.

Do not use the apparatus if it is not earthed.

Be sure that the voltage supply is the same as the one indicated on the characteristics plate.

If you change the plug bear in mind the following:

Blue cable:	Neutral
Brown cable:	Phase
Yellow/green cable:	Earth

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OPERATION

STARTING UP:

- 1- Start up with mains switch (1).
- 2- Check that the steam drainage valve (22) is closed.
- 3- Put in the material to be sterilized.
- 4- Close the cover.
- 5- Programming, this Autoclave permits two different operation systems: MANUAL or AUTOMATIC. Been the number 0 the manual program. Been the number 1 up to 9 the automatic programs.

FOR THE MANUAL PROGRAMMING of functioning mode, temperature, sterilization time and drying time mode; do as follows:

- 1º Put the programme display at 0 by pressing the programme button (17).
- 2º To choose the mode of working press the buttons 5, 6 and 7 depending if you wish liquids, solids or solids+drying.

LIQUIDS: At the end of the sterilization cycle, the Autoclave stops and the temperature gradually goes down and therefore the pressure. Thus avoiding the spilling of liquids.

SOLIDS: At the end of the sterilization cycle, the Autoclave stops and a rapid decompression takes place, emptying the liquid of the tank only if the working pressure is the same or higher than 1 bar.

DRY: At the end of the solids system the Autoclave is emptied to dry the inside for a period of 60 minutes.

- 3º To choose the temperature, press the button (9) and buttons (10) or (11) simultaneously to increase or decrease the sterilization temperature. (Programmable value between 105°C and 134°C).
- 4º To choose the sterilization time, simultaneously press button (11) and buttons (14) or (11) to increase or decrease the sterilization time. (Selected value between 1' and 99').
- 5º To choose the drying time, simultaneously press button (14) two times (automatically the indicator lamp (12) lits up) and buttons (10) or (11) to increase or decrease the drying time. (Selected value between 1' and 99').

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SAFETY PRECAUTION BEFORE PRESSING START (16):

- * Check that the time is different from 0.
- * Check that the temperature is below 100°C.
- * Check that there are no ERRORS.

If there is not water in the autoclave tank, the indicator lamp (3) will light on and will remain on until the tank is filled (the filling will be done automatically).

6º Press the START button (15) and the indicator lamp of the selected mode (5 or 6 or 7) will light up intermittent.

7º From this moment, the buttons (10), (11) and (17) buttons are inactive.

8º The selected temperature (8) and time (13) can be observed at any time by pressing the corresponding button (9) and (14).

9º When the sterilization time is reached, the indicator (4) for cycle end will light up and the acoustic signal will sound.

10º When the cycle is finished always press the STOP button (18).

11º Check that there is no pressure and open THE STEAM VALVE and open the cover.

FOR AUTOMATIC PROGRAMME you must choose any programme except 0 by pressing the button (17) and then pressing the START button (15).

To modify the default programs, do as follows:

Go to the program X, pressing the program button (17) for at least 10 seconds will start the programming menu and the DOOR (2), WATER (3) and END (4) indicator lamps will lit up:

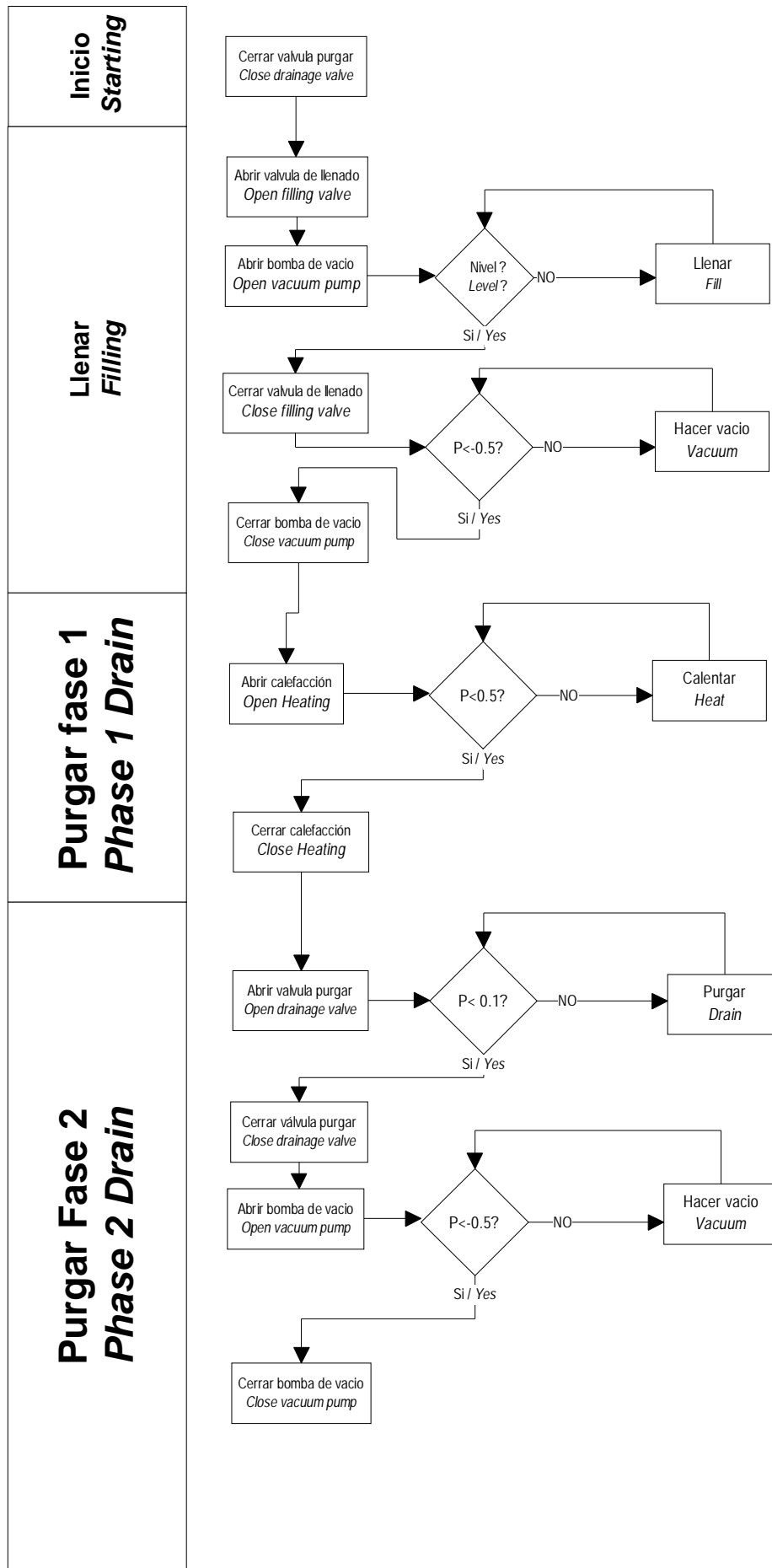
1. Once choosed the programming menu, you can start programming.

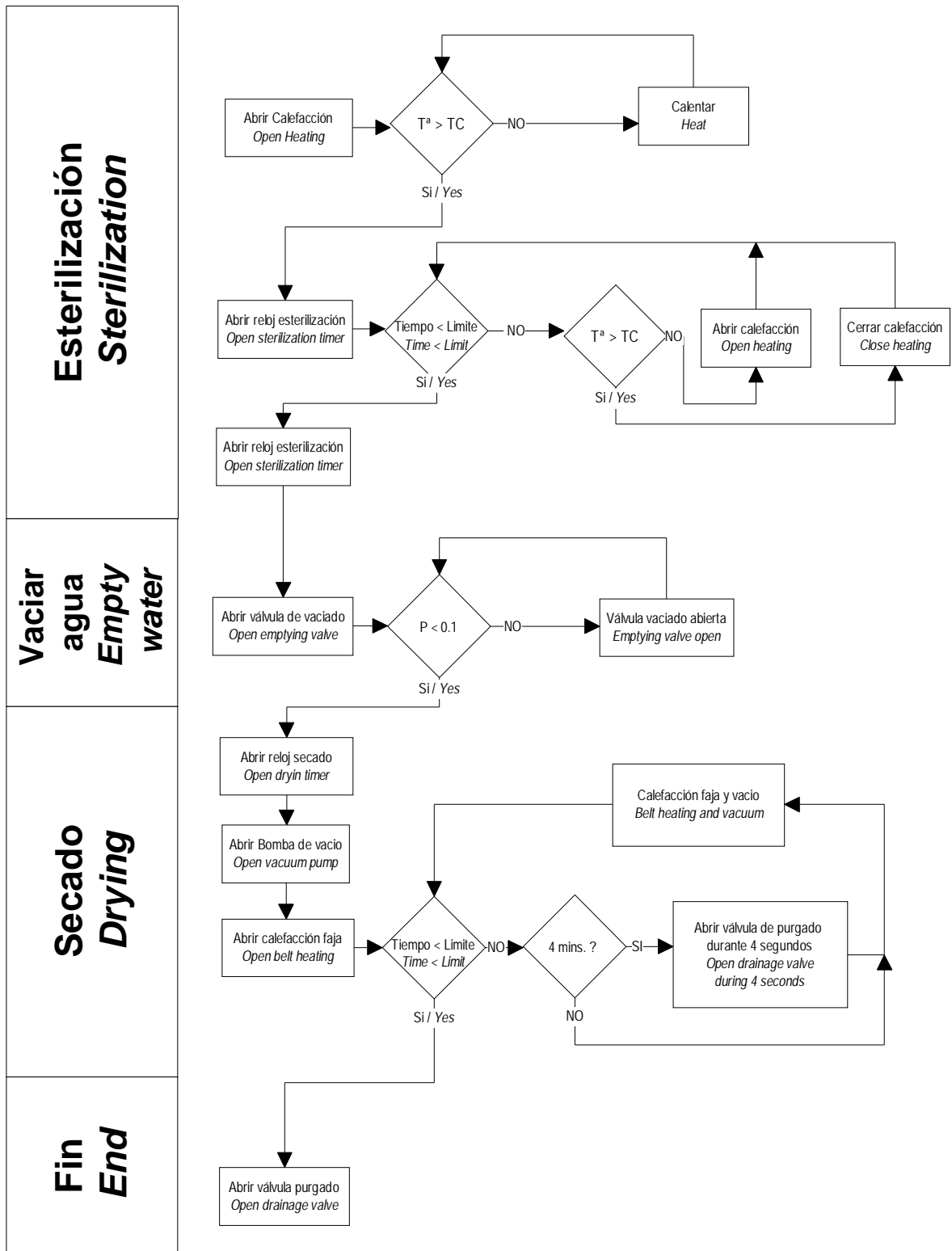
2. To change the program, temperature or time do as usual. And to change the type, temperature, time and drying time following the indications of the points 2, 3, 4 and 5.

3. To save the changes press the STOP button. If for 10 seconds any button is pressed, the menu will go out without saving any changes.

4. The indicator lamps of DOOR (2), WATER (3) and END (4) will lit off.

The default programs will be those determined by J.P. Selecta, s.a.

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PRACTICAL RECOMMENDATIONS FOR A PERFECT STERILIZATION:

1. *The material to be sterilized has to be perfectly clean, free of any incrustations and impurities, so cleaning with a good detergent and distilled water and rinsing with abundant water is recommended.*
2. *Do not overload the tray, baskets or racks, and leave a space between them of 1 or 2 cms to facilitate the steam circulation and the drying process.*
3. *Sterilization of «UNWRAPPED INSTRUMENTS».*
 - 3.1 *Place the instruments directly onto water-repellent paper, avoiding any piling up of one upon another.*
 - 3.2 *Avoid placing instruments of different metal type in the same tray.*
4. *Sterilization of «INSTRUMENTS IN BAGS».*
 - 4.1 *Put one instrument per bag.*
 - 4.2 *Place bags onto water-repellent paper, in vertical position avoiding any piling up of one upon another or the autoclave walls.*
5. *Tubes.*
 - 5.1 *The tubes have to be clean and dried.*
 - 5.2 *Place tubes, if it is possible, without obstructing the both ends.*
 - 5.3 *Take care that the tubes do not touch the autoclave chamber and avoid folding them because the inner diameter can be obstructed.*
6. *Receptacles.*
 - 6.1 *NEVER put the receptacles hermetically closed.*
 - 6.2 *Place the receptacle face down to avoid the water residues.*
7. *Liquids.*
 - 7.1 *Place the liquid to be sterilized in receptacles capable of supporting the sterilization temperature; and over the shelves so you can collect the leakages that can occur.
If you are sterilizing AGAR, don't place it at temperature in boiling phase, because in the first emptying phase it would spill.*

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7.2 The receptacles will be filled to about 2/3 of their capacity. They will NEVER be closed hermetically, they will cover with cotton or some type of tap that makes the air easy outlet and it does not make pressure.

7.3 Please avoid using a receptacle with a narrow neck.

7.4 At the end of the sterilization cycle, leave the autoclave to cool slowly (until the pressure is 0 Kg/cm²).

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SAFETY:

Safety Pressure Controller:

When abnormal overpressure occurs in the sterilization chamber the safety pressure controller acts by stopping the heating and opening the drain electrovalve. The ALARM indicator lamp (20) lights up simultaneously and error 12 appears on the display.

Safety valve:

If abnormal overpressure occurred the safety valve (25) would open and expel the steam.

Said valve is gauged to initiate the opening between 2.2 kg/cm² and 2.5 kg/cm².

Safety Thermostat:

When abnormal increase of the temperature or a considerable decrease in the water level occurs (with the consequent overheating of the heater element), the safety thermostat (27) starts working blocking the heating of the machine, opening the drain electrovalve and the ALARM indicator lamp (20) lights up simultaneously and error 12 appears on the display.

To deblock the machine proceed as follows:

- 1º Stop the machine by pressing the button STOP (18) and by putting the switch in OFF position.*
- 2º Release any pressure with valve (22).*
- 3º Open the cover.*
- 4º Fill the tank with water up to the working level.*
- 5º Reset the safety thermostat (27) by pressing its reset button. The red indicator lamp ALARM (20) will switch off.*

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POSSIBLE ERRORS:

Some erroneous conditions could be the reason for the faulty operation of the Autoclave. The errors can be detected by the microcontroller which also checks that the process is correct.

ERROR 1

Filling phase takes longer than usual.

CAUSES:

- * *Minimum level in lower tank.*
- * *Electric filling valve not on or broken.*
- * *Filling selecting valve in central position.*
- * *Short-circuit in the float level of filling.*
- * *Float level of filling blocked at the top.*
- * *Failure in the control panel.*

ERROR 2

It takes longer to reach the desired temperature than usual.

CAUSES:

- * *Cut in the heating elements.*
- * *Failure in the solid-relay.*
- * *Failure in the control panel.*

ERROR 3

Drainage phase takes longer than usual.

CAUSES:

- * *Obstructed draining filter.*
- * *Draining electrovalve not switched on or broken.*
- * *Draining selecting valve in central position.*
- * *Failure in the control panel.*

ERROR 4

Open probe.

CAUSES:

- * *Broken probe.*
- * *Connections failure.*

ERROR 5

Temperature above 136°C

CAUSES:

- * *Short-circuit in the float level of filling.*
- * *Loss of liquid level.*
- * *Probe failure.*

ERROR 6

Pressure above 2.4 bar.

CAUSES:

- * *Temperature probe failure.*
- * *Control circuit failure.*

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ERROR 7

Temperature 2°C below the set temperature.

CAUSES:

- * Loss of steam though:
 - Drainage valve.
 - Filling valve.
 - Vacuum valve.
- * Due to bad closure or remaining switched on.
- * Failure of control circuit.

ERROR 8

Safety relay activated.

CAUSES:

- * Electricity cut.
- * Safety thermostat activated.
- * Safety pressure switch activated.
- * Failure in control circuit.

MAINTENANCE

Before removing the side covers disconnect the apparatus from the mains.

The control panel must only be manipulated by authorized personnel.

CLEANING:

For the cleaning of the different parts use the following products:

- Cleaning of stainless steel: Alcohol.
- Cleaning of plastic: Alcohol with cotton duster.

Cleaning of pipes with lime: Solution of 50% of Hydrochloric acid and 50% of water.

Clean the emptying filter of the tank, placed at the lower part of the control panel.

It is advisable to change the water after 50 uses or once a month.

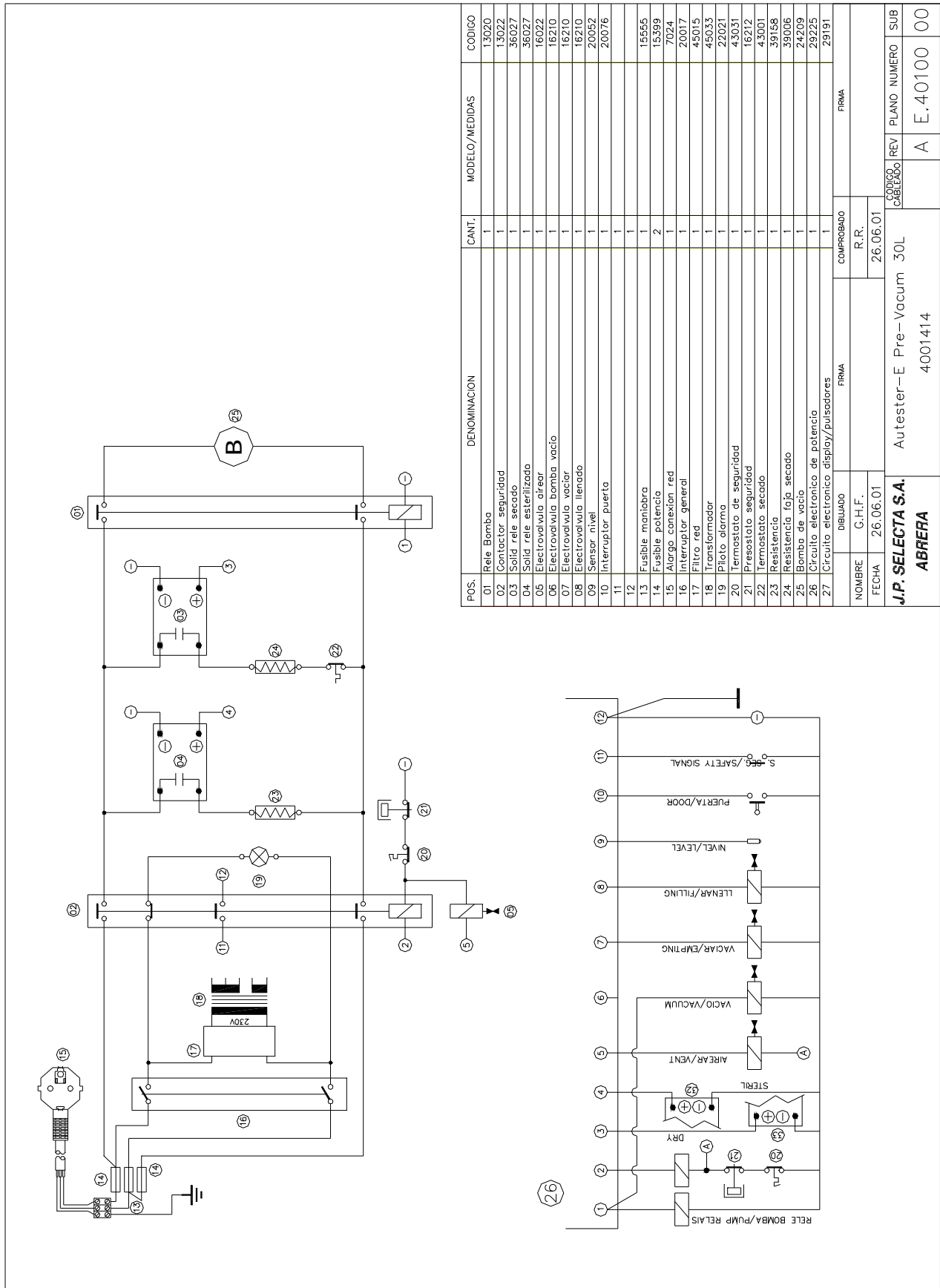
GUARANTEE

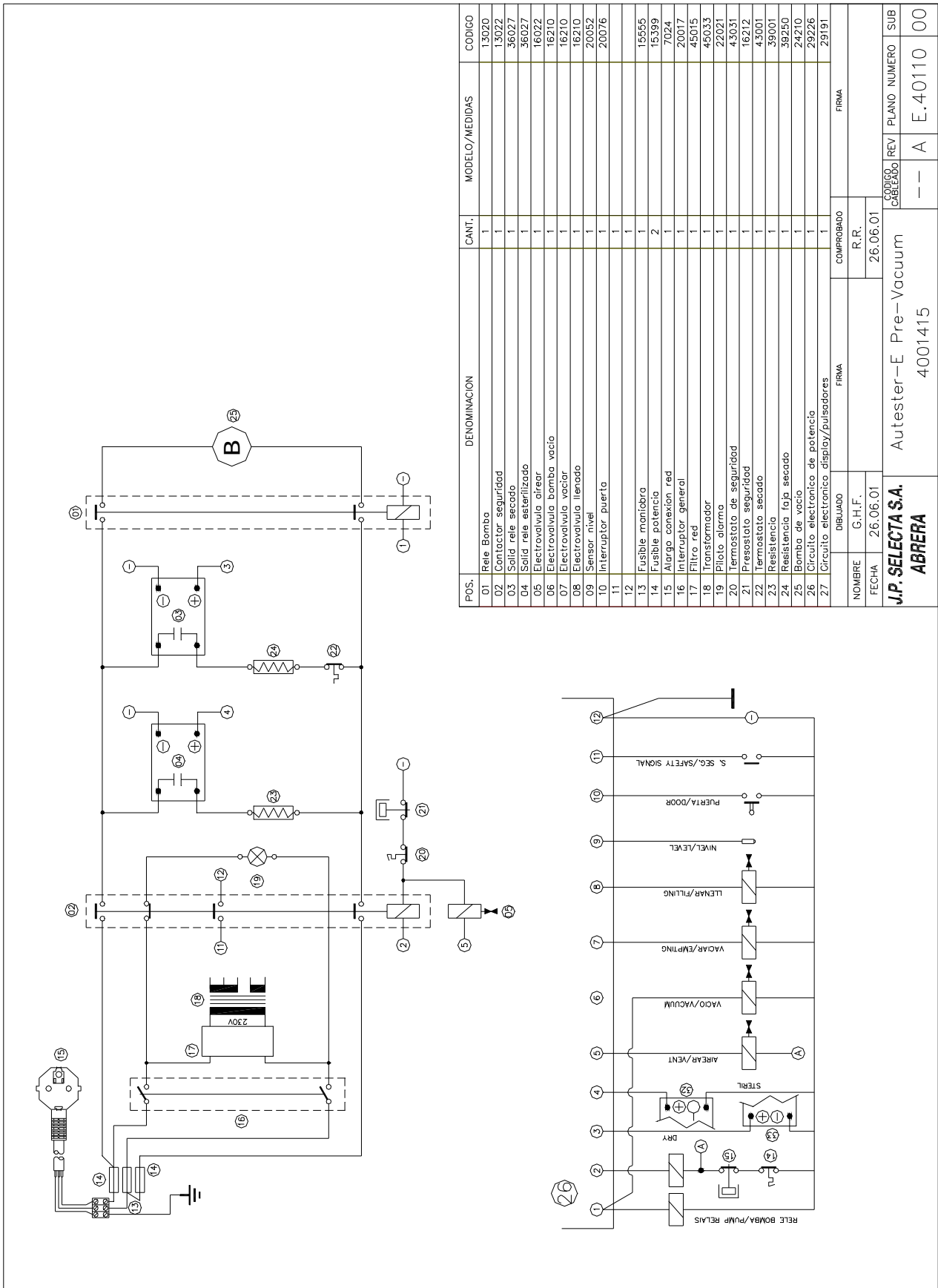
This product is guaranteed for one year. The guarantee does not cover damage caused by incorrect use or causes beyond the control of J.P. SELECTA, S.A..

Any manipulation of the apparatus by personnel not authorized by J.P. SELECTA, S.A. cancels the guarantee automatically.

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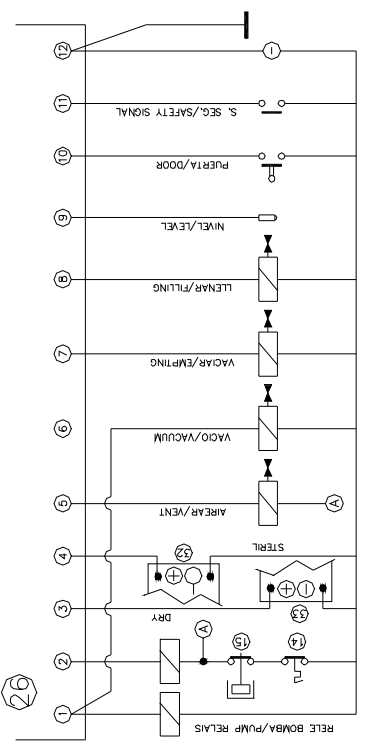
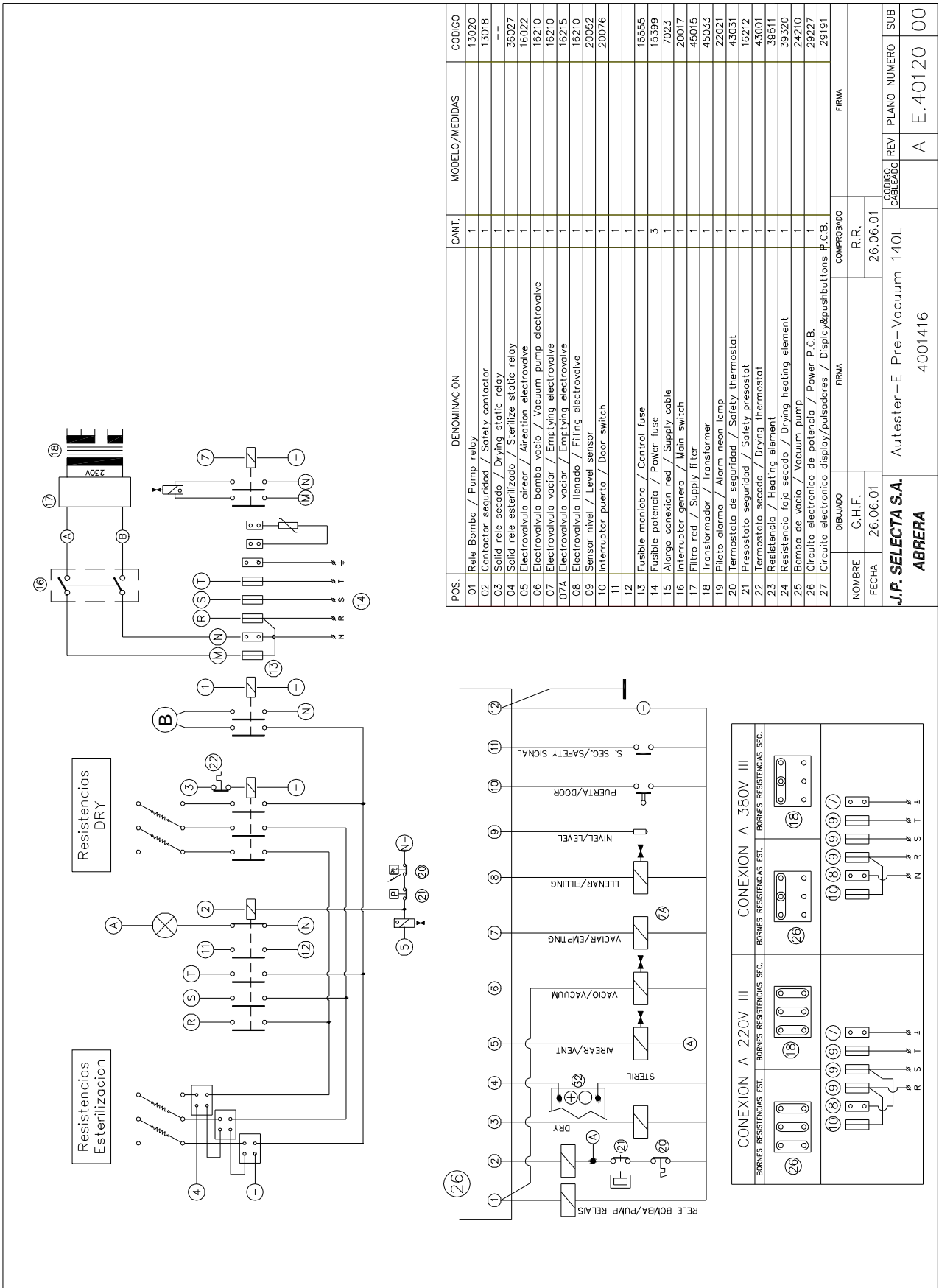
To guarantee the safety of the equipment, the spare parts must be purchased from J.P.SELECTA, S.A.

		30 LITROS	75 LITROS	140 LITROS
Descripción	Description	4001414	4001415	4001416
Circuito control	Control circuit	29225	29226	29227
Circuito potencia	Power circuit	29191		
Contactador seguridad	Safety contactor	13022		13018
Relé	Relay	13020		
Contactador	Contactor	-	-	13019
Relé steril.	Steril relay	36027		
Relé DRY	Dry relay	36027		-
Relé electroválvula	Electrovalve relay	-	-	36030
Fusible (maniobra)	Fuse (operation)	15555		
Fusible (potencia)	Fuse (power)	15399		
Manovacuómetro	Pressure and vacuum gauge	16009		
Presostato (seguridad)	Safety pressure switch	16212		
Electroválvula N.A. (purgado)	Electrovalve N.A. (drainage)	16022		
Electroválvula N.C. (vaciado, secado y llenado)	Electrovalve N.A. (emptying, drying an filling)	16210		
Electroválvula N.C. (vaciado)	Electrovalve N.A. (emptying)	-	-	16215
Interruptor/selector	Switch / Selector	20017		
Junta tapa	Lid gasket	21015	21014	21061
Piloto rojo (alarma)	Red indicator lamp (alarm)	22021		
Bomba vacío	Vacuum pump	24209	24210	
Válvula (selector llenado)	Valve (filling selector)	34094		
Válvula (selector vaciado)	Valve (emptying selector)	34094		34578
Válvula vapor	Steam valve	34556		34058
Válvula retención	Retaining valve	34557		34585
Válvula seguridad	Safety valve	34200		34128
Resistencia esterilización	Sterilization heater element	39158	39001	39511
Resistencia secado	Drying heater element	39006	39250	39320
Rueda	Wheel	40007		40026
Termostato seguridad	Safety thermostat	43031		
Termostato secado	Drying thermostat	43001		
Sonda Pt100	Pt100 probe	43071		
Transformador	Transformer	45033		
Boya nivel	Level float	20052		

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