

Title: Defibrillator – Medtronic AED	Date: January 8, 2022
By: PhysioControl	DISCLAIMER: THIS PROCEDURE PROVIDED "AS IS" AND WITH POSSIBLE FAULTS. USER MUST VERIFY BEFORE USE. NEITHER PROVIDER NOR WEBSITE ASSUMES ANY RESPONSIBILITY FOR ITS USE.
File = AED Medtronic.doc	

1. General
Service Guideline for: Medtronic Lifepak AED

2. Reference Documents
Operator’s Manual 3201686-011
Service Manual - NA

3. Accessories: Unexpired Pad Set, Battery
Note: Pad Set usually sold with new battery.

4. Battery Information
The Units are often shipped with battery uninstalled.
Battery quality evaluation occurs automatically during the self test.

5. Basic PM Procedure

With a good battery, turn on unit by briefly holding down the front yellow button shown in the photo. Wait for the “OK” to appear in display.

The following has been copied from the Operator’s manual listed above. *It waives the need to deliver a measurable known joule pulse.*



MAINTAINING A STATE OF READINESS

“Your LIFEPAK CR Plus or LIFEPAK EXPRESS defibrillator does not require routine maintenance. It performs an automatic self-test once a week and every time you turn it on. The electrode indicators briefly flash during the test. If the automatic self-test detects a condition that requires attention, the OK symbol in the readiness display will fade and either the CHARGE-PAK symbol, the ATTENTION symbol, or the WRENCH symbol will appear, depending on the type of condition detected.

On a regular basis, you should do the following:

- Check to make sure that the OK symbol is visible in the readiness display.
- Check the Use-By-date on the electrode packet (visible through the defibrillator lid in the upper right corner) and all other electrode packets. If the date has passed, replace the electrode packet and the battery charger.
- Check other emergency supplies that may be stored with your defibrillator. When establishing your local inspection schedule, consider how often your defibrillator will be used and how familiar the operators are with using a defibrillator. For example, if the defibrillator is used only rarely, monthly inspections may be appropriate. An inspection checklist is provided in Appendix C. “