


Neu ab:
New as of:
Nouveau depuis:
Nuevo a partir de:


05.2007

Sirona.
The Dental Company



C8+

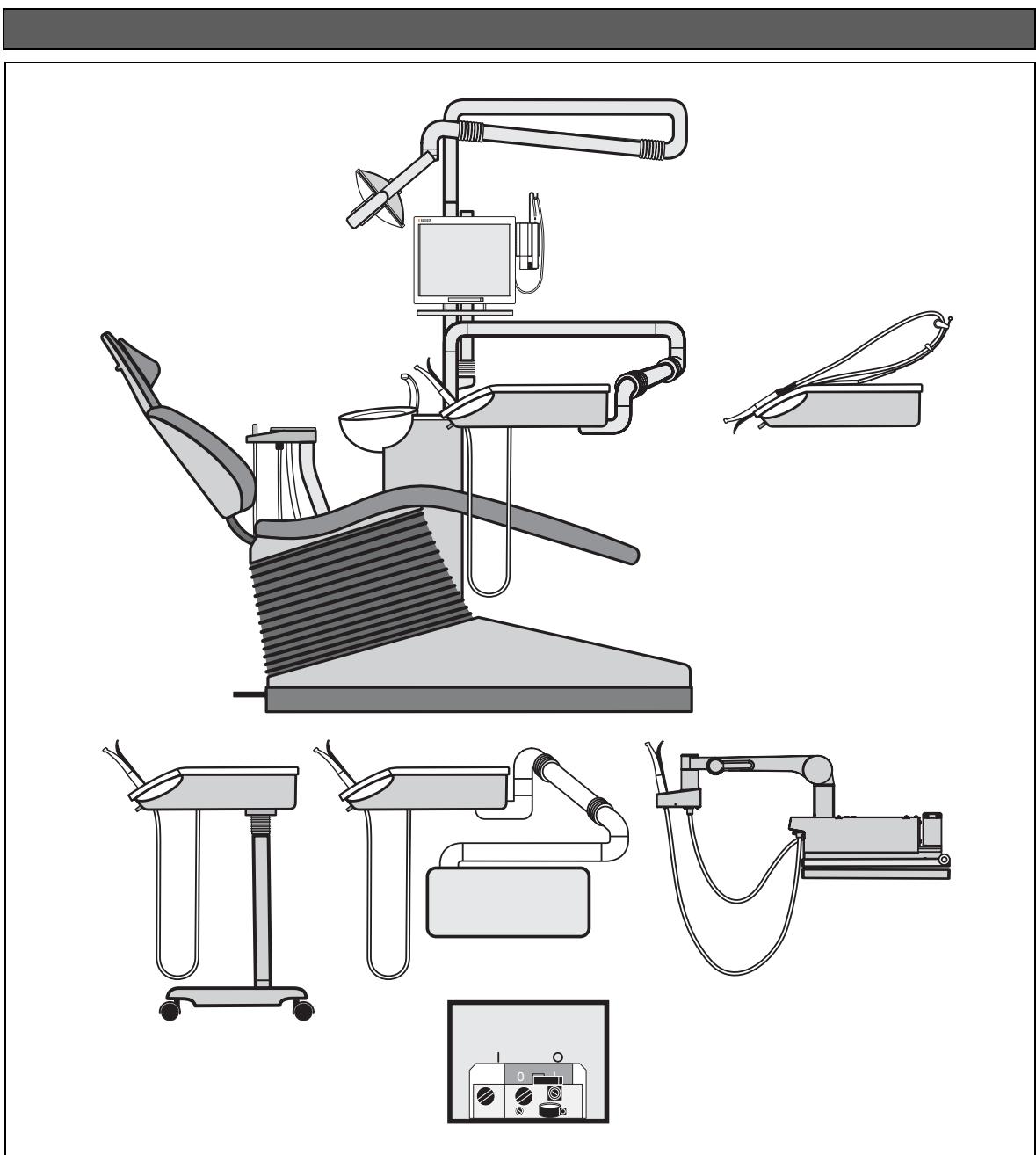




 C8+

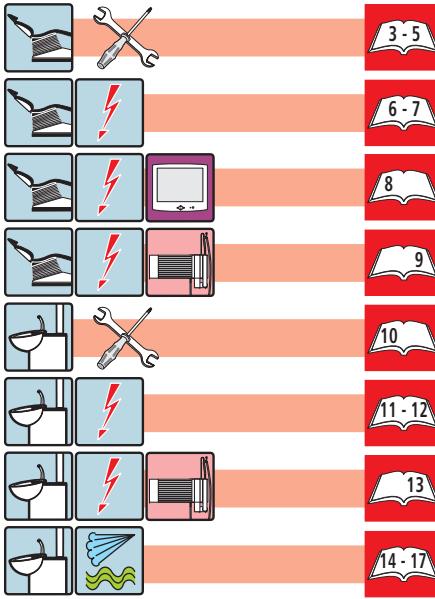

 C8+ Turn

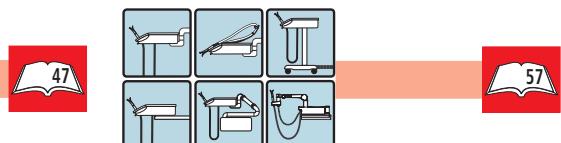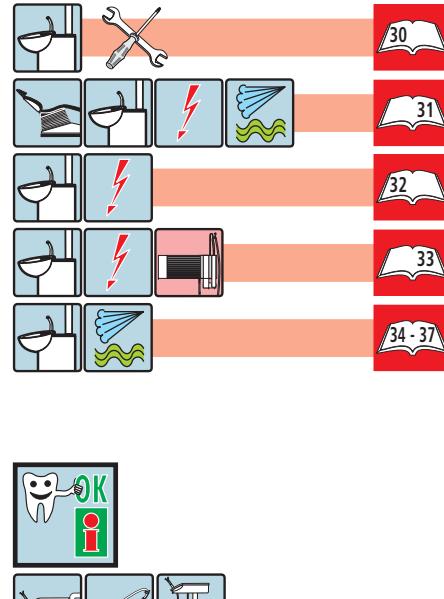


 C8+ Turn

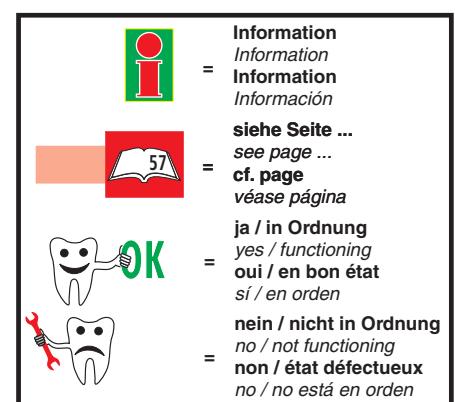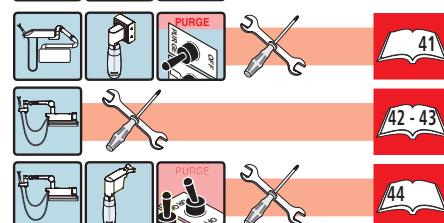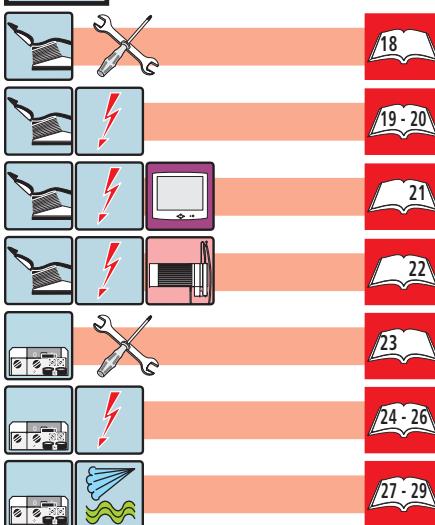



C8+, C8+ Turn





breite Rückenlehne
wide backrest
dossier large
respaldos anchos

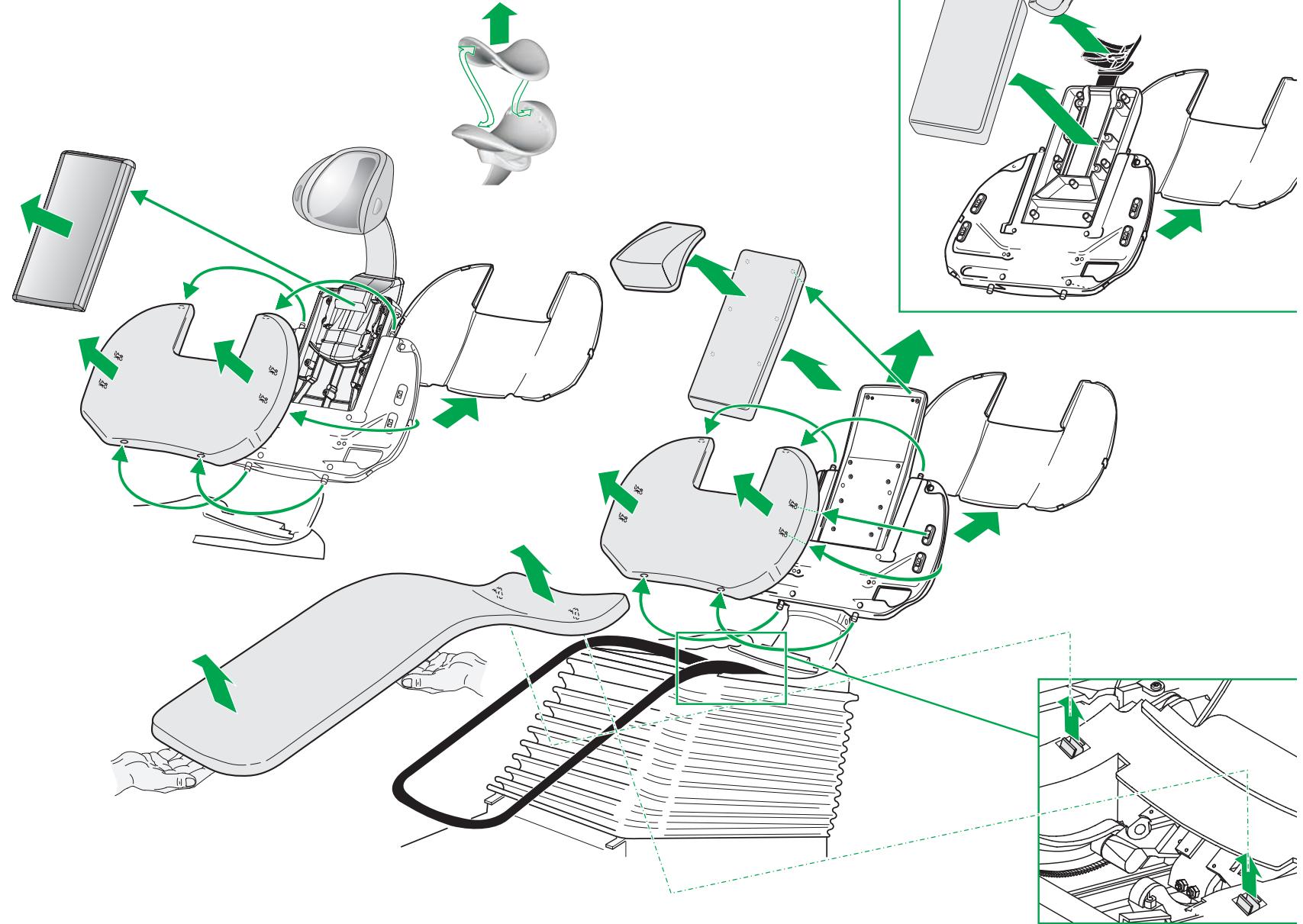






C8+, C8+ Turn


schmale Rückenlehne
narrow backrest
dossier étroit
respaldos estrechos

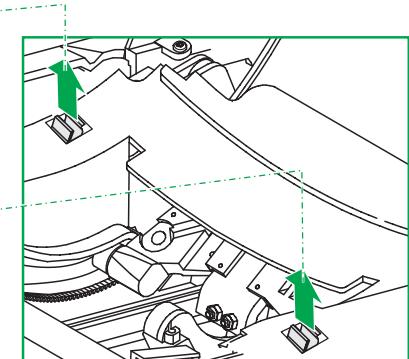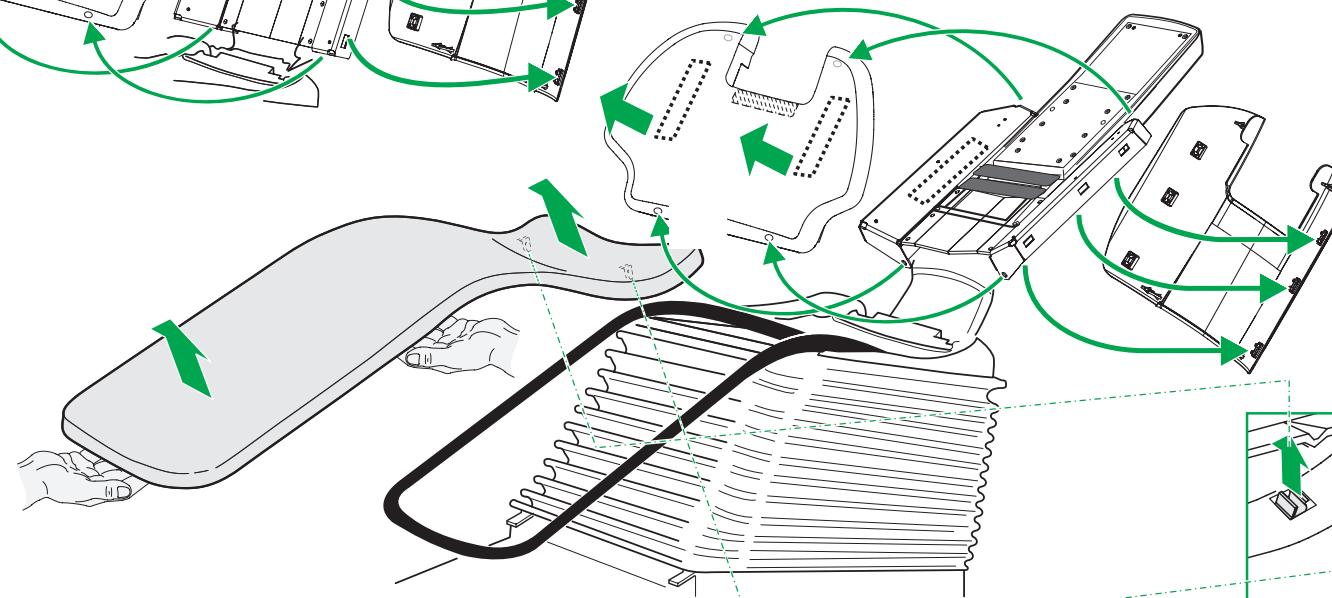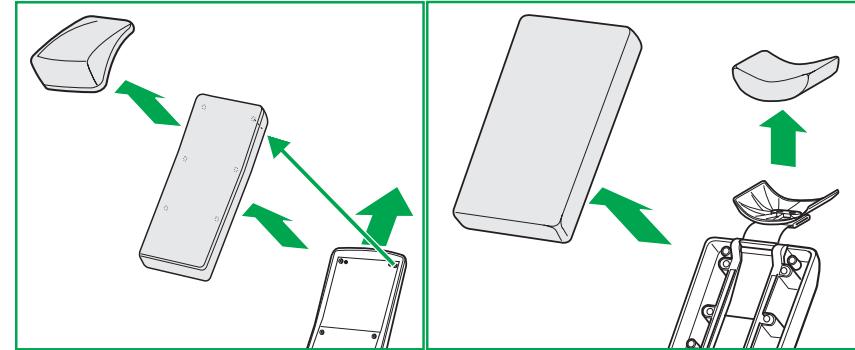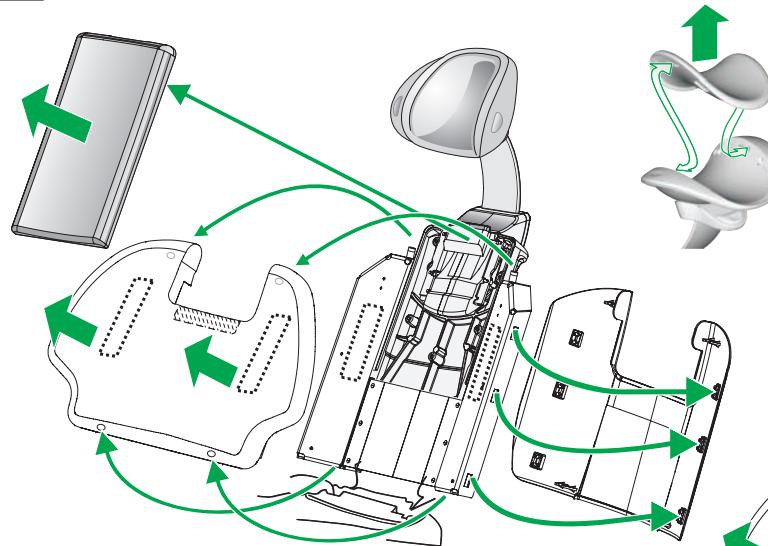





C8+

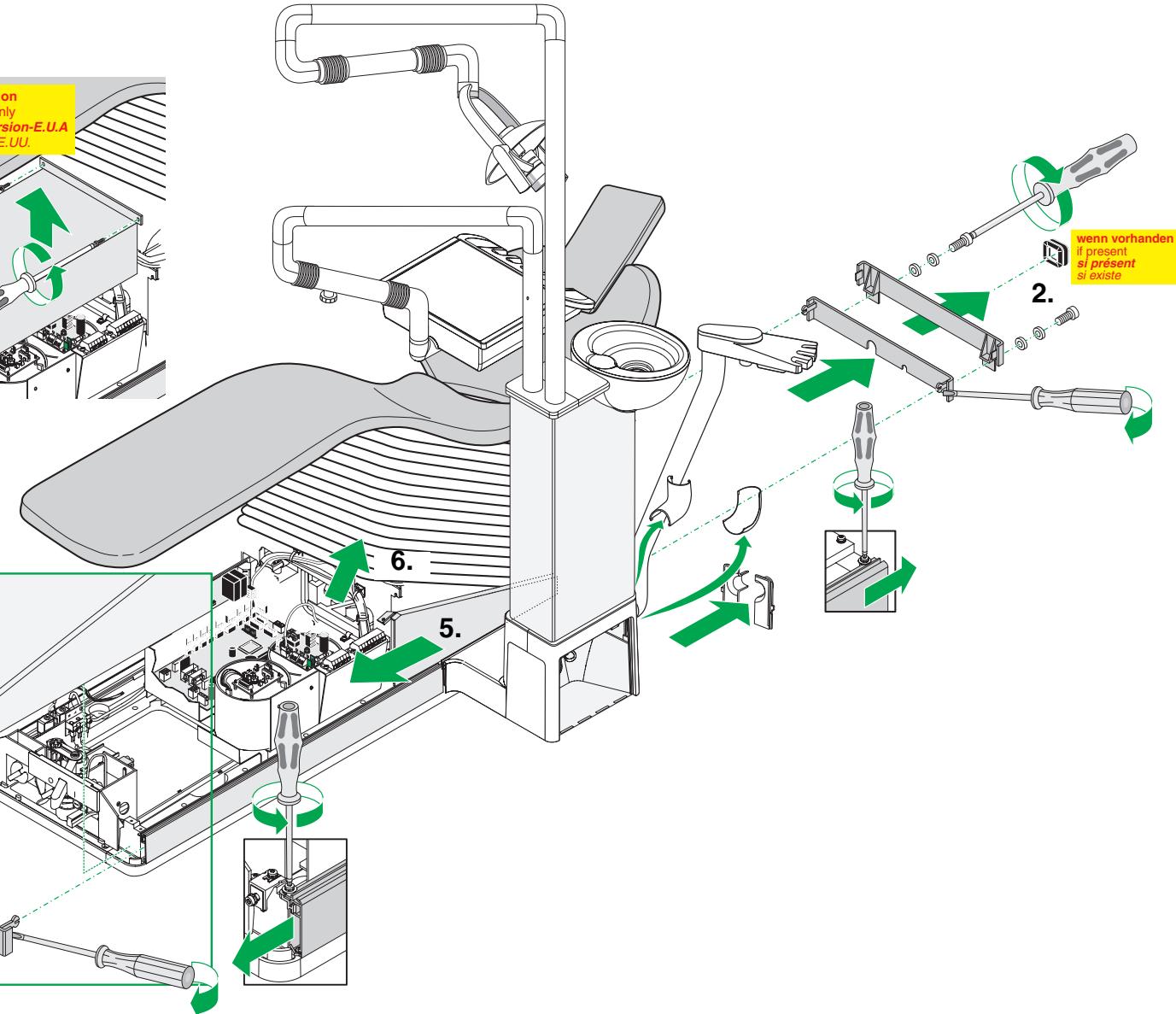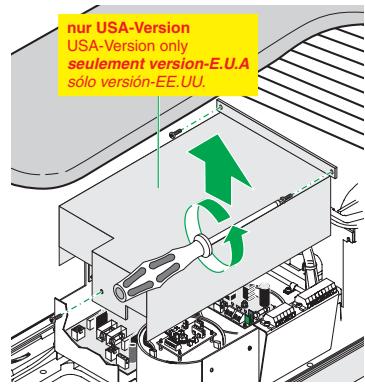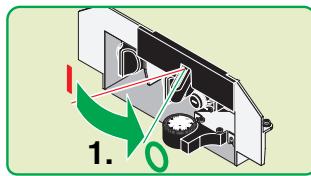

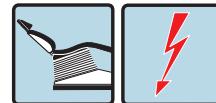

C8+

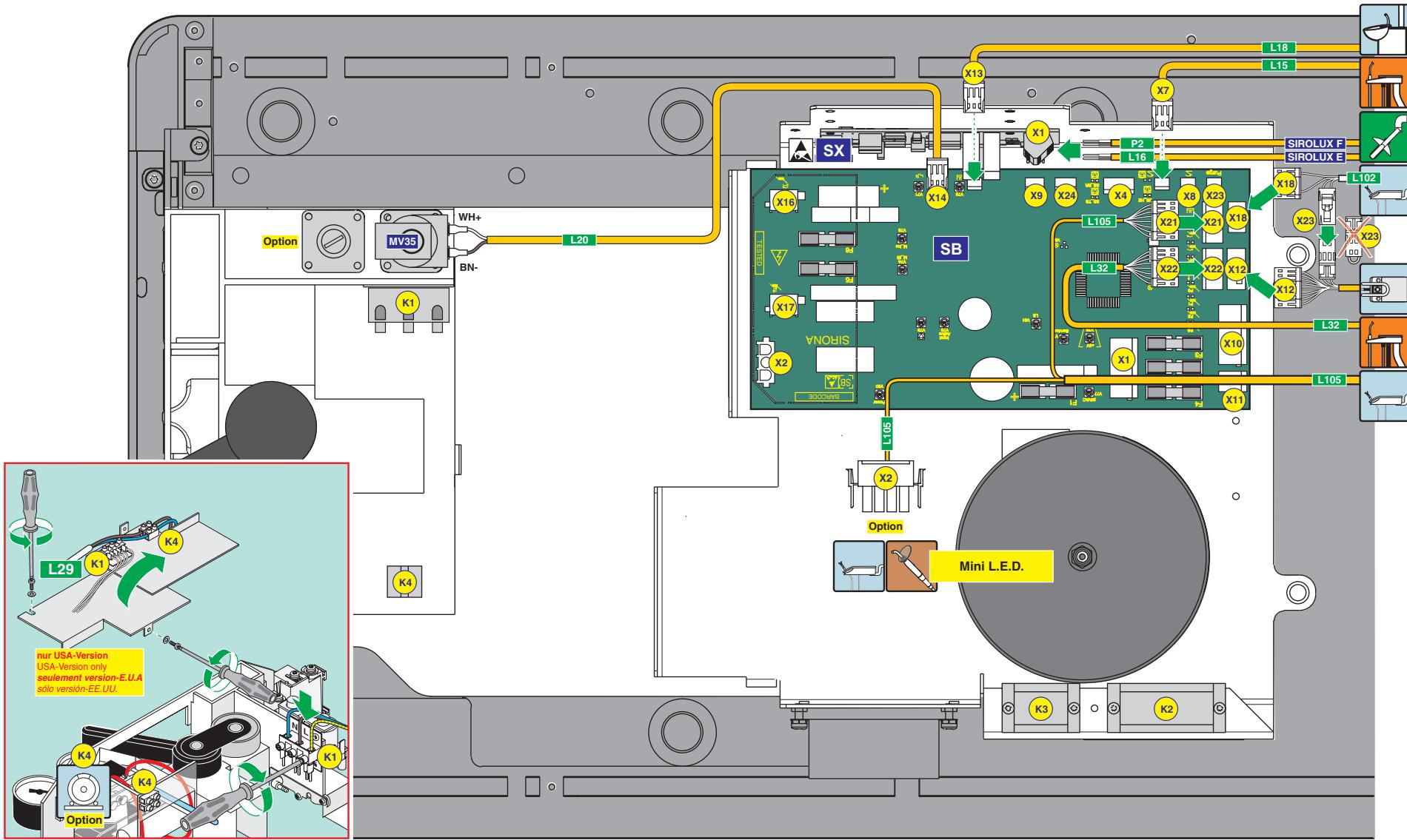

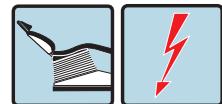

C8+



Amalgamabscheider
Amalgam separator




Desinfektionssystem
Disinfection system






Wasserheizer
Water heater




Mini L.E.D.

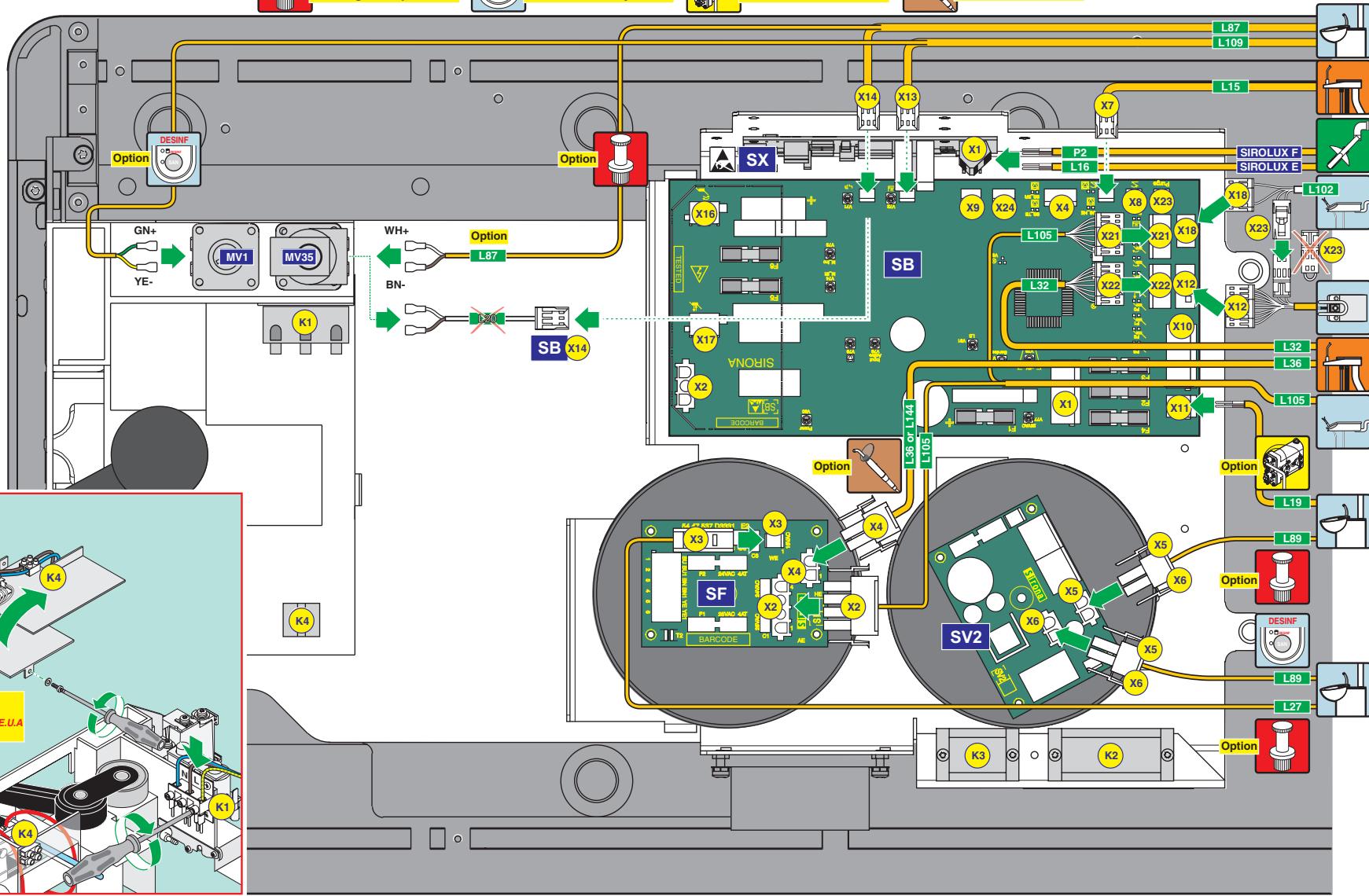


C8+, option video

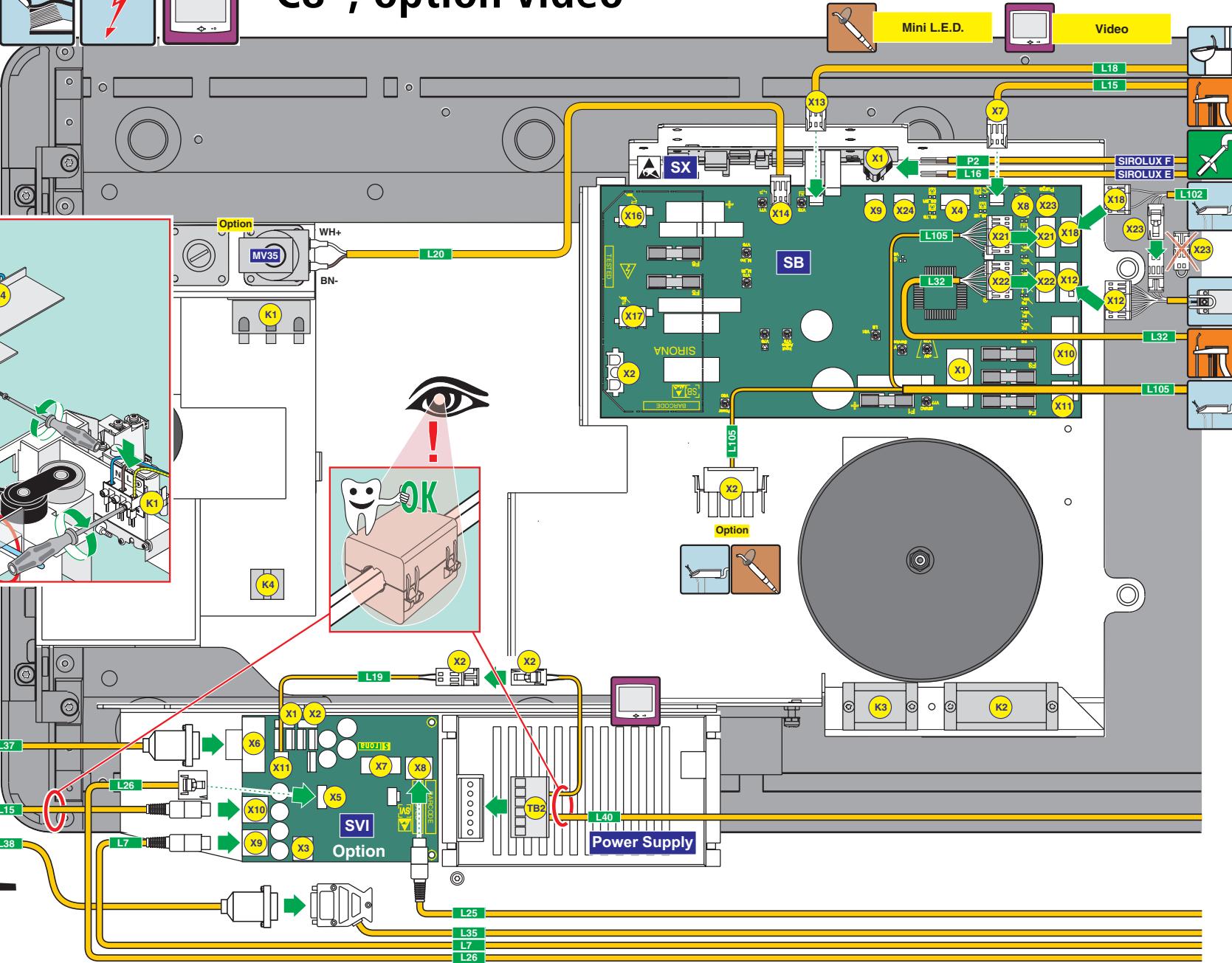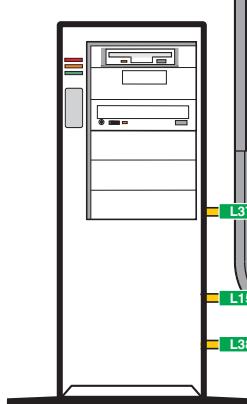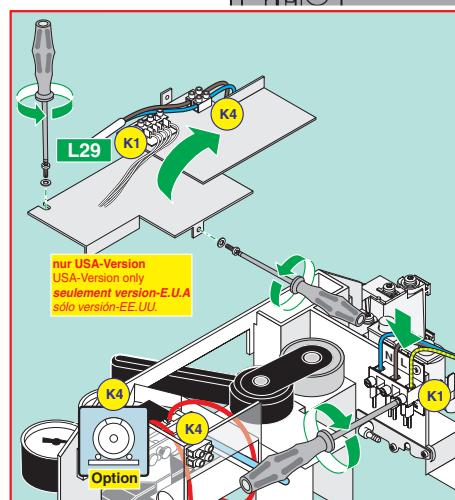

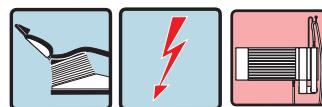



C8+, option chairline

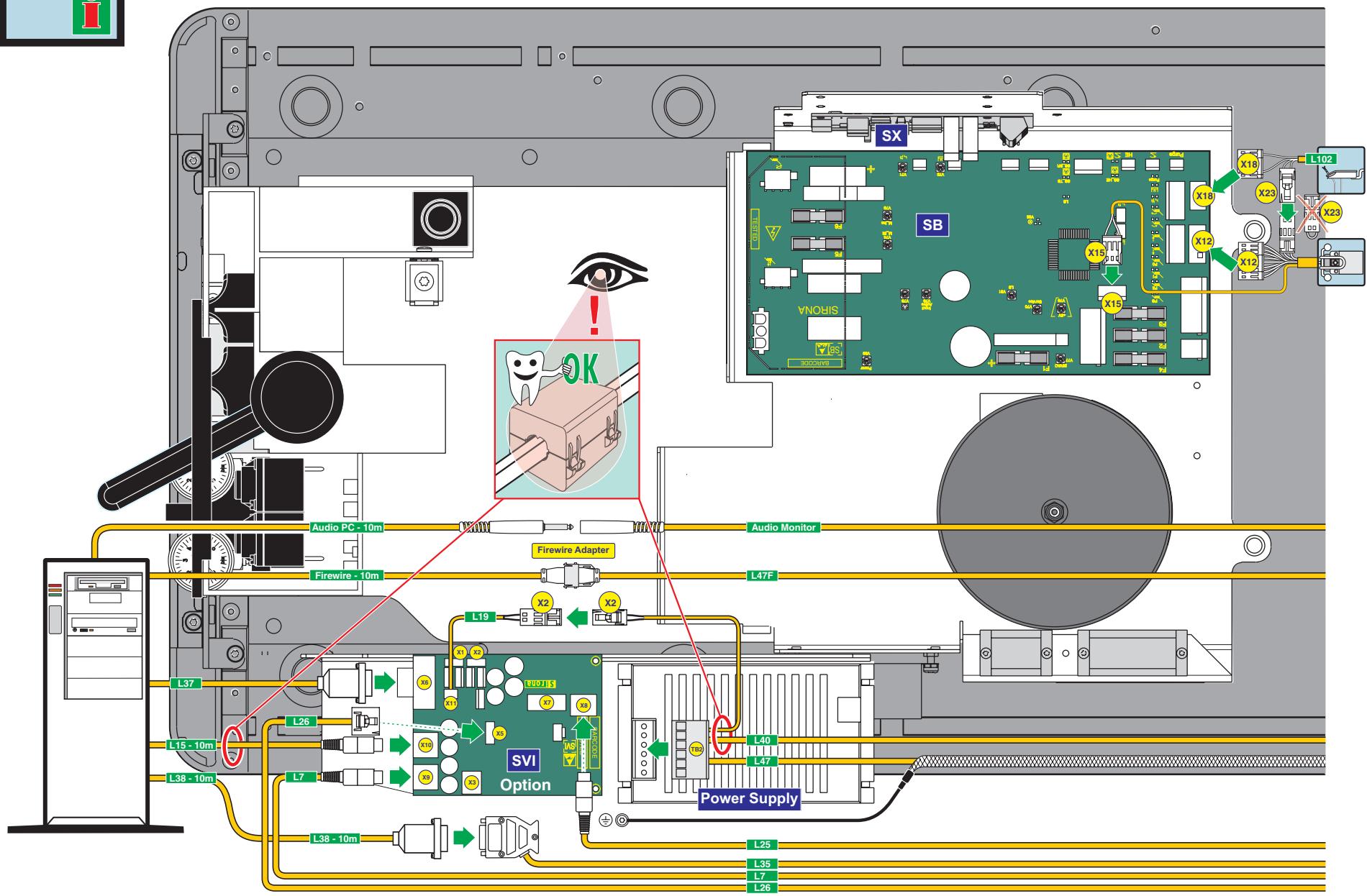




C8+

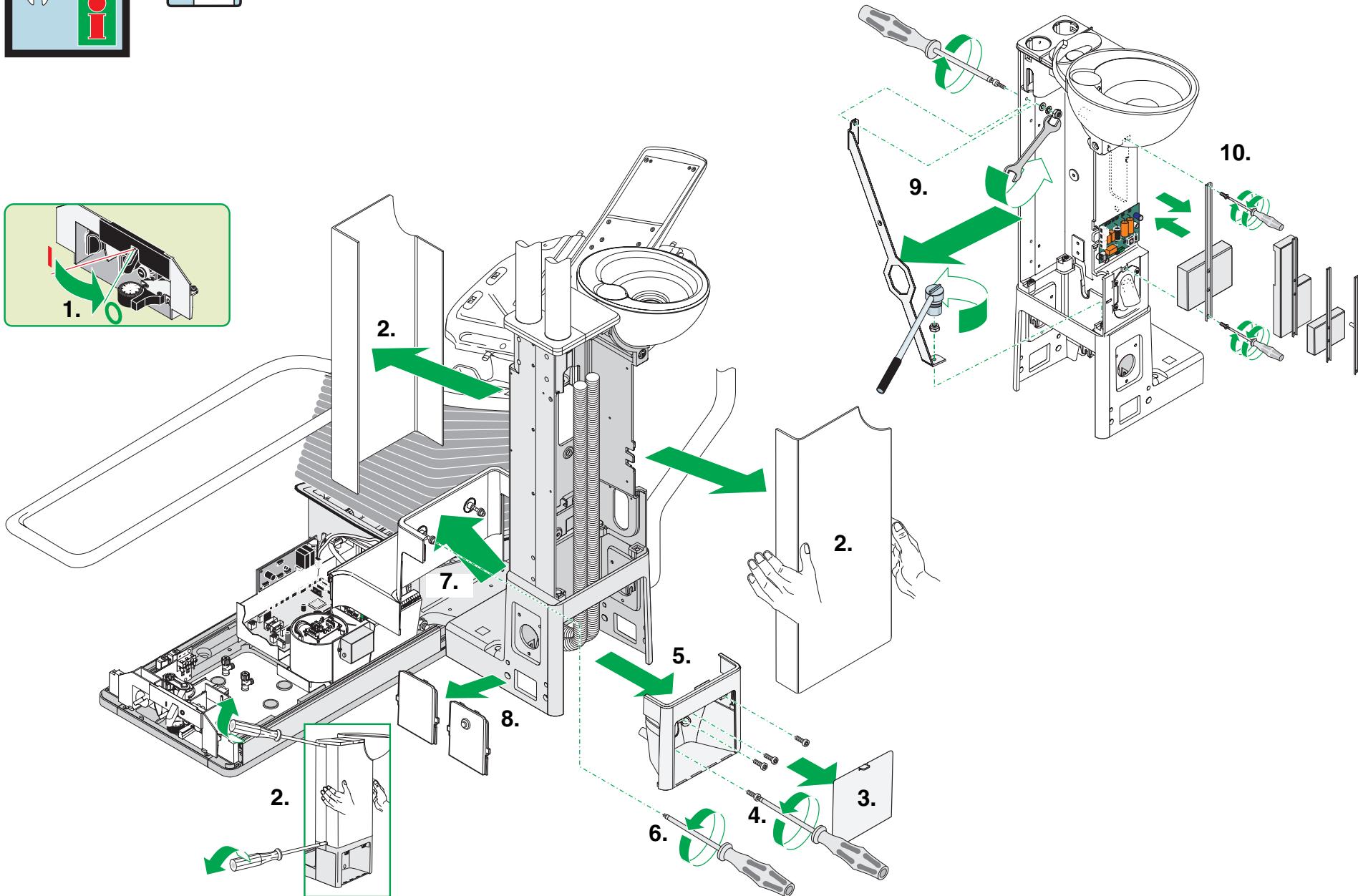

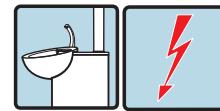


C8+

- Luftstrahlpumpe
Air Venturi
Option
- PURGE
Purge
Option
- Wasserheizer
Water heater
Option
- Helferinelement
Assistant Element
- SIROLUX
- Mini L.E.D.
Option
- Amalgamscheidner
Amalgam Separator
Option

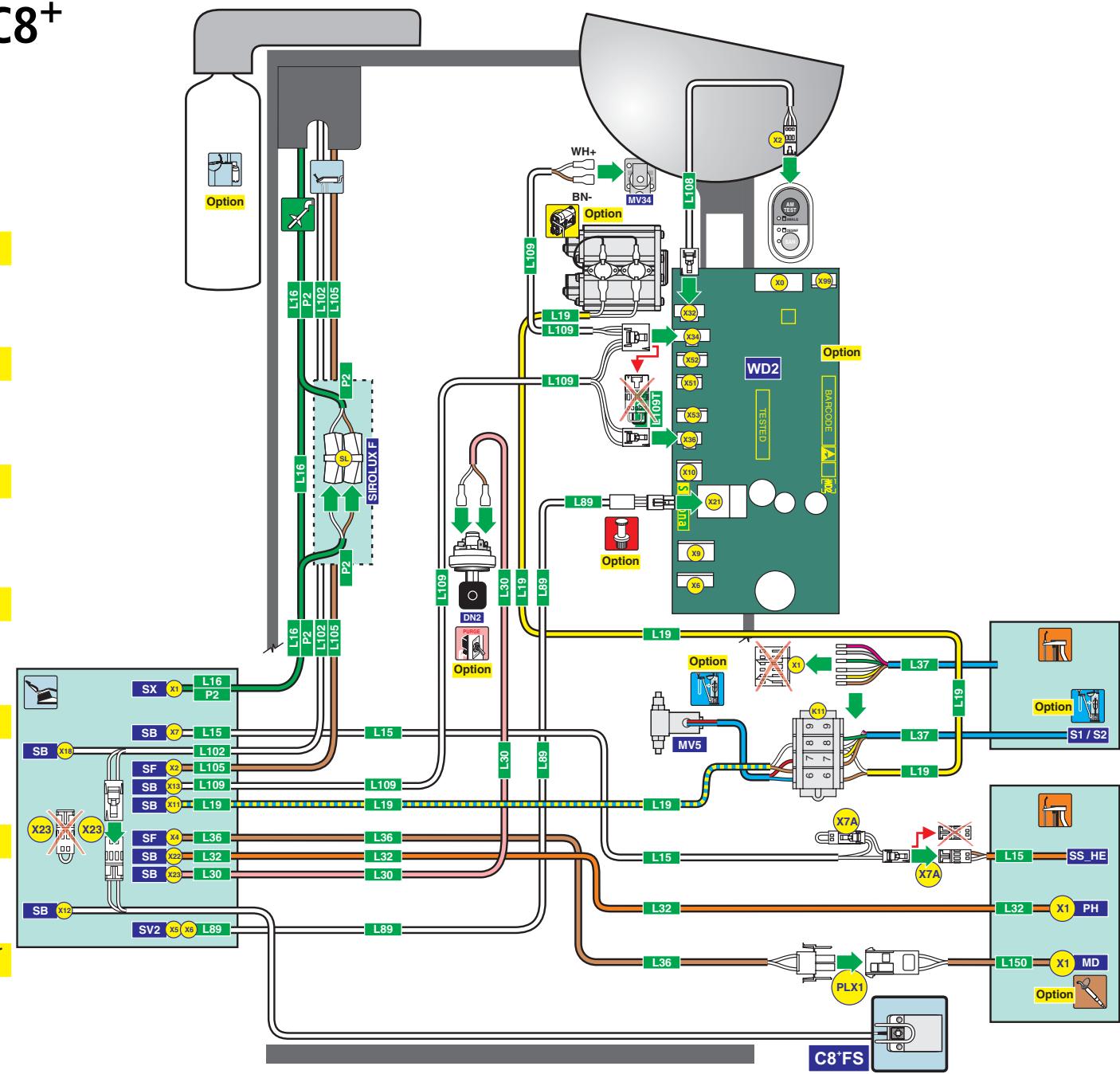

C8+

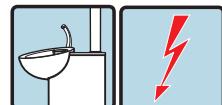


Amalgamscheider
Amalgam Separator
Option



Purge
Option




Helferinelement
Assistant Element

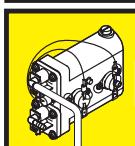


Wasserheizer
Water heater
Option




SIROLUX

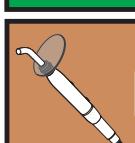


Mini L.E.D.
Option

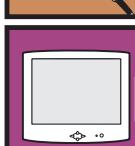


Video
Option

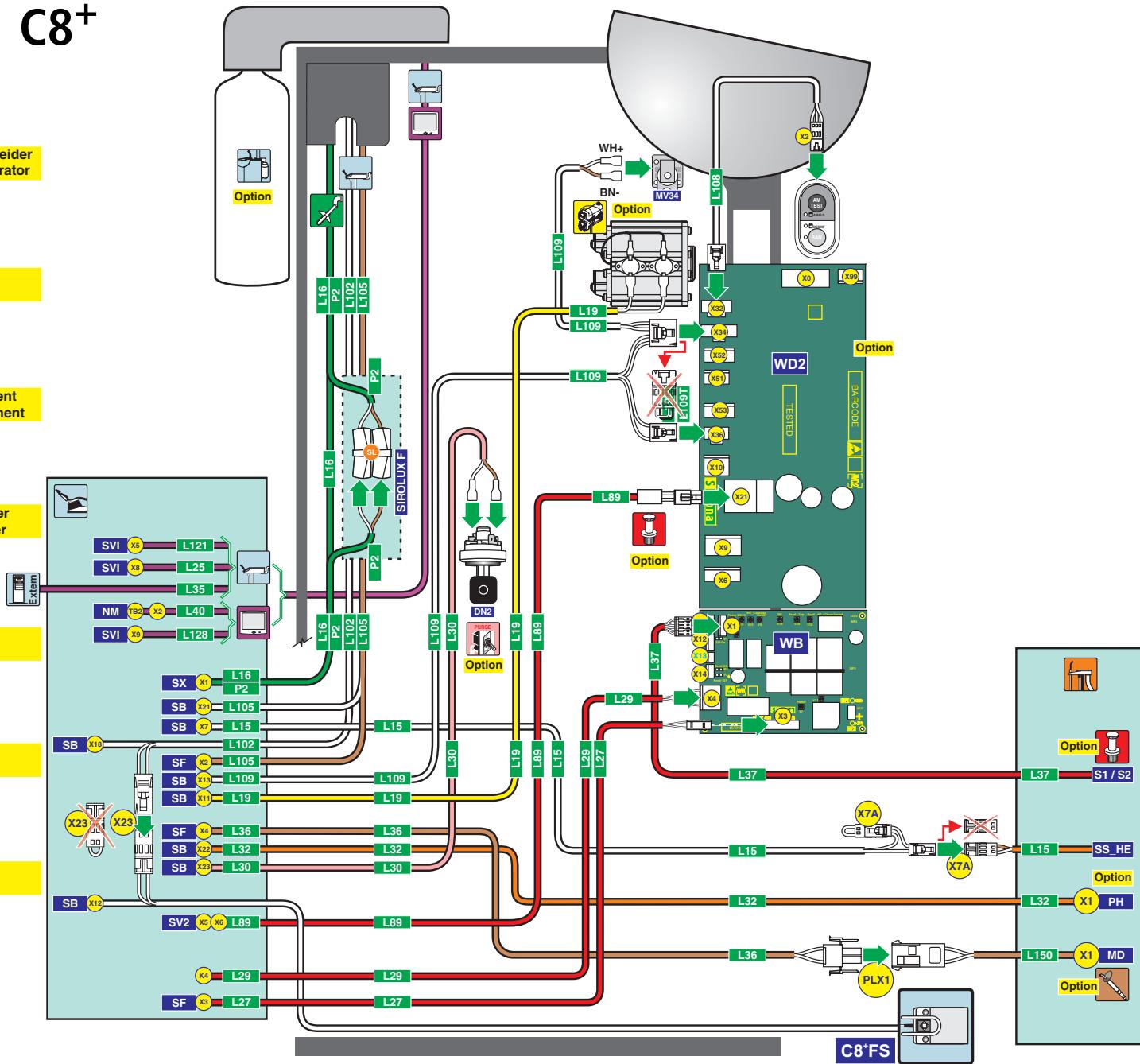

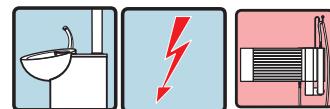


C8+, option chairline

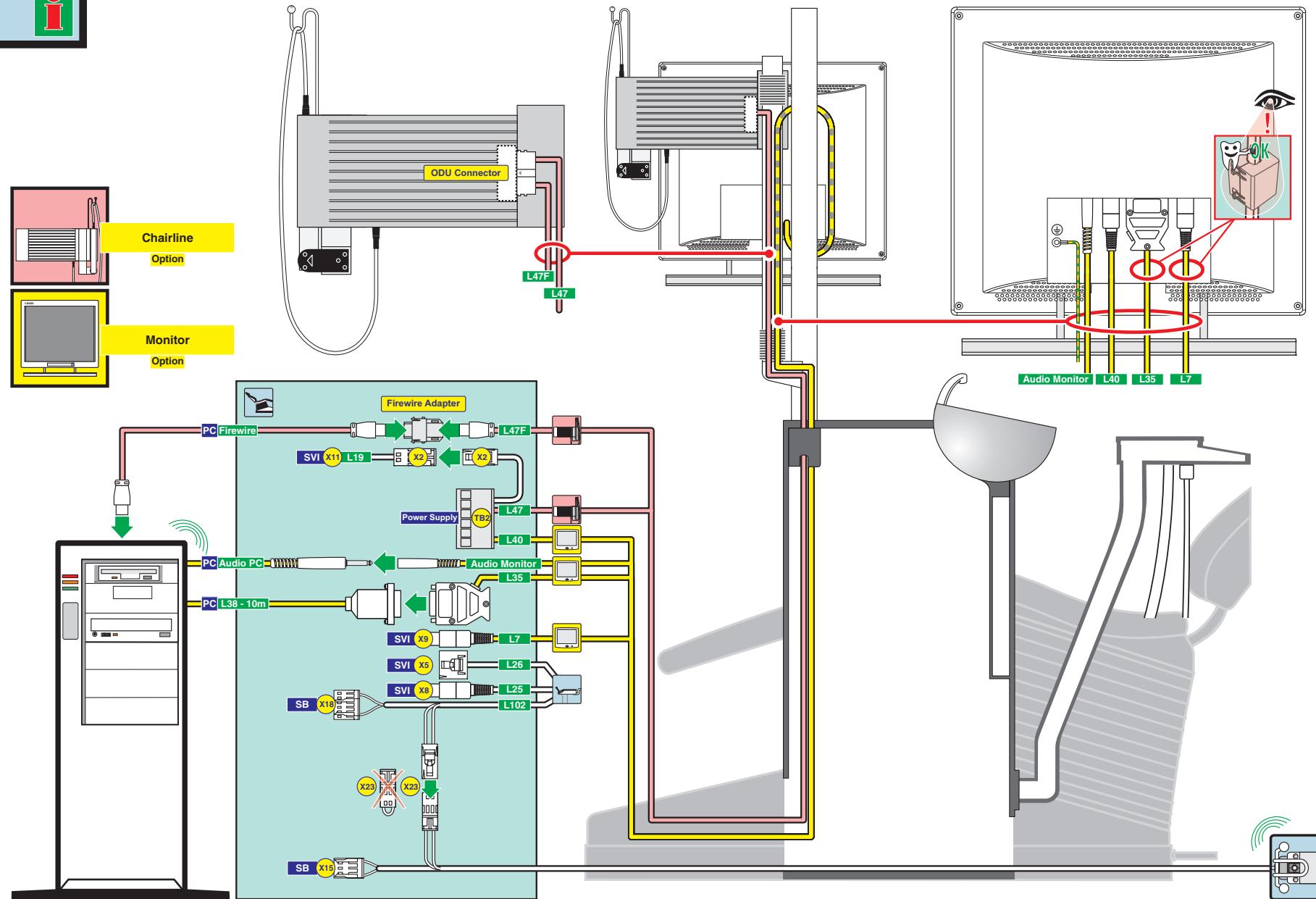




C8+

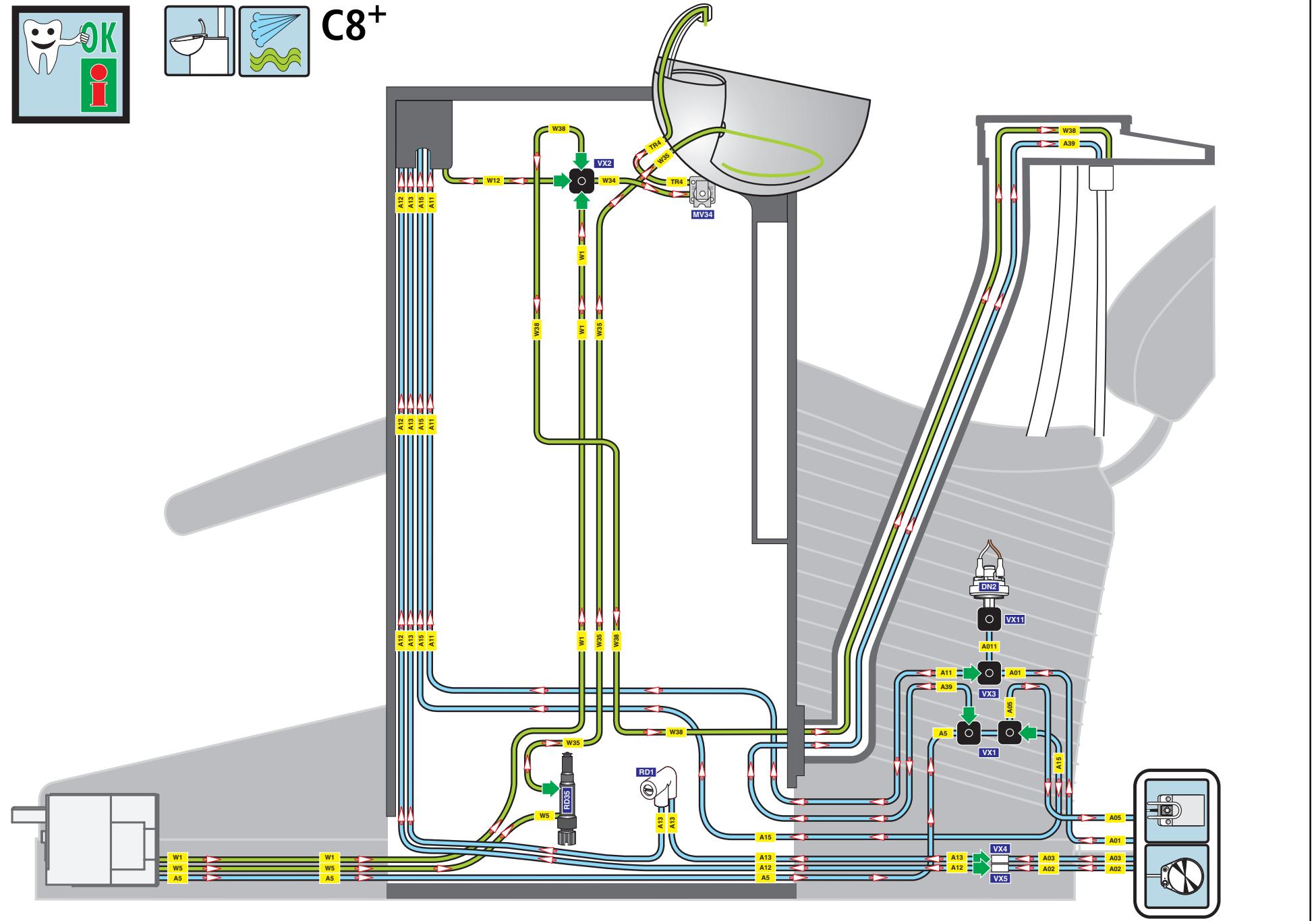


C8+

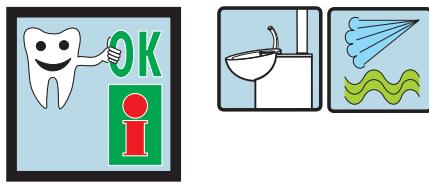




Wasserflasche
Water Bottle
Option



Purge
Option

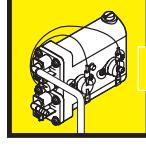

Wasserheizer
Water heater
Option

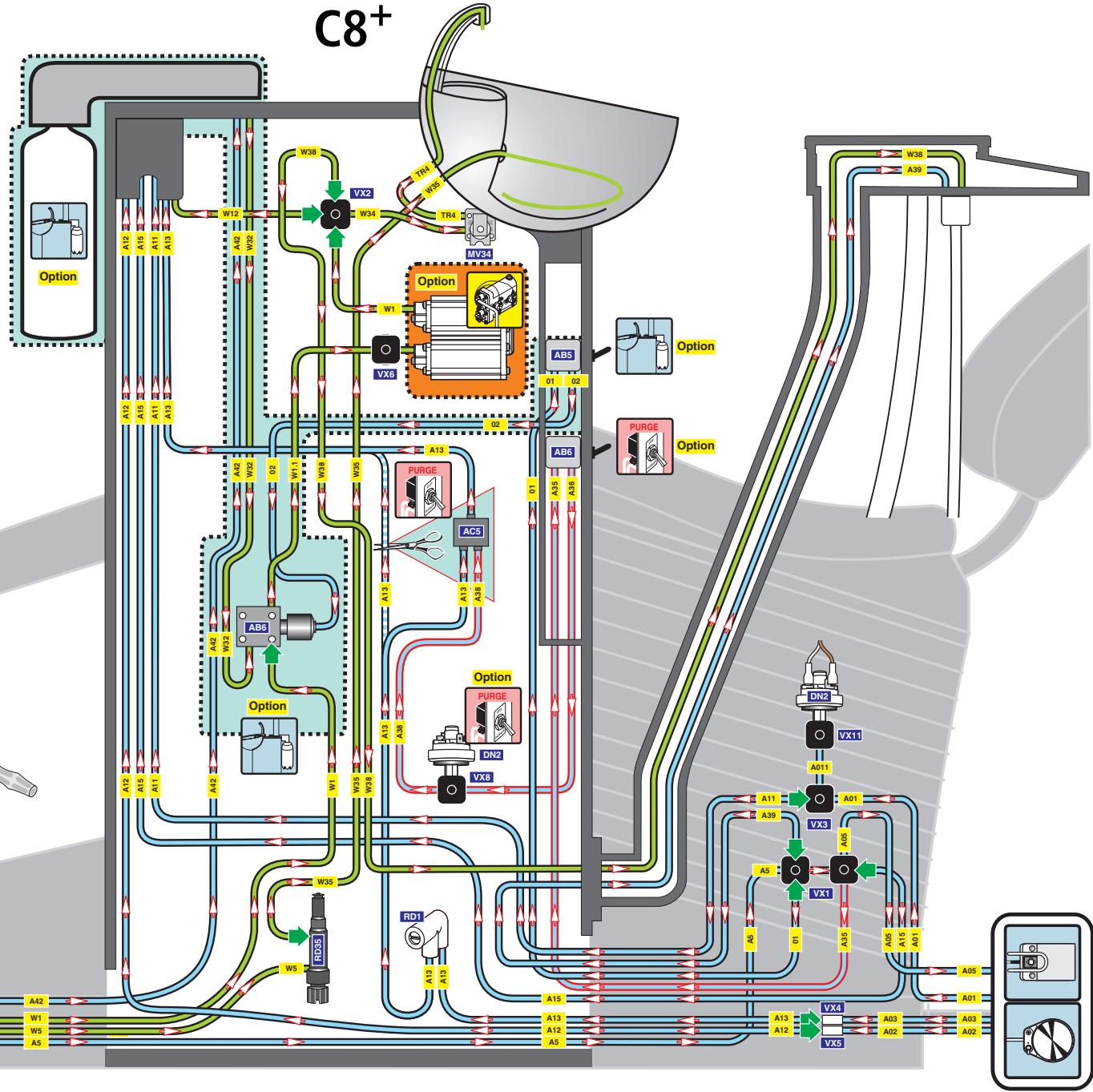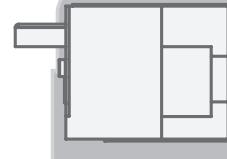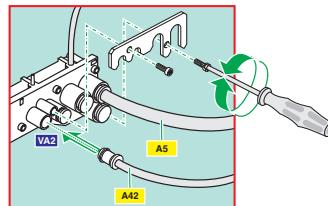


C8+






Wasserflasche
Water Bottle
Option



Purge
Option



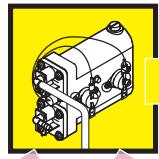


Wasserheizer
Water heater
Option

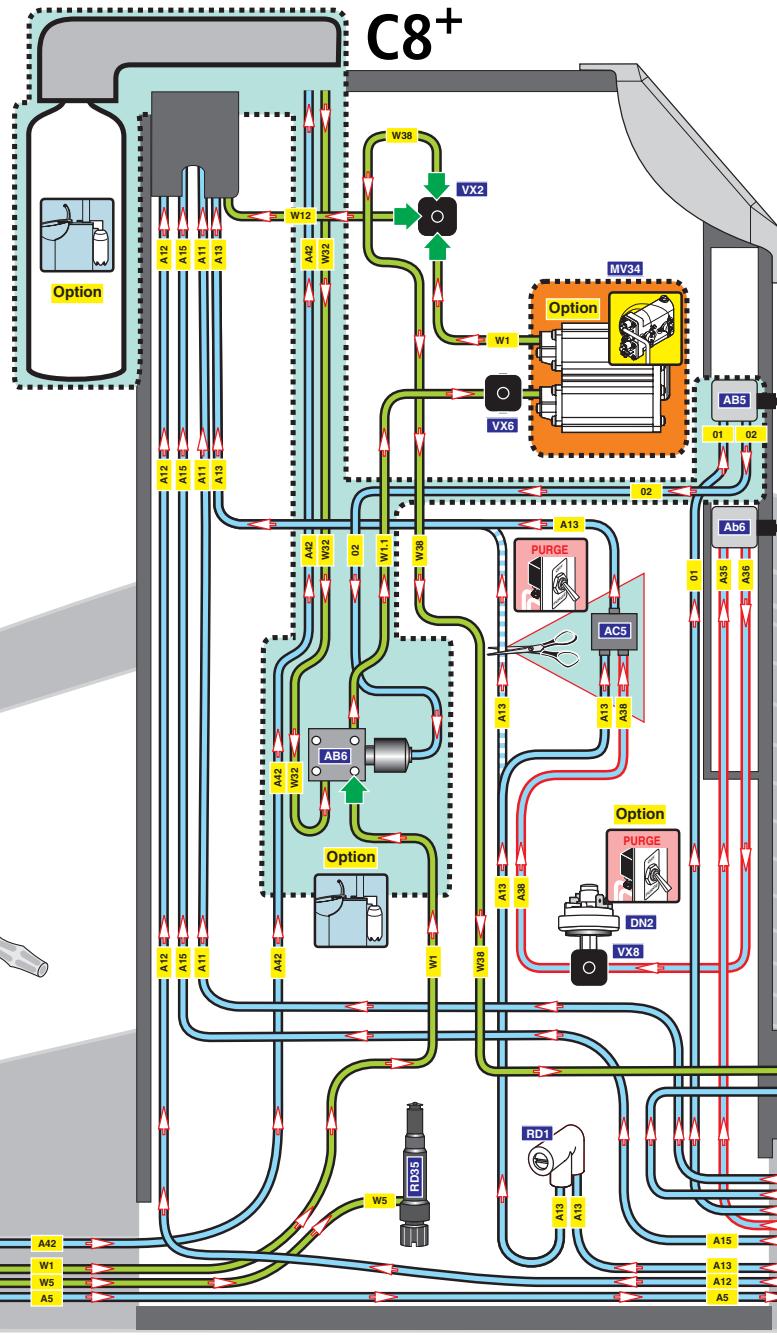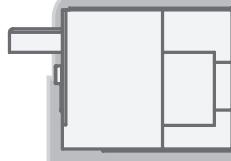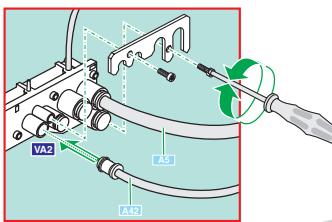

Option

Purge
Option

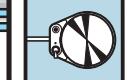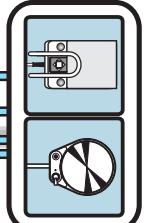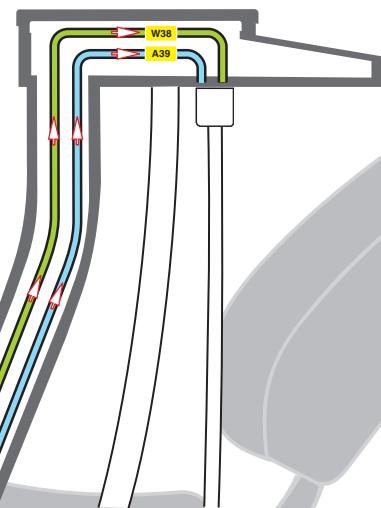






C8+



Desinfektionssystem
Disinfection System
Option

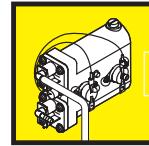

Wasserheizer
Water heater
Option

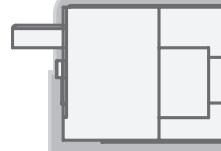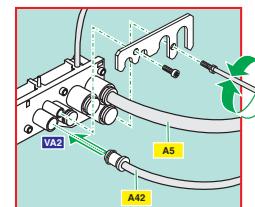


A42

W1








W5

A5

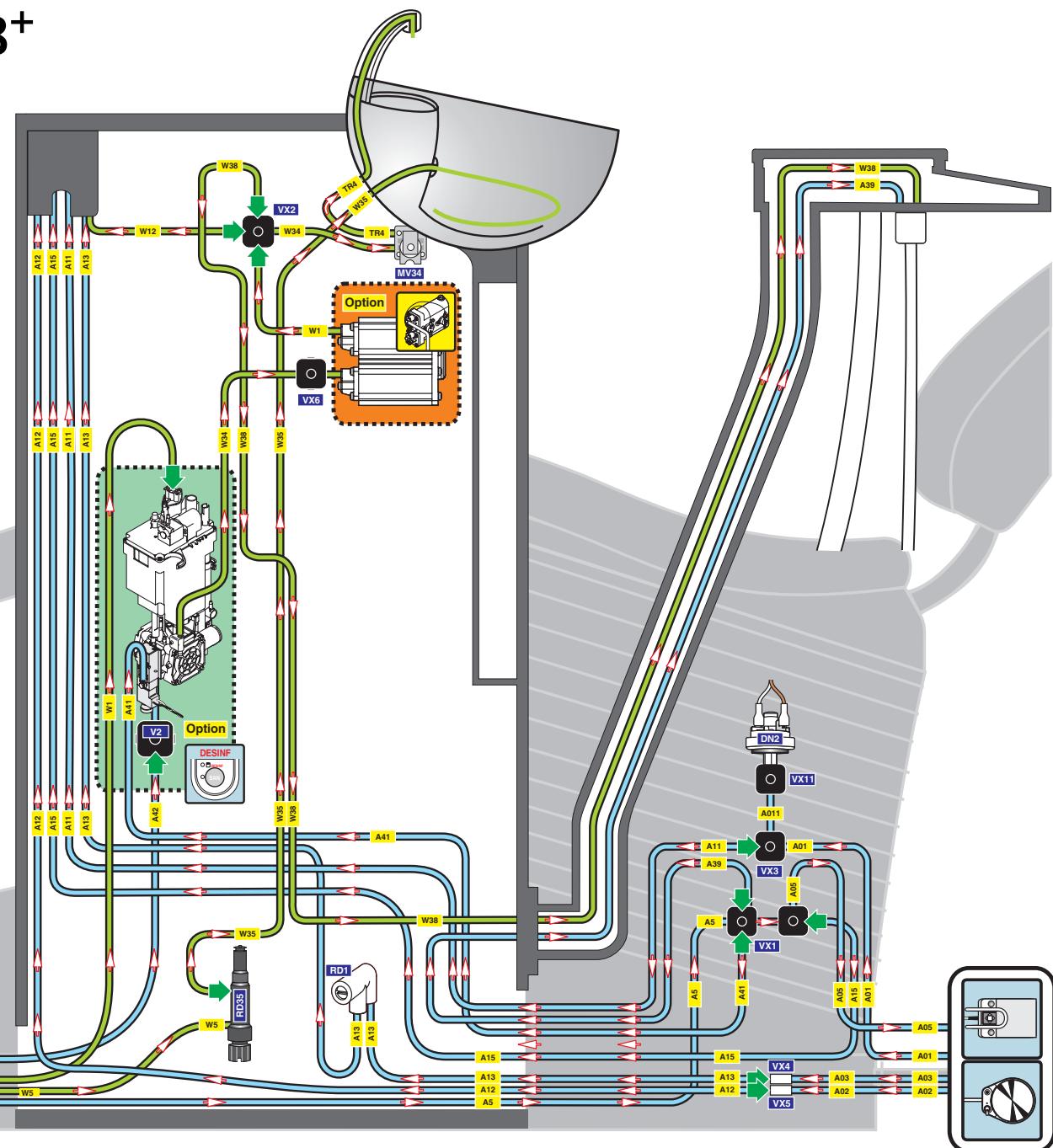



C8+ Turn

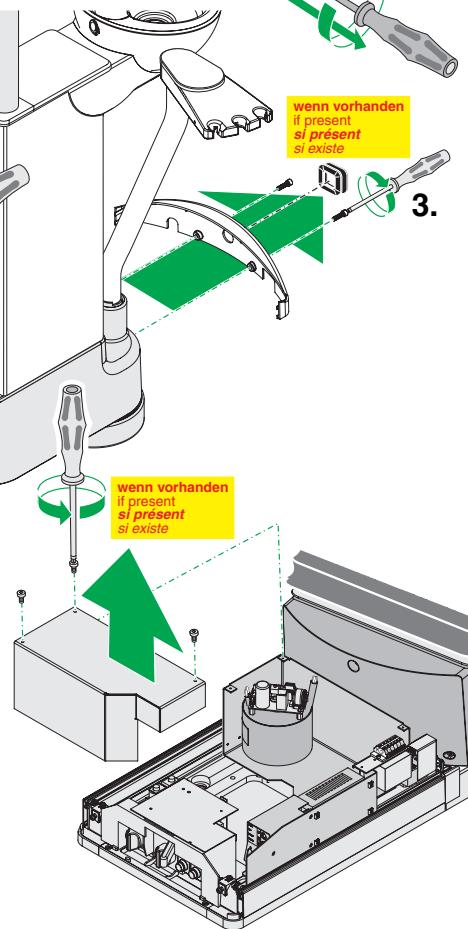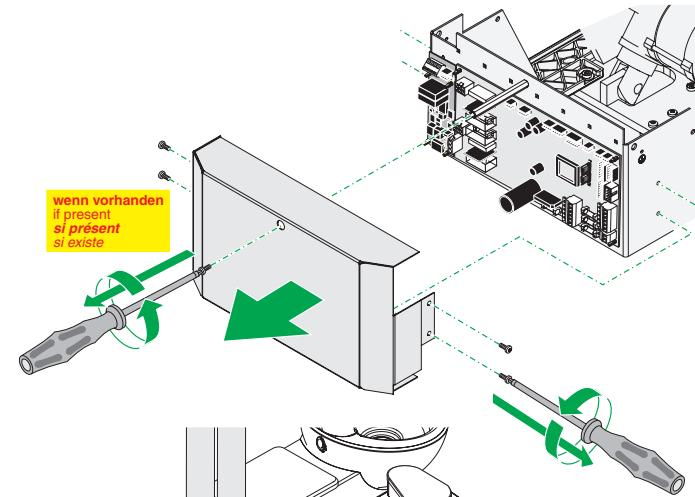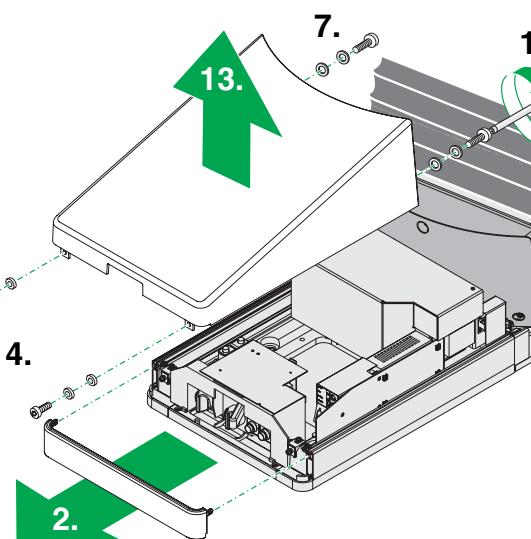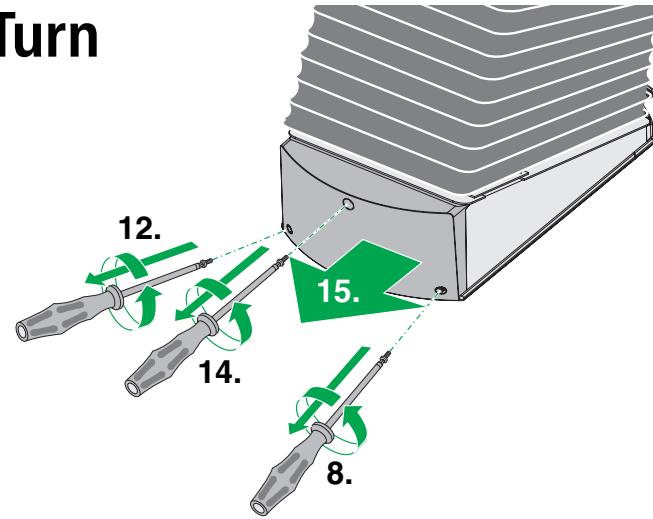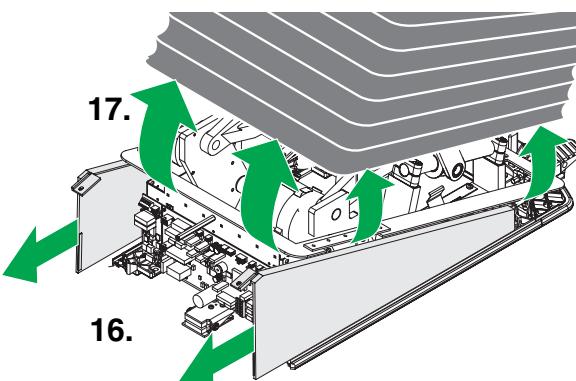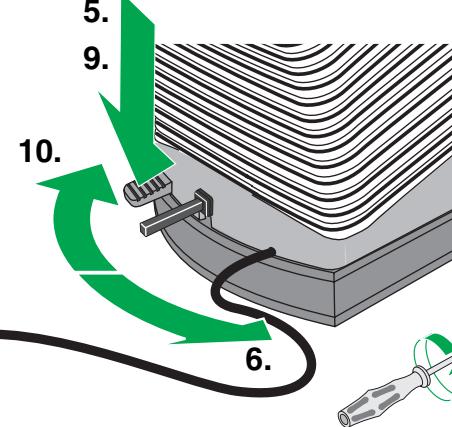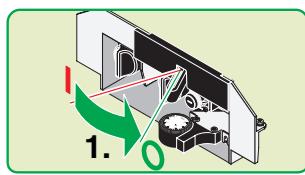


wenn vorhanden
if present
si présent
si existe


wenn vorhanden
if present
si présent
si existe

wenn vorhanden
if present
si présent
si existe



C8+ Turn

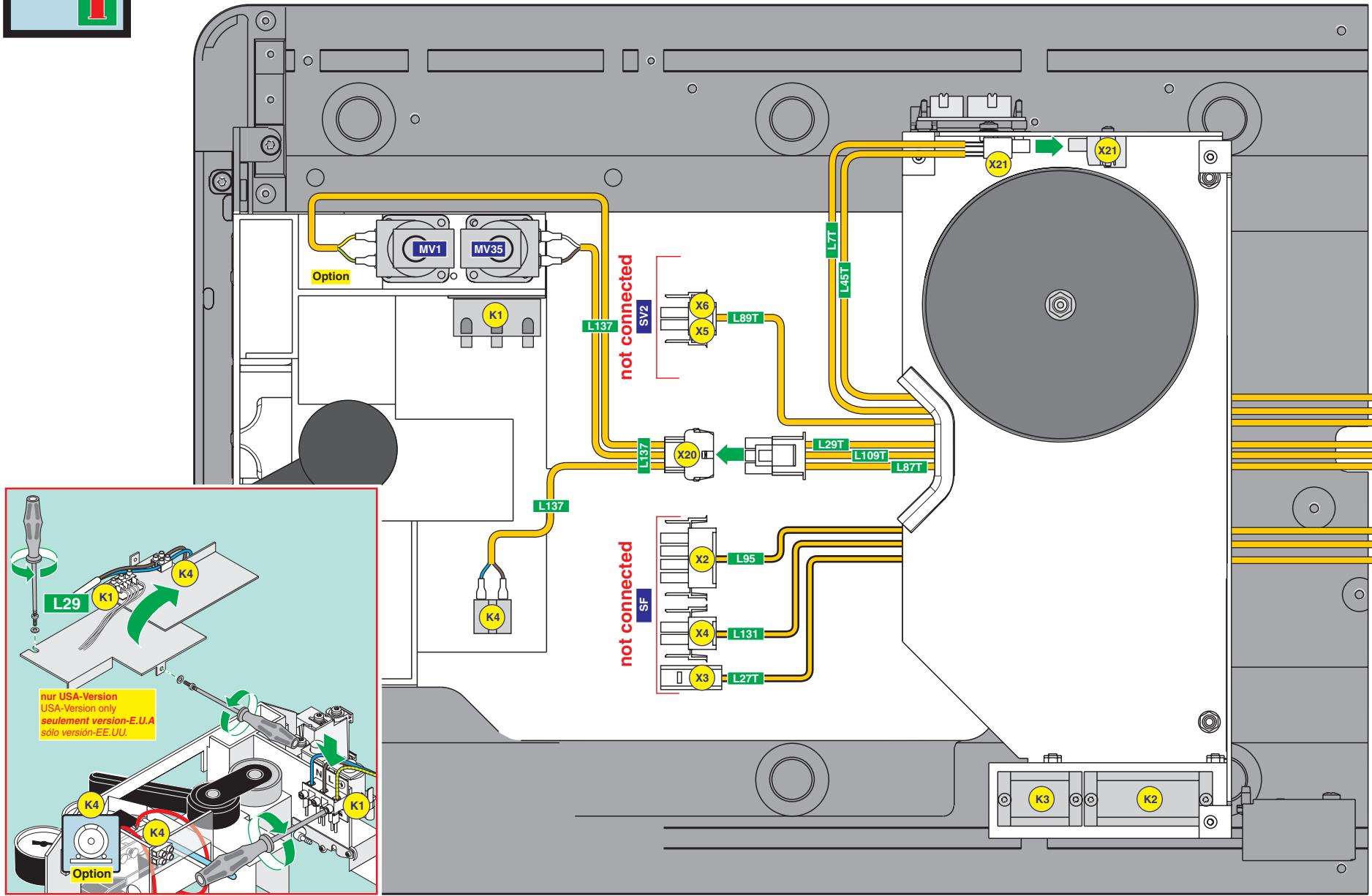

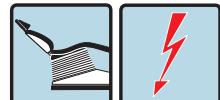



C8+ Turn



Amalgamabscheider
Amalgam Separator





Desinfektionssystem
Disinfection System

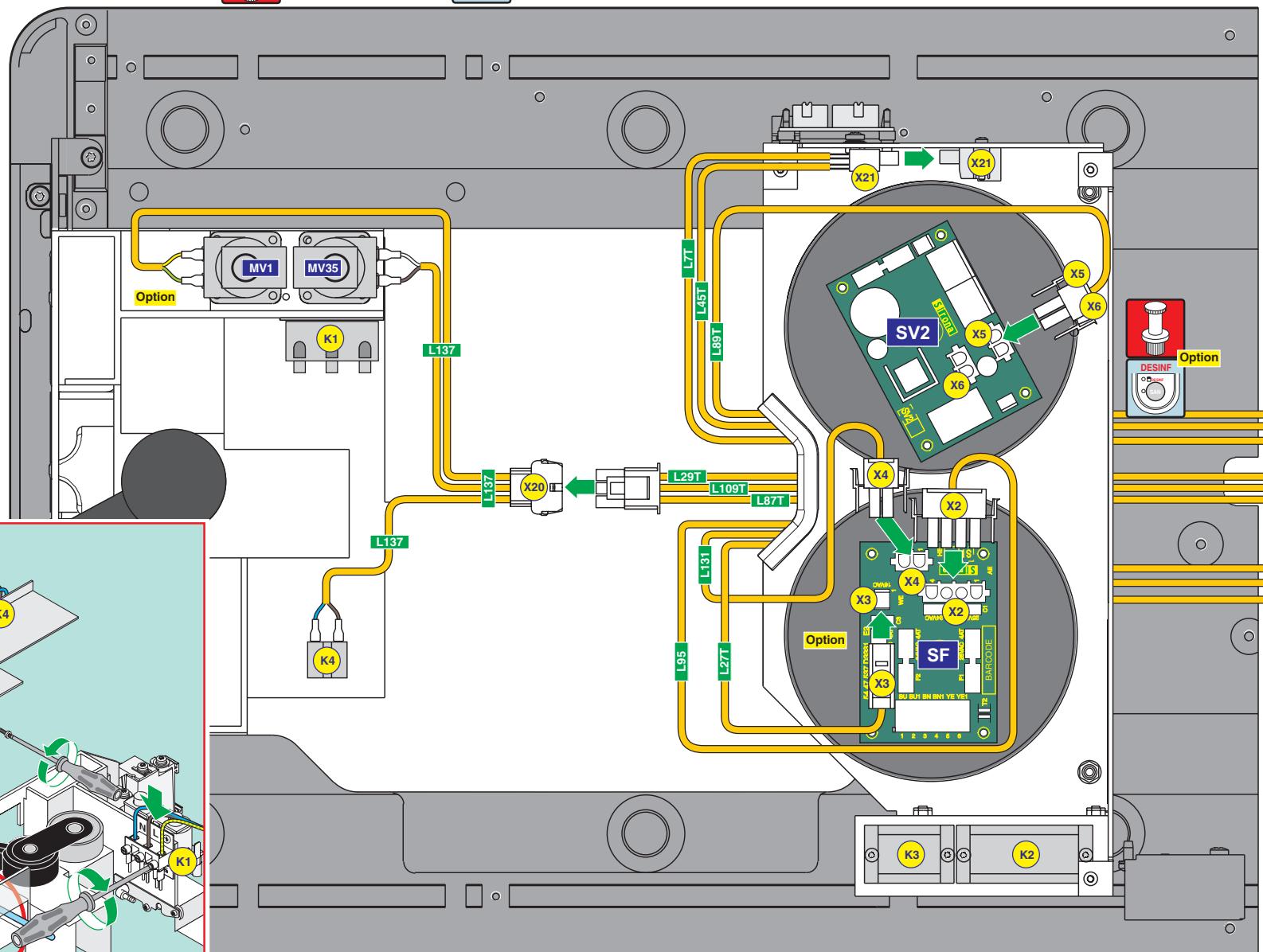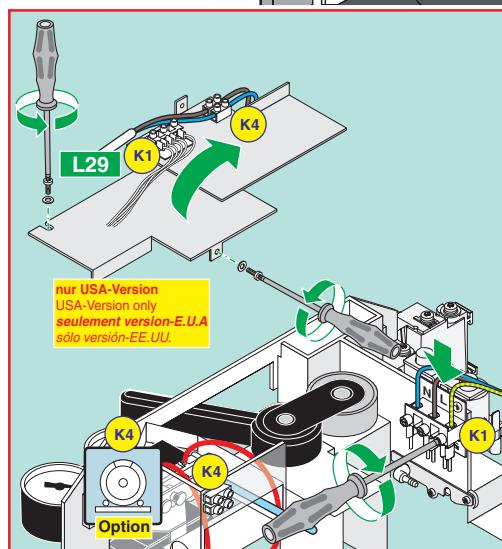




C8+ Turn, option video

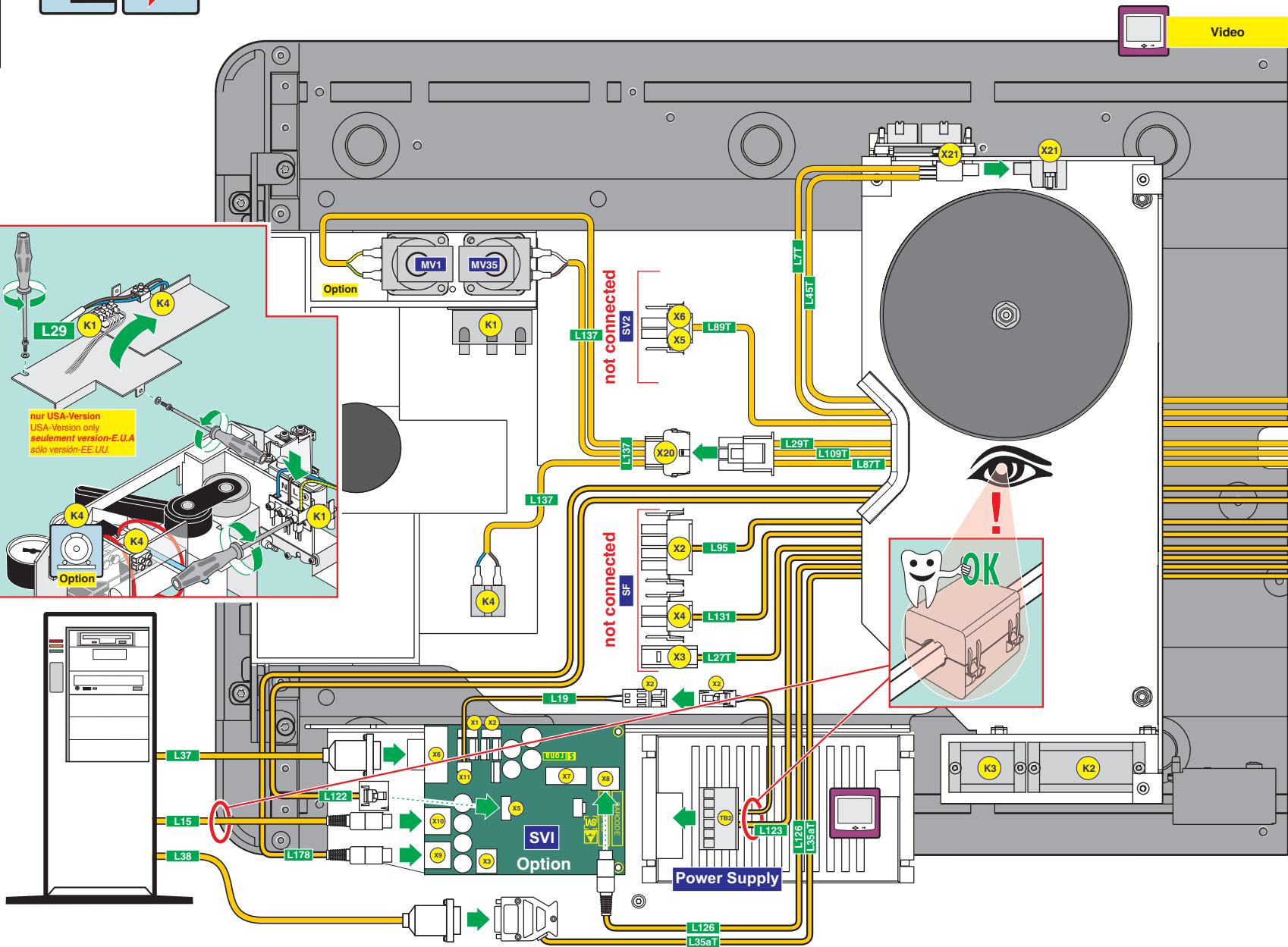


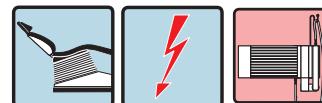


C8+ Turn, option chairline


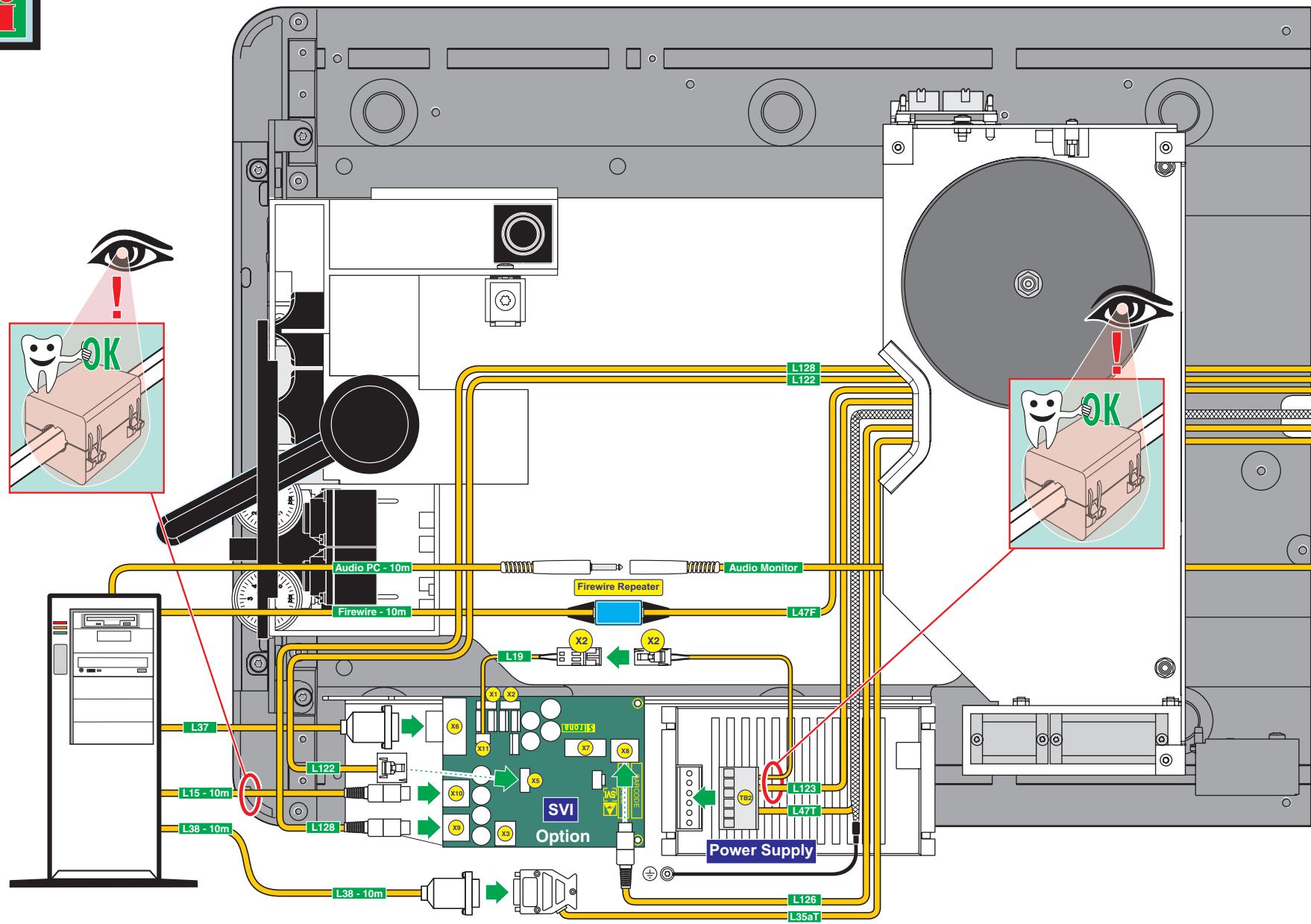




Junction box

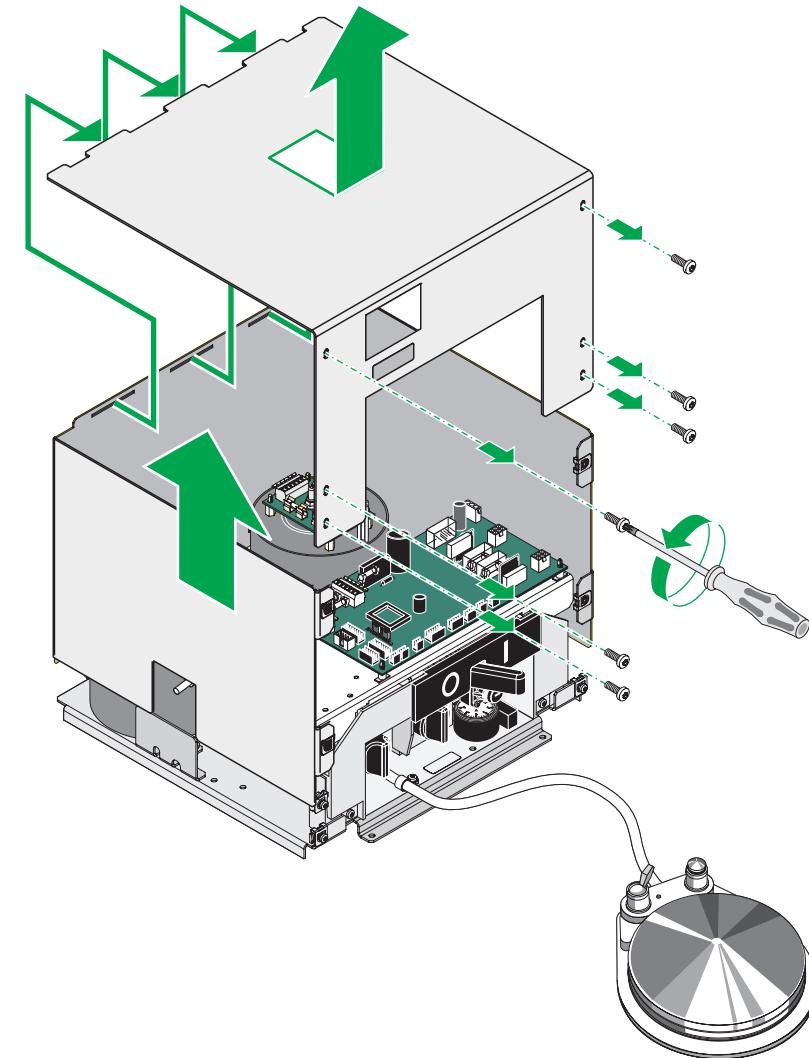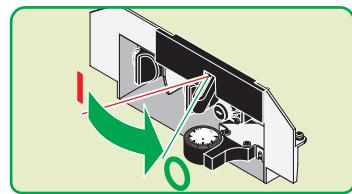


Junction box

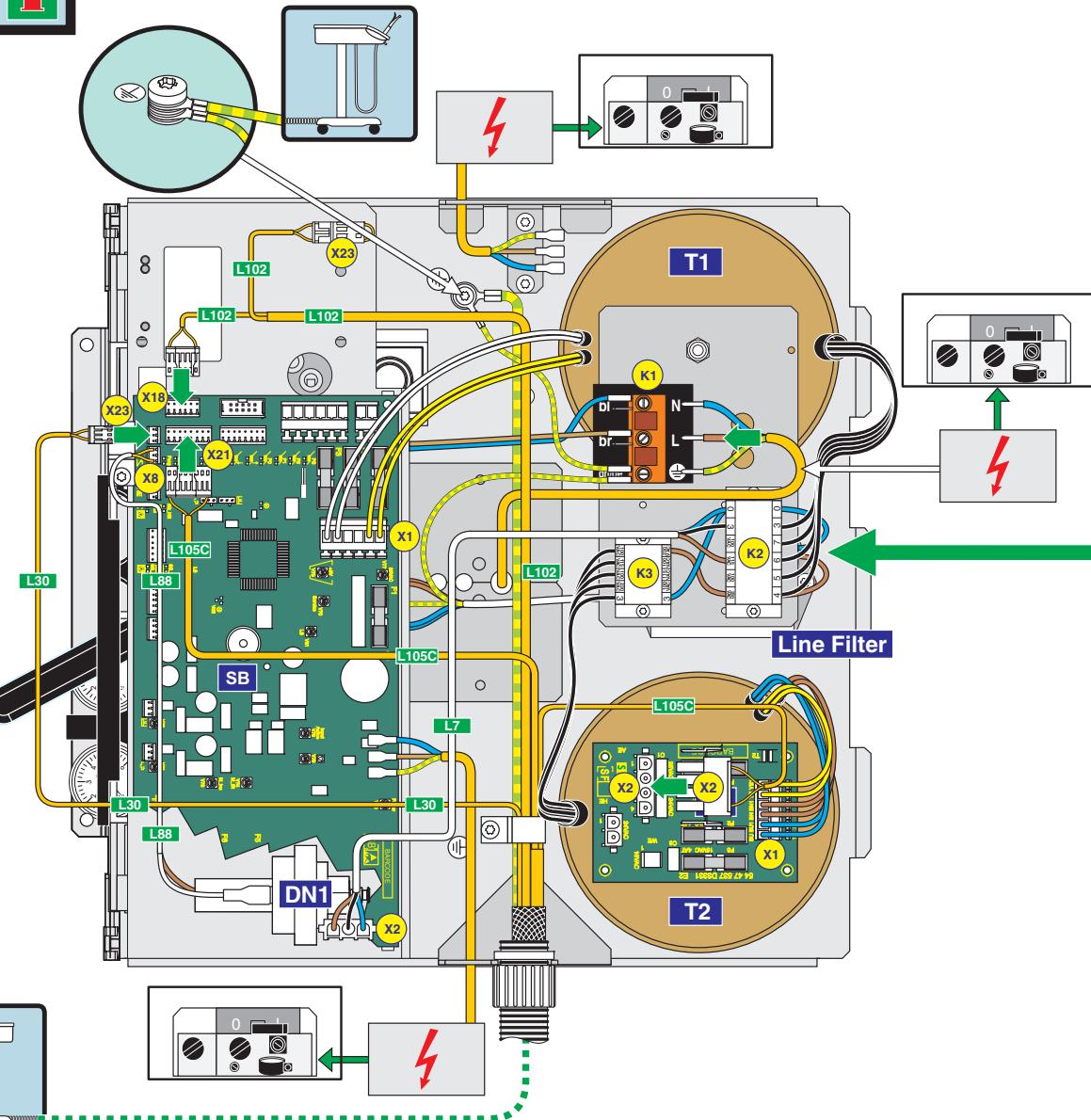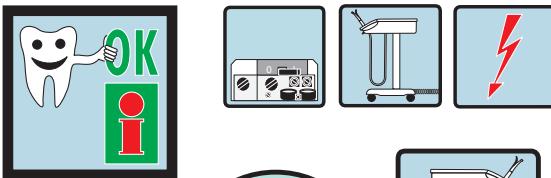


230VAC

BU: K2.0-K3.3
BN: K2.230VAC-K3.230VAC

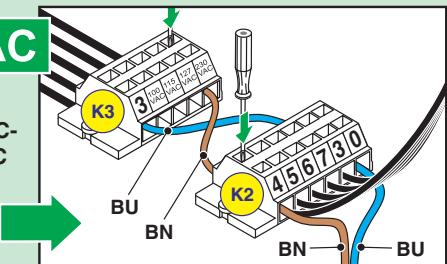


127VAC

BU: K2.0-K3.3
BN: K2.127VAC-K3.127VAC

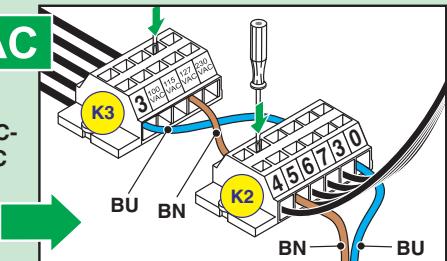


115VAC


BU: K2.0-K3.3
BN: K2.115VAC-K3.1 15VAC

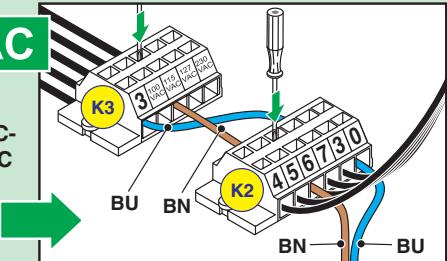



100VAC

BU: K2.0-K3.3
BN: K2.100VAC-K3.100V AC

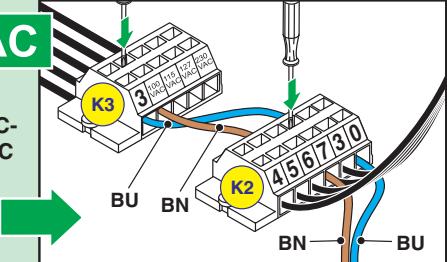


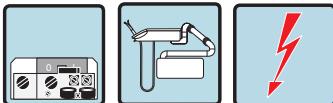


Junction box


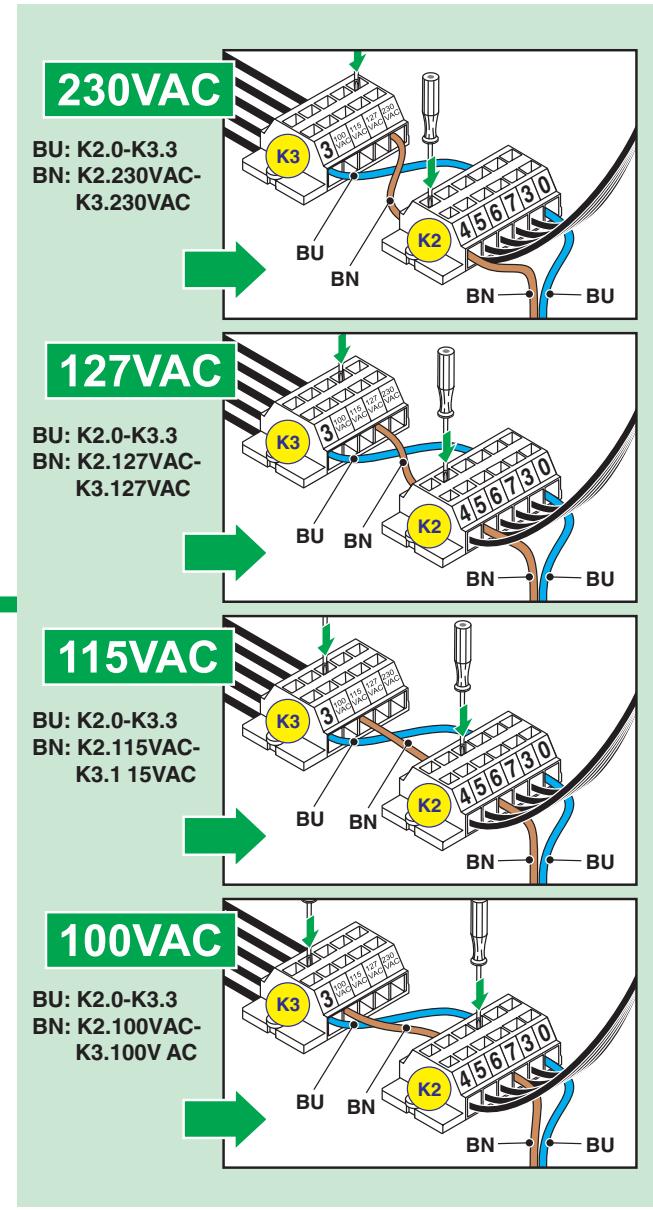
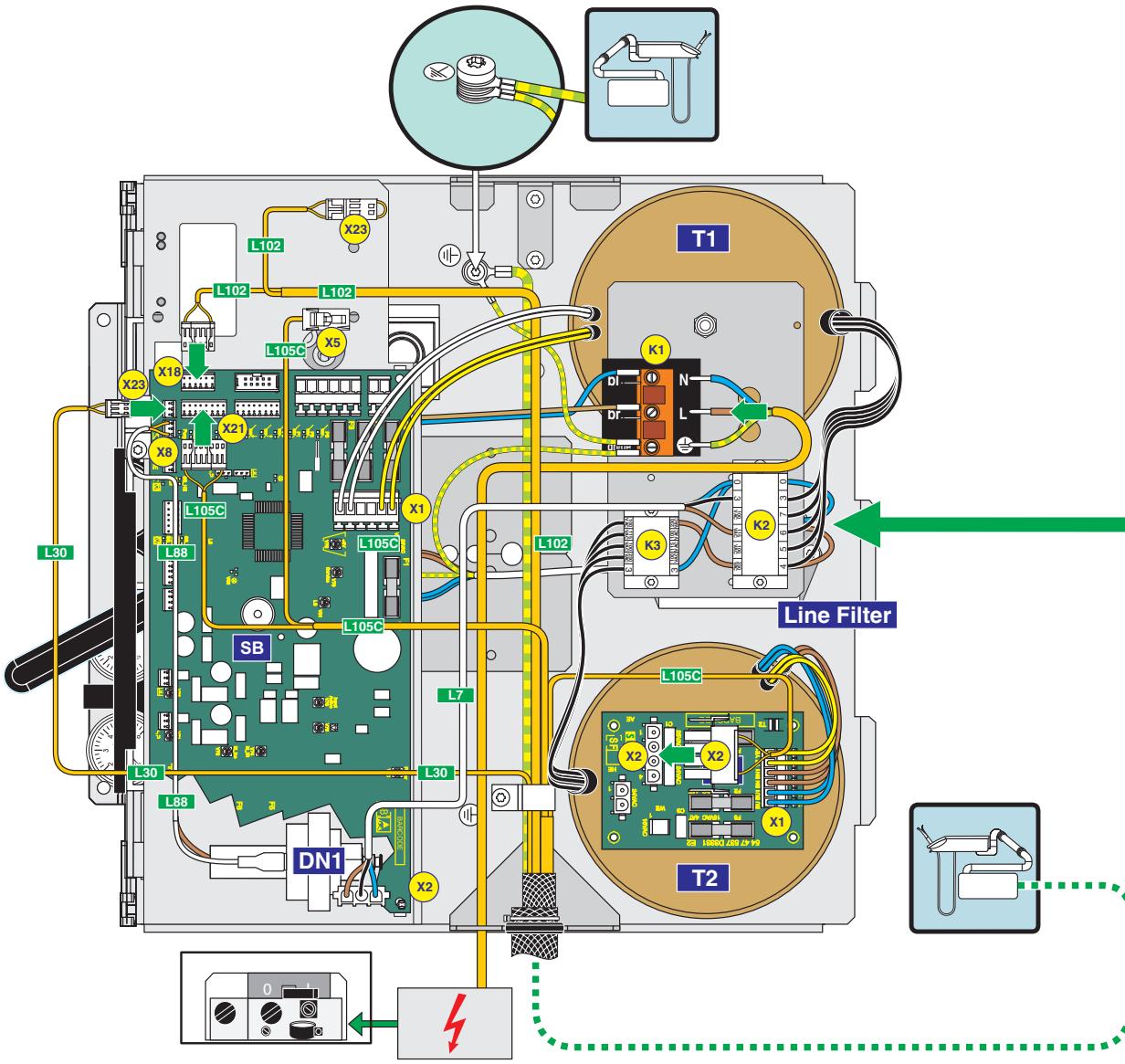





Junction box

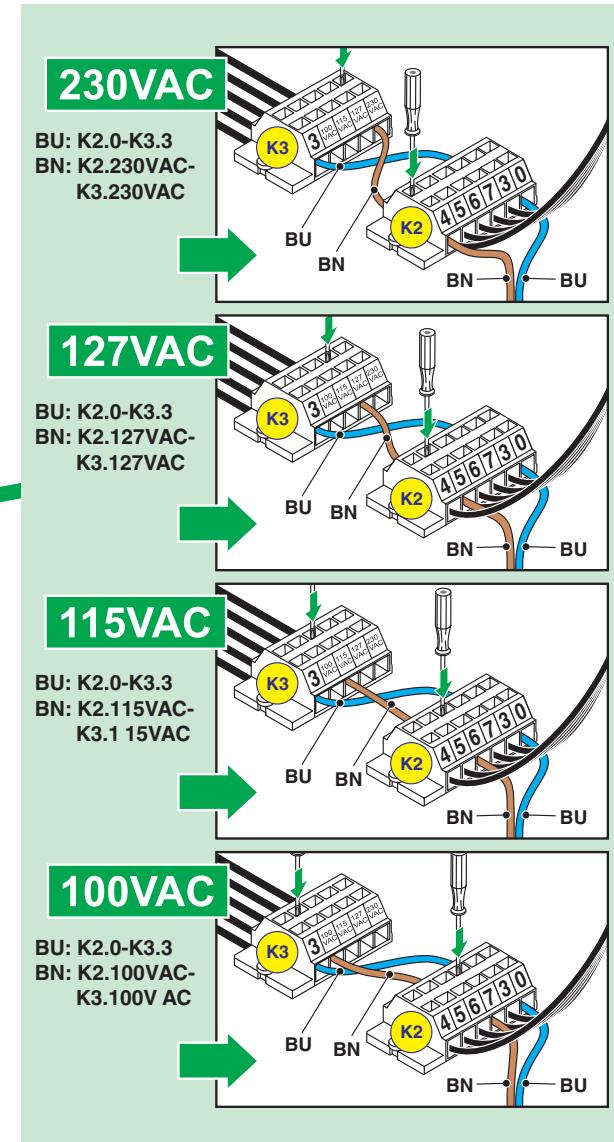
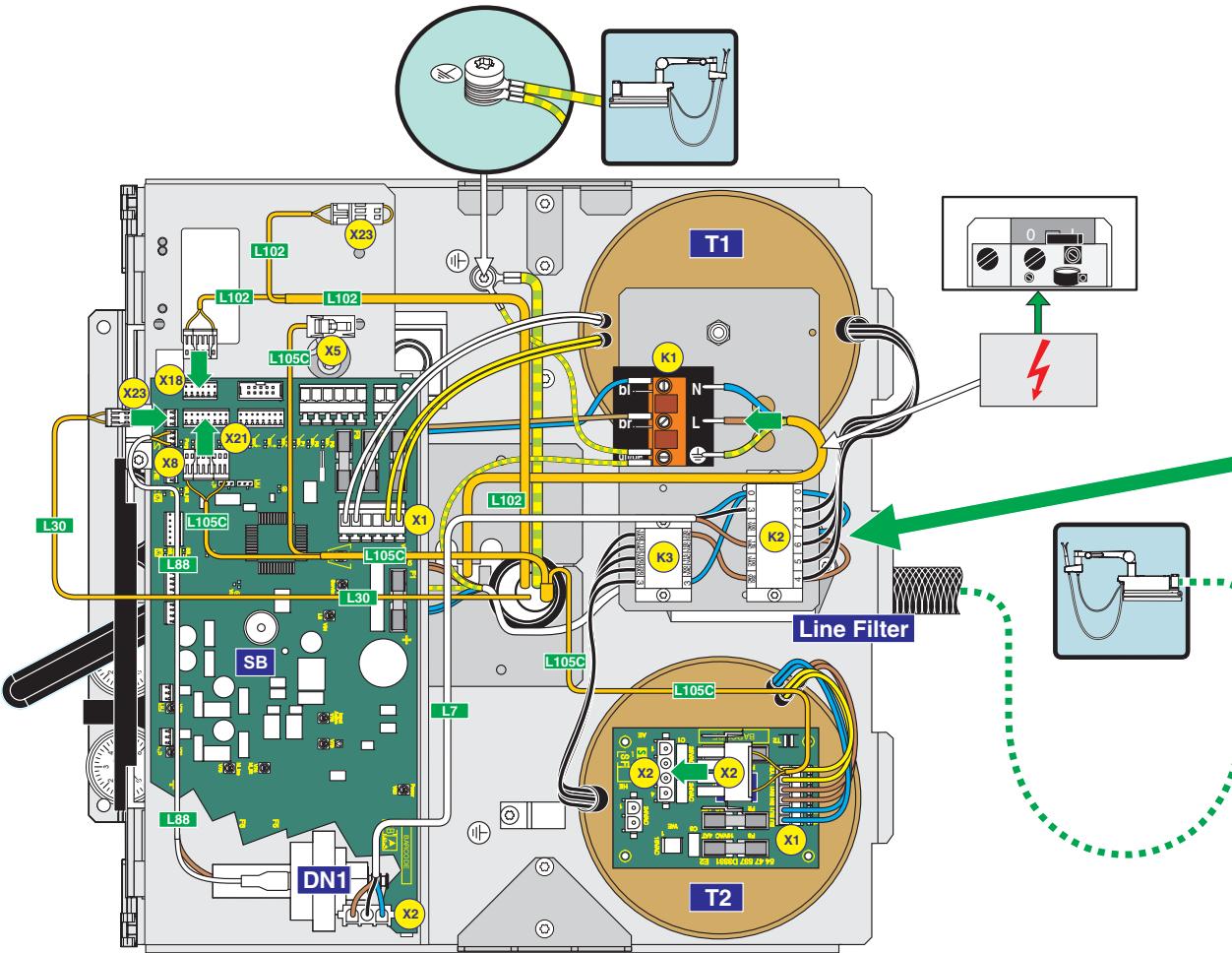


Junction box

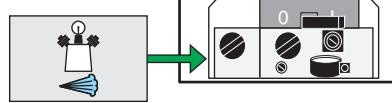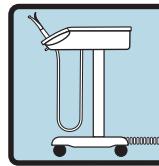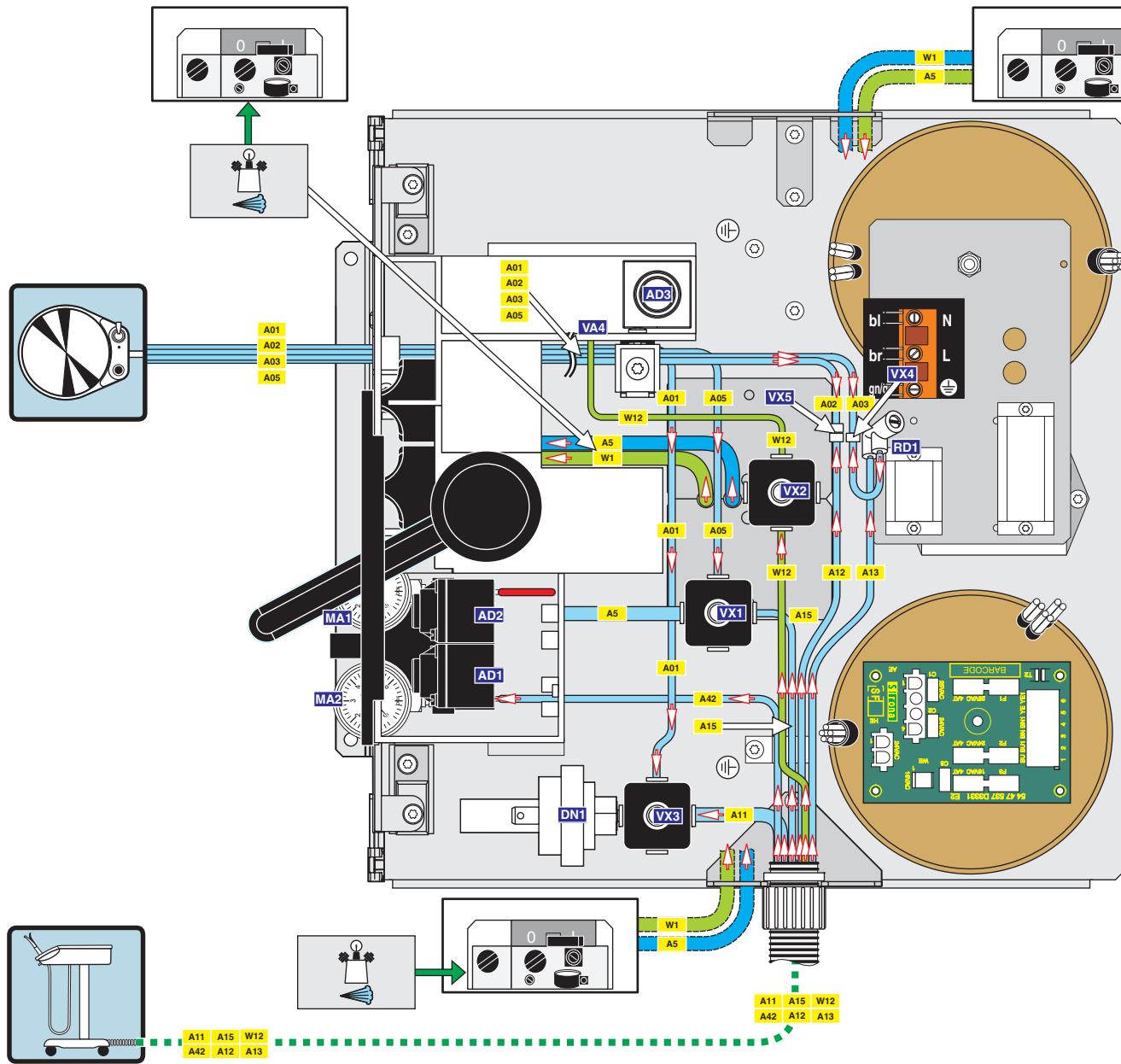



A11 A15 W12
A42 A12 A13

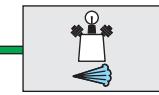

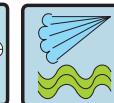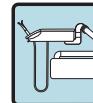


Junction box

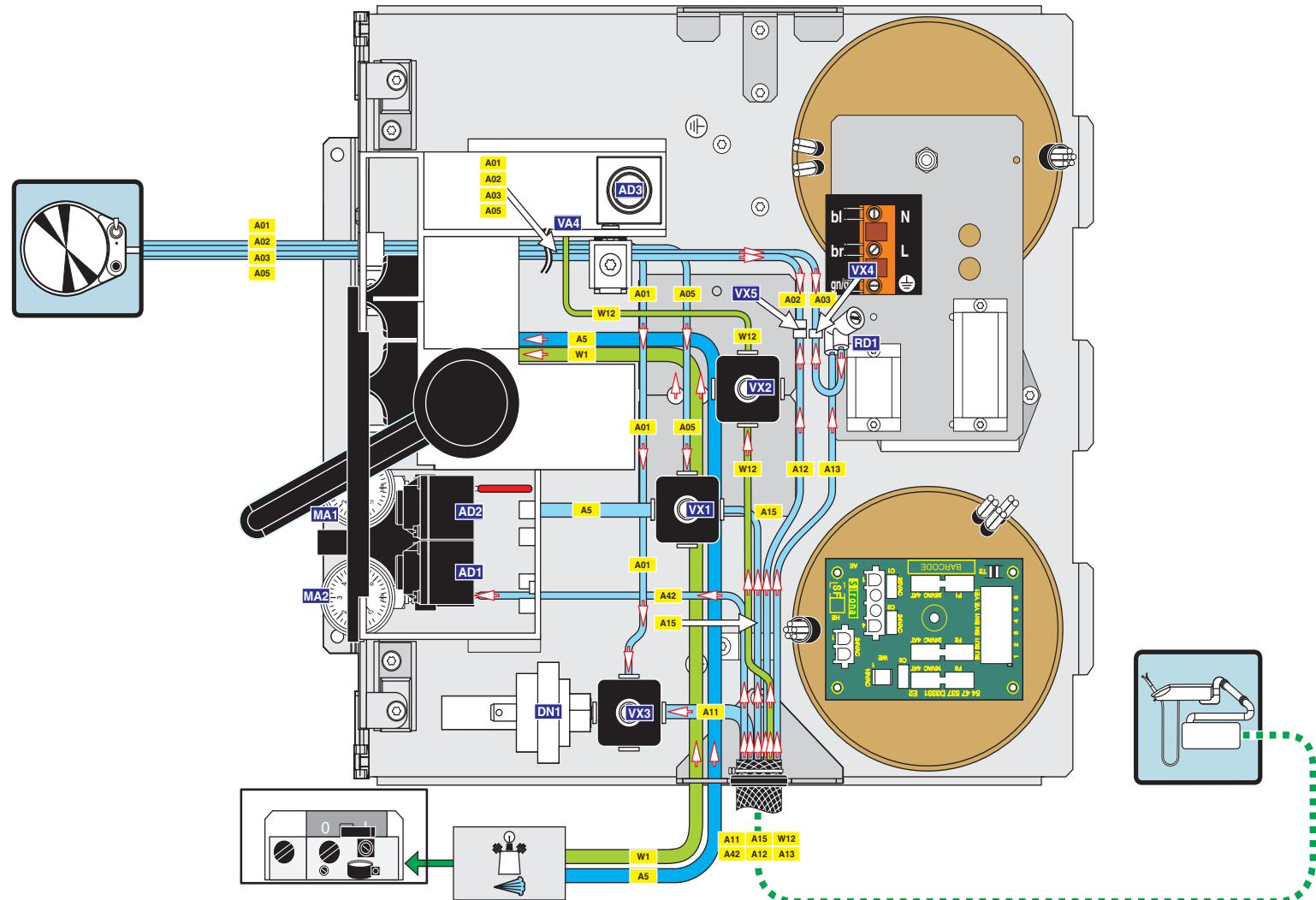



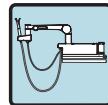



Junction box

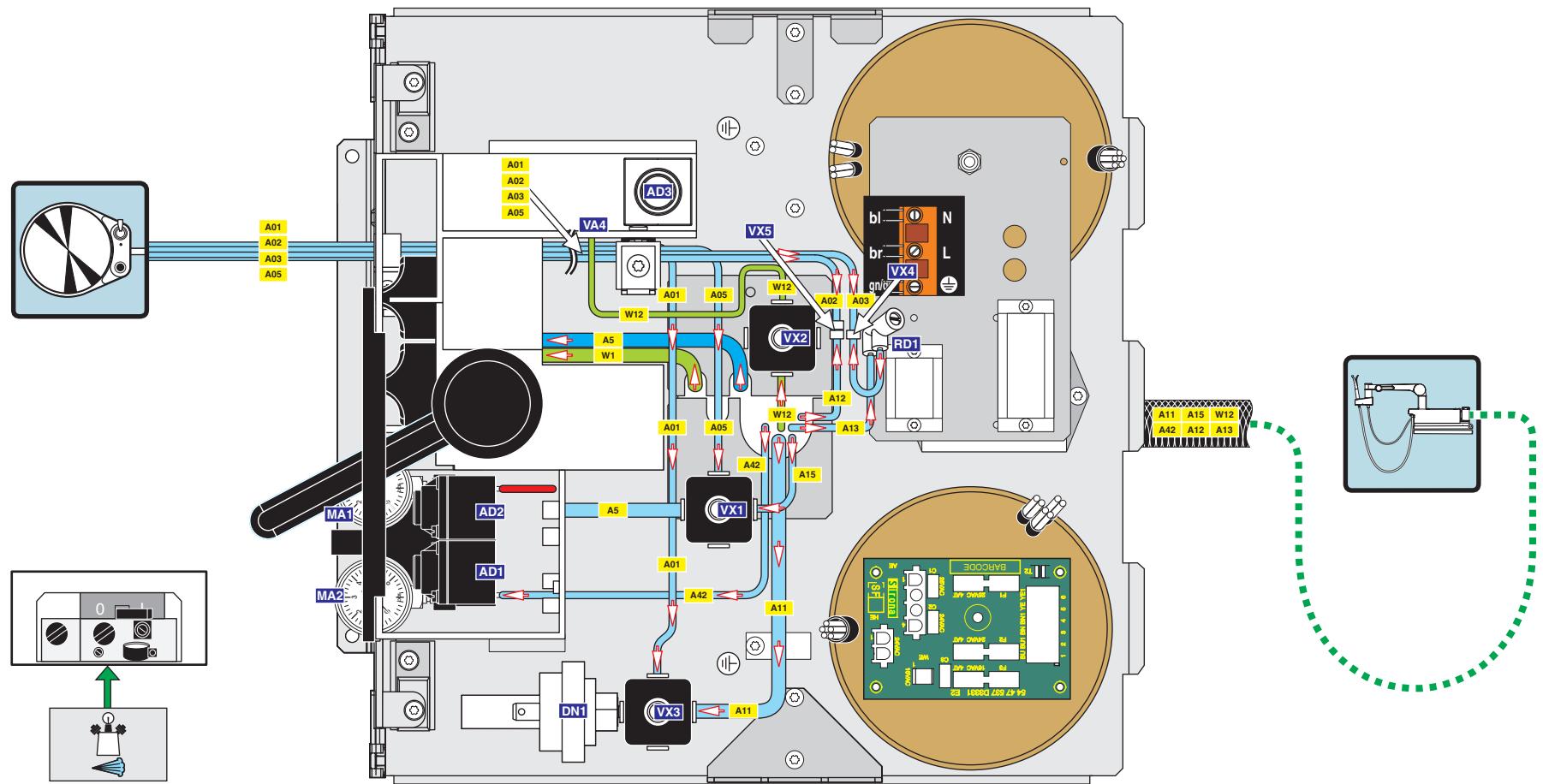






C8+ Turn

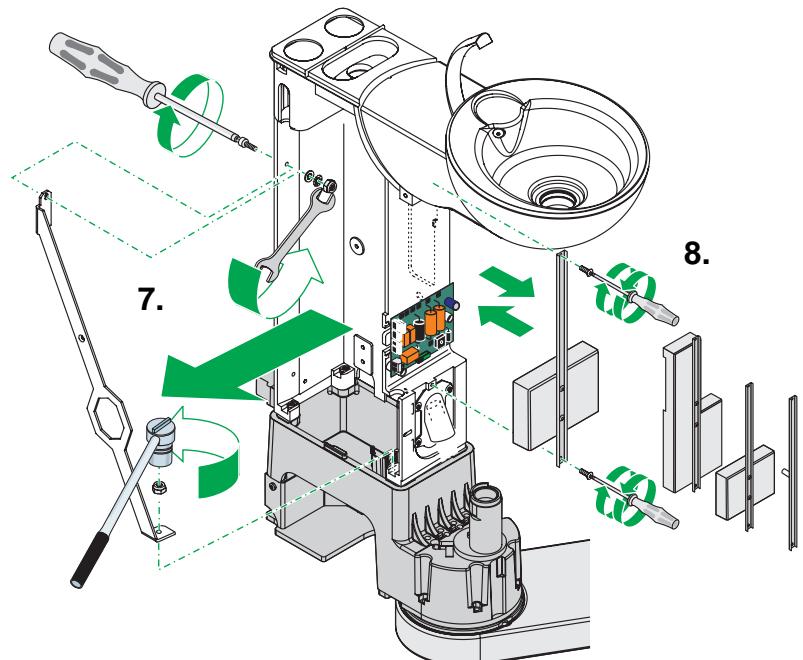
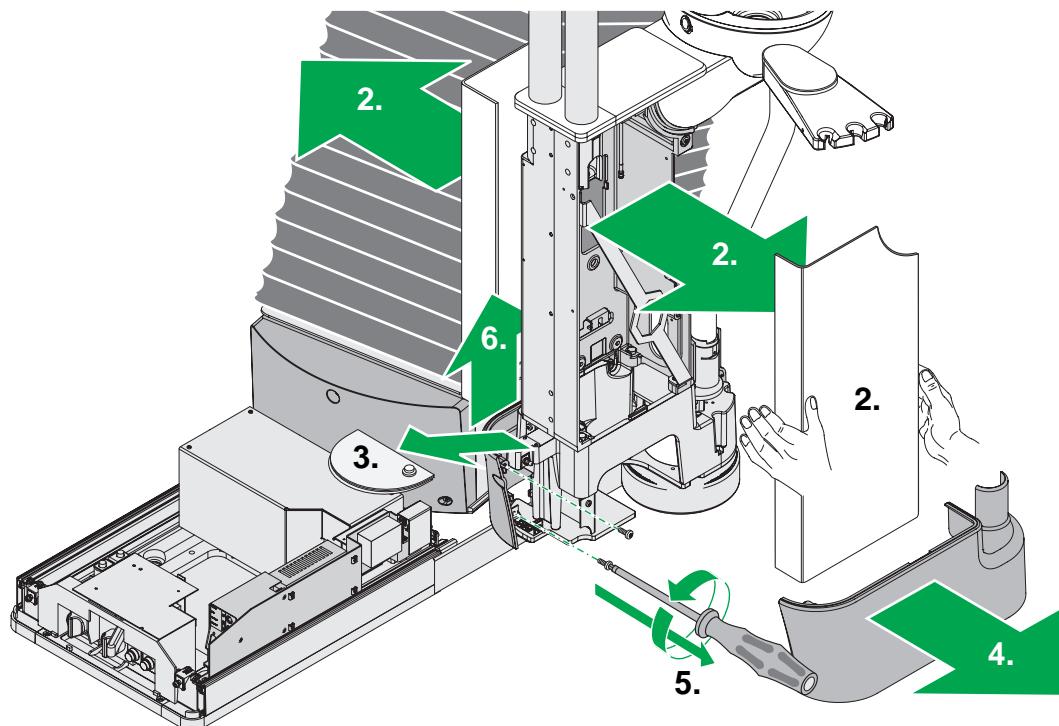
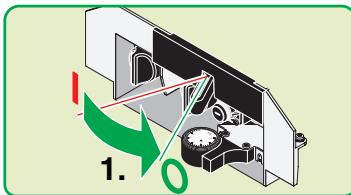


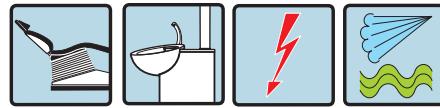

C8+ Turn

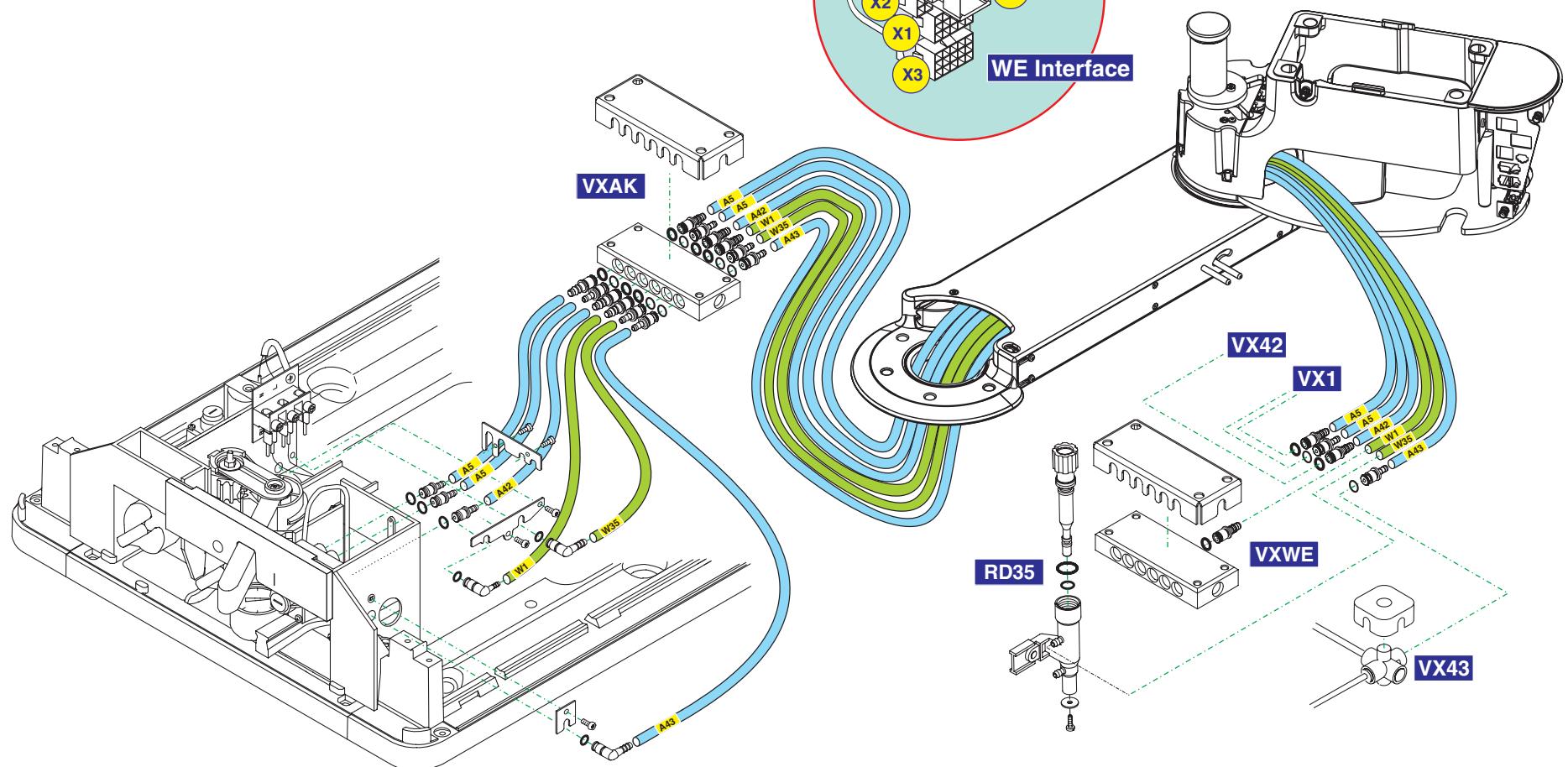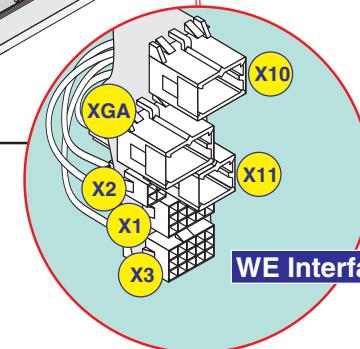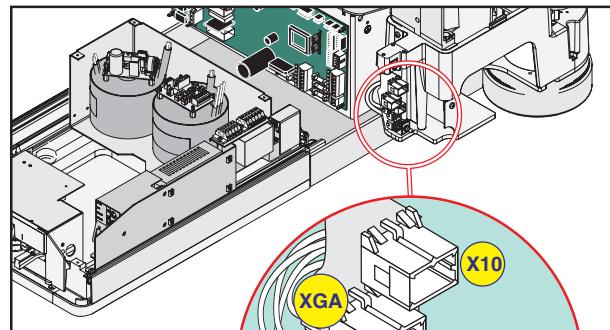


C8+ Turn

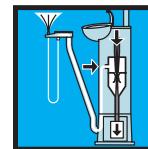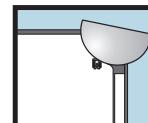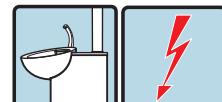

Luftstrahlpumpe
Air Venturi
Option



Purge
Option

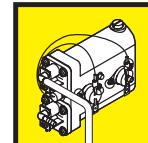


Wasserheizer
Water heater
Option



Helferinelement
Assistant Element




SIROLUX

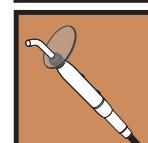




Mini L.E.D.
Option



Amalgamscheidner
Amalgam Separator
Option

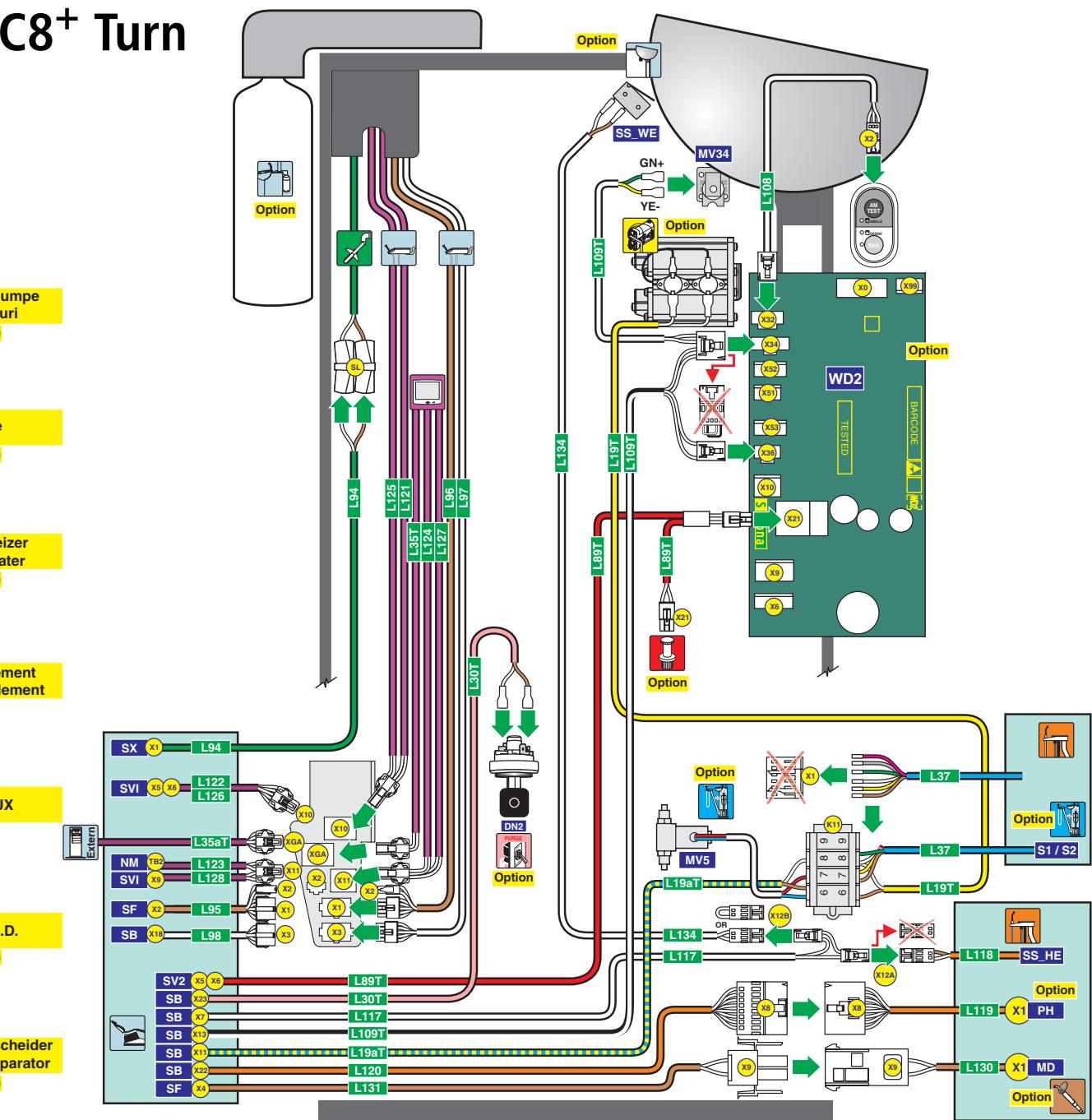


C8+ Turn, option chairline

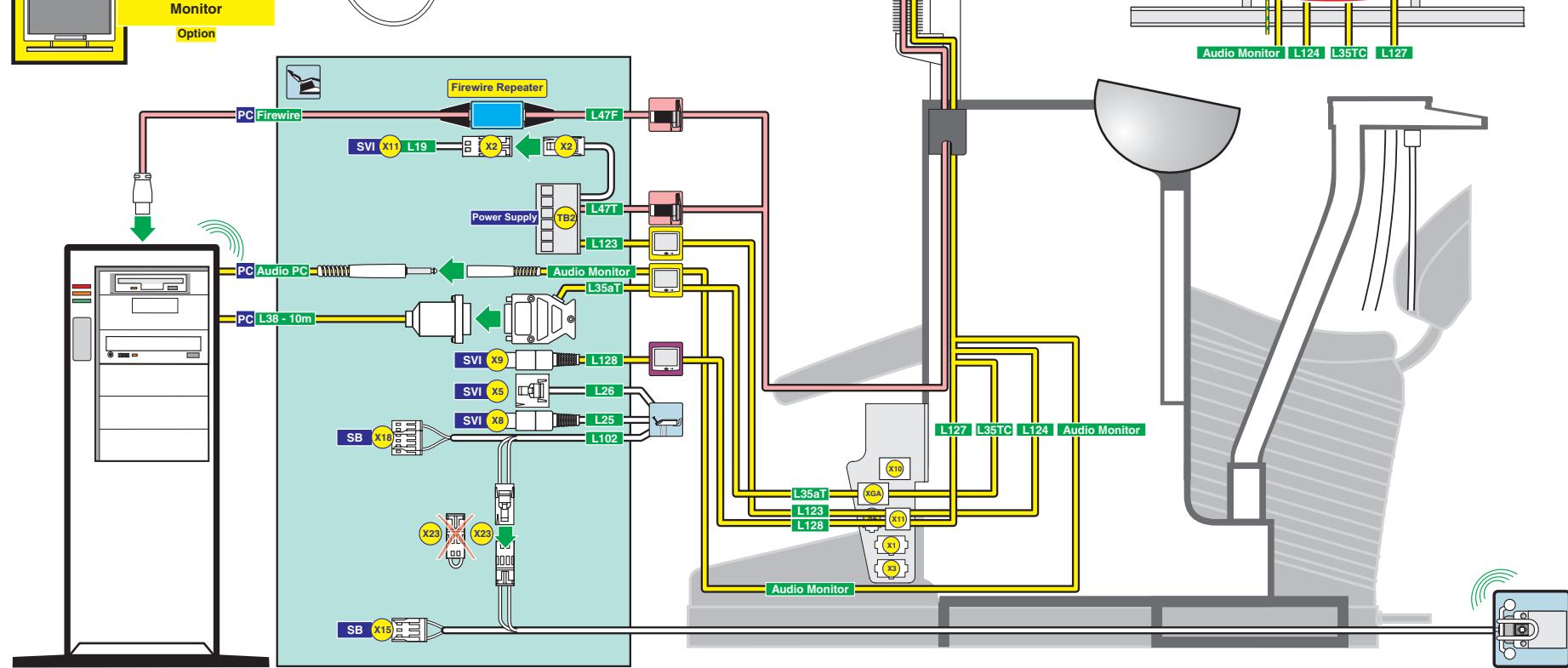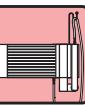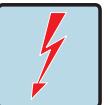



C8+ Turn

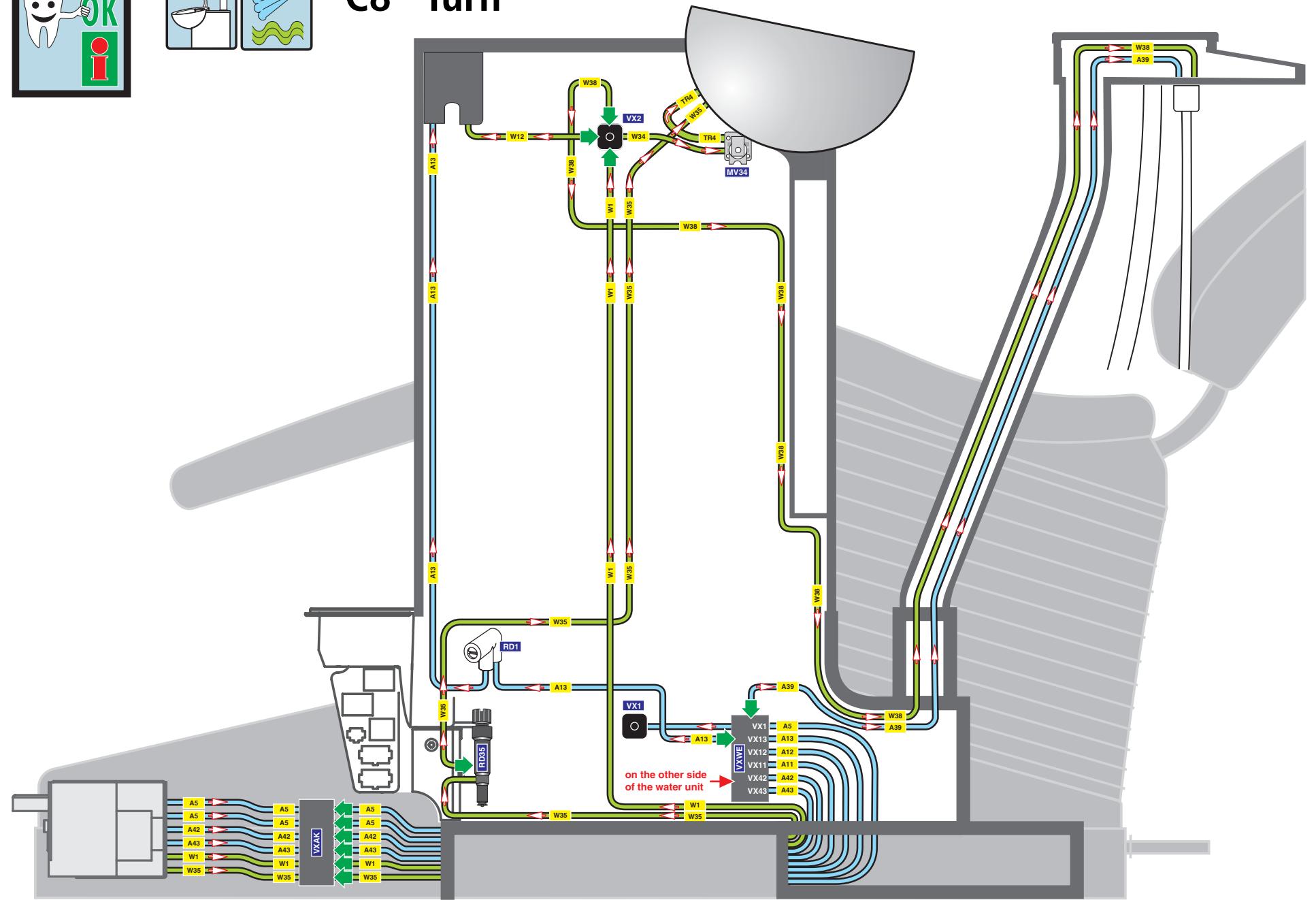


C8+ Turn




Wasserflasche
Water Bottle
Option




Purge
Option

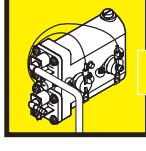

Wasserheizer
Water heater
Option

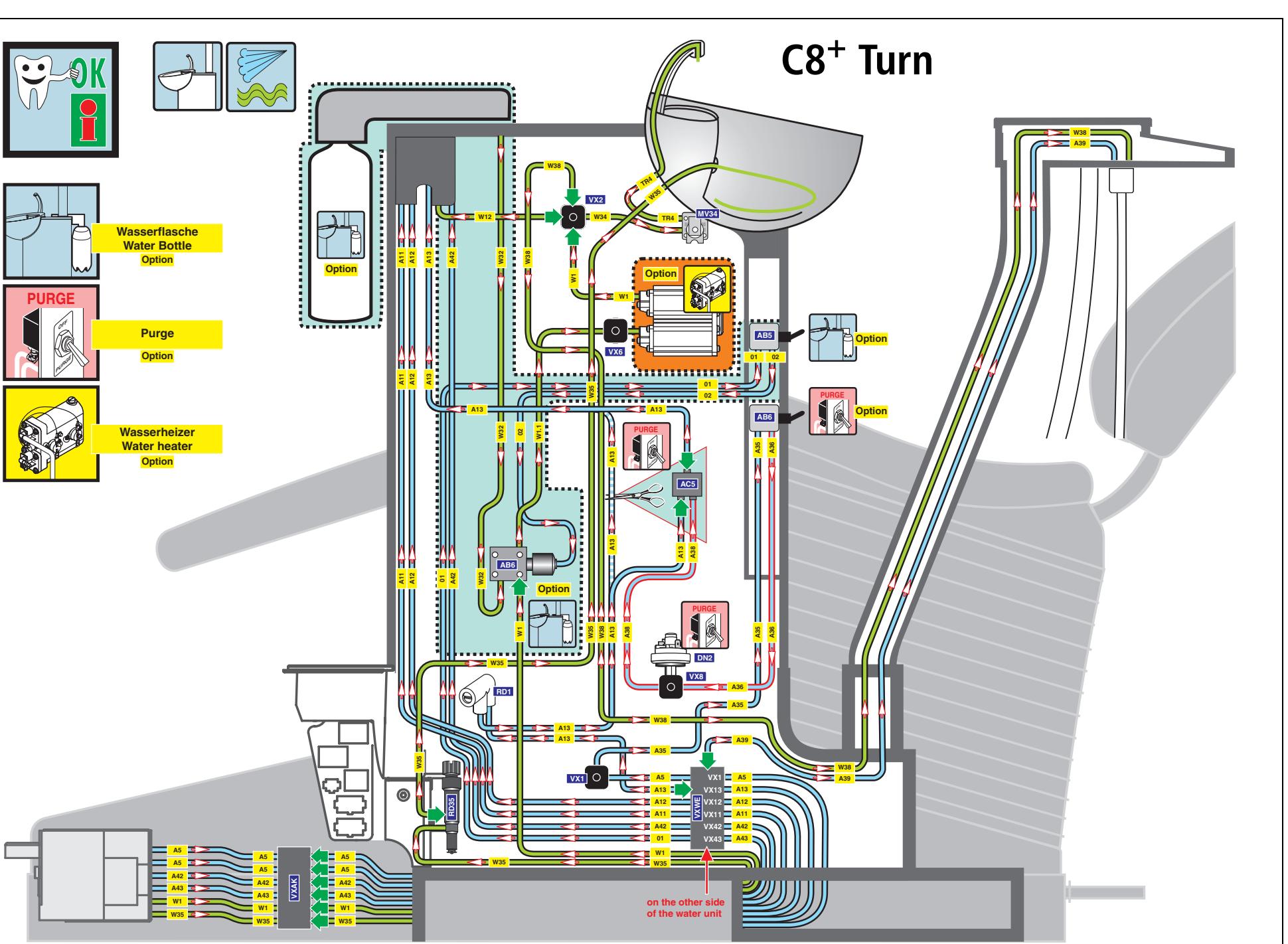

C8+ Turn

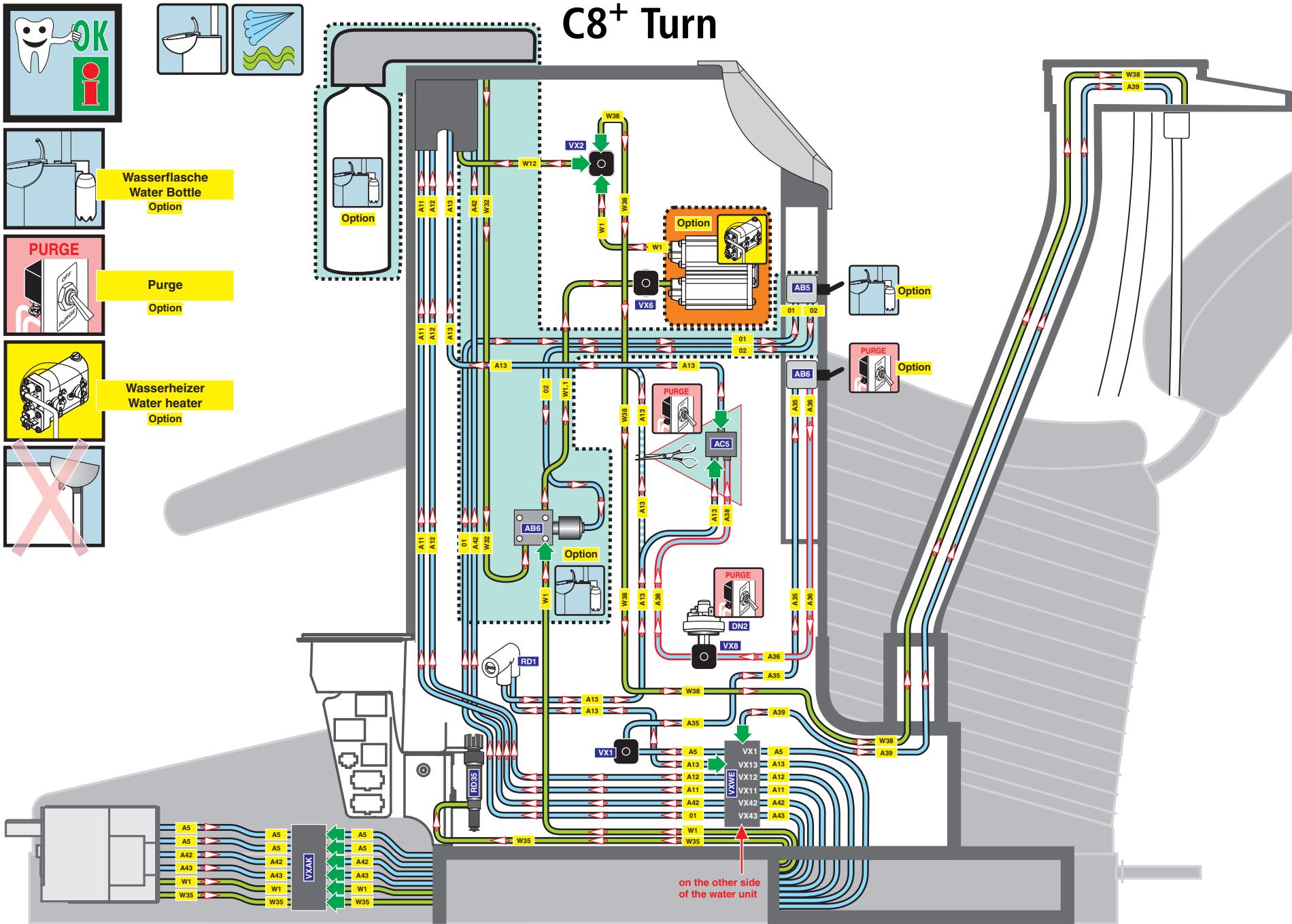






C8+ Turn







Desinfektionssystem
Disinfection System
Option


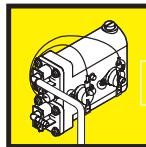





Wasserheizer
Water heater
Option

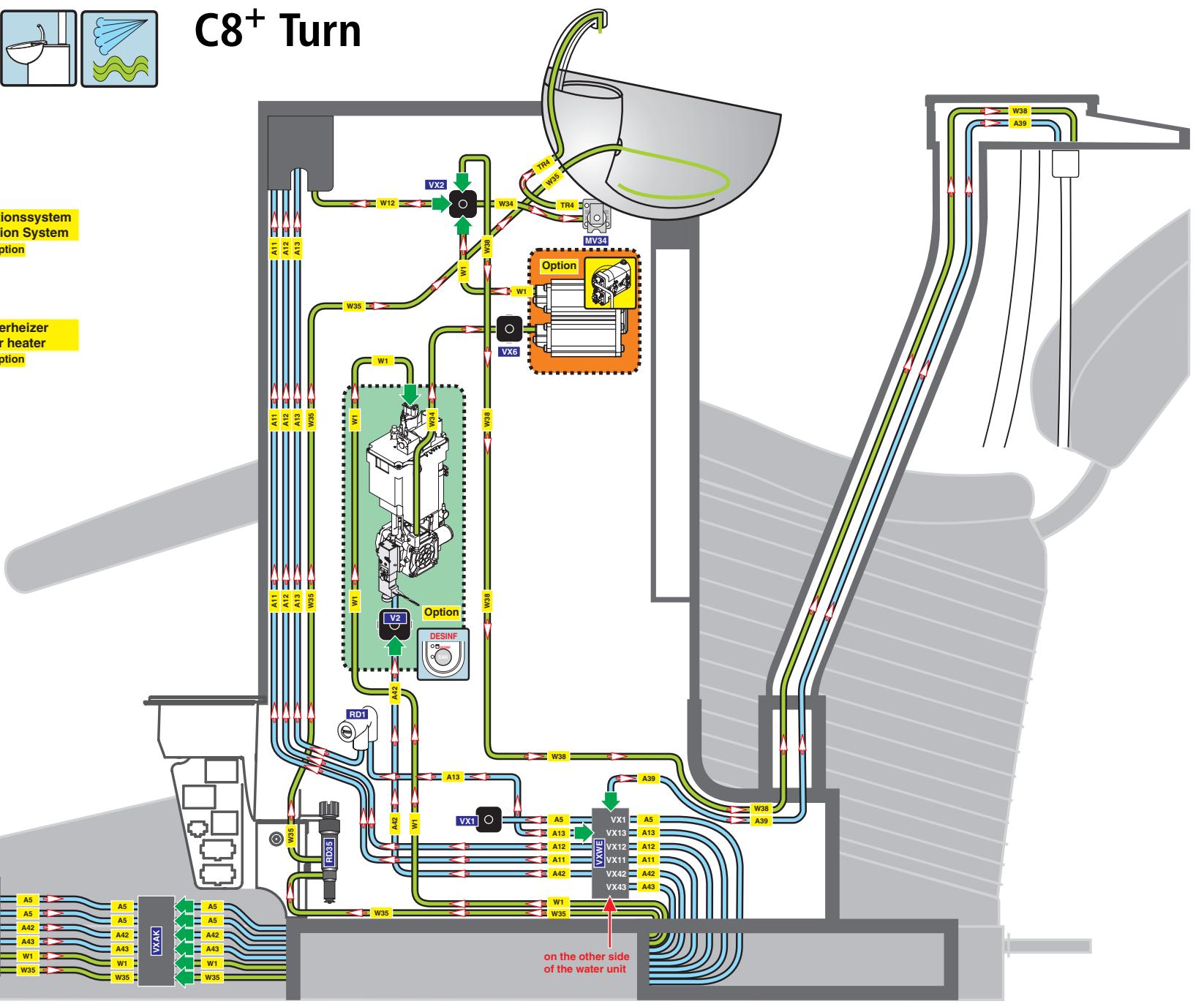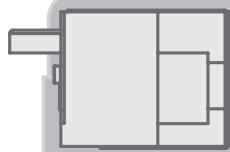


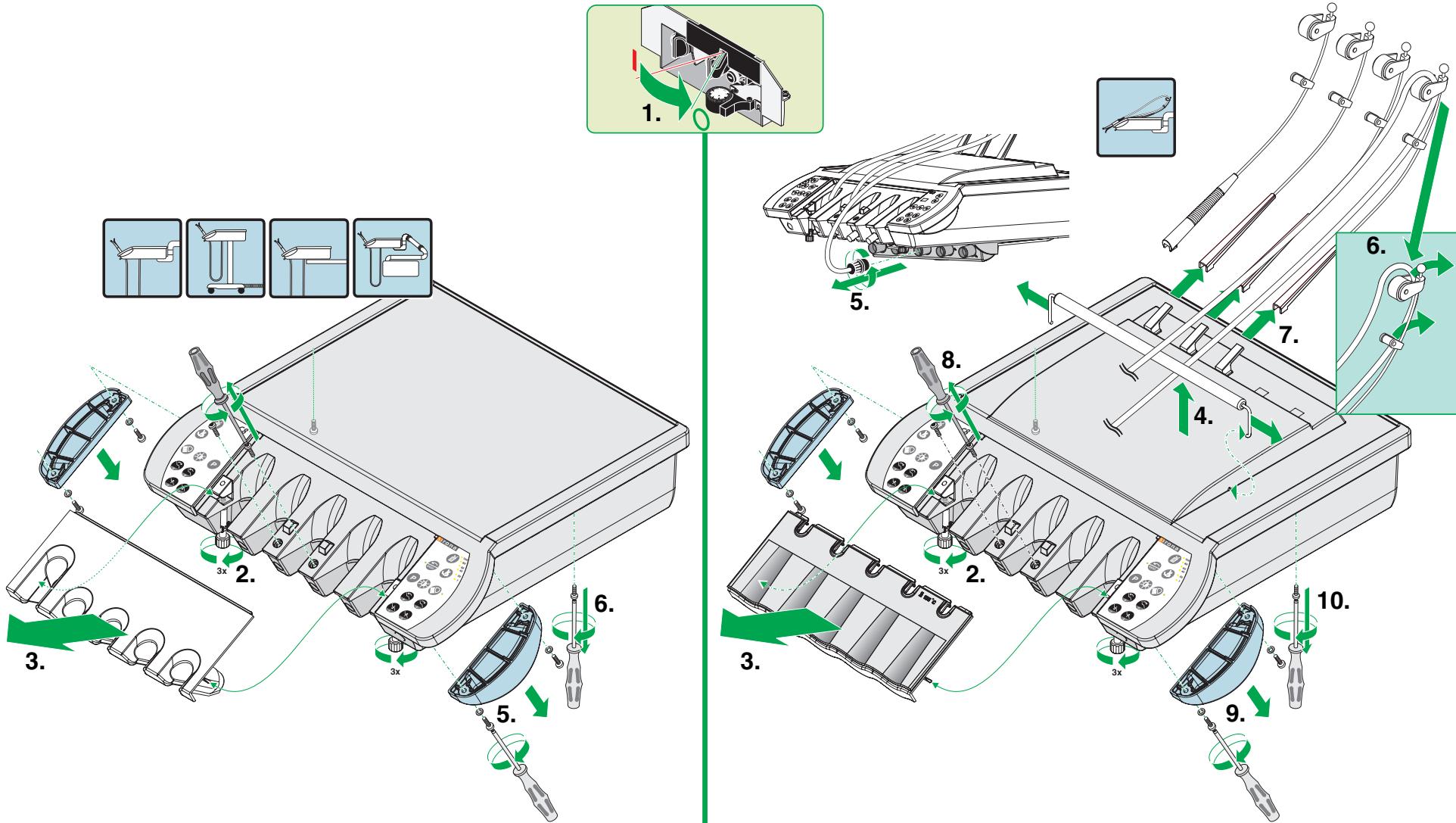
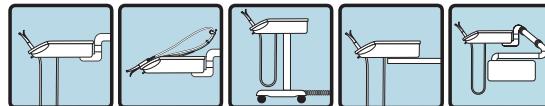





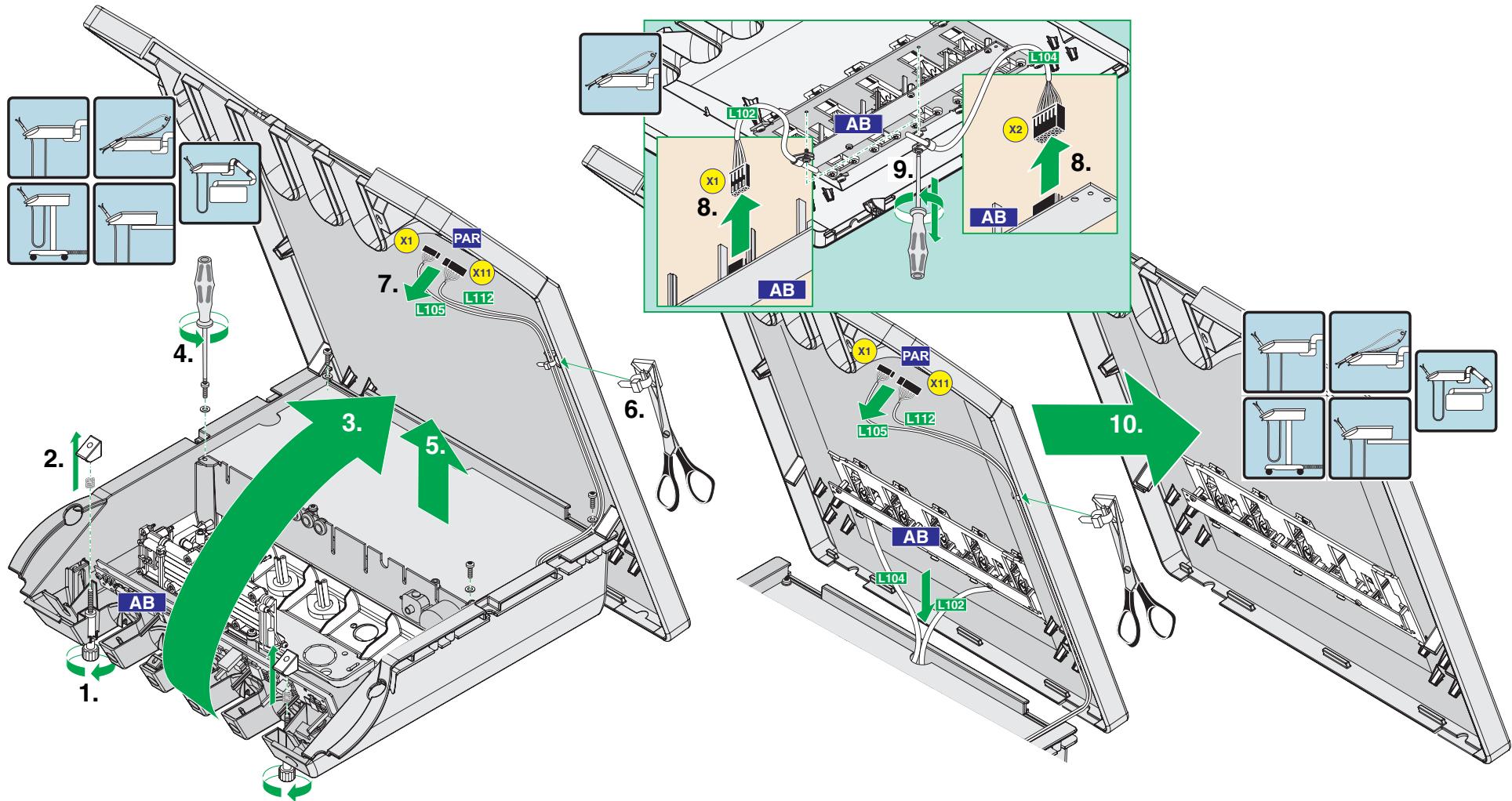
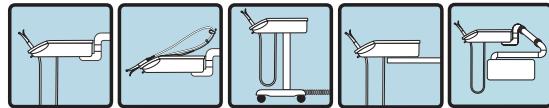






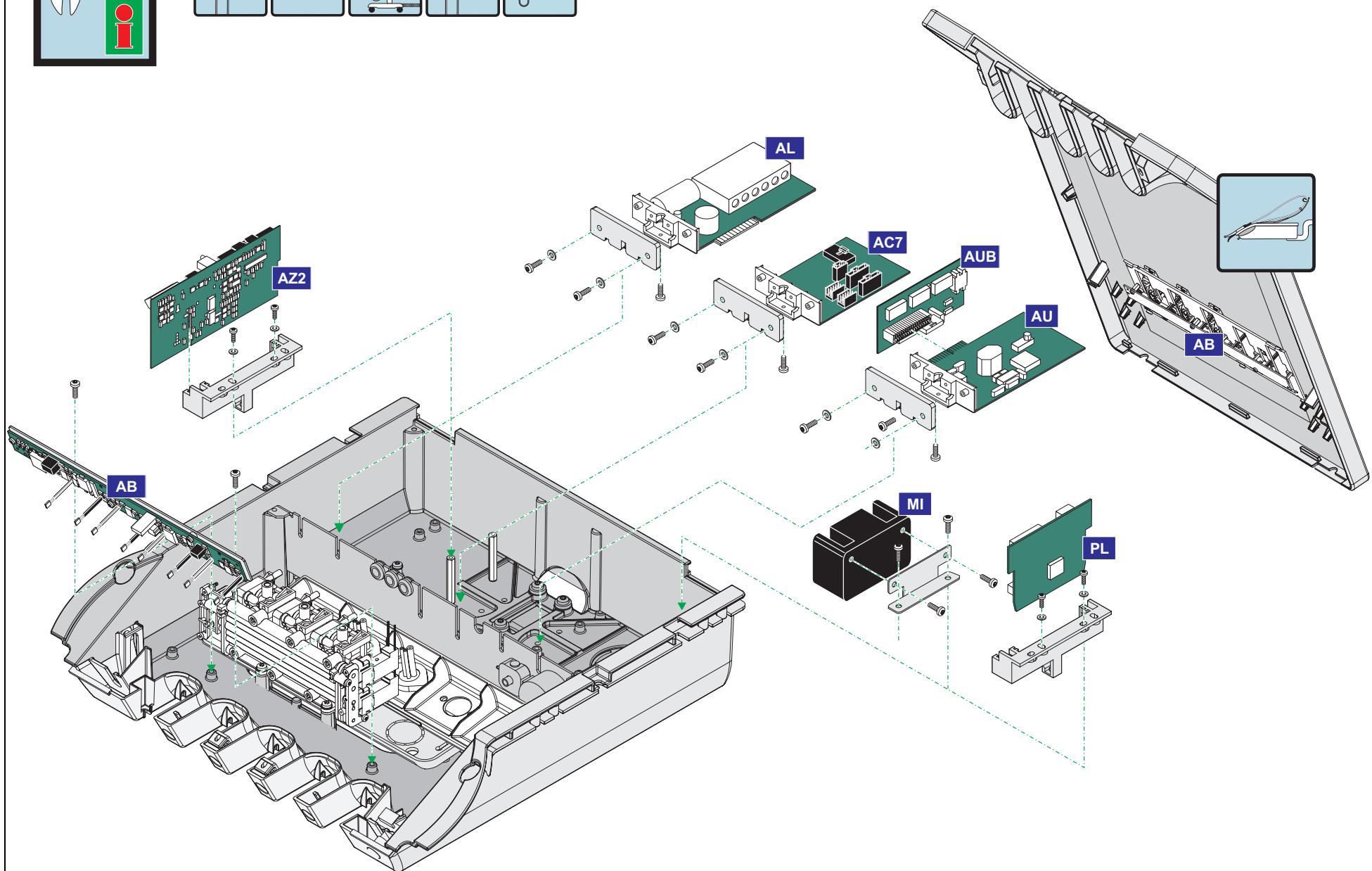
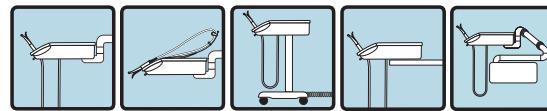


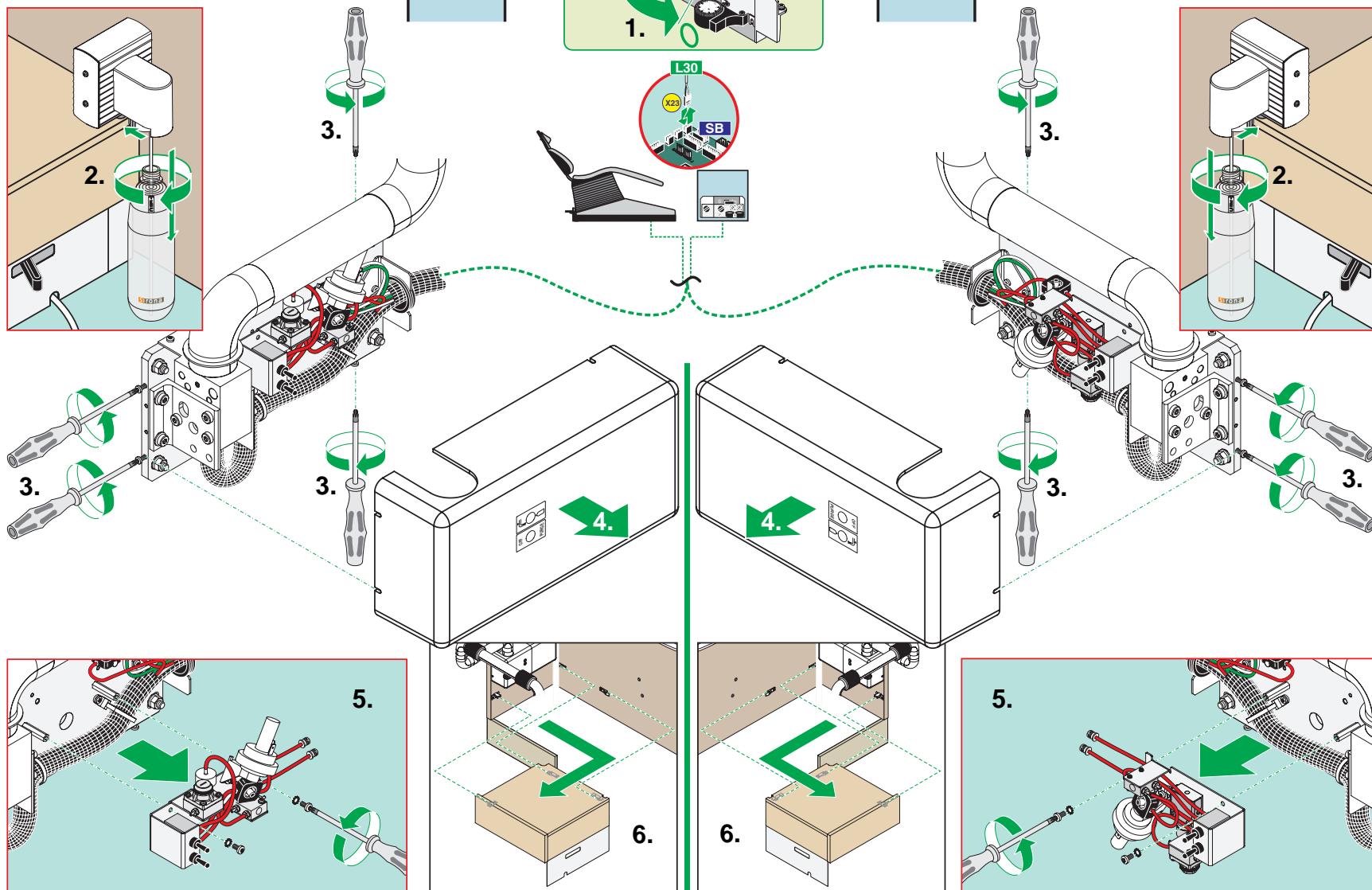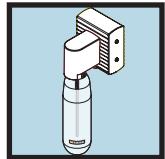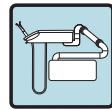



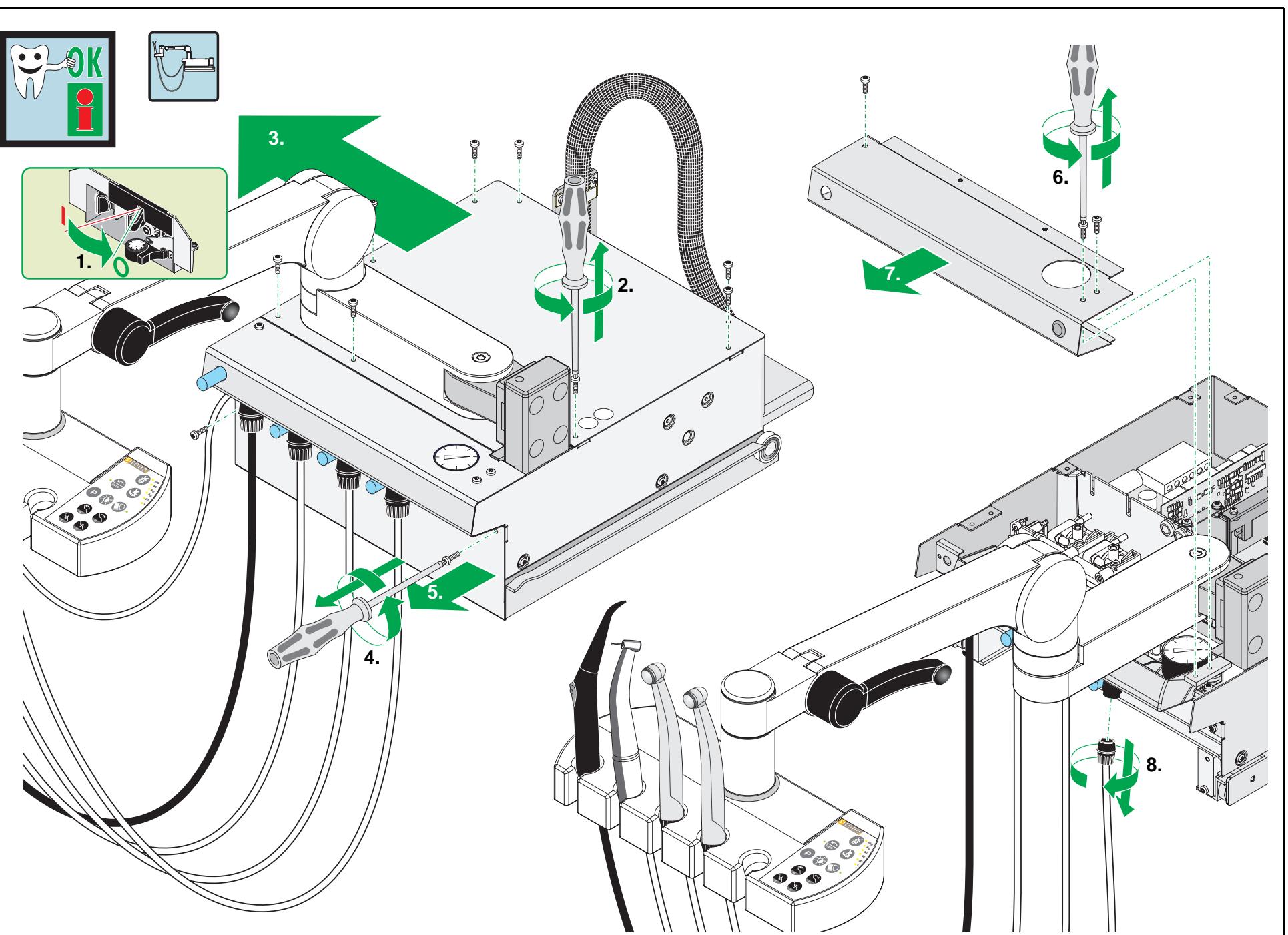


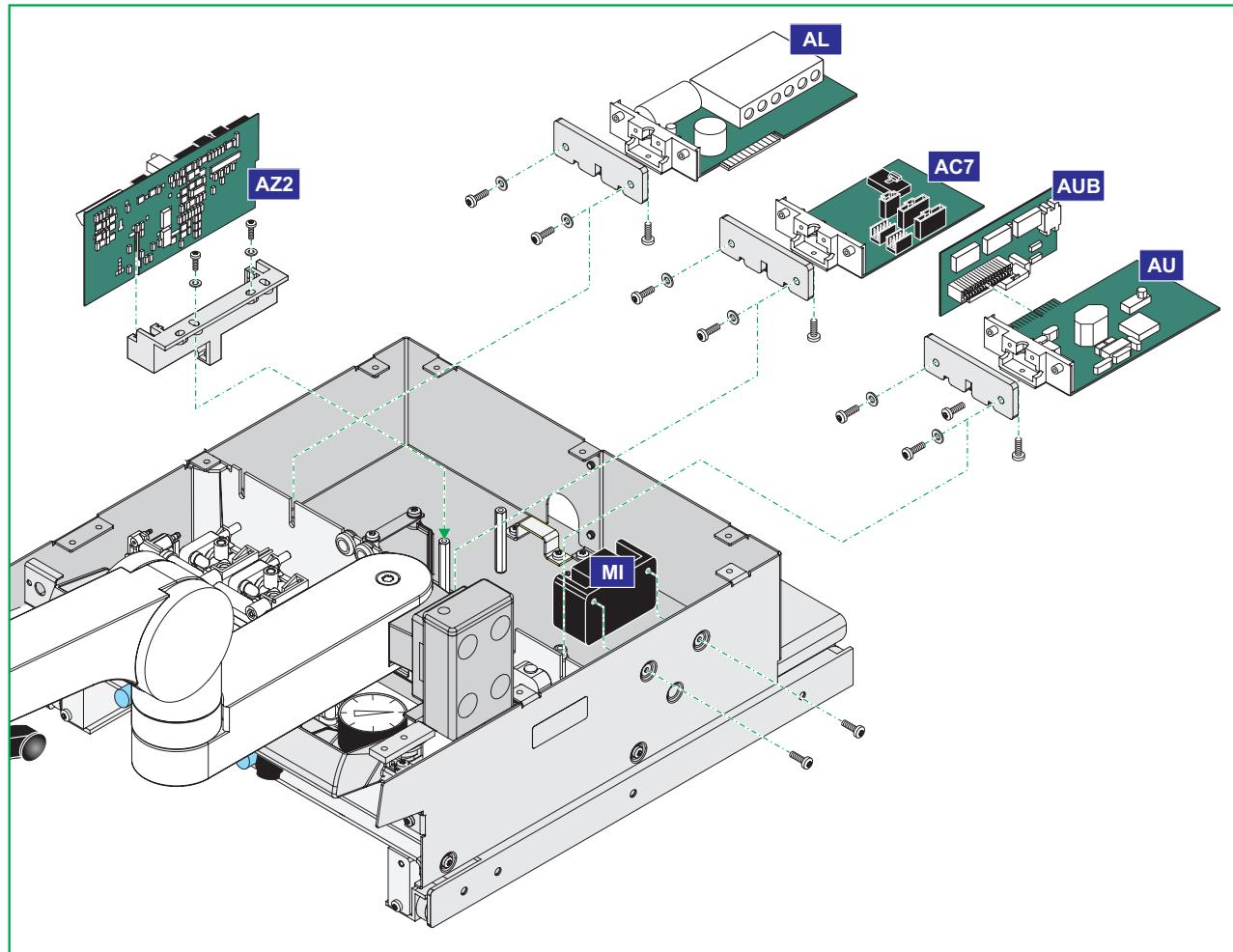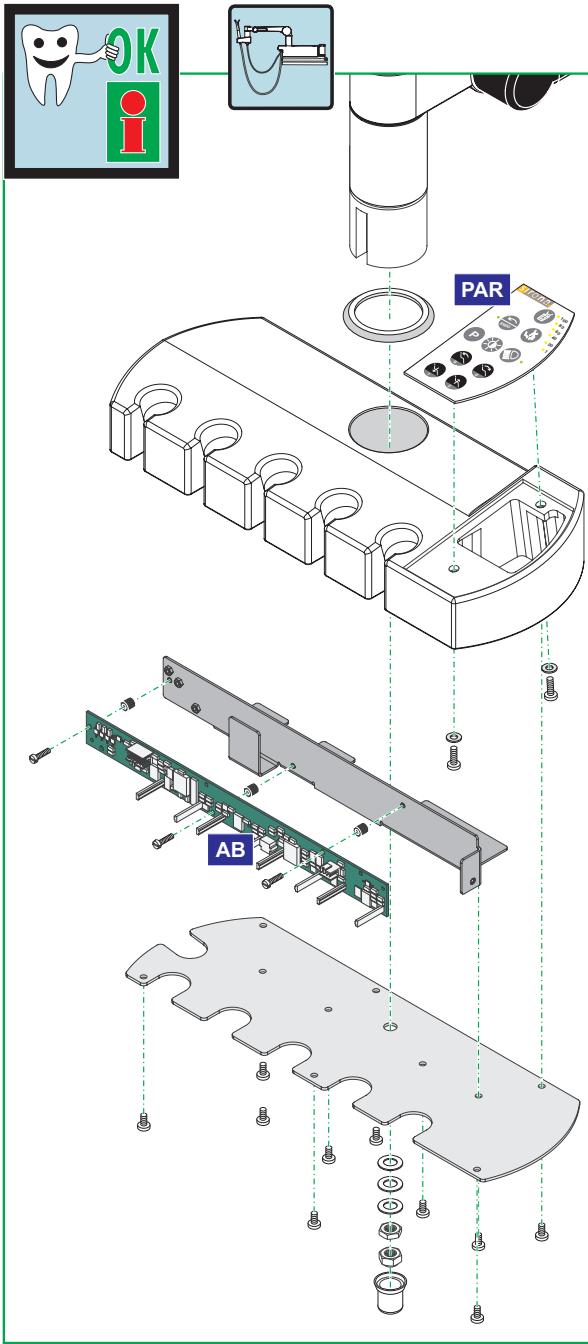

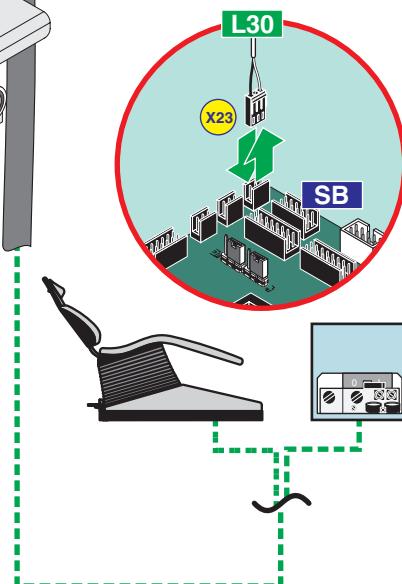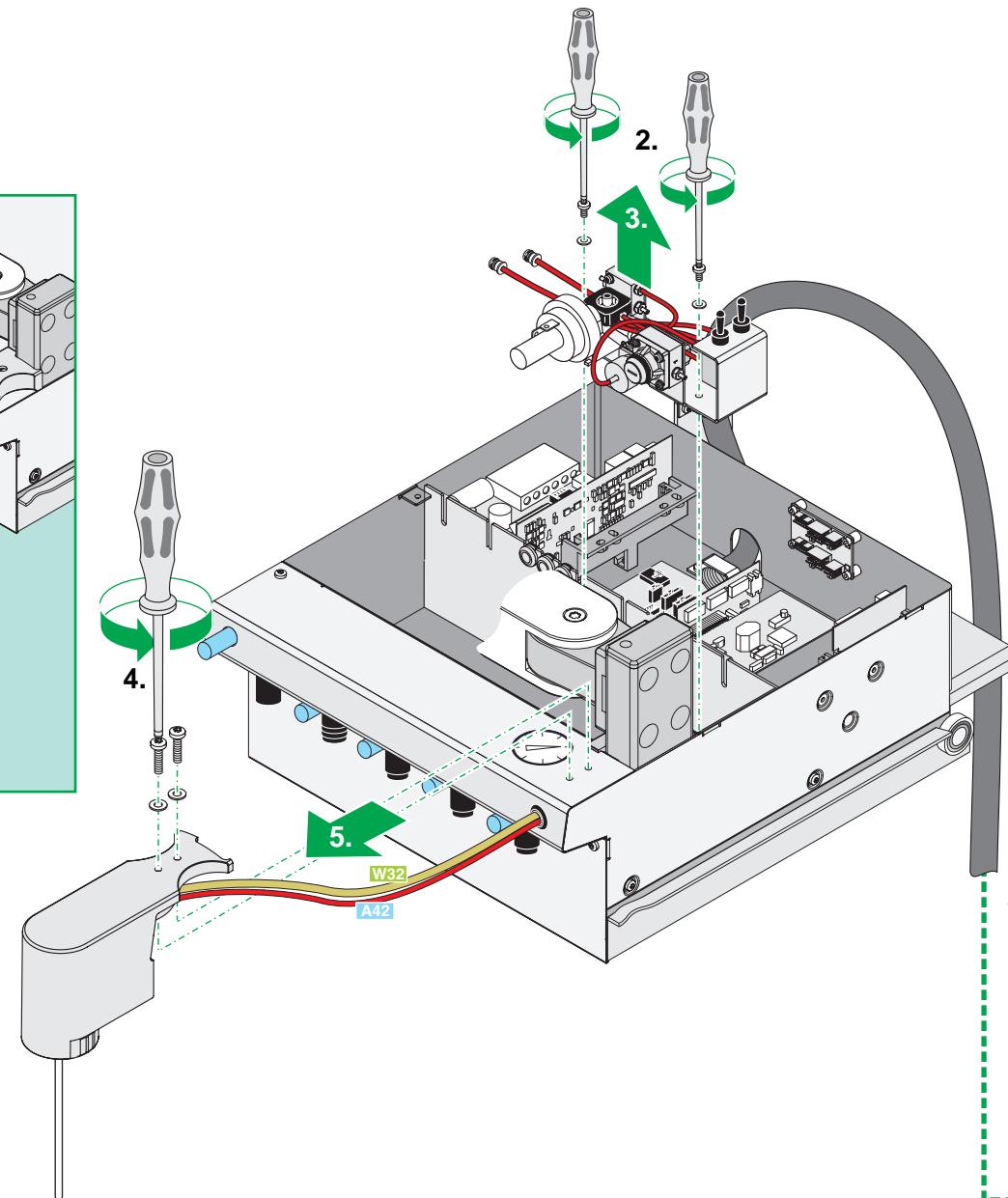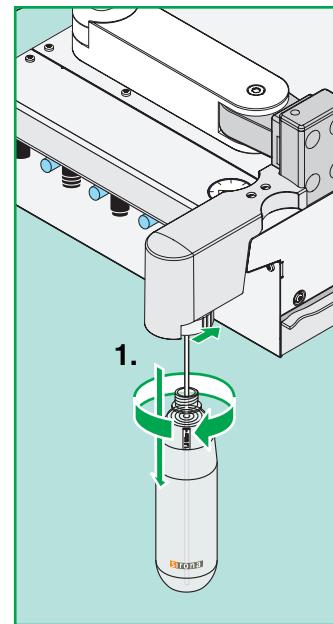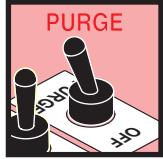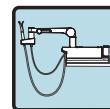

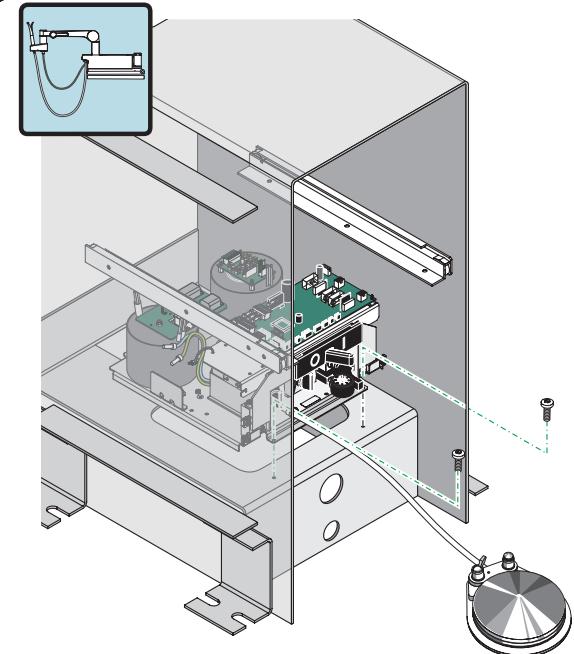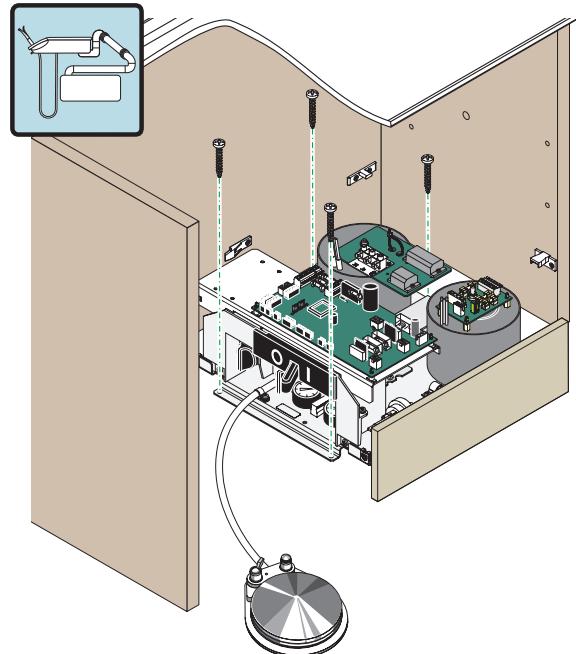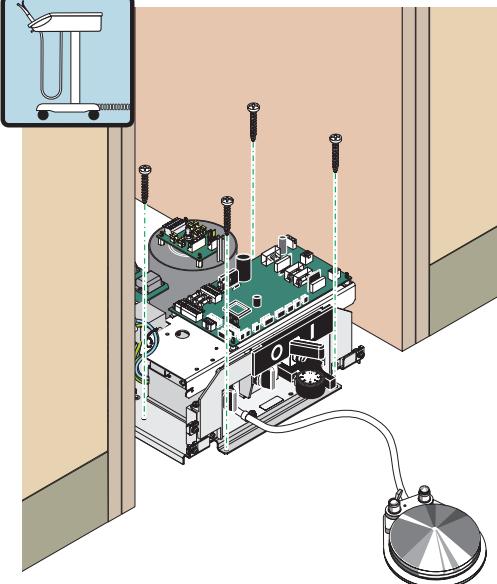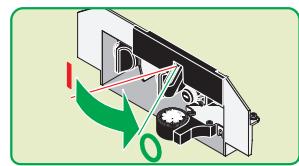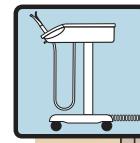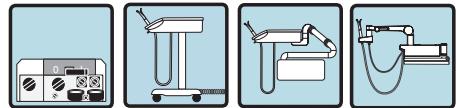


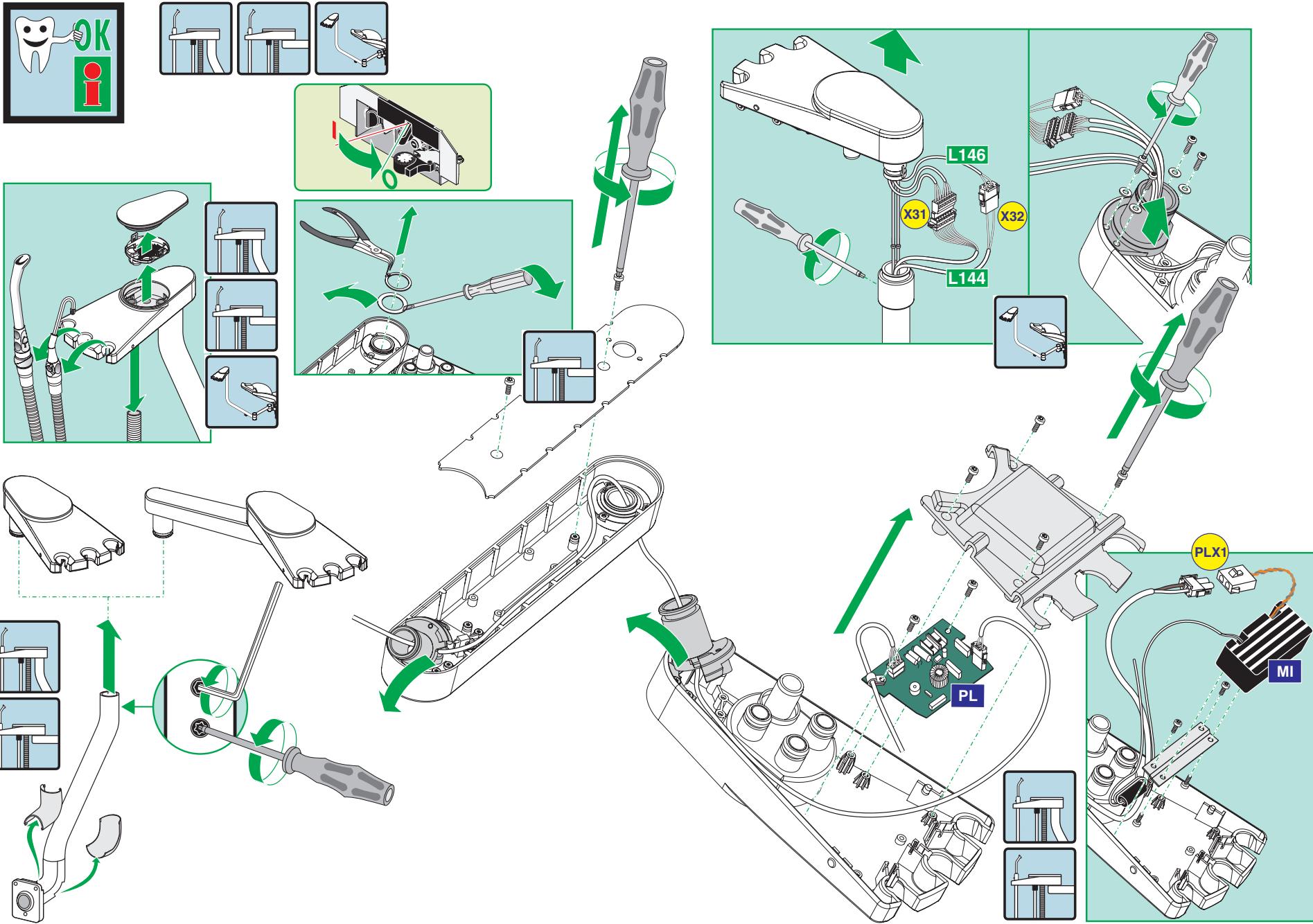

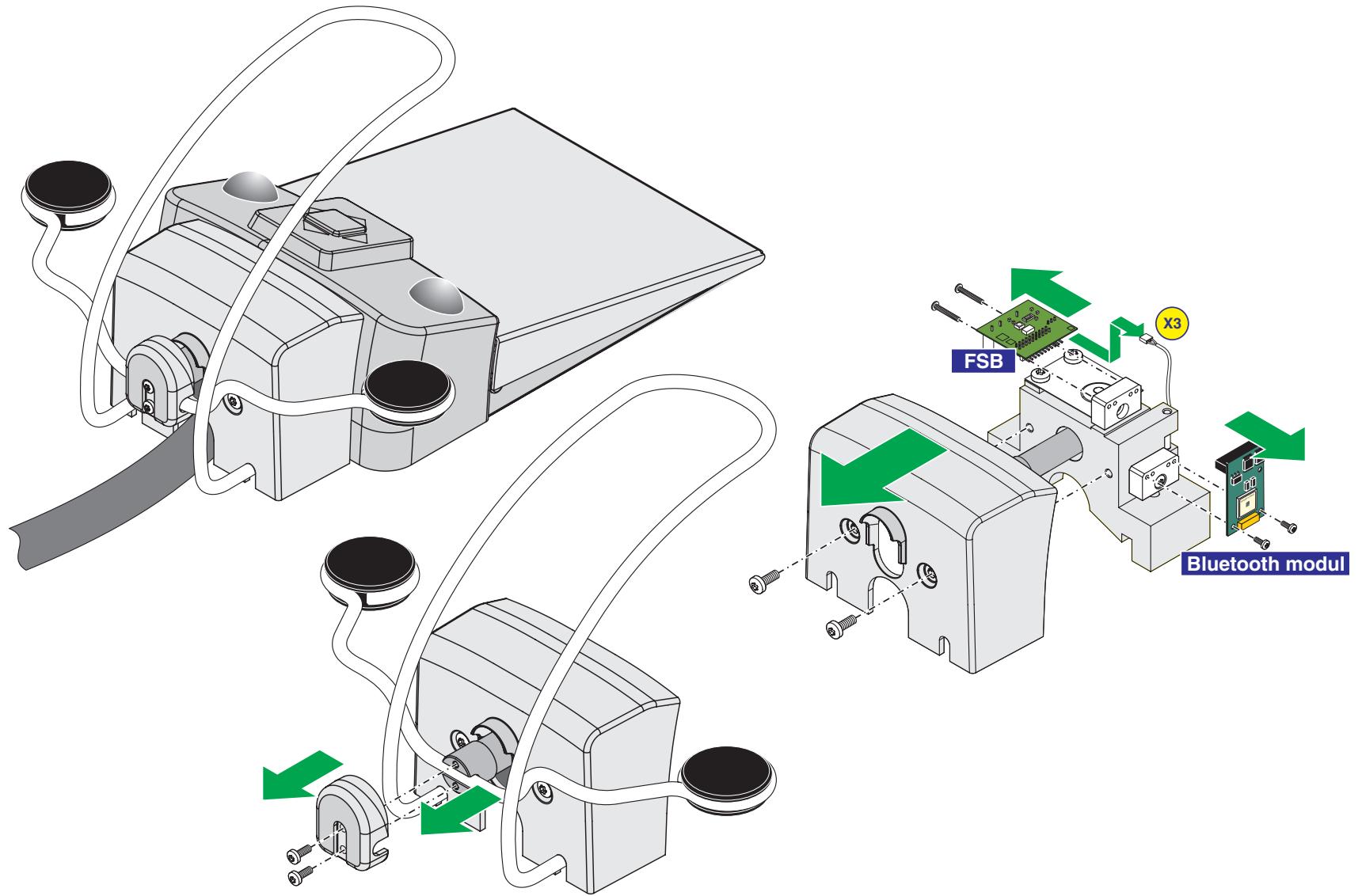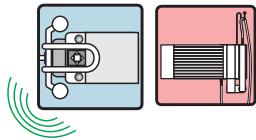





SIROLUX E

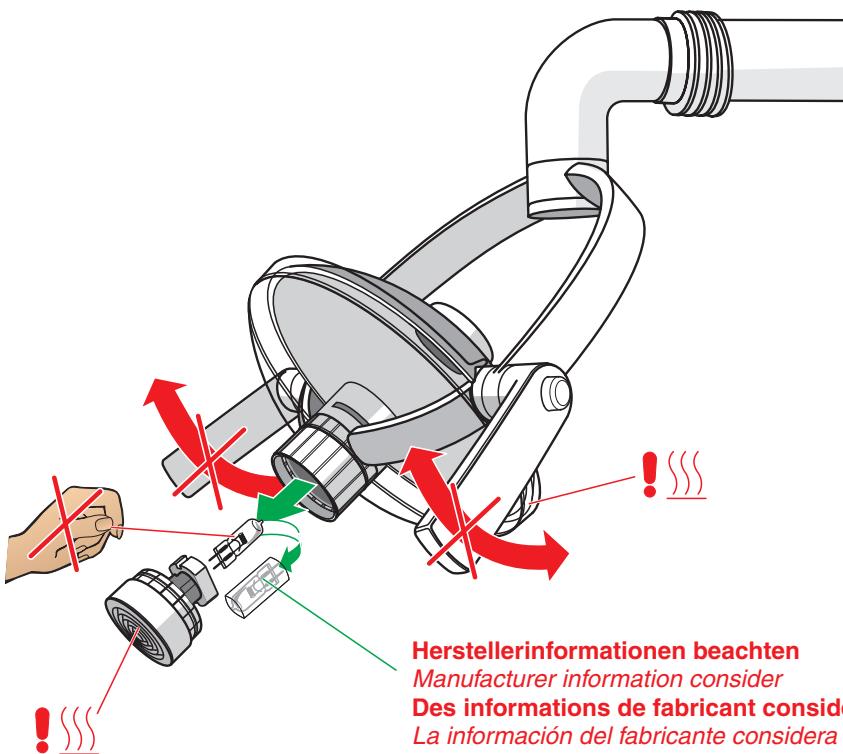

Herstellerinformationen beachten
Manufacturer information consider
Des informations de fabricant considérer
La información del fabricante considera

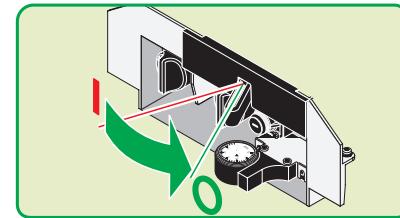

SIROLUX FANTASTIC

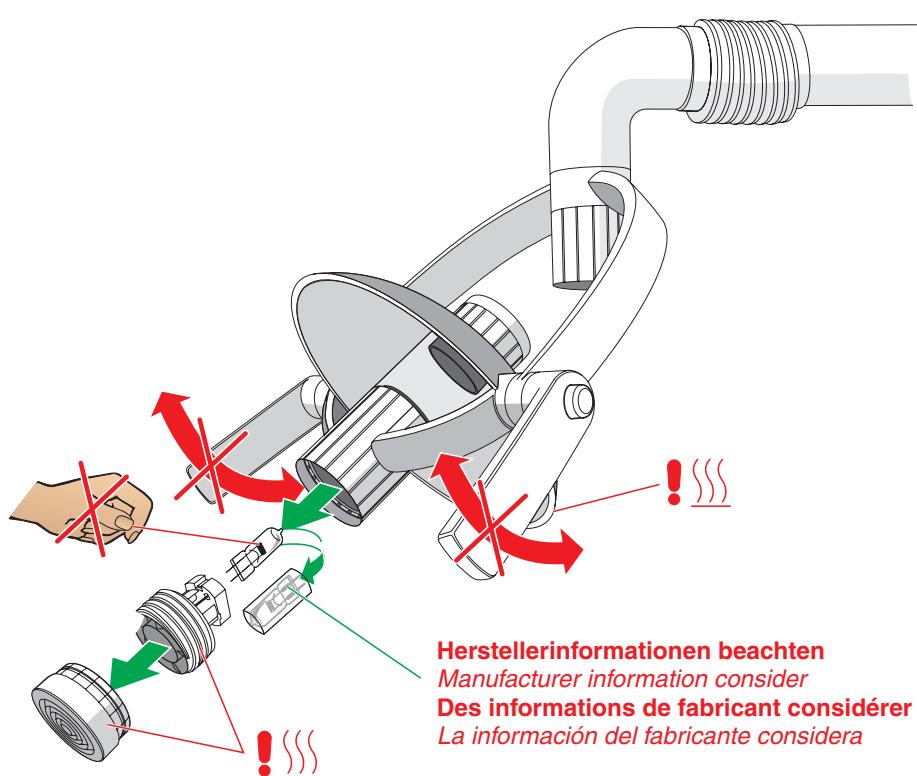

Herstellerinformationen beachten
Manufacturer information consider
Des informations de fabricant considérer
La información del fabricante considera



SB
Stuhl-Steuerung
Chair-Control
Commande du fauteuil
Control del sillón

SV2
24V-Versorgungsplatine
24V-Support
Platine d'alimentation 24V
Platina de alimentación 24V

SF
Sicherungsplatine
Fuseboard
Carte fusibles
Platina de fusibles

SX
SIROLUX-Steuerung
SIROLUX-Control
Commande de SIROLUX
Control de SIROLUX

KA
Anschluss Kompakt
Connection Kompakt
Raccordement Kompakt
Conexión platica Kompakt

SVI
PC/Video-Interface
PC/video interface
Interface vidéo /PC
Interfaz PC7Video



FSB
Verbindungsplatine Bluetooth
Bluetooth connection board
Platine de liaison Bluetooth
Platina de conexión Bluetooth

Funk-Modul (Bluetooth)
Radio module (bluetooth)
Module radio (bluetooth)
Módulo radio (bluetooth)

Bluetooth modul

SB
Stuhl-Steuerung
Chair-Control
Commande du fauteuil
Control del sillón

SF
Sicherungsplatine
Fuseboard
Carte fusibles
Platina de fusibles

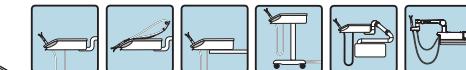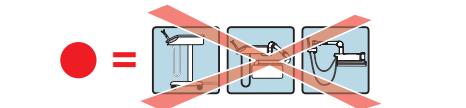

AB
Instrumenten-Steuerung
Instrument-Control
Commande des instruments
Control de los instrumentos

AC7
Kameramodul
Camera module
Module caméra
Módulo de cámara

AL
Instr.-Motor-Steuerung
Instrument-Motor-Control
Commande moteur d.instruments
Control el motor del instrumento

AUB
Ultraschallversorgung
Ultrasonic-Supply
Alimentation de ultrasons
Alimentación de ultrasonidos

AU
Ultraschall-Modul
Ultrasonic module
Module ultrasons
Modulo de ultrasonidos

AZ2
Optionsversorgung
Option-Support
Commande de option
Control de opción

MI
Mini L.E.D. -Versorgung
Mini L.E.D.-Support
Alimentation Mini L.E.D.
Alimentación Mini L.E.D.

PAR
Tastatur
Keyboard
Clavier
Teclado

Film betrachter Versorgung
Film viewer-Supply
Alimentation de négatoscope
Alimentación de negatoscopio



WD2

Desinf.-steuerung

Disinfection system-Control

Commande de syst. de désinfect.

Control de sistema de desinfección

WA2

Amalgamabscheider-Steuerung

Amalgam separator-Control

Commande - séparateur d'amalg.

Control de separador de amalgama

WB

Separier.-Steuerung




Automatic separator-Control

Commande de autom. séparateur

Control de automatismo separador



MI
Mini L.E.D. -Versorgung
Mini L.E.D.-Support
Alimentation Mini L.E.D.
Alimentación Mini L.E.D.

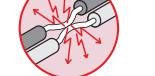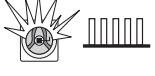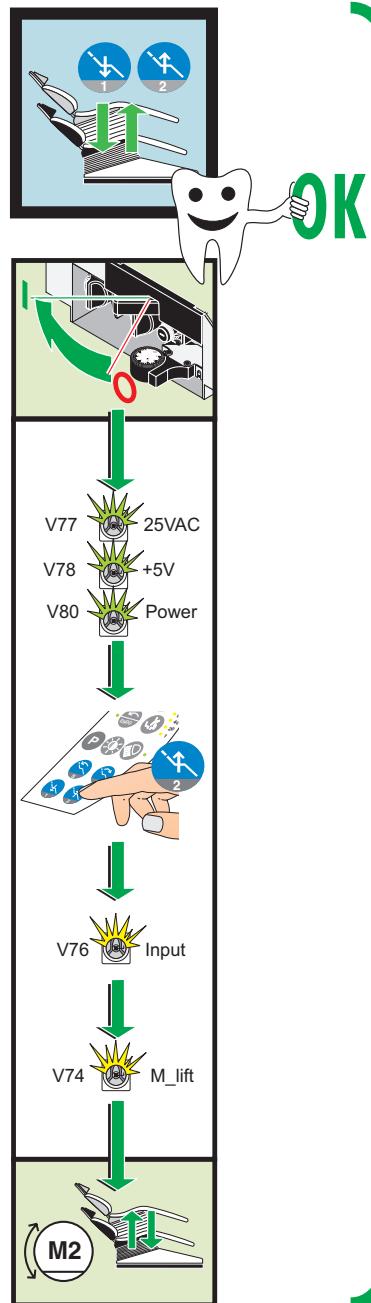

Information
Information
Information
Información

siehe Seite ...
see page ...
cf. page
véase página

muß leuchten / Betriebszustand in Ordnung
must light up / operating state in order
= doit s'allumer / bon état de fonctionnement
tiene que estar iluminado / Estado operacional en orden

darf nicht leuchten / Fehlerzustand
must not light up / error state
= ne doit pas s'allumer / défaut
no puede estar iluminado / Defectuoso

Status / Information
Status / Information
état /information
Revisión / Información

LED leuchtet
LED lit up
DEL allumée
LED iluminado

LED blinkt
LED flashing
DEL clignote
LED en intermitencia


LED AUS
LED OFF
DEL éteinte
LED APAGADO

ja / in Ordnung
yes / functioning
oui / en bon état
sí / en orden

nein / nicht in Ordnung
no / not functioning
non / état défectueux
no / no está en orden

Unterbrechung
Disturbance
interruption
Interrupción

Kurzschluß
Short circuit
court-circuit
Cortocircuito

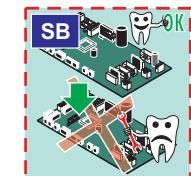

prüfen (sehen ob ...)
Check (see whether ...)
vérifier (contrôle visuel ...)
comprueba (mire si ...)










prüfen (hören ob ...)
Check (listen whether ...)
vérifier (contrôle auditif ...)
comprueba (oiga si ...)

Function Pad überbrücken
Jumper function pad
court-circuiter Function Pad
Puentear la función Pad


Auswechseln
Replace
échanger
cambiar

O-Ringe leicht fetten
Lightly lubricate the O-rings
Graisser légèrement les joints toriques
Lubricar ligeramente los anillos toroidales






C8+

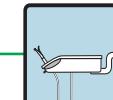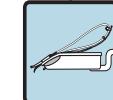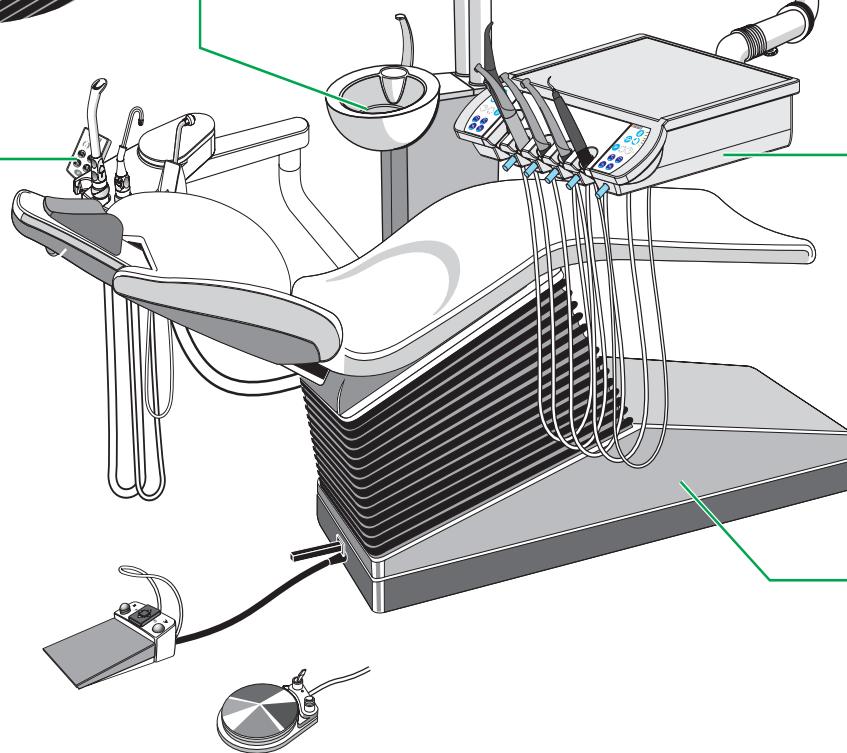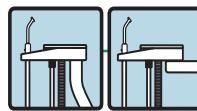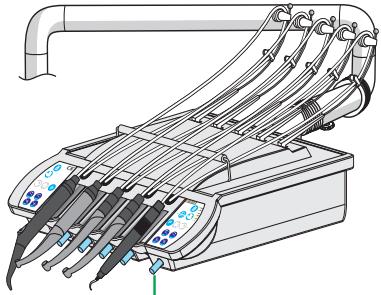
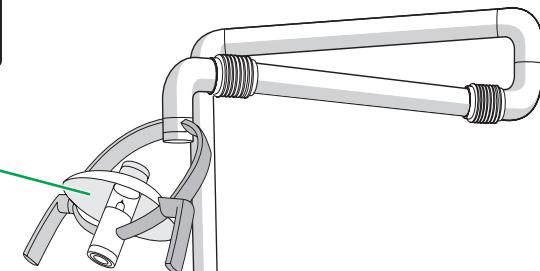
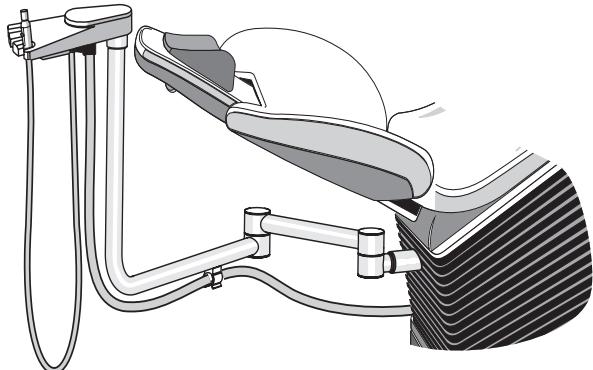
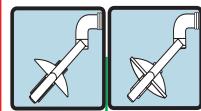
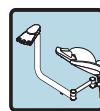



C8+ Turn










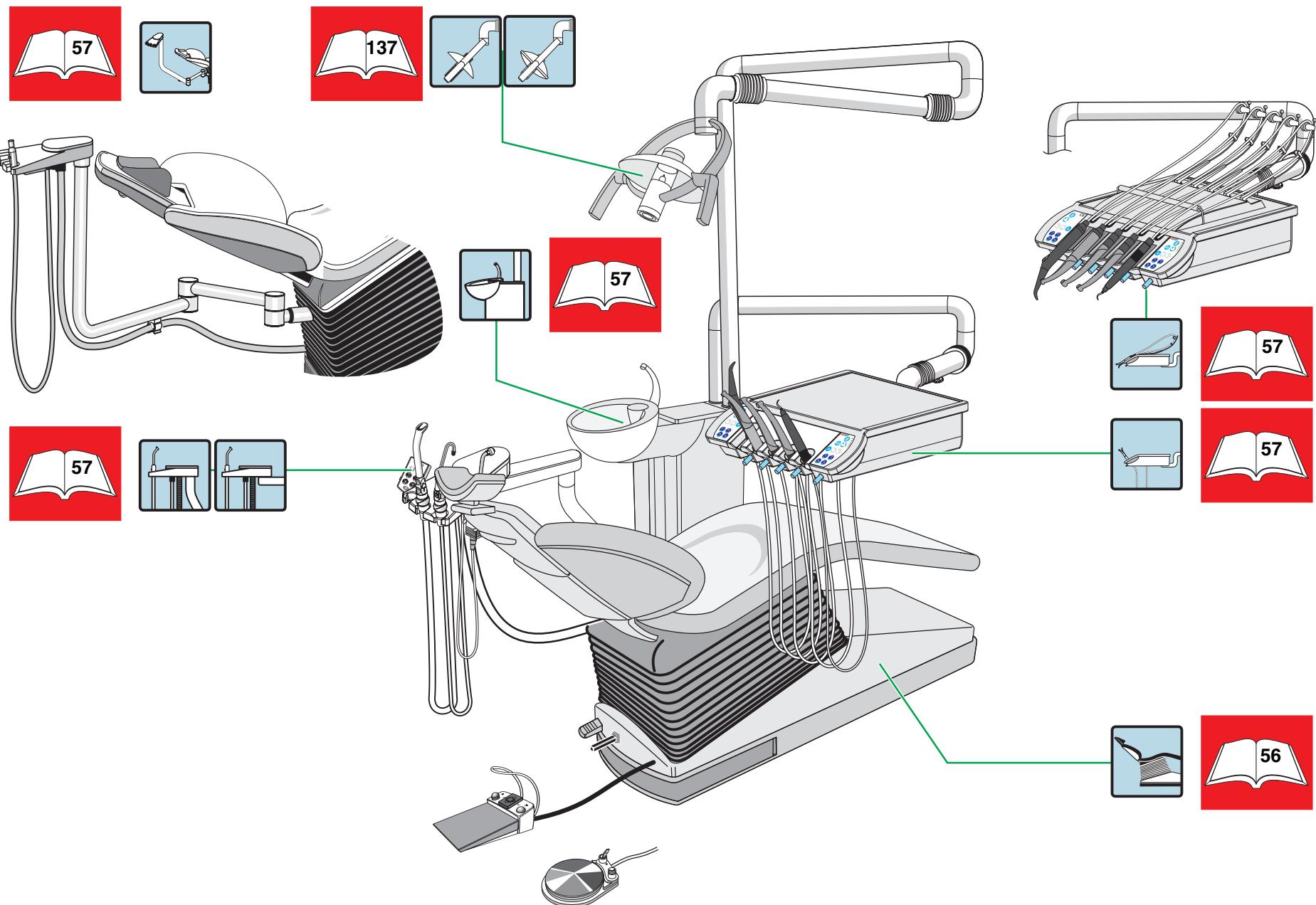










C8+

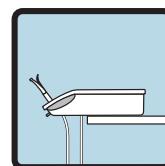
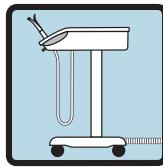
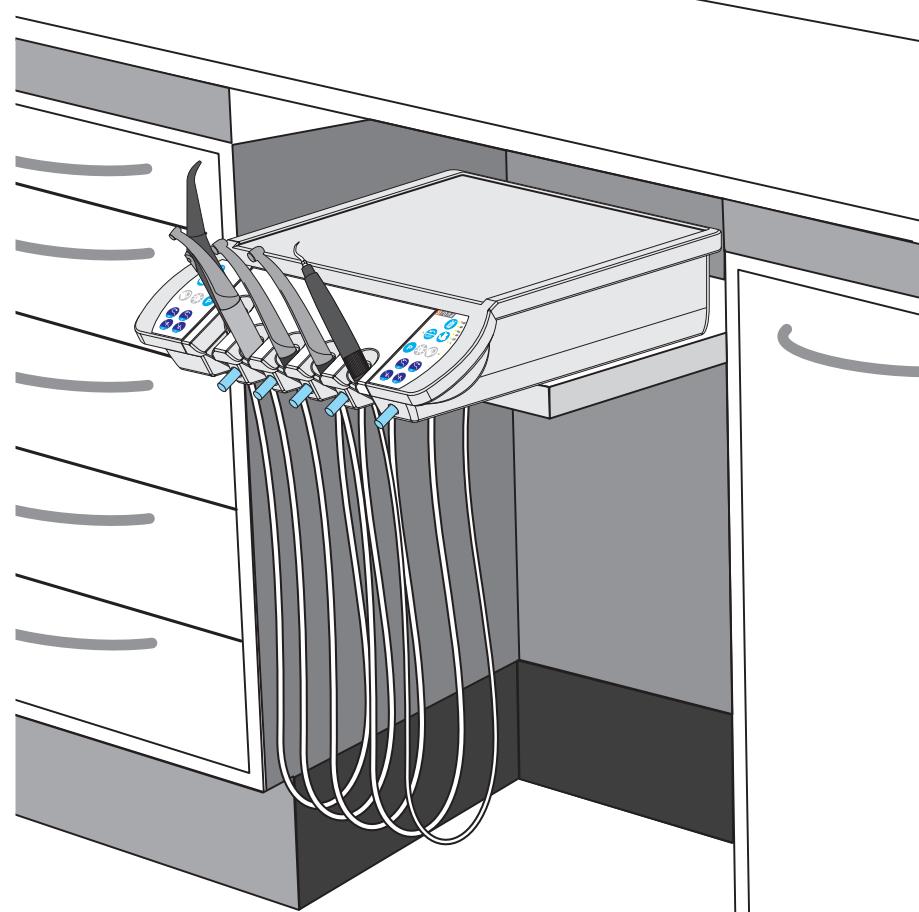
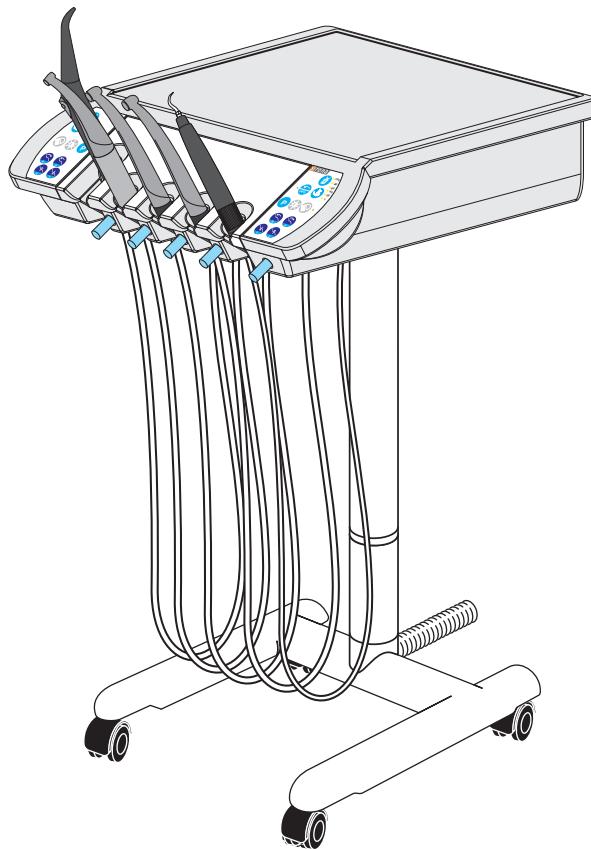
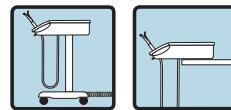


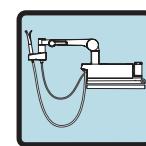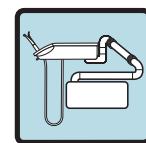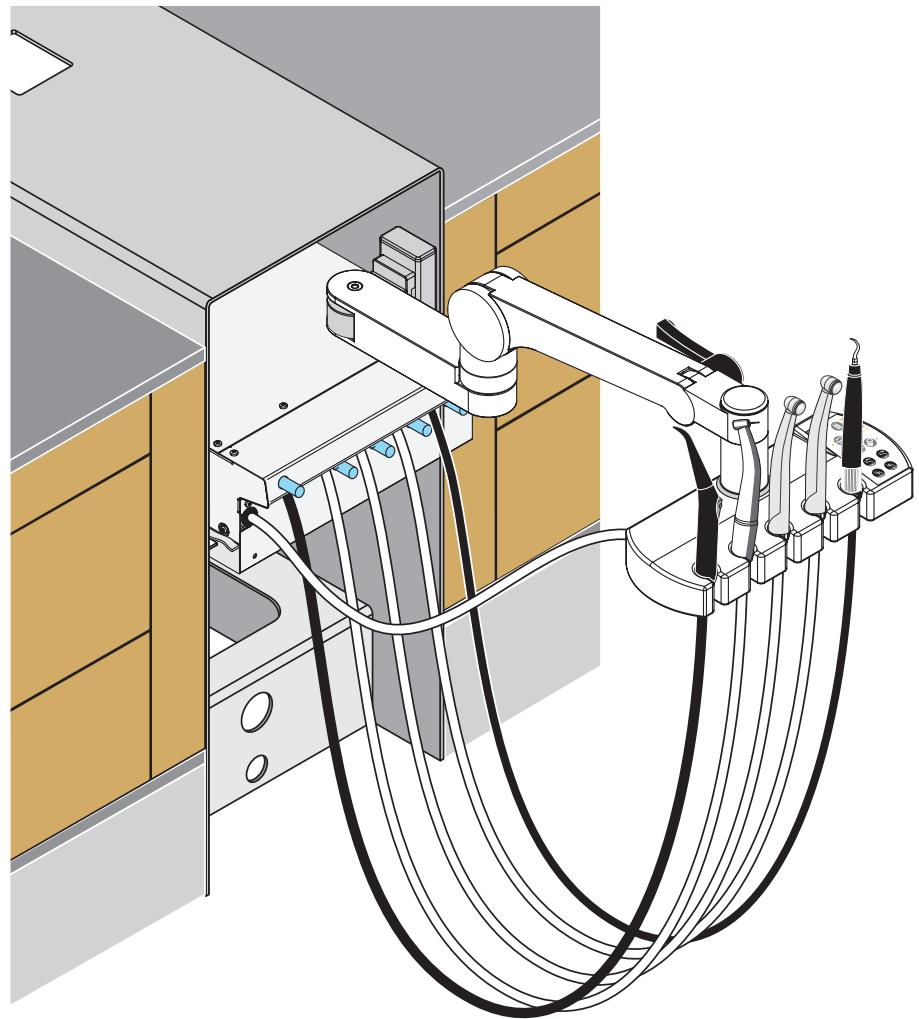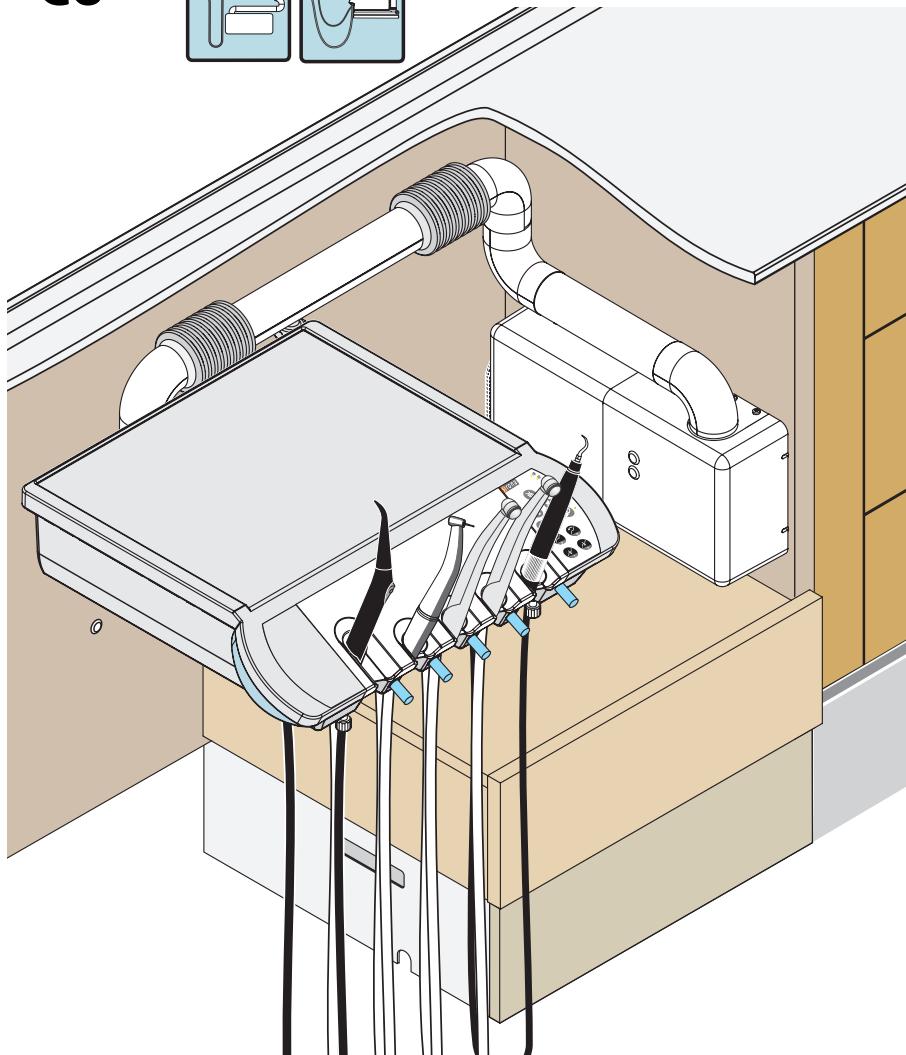
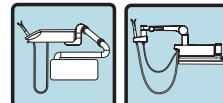
C8+

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

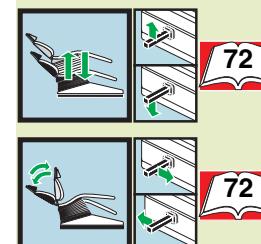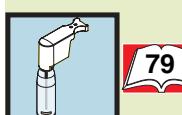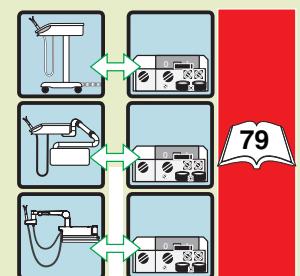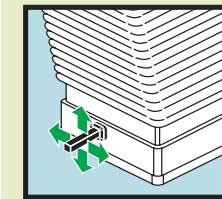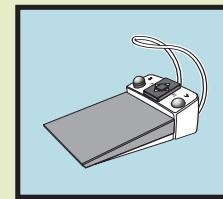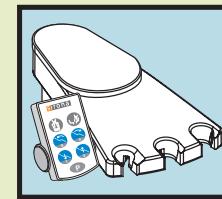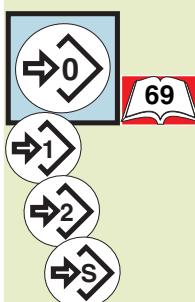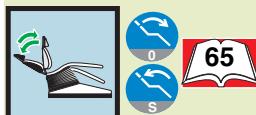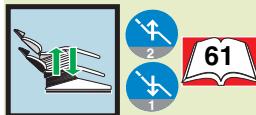
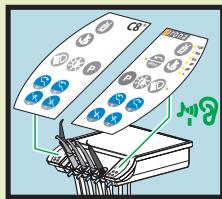


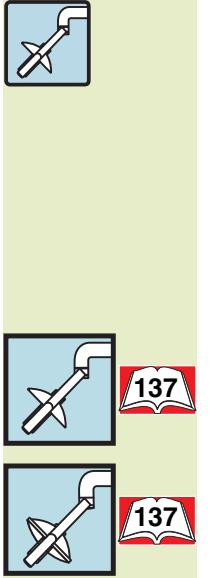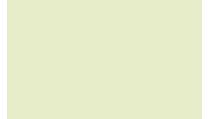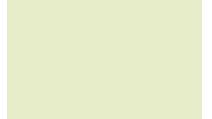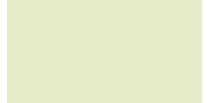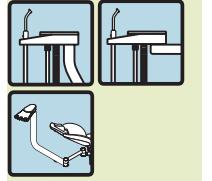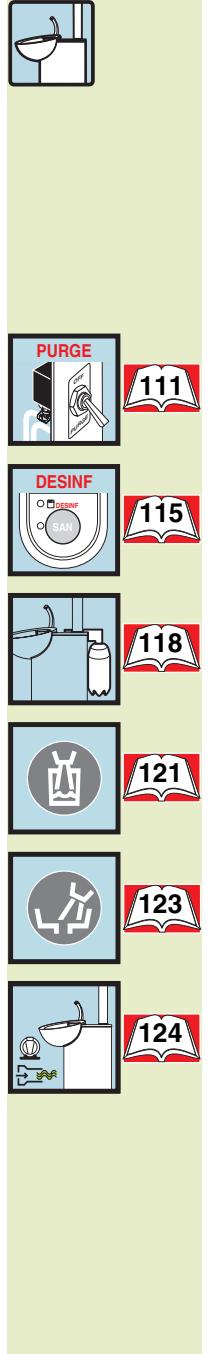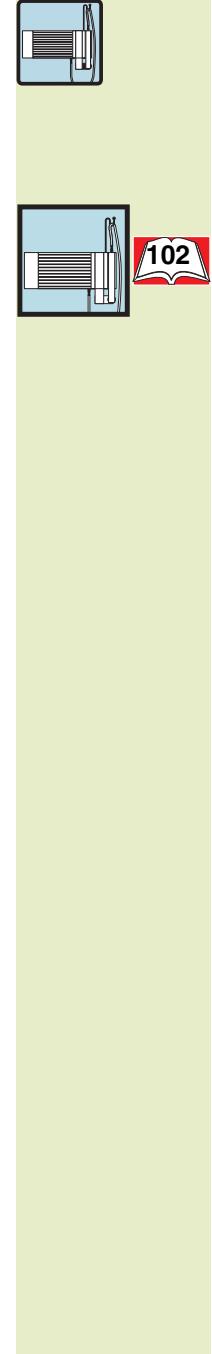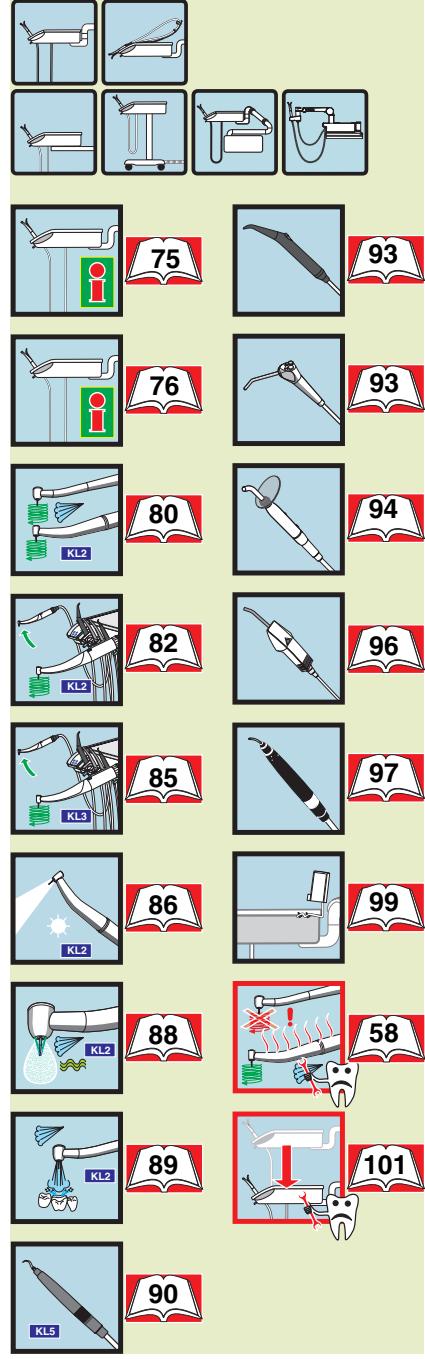

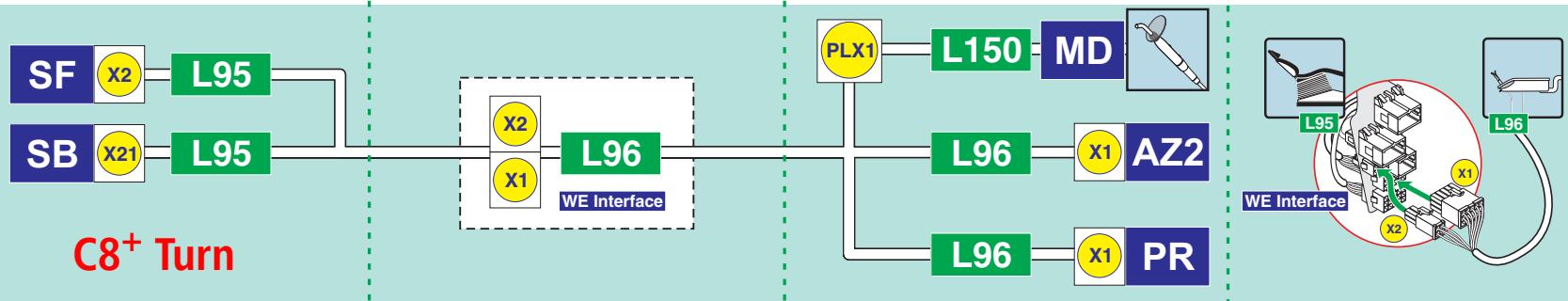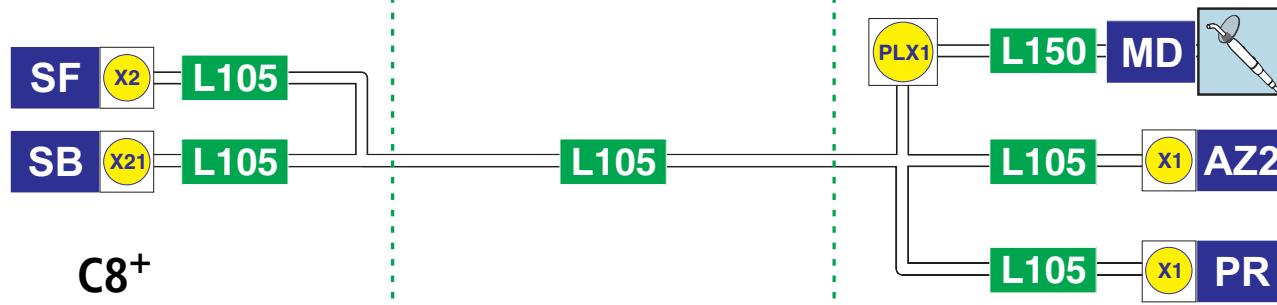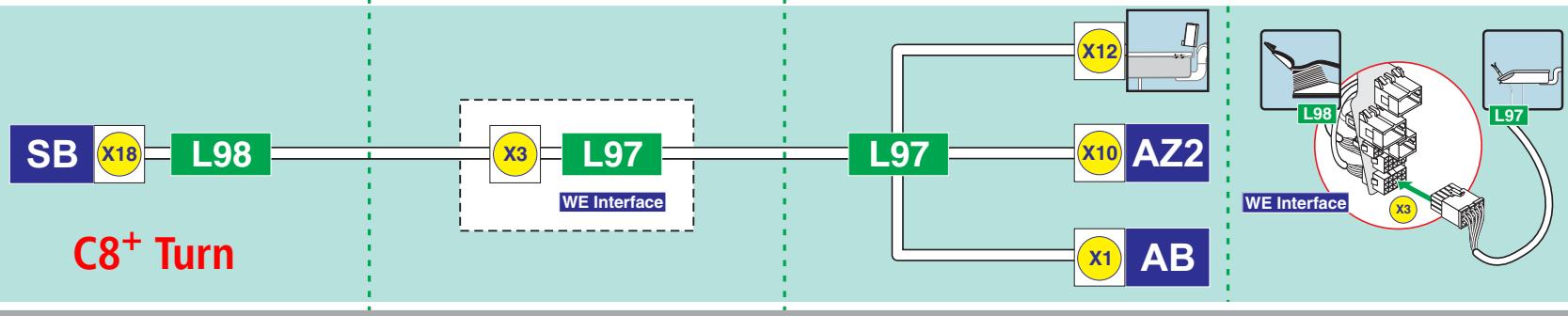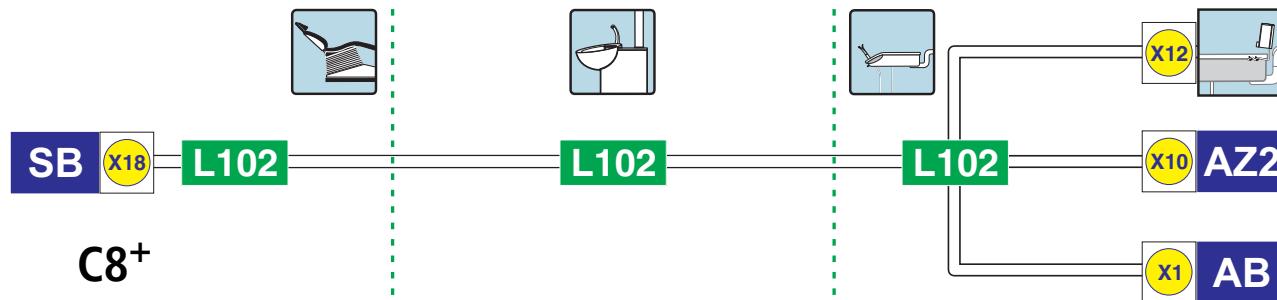
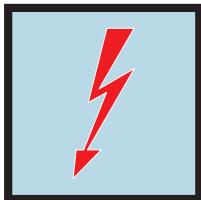


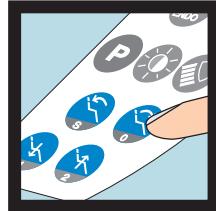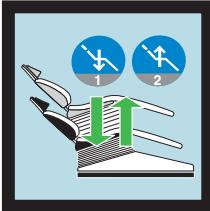



C8⁺



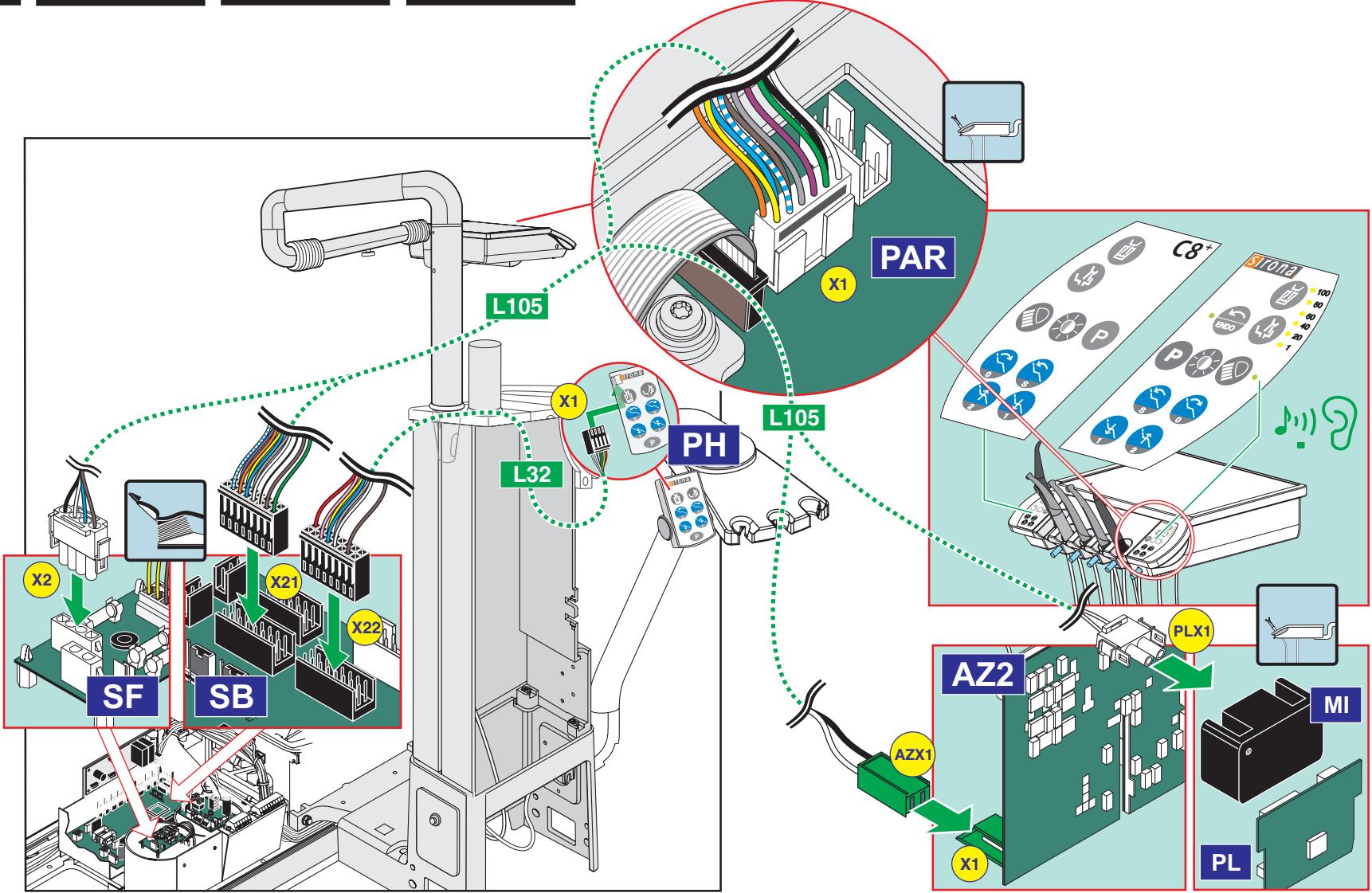
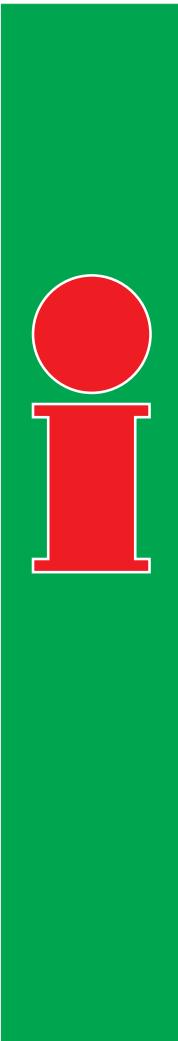







C8+ Turn

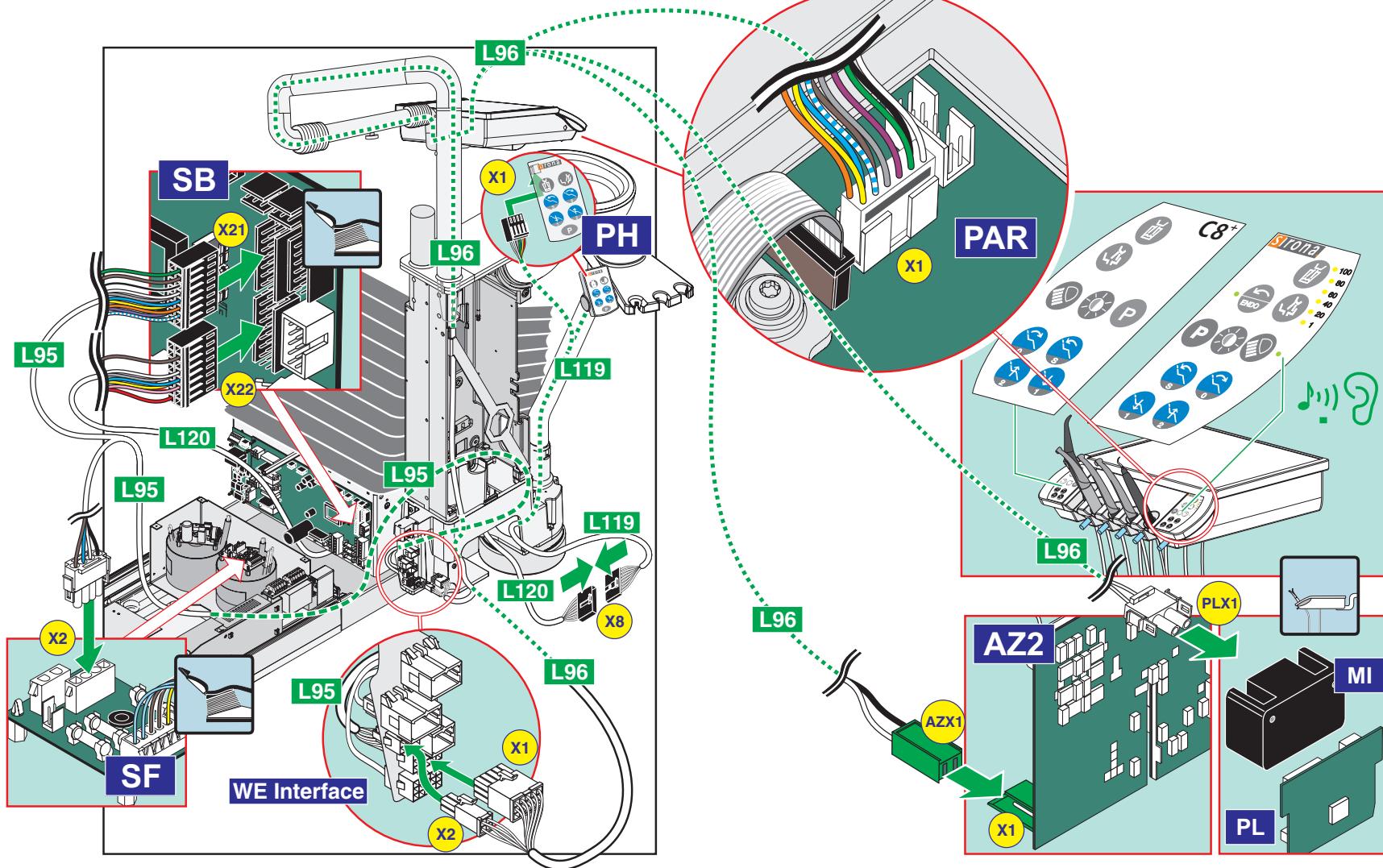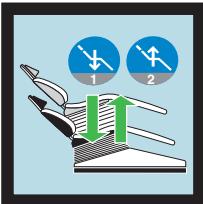


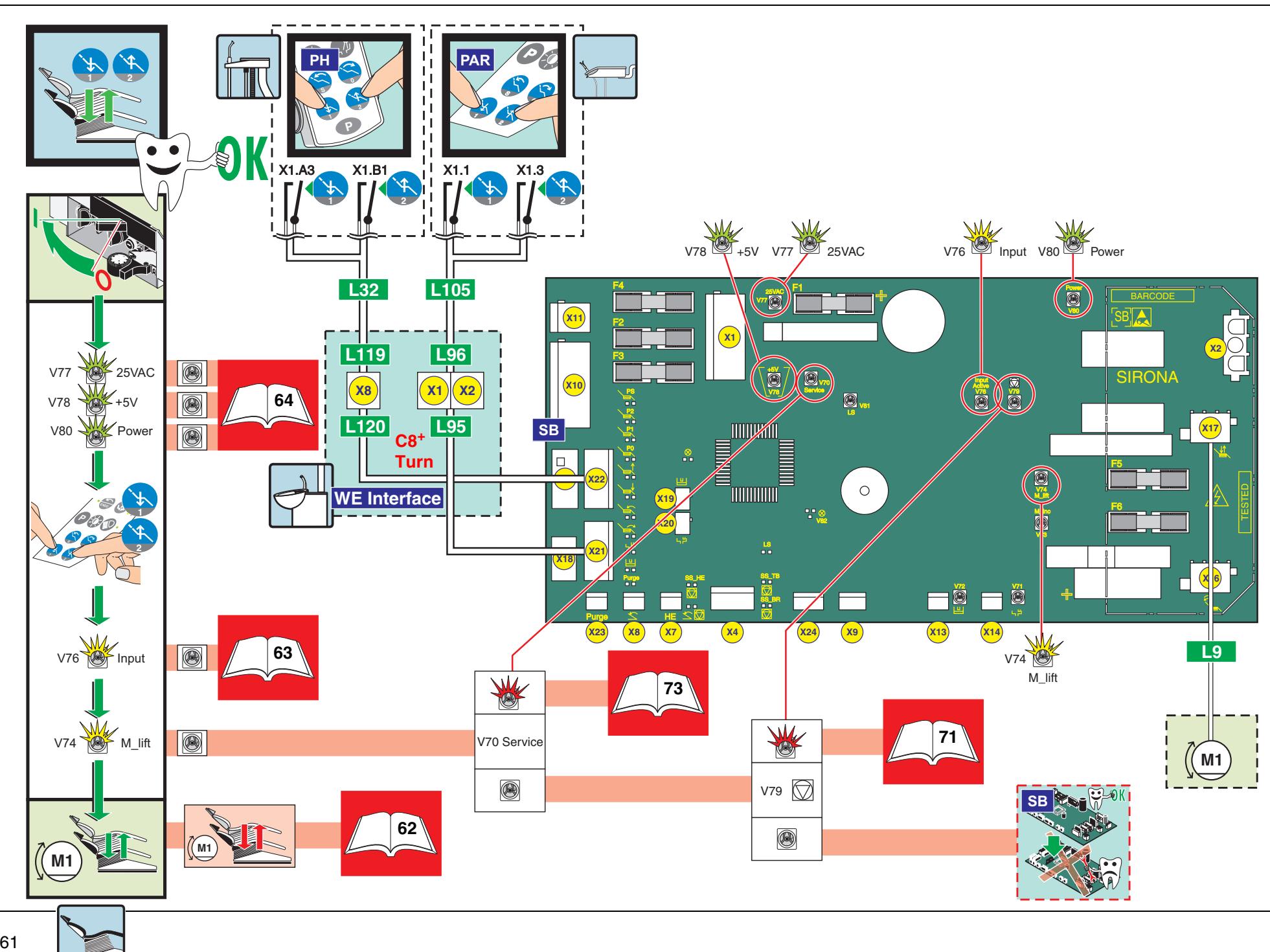


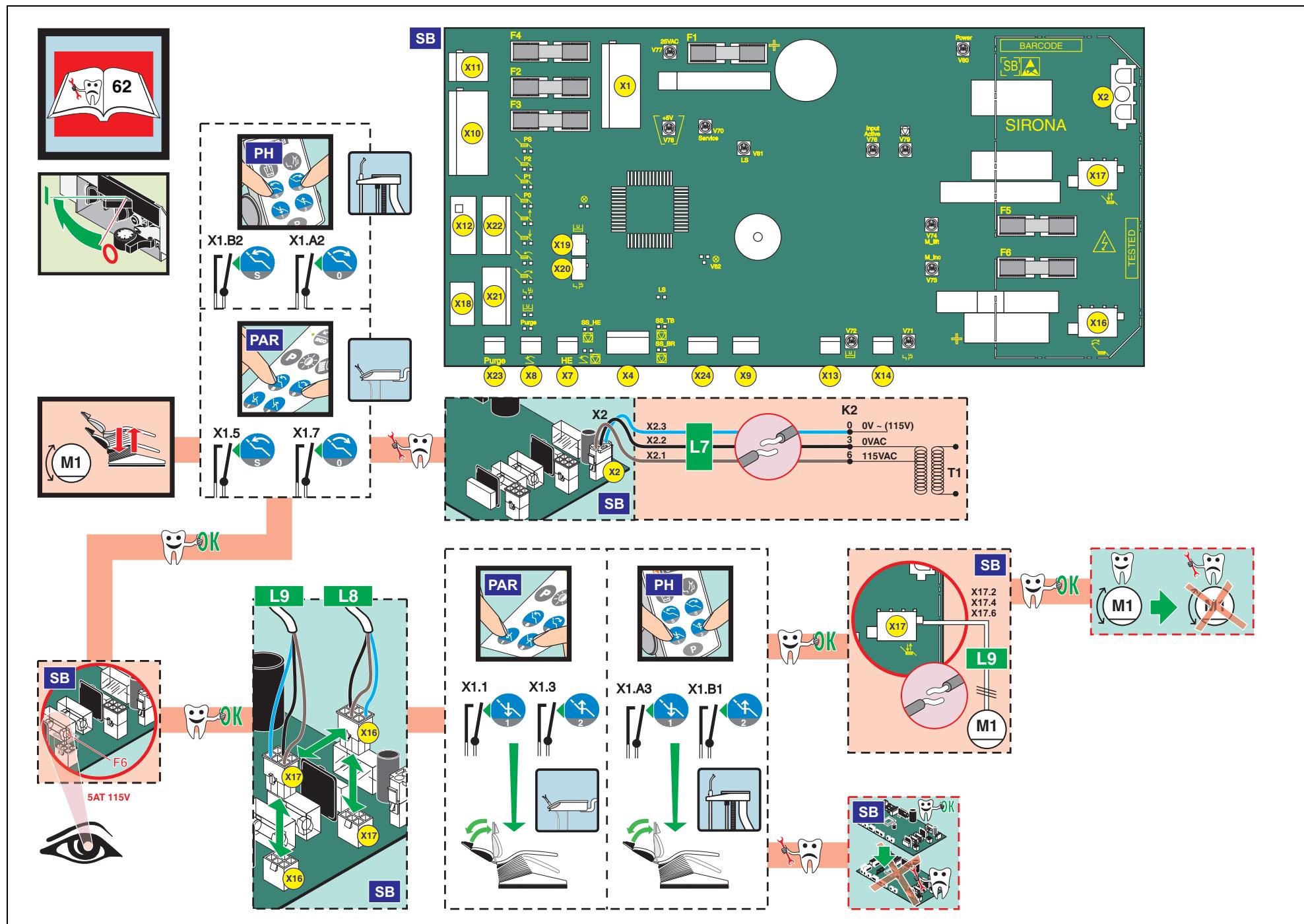





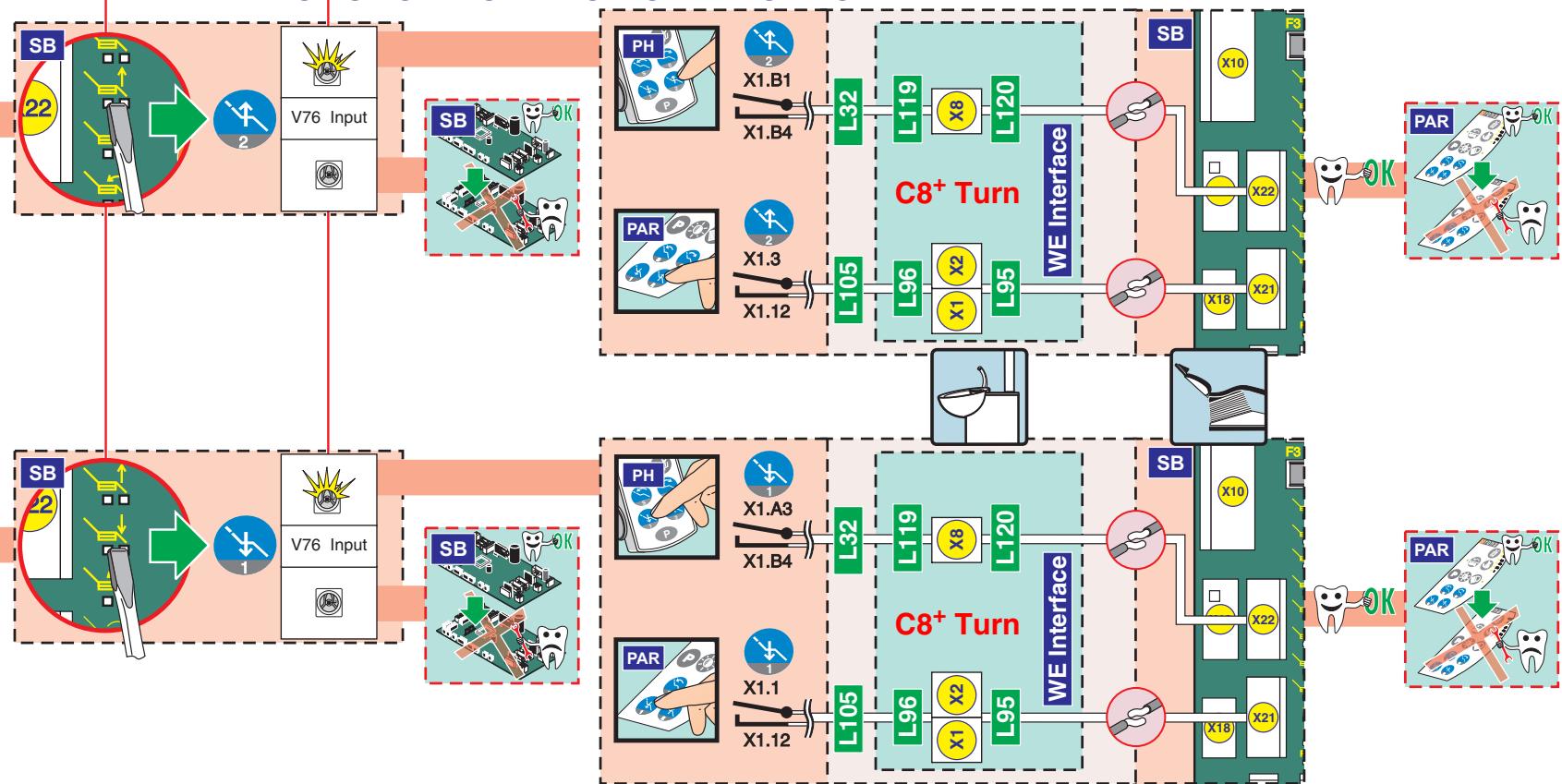
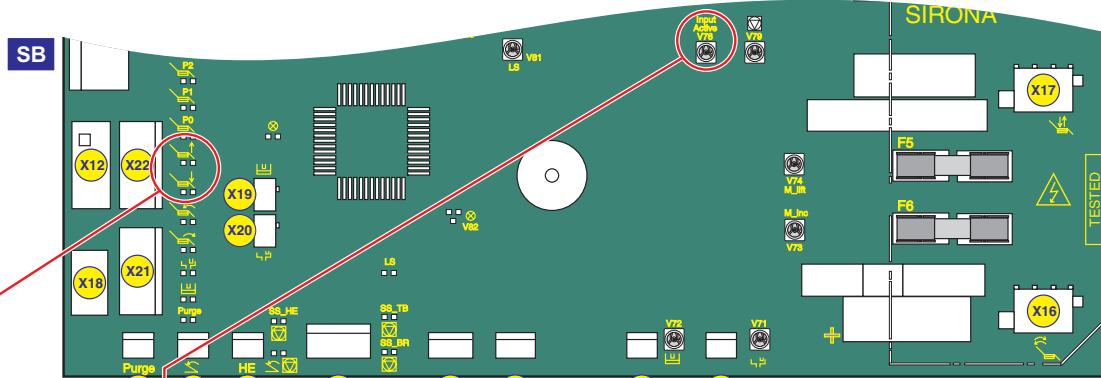
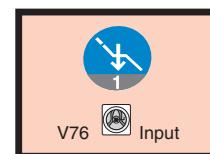
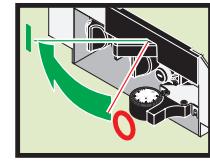


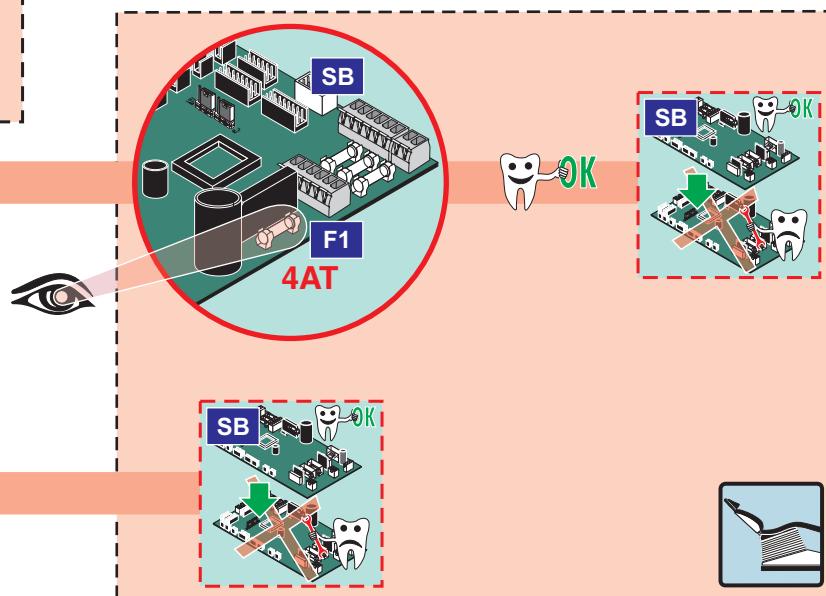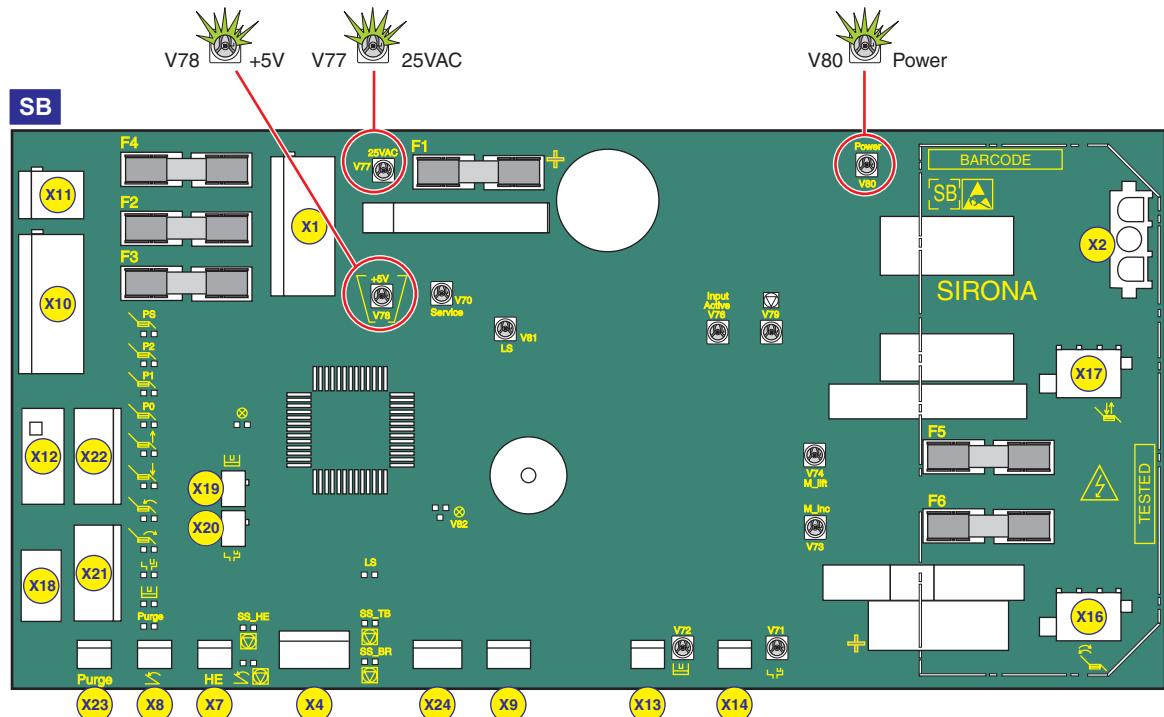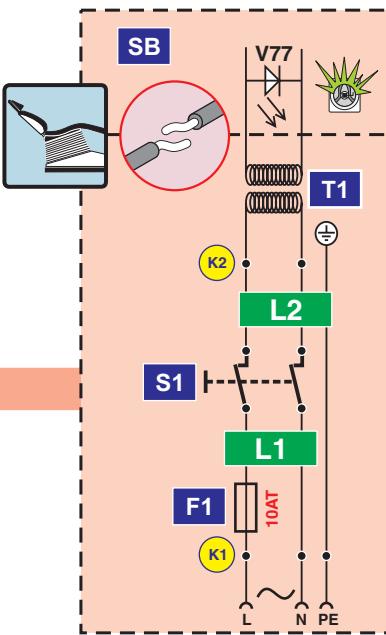
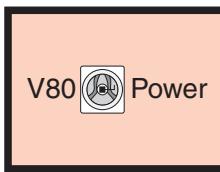
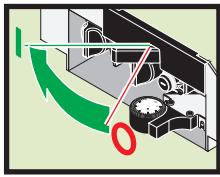


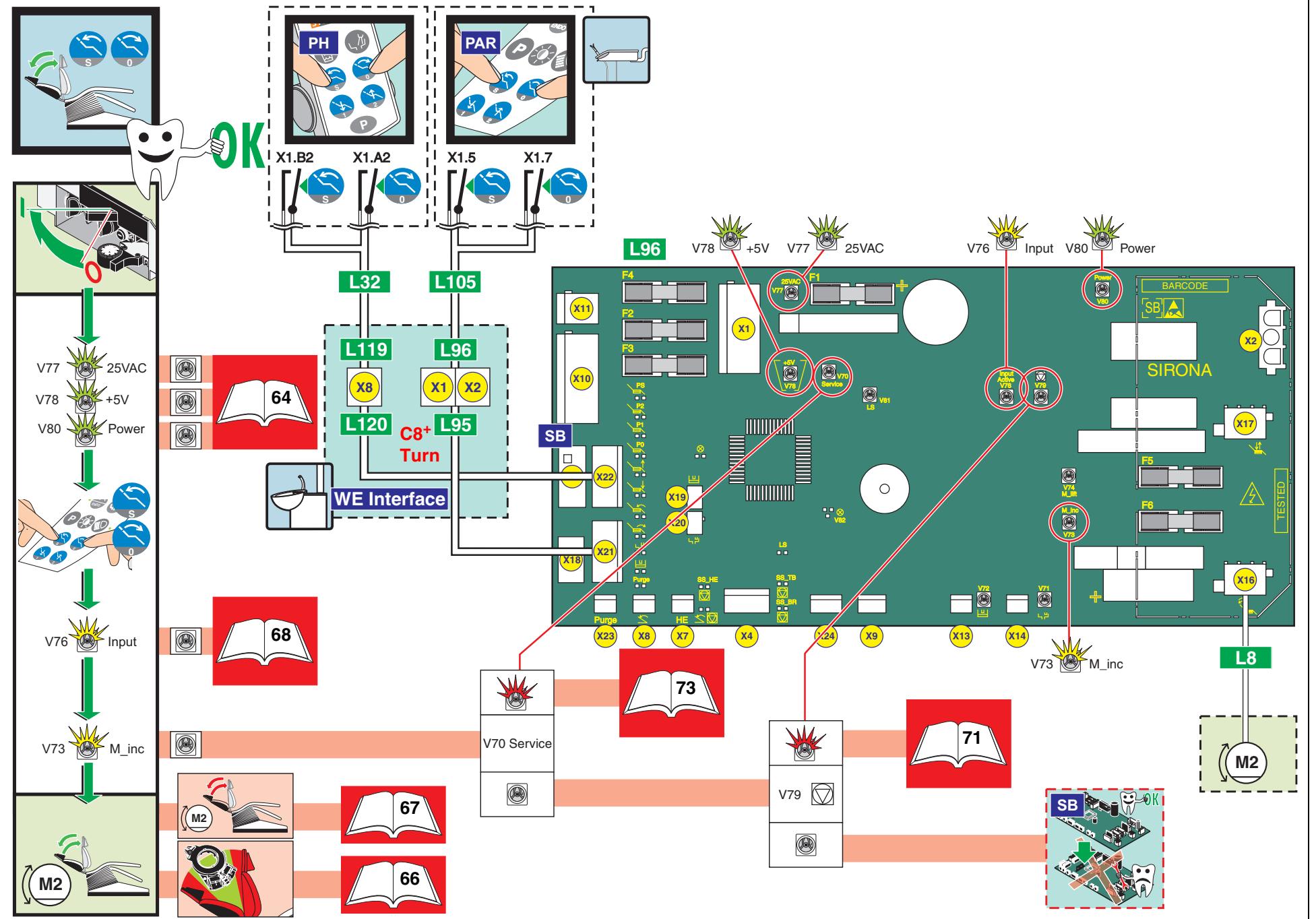



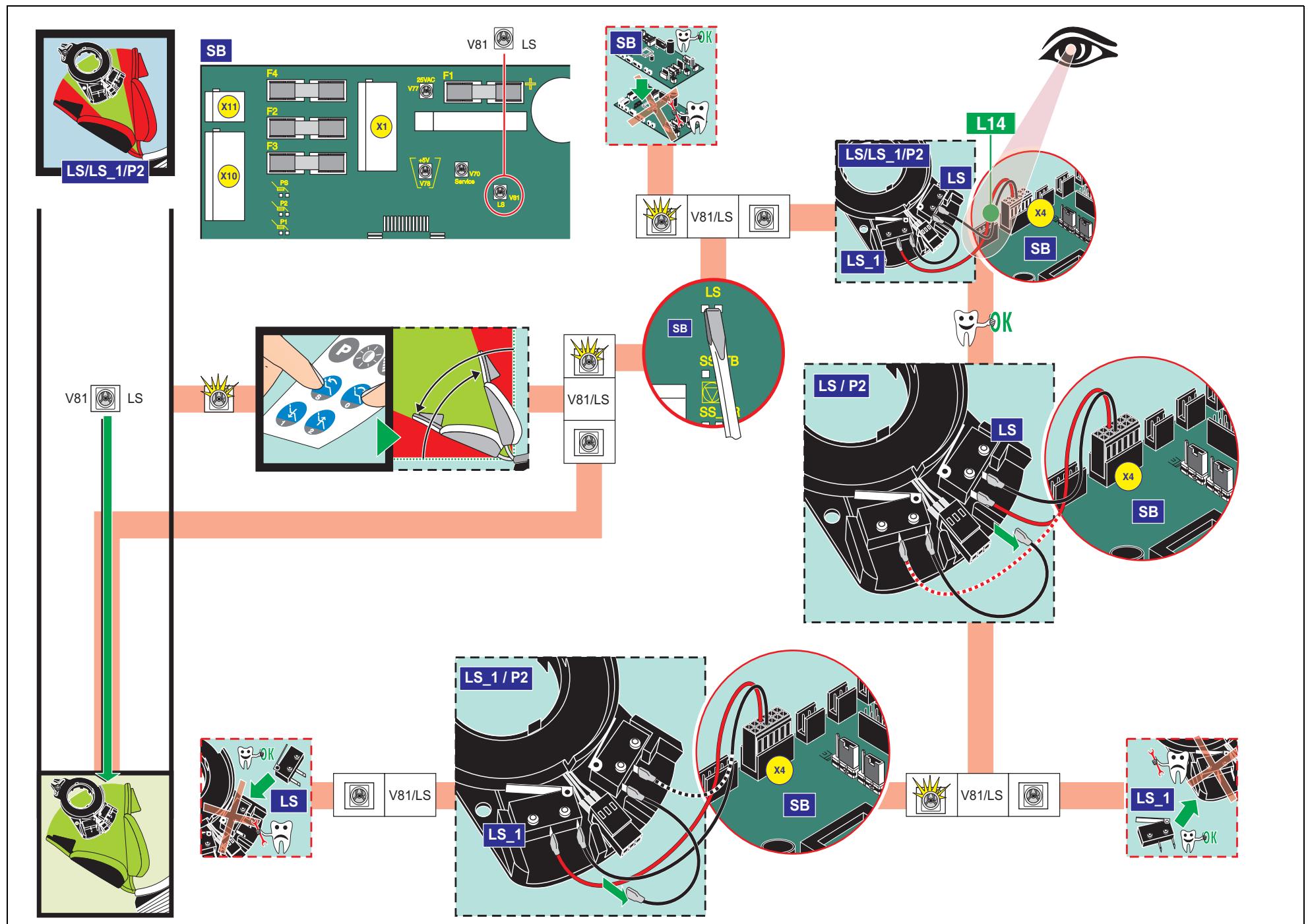


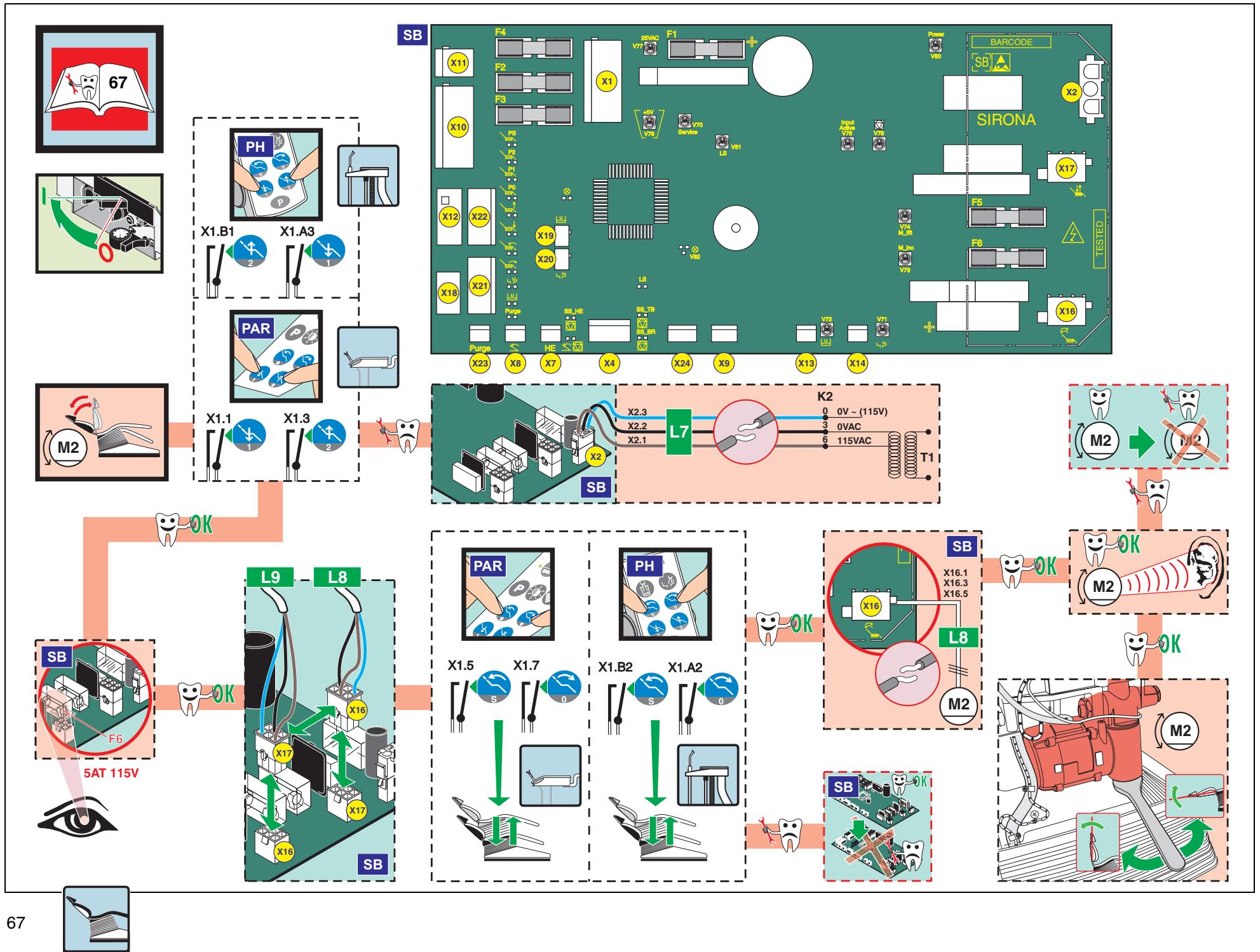


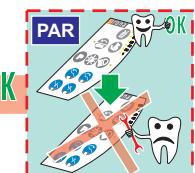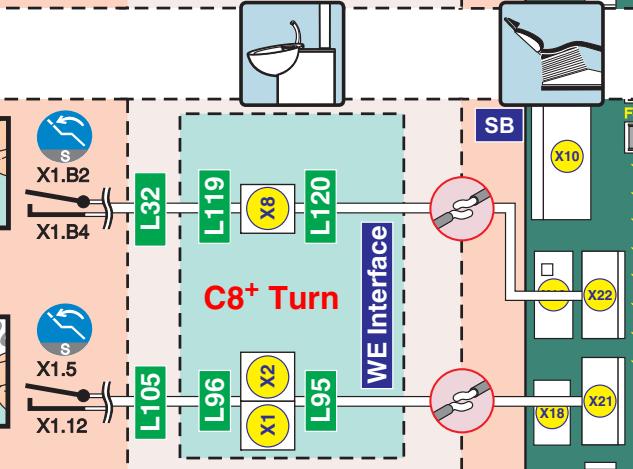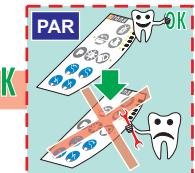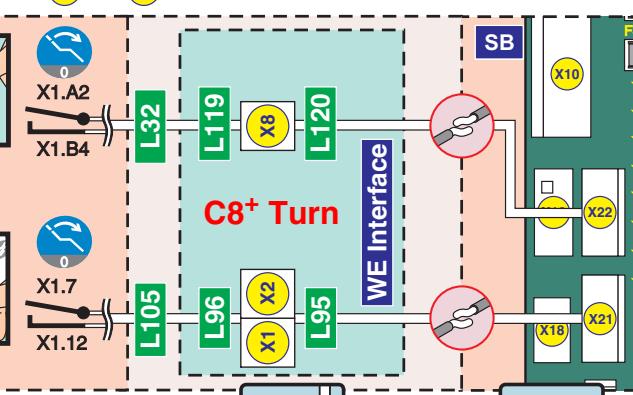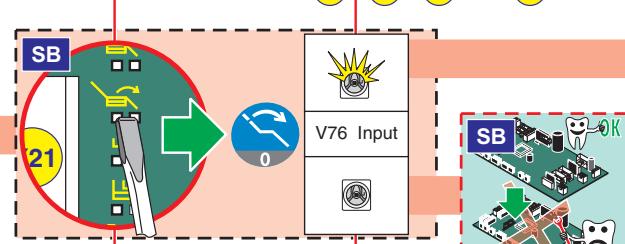
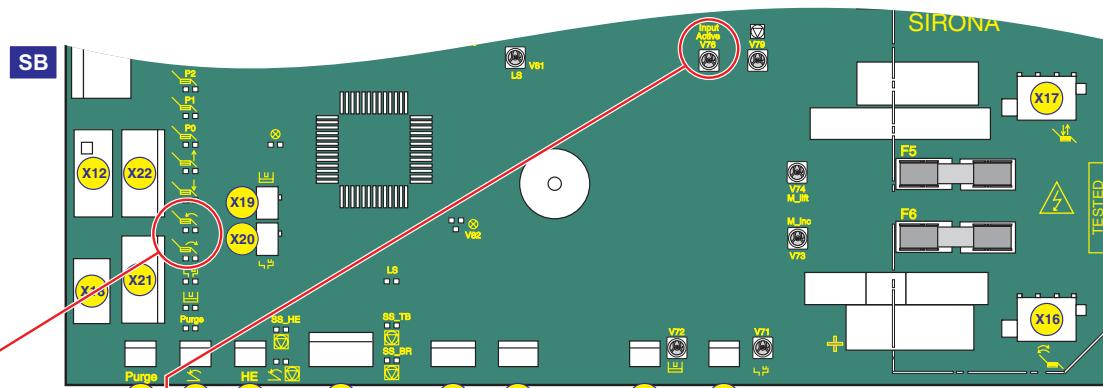
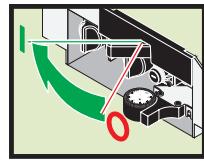


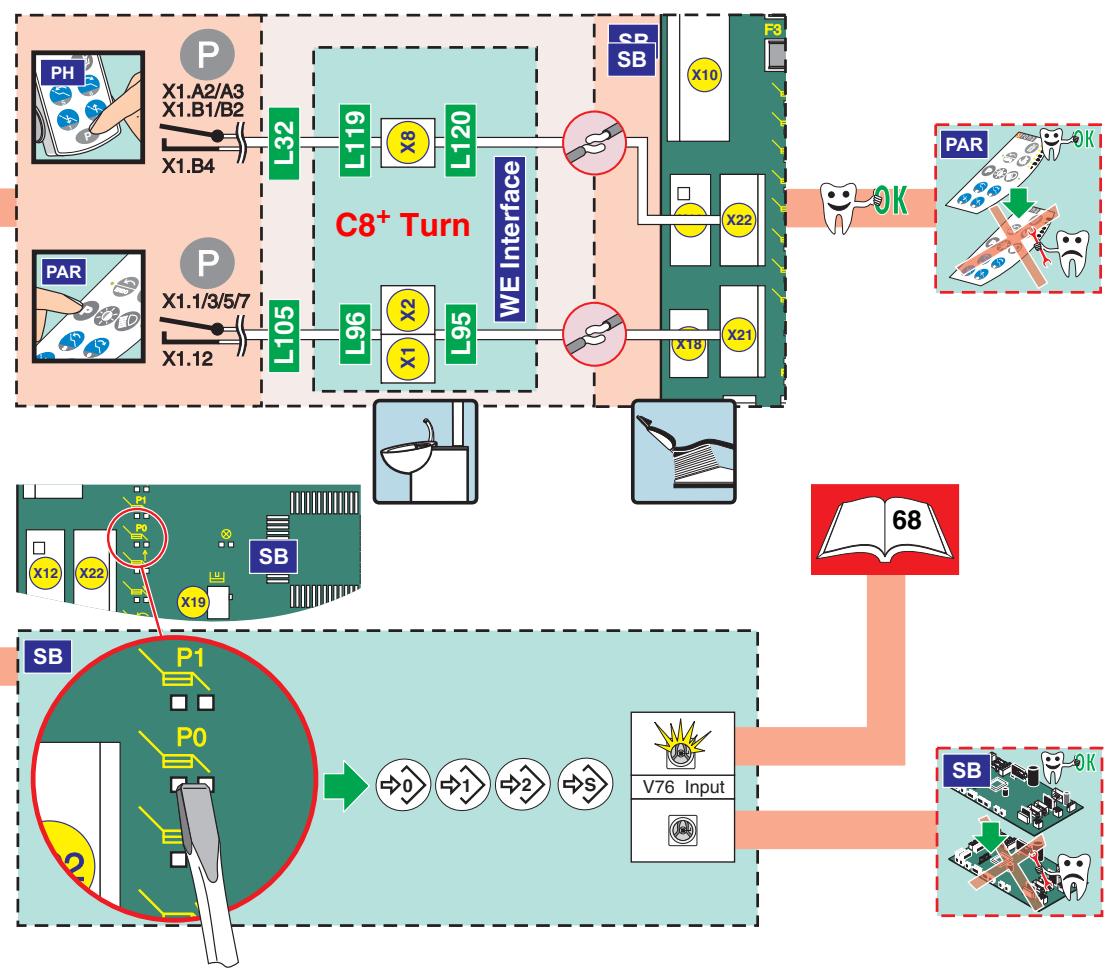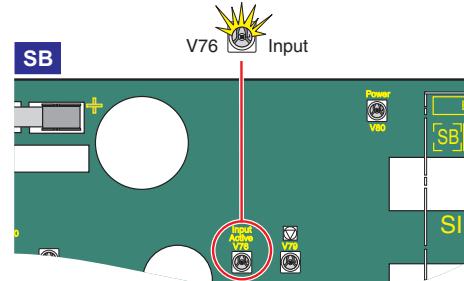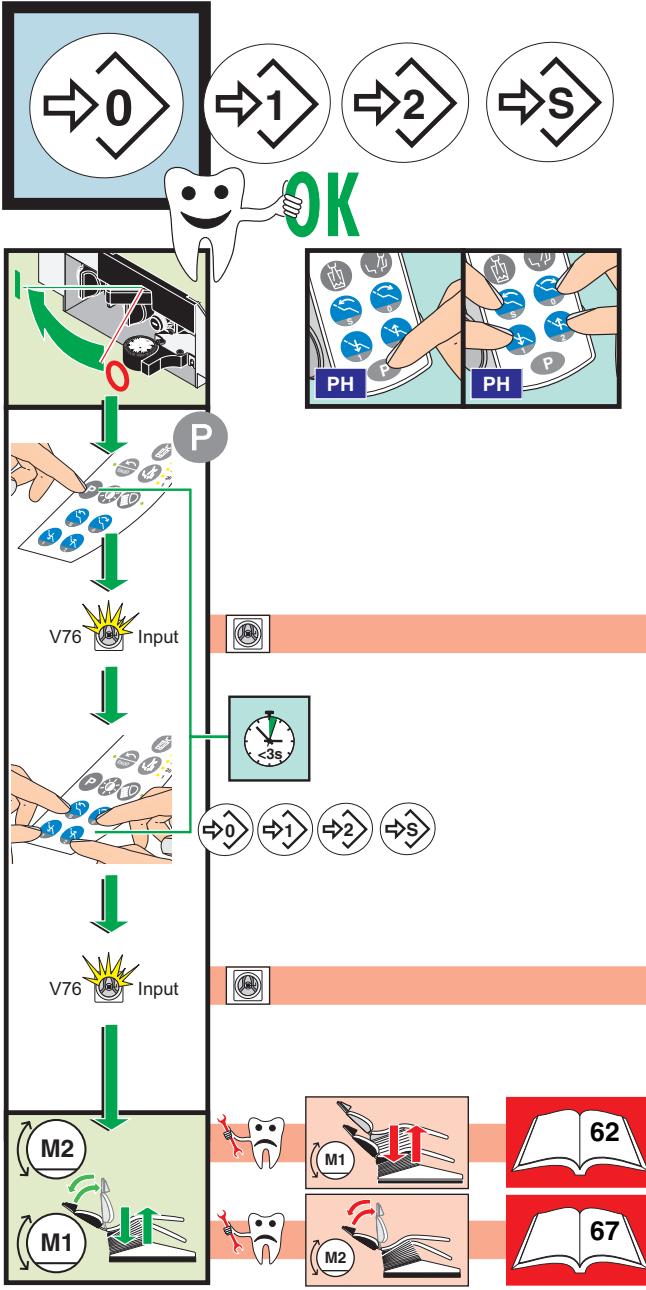









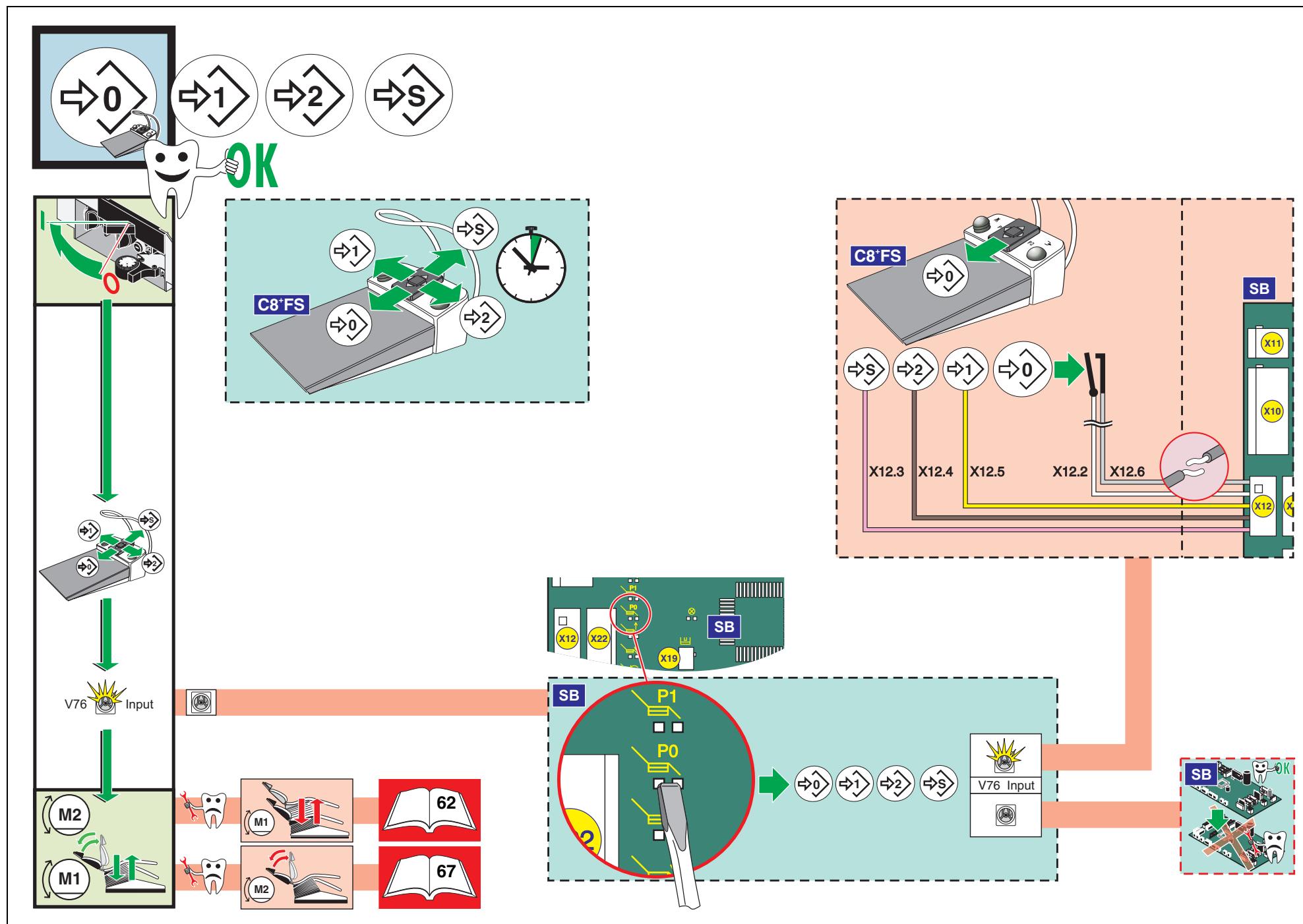


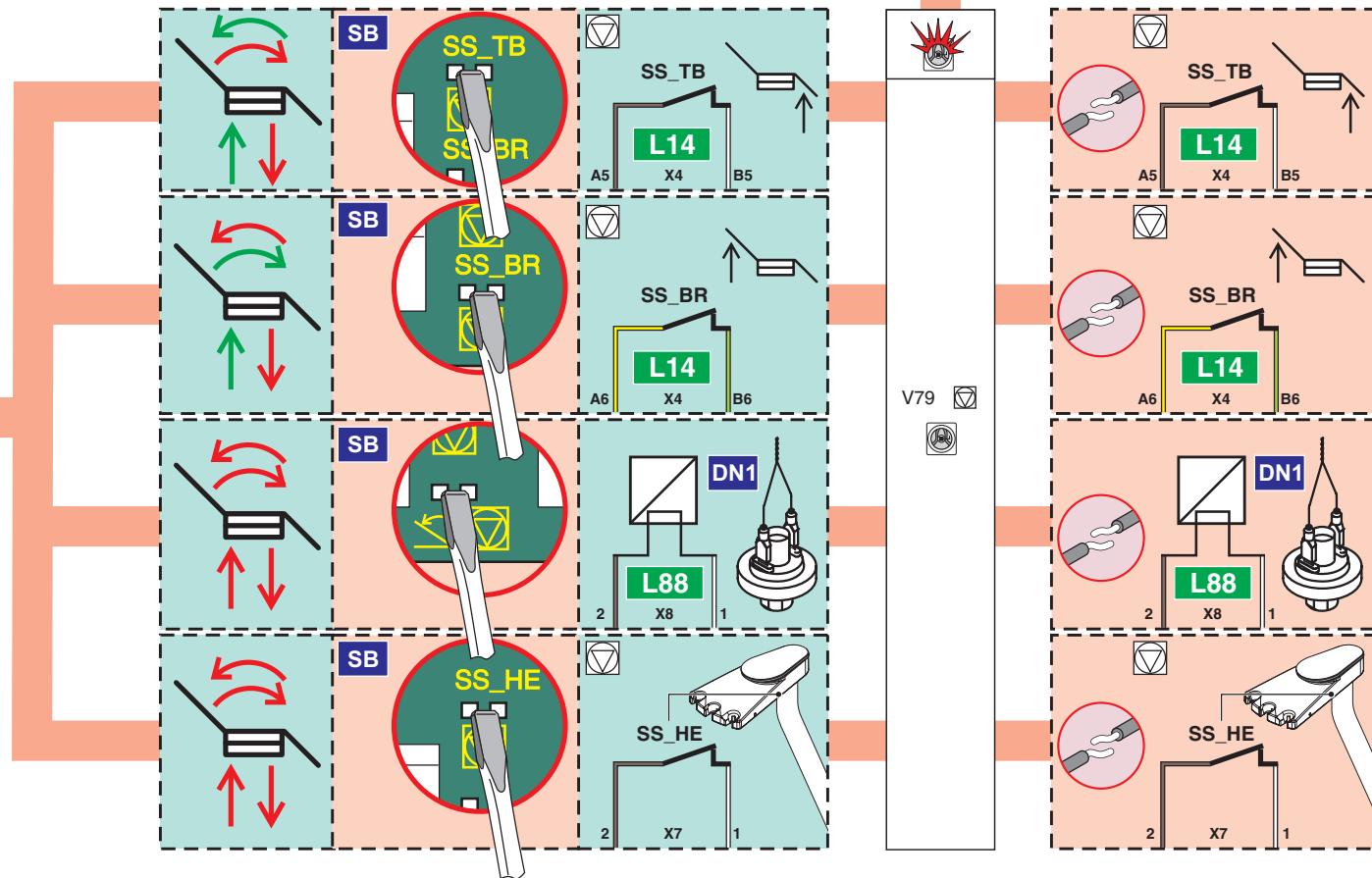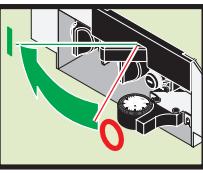


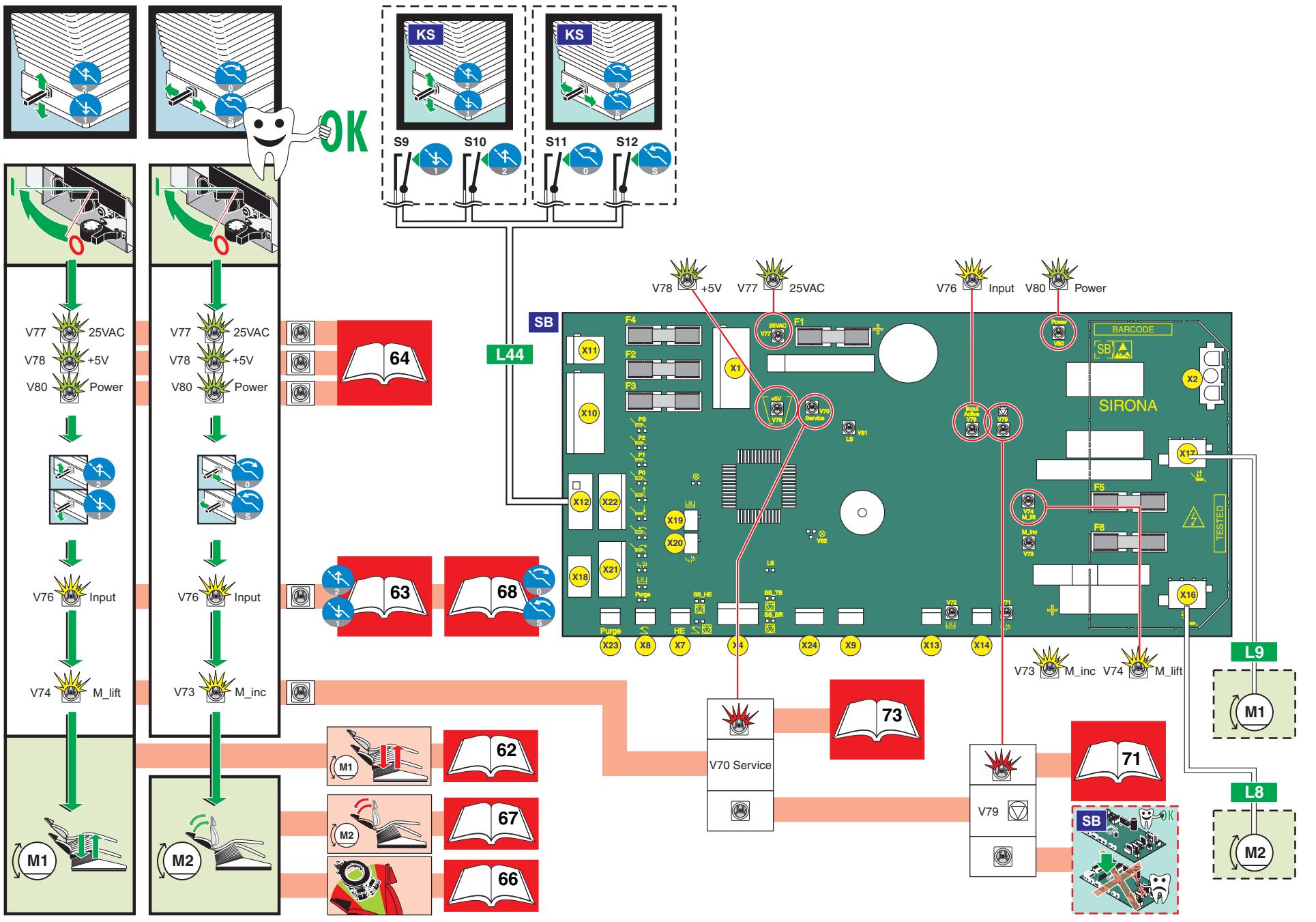








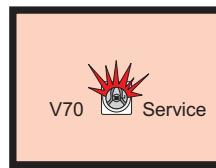
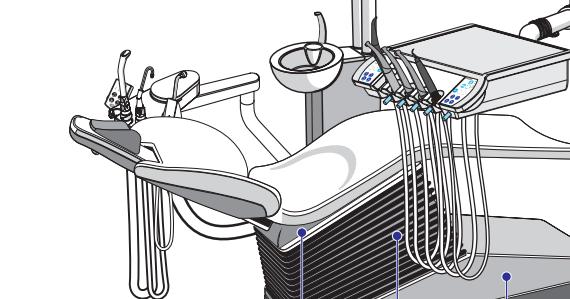
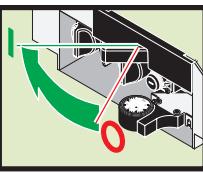


SB

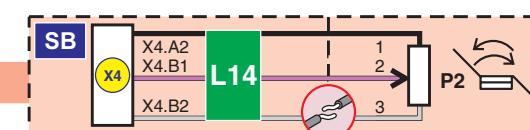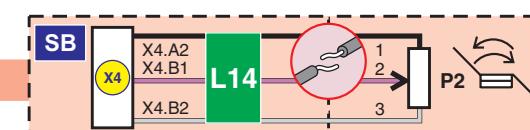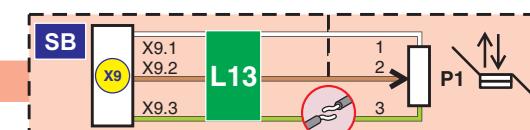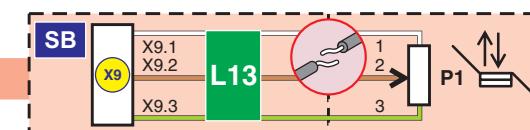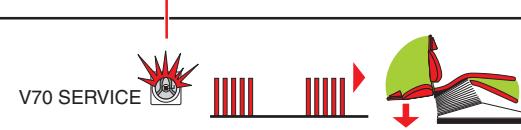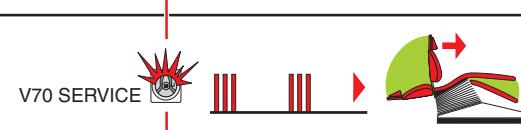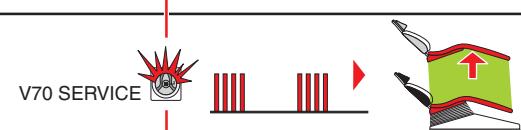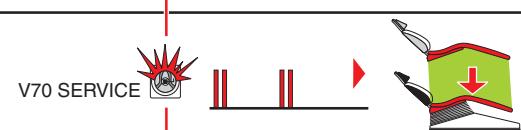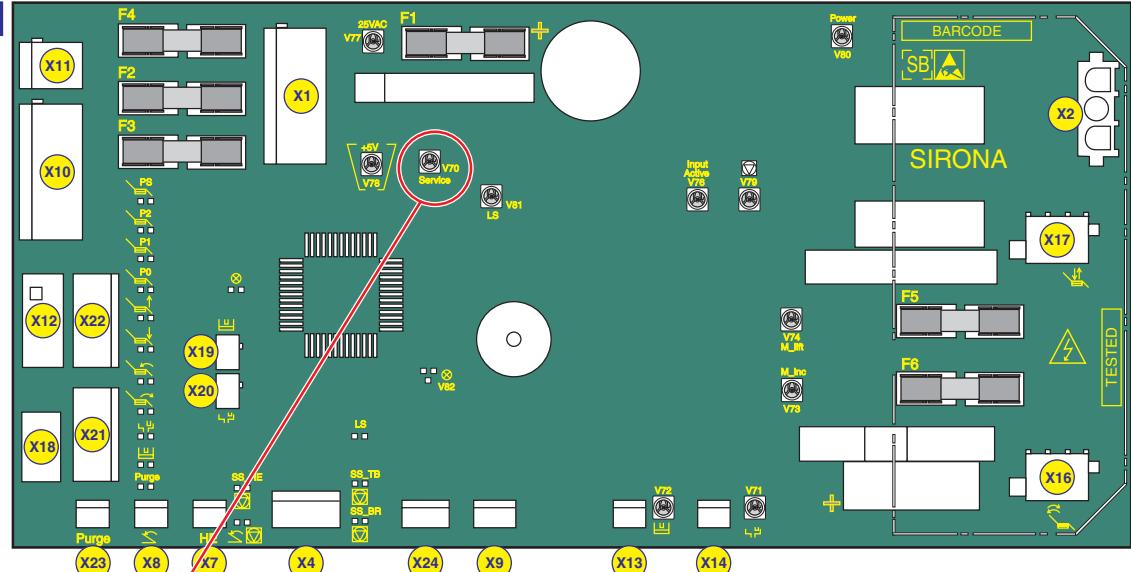

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


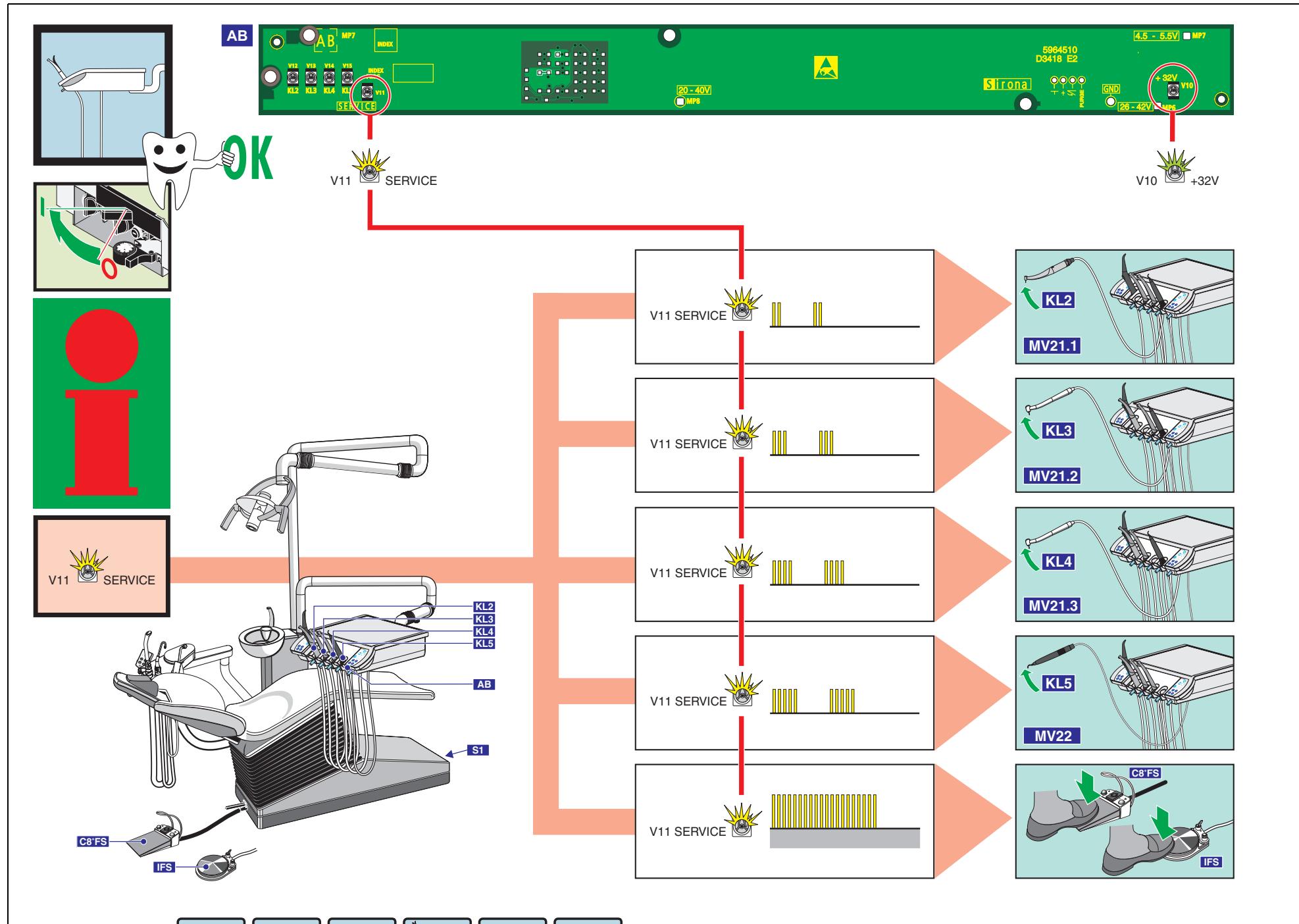

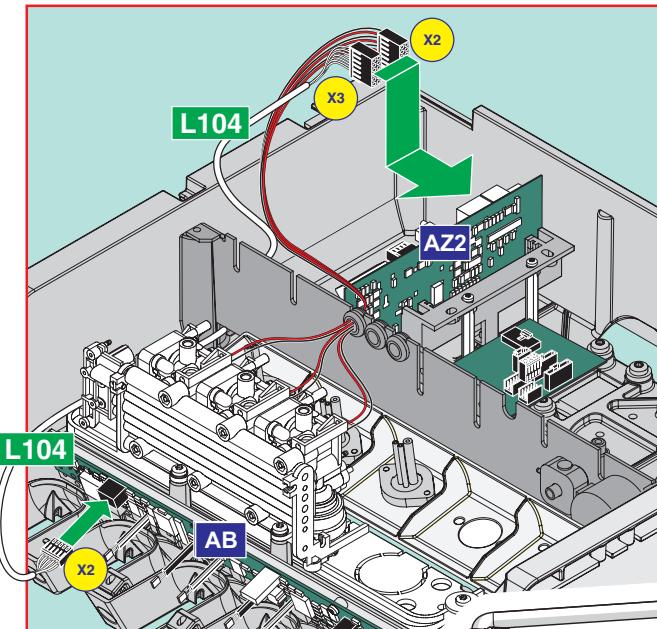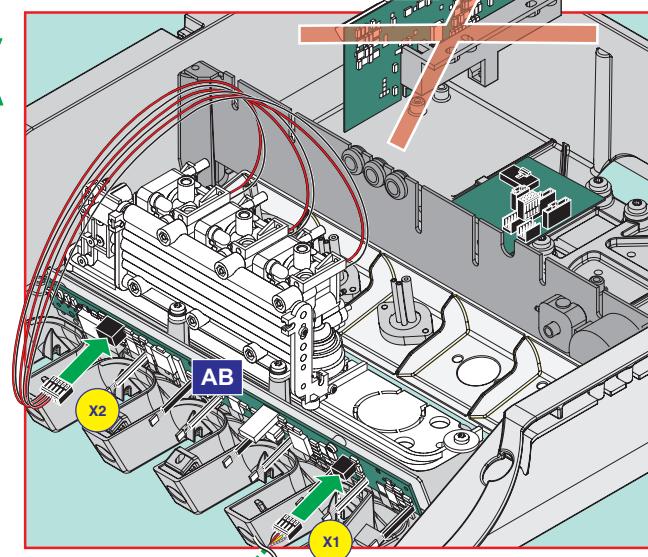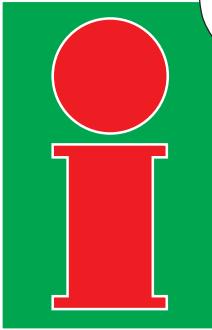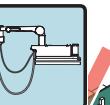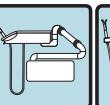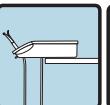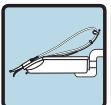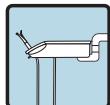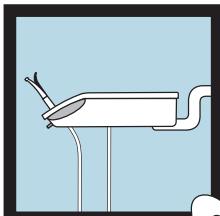



V8 SERVICE













AZ2

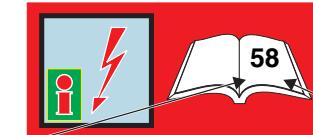






L102




L102


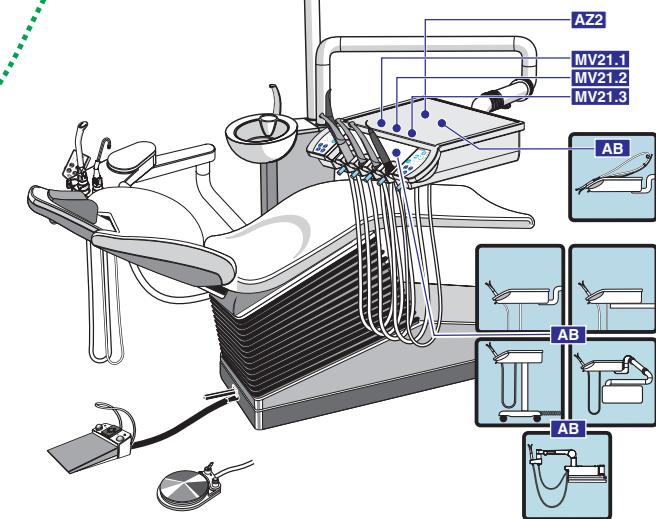
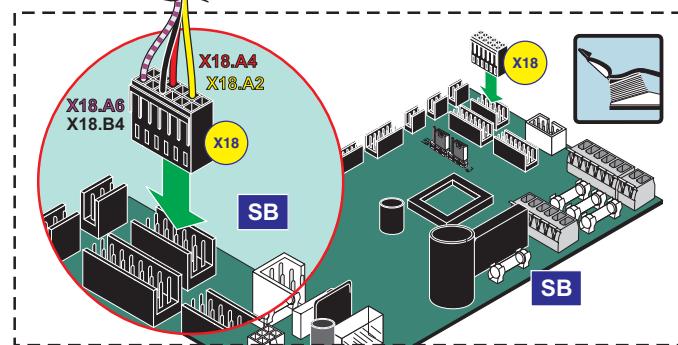


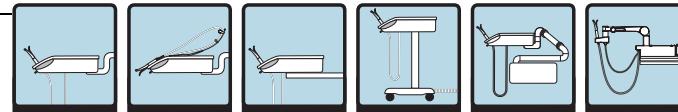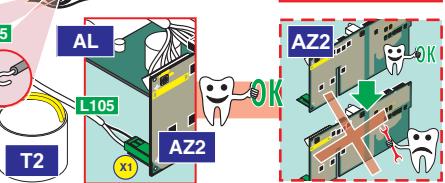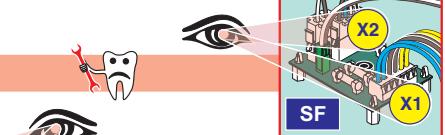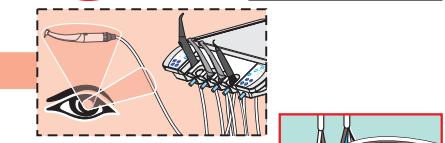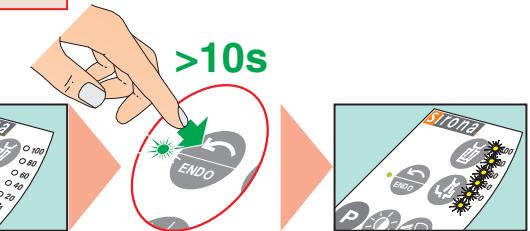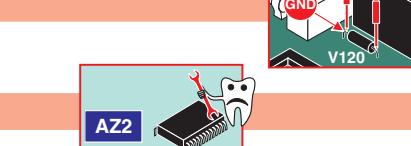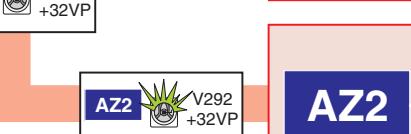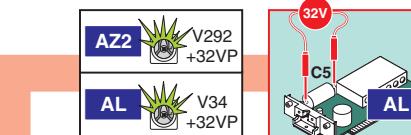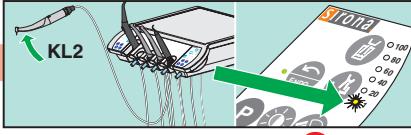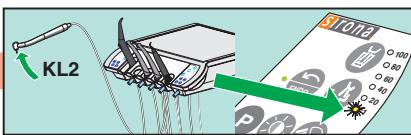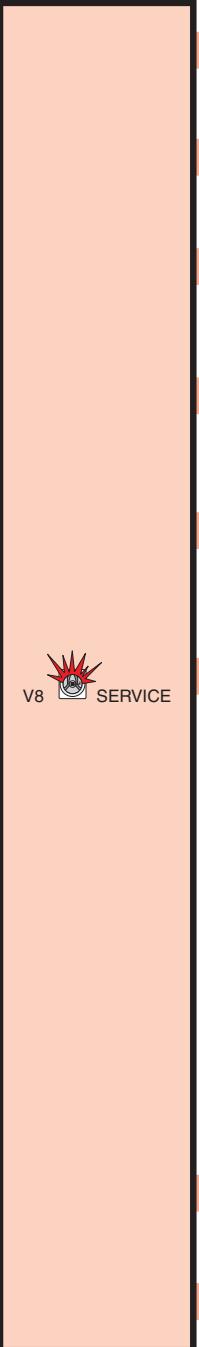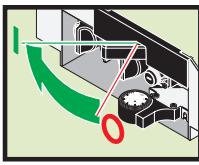

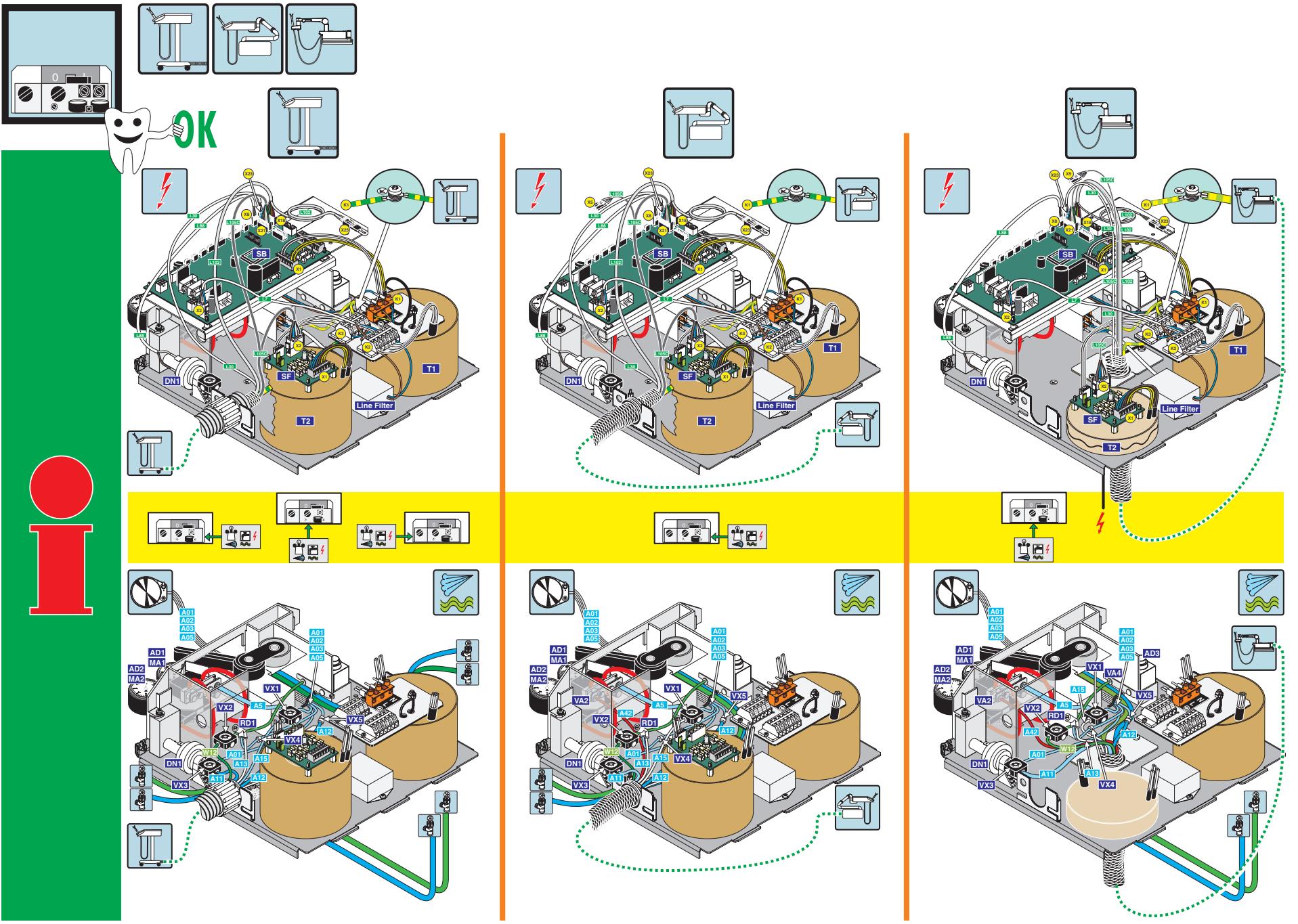

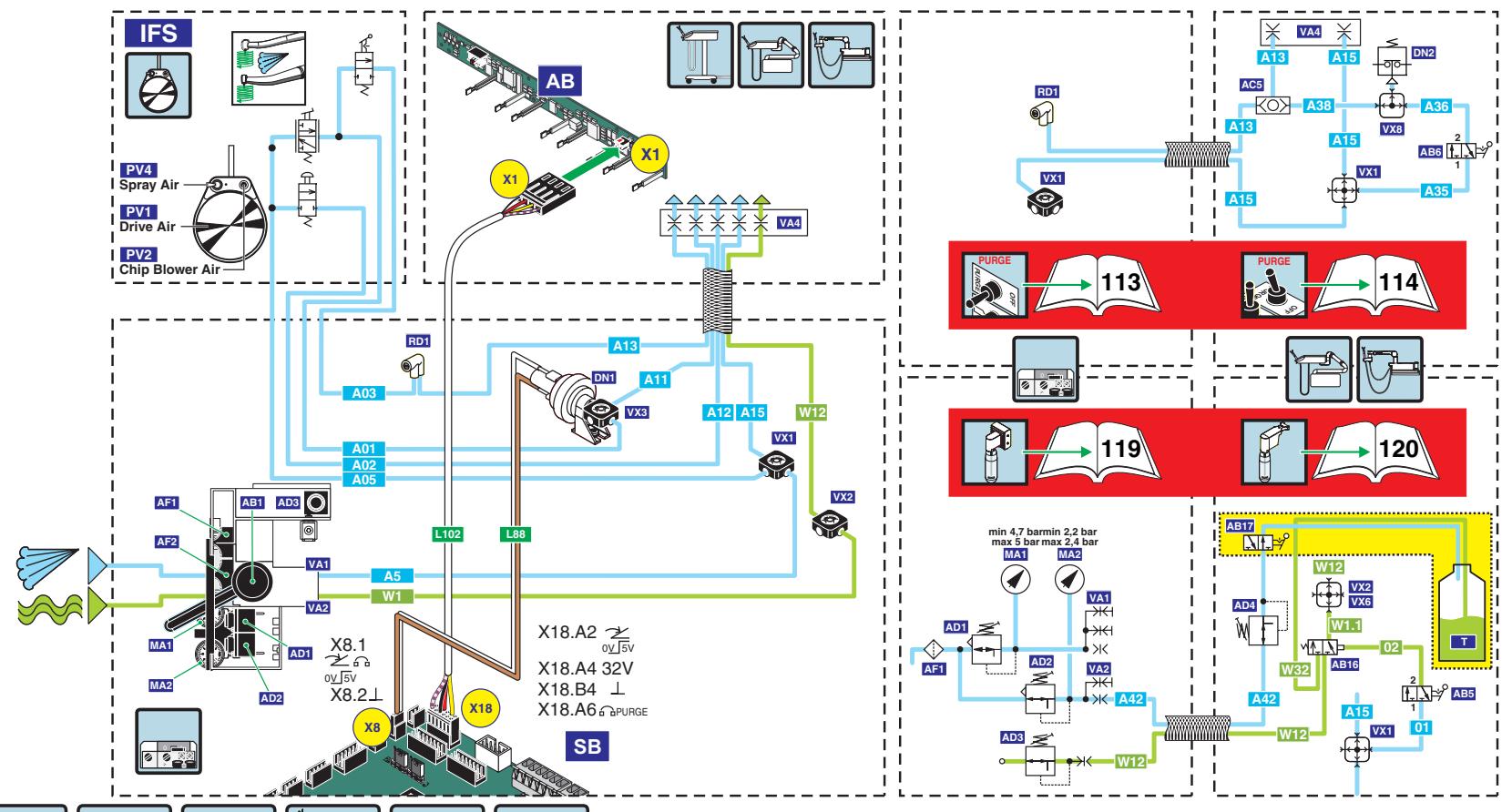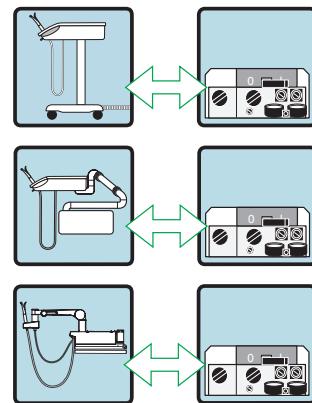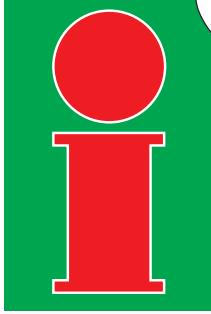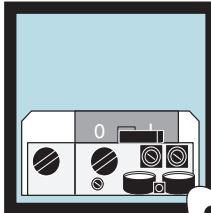

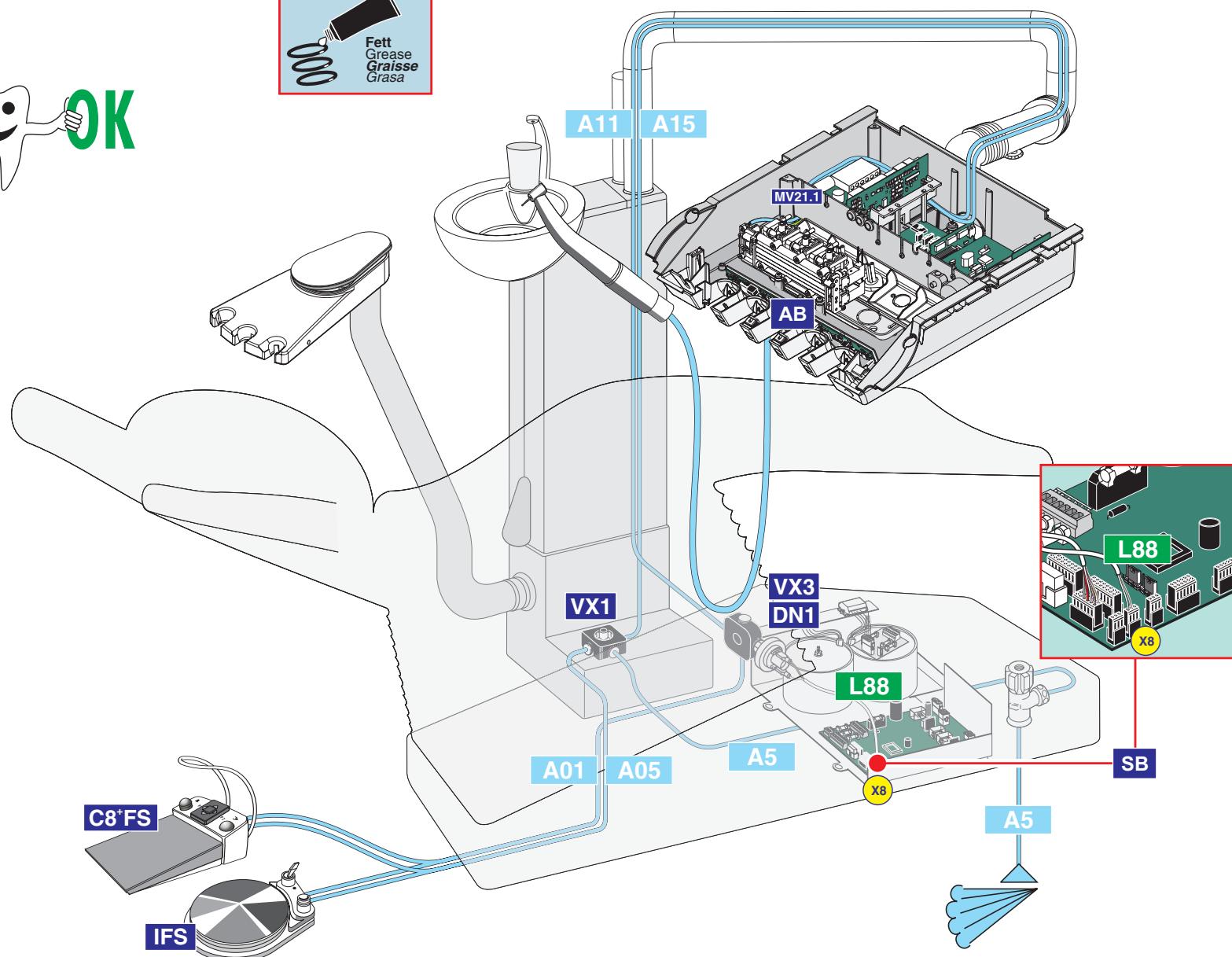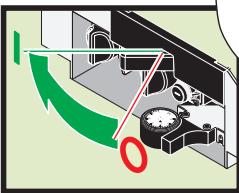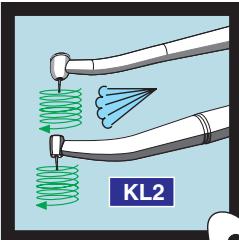

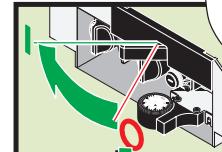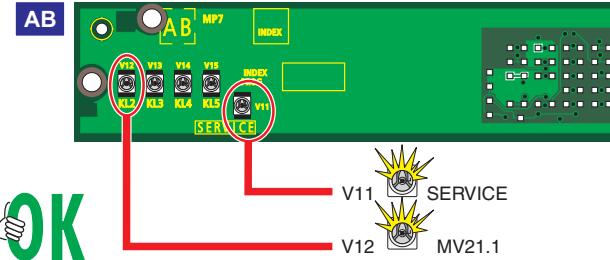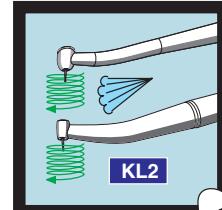

V77/SB 25VAC

V78/SB +5V

V80/SB Power

V10/AB +32V

V11/AB SERVICE

KL2

V11/AB SERVICE

KL2

MV21.1

DN1 0V/5V

V11/AB SERVICE

KL2

64

L102

92

AB

OK

58

A03
A01
A02
A05

A5

L102

L88

X8.1

X8.2

SB

X18.A2




X18.A4


X18.B4








X18.A6


PURGE




77

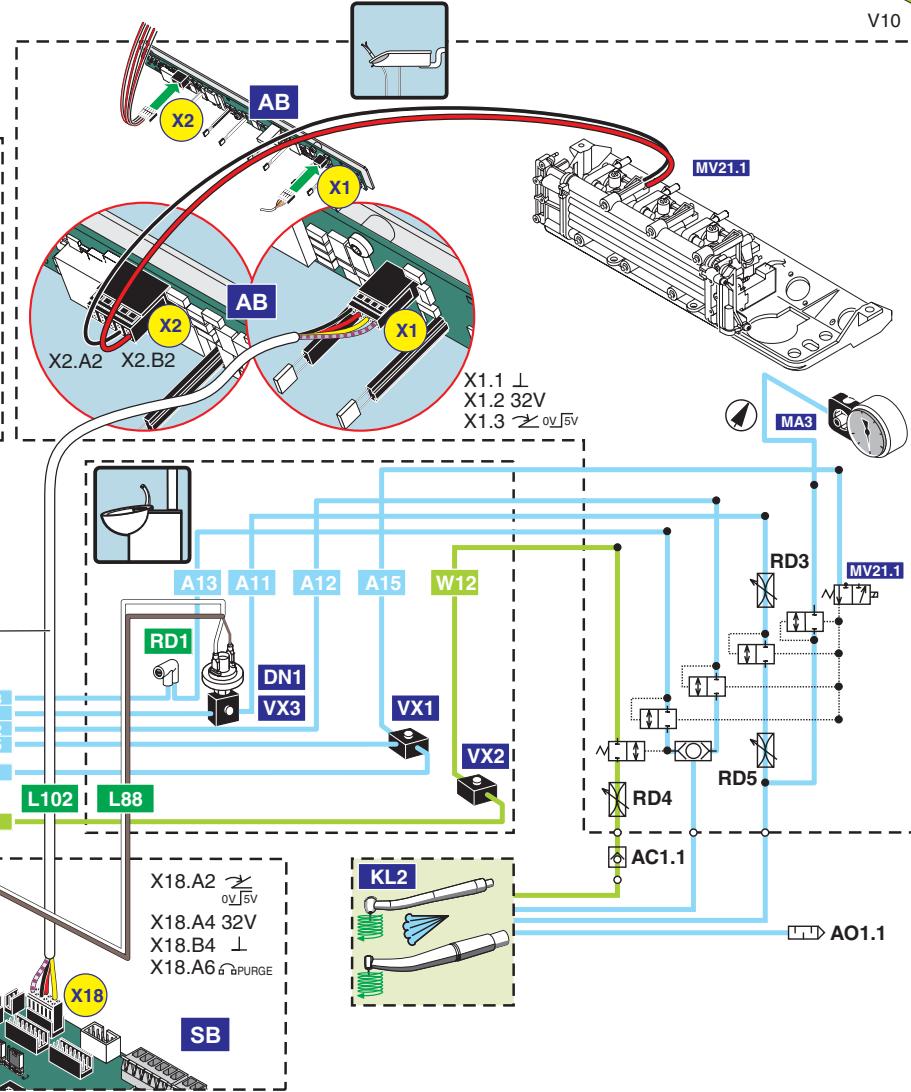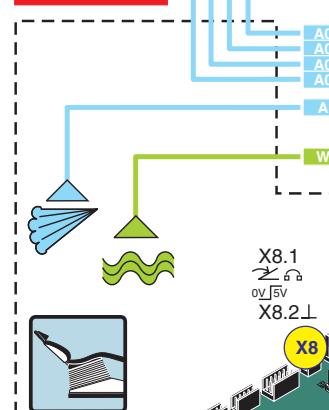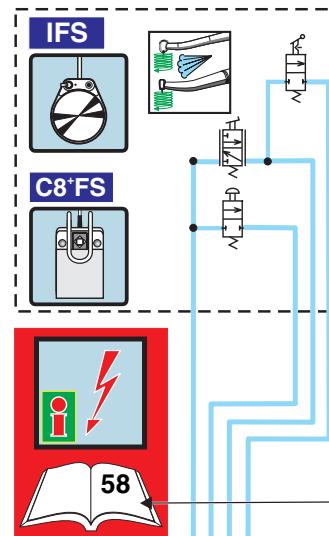

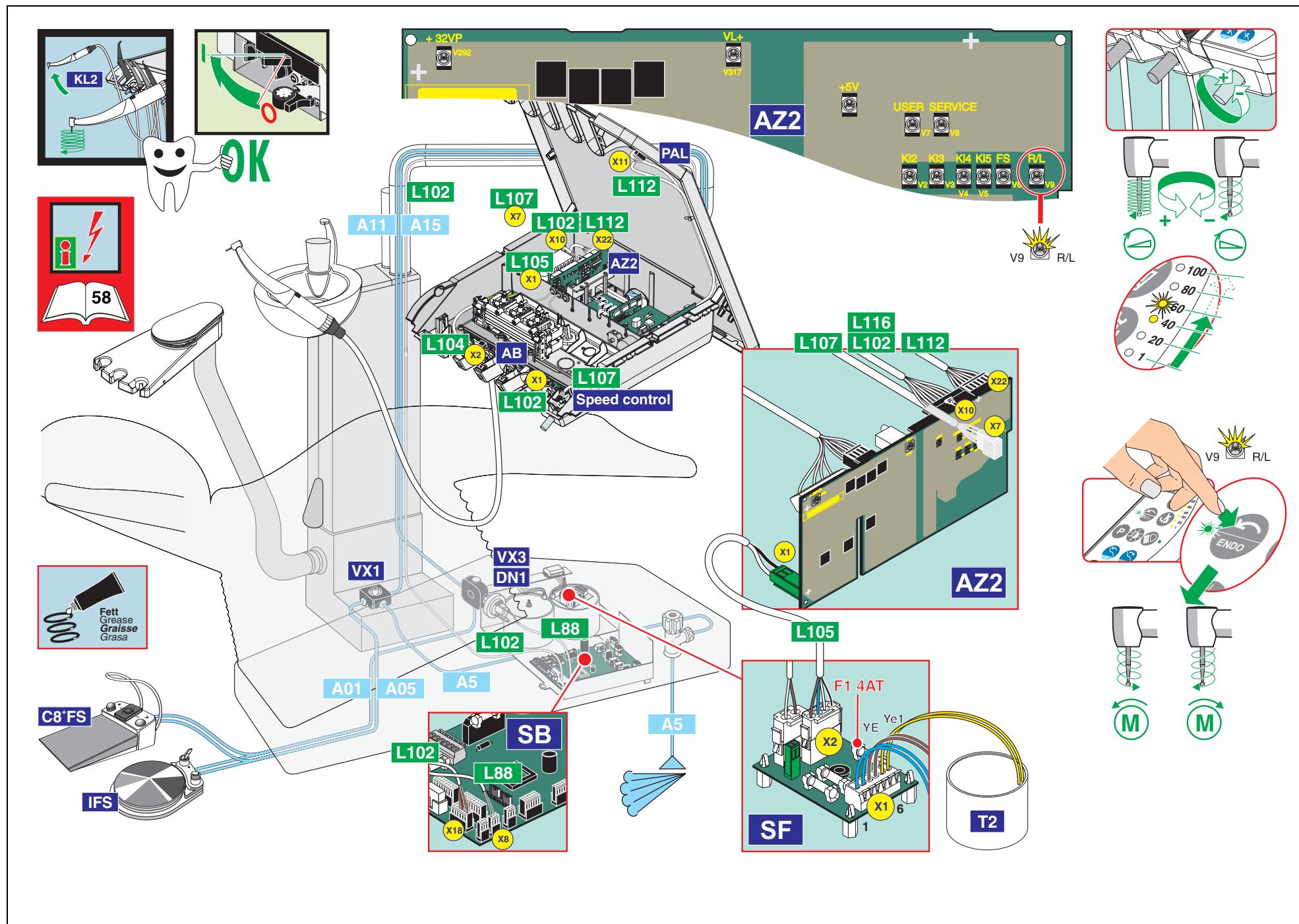

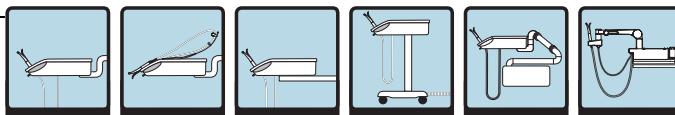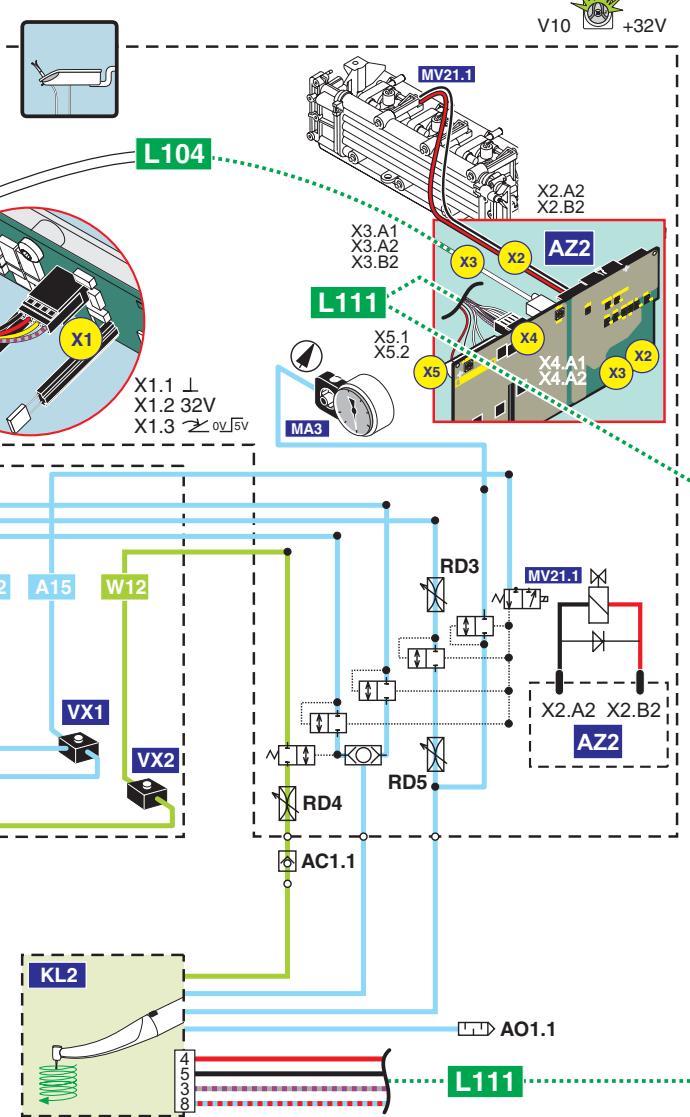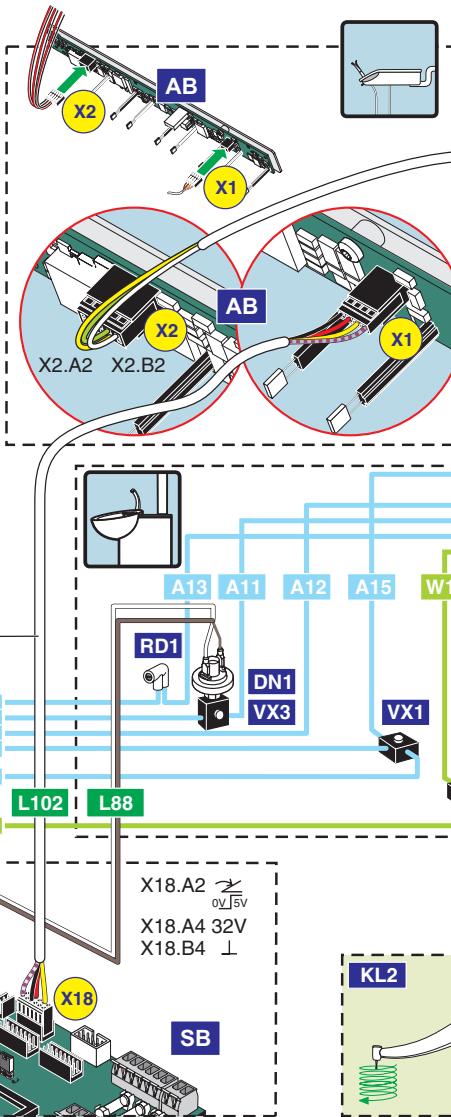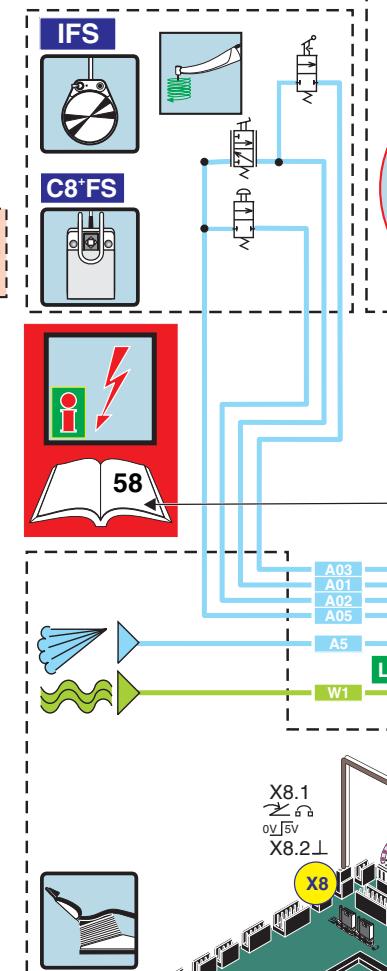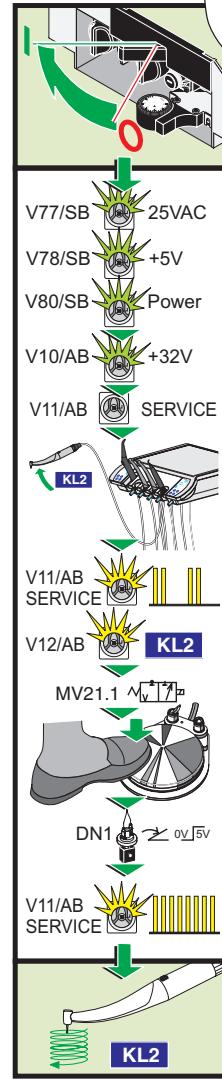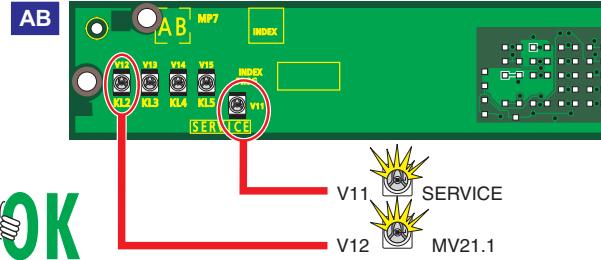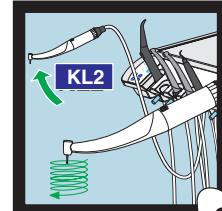

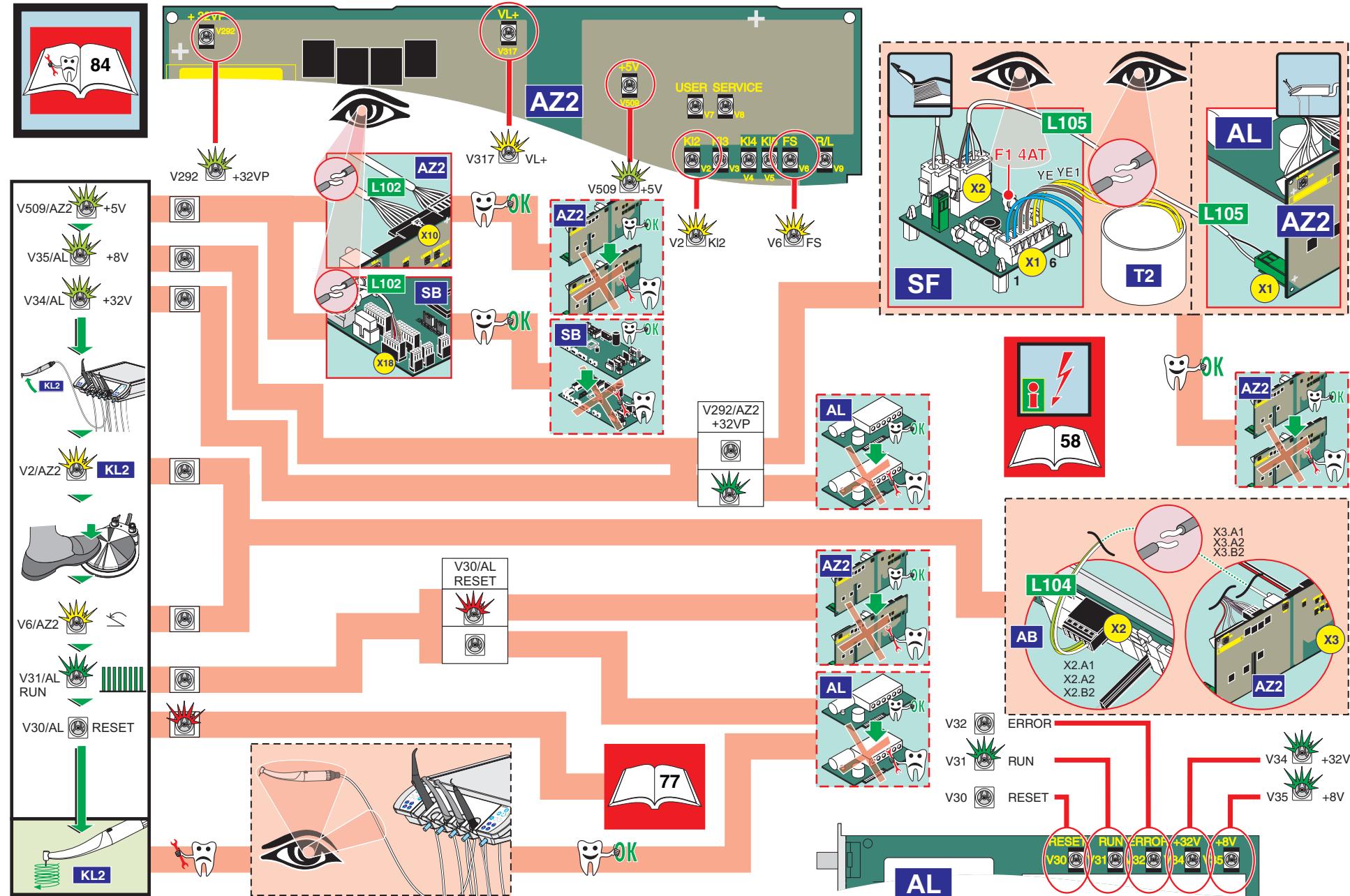

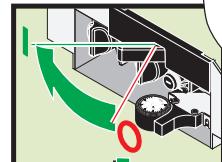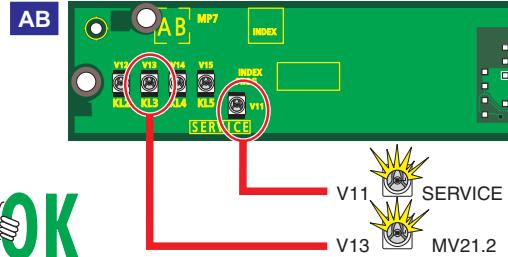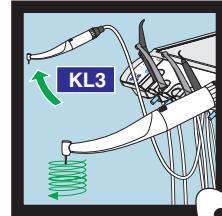

V77/SB 25VAC

V78/SB +5V

V80/SB Power

V10/AB +32V

V11/AB SERVICE

KL3

V11/AB SERVICE

KL3





MV21.2


DN1







DN1 0V 5V




V11/AB SERVICE

KL3

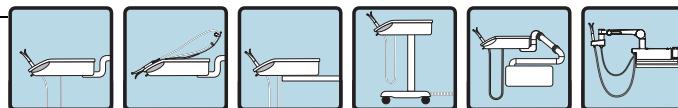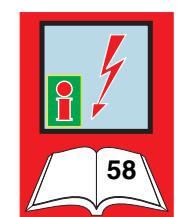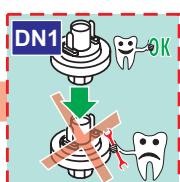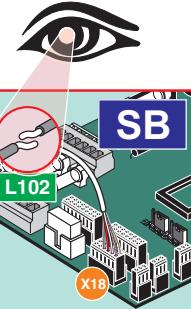

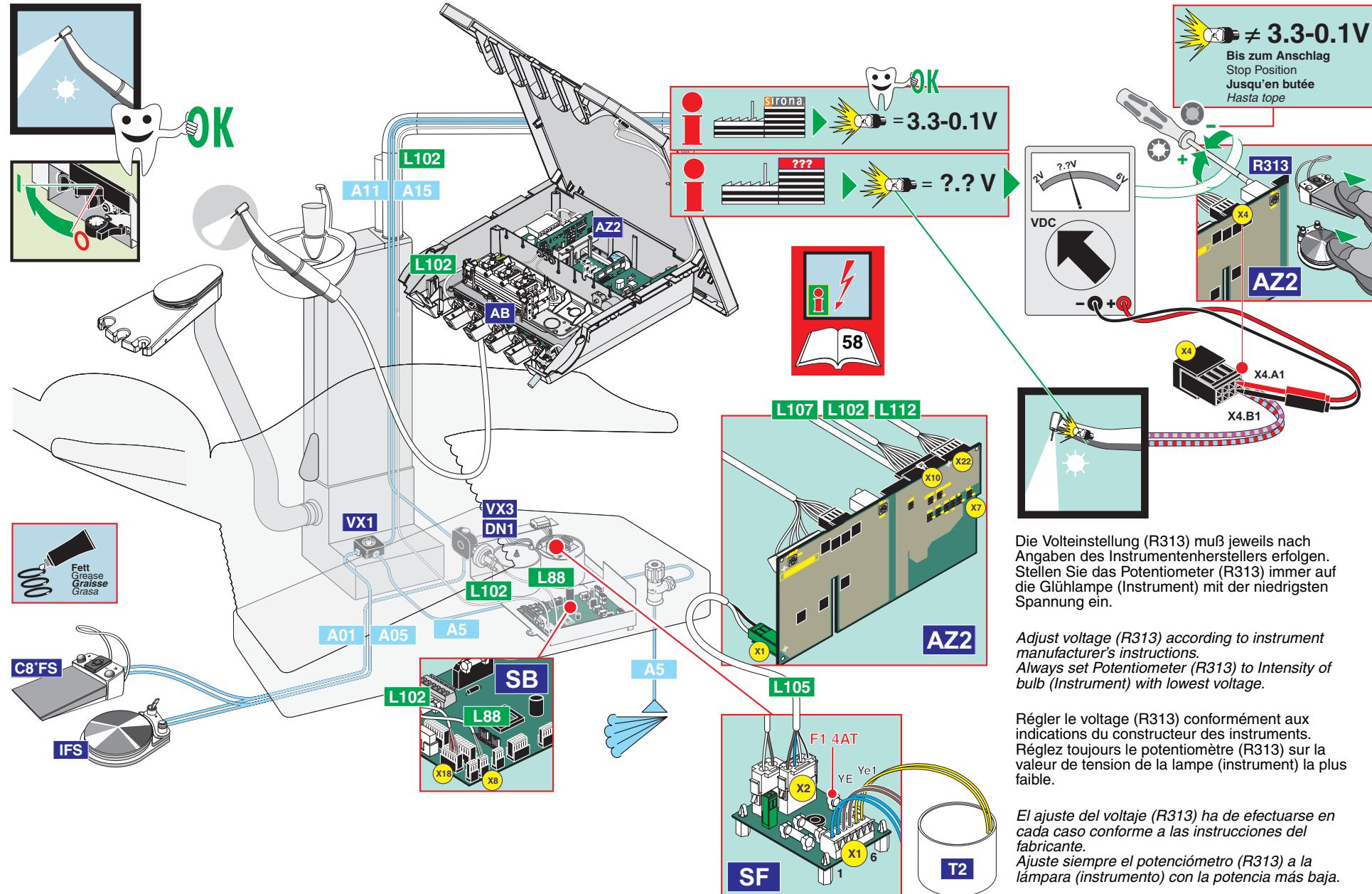

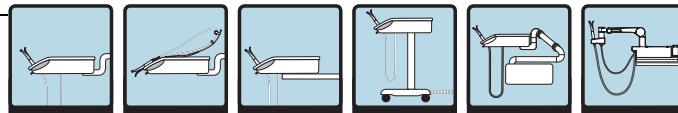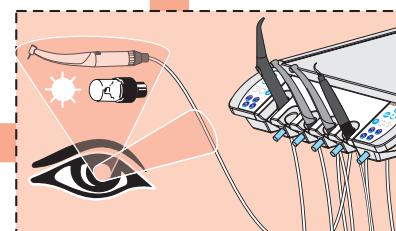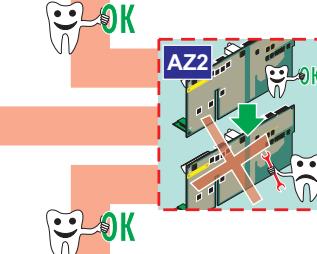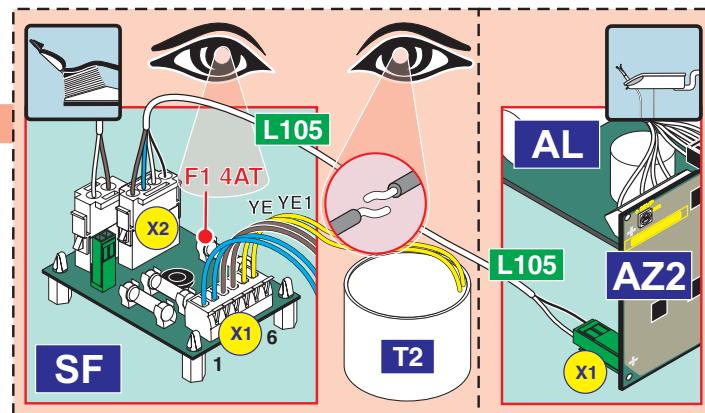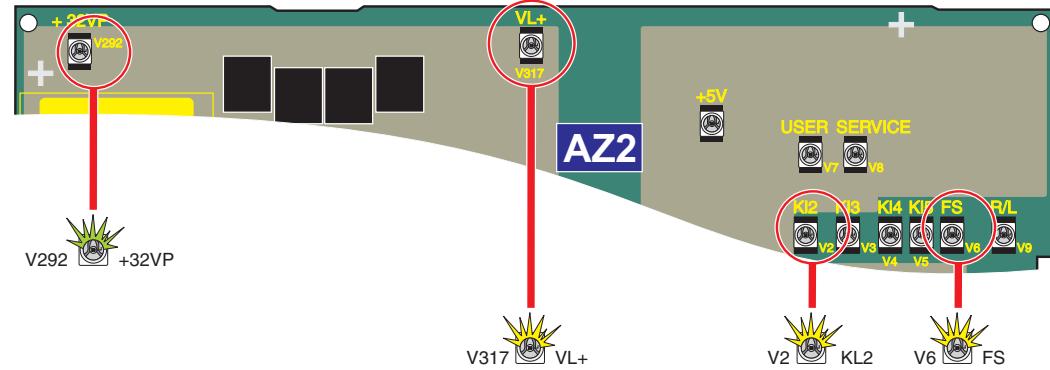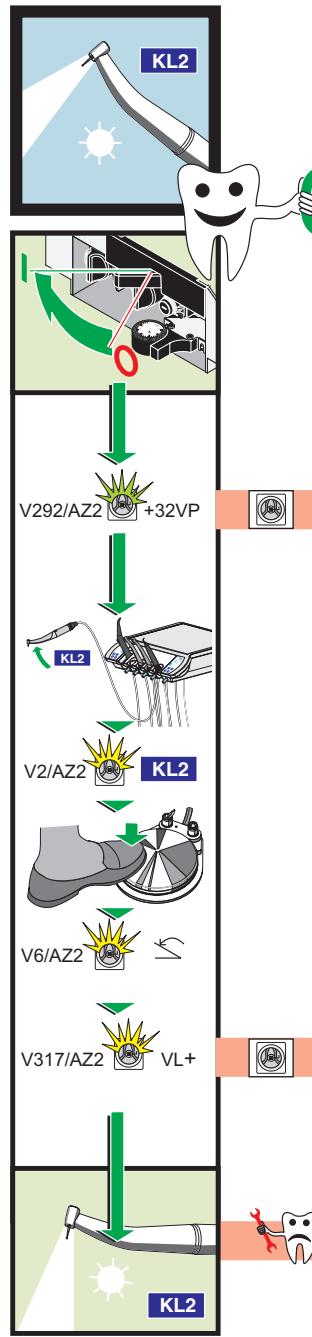

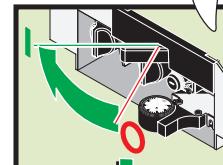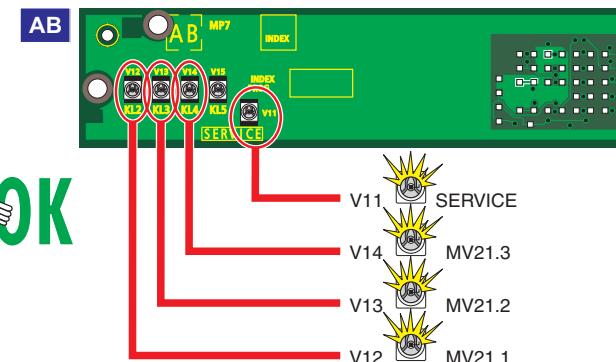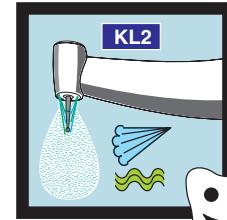

V77/SB 25VAC
V78/SB +5V
V80/SB Power
V10/AB +32V
V11/AB SERVICE



KL2






V11/AB SERVICE

KL2

MV21.1

KL2



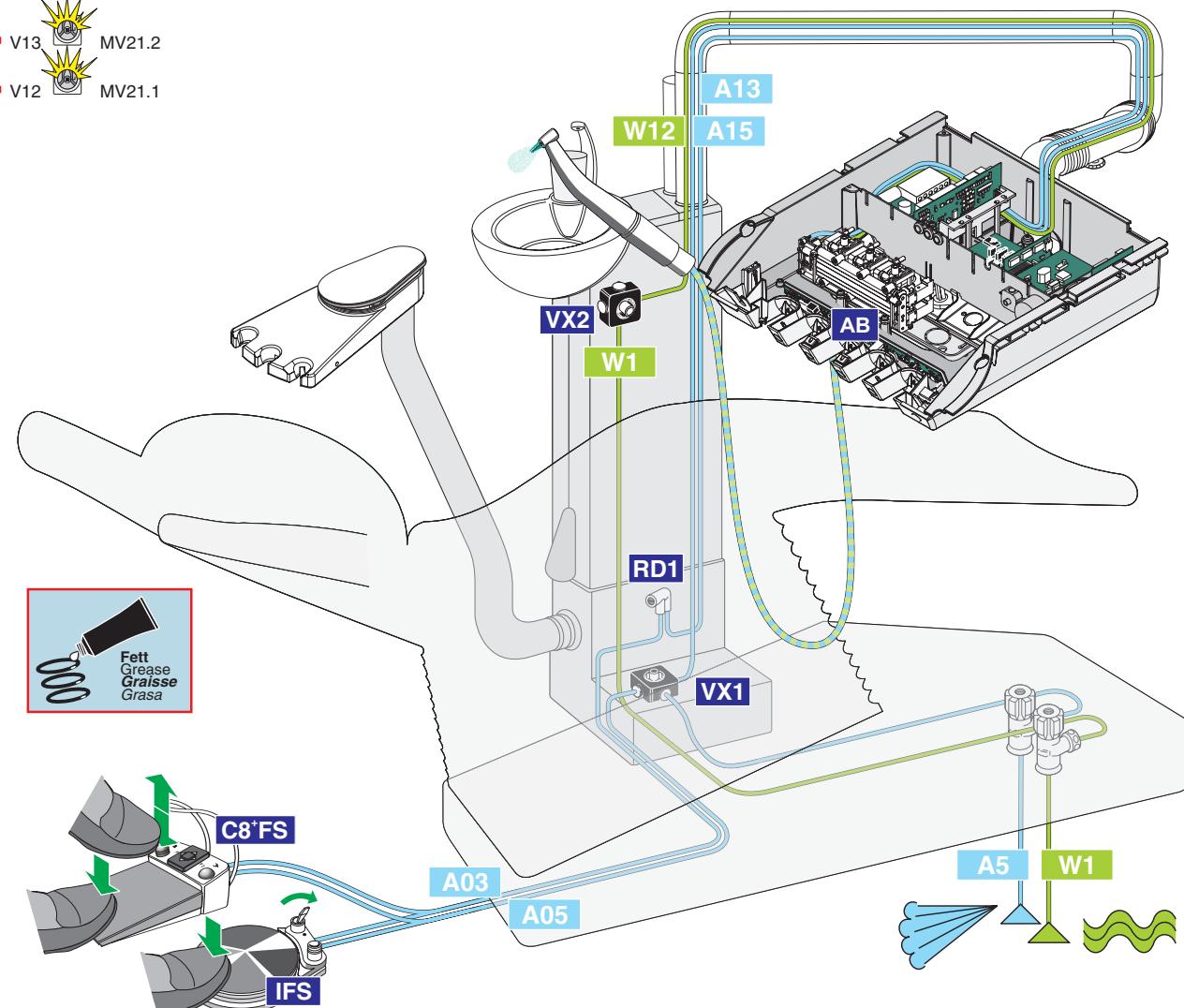


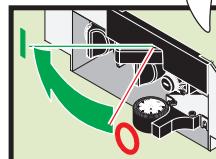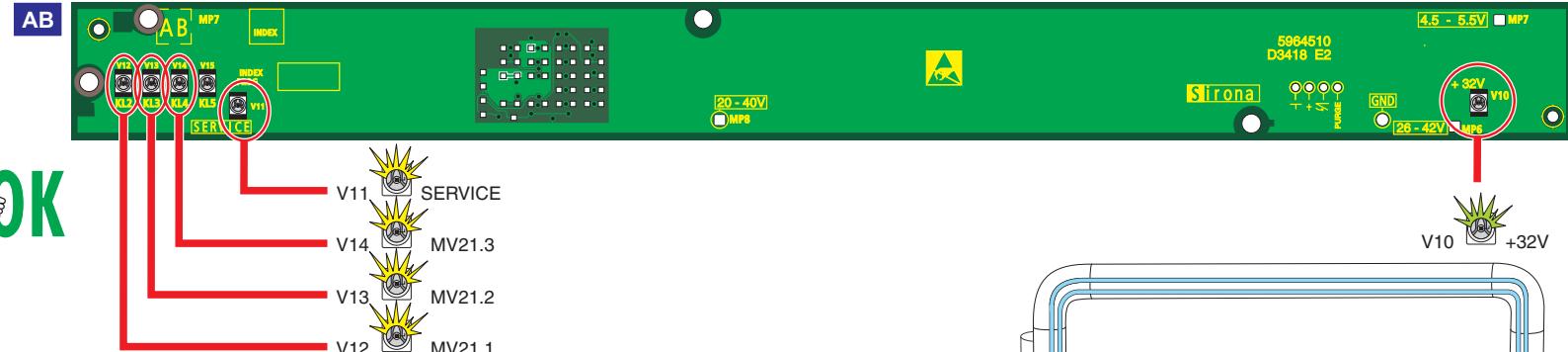



- V77/SB 25VAC
- V78/SB +5V
- V80/SB Power
- V10/SB +32V
- V11/AB SERVICE
- KL2





- V11/AB SERVICE
- KL2
- MV21.1

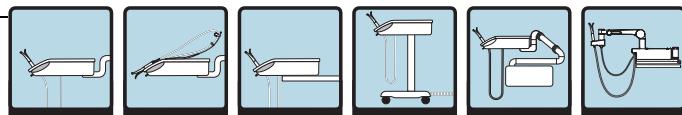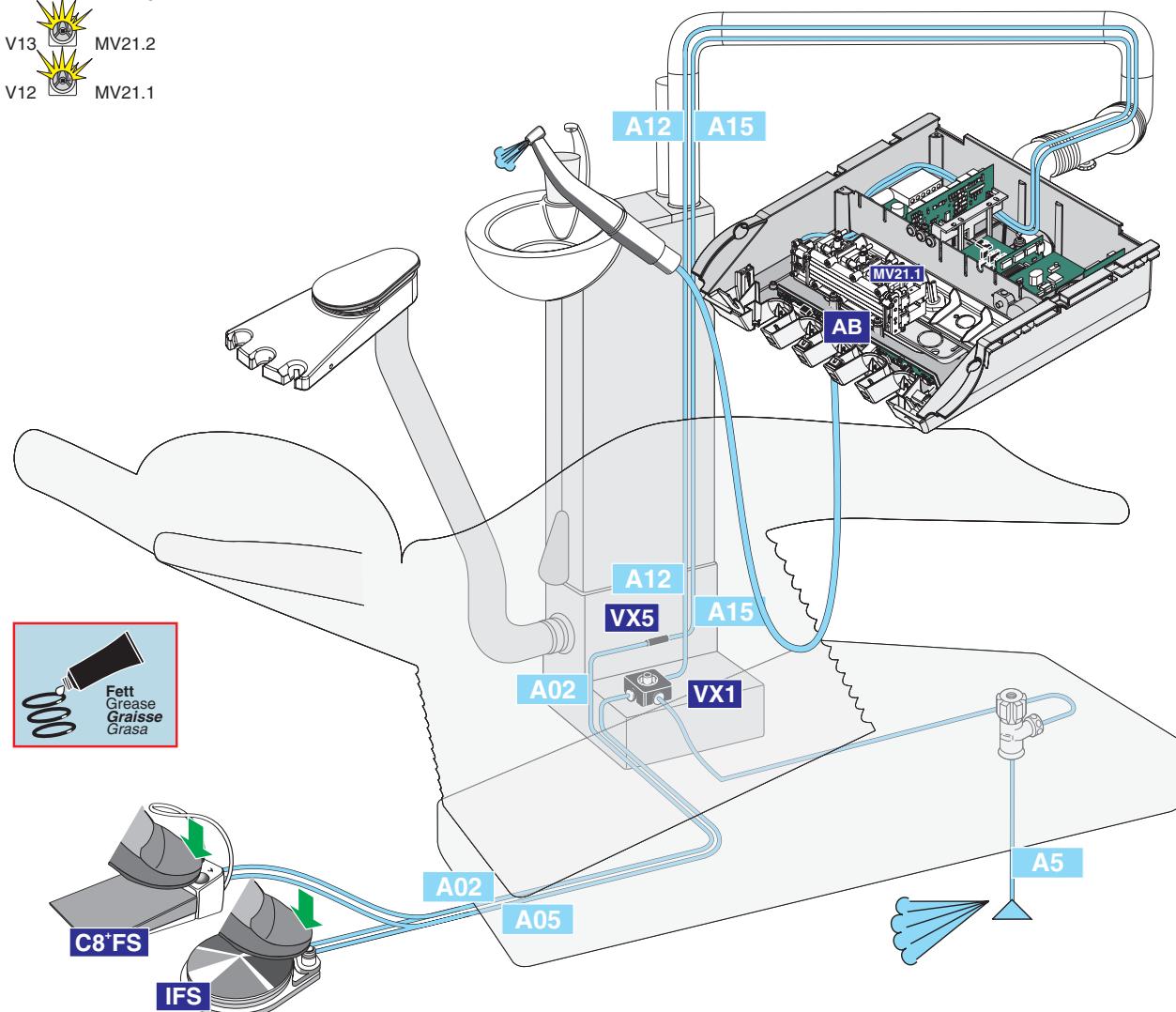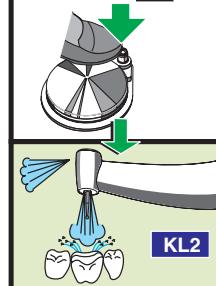


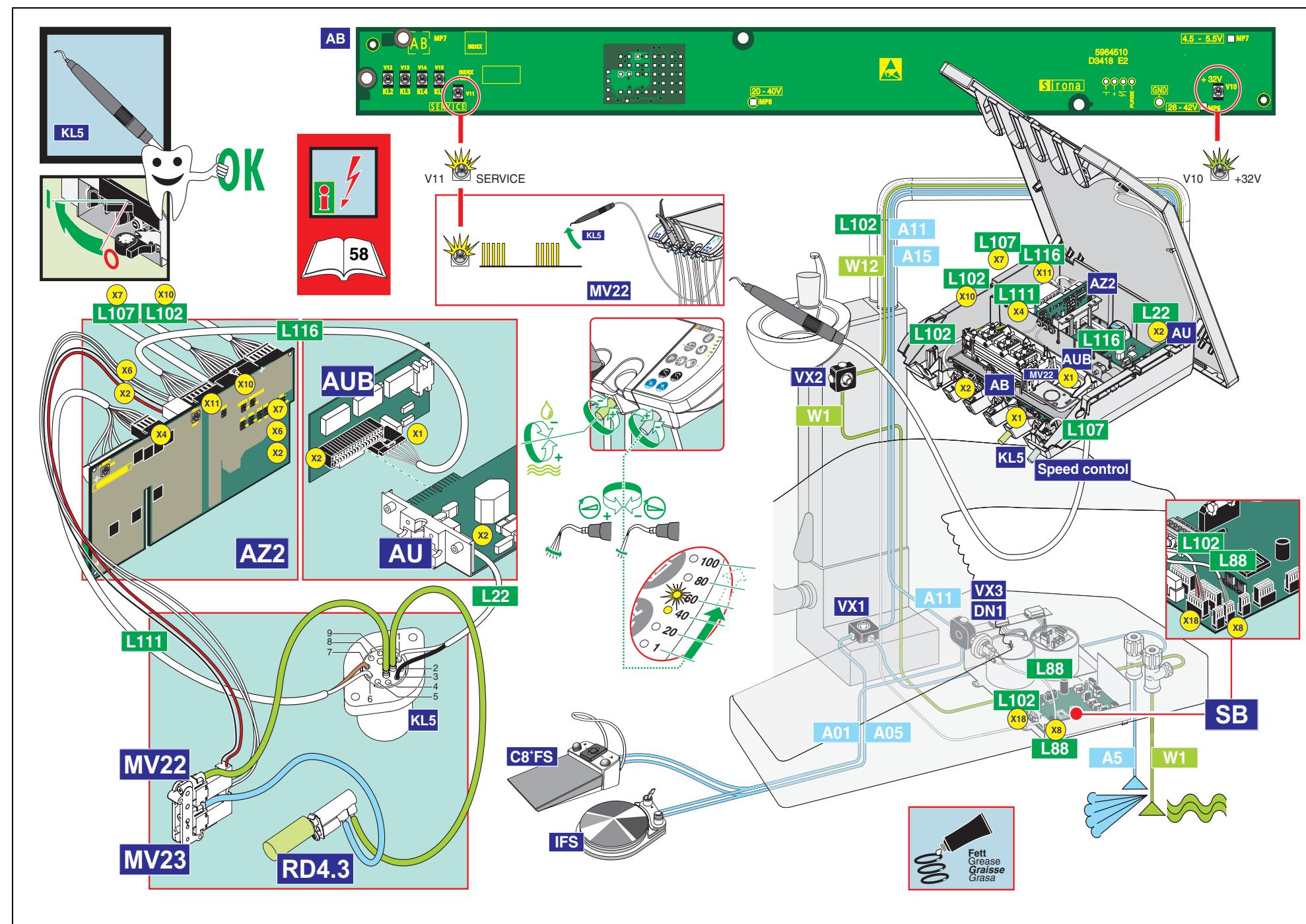

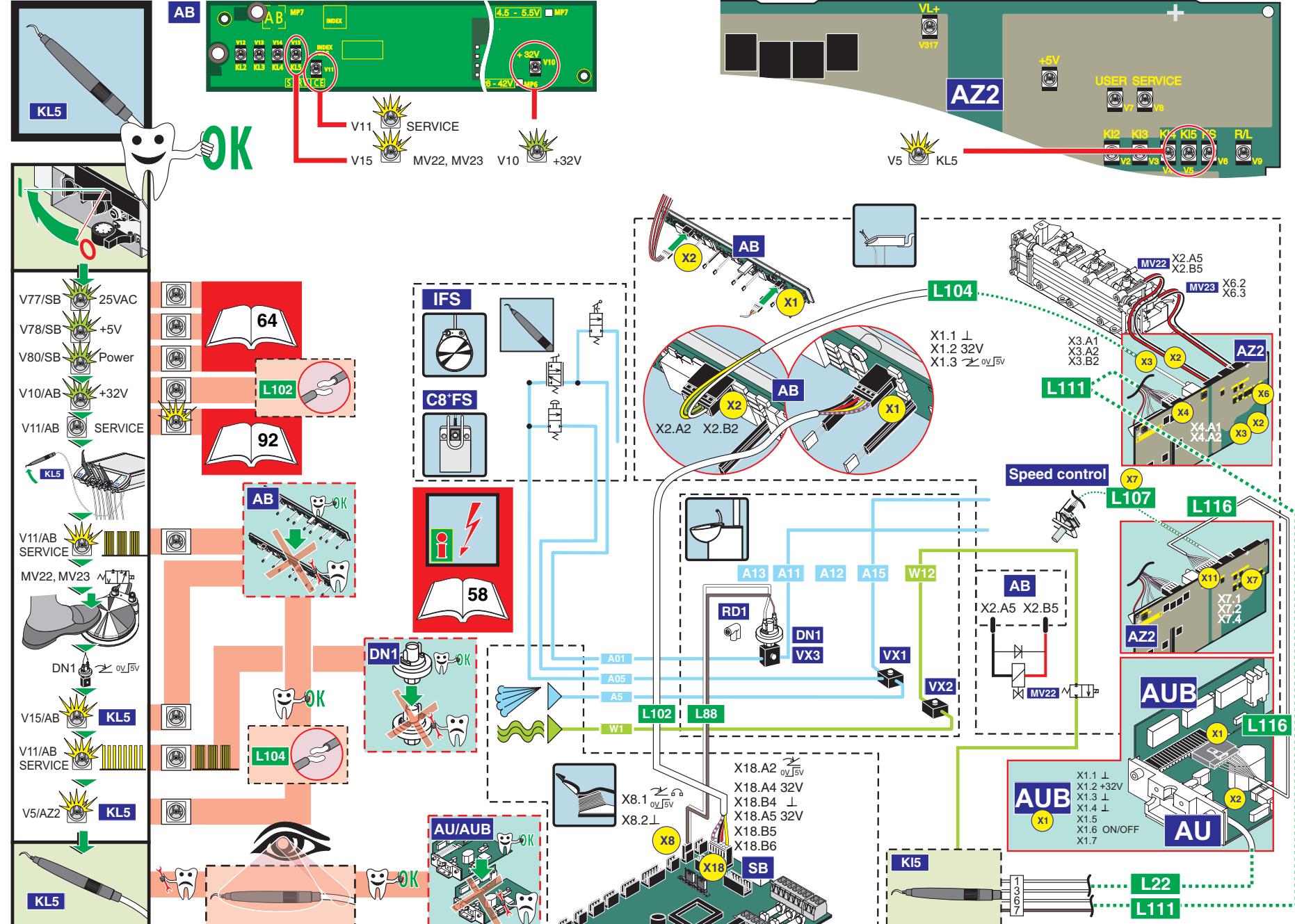


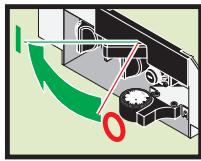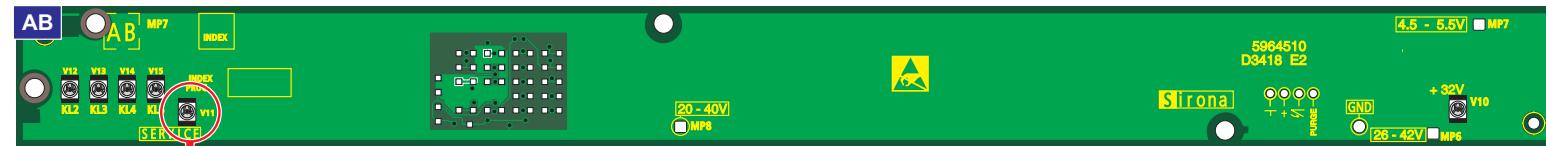


V11 SERVICE

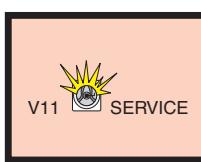


V11 SERVICE

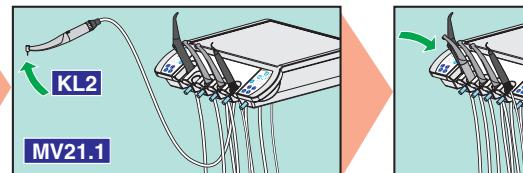


V11 SERVICE

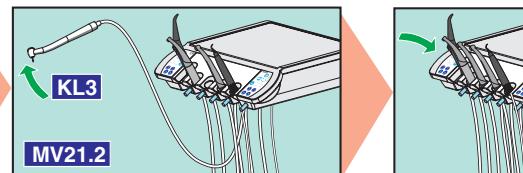



V11 SERVICE








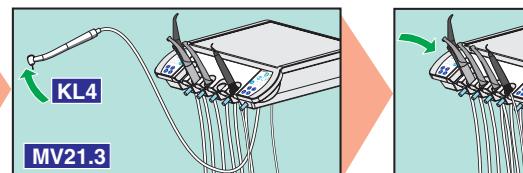



V11 SERVICE

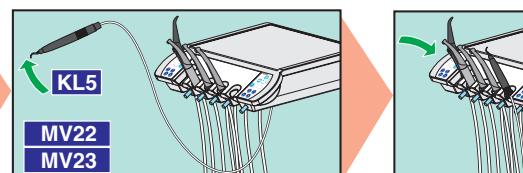






V11 SERVICE


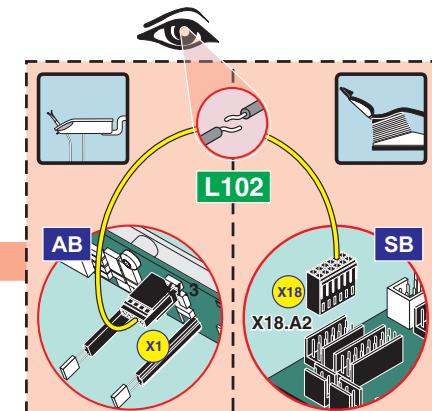
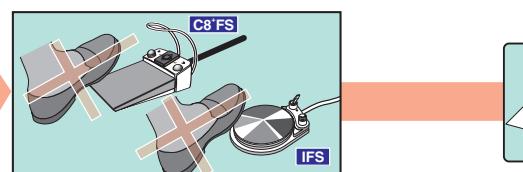


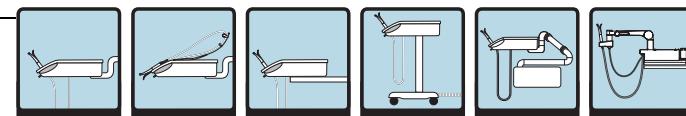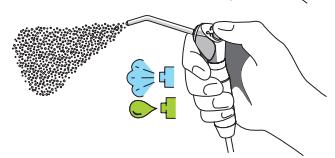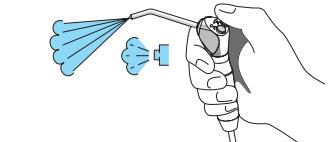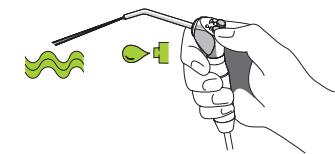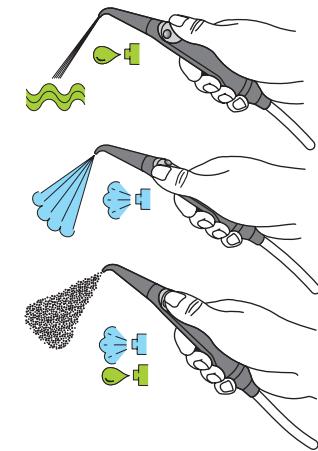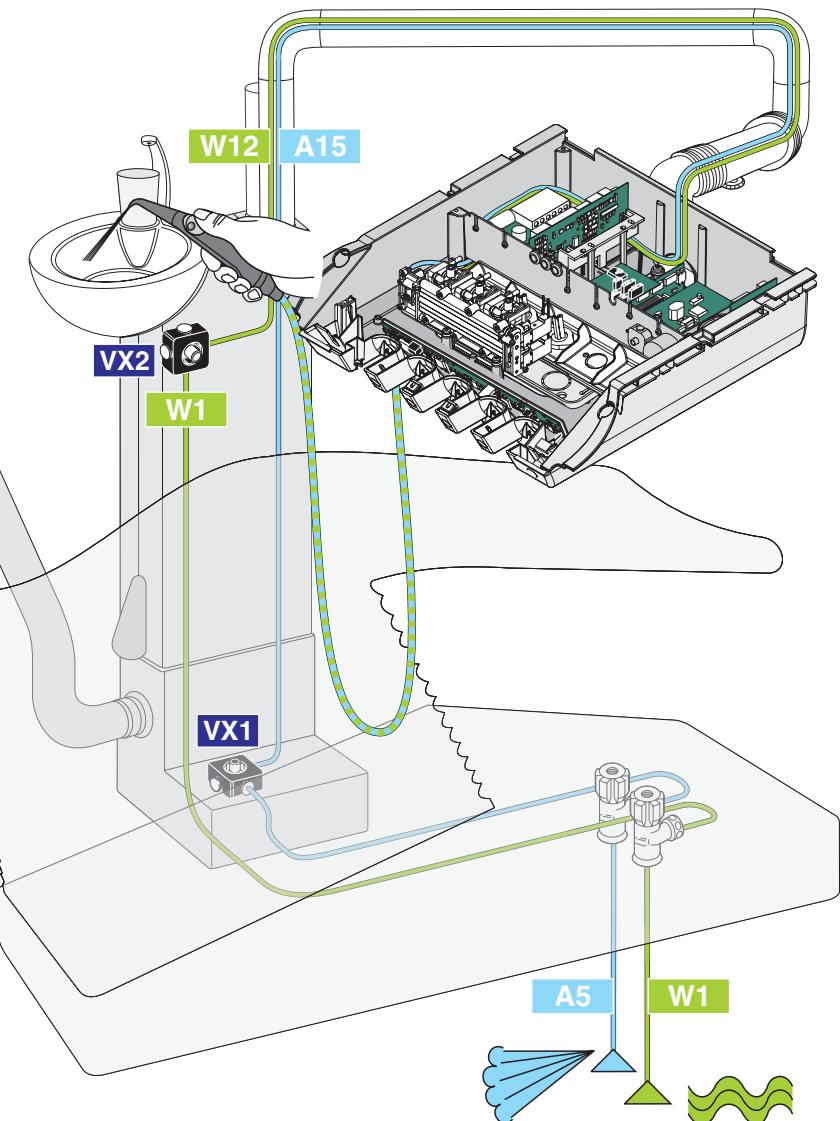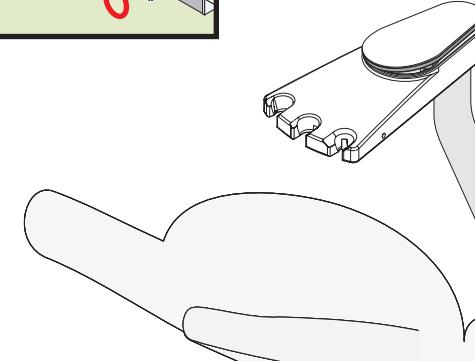
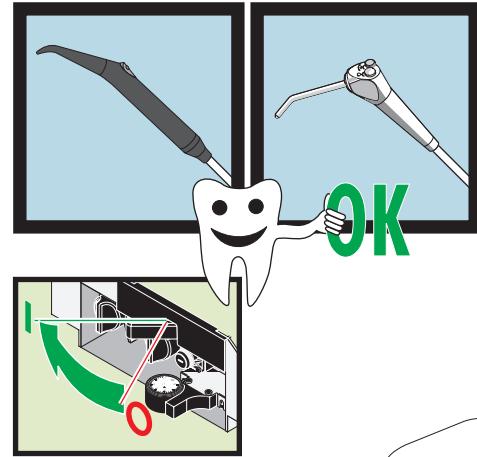









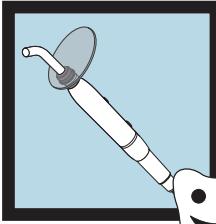



OK



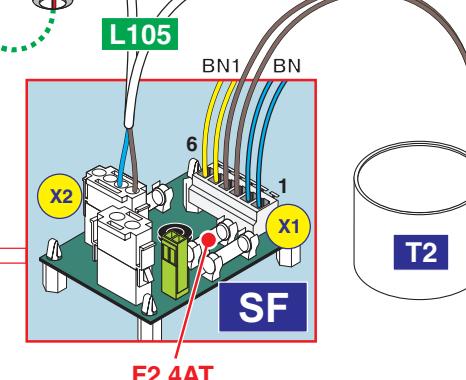
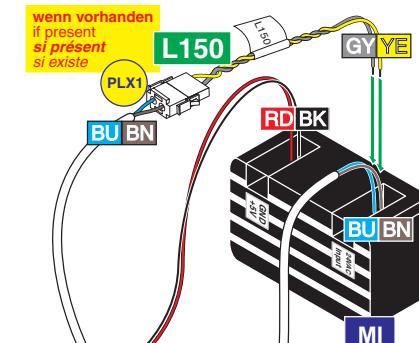
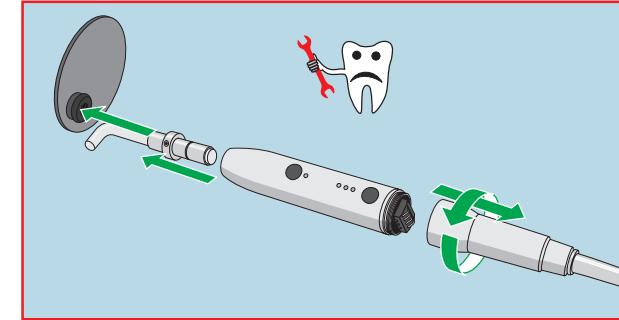
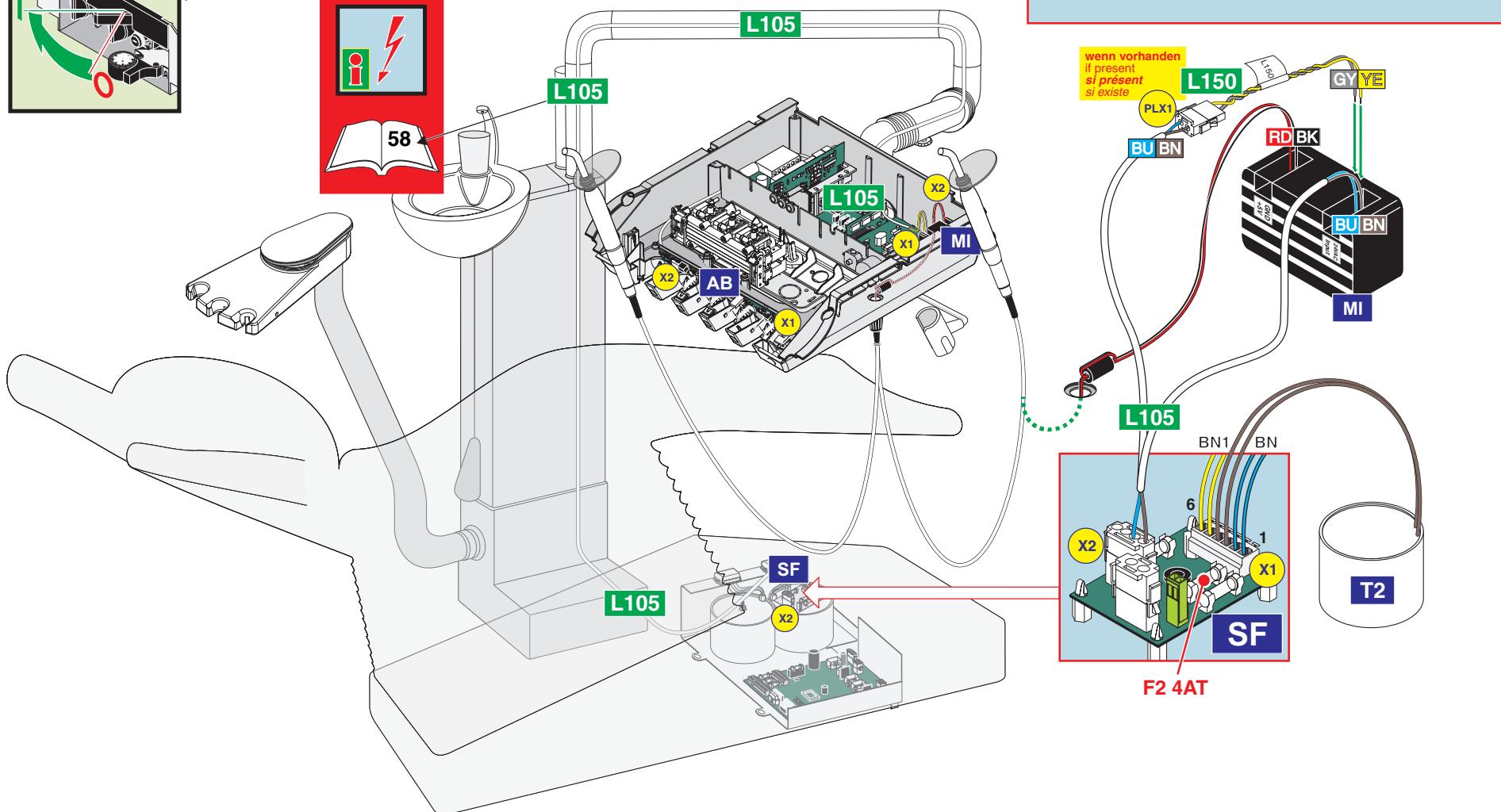
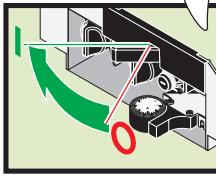


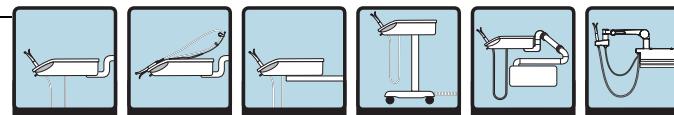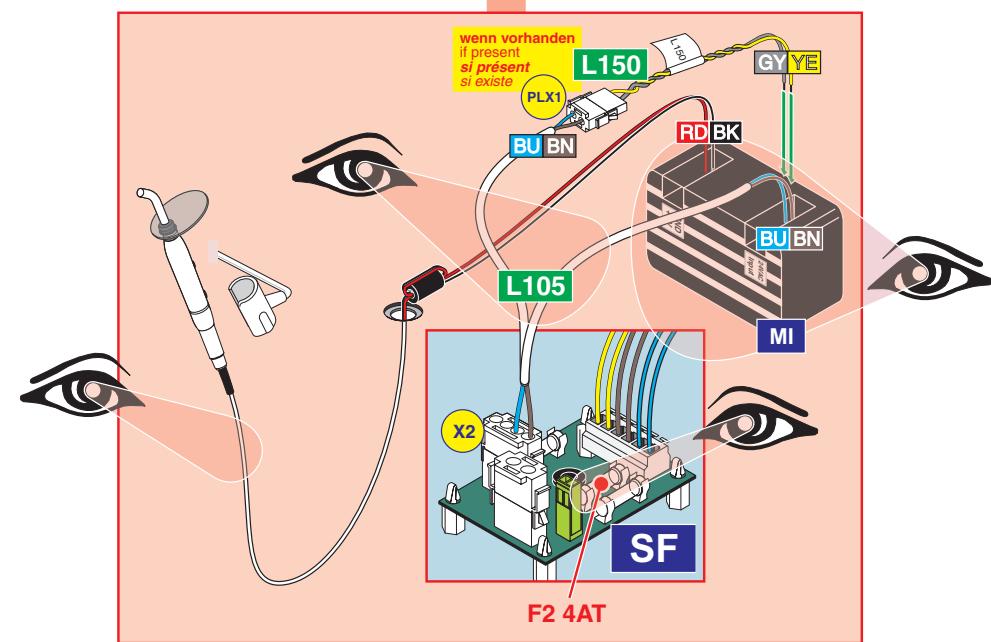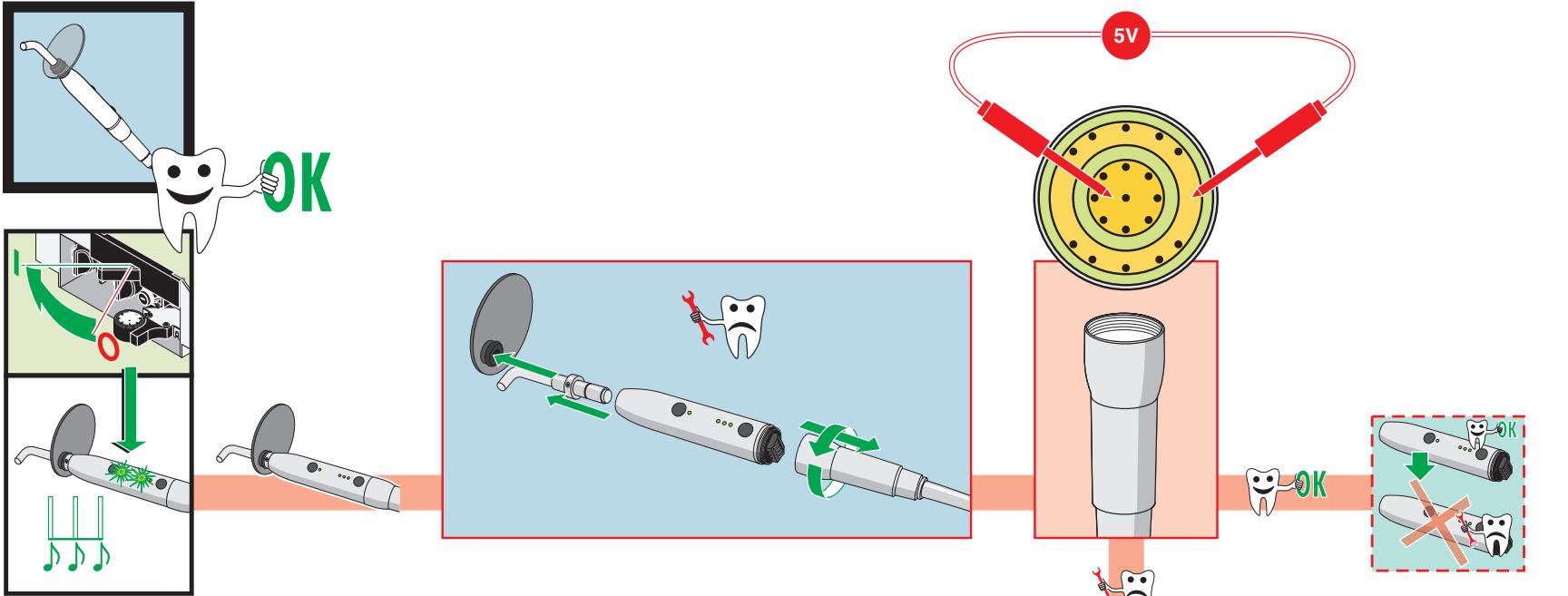

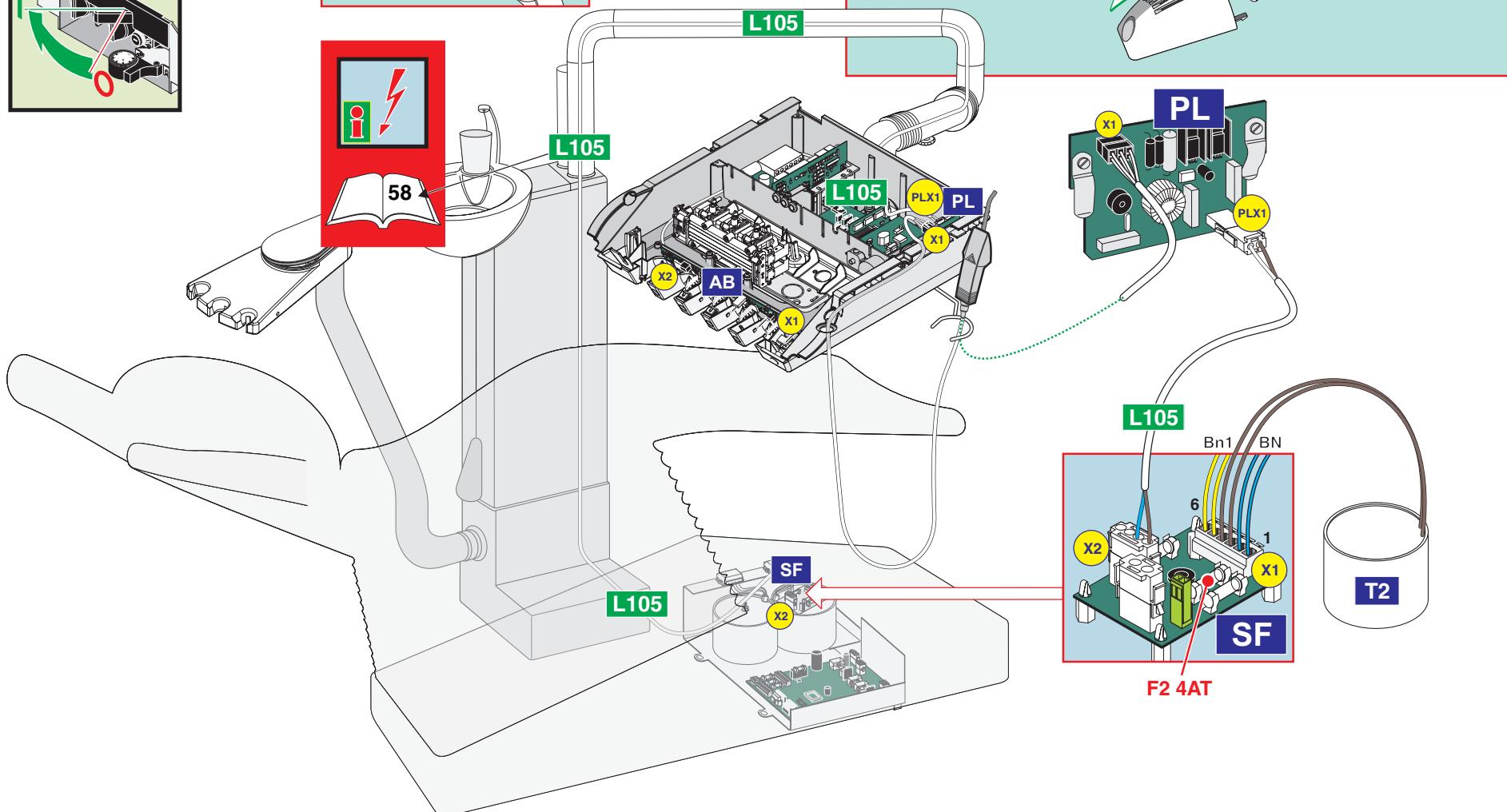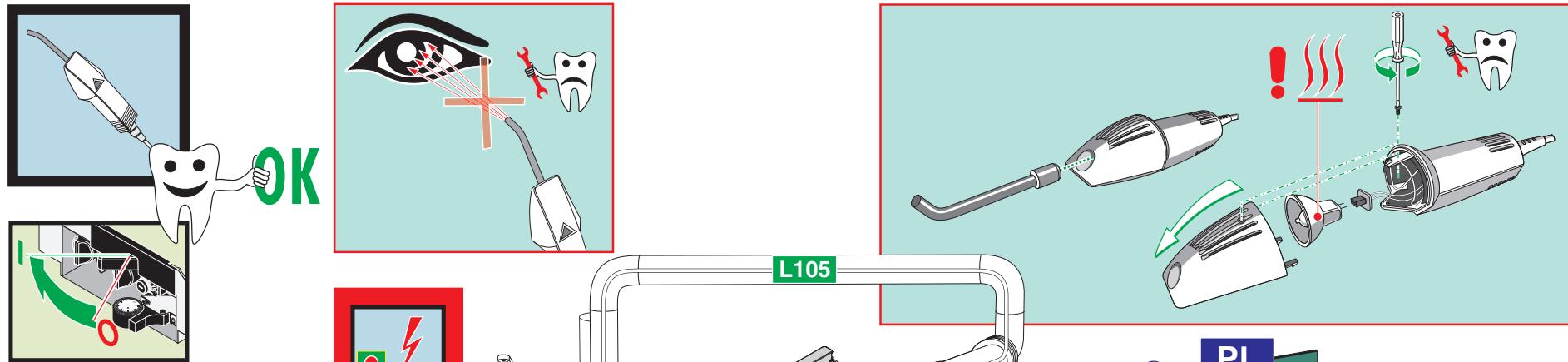

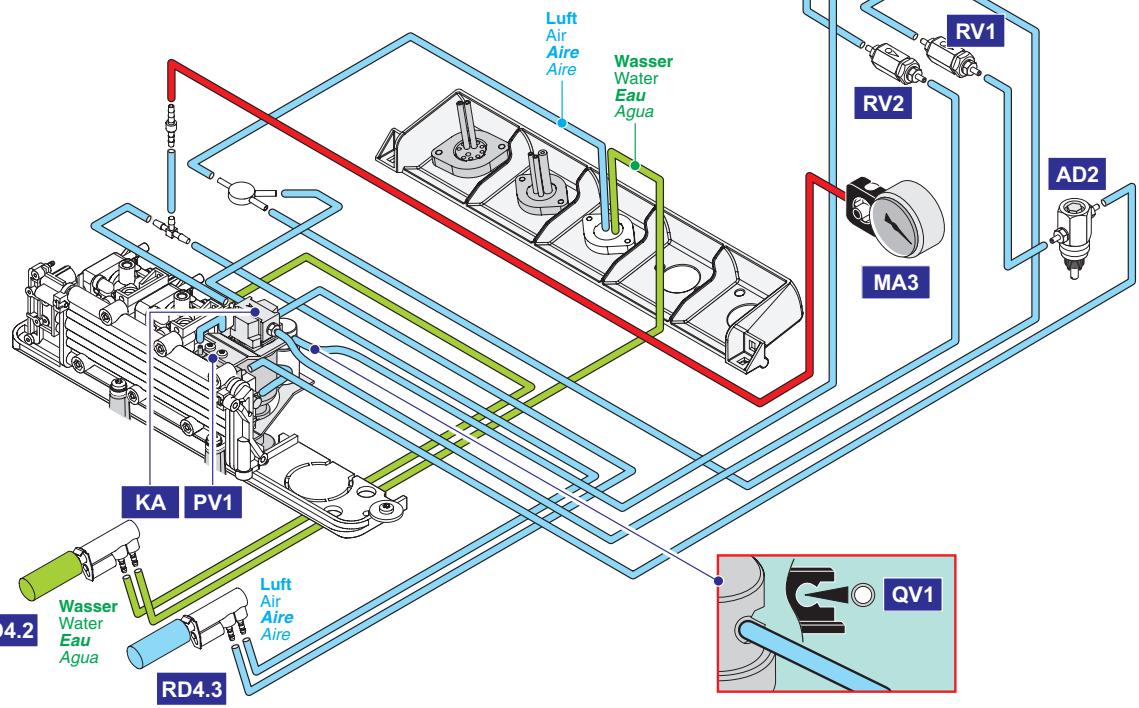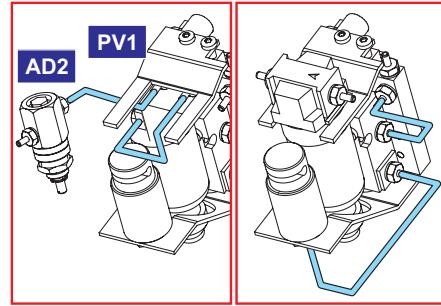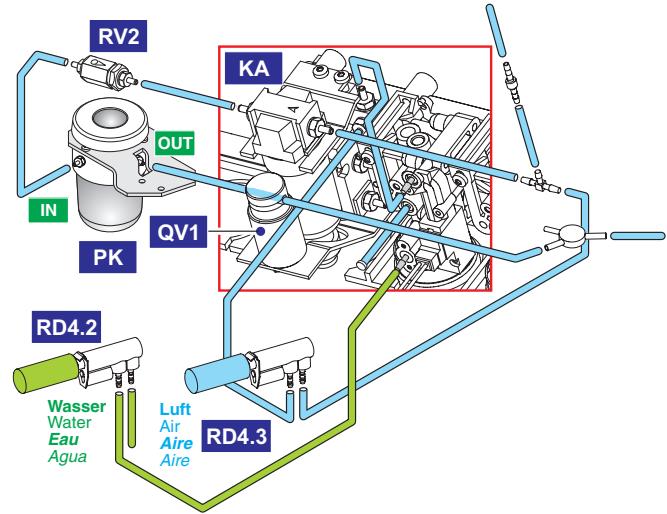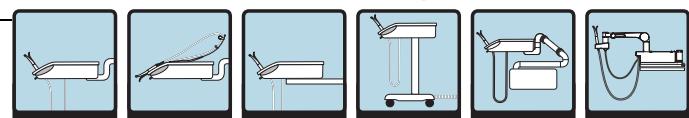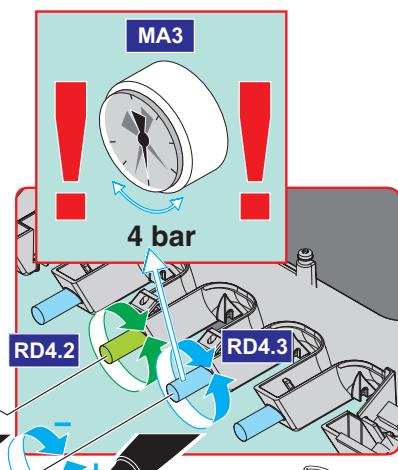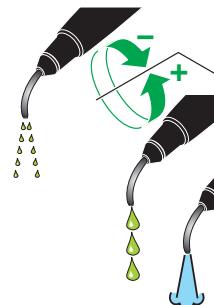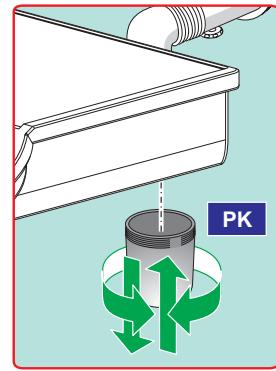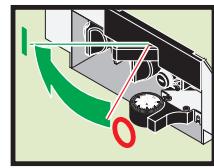

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

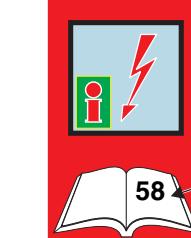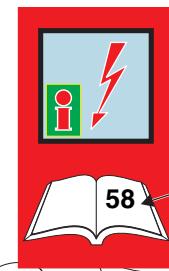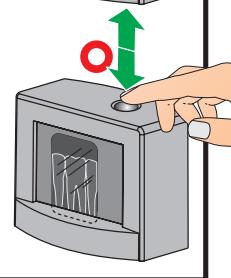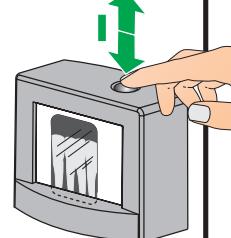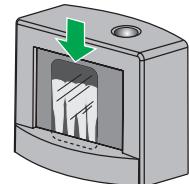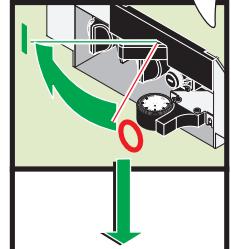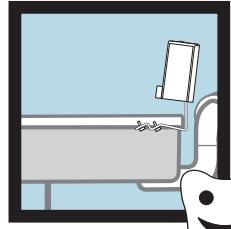

L102


L102

L102

X2

AB



X1




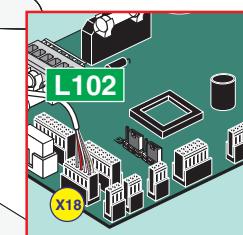








SB



L102









X1B


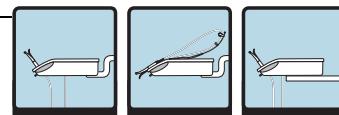
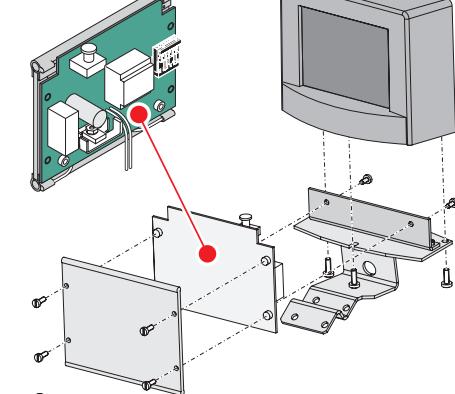


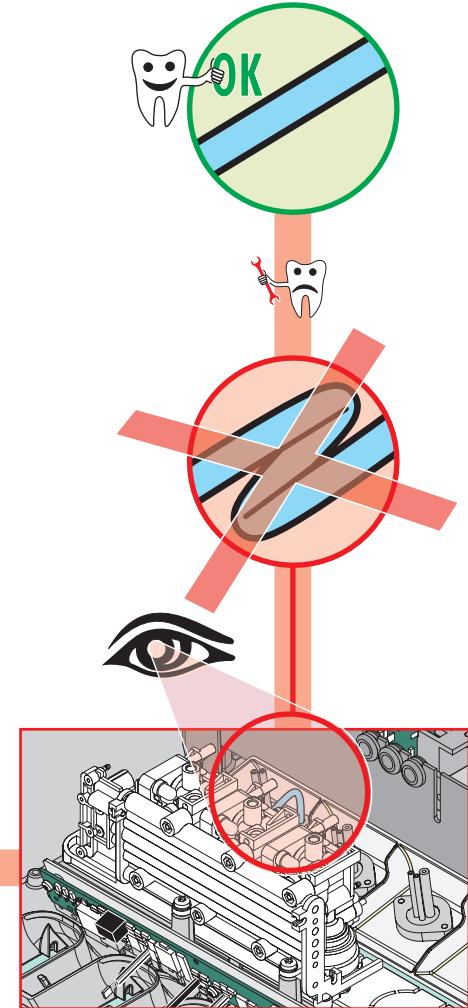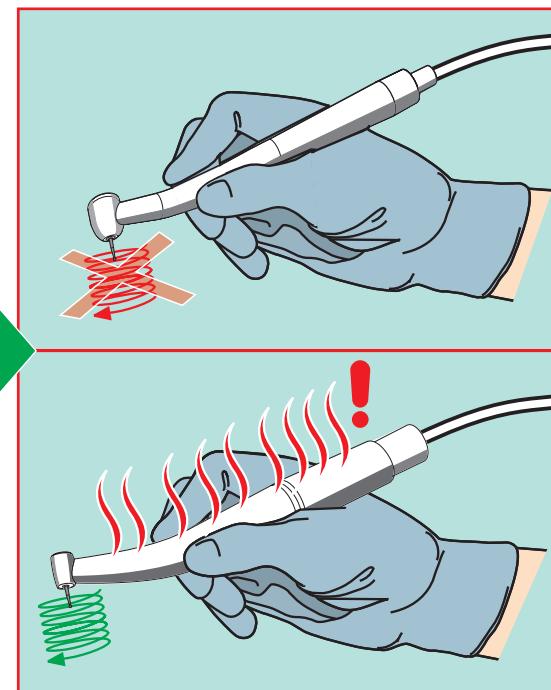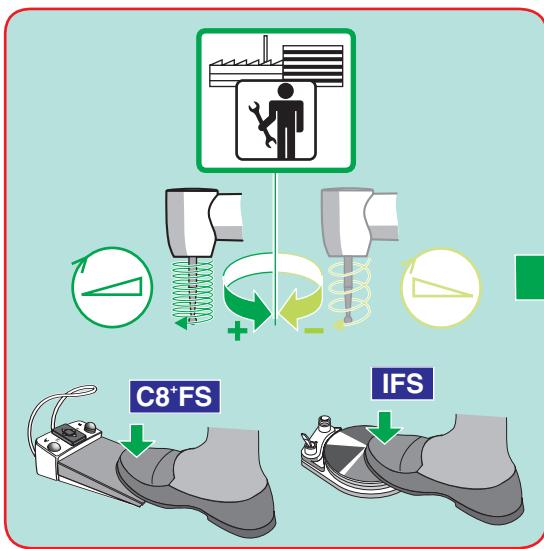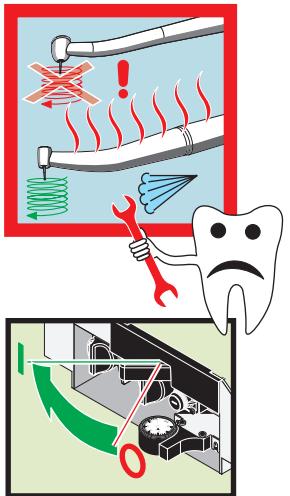

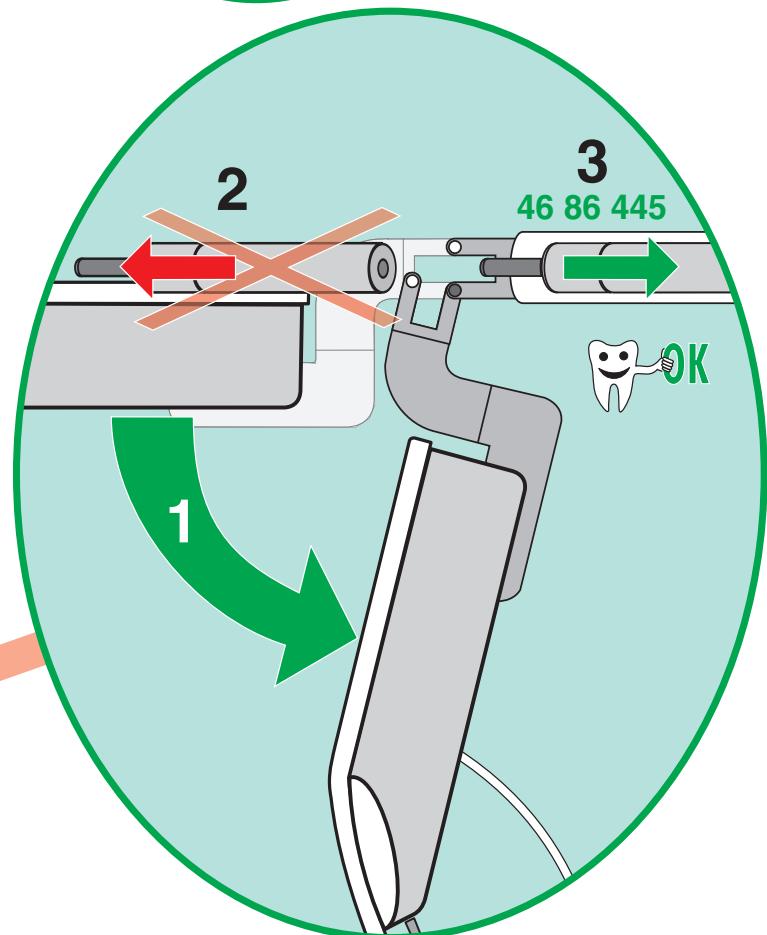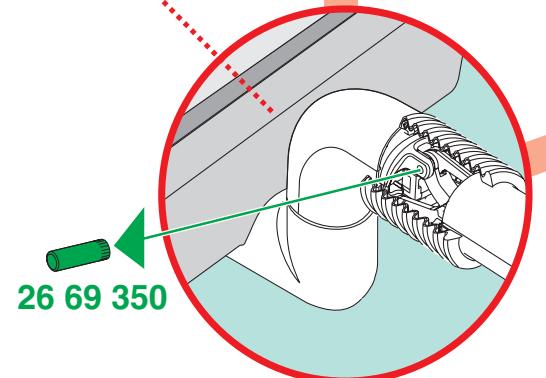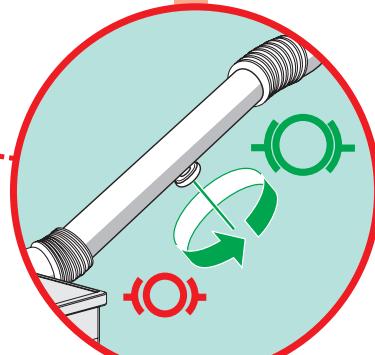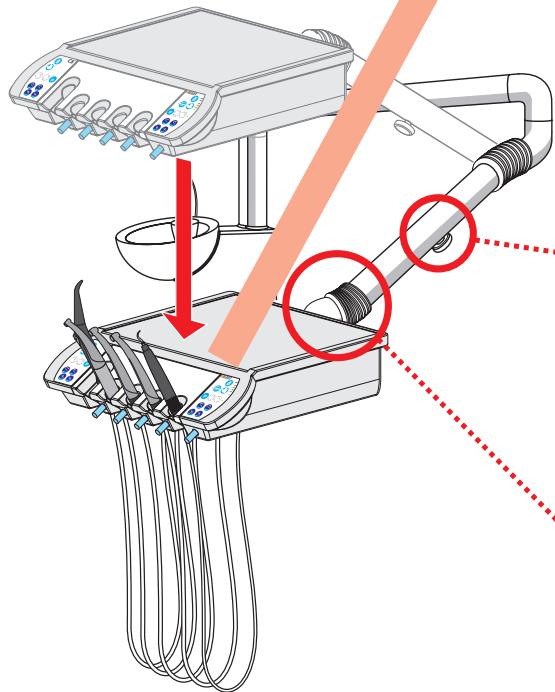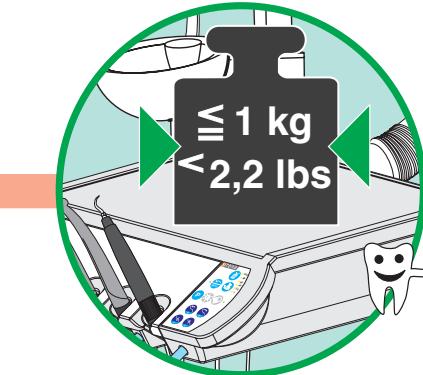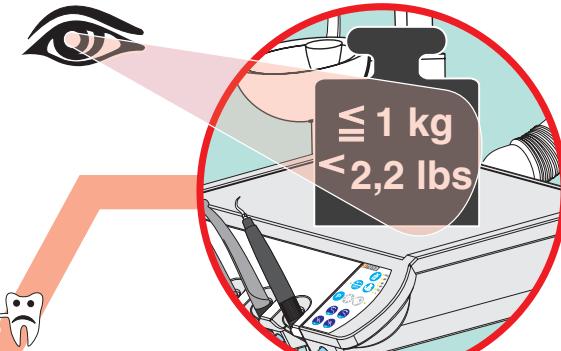


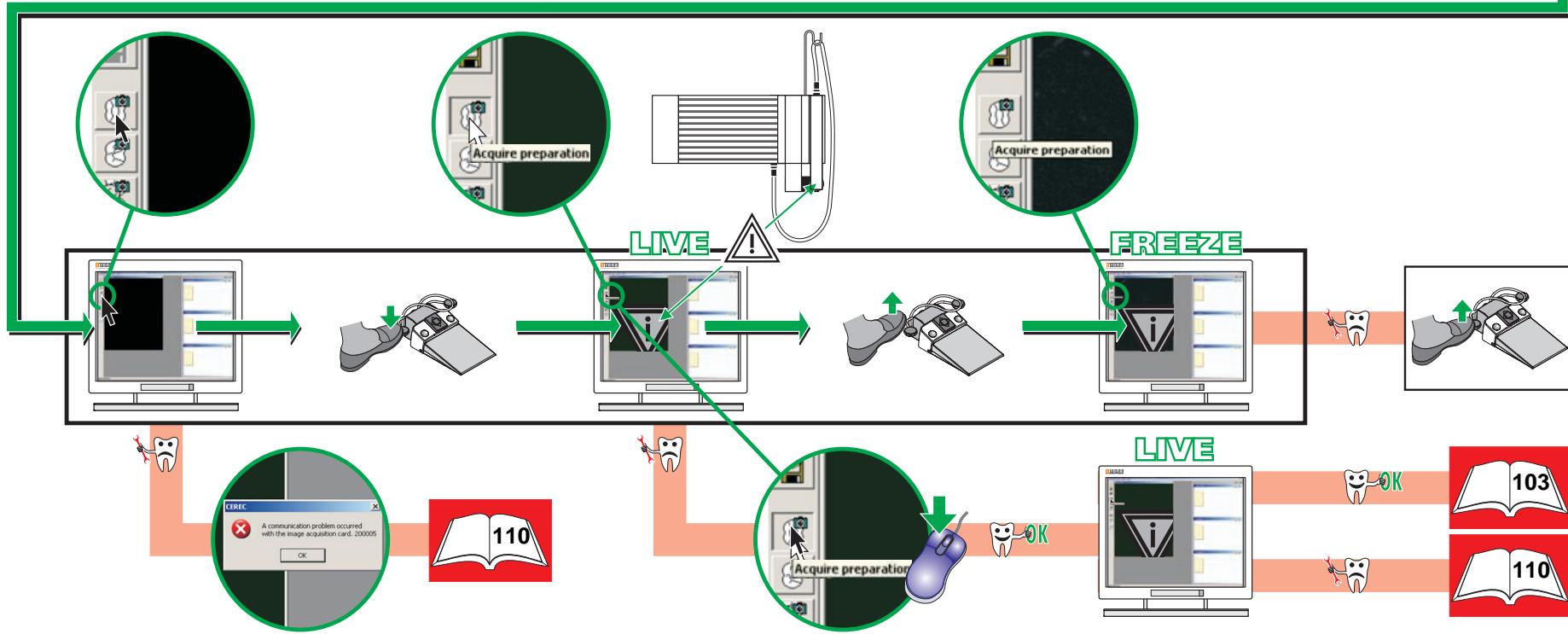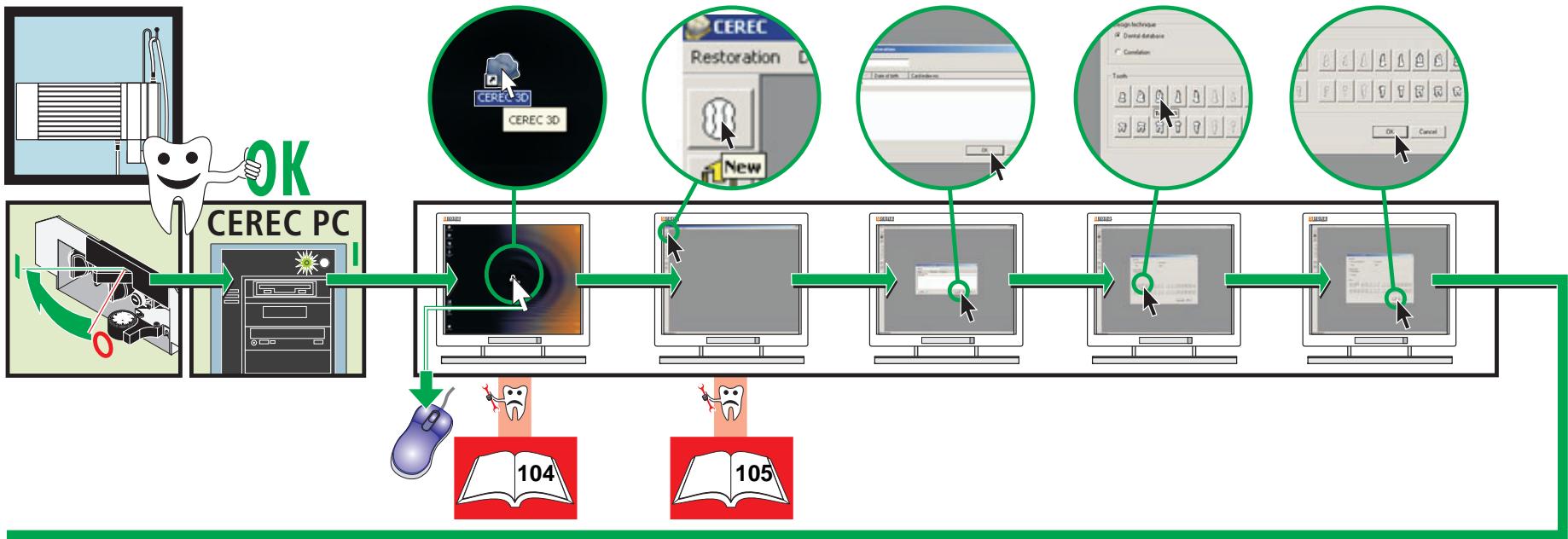

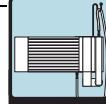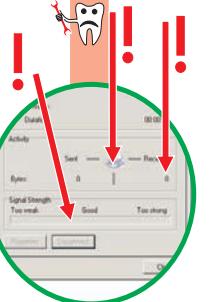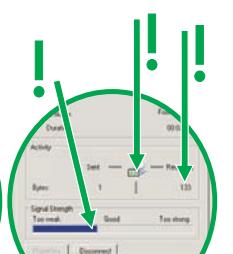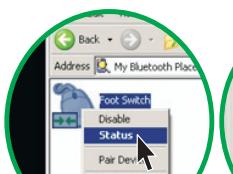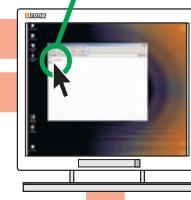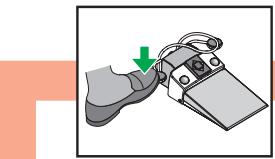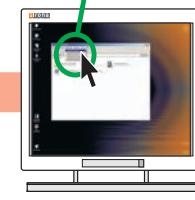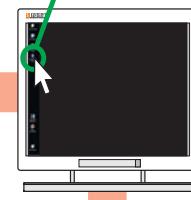


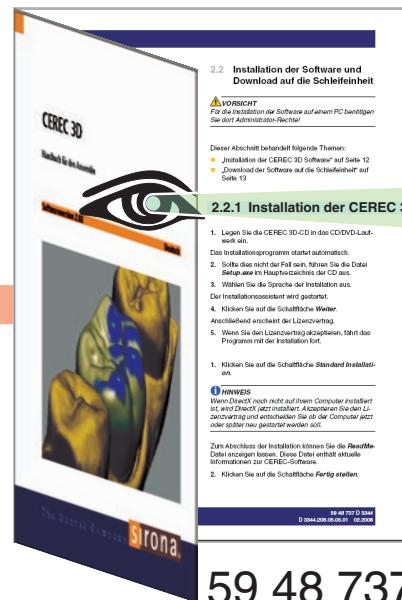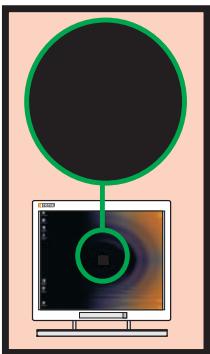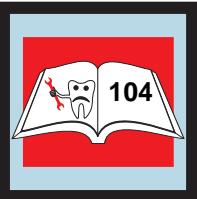


59 48 737

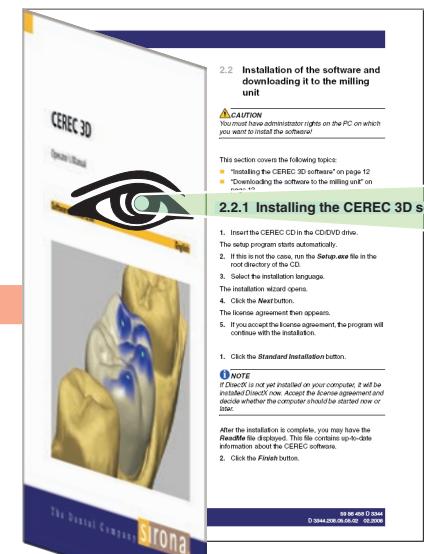


59 56 458

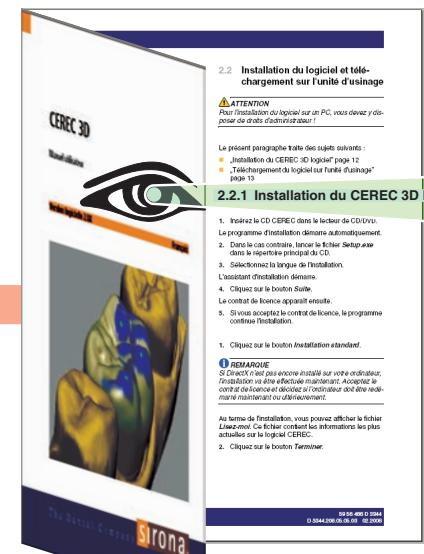




59 56 466

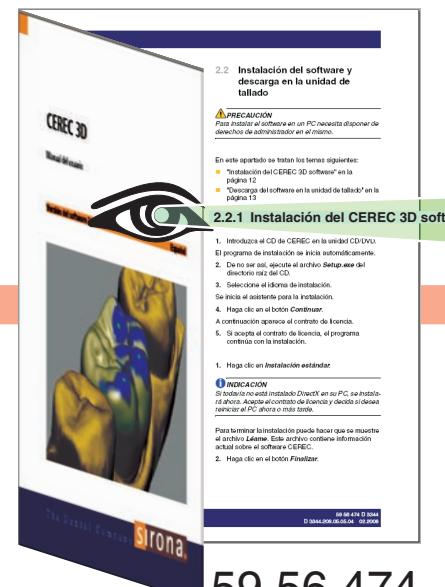


59 56 474

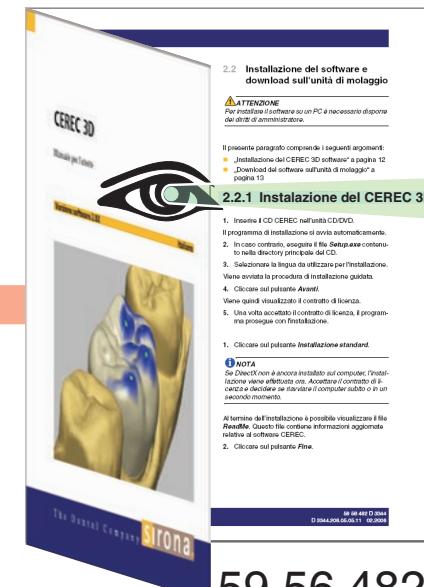


59 56 482

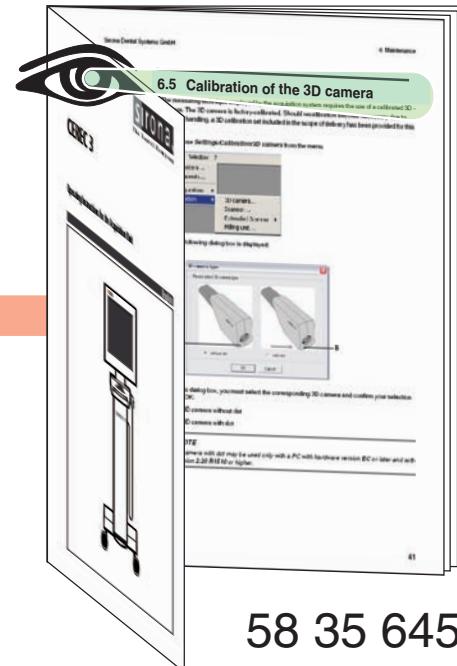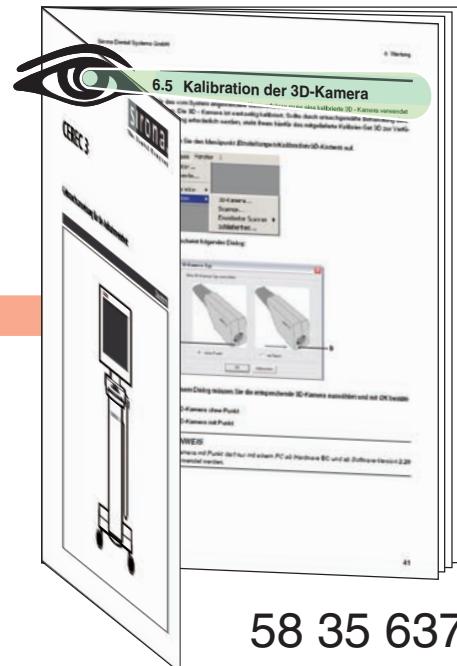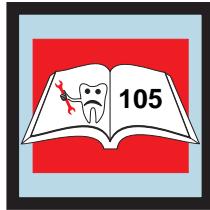


58 35 637

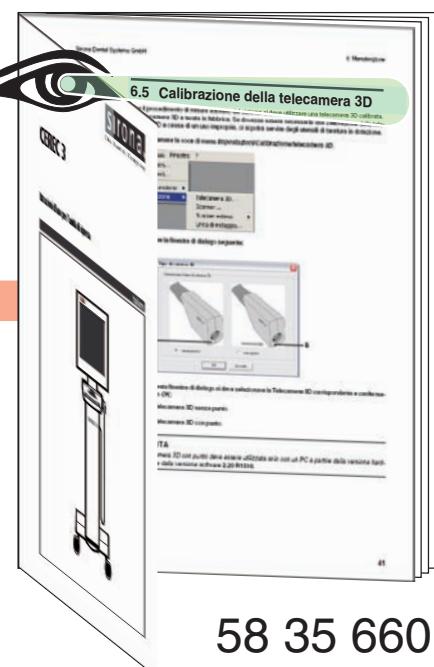


58 35 678

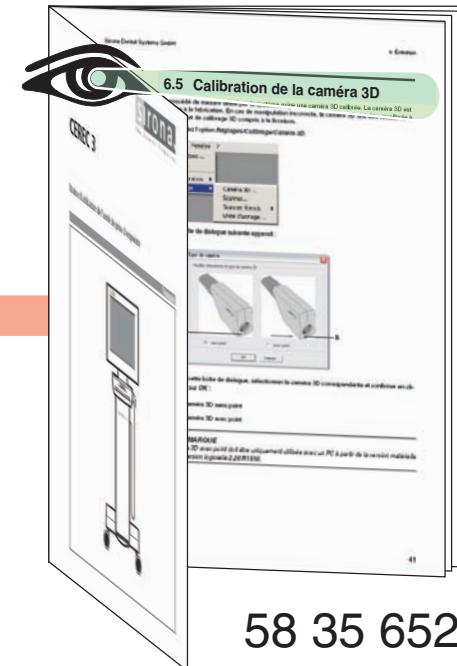


58 35 652


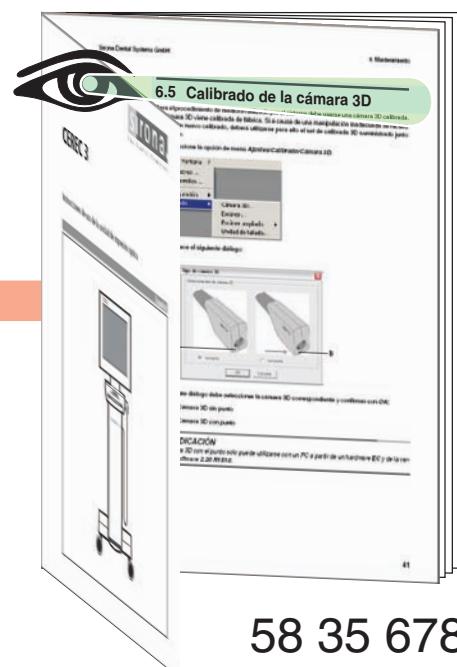





58 35 660

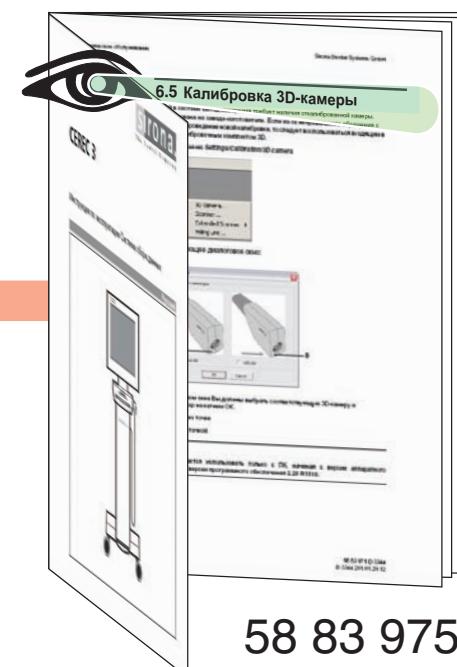




58 83 975

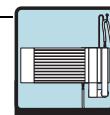

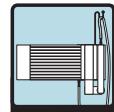
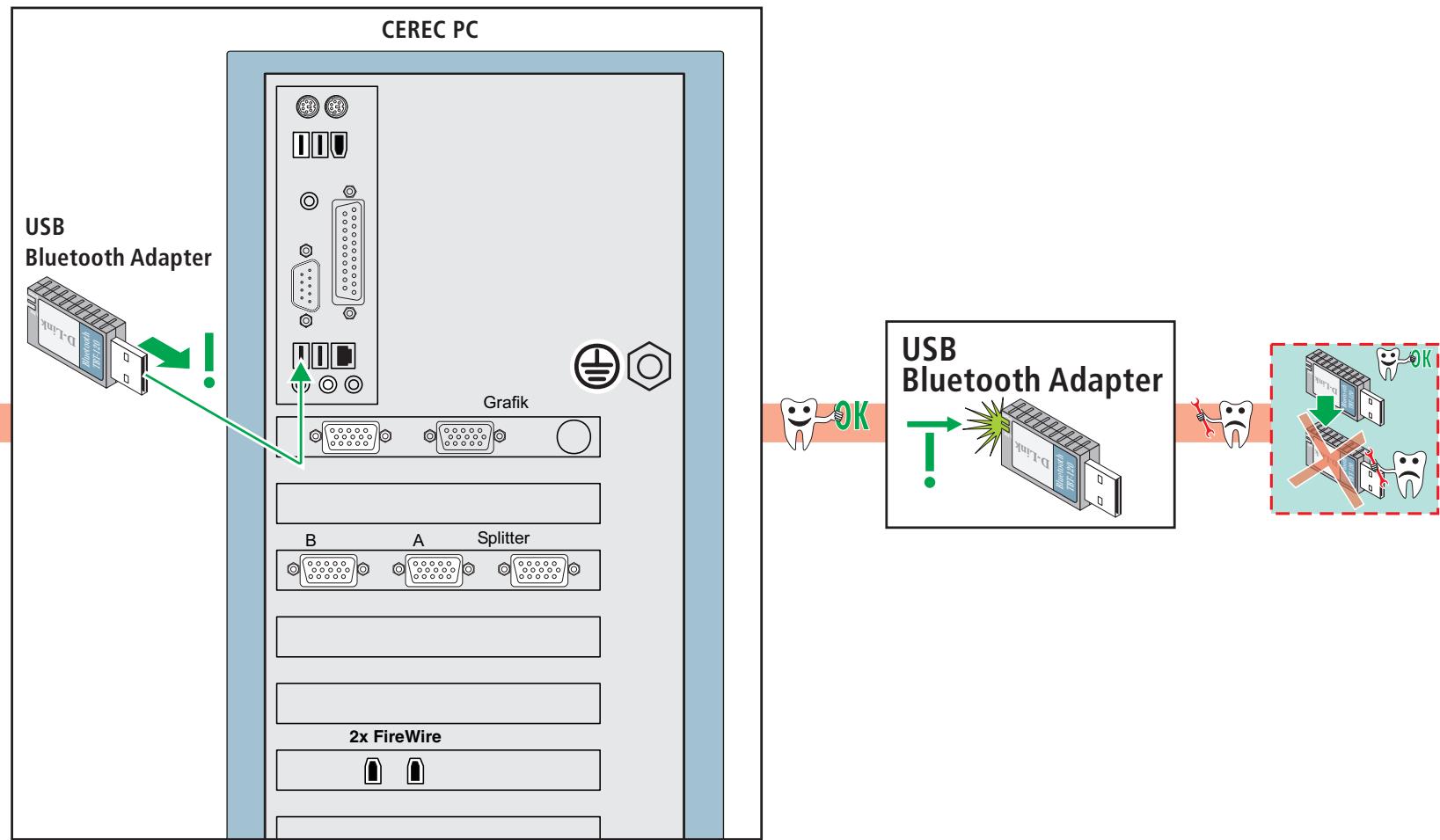


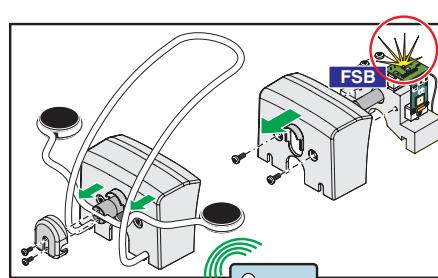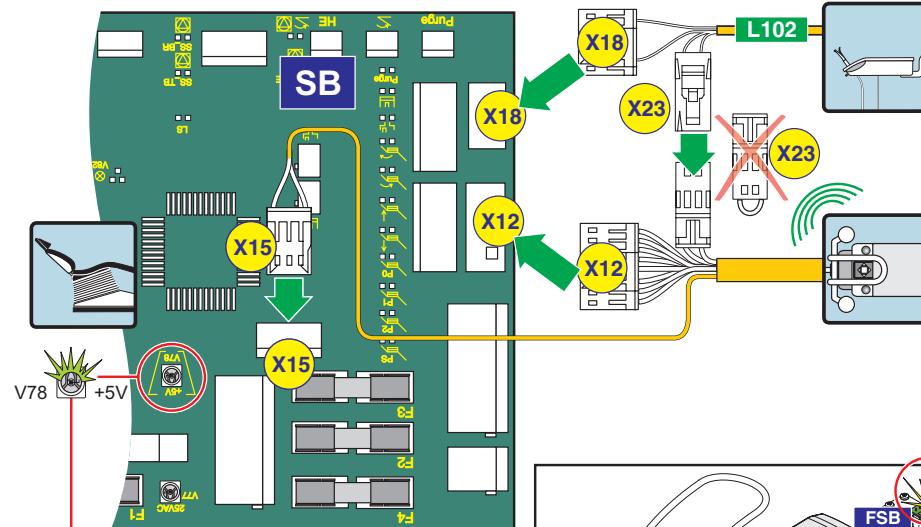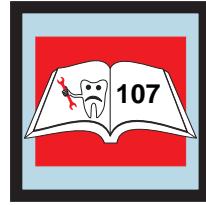


V1 FS_activ

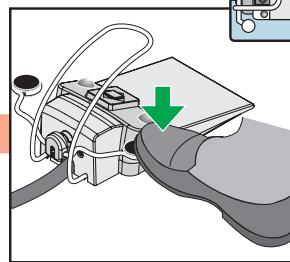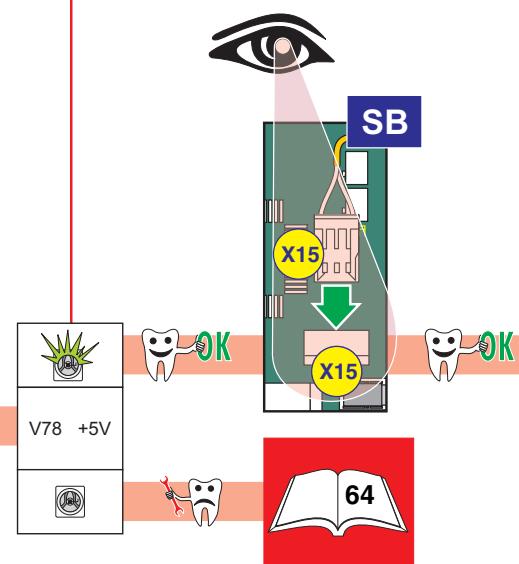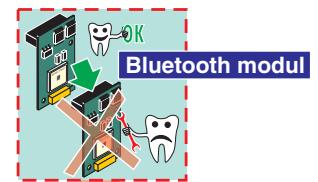


V1 FS_activ

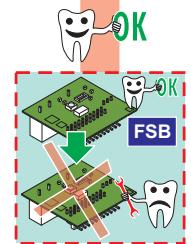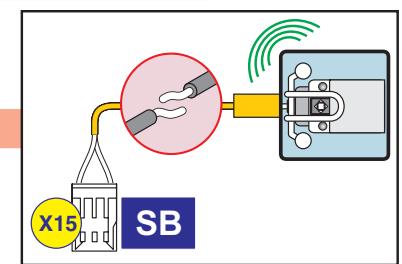


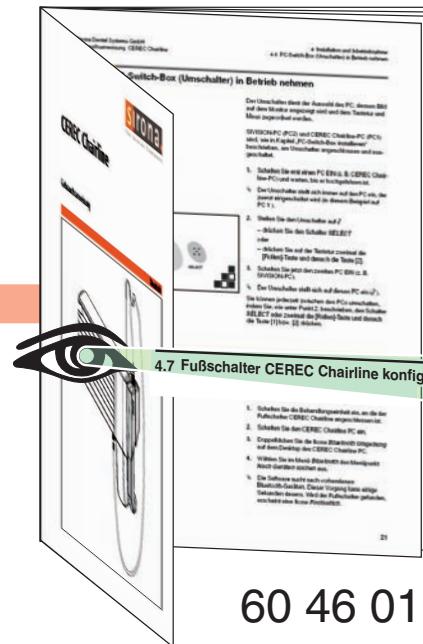


60 46 010

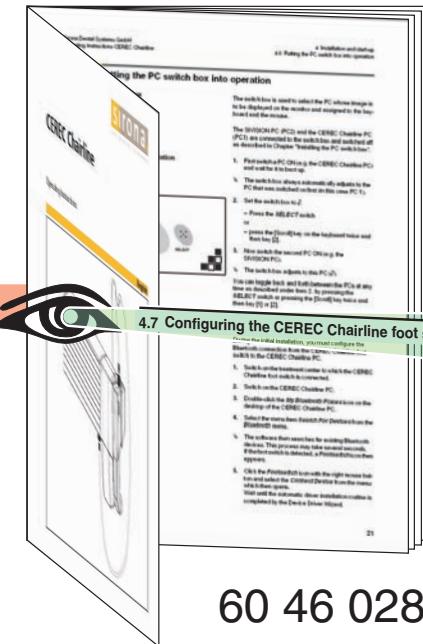


60 46 028

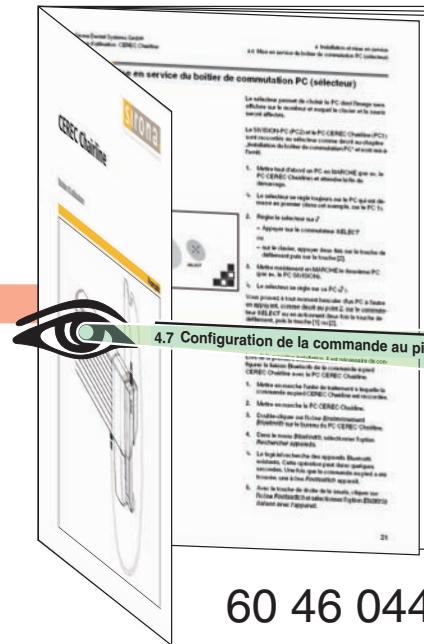


60 46 044


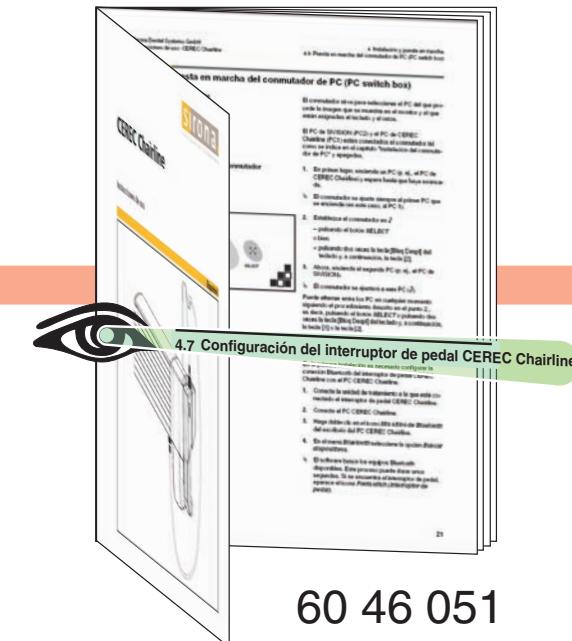





60 46 051








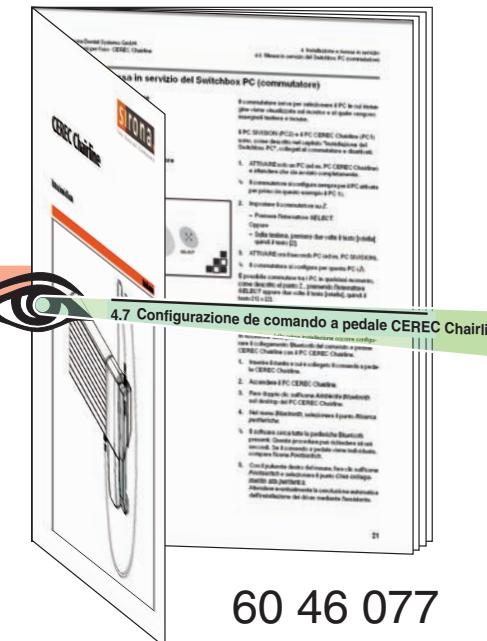


60 46 077

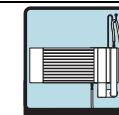

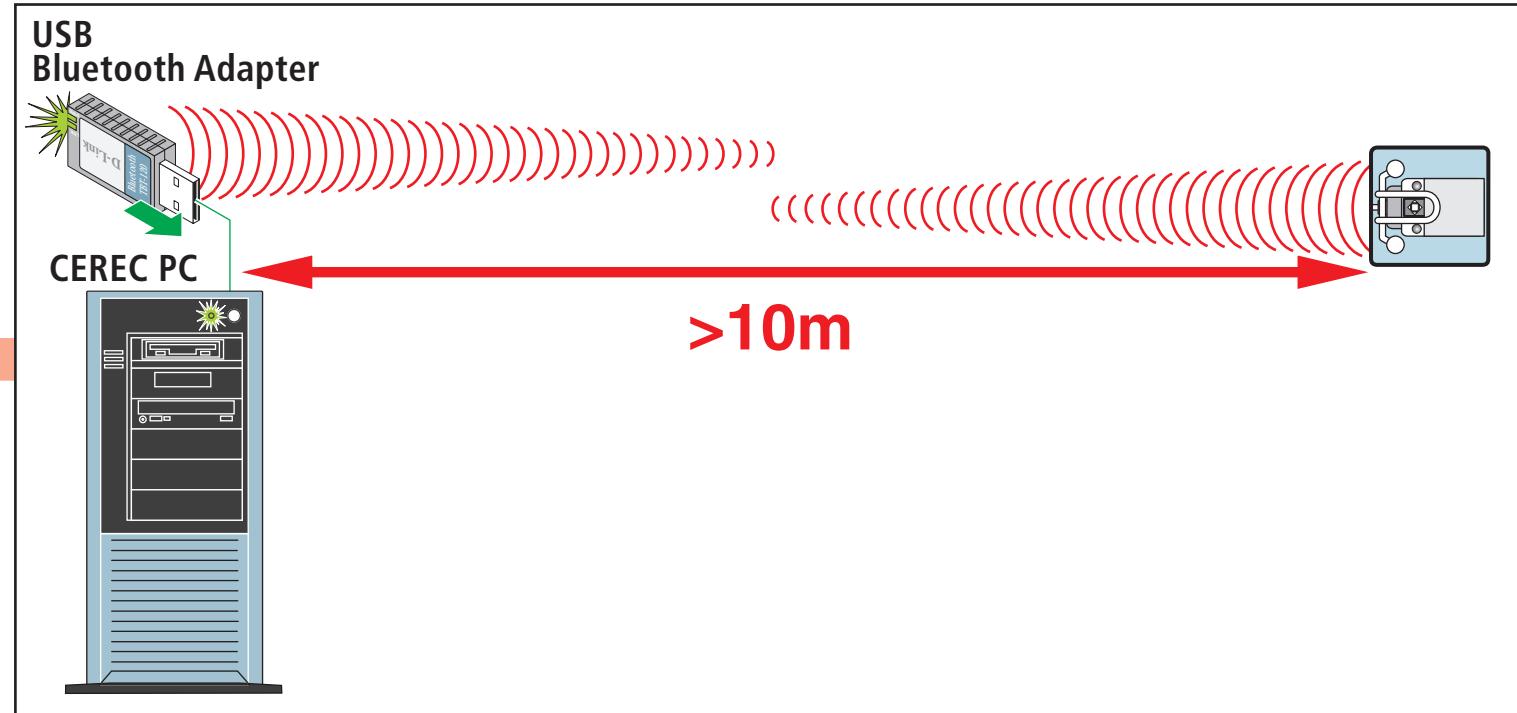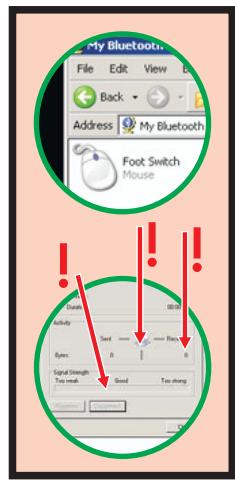

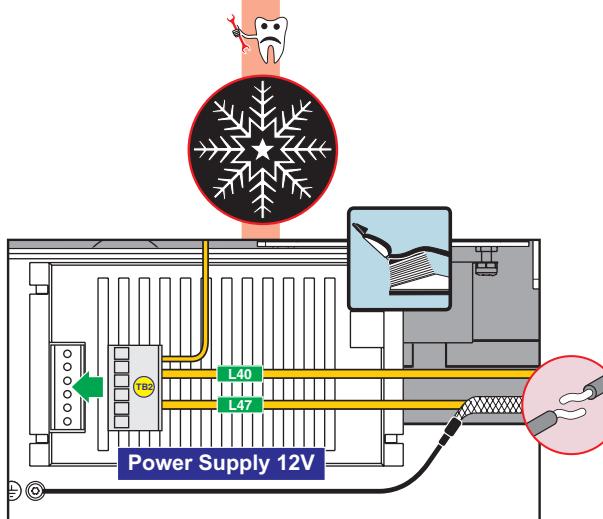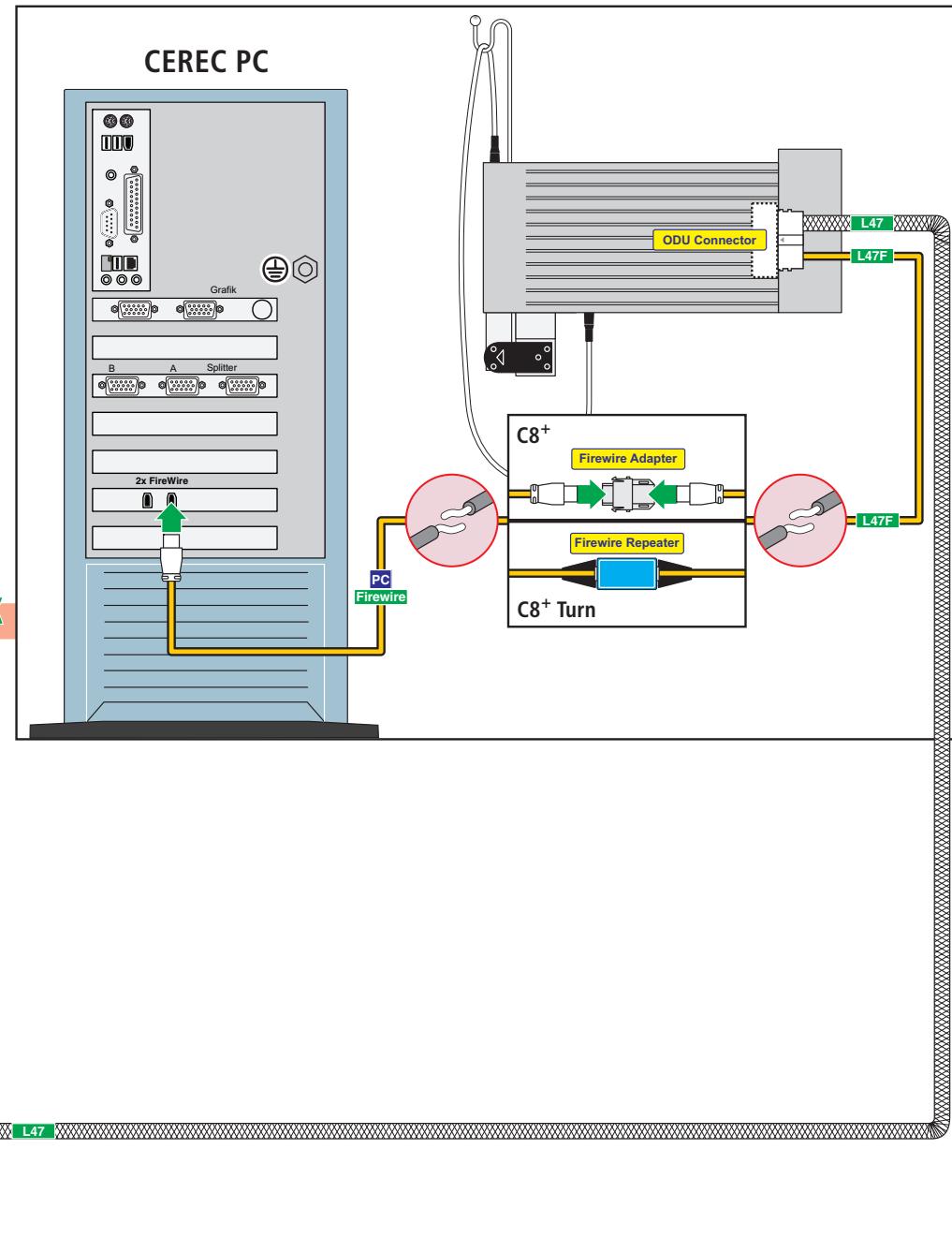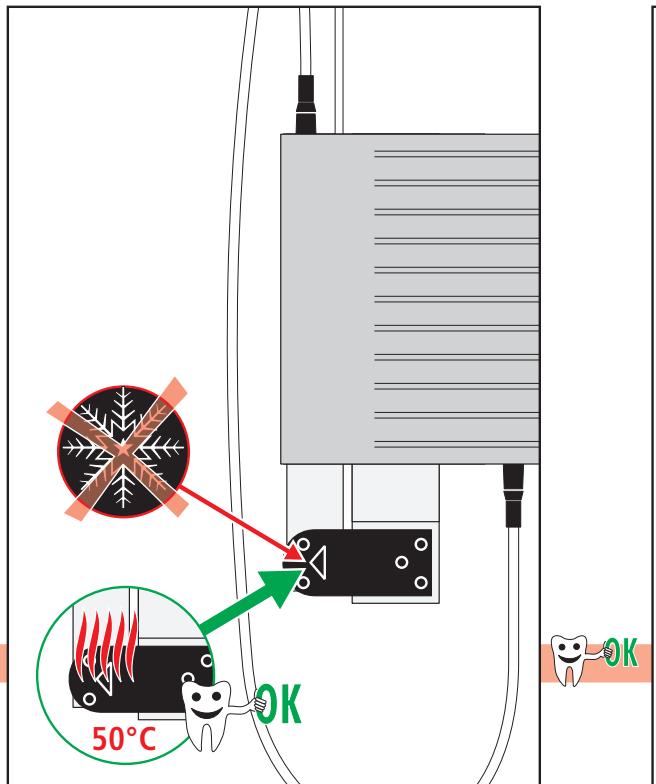


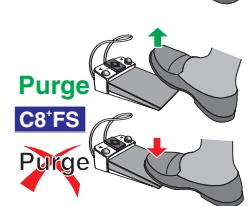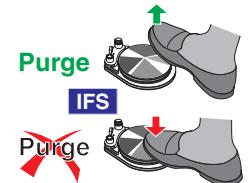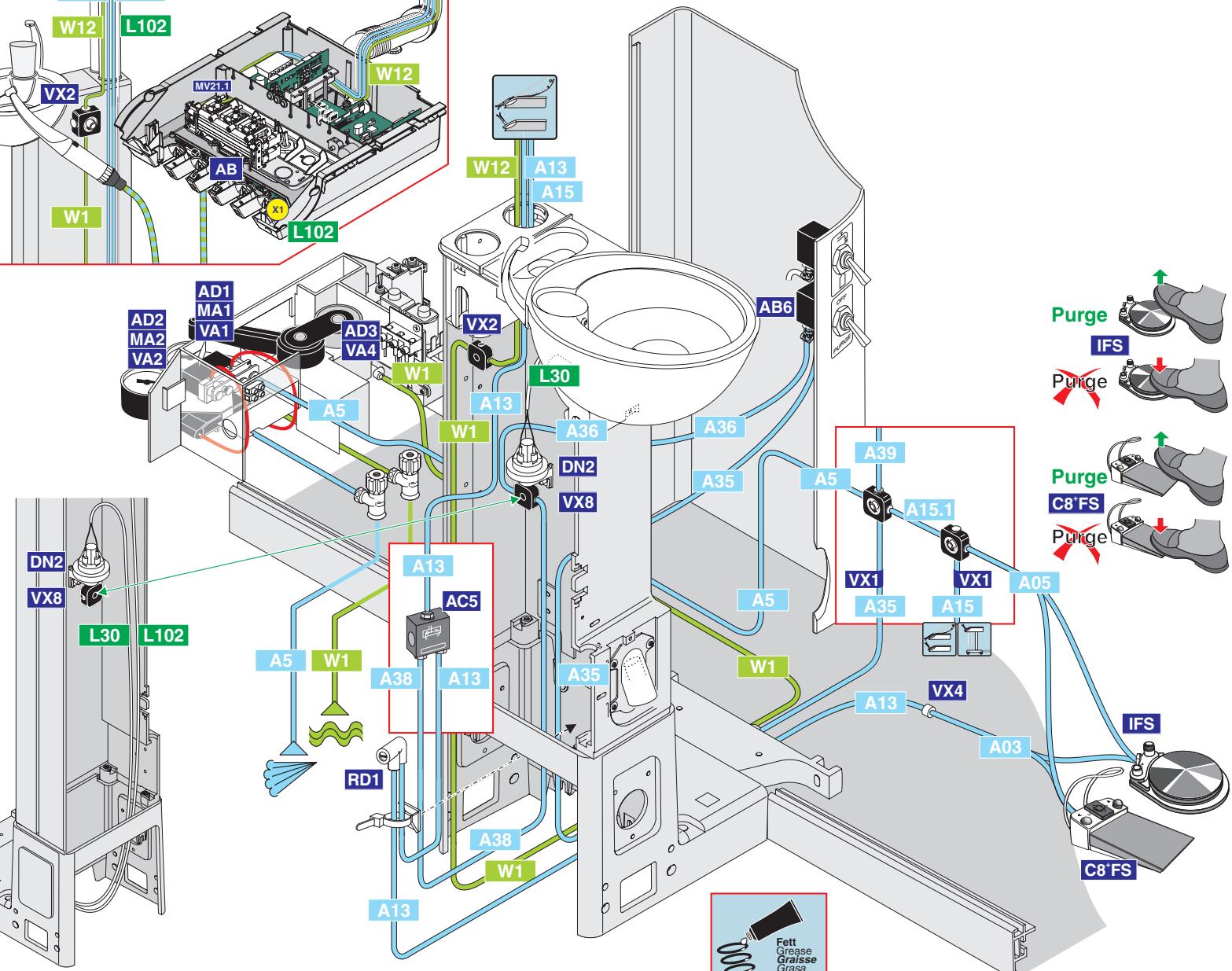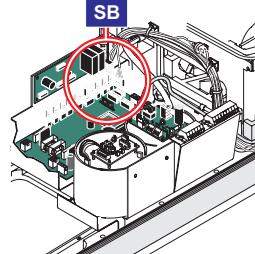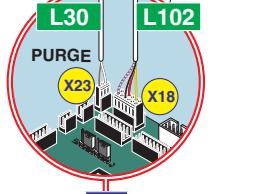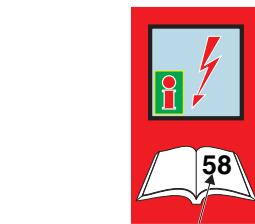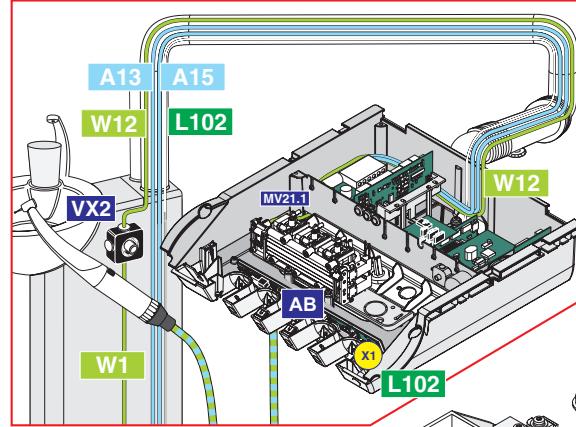
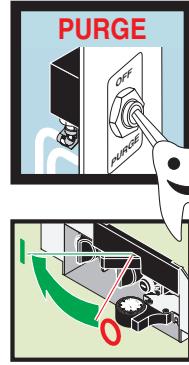



Purge

C8'FS

Purge

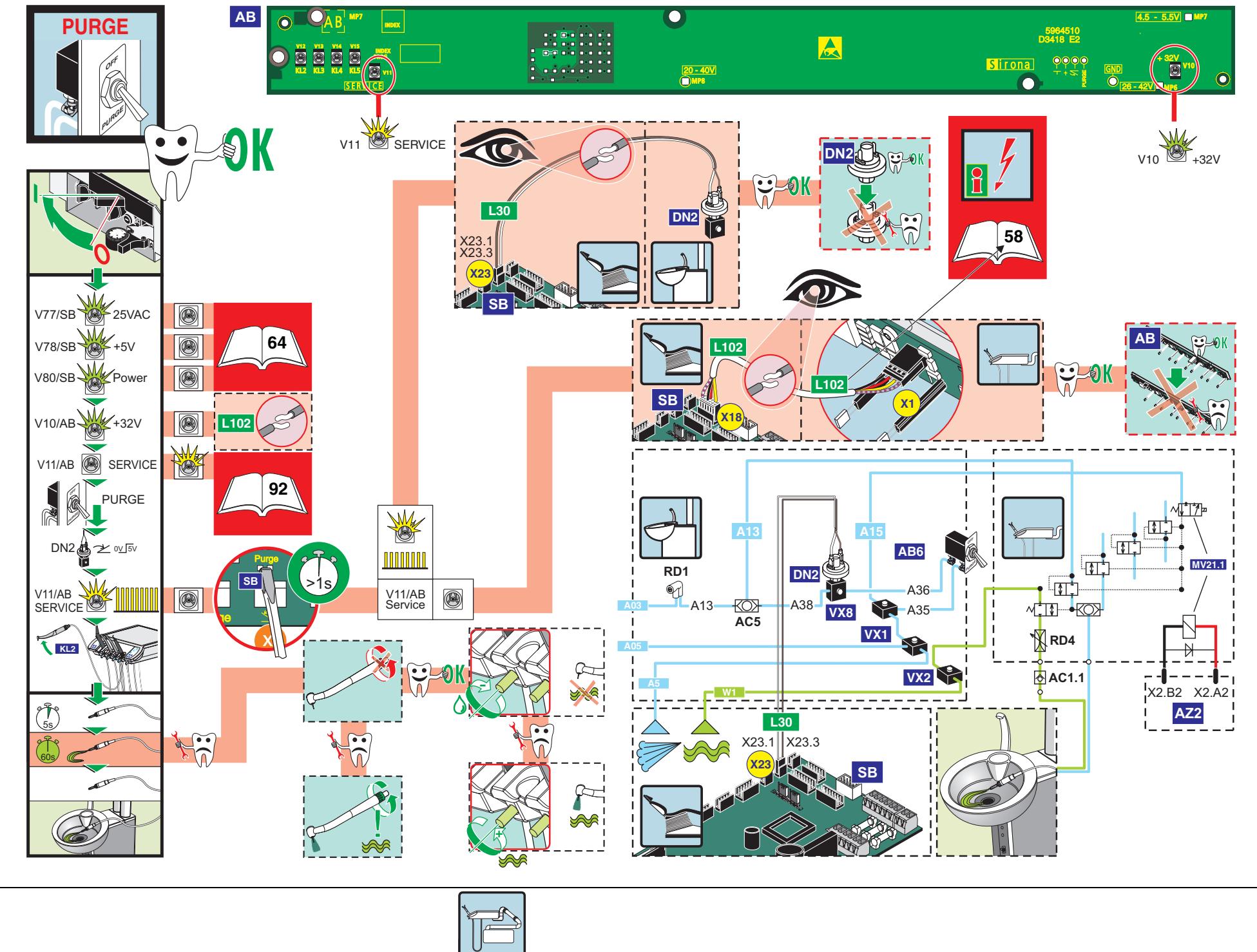

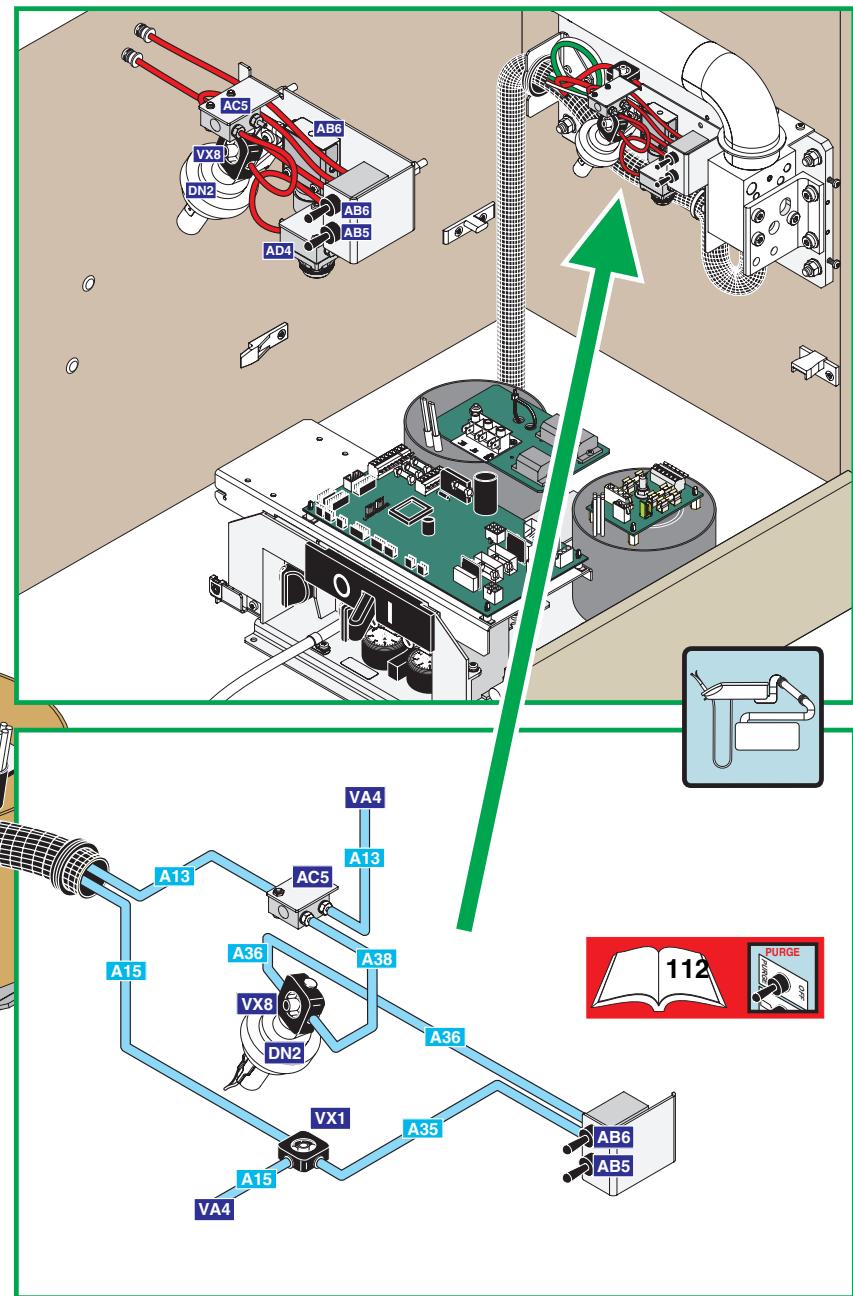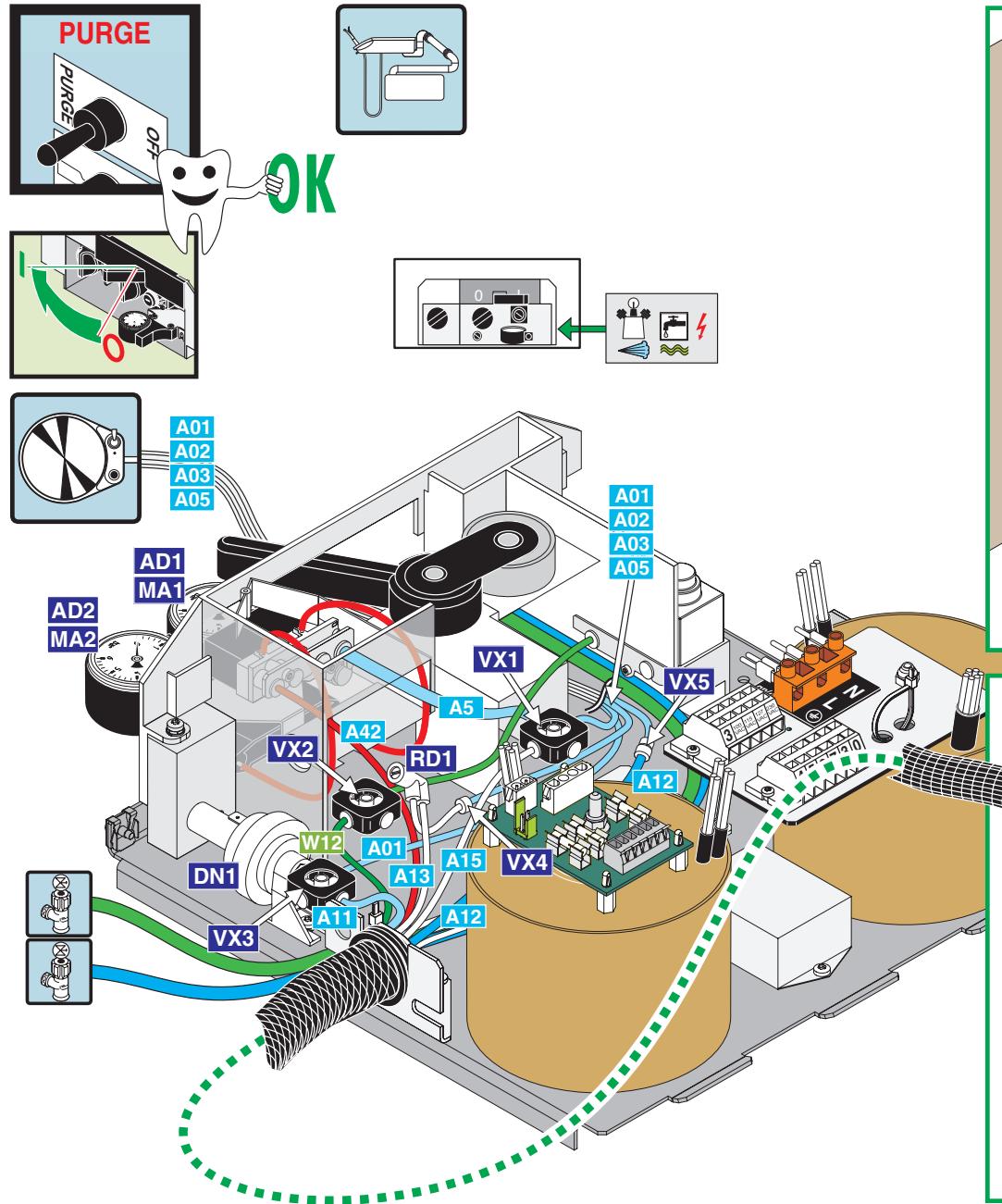

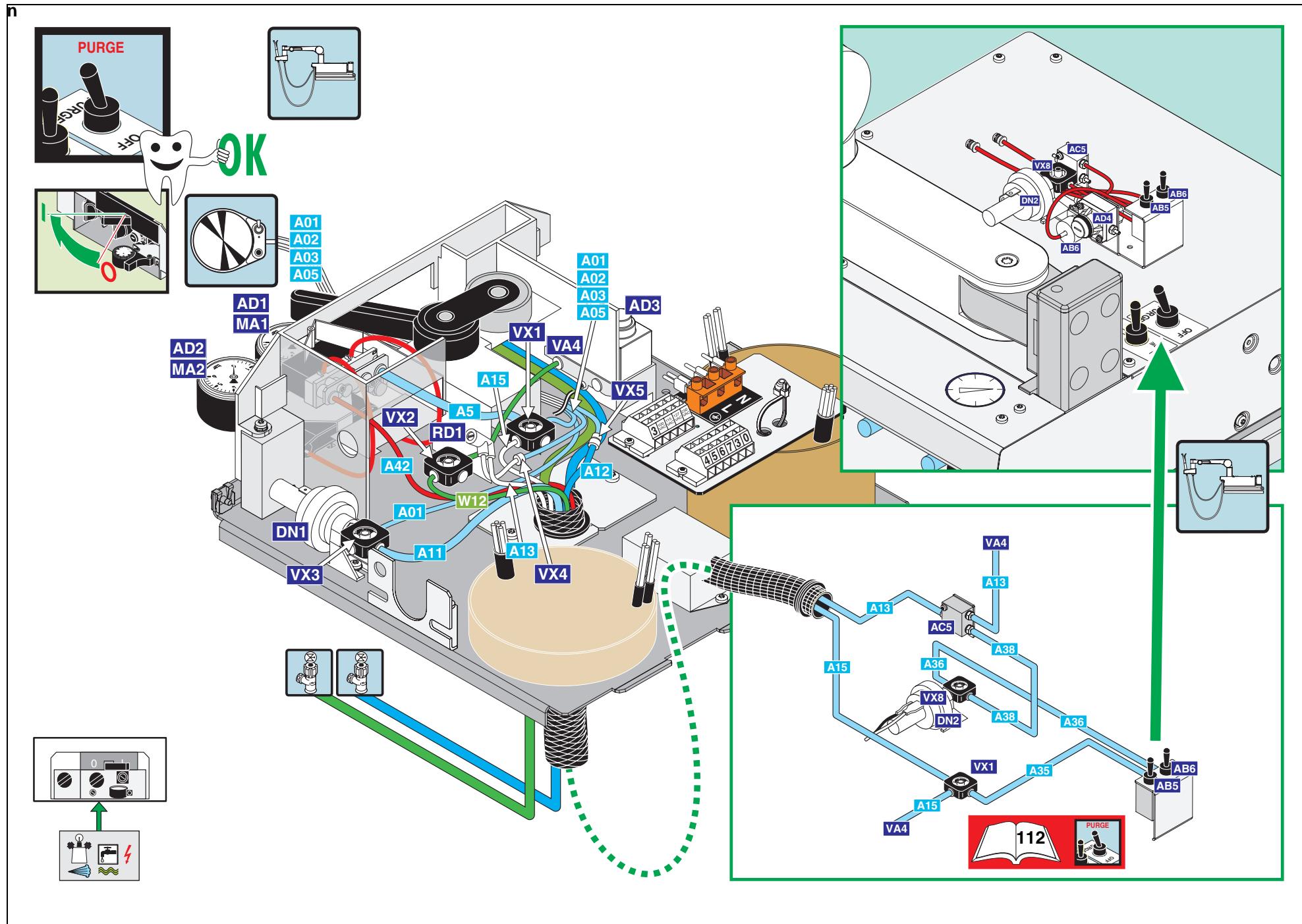





L29

U.S. only




K4




OK

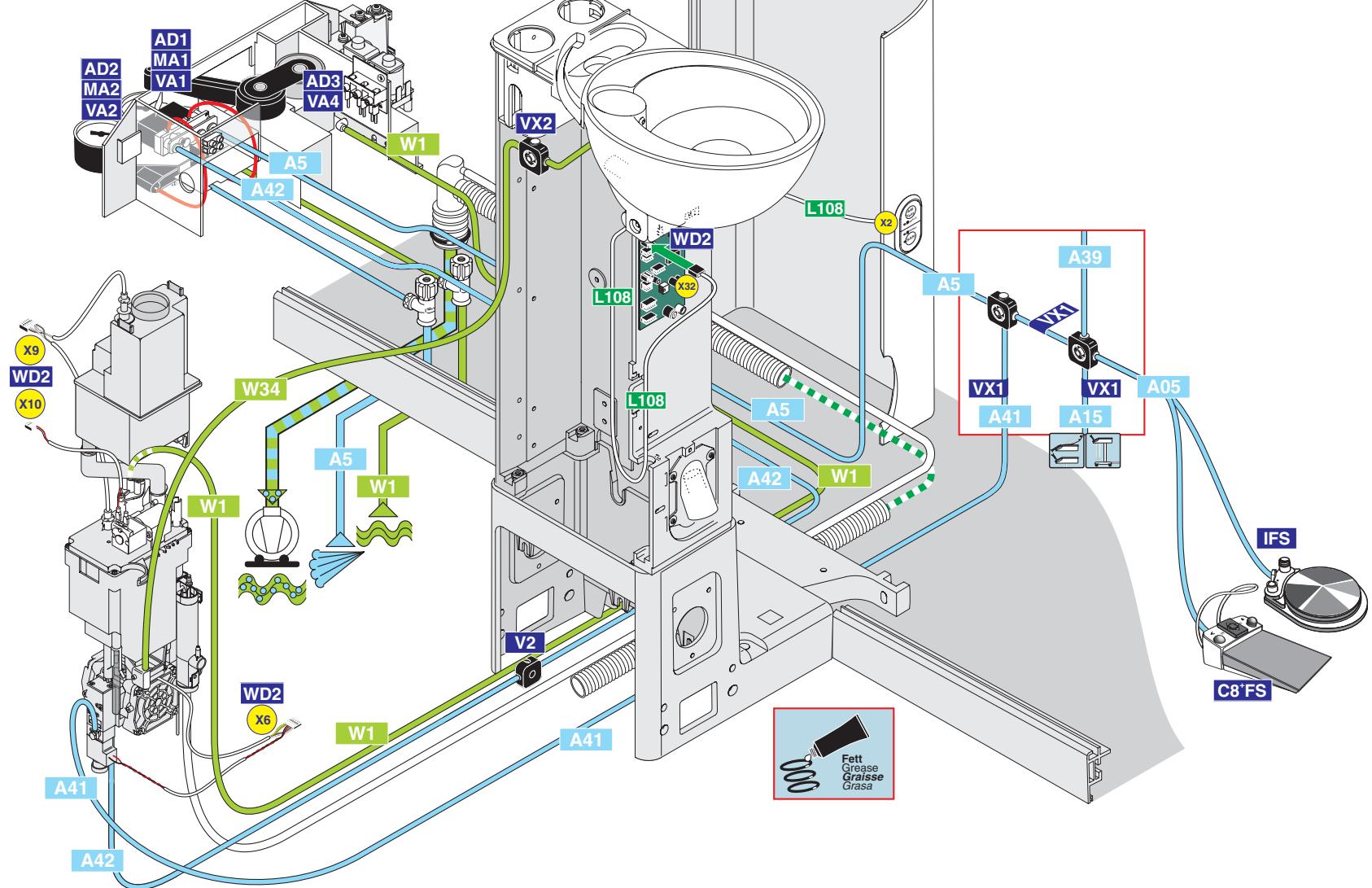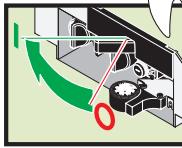








OK

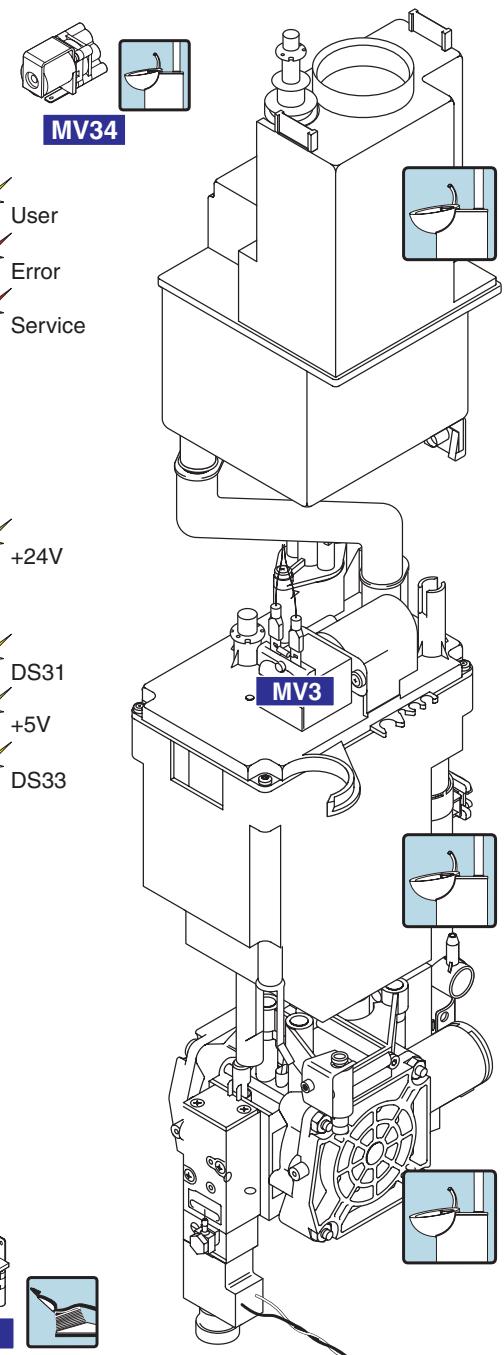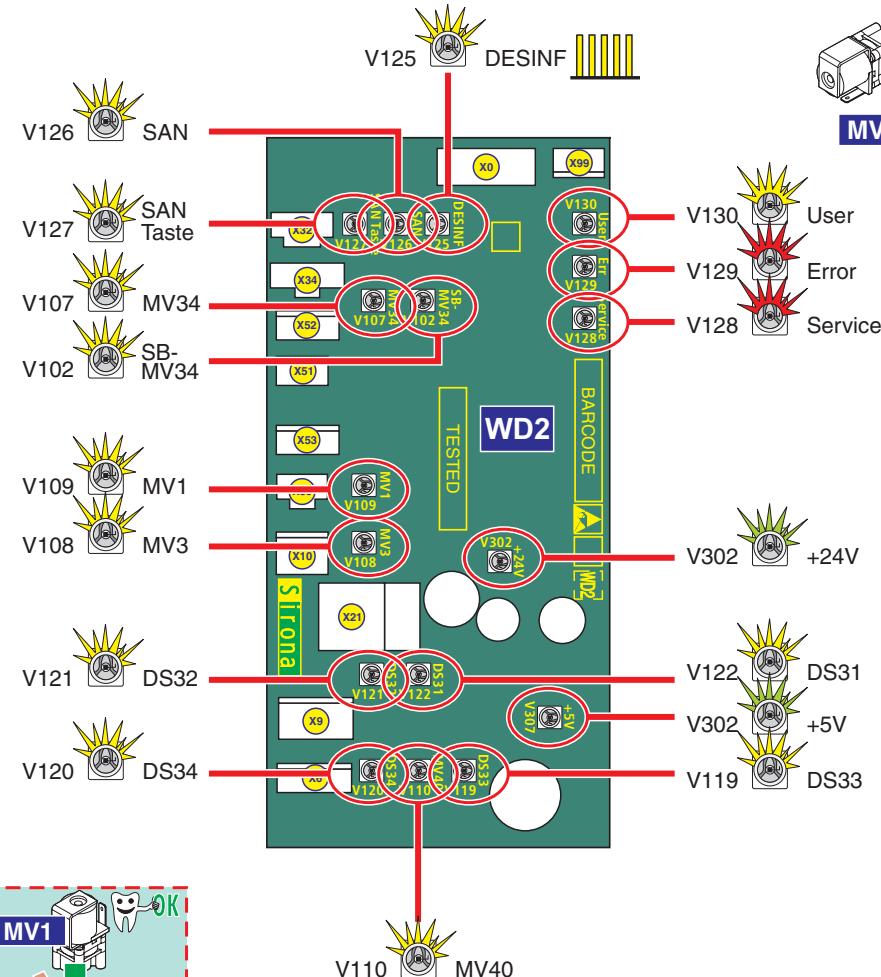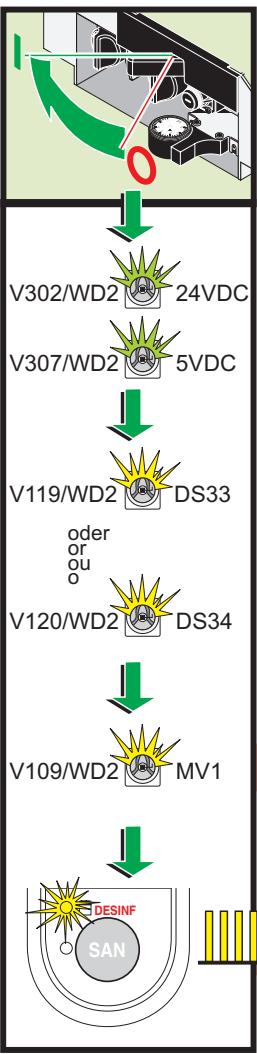




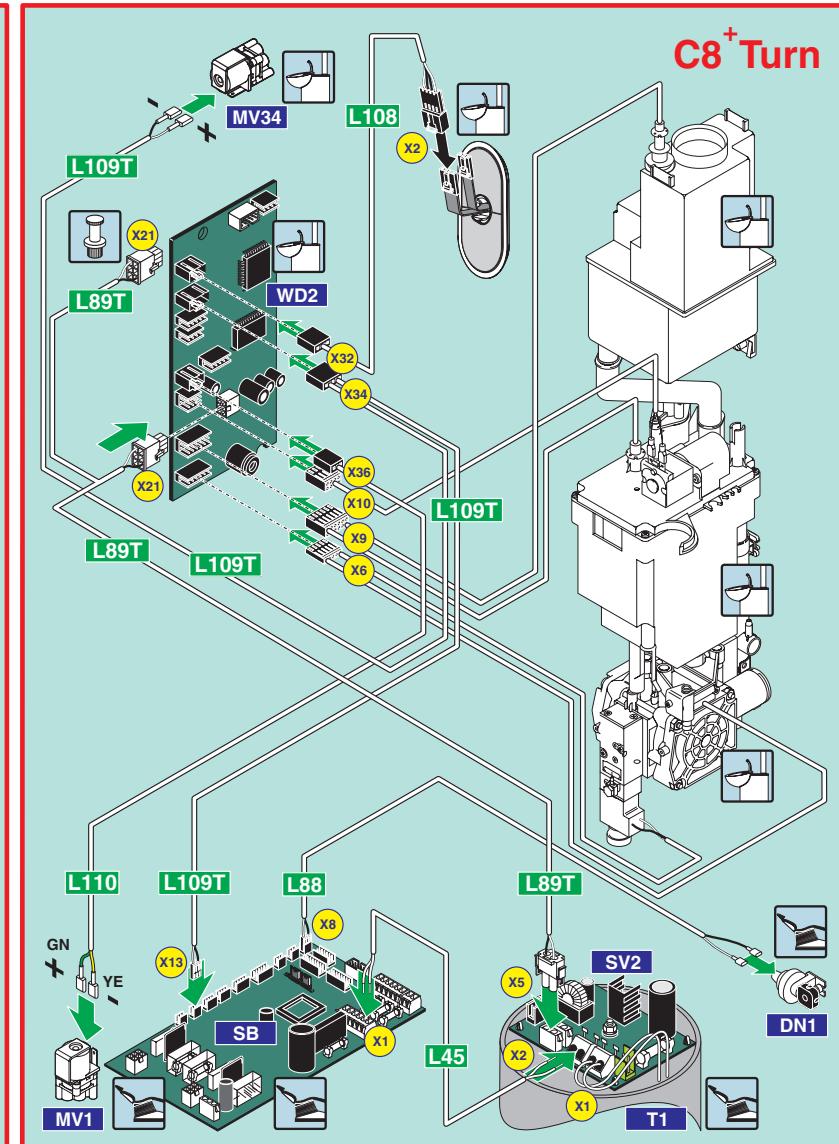
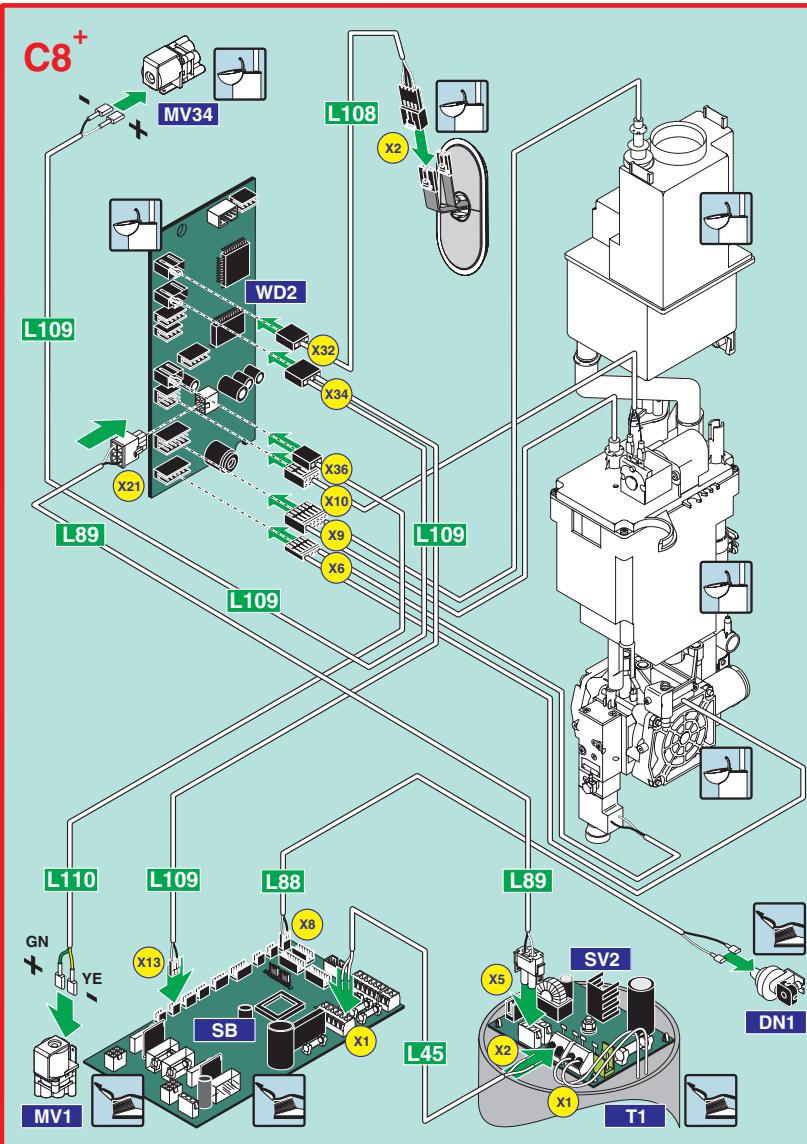
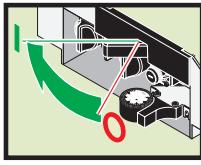


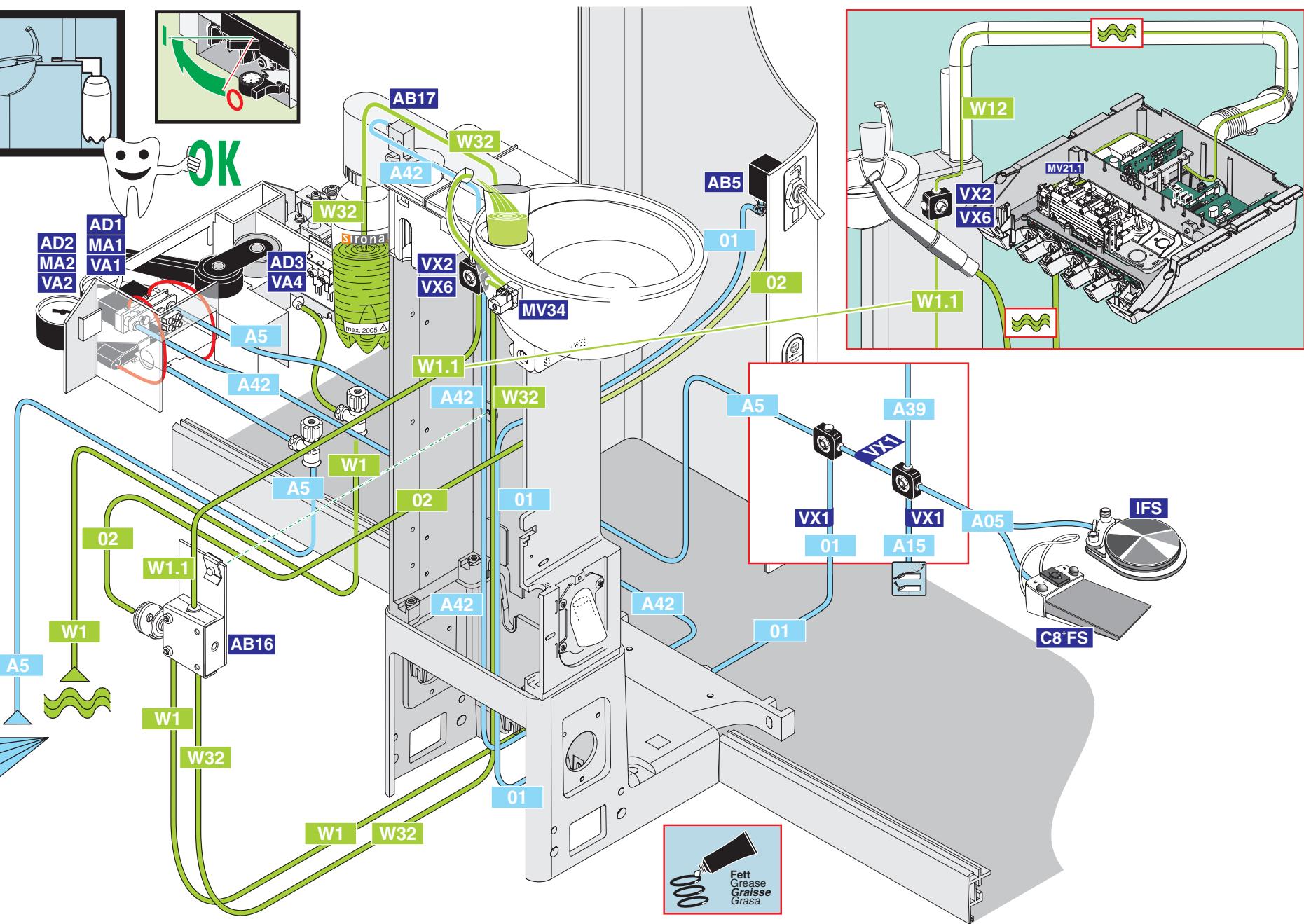


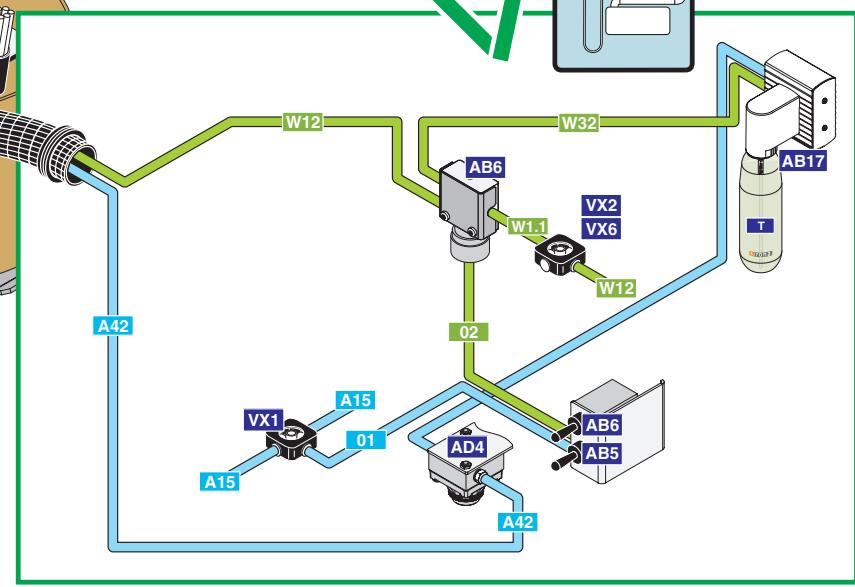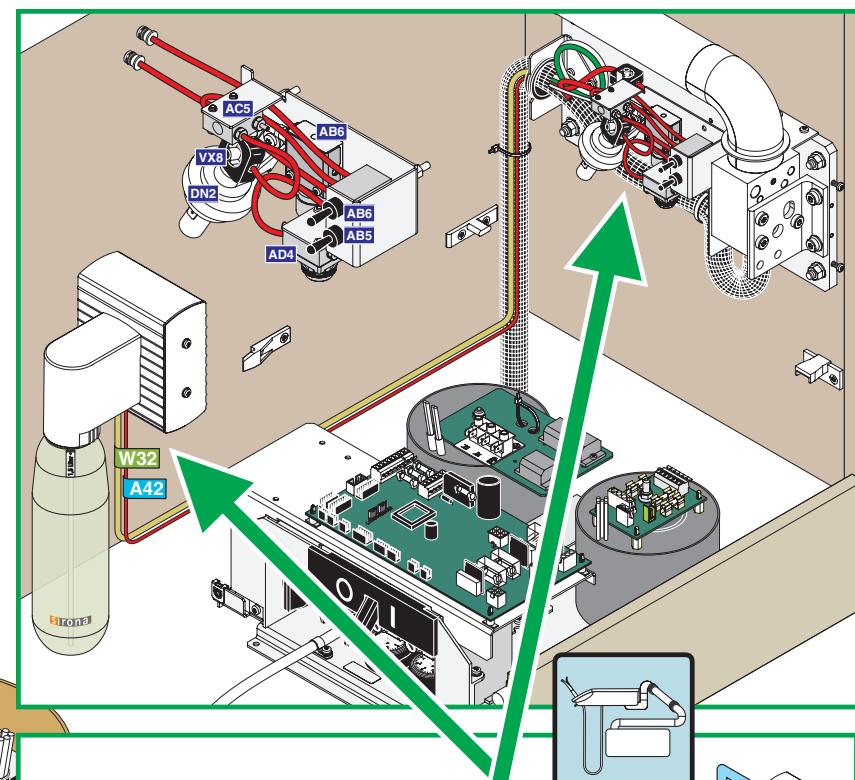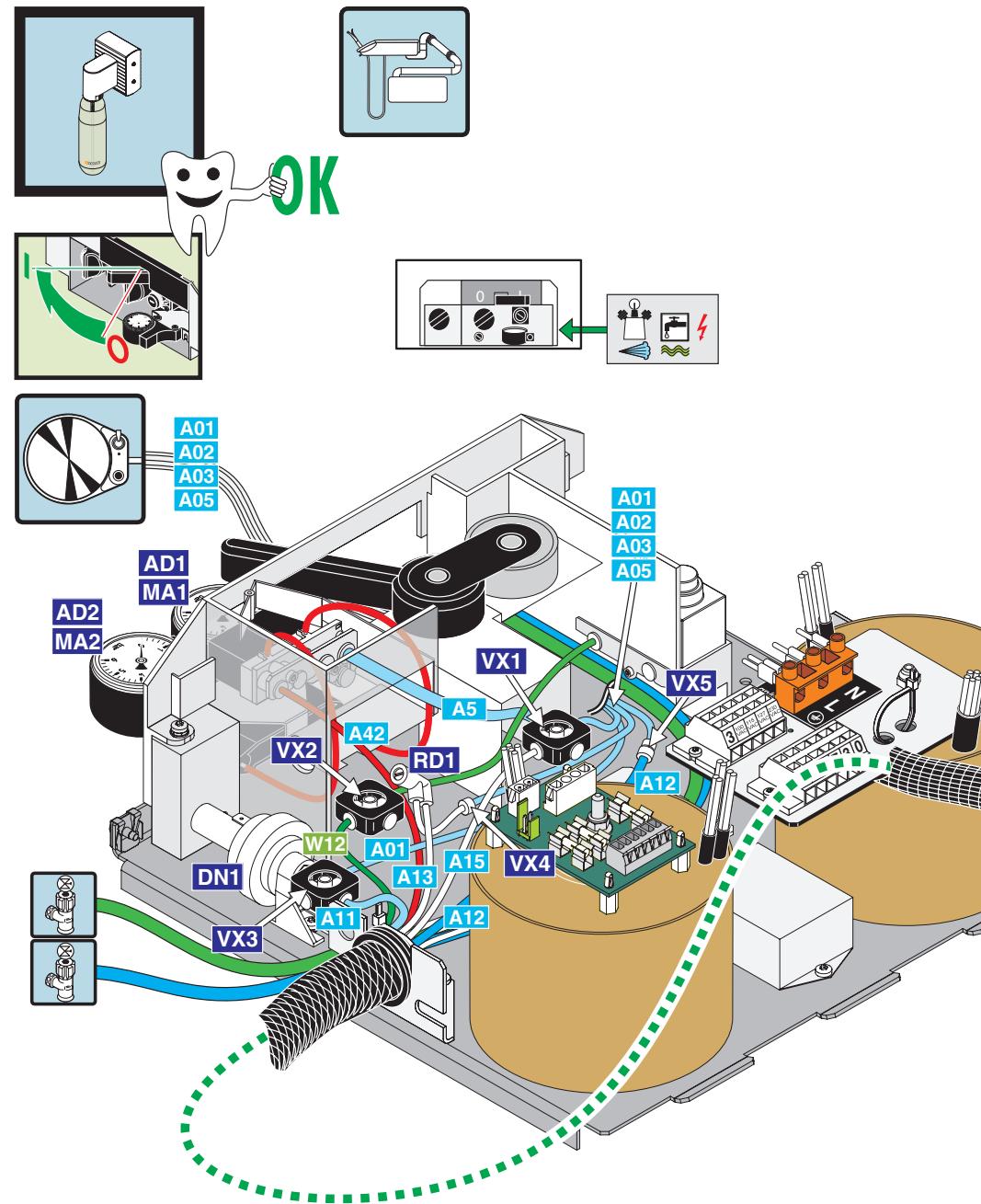


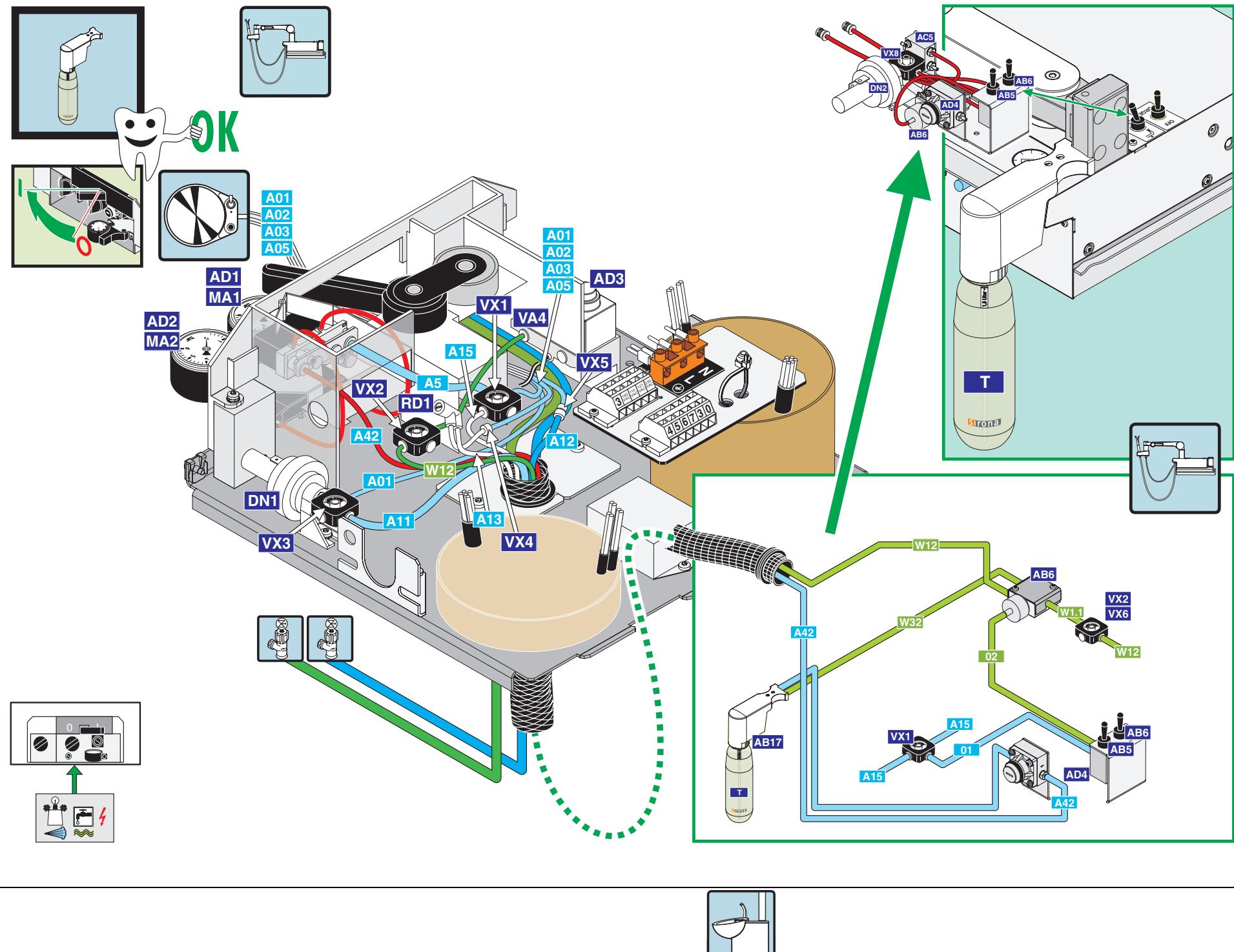


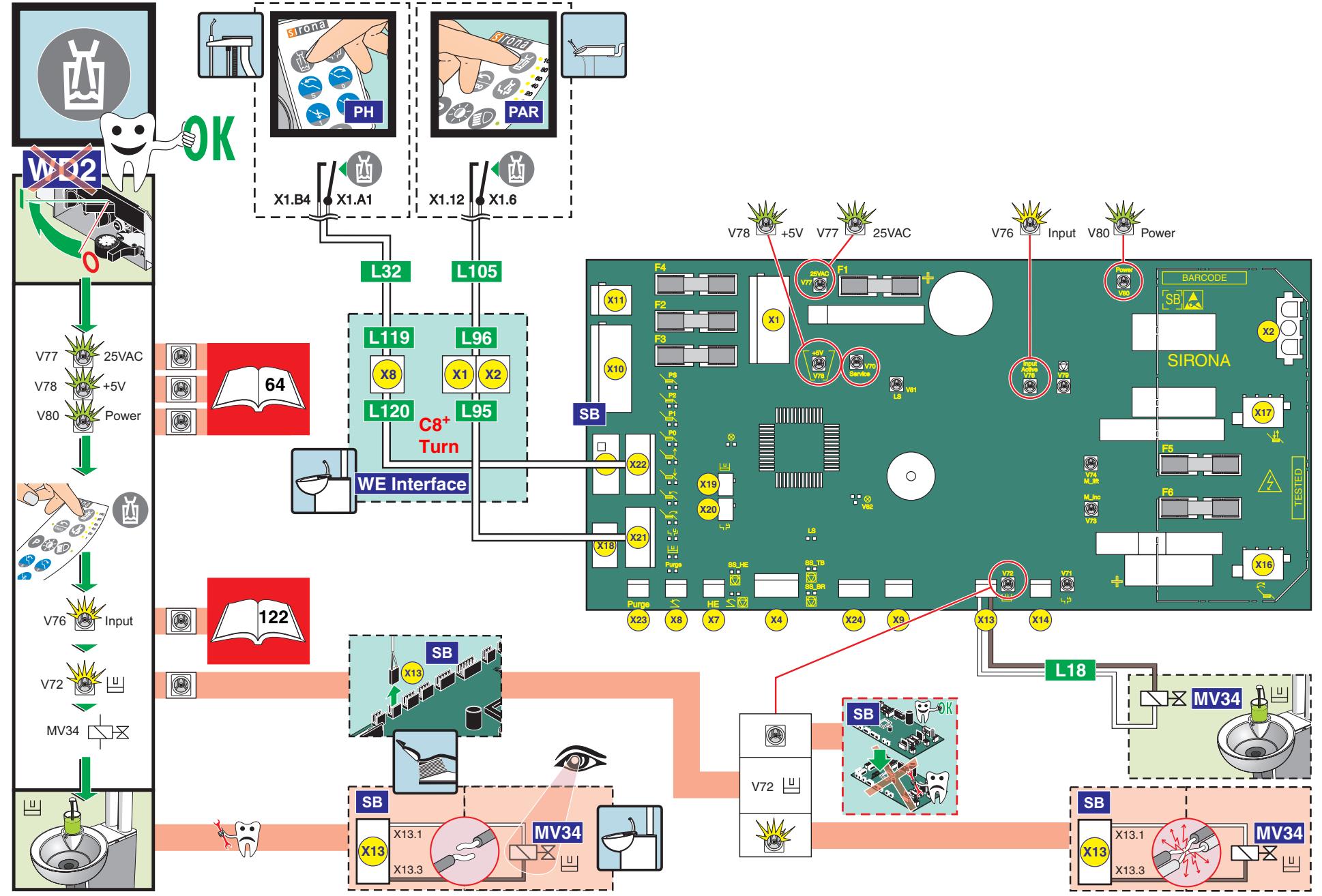


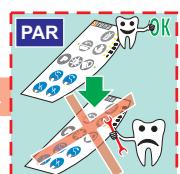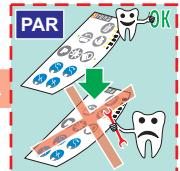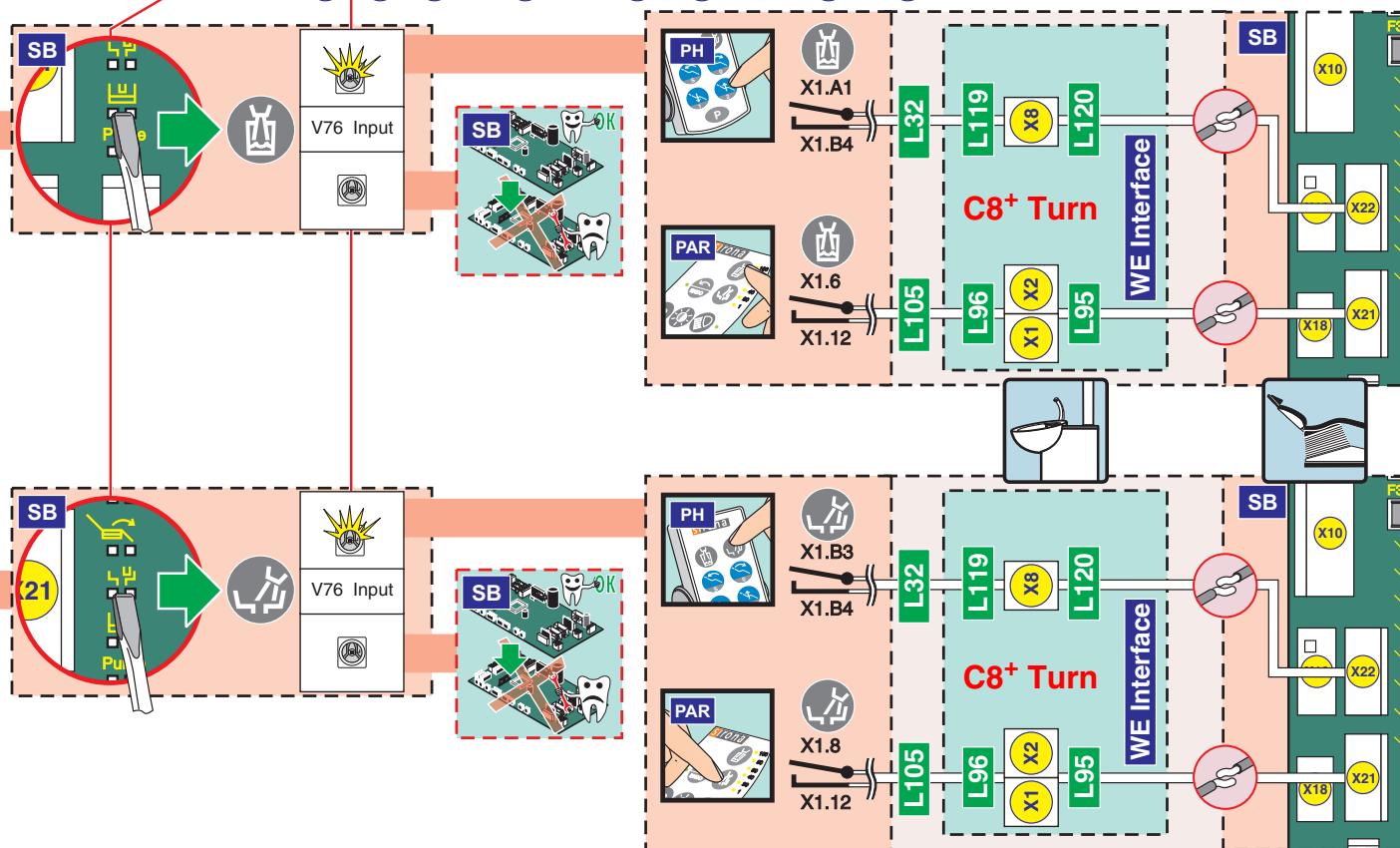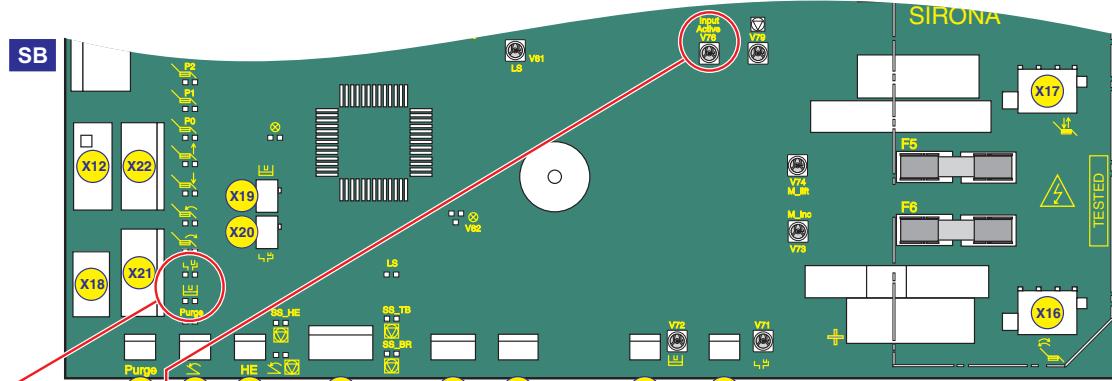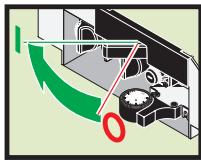





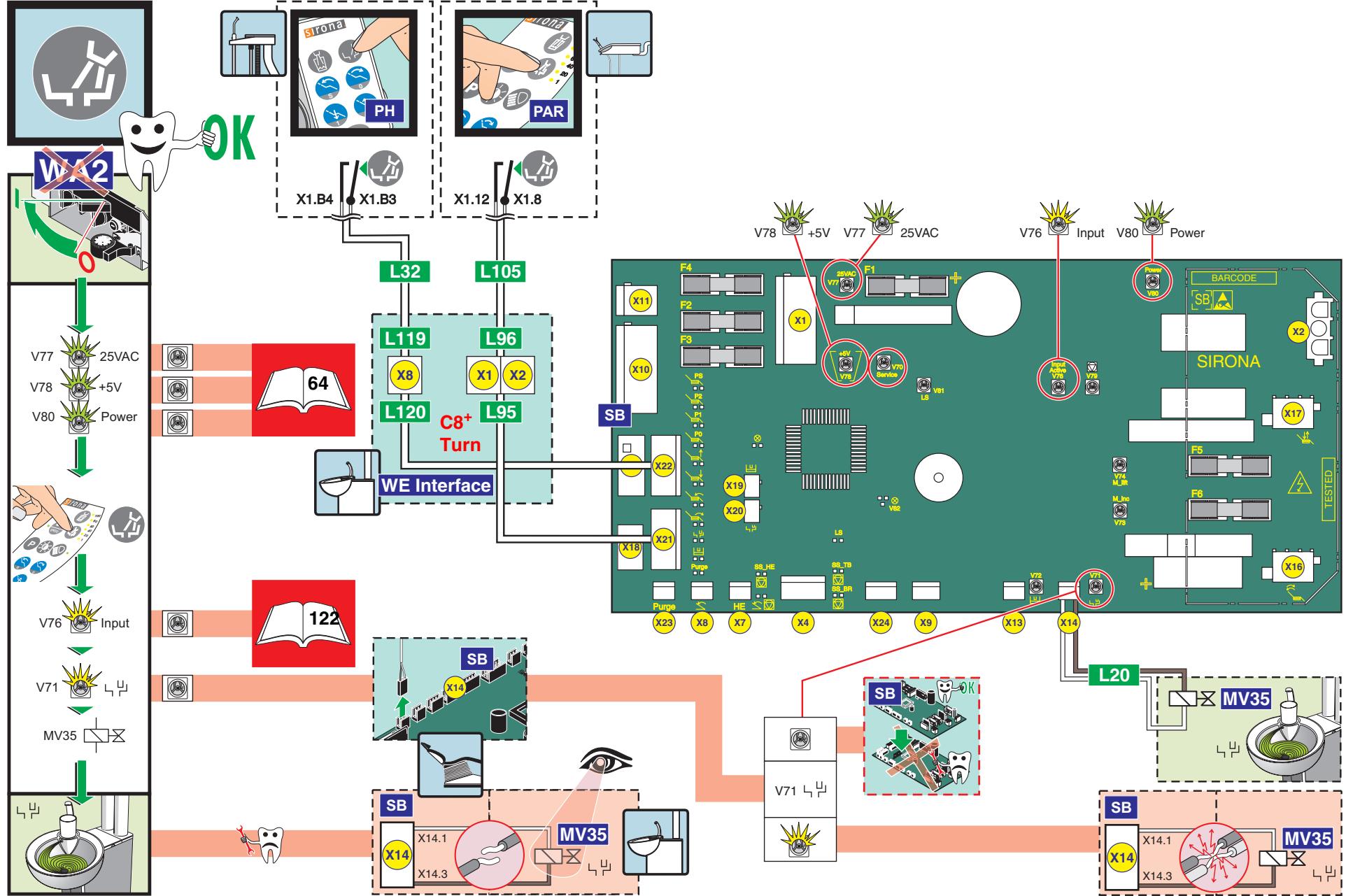




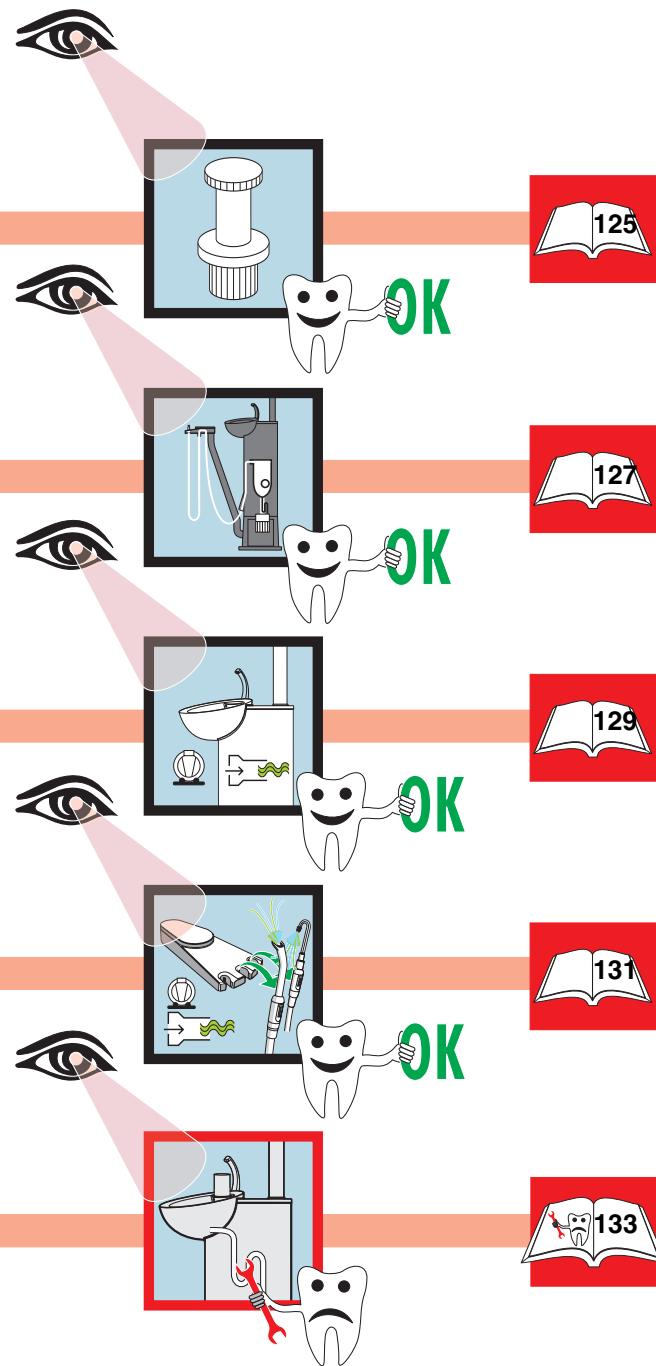



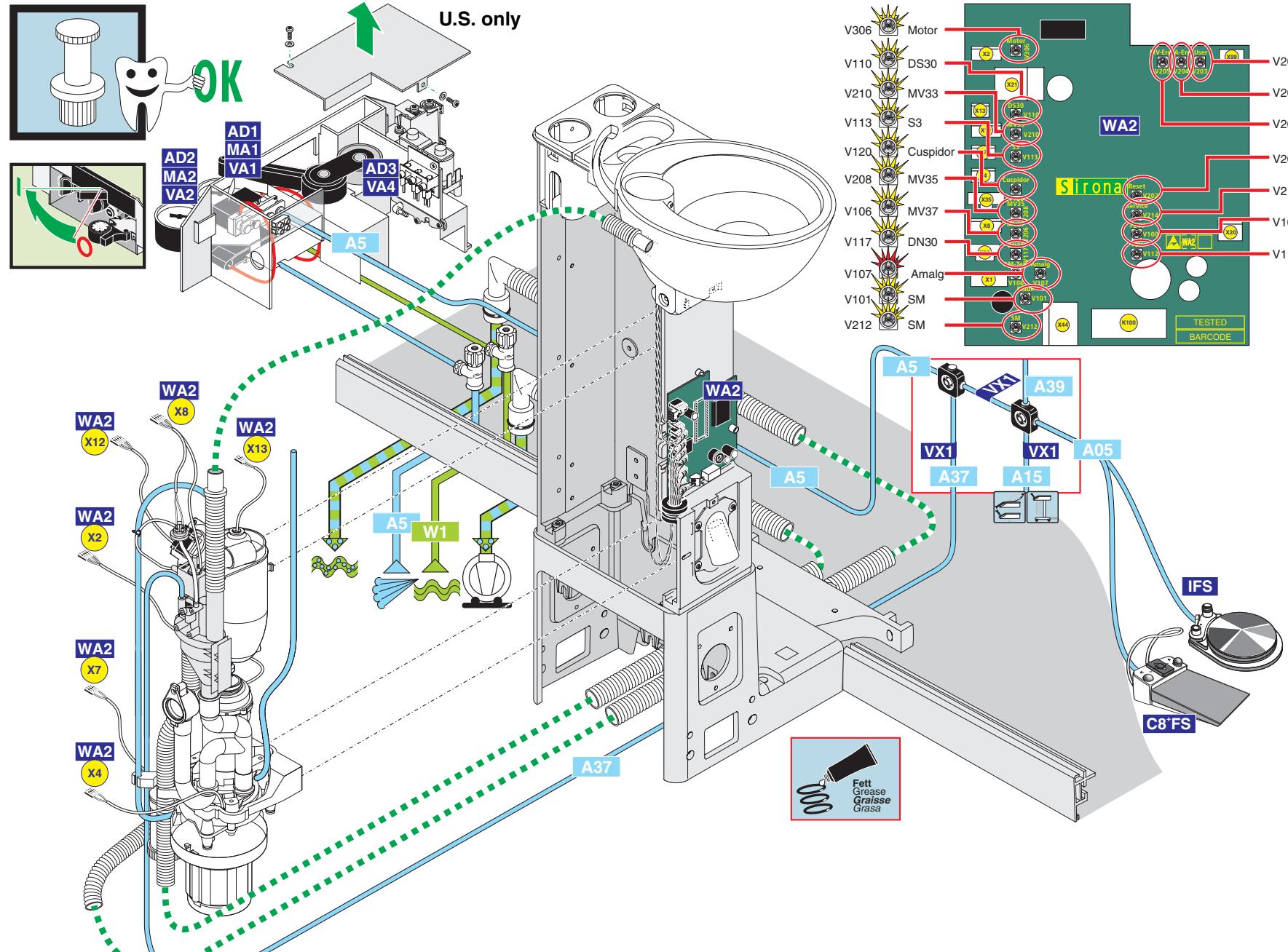


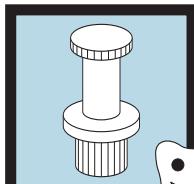

OK

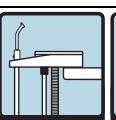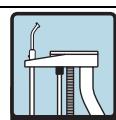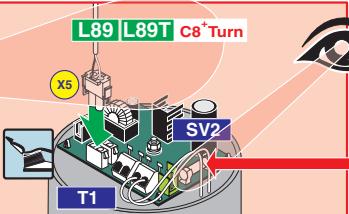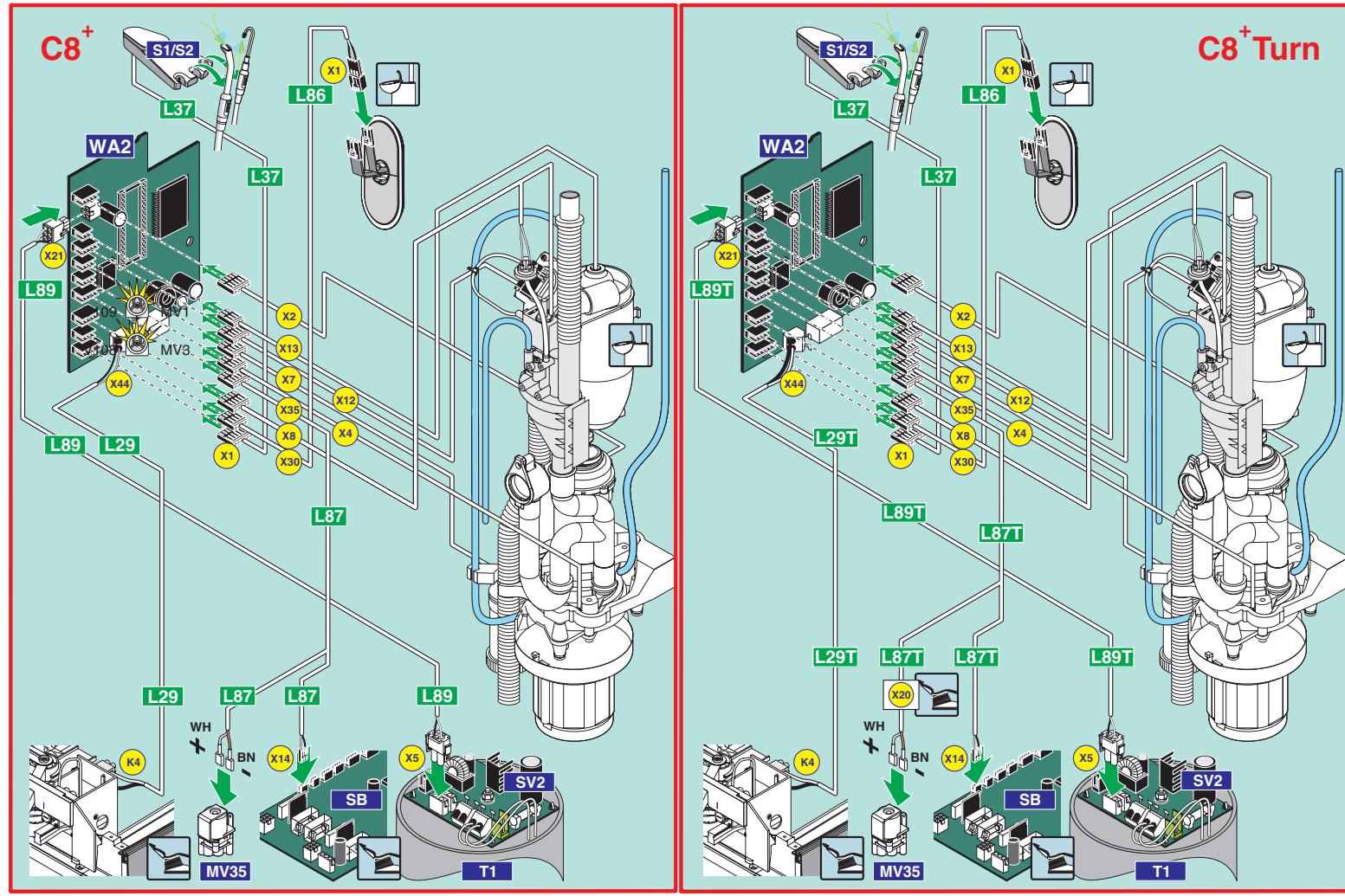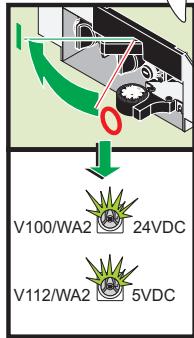

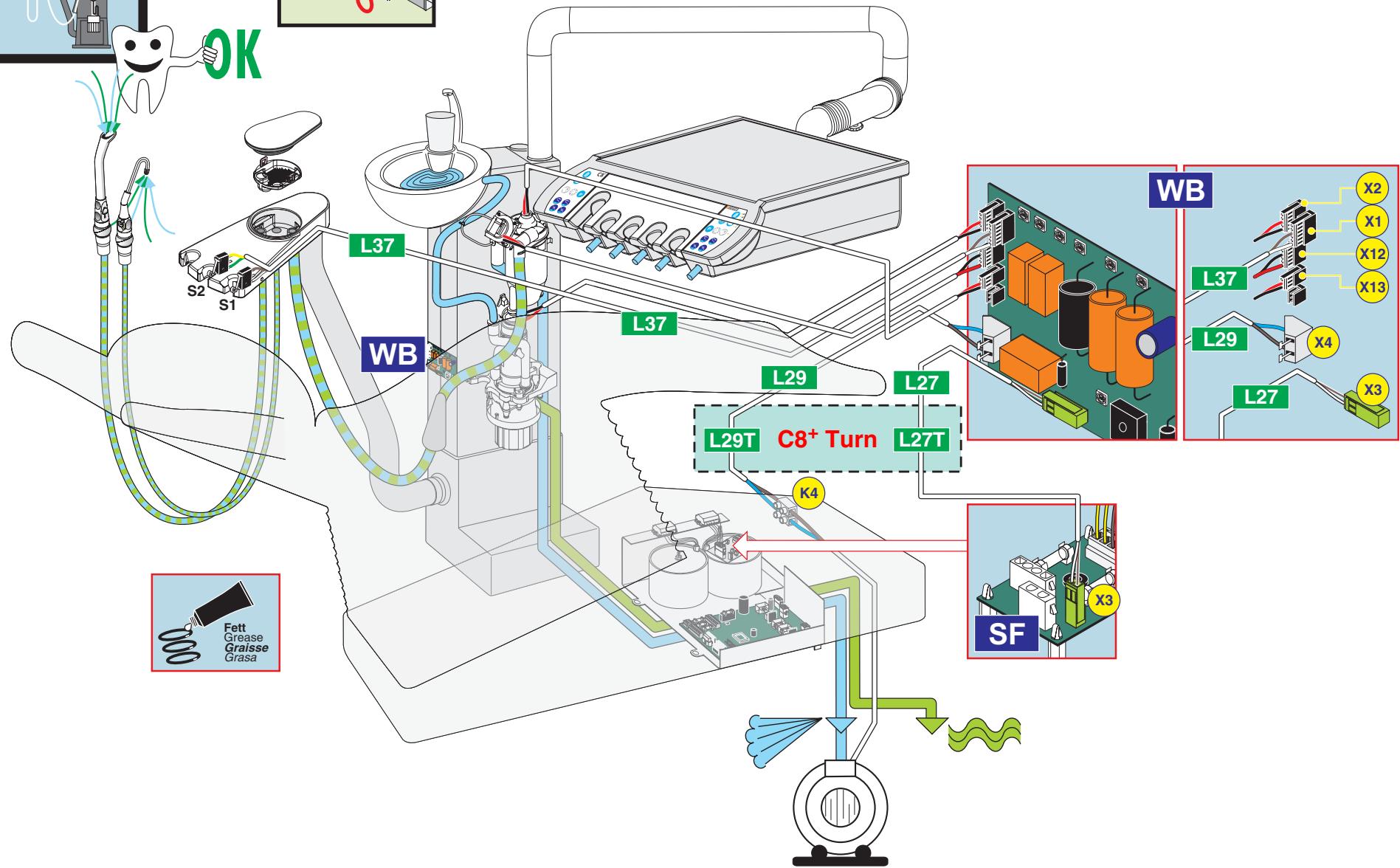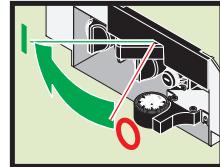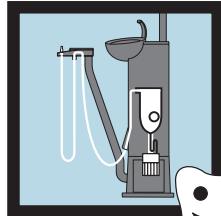

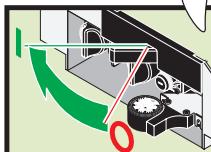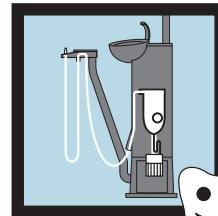

V77/SB 25VAC

V78/SB +5V

V80/SB Power

V7/WB Power

S1/S2







V8/WB ON/OFF


V19/WB SM



V17/WB Pump



V20/WB Reed.Sep.

V9/WB MV

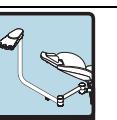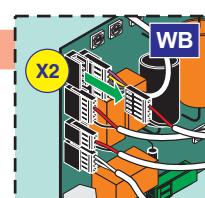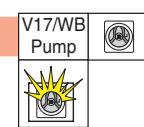
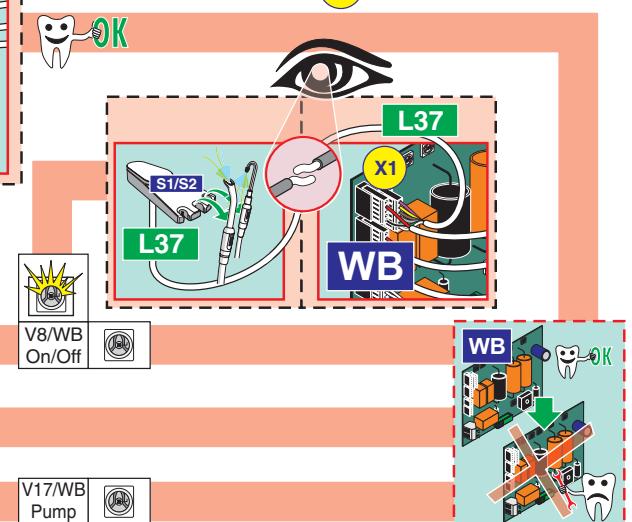
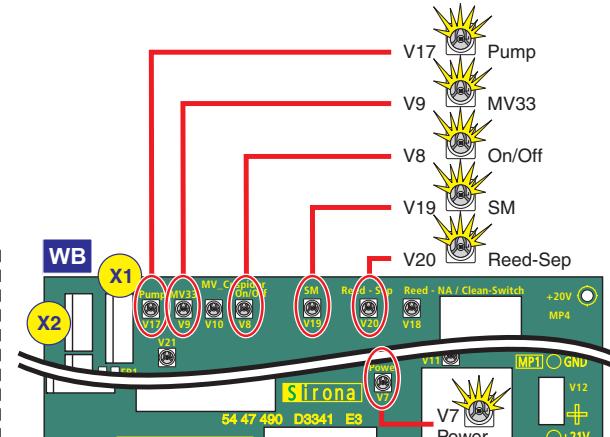
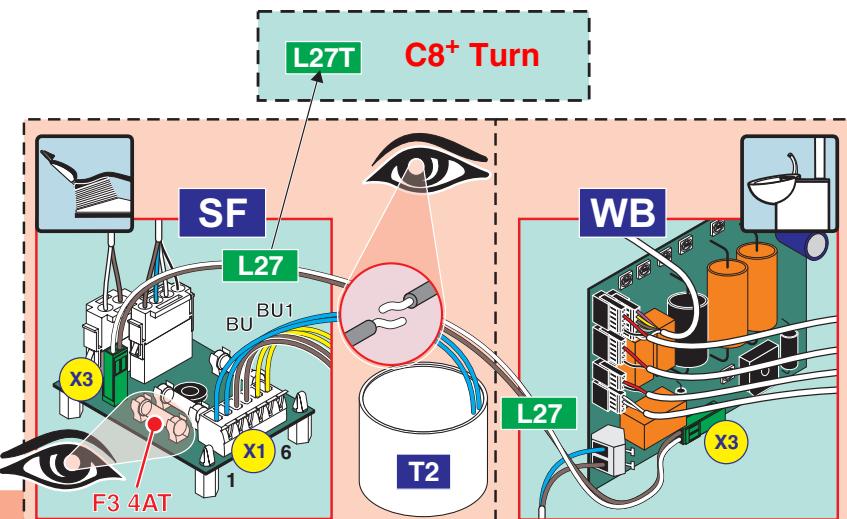










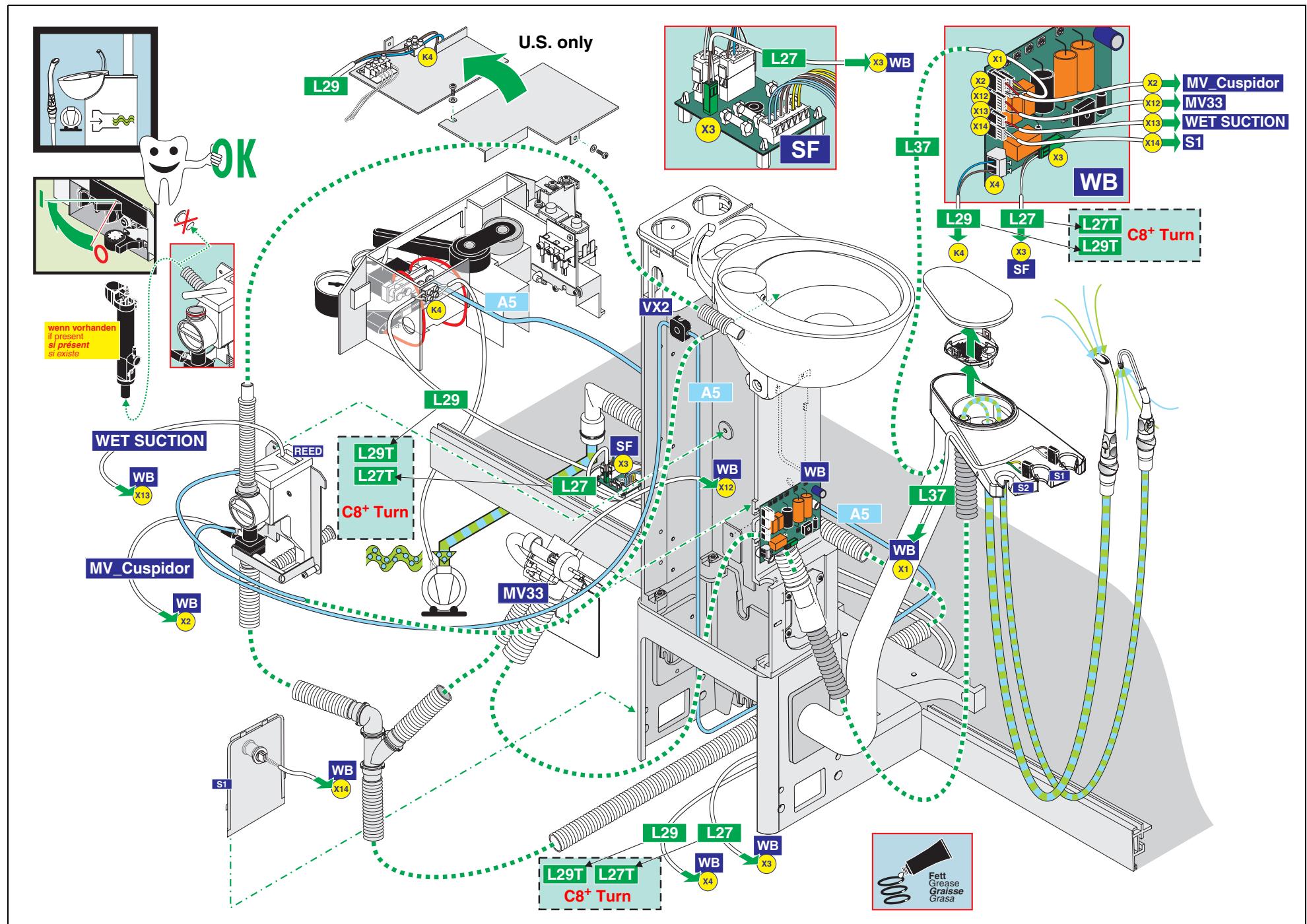


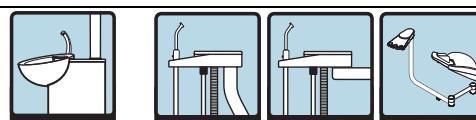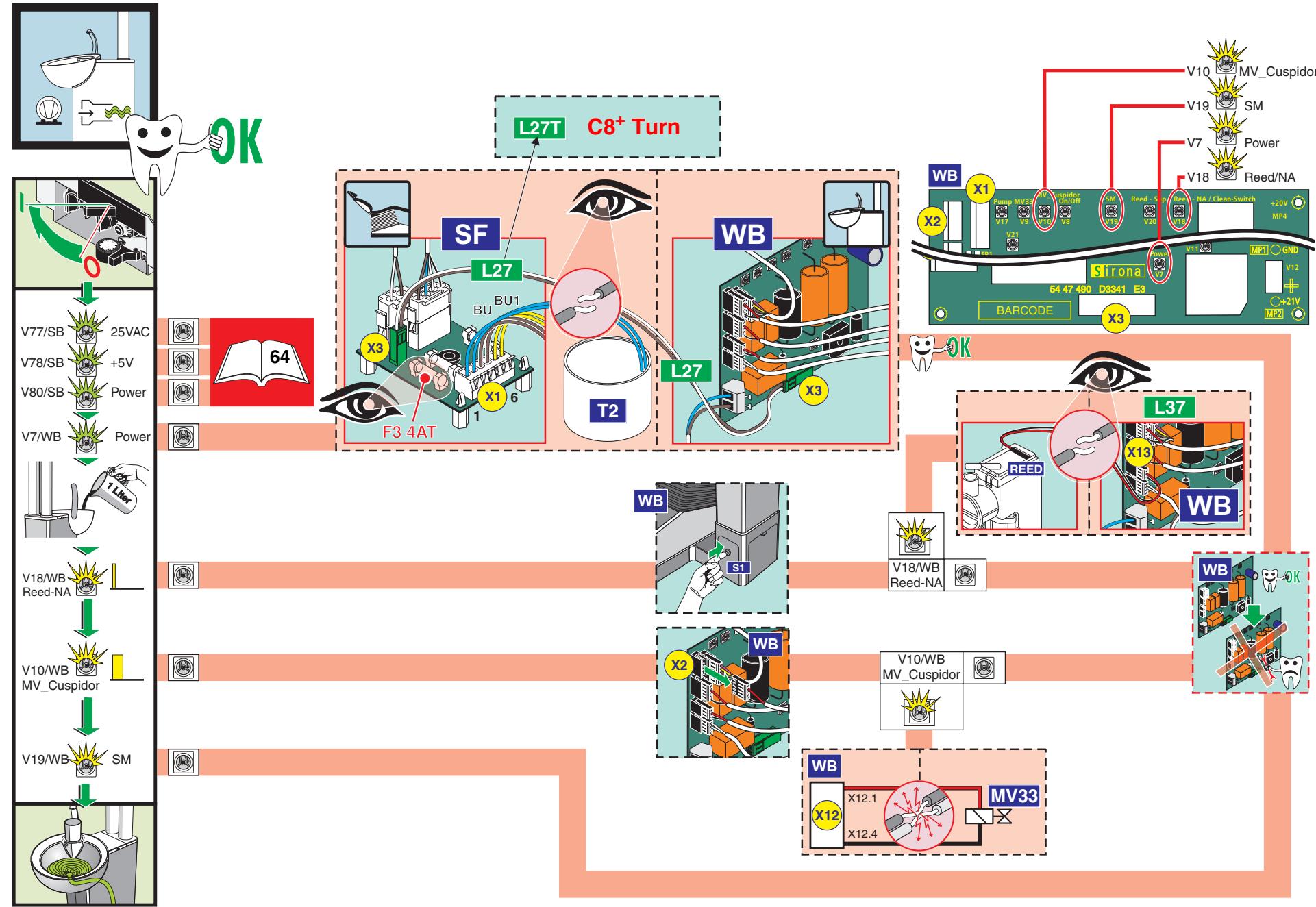

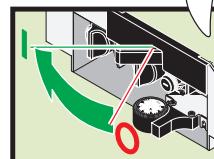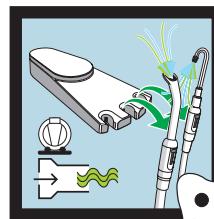


- V77/SB 25VAC
- V78/SB +5V
- V80/SB Power
- V7/WB Power
- S1/S2
- V8/WB ON/OFF
- V19/WB SM
- V9/WB MV33

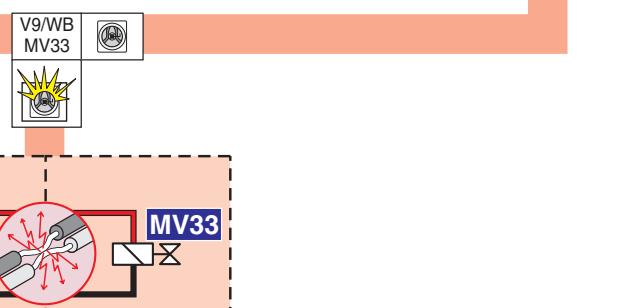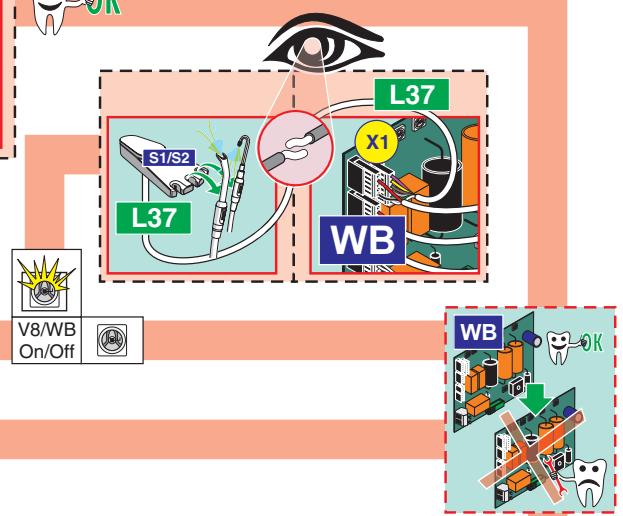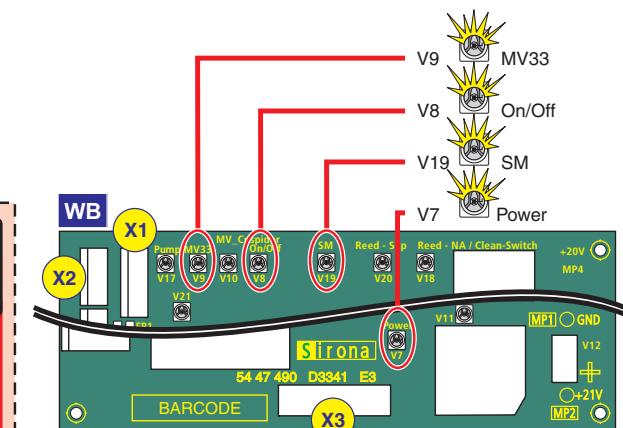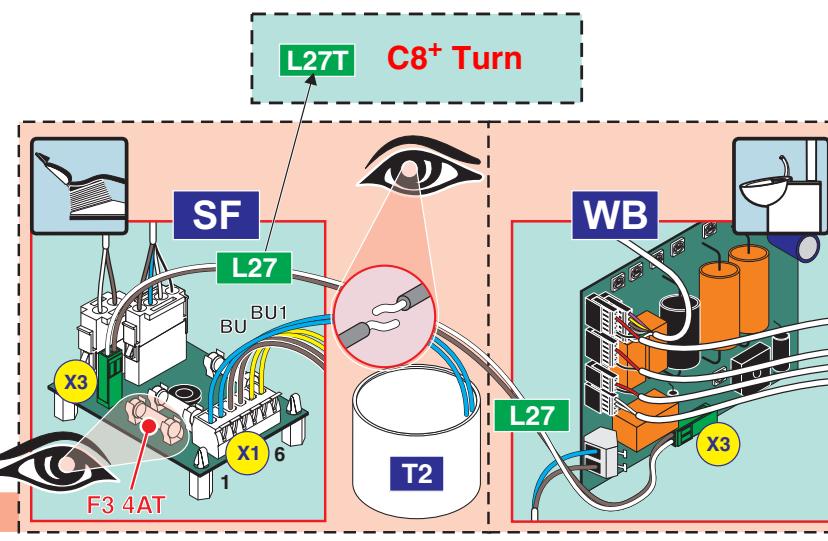

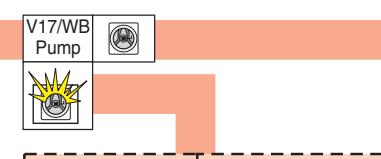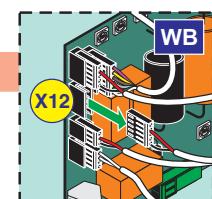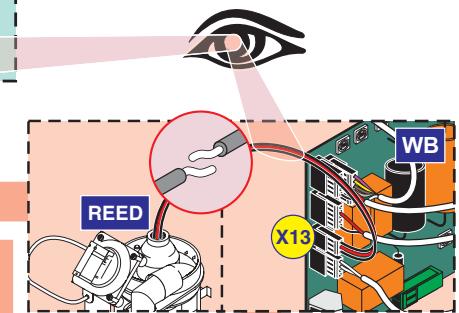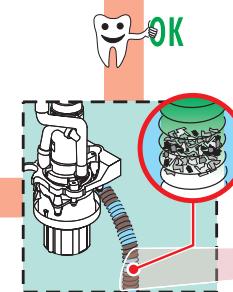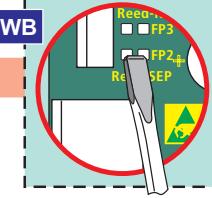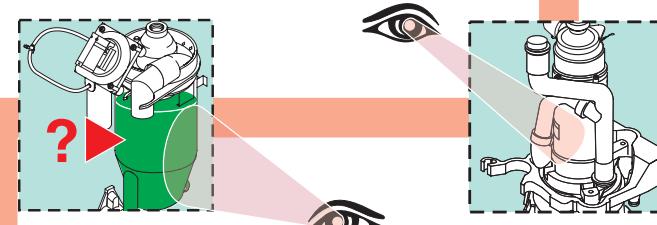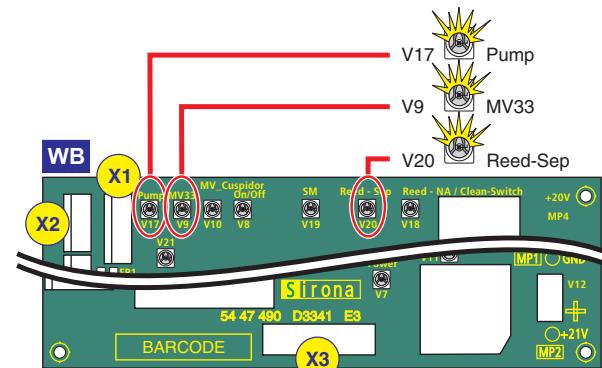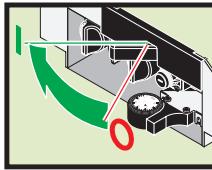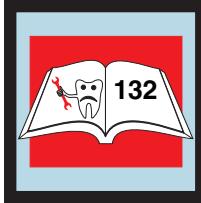


WB





MV33

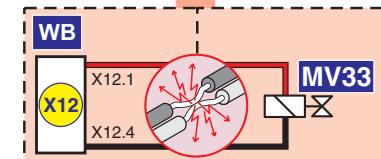









WB

MV33

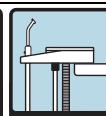


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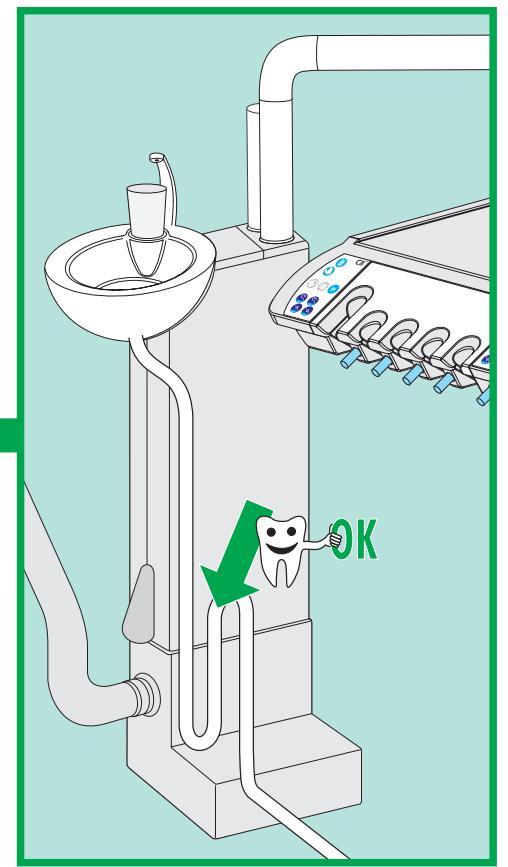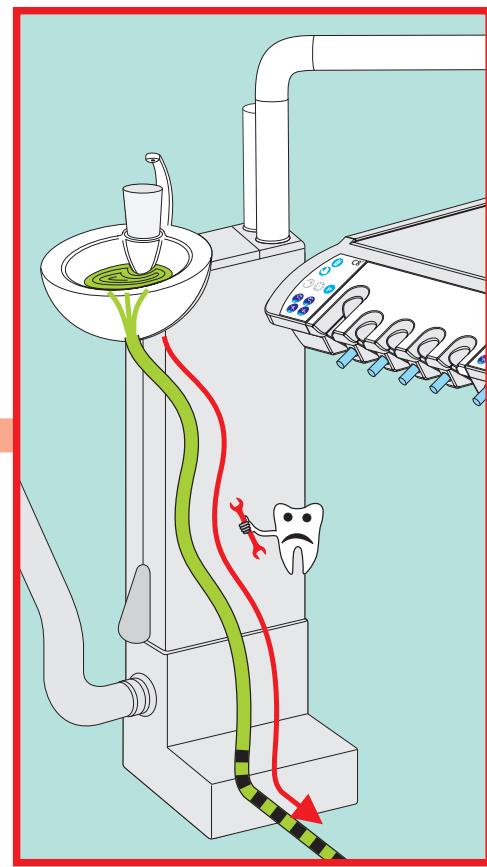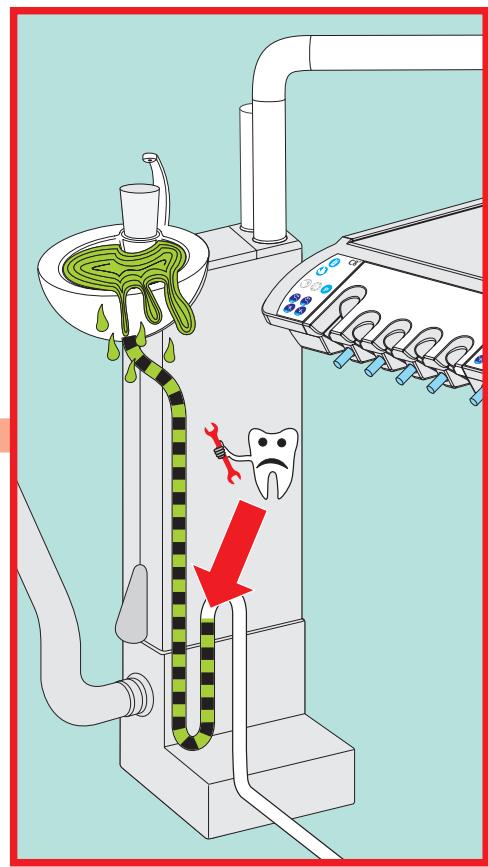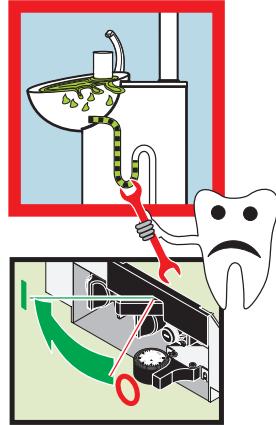


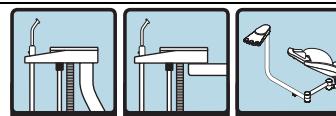

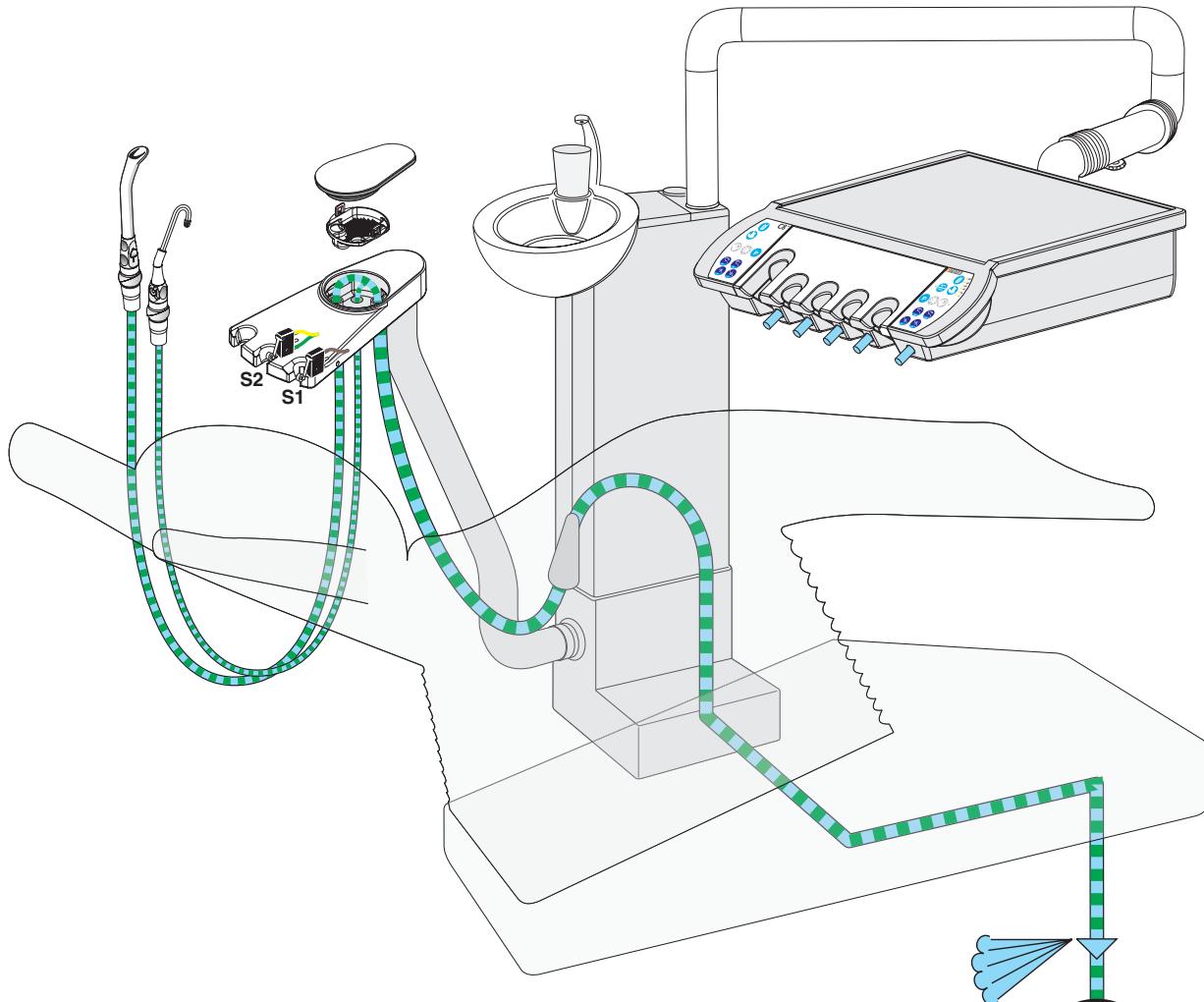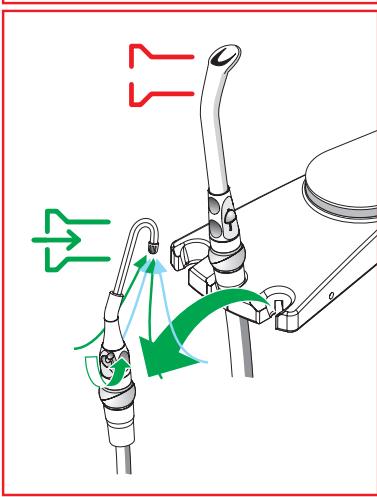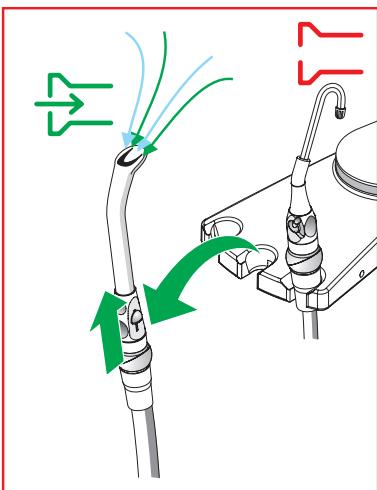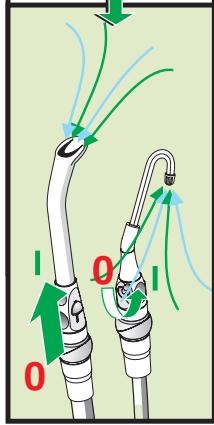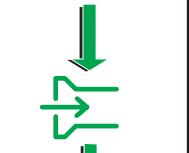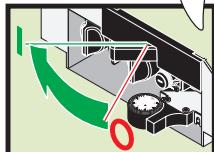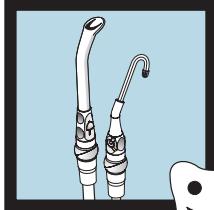



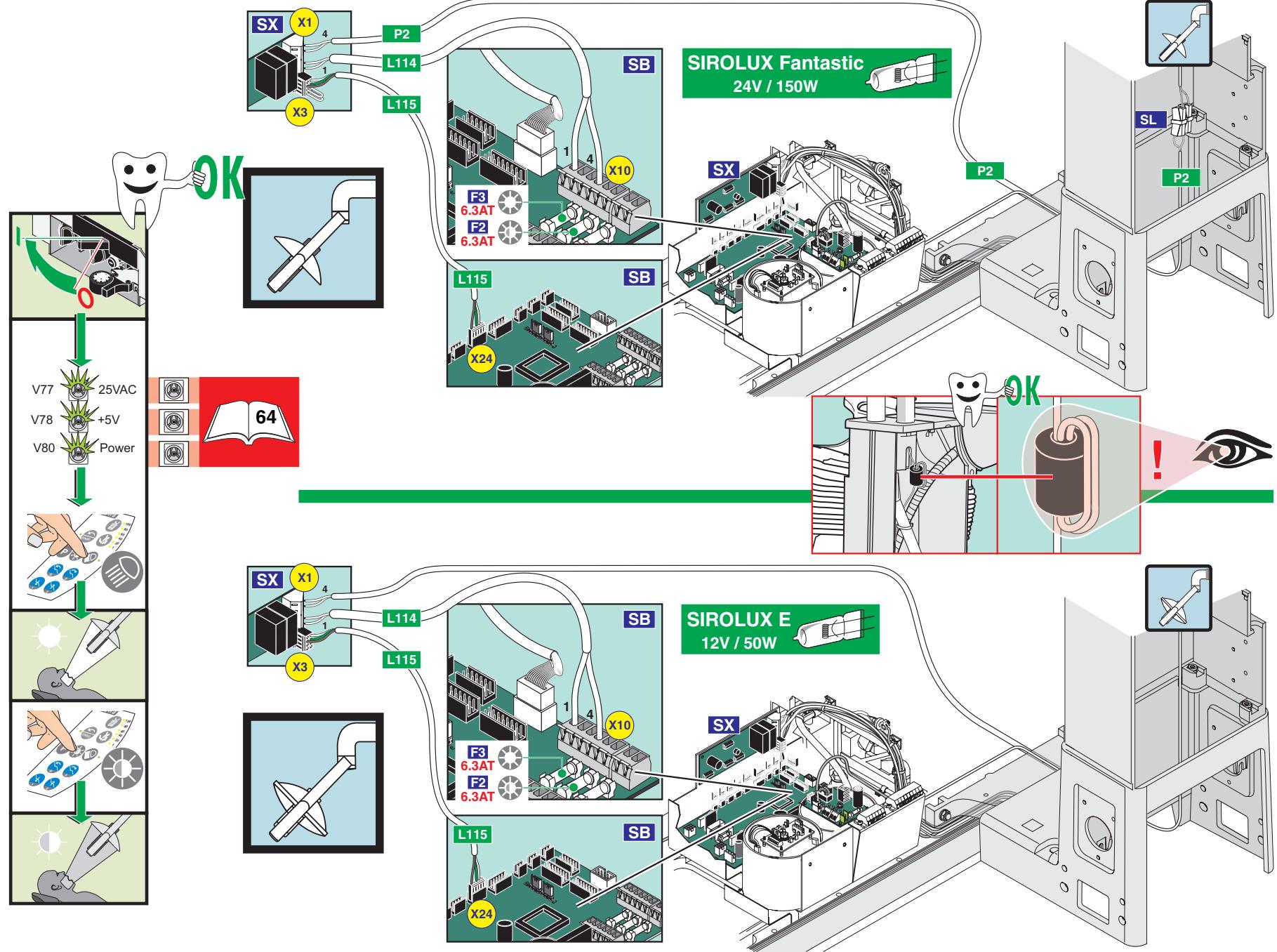



Änderungen im Zuge technischer
Weiterentwicklung vorbehalten

We reserve the right to make any
alterations which may be due to
technical improvements.

Sous réserve de modifications
dues au progrès technique.

Reservados los derechos de modi-
ficación en virtud del progreso téc-
nico.

© Sirona Dental Systems GmbH 2007
D 3417.076.01.03.02 05.2007

Sprache: englisch
Ä.-Nr.: 108 389

Printed in Germany
Imprimé en Allemagne

Sirona Dental Systems GmbH

Fabrikstraße 31
64625 Bensheim
Germany
www.sirona.com

in the USA:

Sirona Dental Systems LLC
4835 Sirona Drive, Suite 100
Charlotte, NC 28273
USA

in Canada:

Sirona Canada
3250 Ridgeway Drive - Unit 5
Mississauga, Ontario L5L 5Y6
Canada

Bestell-Nr.
Order No

59 93 485 D 3417