
C1⁺

Pflege und Reinigung durch das Praxisteam
Care and Cleaning by the Practice Team
Entretien et nettoyage par l'équipe du cabinet
Cuidado y limpieza por parte del personal de la consulta

Deutsch / English / Français / Español

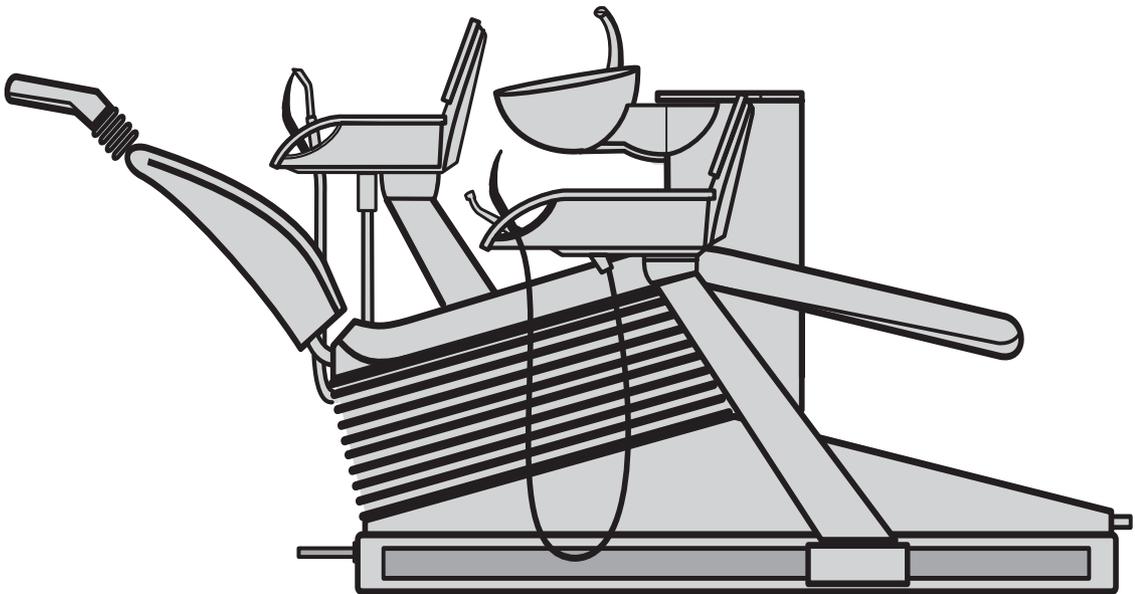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
Bestell-Nr. **59 06 511 D 3386**

C1⁺

Care and Cleaning by the Practice Team

English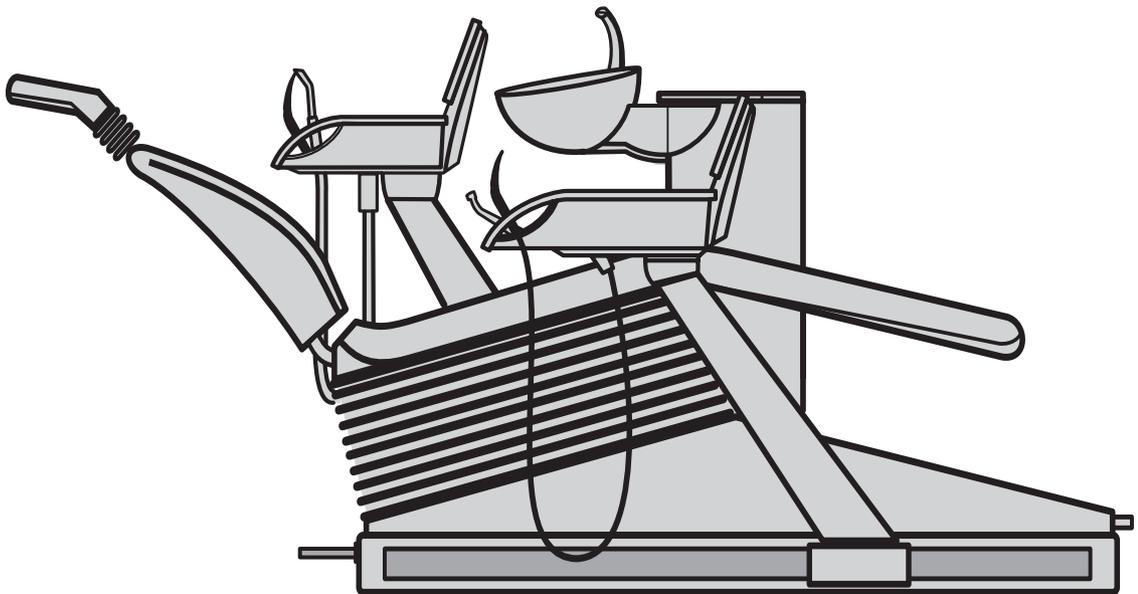

Care and Cleaning by the Practice Team

Dear User,
To maintain the value and safe functioning of your treatment center it is necessary to care for, to clean and to disinfect this regularly.

The work steps you have to perform are described in this document.

Maintenance of the treatment center by the service engineer

Despite the outstanding quality of your treatment center and the regular care performed by the practice team, inspections and maintenance by the service engineer are required at predetermined intervals to ensure its operational reliability and safety.


Please refer to your operating instructions for details.

Your C1⁺ Team

Contents

1	Connection box	6
2	Cuspidor.....	7
3	Vacuum system	8
	3.1 Care	8
	3.2 Disinfection when using dry aspiration.....	8
	3.3 Disinfection when using wet aspiration	10
	3.4 Suction handpieces.....	11
	3.5 Cleaning the suction hoses.....	12
	3.6 Thermodisinfecting the suction hoses.....	13
4	Amalgam separation	14
5	Instrument hoses.....	15
	5.1 Removal/replacement of instrument hoses.....	15
	5.2 Turbine hose	15
6	Disinfection system.....	16
	6.1 Microbiological water test.....	16
	6.2 Sanitation	17
	6.3 Aborting the sanitation process.....	22
7	Surfaces / O-rings.....	23
8	POLYLUX fiber optic light handpiece.....	25
	8.1 Care and cleaning of the handle sleeve.....	25
	8.2 Care and cleaning of the light guide	25
9	SIROCAM 3 / SIROCAM C video camera and monitor	26
	9.1 Care and cleaning of the SIROCAM 3	26
	9.2 Care and cleaning of the SIROCAM C	27
	9.3 Care and cleaning of the monitor	27
10	Checking the safety switches	28

English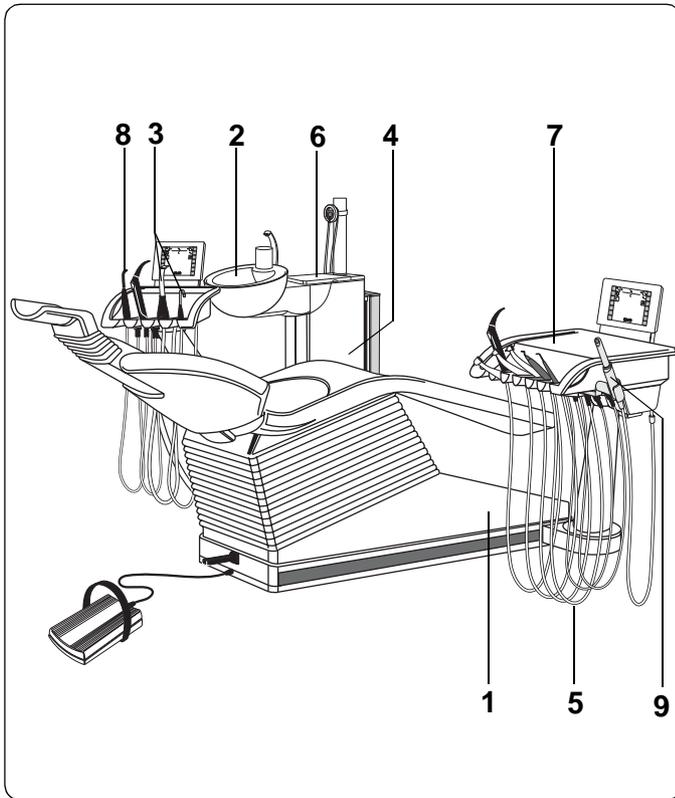

Overview


The numbers indicated in the overview correspond to the chapter numbers in the table of contents. Thus you can easily find the work to be performed in this document.

- 1 Connection box
- 2 Cuspidor
- 3 Vacuum system
- 4 Amalgam separation
- 5 Instrument hoses
- 6 Disinfection system
- 7 Surfaces
- 8 POLYLUX fiber optic light handpiece
- 9 SIROCAM 3 / SIROCAM C video camera monitor
- 10 Checking the safety switches

Please carry out care and maintenance of the following products as described in the relevant operating instructions supplied in each case.

- **SPRAYVIT** syringe,
- **T1 TITAN, T1 CLASSIC** and **T1 LINE** straight and contra-angle handpieces,
- **Electric motors,**
- **Turbines,**
- **SIROSONIC L** scaler,
- **SIROTOM** electrosurgical unit,
- **SIROCAM 3** video camera,
- **SIROLUX FANTASTIC** unit lamp

1 Connection box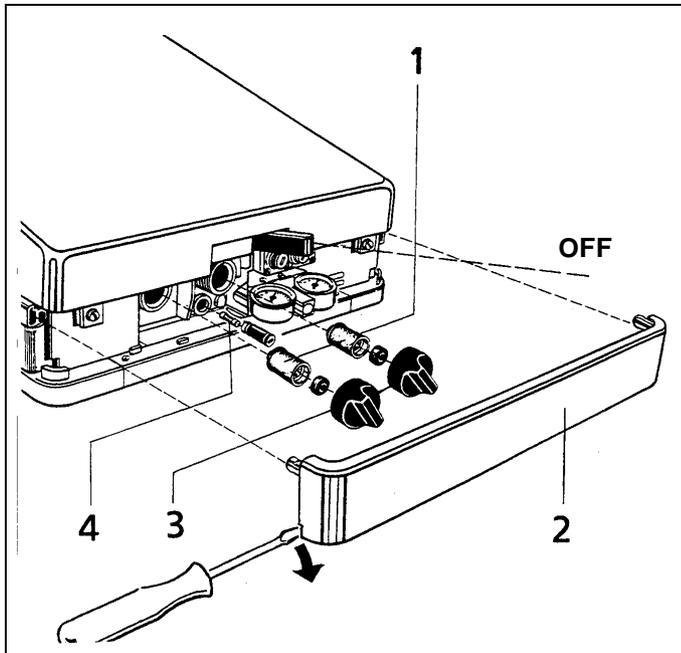

Should you notice changes of the media flows, please check the filters for water and air (1) for permeability. Replace them if necessary.

Replace the filters (1) for water and air.

To do so, first turn the main switch OFF. Pry off the cover (2) with a screwdriver.

Remove the screw caps (3) and replace the filters.

Filter

Order No. 14 43 436

The **main fuse (4)** is located below the air filter.


Unit main fuse for 230V AC:

T 6.3A H, 250V AC, Order No. 10 77 452

Unit main fuse for 100V AC / 115V AC:

T 10A H, 250V AC, Order No. 10 77 460

2 Cuspidor

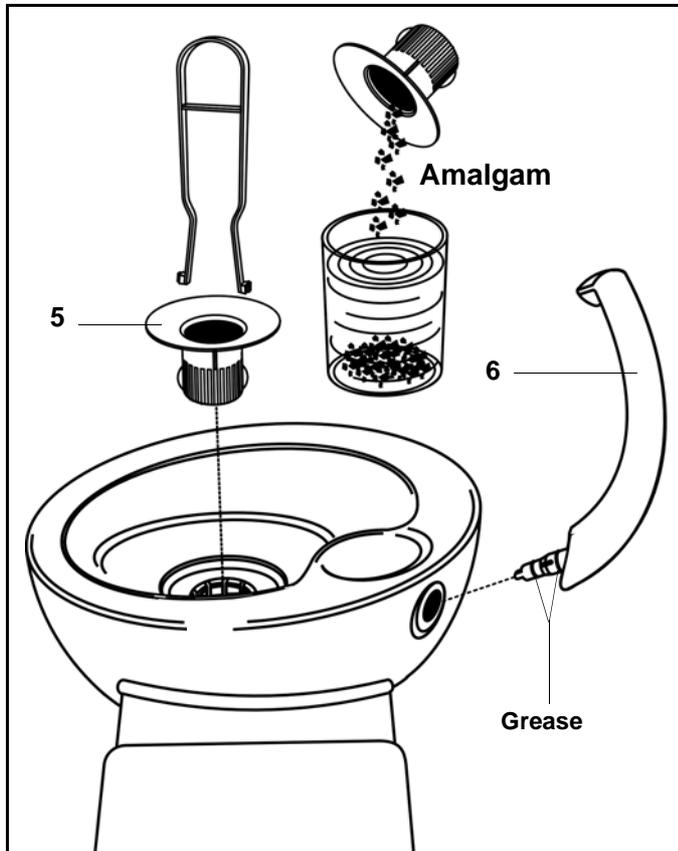After each patient

Clean the cuspidor.

ATTENTION

Use only cleaning agents which have been approved by Sirona!

ATTENTION

Do not use any domestic cleaning and care agents. Foaming domestic cleaning agents lead to foam and water being drawn into the dry aspiration system, resulting in damage to the suction machine.

Daily before the mid-day break and after the practice has closed

Remove the gold trap (5) and clean it.

Store amalgam residues under water in a separate closed container. When the amalgam rotor is replaced (see page 14), put the residues into the rotor for disposal.

The cuspidor must be filled with at least 1 liter of water for rinsing after the cleaning process.

ATTENTION

Amalgam residues in the gold trap must not be disposed of in the cuspidor or the sink!

Weekly

Pull out the tumbler outlet (6) or the hydrocolloid adapter, if any, and clean it.

Before refitting the tumbler outlet or the hydrocolloid adapter, lightly lubricate the O-rings.


ATTENTION

Use only lubricants which have been approved by Sirona!

3 Vacuum system

3.1 Care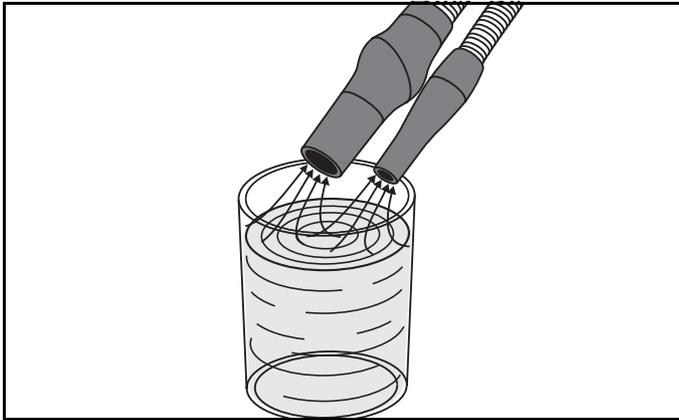


After each patient

To ensure that the **vacuum system** is always ready for use, a large glass of cold, clear water must be aspirated through the used suction hoses **after each patient**, especially after each intervention during which the patient loses blood and after using hydrogen peroxide (H₂O₂).

In **long-lasting treatments** a glass of water must be aspirated at least **every 60 min.**

3.2 Disinfection when using dry aspiration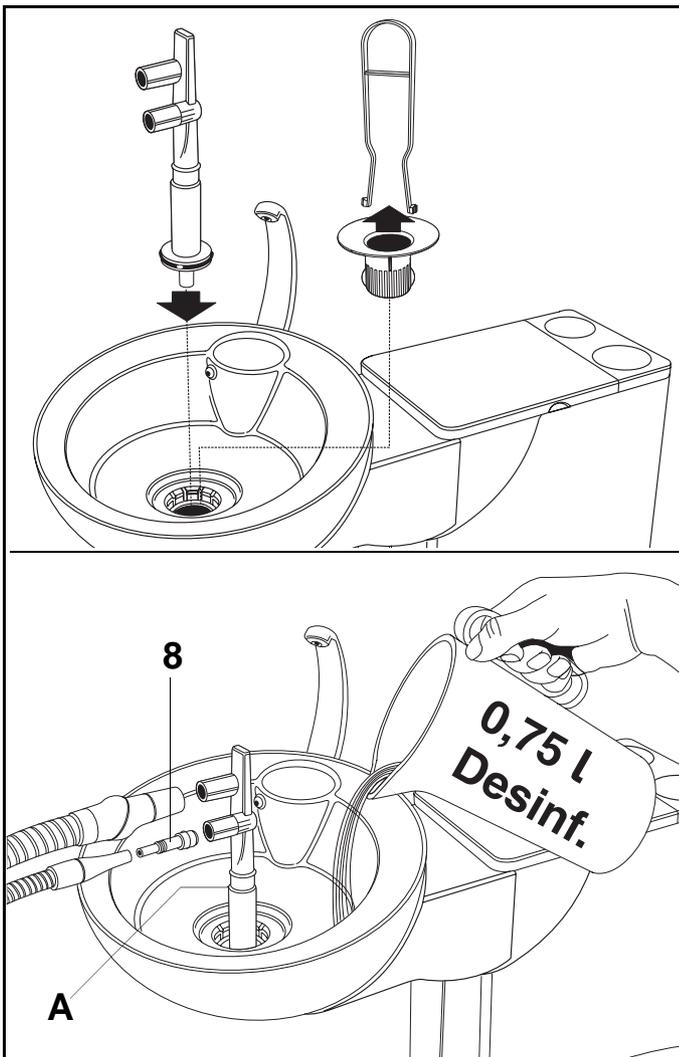

Daily before the mid-day break and after the practice has closed

The vacuum system is subjected daily to germ-carrying secretions, saliva and blood.

For hygienic reasons the system must therefore be disinfected with a detergent/disinfectant approved by Sirona.


ATTENTION

Use only cleaning and disinfecting agents which have been approved by Sirona !


Do not use any detergents or other disinfectants!

- Prepare the cleaning solution in a separate container according to the manufacturer's instructions and **mix thoroughly**.
- Remove the gold trap.
- Clean the cuspidor.
- Plug in the cleaning adapter up to the stop. For the saliva ejector hose, insert the intermediate piece (8) in the cleaning adapter.
- Fill the cleaning solution up to the filling limit (A) in the cuspidor (approx. 0.75 liters).
- Remove the suction hoses from their receptacles and attach them to the side of the cleaning adapter, if possible simultaneously.
- The cleaning solution is aspirated.
The cleaning process takes several minutes.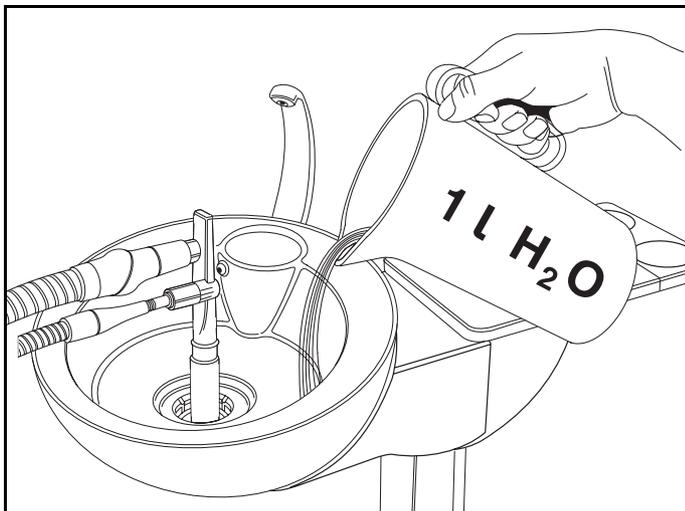

- After the cleaning process, at least 1 liter of water must be filled into the cuspidor and aspirated in order to remove any disinfectant residues from the suction hoses.
- At the end of the cleaning process, pull off the hoses and deposit them in their receptacles.
- Remove the cleaning adapter.
- Finally, refit the gold trap.

3.3 Disinfection when using wet aspiration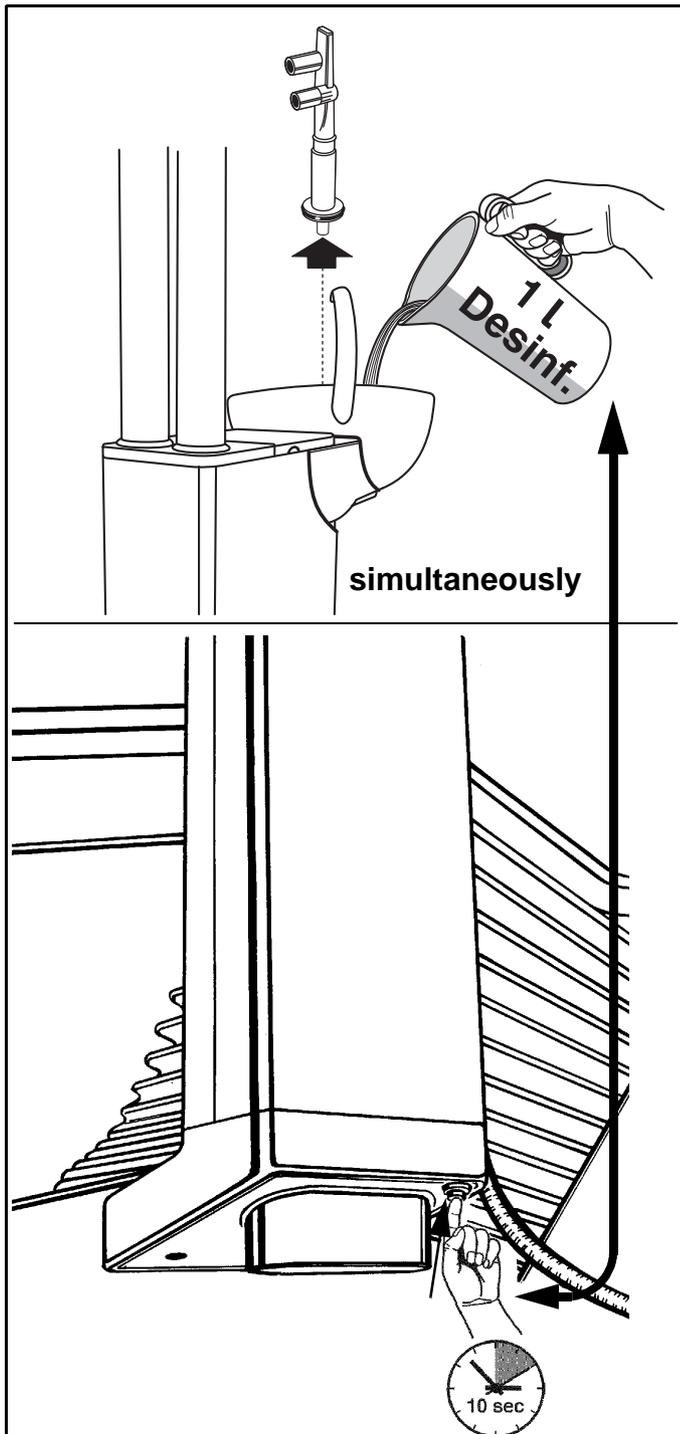

With this system, the automatic separating device and, where appropriate, the amalgam separator are not integrated into the water unit. Both separation processes are performed centrally.

Daily before the mid-day break and after the practice has closed

- Flush through the vacuum system as described under **3.2 Disinfection for dry aspiration** with a suitable cleaning and disinfection agent.
- After the liquid has been aspirated through the two suction hoses, remove the cleaning adapter.
- After this, pour 1 liter of disinfectant solution into the cuspidor while **simultaneously** pressing the button (arrow) for approx. 10 seconds. Pressing the button enables the cleaning and disinfectant liquid from the cuspidor to reach the vacuum line via the cuspidor valve.
- Finally, refit the gold trap.


ATTENTION

Use only cleaning and disinfecting agents which have been approved by Sirona!

Do not use any detergents or other disinfectants!

3.4 Suction handpieces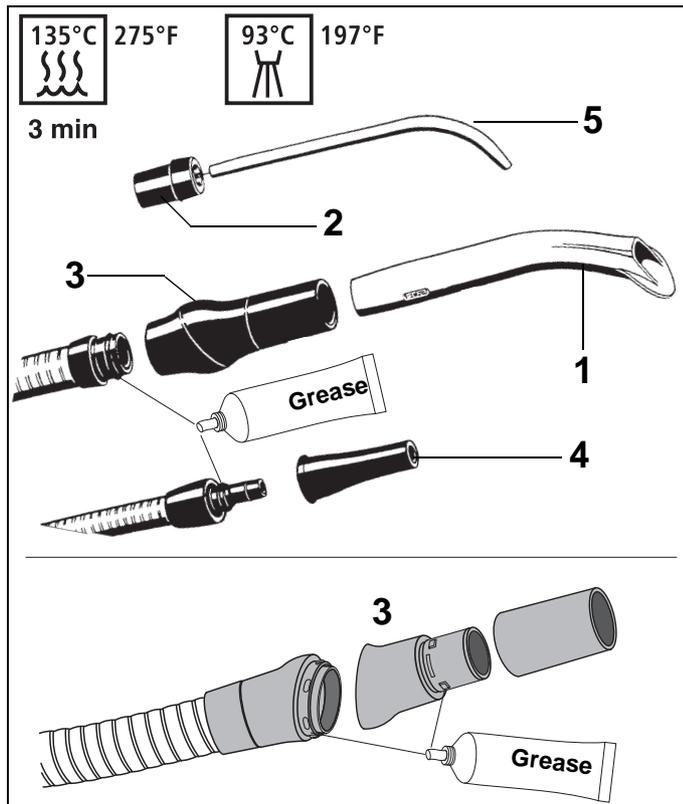

Sterilization

The suction cannula (1), the intermediate piece (2), the suction handpiece (3), the saliva ejector handpiece (4) and the surgical suction cannula (5) can be sterilized.

Sterilize only in an autoclave at 135°C, 2.1 bar.

Thermodisinfection

All parts can be thermodisinfected at 93°C (197°F).

Disinfection

All parts can be sprayed or wiped with a disinfecting solution.

ATTENTION

Use only cleaning agents which have been approved by Sirona !

Grease the O-rings at the joints afterwards!

Weekly


Separate the suction handpiece at the joint, clean the handpiece, and grease the joints.

ATTENTION

Use only lubricants which have been approved by Sirona!

3.5 Cleaning the suction hoses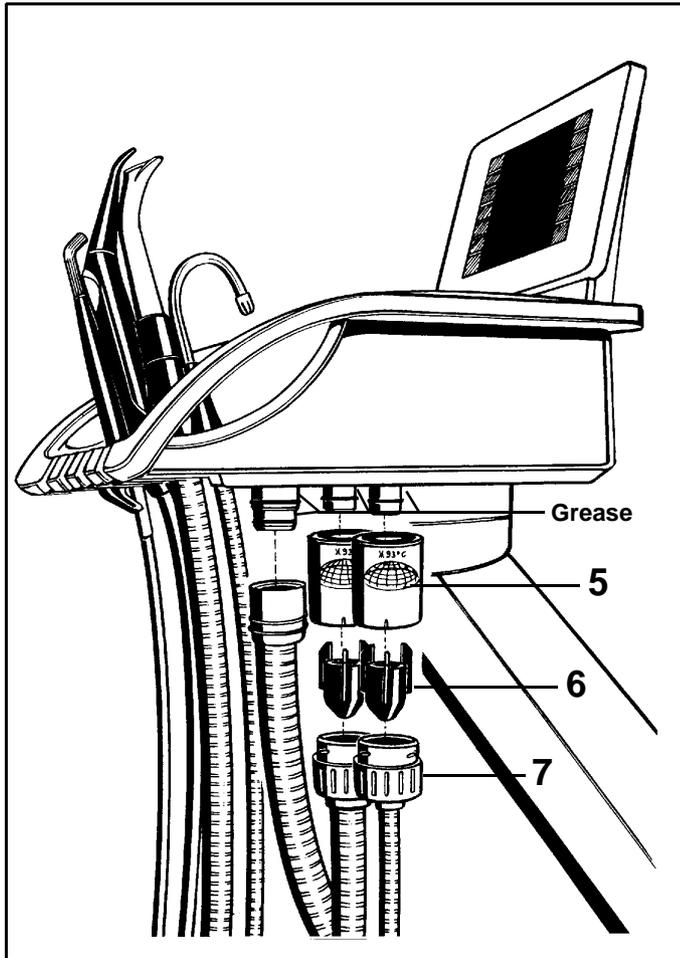

The hoses of the saliva ejector and suction cannula as well as the connection hose to the water unit can be pulled off from the assistant element for rinsing under running water or thermodisinfection.

Switch the unit OFF first!

Unscrew the suction hoses from the top parts (5) of the filter housings. Rinse the hoses under running water.

The top parts of the housings (5) with **filter inserts** must be **thoroughly cleaned every day**.

The top parts of the filter housings can be thermodisinfectated.

Remove the amalgam residue **collectors** (6) from the lower parts (7) if the suction strength decreases, otherwise **once a day**.

Store amalgam residues under water in a separate container. When the amalgam rotor is replaced (see page 14), put the residues into the rotor for disposal.

ATTENTION

Amalgam residues must not be disposed of in the cuspidor or the sink!

When the surfaces of the hoses have become sticky from disinfectants, clean them with regular dish washing fluid and then rub talcum over the hoses, if necessary.

Before refitting the hoses, grease the O-rings at the connection pieces.


ATTENTION

Use only lubricants which have been approved by Sirona (see page , General Notes)!

The hoses must engage firmly.

If the unit is equipped with a 3rd suction hose, clean it the same way.

3.6 Thermodisinfecting the suction hoses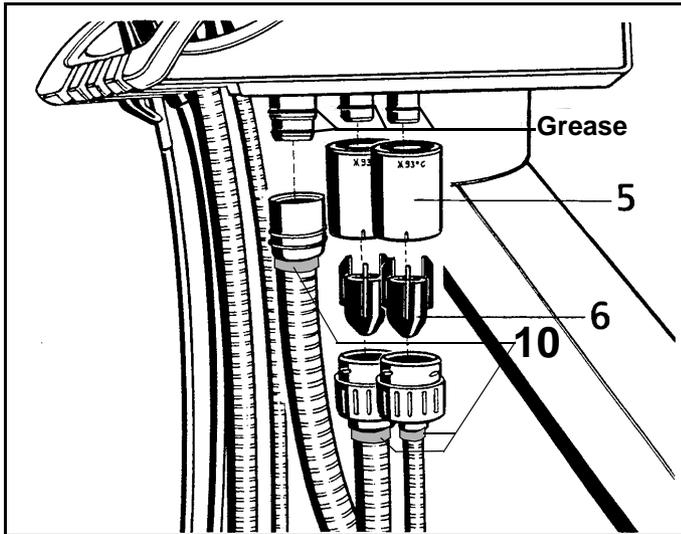

The unit is equipped as standard with suction hoses that **cannot be thermodisinfecting**.

However, it may have suction hoses that **can be thermodisinfecting** as special accessories. These are then marked in turquoise color (10).


! ATTENTION

Only hoses marked with a turquoise ring (10) can be thermodisinfecting!

First switch the treatment center OFF.

Pull off the suction hoses and the connection hose from the assistant element and the water unit.

Remove the top parts (5) of the filter housings and the amalgam residue collector (6).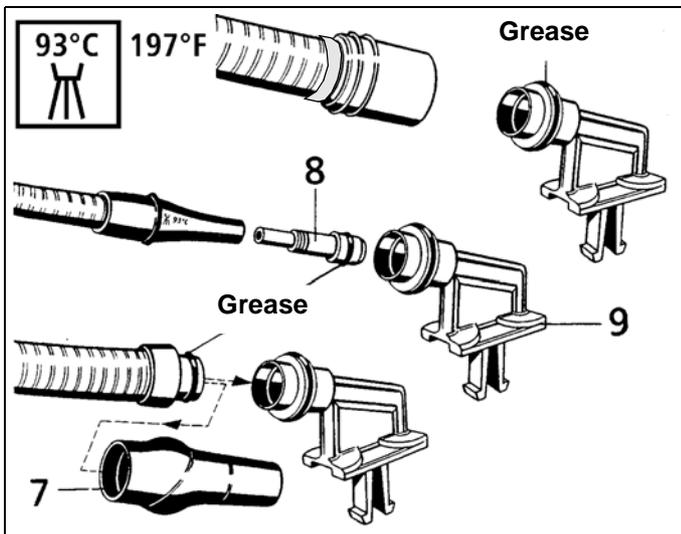


Pull off the handpiece (7) from the suction hose.

For the saliva ejector hose, insert the intermediate piece (8) in the hose holder (9).

Grease the O-rings.

! ATTENTION

Use only lubricants which have been approved by Sirona!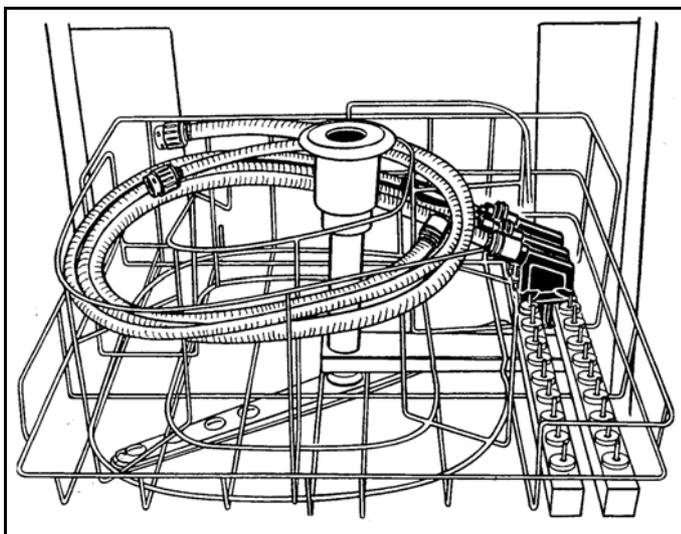

Fit the hose holders onto the rails in the thermodisinfecting (by Miele) as shown.

Connect the suction hoses to the hose holders and place them in the wire basket (by Miele) for thermodisinfection.



Thermodisinfect at max. 93 °C.

Before reconnecting the hoses to the assistant element, lubricate the O-rings at the connection pieces .

The hoses must engage firmly.

Hose holder for
Miele thermodisinfecting
Order No. 89 18 757

4 Amalgam separation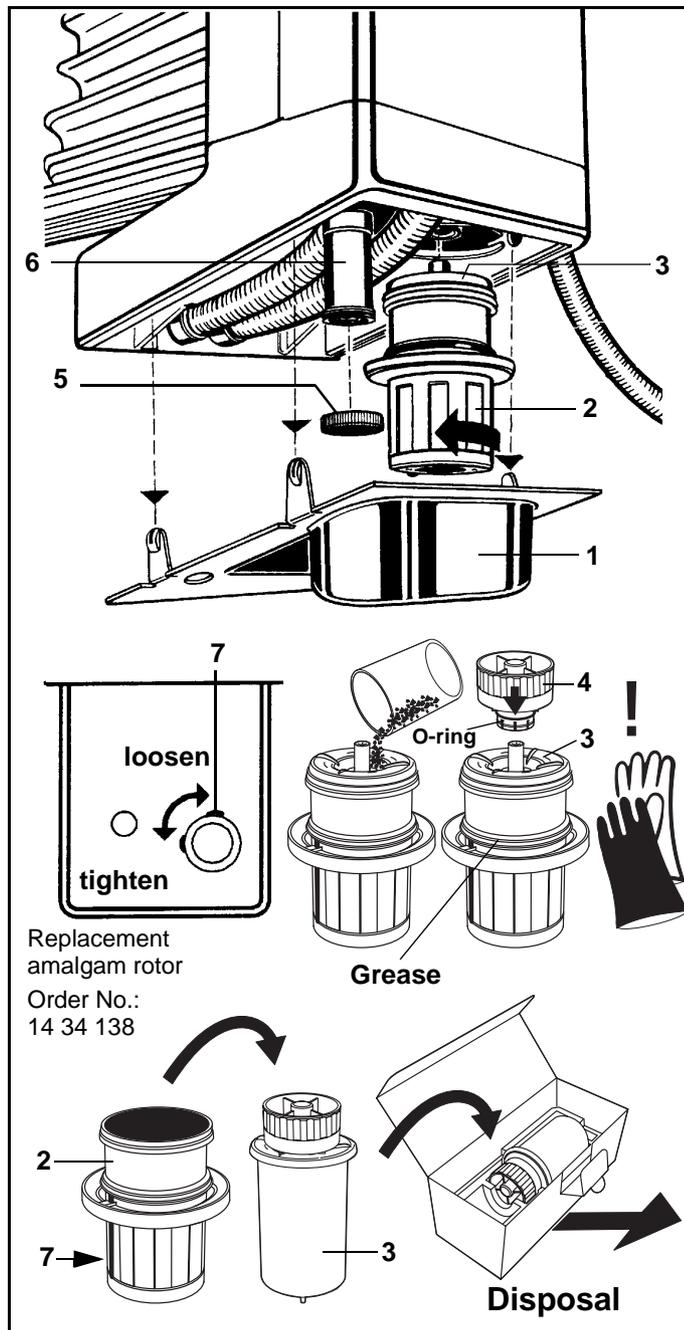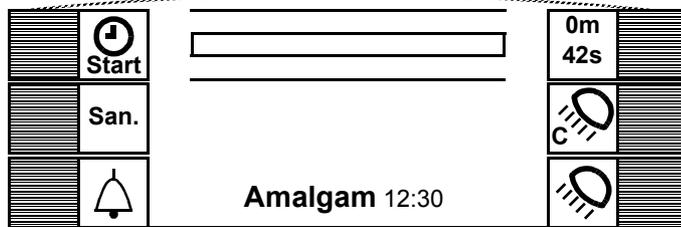

When the message **Amalgam** lights up on the display of the dentist element, proceed as follows:

- **Disinfect** as described in chapter 3 “**Vacuum system**”.
- **Pull the lower cover (1)** off from the water unit.
DO NOT turn the unit off!

- **Turn the lower part (2) of the amalgam separator with rotor (3)** in the direction of the arrow until it can be removed easily.
The message **Amalgam flashes** on the displays.
- Put any amalgam residues from the suction hoses and the cuspidor into the dismantled rotor for disposal (see pages 7 and 12).
- **Hold the bottom part upright and click on the transport cap (4)**. This releases a binding agent. Therefore, **do not remove the transport cap again!**
- Remove the **rotor and cap** from the lower part and place it in the special packing for disposal.

(Send it to one of the companies listed on the insert enclosed with the new amalgam rotor.)

- Insert the **new rotor (3)** in the lower part (2).
Do not install a used rotor!
- Reinstall the lower part and turn it in the opposite direction of the arrow **up to the stop**. The bar (7) should point to the chair at the start and to the siphon (6) after it is screwed tight.
If after insertion of the rotor Amalgam still flashes on the displays, then the lower part is not correctly locked in place!
- Refit the **cover (1)**.

NOTE: Independently of the **AMALGAM** message lighting up, the rotor (3) in the amalgam separator must be replaced at least **once per year**, otherwise unpleasant odors can develop.

Siphon

Clean the siphon (6) in case of poor drainage of the cuspidor, otherwise **once a month**.

To do so, first open the cover (1) and unscrew the cap (5).

Have a collector ready to catch the contents!


Reinstall the cap and cover.

i NOTE

Always wear gloves when performing these tasks!

5 Instrument hoses

5.1 Removal/replacement of instrument hoses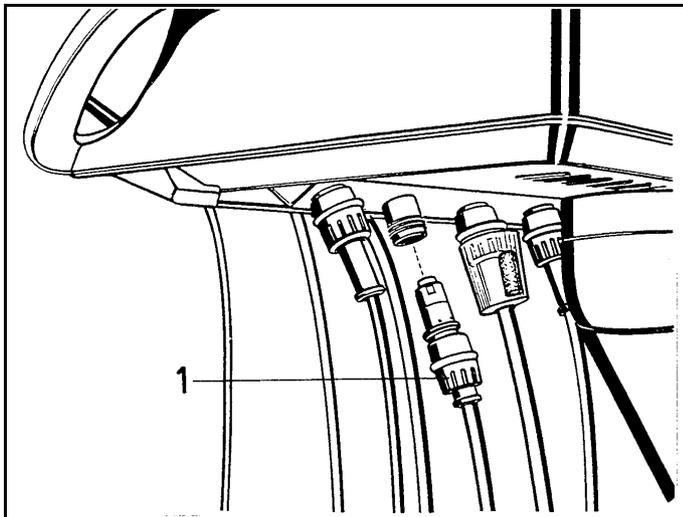

First switch the unit OFF!


Unscrew the cap nut (1) and then simply pull off the instrument supply hoses.

i NOTE

Only SIROTOM, SIROSONIC L, POLYLUX or SIROCAM 3 or SIROCAM C may be connected to the right coupling piece.

Only SIROSONIC L or a 4th drill drive (only with expansion set) may be connected to the second coupling piece from the right.

5.2 Turbine hose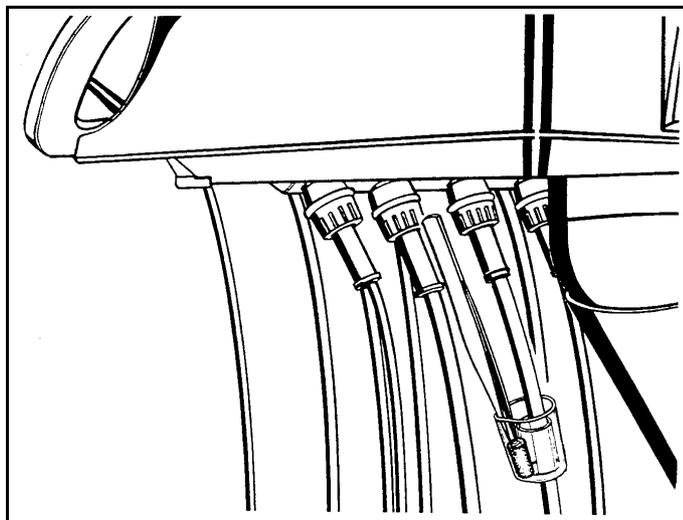

At the unit end of the turbine hose, return air leaks out with a small amount of turbine oil. This oil is absorbed by the cotton wool roll in the transparent collecting cup.


We recommend replacing the cotton wool roll **every week**.

To do so, push the cup down and remove the cotton wool roll.

Cut off and insert a new cotton wool roll.
Push the cup back into place.

6 Disinfection system

6.1 Microbiological water test

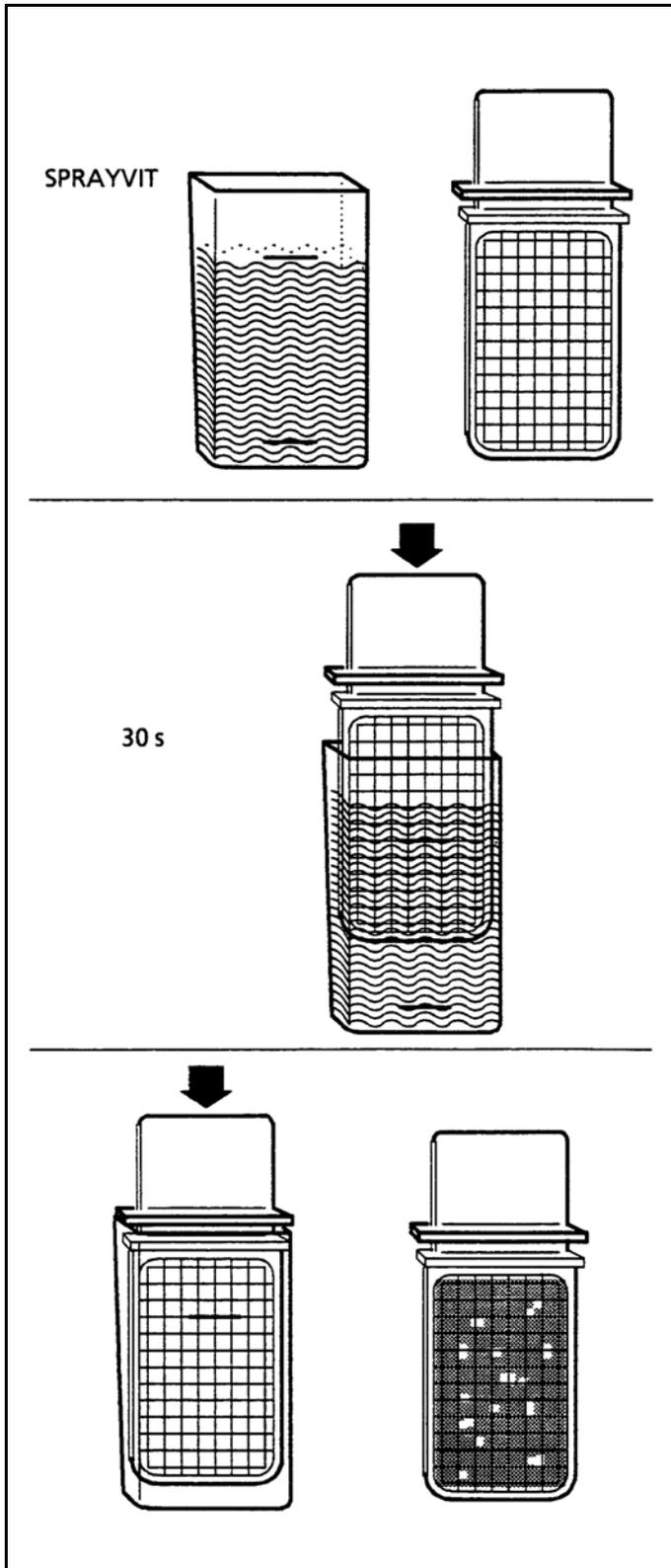Must be performed after a working pause of more than one week.

Use a "Total Count Tester" for this test. The first three testers will be supplied free of charge if you return the attached coupon.

Please indicate your address and the serial number of the chair (see "Installation Report / Warranty Passport"). Additional testers can be ordered under Order No. 58 53 775.

ATTENTION

The "Total Count Tester" has a maximum durability of 1 year after receipt.

The cardboard disk contains a dehydrated nutrient medium. It is activated by the sample and serves as culture medium for a number of bacteria. The number of germs provides information about the hygienic condition of the water.

Performing the test

- Allow water to run out of the SPRAYVIT syringe for about 1 minute.
- Then take a water sample from this syringe. Fill in water up to the upper marking of the container. To neutralize the disinfecting agent of the water sample, add approx. 1.5 g of fixing salt (sodium thiosulfate). Do not touch the inside of the container and the part of the tester to be immersed.
- Immerse the tester in the filled container for 30 seconds. The cardboard disk with the nutrient medium will now absorb 1 ml of the water sample.
- Remove the tester and shake off excessive water. Empty the container entirely.
- Place the tester into the empty container for incubation: Keep for either two days at room temperature or for 24 hours at a temperature of 35°C.
- Then count all germs found on the surface of the tester.

If the number of germs significantly exceeds 100, then sanitation is required (see next page).


6.2 Sanitation

Special measures for additional devices

Special devices that are connected to the chair via the media block are treated with disinfected water with a higher hydrogen peroxide concentration (1.4%) during sanitation.

ATTENTION

If the connected devices are not suitable for this higher sanitation concentration, Sirona recommends to disconnect them from the treatment center during sanitation and to perform a separate sanitation in accordance with the manufacturer's specifications.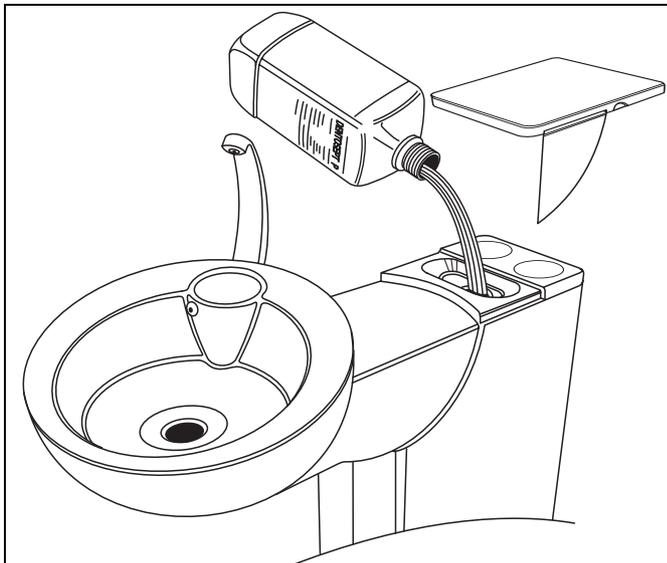

Preparation of sanitation

Perform sanitation every 4 weeks, or if the germination index significantly exceeds 100 germs per ml (see section "Microbiological water test"). Only in this way can you surely and effectively combat the formation of biofilm in the water paths.

- It takes approx. 10 minutes to prepare sanitation.
- If **Desinf.** flashes on the display of the dentist element, **DENTOSEPT P** must be filled in first (see Operating Instructions).

ATTENTION




In Japan the disinfectant OXYDOL available there must be used instead of DENTOSEPT P. OXYDOL must be mixed 1:1 with distilled water before it is filled into the disinfection system.

The flashing **Desinf** message then disappears.

DENTOSEPT P

1 carton = 6x1 liter, Order No. 33 18 156

- **Maximum water flow** must be set on all drill drives as well as on the SIROSONIC L. Check this before every sanitation.
- All instruments and suction hoses must then be placed in their receptacles.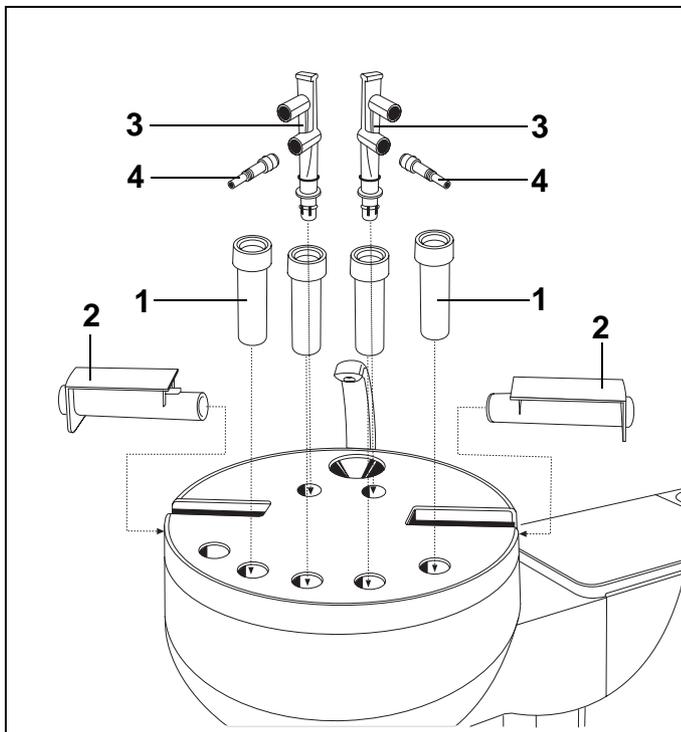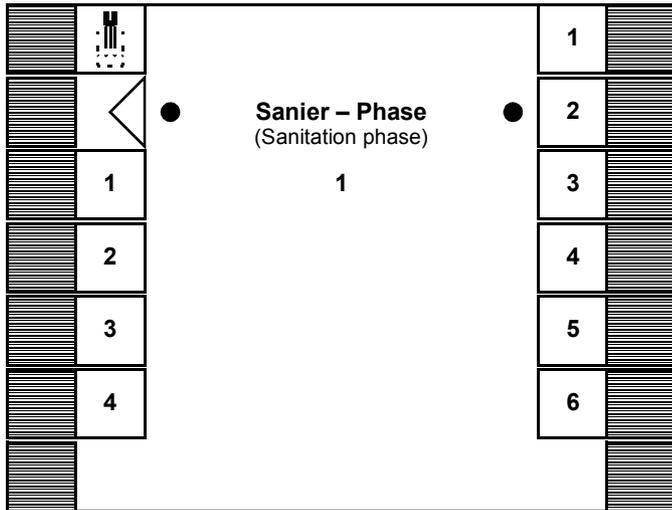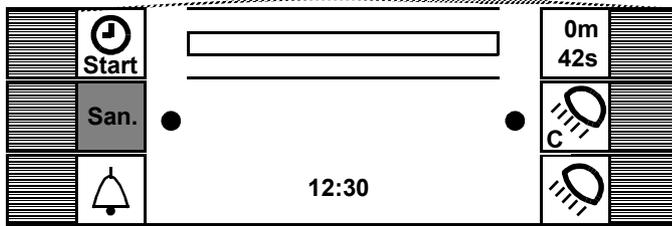

Activating the sanitation

- In the **Chair program dialog** select the **assistant dialog**.
- In the assistant dialog, activate the **San.** key for approx. 3 seconds.

Activation is only possible if there is sufficient **DENTOSEPT P** in the refill container and if all instruments are fitted firmly in their instrument receptacles.

If necessary refill **DENTOSEPT P** and check the correct fit of the instruments.

The **sanitation dialog** appears.

Sanitation phase 1

During the sanitation procedure, the displays indicate **sanitation phases 1 to 6**.

The left key row identifies the receptacles on the assistant element.

The right key row identifies the receptacles on the dentist element.

Preparation of the sanitation hood

- Place the sanitation hood with lid on the cuspidor. The sanitation hood vessel must always be clean!
- Insert the cleaning adapters for the instrument couplings (1), SPRAYVIT valve bodies (2) and suction hoses (3) up to the stop in the sanitation hood, as shown.

The adapters for the instrument couplings are color-coded to match the hose couplings.

SL motor: **green**

Electric motors with interface acc. to ISO 3964 such as

EL 1 motor: **blue**


Turbine: **white**

SIROSONIC L: **red**

- For the saliva ejector hose, insert the intermediate piece (4) in the cleaning adapter.

i NOTE

To prevent the hoses from twisting and stretching, the arrangement of the cleaning adapters from left to right should correspond to the arrangement of the instruments on the dentist element.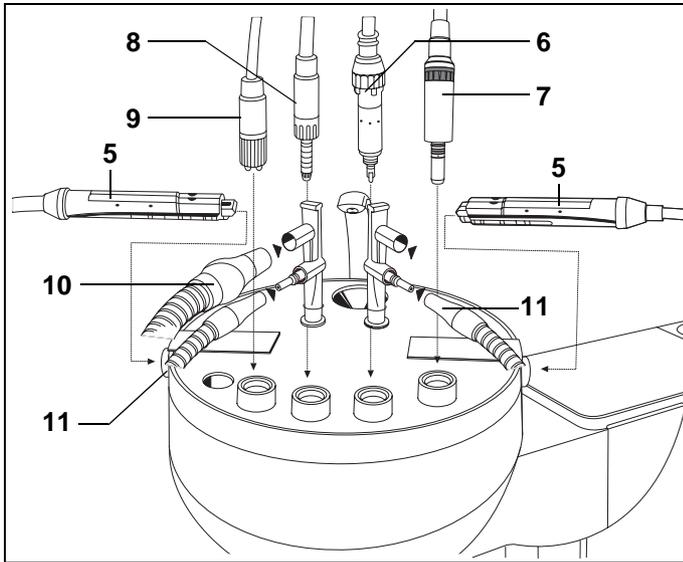


The water paths must always be sanitized without the dental instruments attached!

- Pull the dental instruments off the couplings and the SPRAYVIT sleeves off the valve bodies.
- Insert the SPRAYVIT hoses with valve bodies (5) from the dentist element and assistant element up to the stop in the adapters at the right and left (valve lever facing up, lock-in button facing down).
- Snap into place or insert up to the stop the couplings of the
 - SL motors (6), electric motors with interface acc. to ISO 3964, such as
 - EL 1 motor (7),
 - turbine (8) and
 - SIROSONIC L (9) in the corresponding color-coded adapters.

The color ring on the instrument hose is located on the cap nut.

- Pull off the suction nozzles. Attach the suction hose (10) and the saliva ejector hose and, if available, the 3rd suction hose (11) to the side of the cleaning adapter.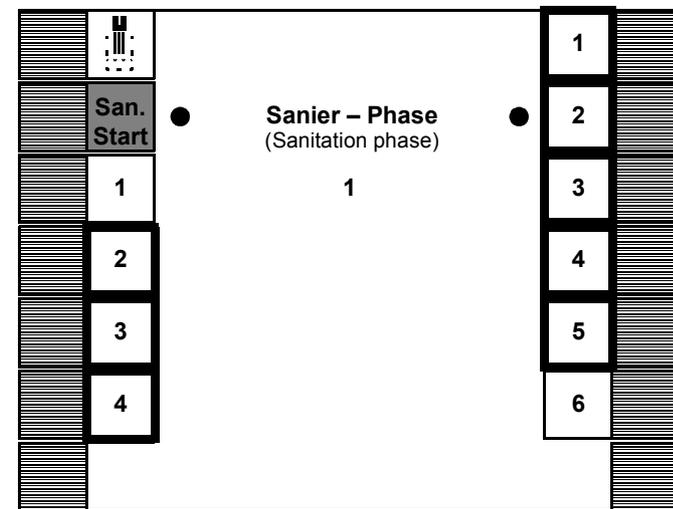

Start of sanitation


After the instruments and hoses have been removed from their receptacles, the number symbol of the corresponding receptacle is marked with a bold frame (left: assistant element, right: dentist element).

The numbers in the left and right row of numbers on the display correspond to the receptacles of the assistant and dentist element (counting the receptacles in each case from the left to the right).

All instruments which are not removed or do not carry water (POLYLUX, SIROTOM, SIROCAM) are not included in the sanitation. The corresponding number symbols appear with a normal frame.

- Activate the **San. Start** key.

San. Stop appears. The sanitation process starts.

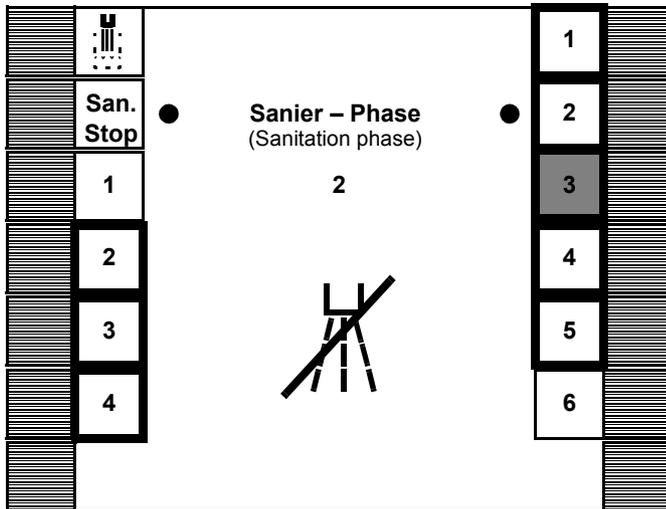Sanitation phase 2

First the program checks whether maximum water flow has been set for the instruments.

If there is no water flow, an acoustic signal sounds. The symbol of the corresponding receptacle flashes and the display shows the symbol:

(Example receptacle 3 on the dentist element).

In this case open the water regulator of the corresponding hose on the instrument coupling. Leave the relevant hose plugged into the adapter in this case. As soon as the system detects that water is flowing, sanitation is continued automatically.

i NOTE

*If you press the flashing key after the acoustic signal sounds without opening the water regulator, the sanitation program continues. **The corresponding water path is not sanitized.***


Sanitation phase 3

The water is pumped out from the water tank of the water unit via the tumbler filler.

Sanitation phase 4

The container is now automatically filled with DENTOSEPT P.

This is then used for rinsing the instrument hoses, the SPRAYVIT hoses with valve bodies and the tumbler filler. After the individual rinsing process the corresponding receptacle symbol appears dark.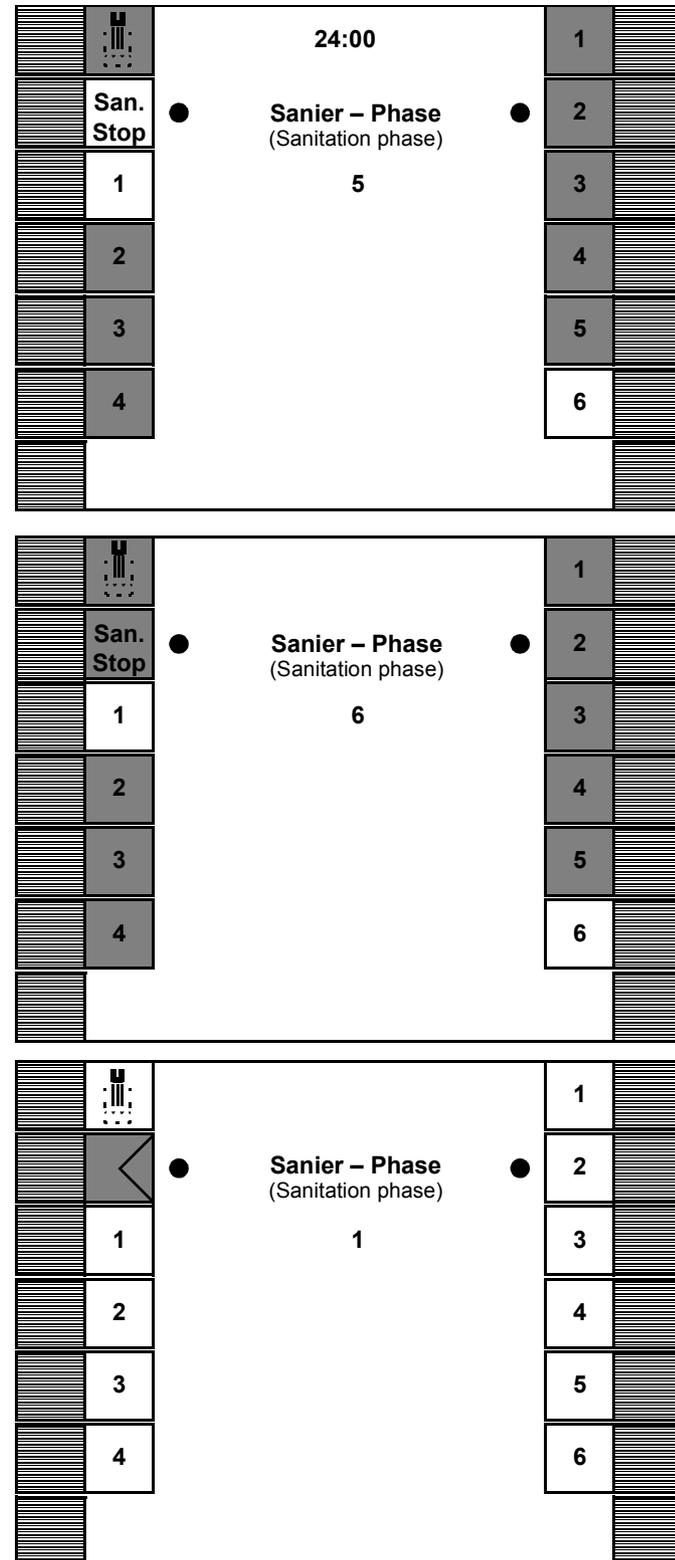

Sanitation phase 5

If all number symbols of the instruments removed for sanitation are dark, the reaction time of **24:00** hours appears on the top of the display.

- The unit must now be switched **OFF** for **at least 24 hours**, but **not longer than three days**.
Leave all hoses plugged unchanged in the adapters of the sanitation hood.
- Switch the unit back **ON** after a **minimum of 24 hours**.

i NOTE

If you have blocked the water supply for the treatment center after sanitation phase 5, the water supply must be opened again prior to renewed start-up of the treatment center (i.e. after 24 hours). Switch the unit back ON after this.

Leave all hoses plugged into the adapters of the sanitation hood. The sanitation program then automatically runs to its completion.

If you switch on the treatment center before the 24 hours have elapsed (the missing time is then indicated on the display), you must activate the "San. Stop" key after switching on the unit. The sanitation program then runs to its completion.


Sanitation phase 6

The hoses and the tumbler filler are rinsed with water. This takes several minutes. The receptacle symbols are again marked with a bold frame after rinsing.

- As soon as **sanitation phase 1** appears again on the display, the hoses can be placed in the corresponding receptacles. The "Back" arrow then appears at the top left on the display.
- Now activate the "Back" arrow to leave the sanitation dialog.

The unit is now ready for operation again.




6.3 Aborting the sanitation process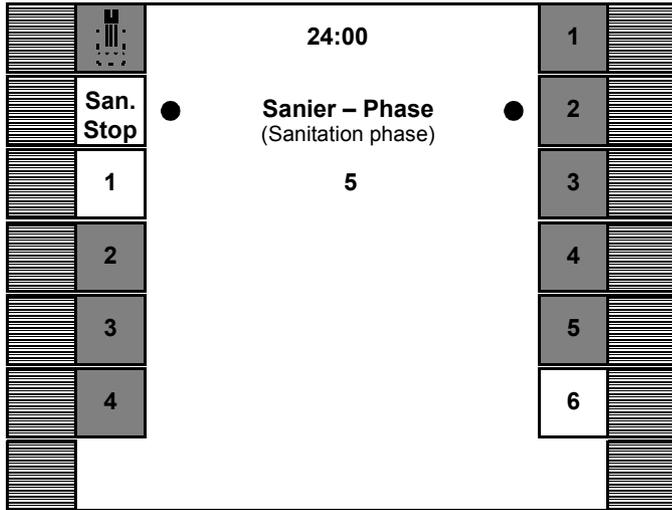

During the sanitation process the displays show **San. Stop**. With this key the sanitation can be aborted at any time.

The hoses are then rinsed with water as described above. When the rinsing process has been completed, the "Back" arrow appears on the display. After the instruments have been deposited in their receptacles, the unit is again ready for operation.

7 Surfaces / O-rings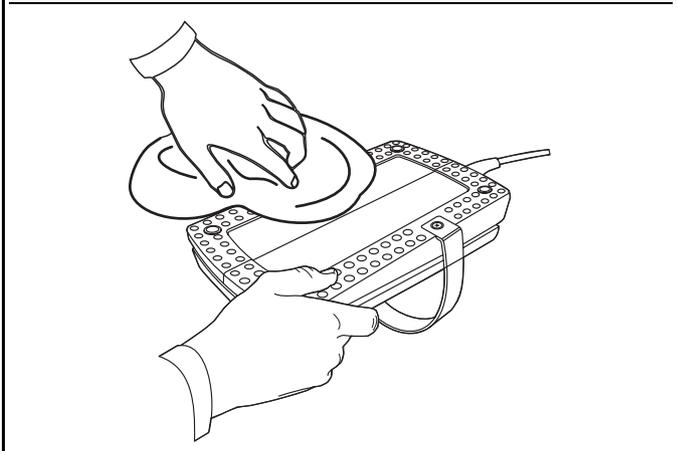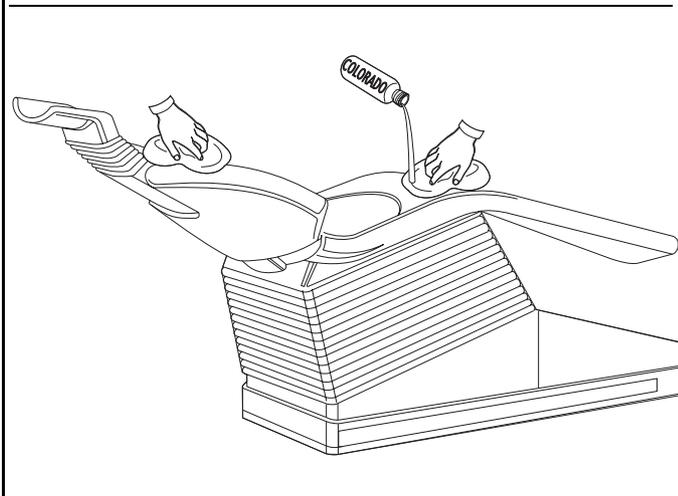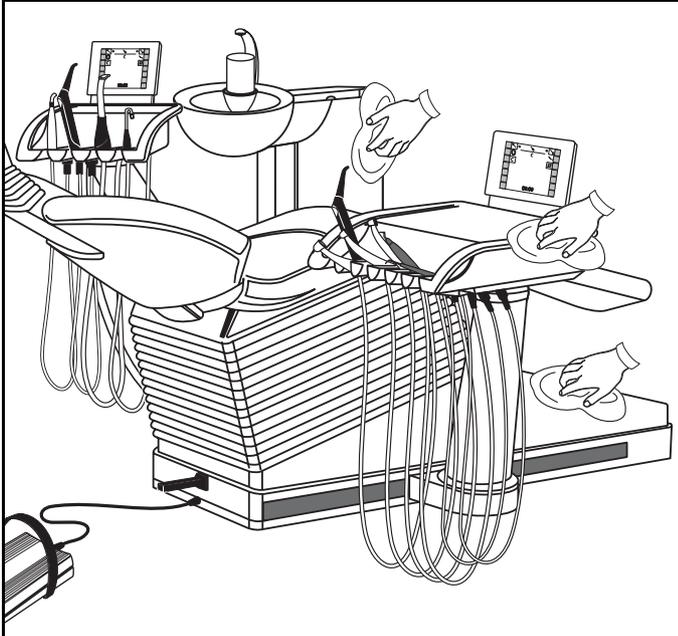

Disinfecting

The surfaces (apart from upholstery) can be sprayed and wiped with surface disinfection agents.

⚠ ATTENTION

Use only disinfecting agents which have been approved by Sirona!

⚠ ATTENTION

If the keys on the displays are touched (if the system is powered on) the settings can change!

Cleaning

Remove dirt and disinfectant residues **regularly** using a mild, commercially-available cleaning agent.

Please do not use any **colored cloths**, since they may cause discoloration of the unit surfaces, e.g. in combination with disinfectants!

Do not allow liquids to penetrate into the ventilation slots!

Upholstery

Care of the synthetic leather upholstery must be performed weekly.

Cleaning:

⚠ ATTENTION

Use only cleaning agents which have been approved by Sirona!

Disinfection:

The upholstery must be disinfected with alcohol (50% ethanol/water mixture).

Care of the O-rings

- Use only approved lubricants (see page 2, General Notes)
- Do **not** use vaseline or silicone grease. They cause damage to the O-rings!


Foot switch

Clean the bottom plate using a damp cloth.

Protection from medicaments

Due to their high concentrations and the substances they contain, many medicaments can dissolve, etch, bleach or discolor surfaces.

To prevent damage, wipe off medicaments **immediately** with a damp cloth!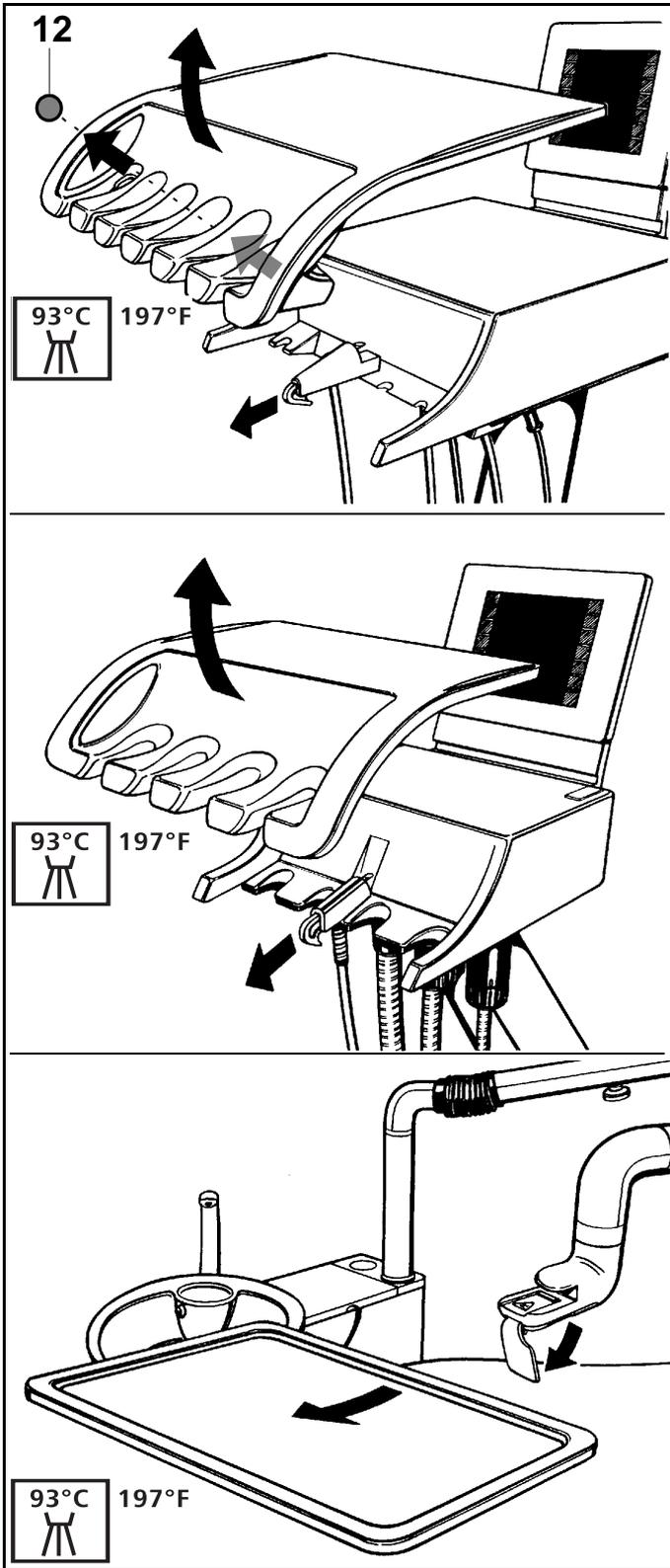

For easier cleaning or thermodisinfection at 93°C the instrument holders of the dentist and assistant element as well as the instrument tray can be removed.


! ATTENTION

The instrument holders of the dentist and assistant element must be thermodisinfectated **separately** to prevent possible discoloration.

A stopper (12) that may be in an unused instrument receptacle can simply be pushed out from the back for cleaning or thermodisinfecting.

8 POLYLUX fiber optic light handpiece

8.1 Care and cleaning of the handle sleeve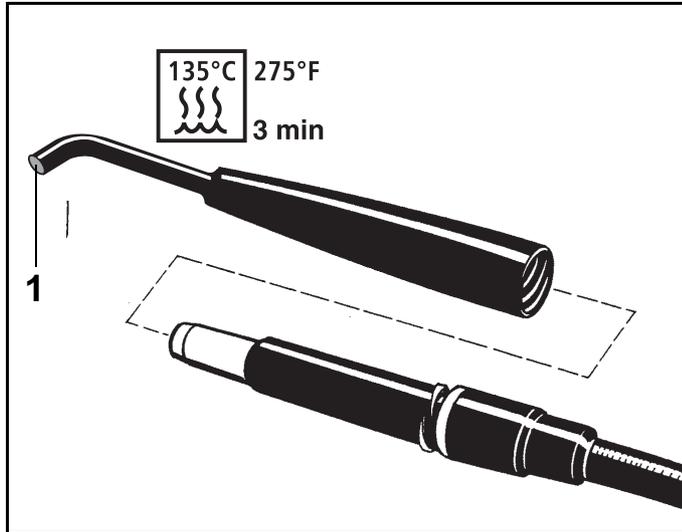

The handle sleeve with the fiber optic rod can be unscrewed (counterclockwise) from the handpiece for cleaning and sterilizing.

Sterilize only in an autoclave at 135°C, 2.1 bar.

! ATTENTION

NEVER exceed 150°C! DO NOT subject the handle sleeve to ultrasonic cleaning or chemical vapor sterilization!


Prior to sterilization

clean the light exit surface (1) with alcohol (50%).

! ATTENTION

DO NOT place the handle sleeve in disinfectant solutions!

8.2 Care and cleaning of the light guide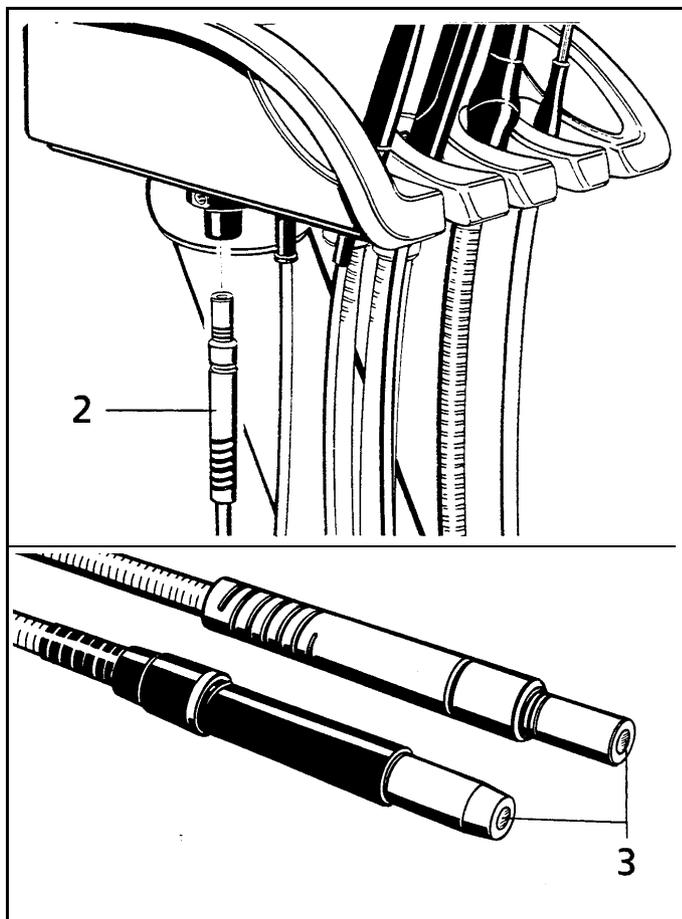

The light guide (handpiece hose) can be pulled out for cleaning and disinfecting from the lamp module under the dentist or assistant element.

! ATTENTION

Always grasp the gray anti-kink sleeve (2) of the light guide, and NEVER the hose!

Do not squeeze, twist or kink the handpiece hose!
Do not exceed a minimum bending radius of 80 mm.

Clean

with alcohol (50%) or with an aqueous detergent solution.

Clean the light entry and exit surfaces (3) with alcohol (50%).

Disinfection by wiping

is acceptable.


! ATTENTION

NEVER sterilize or thermosanitize the light guide!

DO NOT place the light guide in a liquid bath or subject it to ultrasonic cleaning!

9 SIROCAM 3 / SIROCAM C video camera and monitor

9.1 Care and cleaning of the SIROCAM 3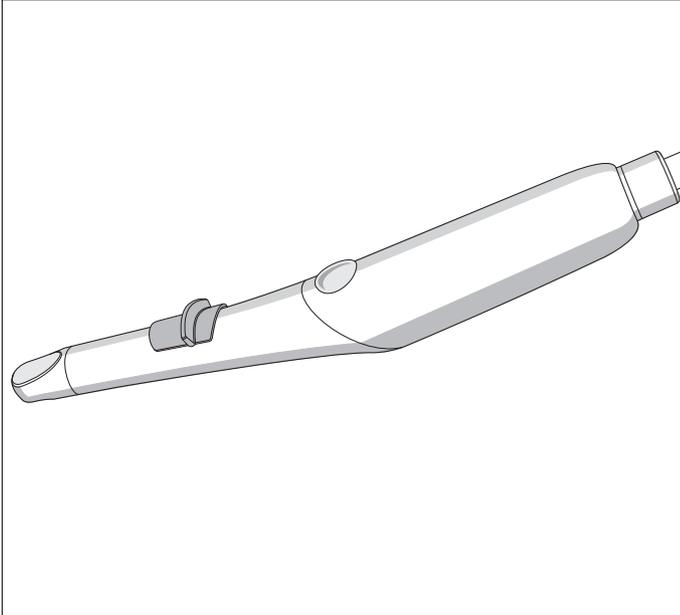

The ergonomically shaped titanium sleeve of the camera has few joints. Its shape takes hygienic requirements into account and has no places difficult to reach.

Clean

Remove dirt and disinfectant residues **regularly** using a mild, commercially-available cleaning agent.

Please do not use any colored cloths, since they may cause discoloration of the surfaces, e.g. in combination with disinfectants!

Disinfecting


The camera can be disinfected by wiping.

ATTENTION

Use only cleaning agents which have been approved by Sirona!

Wipe off SIROCAM 3 with a soft cloth.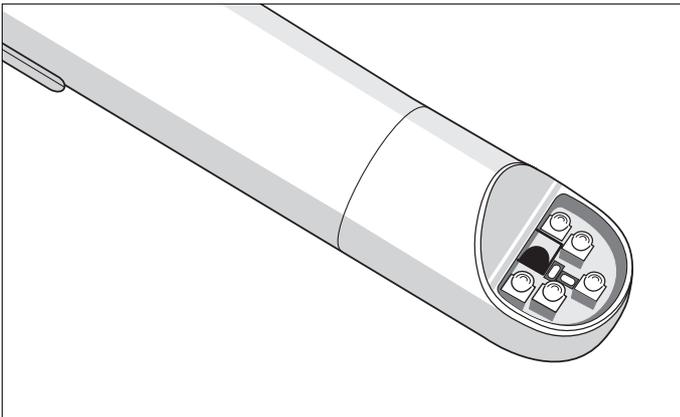

Protect the lens window against scratching and clean it with a lint-free soft cloth.


ATTENTION

The camera handpiece with the cable CANNOT be sterilized or thermoisinfected!

The camera MUST NOT be sprayed or dipped in liquids!

9.2 Care and cleaning of the SIROCAM C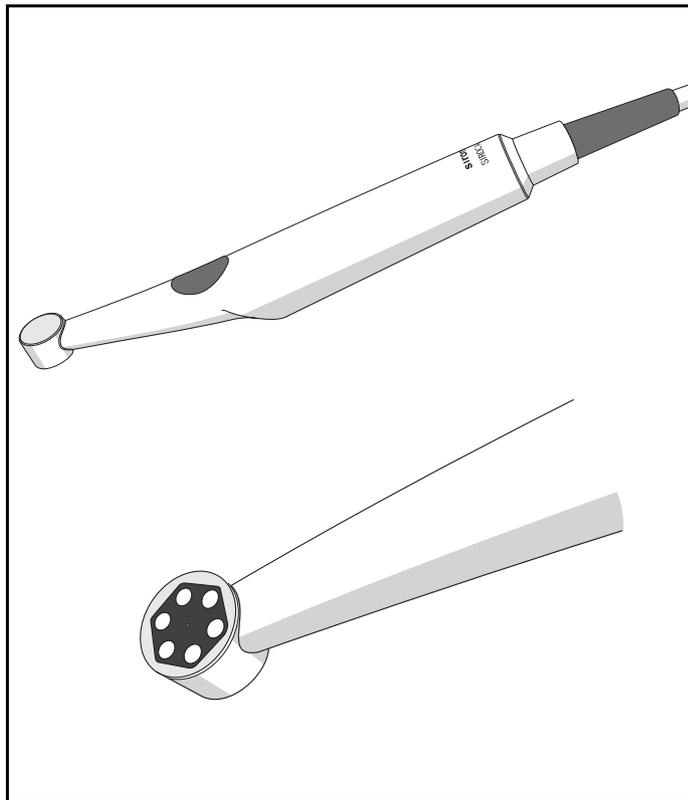

Cleaning

Remove dirt and disinfectant residues **regularly** using a mild, commercially-available cleaning agent.

Please do not use any colored cloths, since they may cause discoloration of the surfaces, e.g. in combination with disinfectants!

Disinfecting

The camera can be disinfected by wiping.

ATTENTION

Use only cleaning agents which have been approved by Sirona!


Wipe off SIROCAM C with a soft cloth.

ATTENTION

*The camera handpiece with the cable CANNOT be sterilized or thermosdisinfected!
The camera MUST NOT be sprayed or dipped in alcohol!*

9.3 Care and cleaning of the monitor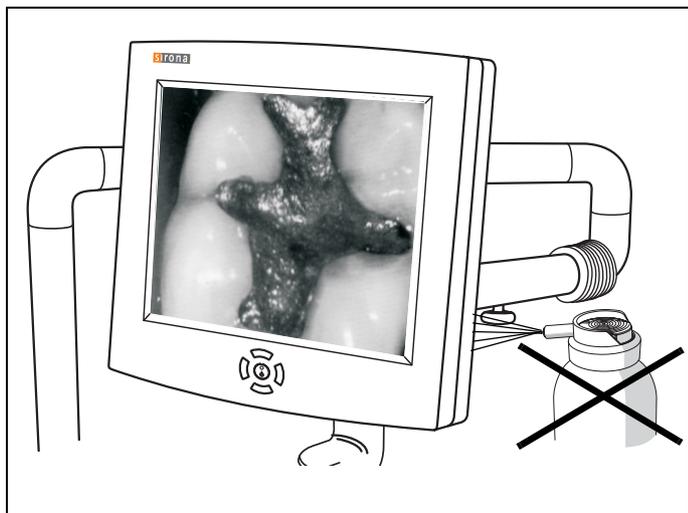

Cleaning

Remove any dirt and disinfectant residues **regularly** using a mild commercial cleaning agent.

Do not use any colored cloths for cleaning, since they may cause discoloration of the surfaces, e.g. in combination with disinfectants!

Disinfecting

The glass screen and housing of the monitor can be disinfected by wiping them with a soft cloth.

ATTENTION


Use only cleaning agents which have been approved by Sirona !

ATTENTION

Never spray the monitor with a disinfecting or cleaning agent!

10 Checking the safety switches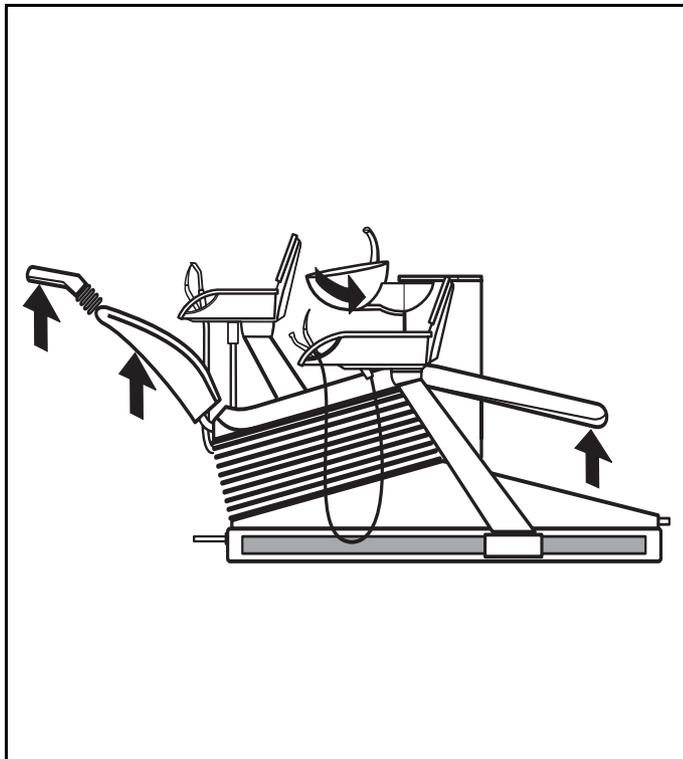

To ensure the safety of the practice team and the patients, please check the safety switches installed in the unit **every 6 months**.

Move the chair or the backrest down for this check.

- **Raise the leg support:**
The chair movement stops, then there is a corrective movement upwards for around 3sec.
- **Raise the backrest:**
The backrest movement stops, then there is a corrective movement upwards for around 3sec.
- **Raise the motor-adjustable headrest:**
The backrest movement stops, then there is a corrective movement upwards for around 3sec.
- **Swing in the swiveling cuspidor (optional):**
The chair movement stops.

ATTENTION

If one of the safety switches does not stop the chair movement or the backrest movement, the treatment center must not be used for dental work!

Please contact your service engineer.

We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH 2001
Sirona Dental Systems GmbH