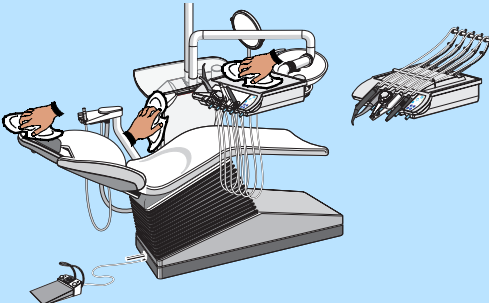

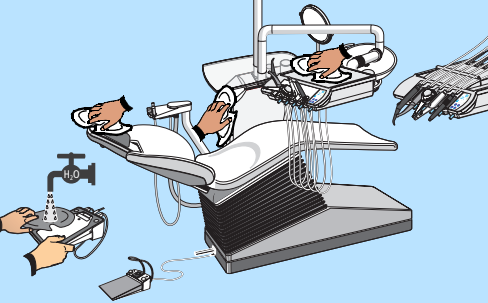

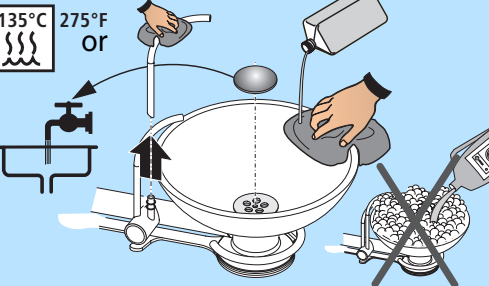

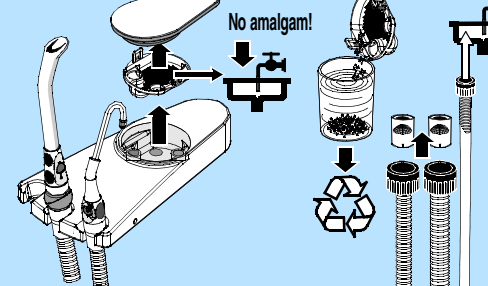

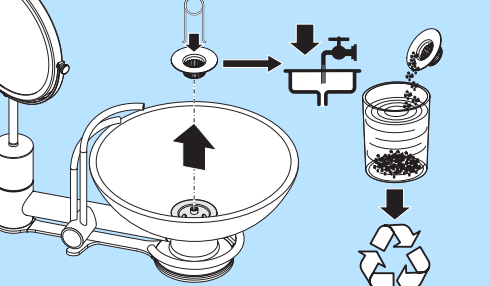
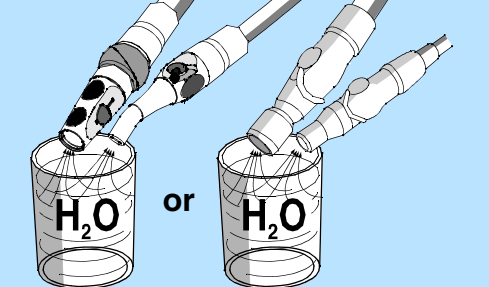
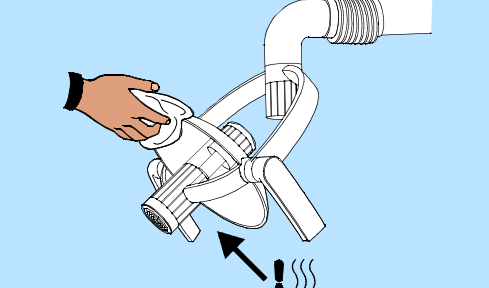

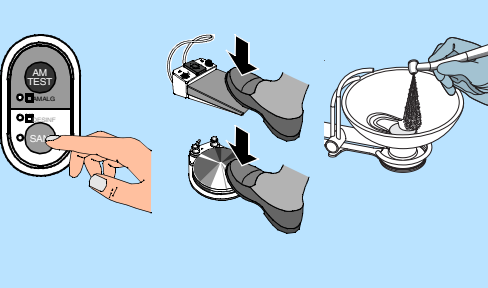
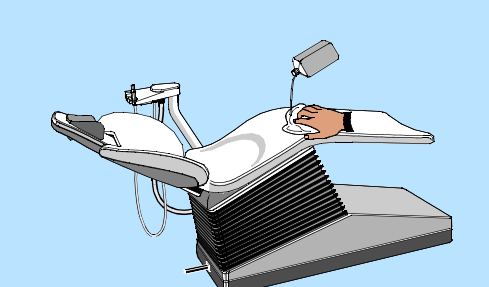
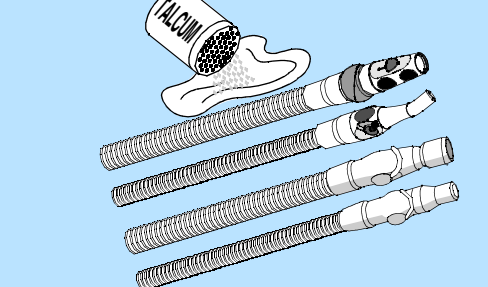
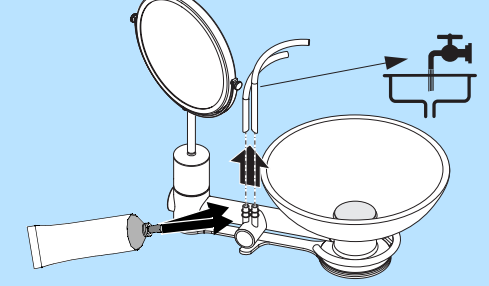
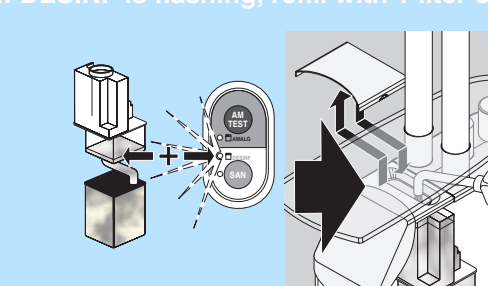
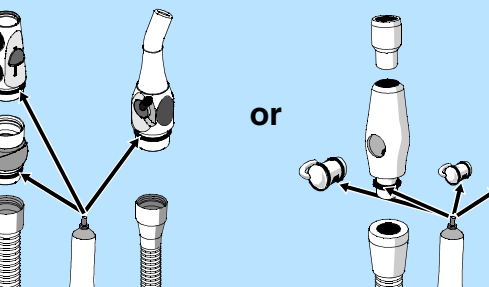
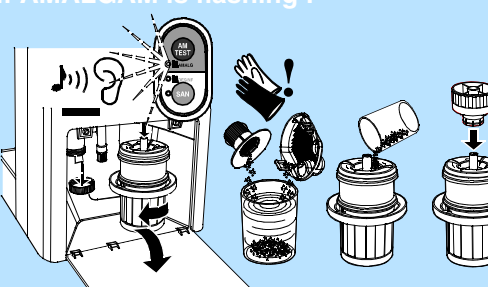


After each patient		Daily after consulting hours	
	<p>Disinfect surfaces and clean upholstery.</p> <p> Spray/ wipe disinfection</p> <p>CAUTION</p> <p>Use only cleaning agents approved by SIRONA ! See document on Cleaning and care agents - 59 70 905.</p>		<p>Disinfect surfaces and clean upholstery.</p> <p> Spray/ wipe disinfection</p> <p>CAUTION</p> <p>Use only cleaning agents approved by SIRONA ! See document on Cleaning and care agents - 59 70 905.</p>
	<p>Clean the cuspidor.</p> <p> Spray/ wipe disinfection</p> <p>CAUTION</p> <p>Use only cleaning agents approved by SIRONA ! See document on Cleaning and care agents - 59 70 905.</p> <p>Do NOT use household detergents or acidic agents!</p>		<p>Cleaning the filter</p> <p> Recycling</p>
	<p>Remove coarse drain filter and clean (al amalgam separator option).</p>		
	<p>Aspirate a large glass of water especially:</p> <ul style="list-style-type: none">• after interventions during which the patient loses blood• after using H₂O₂• and at least every 60 min. during prolonged treatments		

Weekly		Monthly or as required	
	<p>SIROLUX FANTASTIC</p> <p> Spray/ wipe disinfection</p> <p>CAUTION</p> <p>Use only cleaning agents approved by SIRONA ! See document on Cleaning and care agents - 59 70 905.</p>		<p>Sanitation*</p>
	<p>Upholstery</p> <p>The upholstery must be disinfected with alcohol (50% ethanol/water mixture).</p> <p>CAUTION</p> <p>Use only cleaning agents approved by SIRONA ! See document on Cleaning and care agents - 59 70 905.</p>		<p>Suction handpieces</p>
	<p>Pull off tumbler inlet and fountain and clean (if needed, disinfect).</p> <p>CAUTION</p> <p>Use only cleaning agents approved by SIRONA ! See document on Cleaning and care agents - 59 70 905.</p>		<p>If DESINF is flashing, refill with 1 liter of DENTOSEPT P !*</p>
	<p>Separate suction handpieces at joint; then clean and grease points of separation.</p> <p>CAUTION</p> <p>Use only cleaning agents approved by SIRONA ! See document on Cleaning and care agents - 59 70 905.</p>		<p>If AMALGAM is flashing !*</p> <p>at least 1x annually</p>