Installationsanleitung

Installation Manual

Manual de instalación

KARL STORZ AIDA® compact NEO

KARL STORZ AIDA® compact NEO

KARL STORZ AIDA® compact NEO

I. Allgemeines

KARL STORZ AIDA® compact NEO

I. Allgemeines

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KARL STORZ AIDA® compact NEO

I. General

Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern, high-quality piece of equipment from KARL STORZ.

KARL STORZ instruments and equipment are for use only by qualified medical personnel who are trained in their use. All electrical installations at the location of use should meet applicable national and local electrical codes.

Refer servicing to duly authorized KARL STORZ service personnel.

Always use genuine replacement parts from KARL STORZ. To determine which replacement parts are required please refer to the enclosed replacement parts list. Repair and calibration of this device requires special tools and gauges; certain internal adjustments must not be altered.

For further information, please consult this manual or contact:

KARL STORZ GmbH & Co. KG
Mittelstraße 8, D-78532 Tuttlingen
Postfach 230, D-78503 Tuttlingen
Germany

Phone: +49 7461 708-0
Fax: +49 7461 708-105
E-Mail: info@karlstorz.de
Web: www.karlstorz.com

Warranty

All KARL STORZ instruments and equipment are warranted to be free from defects in workmanship and materials for two (2) years from date of sale, unless otherwise specified; any instruments or equipment with such defects during the applicable warranty period will be promptly repaired or replaced at no charge to the customer. KARL STORZ shall not be liable, expressly or implicitly, for:

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I. General

KARL STORZ AIDA® compact NEO

- Any damages which might arise or be caused, whether by the customer or by any of the users of the instrument or equipment, as a result of:
 - misuse, mishandling, and/or improper operation,
 - repairs, modifications and/or alterations performed other than by KARL STORZ or a KARL STORZ authorized repair facility, or
 - use in combination with adapters and/or equipment, or use in any manner or medical procedure, other than those for which it is designed; and
- Particular damage, errors in data transfer such as data loss, damage or alteration, direct and/or consequential damage, regardless of nature and cause, which arise from the sale or use of the instrument and/or unit.

THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS, IMPLIED, AND/OR STATUTORY, INCLUDING, BUT NOT LIMITED TO, WARRANTIES OF MERCHANTABILITY, FITNESS, AND/OR OF SUITABILITY FOR A PARTICULAR PURPOSE, AND OF ALL OTHER OBLIGATIONS OR LIABILITIES ON KARL STORZ' PART.

KARL STORZ neither assumes nor authorizes any person to assume for it any other liabilities in connection with the sale of said instrument and equipment. To insure proper use, handling, and care of instruments and equipment, consult the applicable product literature, catalog, brochure, instruction manual, teaching film and other materials which are included with the product and/or otherwise available from KARL STORZ, at no charge, upon request.

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KARL STORZ AIDA® compact NEO

Maintenance and Repair

KARL STORZ recommends that all equipment be checked and inspected once a year by KARL STORZ, or by an authorized agent. All services such as modifications, repairs, calibrations, and/or readjustments may only be performed by KARL STORZ or by an authorized agent.

CAUTION:

Repairs may only be performed by qualified technicians trained in electrical or electronic engineering, in compliance with the relevant occupational, safety and accident prevention regulations.

Always unplug the equipment before performing any repairs.

Safety Testing based on DIN EN 62353, whichever may apply, must be performed after servicing has been completed.

By making the enclosed technical information available, KARL STORZ does not authorize any service or repair by unauthorized service personnel. Tampering with the instruments or equipment, or unauthorized service or repair of the unit nullifies and voids the warranty.

Reservation of Rights

This documentation is the sole and exclusive property of KARL STORZ and may neither be copied nor passed on to third parties without the express written authorization and approval of KARL STORZ.

KARL STORZ reserves the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on the part of KARL STORZ to submit prior notice thereof.

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KARL STORZ AIDA® compact NEO

Trademarks

KARL STORZ-SCB®, KARL STORZ AIDA® and SCBlink® are registered trademarks of KARL STORZ.

Microsoft® and Windows® are trademarks or registered trademarks of Microsoft Corporation.

Property rights

This product is protected in the USA by (at least one of the following) US Patent No(s) 5,788,688; 6,397,286; 6,484,221; 6,824,539.

- © All product pictures, photos and descriptions and texts are the intellectual property of KARL STORZ GmbH & Co. KG. Utilization and duplication by third parties have to be expressly authorized by KARL STORZ GmbH & Co. KG. All rights reserved.

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Software User Rights

1. With respect to KARL STORZ products containing software components, the Customer has a non-exclusive License, which is limited by the meaning of the following terms and conditions, to use the programmed logic, computer programs and/or software supplied by KARL STORZ and incorporated in KARL STORZ products ("Software").

2. This License applies for such products sold to the Customer only to the form delivered to Customer and for the sole purpose of operating such products in accordance with written instructions by KARL STORZ. Use of the Software in connection with other products bought by the Customer from KARL STORZ or from third parties is prohibited.

3. A transfer of this License by the Customer is permissible only if the Customer provides the buyer's name to KARL STORZ, and said buyer agrees to comply with the terms and conditions regarding the usage of the software as set forth in this License. In the case of a transfer, the Customer's License shall terminate.

4. All copyrights to the Software, as well as all modifications, enhancements, and upgrades thereto, will remain the property of KARL STORZ. In particular, the Customer may not duplicate, copy, reverse-engineer, decompile, or disassemble the Software or in any way modify the same. The Customer has no right to create derivatives of the software, and the Customer may not copy, decipher or re-create the source code of the Software. § 69g paragraph 2 of the Copyright remains unaffected by this. [This refers to German Copyright Law.]

5. Any modification or enhancement of the Software by the Customer in violation of this Agreement is considered a violation of this License; KARL STORZ immediately shall acquire all property rights and copyrights to such modifications or enhancements. The Customer hereby in advance transfers an exclusive, free, and unlimited license to use such modifications and enhancements to KARL STORZ.

6. KARL STORZ does not recommend use of the Software in connection with other software not supplied by KARL STORZ for the respective products. KARL STORZ is not liable for software malfunctions resulting from the use of software not supplied or recommended by KARL STORZ in connection with products supplied by KARL STORZ.

7. In the event of a failure of the Customer, or its agents, employees or representatives, to comply with any terms and conditions of the License herein granted, the License will, without termination or any further action by KARL STORZ or any other party, immediately terminate.

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KARL STORZ AIDA® compact NEO

1 Equipment

1.1 Basic equipment for KARL STORZ AIDA® compact NEO

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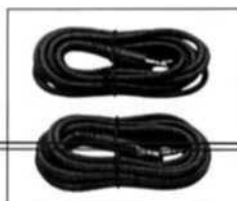
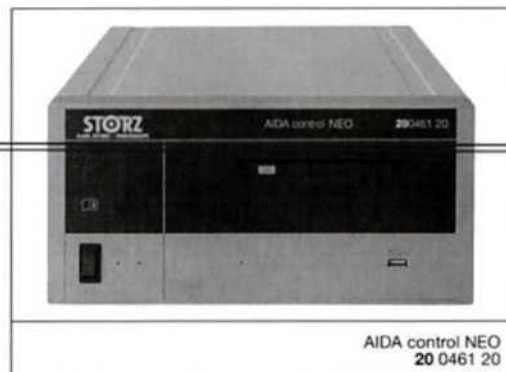
AIDA compact NEO software
20 040402-20 /
with Communication 20 040402-21

Connecting cable
20 2210 70

AIDA control NEO
20 0461 20

KARL STORZ USB stick
20 0402 75

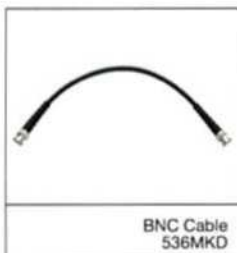
PS/2 compact keyboard
20 0902 34 /B/F/I/P/R/S/U/DK/PL

PS/2 mouse
20 0902 33

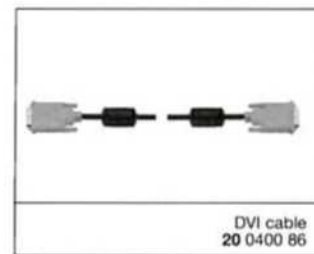
AIDA compact NEO
instructions folder 96206169D

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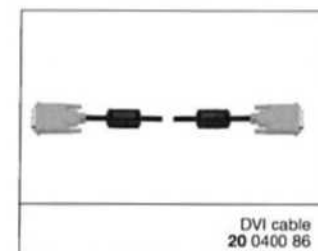
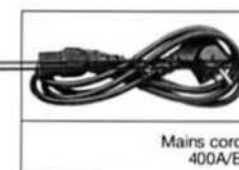
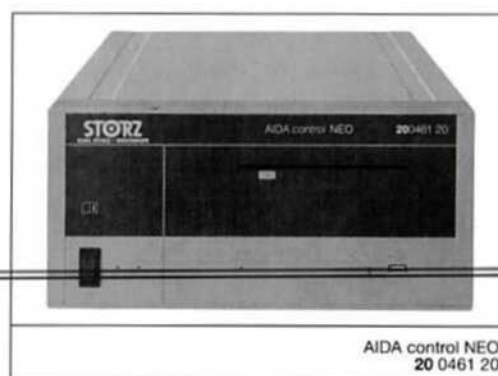
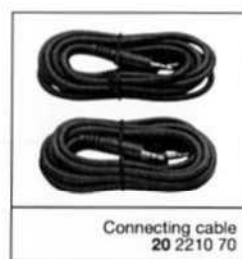
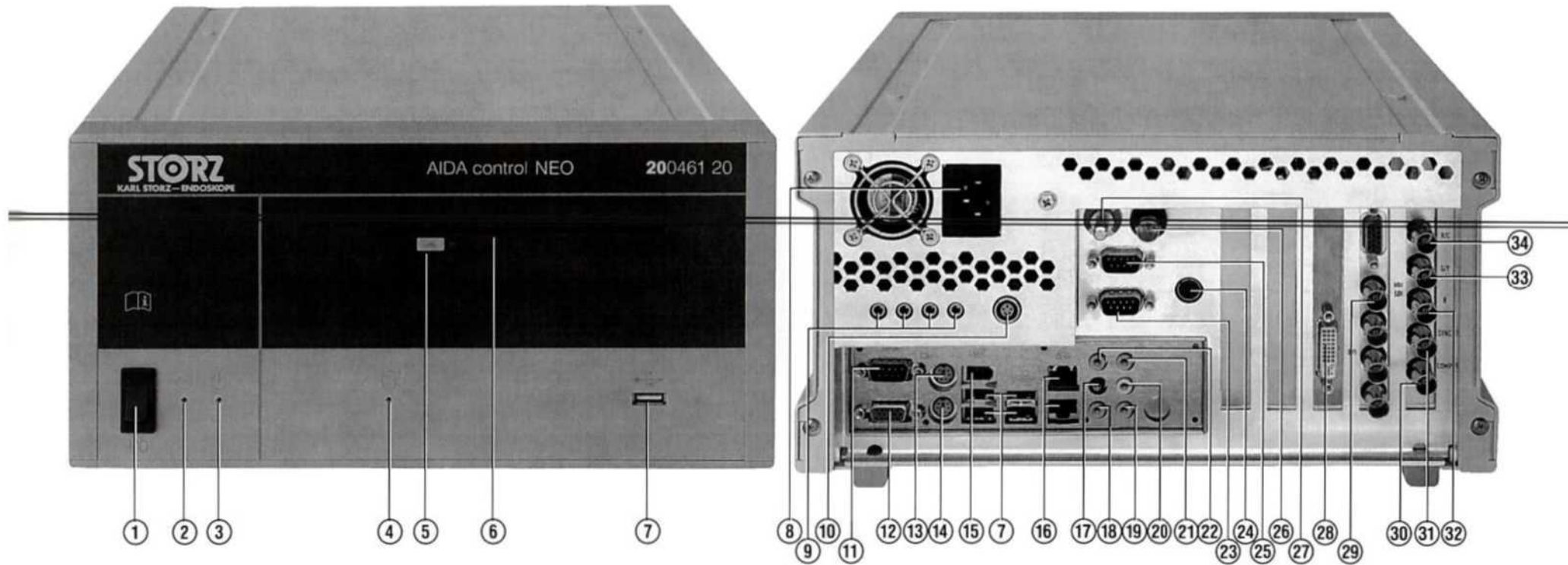
1.2 Photographs of the unit, controls, connectors and their uses

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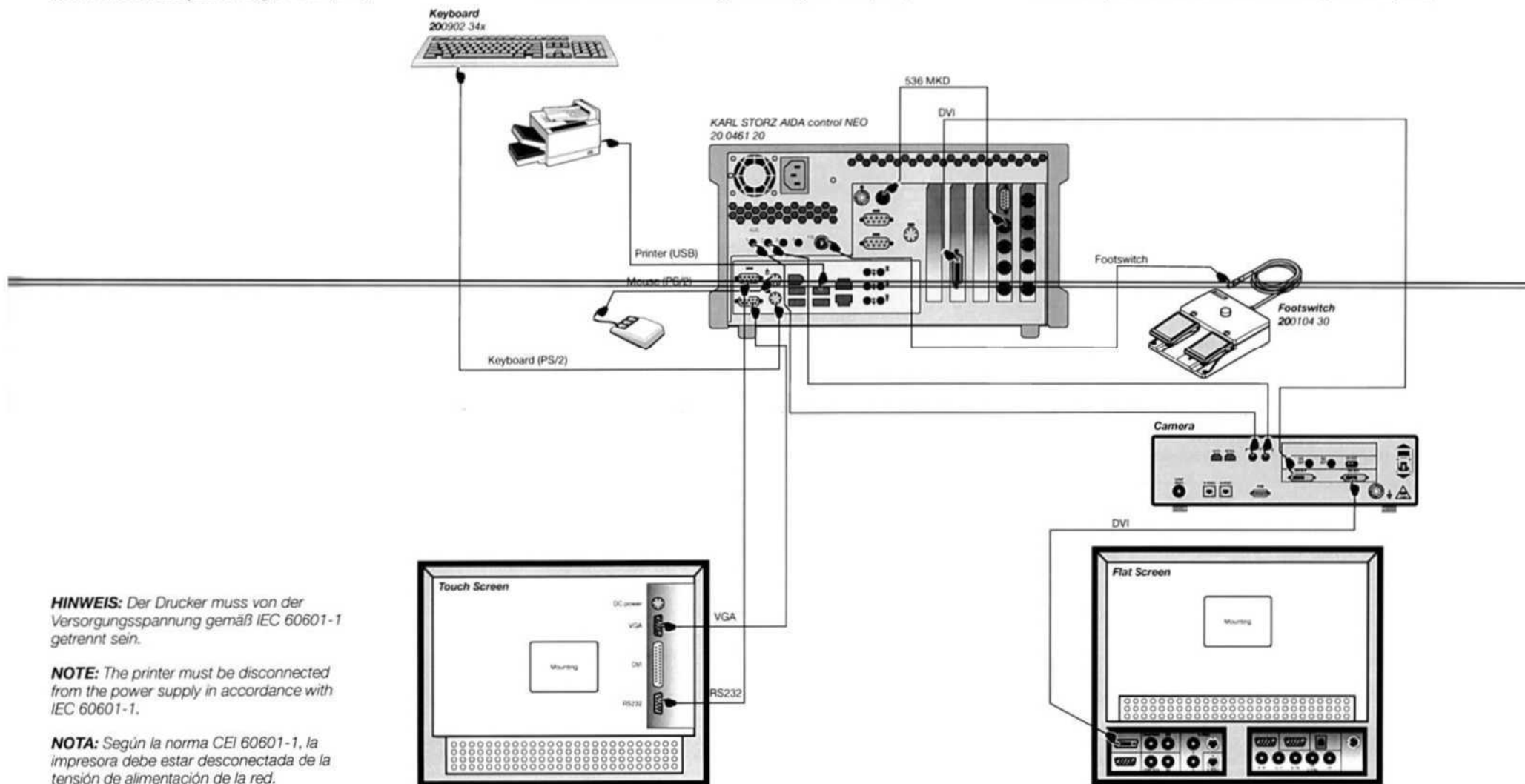
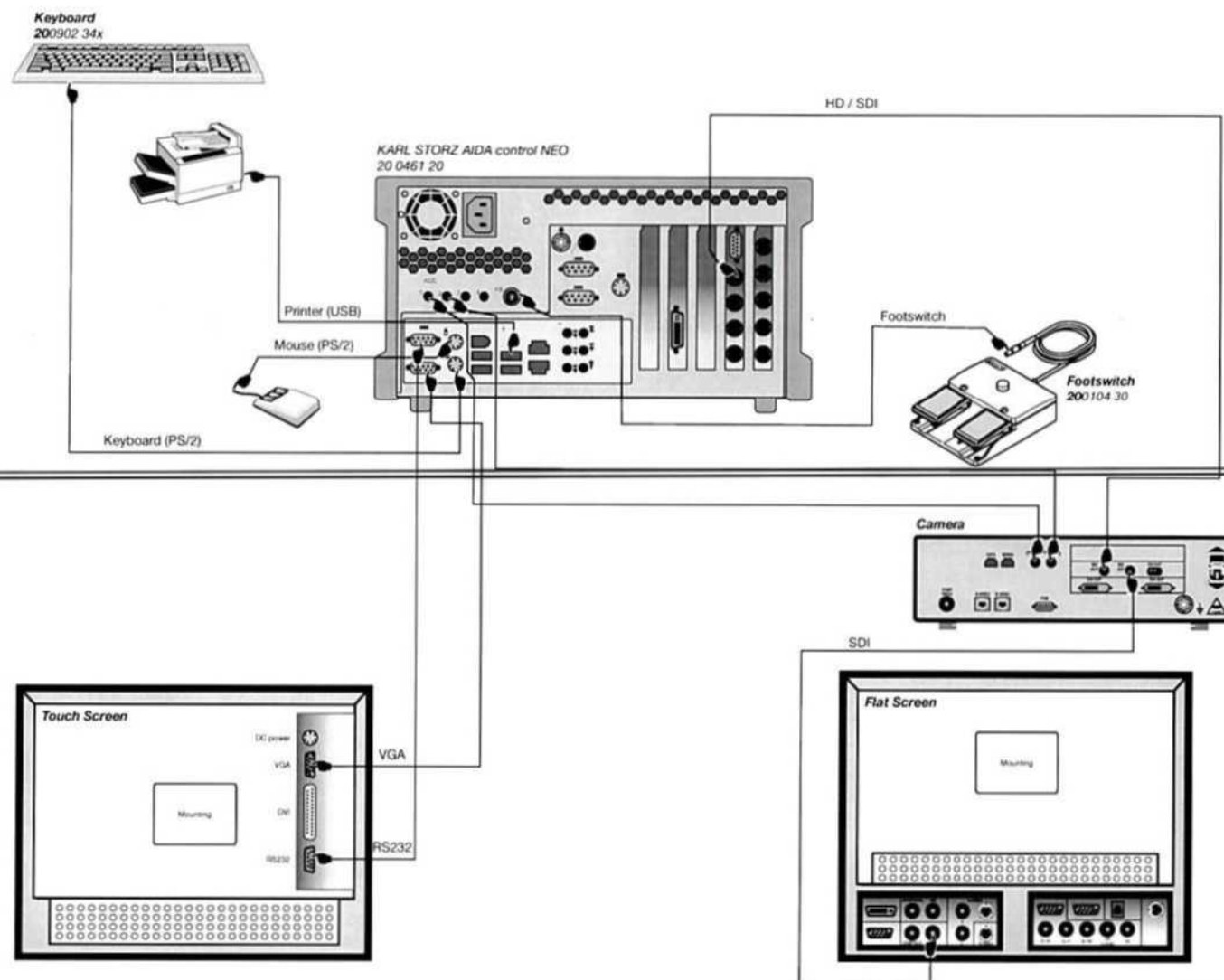
NOTE: The printer must be disconnected from the power supply in accordance with IEC 60601-1.

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2 Connecting the peripherals

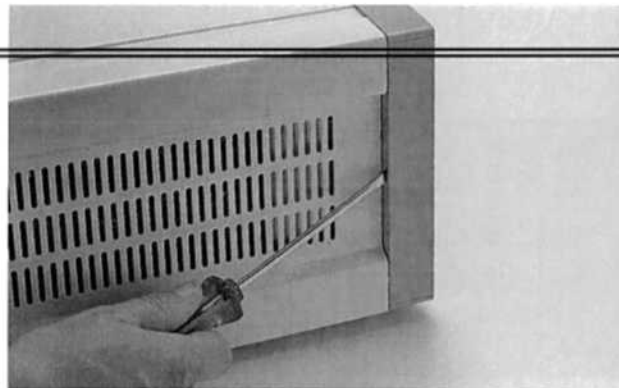
When setting up the PC system observe the instructions provided with the PC and monitor and the Connection diagram in Section 1.4.

2.1 Connect USB hardlock

To be able to start the licensed KARL STORZ AIDA® compact NEO software, the license dongle must be inserted.

If no USB hardlock is present, the error message **No hardlock found**. Program shuts down, appears, and the KARL STORZ AIDA® compact NEO software cannot be started.

1. Insert a screwdriver or similar tool into the recess on the rear of the side cover and lift out toward the front.

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2. Pull out the two sliding bolts of the front panel sideways. Tilt the front panel forward.

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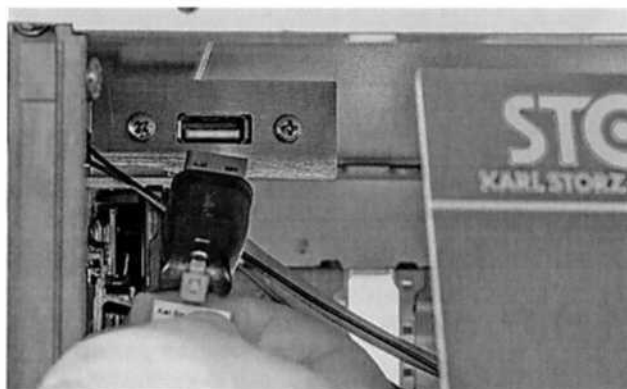
3. Plug the supplied USB dongle into the internal USB port and close the housing again.

NOTE: If the dongle is plugged into an external USB port, it must be removed before transporting the control unit of the computer.

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2.2 Connecting the Image 1 HD (DVI) video signal source

1. Use the cable supplied 20040086 to establish the video connection between the DVI-D video input ⑳ and the DVI output of the video signal source (e.g., endoscopic video camera).
2. Establish a connection between the BNC connector ㉖ and the HD/SDI video input ㉙ using the BNC cable [536MKD].

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2.3 Connecting the Image 1 (SDI) video signal source

1. Use the 536MK cable supplied to establish a video connection between the HD/SDI video input ⑳ and the HD/SDI output of the video signal source (e.g., endoscopic video camera or room camera).

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2.4 Connecting the video signal source (RGB / composite)

1. Use the RGB connection cable to establish the video connection between the R, G, B and Sync ①① - ①④ video inputs and the corresponding video outputs of the video signal source (e.g., endoscopic video camera) ...

Or

... use the 536 MK cable supplied to establish the video connection between the composite video input ①⑩ and the composite output of the video signal source (e.g., endoscopic video camera).

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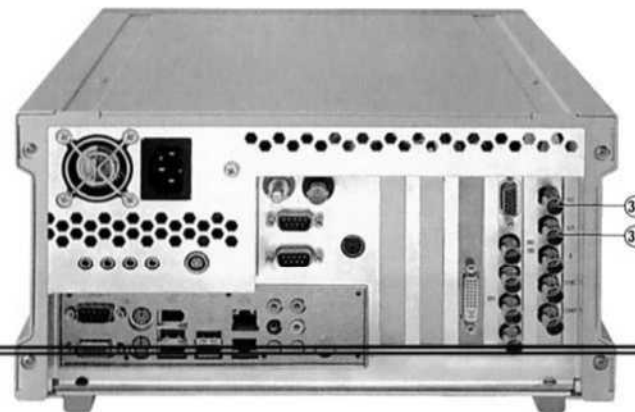
2.5 Connecting the (S-Video) video signal source

1. Use the cable supplied 20040087 to establish the video connection between the R/C 34 and G/Y 33 video inputs and the S-Video video output of the video signal source (e.g., endoscopic video camera).

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2.6 Connecting the headset (optional)

1. Plug the audio jack into the Audio-Out ⑳ socket (green socket) and the microphone jack into the microphone socket ⑲ (pink socket).

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Connecting the peripherals

KARL STORZ AIDA® compact NEO

2.7 Connecting the camera head buttons and footswitch

1. To use the camera head buttons, connect the ACC sockets of the Image 1 CCU to the ACC sockets of the AIDA compact NEO ⑨.

If you wish to use the camera head buttons and footswitch alternatively, you must configure the AIDA compact NEO for the footswitch. You determine the function of the camera head buttons with the ACC cable.

Function assignment with the single pedal footswitch:

- ACC cable to port 2 corresponds to the pedal being fully depressed

Function assignment with the dual pedal footswitch:

- ACC cable to port 1 corresponds to the left pedal being fully depressed
- ACC cable to port 2 corresponds to the left pedal being half depressed
- ACC cable to port 3 corresponds to the right pedal being half depressed
- ACC cable to port 4 corresponds to the right pedal being fully depressed

2. To use a footswitch, connect the footswitch to the footswitch port ⑩.

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3 Software installation

3.1 Configuration of AIDA® compact NEO

The KARL STORZ AIDA® compact NEO software is already pre-installed on the KARL STORZ AIDA® control NEO. To put the system into operation, perform the settings described below.

3.1.1 Language settings for the AIDAUser

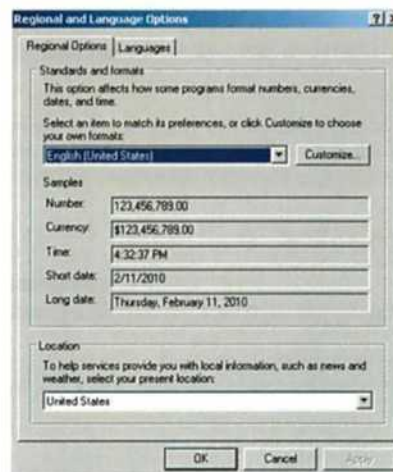
When switched on for the first time, AIDA compact NEO starts automatically and registers you as an AIDAUser.

➤ Entry of the configuration password is requested.

1. Enter the password **0000**.
2. Select the desired operational language.
3. Confirm with **OK**.
4. Close AIDA compact NEO.
5. Set the correct screen resolution for your monitor under Windows.

To customize the system to your place of work and language, perform the settings described below.

1. Click on **Start\Settings\Control Panel** and open the Regional and Language Options by double clicking on **Regional and Language Options**.
- The **Regional and Language Options** window appears.

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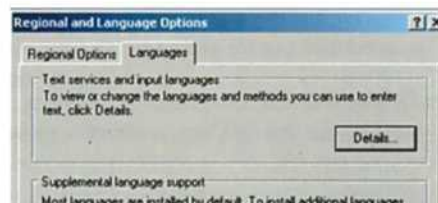
Software installation

KARL STORZ AIDA® compact NEO

2. Select the appropriate number, currency, date and time, etc. settings in the drop-down menu under **Standards and formats**.
3. Click on **Customize...** to select your own formats.
4. Select your place of work from the drop-down menu under **Location**.
5. Switch to the Languages tab and click on **Details...**

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☛ The window **Text Services and Input Languages** opens.

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Software installation

KARL STORZ AIDA® compact NEO

6. Select your input language from the drop-down menu under **Default input language**.
7. If your desired language is not included in the list, click on the **Add** button under **Installed services**, select your language in the **Add input language** window and confirm with **OK**.

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- Your desired language is now activated and appears in the list.
- Select the desired input language and keyboard layout and click on **Apply**.
- Close the **Text Services and Input Languages** window by clicking on **OK**.
- Confirm the settings in the **Regional and Language Options** window by clicking on **OK**.

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☛ Es erscheint das Fenster **Regional and Language Options**.

3.1.2 Language settings for the administrator

1. Log off as an AIDAUser and log on as an **Administrator** by entering the password **admin**.

To customize the system to your place of work and language, perform the settings described below.

2. In order to set the operating system language for administrators, click on **Start\Settings\Control Panel** and open the Regional and Language Options by double clicking on **Regional and Language Options**.

☛ The **Regional and Language Options** window appears.

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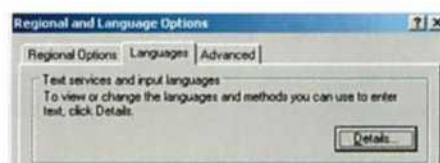
3. Select the appropriate number, currency, date and time, etc. settings in the drop-down menu under **Standards and formats**.

4. Click on **Customize...** to select your own formats.

5. Select your place of work from the drop-down menu under **Location**.

6. Switch to the **Languages** tab and click on **Details...**

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- The window **Text Services and Input Languages** opens.
- 1. Select your input language from the drop-down menu under **Default input language**.
- 2. If your desired language is not included in the list, click on the **Add** button under **Installed services**, select your language in the **Add input language** window and confirm with **OK**.
- Your desired language is now activated and appears in the list.
- 3. Select the desired input language and keyboard layout and click on **Apply**.
- 4. Close the **Text Services and Input Languages** window by clicking on **OK**.

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- 5. Confirm the settings in the **Regional and Language Options** window by clicking on **OK**.
- 6. When logged on as an administrator, check the date, time and time zone. Correct the settings if necessary.
- 7. Continue with *Chapter 4 Network configuration*.

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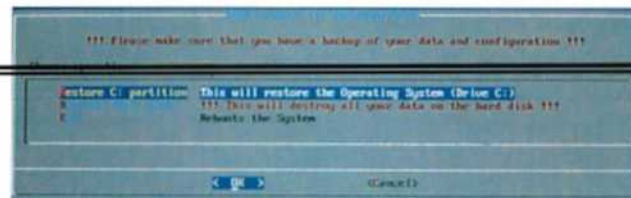
3.2 Installation for upgrades

In order to upgrade from system 20040608 or 20040908 to AIDA compact NEO, proceed as described in the sections below. In the process all data from the hard disk are deleted and the system is reinstalled. Then carry out the settings which are necessary after recovery of the basic system.

3.2.1 Installation of the AIDA compact NEO software

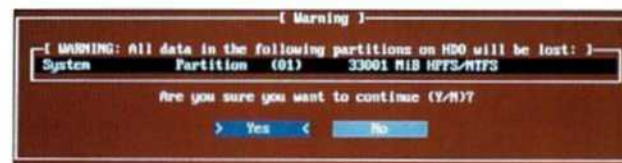
Ensure that you are logged on as an administrator.

1. Save all system data.
2. Insert the bootmaster CD and restart the system.
3. Confirm the two subsequent DOS queries by pressing any key.
 - ☛ The system boot is now automatically reconfigured.
 - ☛ Following successful configuration, the message **Status 00** appears.
4. Insert the recovery DVD and restart the system.
 - ☛ The window **AIDA compact NEO Recovery Disk** appears.

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5. Select **Restore C: partition**.
6. Confirm with **OK**.
 - ☛ The **Warning** window appears.

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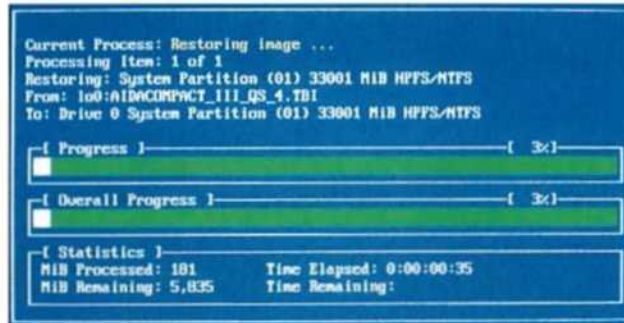
Software installation

KARL STORZ AIDA® compact NEO

- The files are now copied to the hard disk. The current status of the recovery process is displayed.

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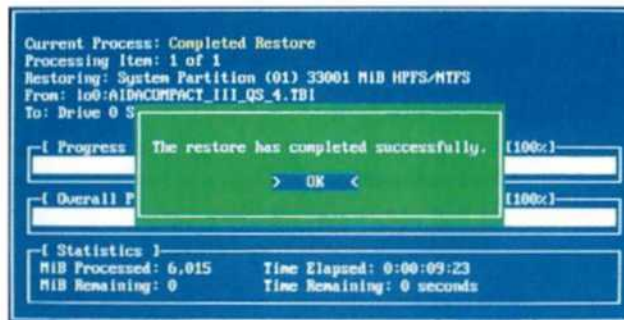
On completion of installation, the relevant message is shown and the CD is ejected.

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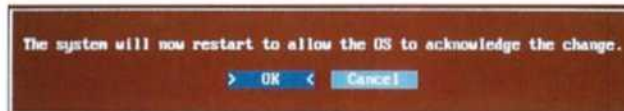
1. Confirm the message by pressing **OK**.

- The message **The system will restart to ...** appears.

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Software installation

KARL STORZ AIDA® compact NEO

3.2.2 Change the operating system language for the administrator user

On conclusion of the installation process and once the system has restarted, you are logged on as an administrator automatically.

1. The following window appears.
Press any key to continue.

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The **Regional and Language Options** window appears.
You can set the language, number, currency, date and time display
settings here.

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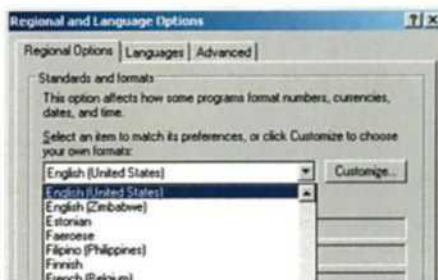
Software installation

KARL STORZ AIDA® compact NEO

2. Select the appropriate format for numbers, currency, date and time, etc. settings in the drop-down menu under **Standards and formats**.

or
Click on **Customize...** to customize the settings.

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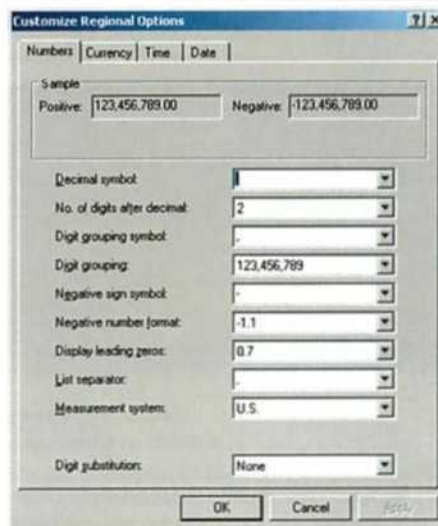
Software installation

KARL STORZ AIDA® compact NEO

- The window **Text Services and Input Languages** opens.
- Select your input language from the drop-down menu under **Default input language**.
- If your desired language is not included in the list, click on the **Add** button under **Installed services**, select your language in the **Add input language** window and confirm with **OK**.

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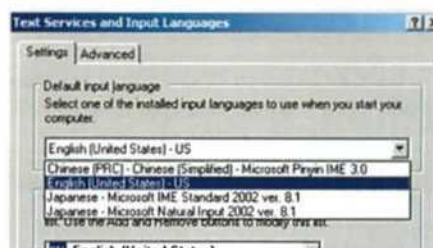
- Your desired language is now activated and appears in the list.
- Select the desired input language and keyboard layout and click on **Apply**.
- Close the **Text Services and Input Languages** window by clicking on **OK**.

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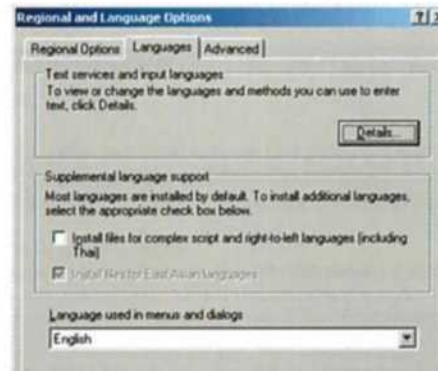
Software installation

KARL STORZ AIDA[®] compact NEO

9. Select the desired operating system language under **Language used in menus and dialogs** and click on **Apply**.
 10. Confirm the settings in the **Regional and Language Options** window by clicking on **OK**.
- ☛ The **NewSID Wizard** opens.

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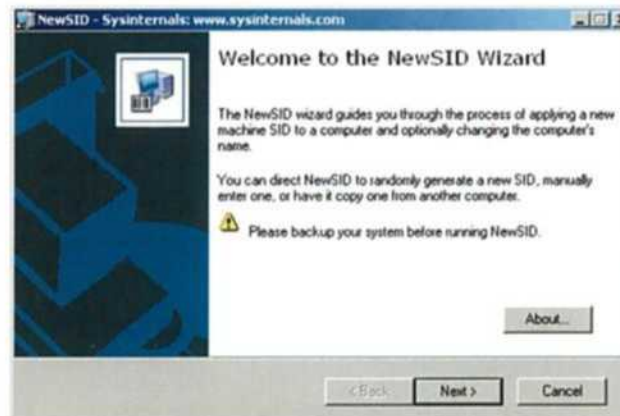
3.2.3 Creation of a new SID, entry of computer name

You are now requested to create a new SID.

1. Click on **Next** in the **NewSID Wizard** window.

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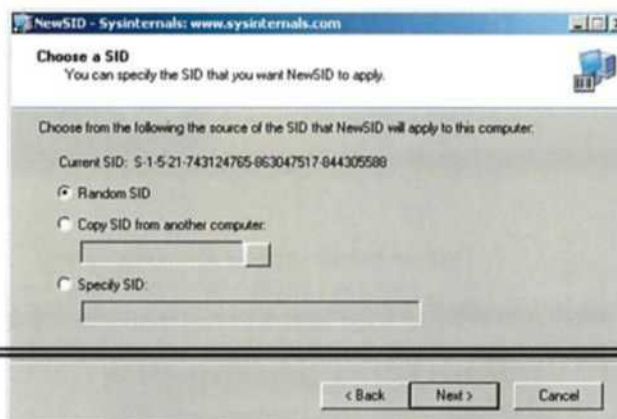
Software installation

KARL STORZ AIDA® compact NEO

2. Ensure that the option Random SID is selected and confirm by clicking on **Next** again.

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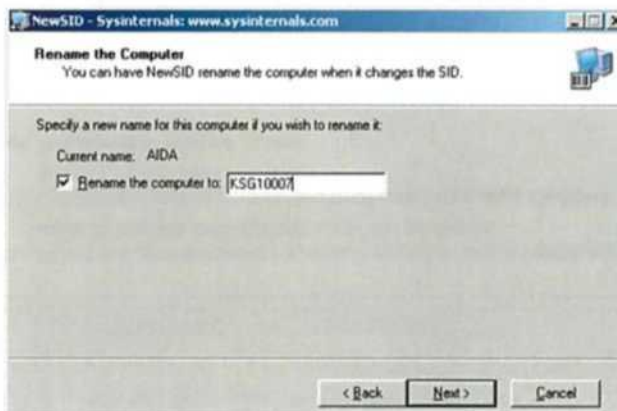
3. Activate the option **Rename the Computer to:** and then enter the name of the computer.

NOTE: Use the serial number of your KARL STORZ AIDA® control NEO computer as the computer name provided that no other name has to be allocated. The serial number is affixed on the rear of the computer.

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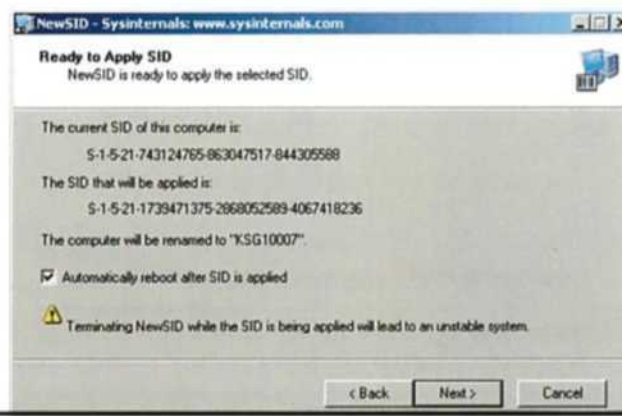
Software installation

KARL STORZ AIDA[®] compact NEO

5. Activate the option **Automatically reboot after SID is applied**.
6. To confirm the selection, click on **Next**.
 - ☛ The new SID and computer name will be applied and the computer will restart.

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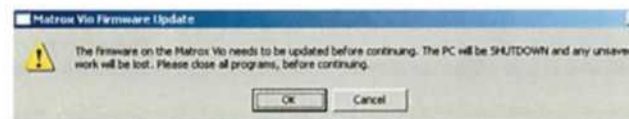
3.2.4 Firmware update

After the system has restarted you are logged on as an AIDAUser automatically.

- ☛ Entry of the configuration password is requested.
1. Enter the password **0000**.
 2. Select the desired operational language.
 3. Confirm with **OK**.
- ☛ The message **Matrox Vio Firmware Update** appears.
4. Start the automatic firmware update of the Matrox Vio card by clicking on **OK**.

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- The update is now carried out. The current status of the process is displayed.

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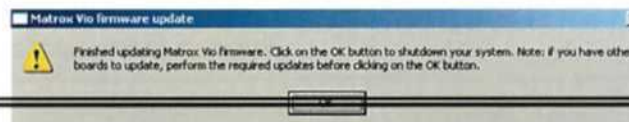
- The relevant message appears once the update is complete.
- Confirm the message by pressing **OK**.
- The system now shuts down.

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- Wait approx. 10 seconds after the system has switched itself off. Then restart the system.
- Set the language settings for the AIDAUser as described in *section 3.1.1*.

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KARL STORZ AIDA® compact NEO

3.3 Recovery

In the event of system recovery, you can carry out a recovery process using the AIDA compact NEO recovery DVD which replaces manual installation of the complete Windows® system and KARL STORZ AIDA® compact NEO software.

Perform the recovery process and then carry out the settings which are necessary after recovery of the basic system as described in sections 3.4.1 to 3.4.3.

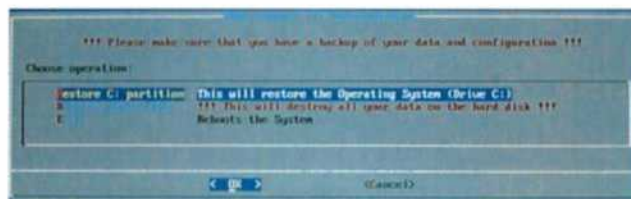
NOTE: Before starting the recovery process, all data should be saved. Although the recovery system offers the possibility to leave the KARL STORZ AIDA® compact NEO data on the system or to overwrite the complete system, it is not possible to guarantee that data will not be lost.

NOTE: The whole backup and recovery process must be carried out by trained IT personnel.

3.3.1 Recovery process

NOTE: In order to restore your user defined settings quickly and simply following repair work or system recovery, we recommend that you firstly save the files AIDA2.ini, FTPserv.ini, SAM.ini, Procedure.csv, DicomServer.csv, Surgeon.dat, Anaesthetist.dat, Operator.dat, Endoscope.dat, Tray.dat and Rename.dat on an external storage medium.

1. Insert the recovery DVD and restart the system.

☛ The window **AIDA compact NEO Recovery Disk** appears. To start the recovery process, select **Restore C: partition**.

In order to additionally delete all data on the hard disk, select **Restore whole disk**.

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Software installation

KARL STORZ AIDA® compact NEO

- ☛ The window **Text Services and Input Languages** opens.
- 5. Select your input language from the drop-down menu under **Default input language**.
- 6. If your desired language is not included in the list, click on the **Add** button under **Installed services**, select your language in the **Add input language** window and confirm with **OK**.

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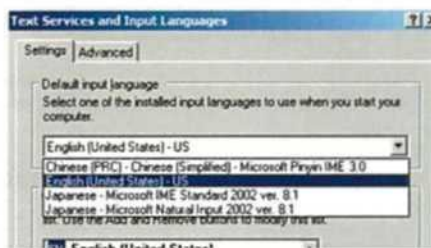
- ☛ Your desired language is now activated and appears in the list.
- 7. Select the desired input language and keyboard layout and click on **Apply**.
- 8. Close the **Text Services and Input Languages** window by clicking on **OK**.

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3.3.3 Creation of a new SID, entry of computer name

- You are now requested to create a new SID.
1. Click on **Next** in the **NewSID Wizard** window.

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7. Now set the language settings for the AIDAUser as described in *section 3.1.1*.

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4 Network configuration

NOTE: The network configuration should only be done by an IT-specialist.

4.1 Setting up network

1. In order to store the KARL STORZ AIDA® compact NEO data on a network drive, first connect your KARL STORZ AIDA® compact NEO system to a close by network connection.

Make sure that you install a network isolator (order no. W20883) or similar technology, in the event that an (optical) isolation between the network and the networked hardware in the medical room is required.

Make sure that you are logged on as a Windows® administrator for the next steps.

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2. Configure the KARL STORZ AIDA® compact NEO system's network properties: domain, IP-address, users.
3. Create a folder on a file server and share it so that KARL STORZ AIDA® compact NEO users have read and write access to this folder.

NOTE: KARL STORZ recommends creating a separate KARL STORZ AIDA® compact NEO user on the KARL STORZ AIDA® compact NEO system with limited local user rights that has write access on the shared network folder and that does an auto-logon when booting the system. In this case the users with the individual logins don't need to have write access to the folder and can therefore not change or delete captured data when working from their own computers.

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4.2 Domain integration

NOTE: Integration in a domain must be performed by an IT specialist only.

To integrate AIDA compact NEO in a domain, create a Windows user for each AIDAUser and perform the following settings for each one.

1. Log on as a domain user.
2. Open the **Batches** file under **C:\Program Files\EloTouchSystems**.
3. Run the application **Disable_PreCalibration_Message.reg** by double clicking.
4. Confirm the subsequent query with **Yes**.
5. Start the **Realtek HD Audio Manager** by double clicking on the symbol in the Windows task bar.
6. Select the **Audio-E/A** tab and click on the tool symbol under **ANALOG**.
7. In the connection settings, deactivate the option **Activate the automatic popup dialog after connecting the device**.

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KARL STORZ AIDA® compact NEO

5 SAM configuration

The following configuration of the KARL STORZ Application Manager is only necessary when using an KARL STORZ AIDA® interface box with KARL STORZ-SCB®.

Please refer to the KARL STORZ AIDA® Interface Box instruction manual (order no. 96206045D) for information on the configuration of the KARL STORZ AIDA® Interface Box.

The SAM files have been installed during KARL STORZ AIDA® compact NEO installation.

Make sure that you are logged on as Windows® Administrator for the next steps.

NOTE: In order to restore your user-defined settings quickly and simply following repair work or system recovery, we recommend that you firstly save the files AIDA2.ini, FTPserv.ini, SAM.ini, Procedure.csv, DicomServer.csv, Surgeon.dat, Anaesthetist.dat, Operator.dat, Endoscope.dat, Tray.dat and Rename.dat on an external storage medium.

5.1 Configure SAM

1. Copy the file C:\Program Files\KARL STORZ\compact\SAM.exe and paste a shortcut in the startup folder for all users.
2. Right click on the Microsoft® taskbar and select Properties.

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SAM configuration

KARL STORZ AIDA® compact NEO

3. Select Auto hide in the Taskbar and Start Menu Properties window and deactivate Always on top.

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4. Start KARL STORZ AIDA® compact NEO.

5. Go to the Setup window (right click - Setup).
6. Activate the SAM Checkbox in the General Settings window.
7. Save the new setting and exit KARL STORZ AIDA® compact NEO.

NOTE: When starting KARL STORZ AIDA® compact NEO the first time, the 'Setup' window will be activated automatically.

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5.2 Entries of the SAM.ini-files

8. Open the file C:\Program Files\KARL STORZ\compact\SAM.ini.

Always leave the first line in SAM.ini empty.

Always put KARL STORZ-SCB® as [Appl1].

Only activate applications that are definitely installed on the KARL STORZ AIDA® compact NEO system.

Up to 4 applications can be defined [Appl1] ... [Appl4], always start with [Appl1] then [Appl2] etc.

Save SAM.ini in the same folder as SAM.exe.

Do not use special characters in SAM.ini.

NOTE: If LOGFILE = 1 then allow all users unrestricted access to the SAMLOGGER file on the C: drive. If the file does not exist, start SAM while you are logged on as Administrator, and change the file properties.

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Values	Description
[SAM]	
IPSCB=192.168.1.1	IP address of KARL STORZ-SCB® computer don't change!
PORT=6070	port number for SCBlink® communication don't change!
PROTOCOL_VERSION=2	2 = communication with SCB control NEO system don't change!
LOGFILE=0	0 = Log-file off (recommended) 1 = Log-file on (Indicates advanced logging. When swichted on samlogger.dat is written to C:\)
[Appln]	first button from the right side n = 1 ... 4 1 = AIDA 2 = AV 3 = SCB 4 = OR1 Checklist

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CAPTION=AIDA	caption for button, if available = 1
PATH=c:\Program Files\KARL STORZ\Compact\compact.exe	path of application's .exe-files
AUTOSTART=0	0 = don't start application automatically 1 = start application when SAM is started
SCBDEVICEID=\$7F02	Identifier for KARL STORZ-SCB® \$7F00 for KARL STORZ AIDA® 2.0 \$7F02 for KARL STORZ AIDA® compact NEO \$7F08 for MagicView 300 VA42B \$7F09 for Terason US \$8F01 for ORChecklist
SCBSERNOM=\$0000	SCBlink®-device type serial number of application
FIV=\$0300	SCBlink® functional interface version of application \$0100 for MagicView 300 VA42B \$0100 for Terason US \$0300 for KARL STORZ AIDA® 2.0 \$0300 for KARL STORZ AIDA® compact II
APPLICATIONNAME=AIDA	SCBlink®-device info application name
NUMBEROFACTUALOBJECTS=0	SCBlink®-device info
MODULEVERSION=0	SCBlink®-device info
APPLICATIONVERSION=0200	SCBlink®-device info
SLEEP=1000	AIDA activation interval, when integrated with Magic View

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6 Communication software

This set-up is delivered with the KARL STORZ AIDA compact HD communication option 20040476-V01. (Set-up contains HL7, DICOM and DICOM video option.)

With the KARL STORZ AIDA® compact HD communication option, simply exchange the USB dongle and configure the settings for DICOM and HL7.

NOTE: For instructions on how to install/exchange USB dongles, please refer to Chapter 2 Connecting the peripherals.

NOTE: The KARL STORZ AIDA compact HD communication systems 20040910, 20040911 and 20041911 are delivered with a DICOM and HL7 interface already installed. This means that no system upgrade is required in this case.

NOTE: In order to restore your user-defined settings quickly and simply following repair work or system recovery, we recommend that

you firstly save the files AIDA2.ini, FTPserv.ini, SAM.ini, Procedure.csv, DicomServer.csv, Surgeon.dat, Anaesthetist.dat, Operator.dat, Endoscope.dat, Tray.dat and Rename.dat on an external storage medium.

- In order to be able to configure the settings for **DICOM** and **HL7**, click on the **Open Config** button and enter the password.

6.1 DICOM interface

Correct network installation is of the utmost importance, please refer to Chapter 4 KARL STORZ AIDA® compact NEO network configuration.

NOTE: Install the interface as described in Chapter 3.2 installation for upgrades.

- Enter the parameters for the DICOM communication into the following mask.

NOTE: Verify the parameters with your clinic's IT department.

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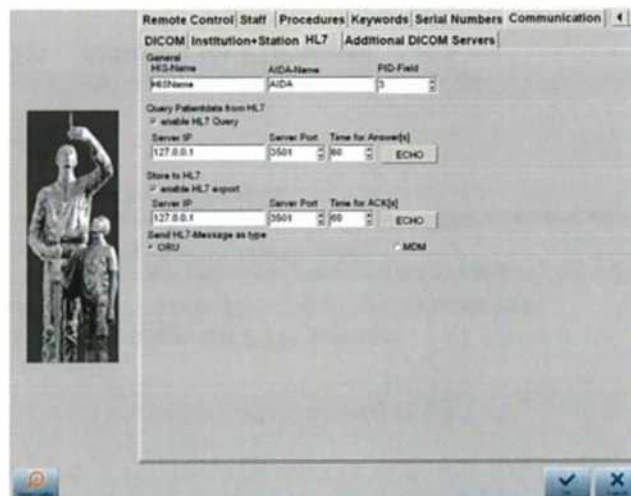
6.2 HL7 Interface (Health Level 7)

1. Enter the HL7 communication parameters in the following screen.

NOTE: Please agree on the parameters with the IT department of your clinic.

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7 Storage on FTP server

Set up the FTP server and configure aida2.ini accordingly.

Create a new user with a new password and an KARL STORZ AIDA® compact data folder on the FTP server.

1. Open the file **C:\Program Files\KARL STORZ\compact\aida2.ini** and insert the following...

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7.1 Aida2.ini for FTP server

Values	Description
[FTP]	
USER=	user that was created on the FTP server (AidaUserName)
PSWD=	password of the above server (AidaUserPassword)

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2. Start KARL STORZ AIDA® compact NEO and go into Setup Screen ([Ctrl] + [Alt] + [S]).

2. Arranque KARL STORZ AIDA® compact NEO y seleccione la ventana Setup ([Ctrl] + [Alt] + [S]).

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3. Enter FTP server and folder into the second page of the Setup Screen **FTP://servername/folder**.
4. Replace 'servername' with the FTP server's name. Replace 'folder' with the name of the folder that has been created on the FTP server.
5. Confirm the new settings.

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Restriction of the memory capacity

KARL STORZ AIDA® compact NEO

8 Restriction of the memory capacity

This setting is recommended, e.g. for saving to a network or when using USB sticks or an external USB hard drive.

When using USB sticks or an external USB hard drive the maximum memory capacity of the medium may not be entered as the maximum size. At least 20 MB memory capacity should be deducted as a reserve.

Capacity details for the various storage media:

CD-ROM: e.g. 650 MB / 700 MB

DVD+R DL: e.g. 4.7 GB

USB Stick: e.g. 1.9 GB

1. Start KARL STORZ AIDA® compact NEO and go into Setup Screen ([Ctrl] + [Alt] + [S]).
2. Select the user defined entry for the max. capacity in the dialogue box for file storage and enter the required capacity.
3. Confirm the new settings.

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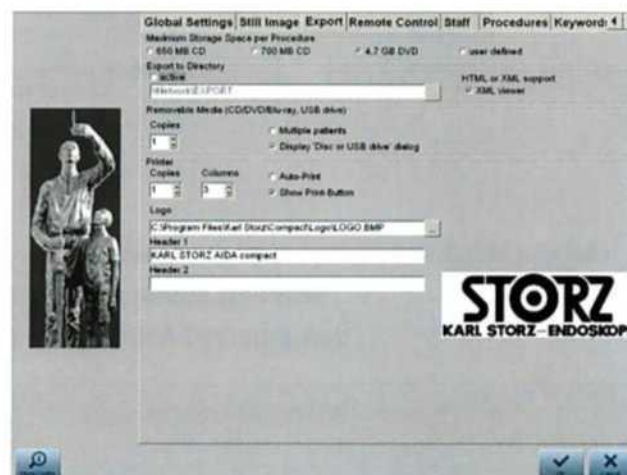
9 Display possibilities

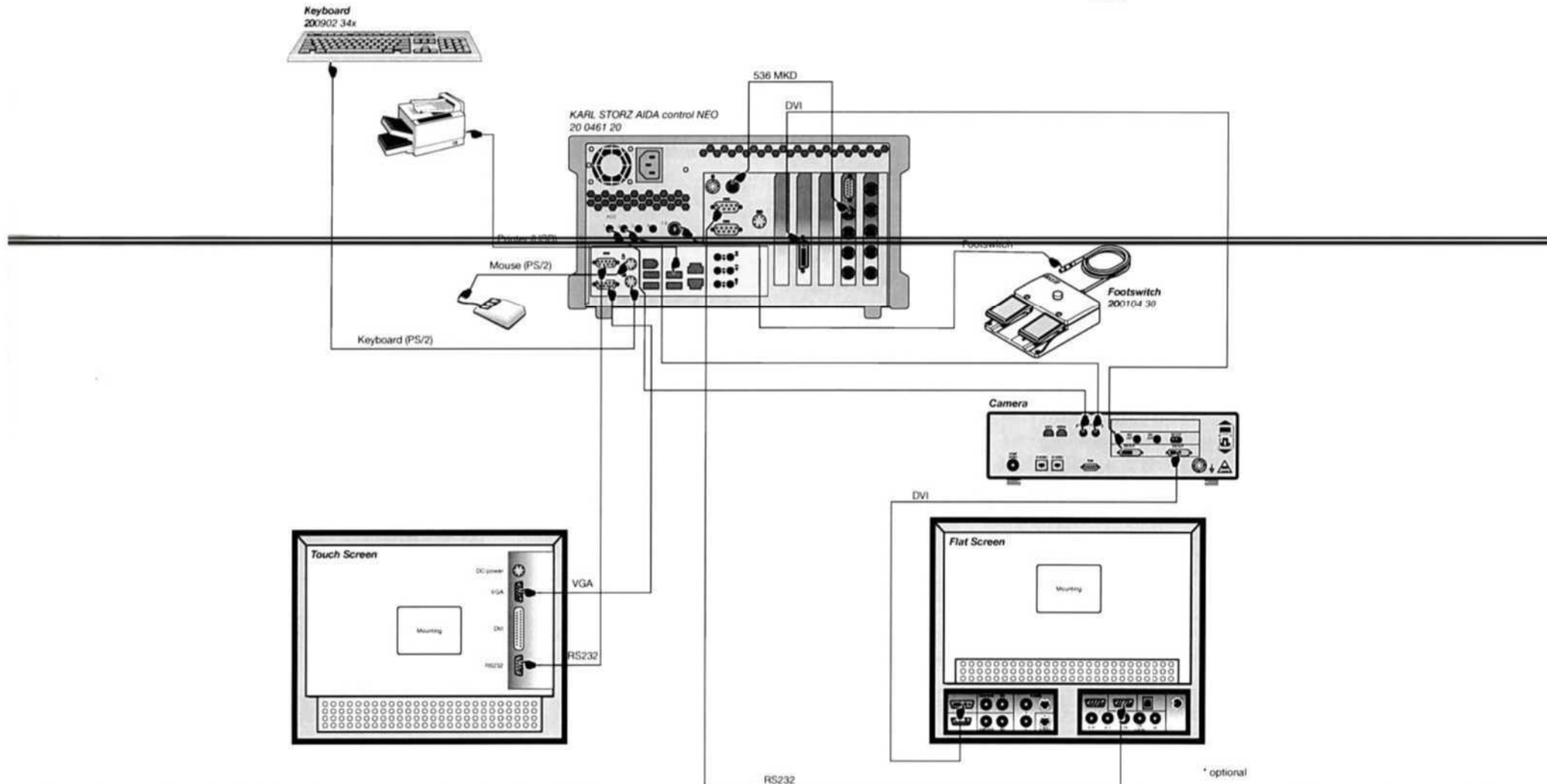
9.1 Live image function

9.1.1 Connection diagram for the live image function

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9.1.2 Setting up the recording/live image function

The following settings enable a function, which shows a white dot for still image capture and a flashing green dot for running video capture on the KARL STORZ 9419/9523 monitors.

NOTE: A connected RS232 cable and the correct 9419N / 9419NB / 9523N / 9523NB monitor is required for this.

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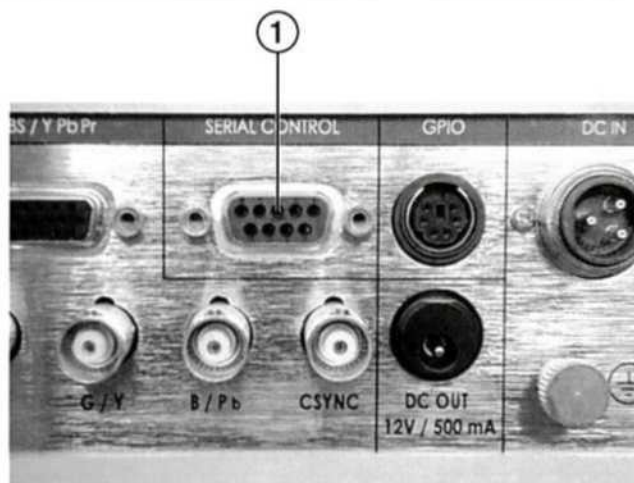
1. Use the RS232 cable 200400 73 to connect the serial connection on the 9523N/9523NB monitor ① to the COM 2 connection on the KARL STORZ AIDA® control ②.

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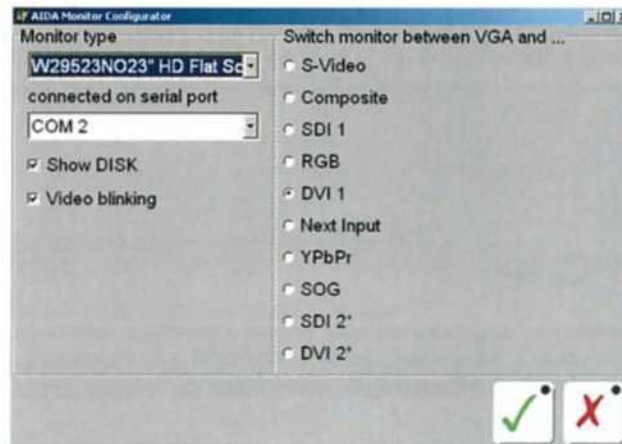
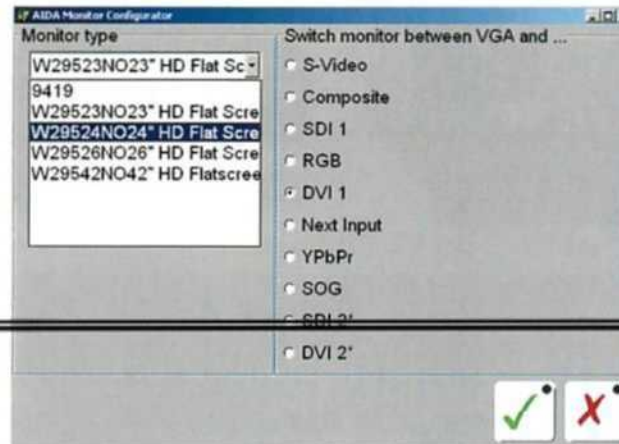
2. Start the program National.exe and select the type of monitor. Activate the check box Show DISK.

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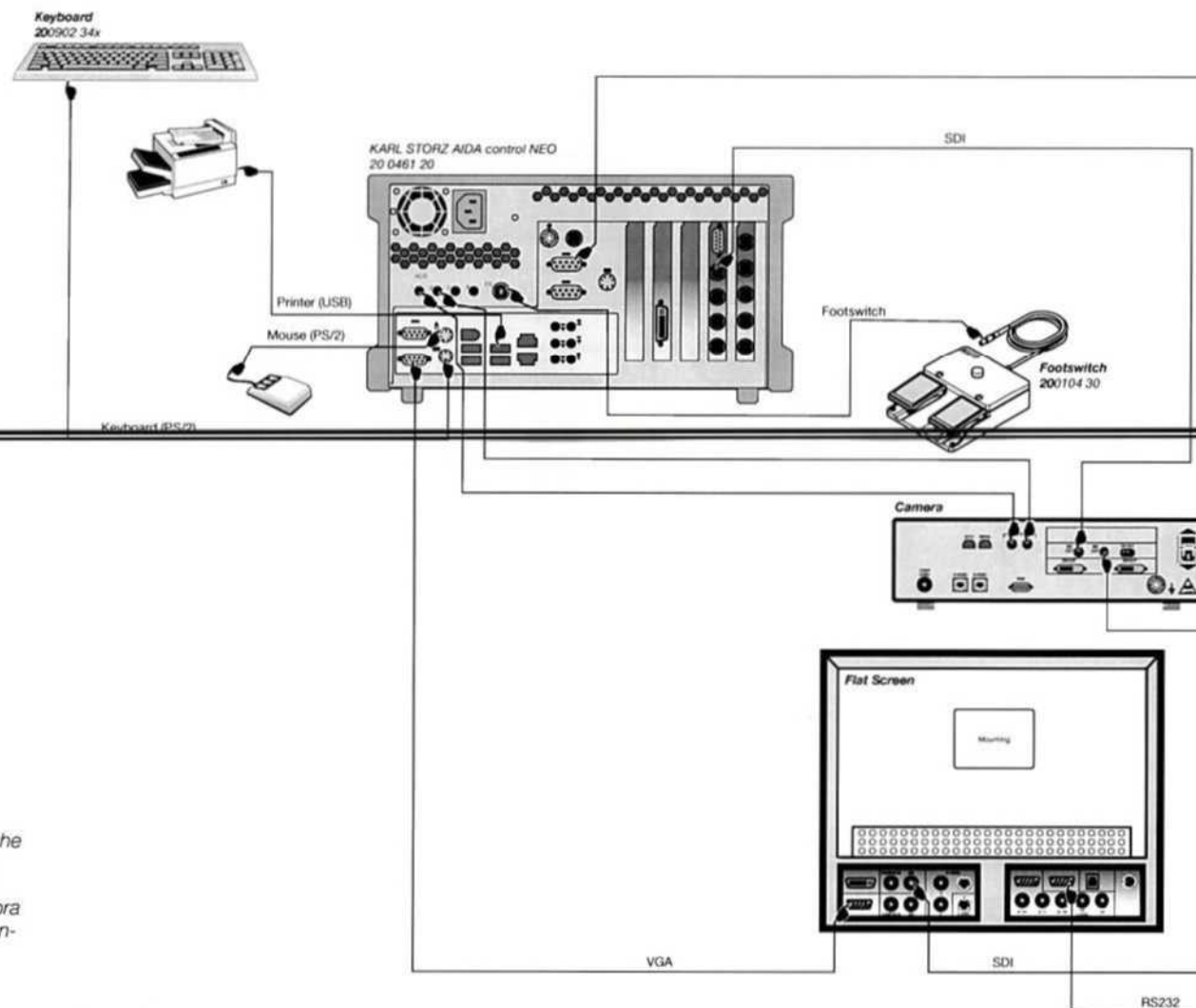
9.2 Toggle video inputs

9.2.1 Connection diagram for toggle video inputs with a camera system

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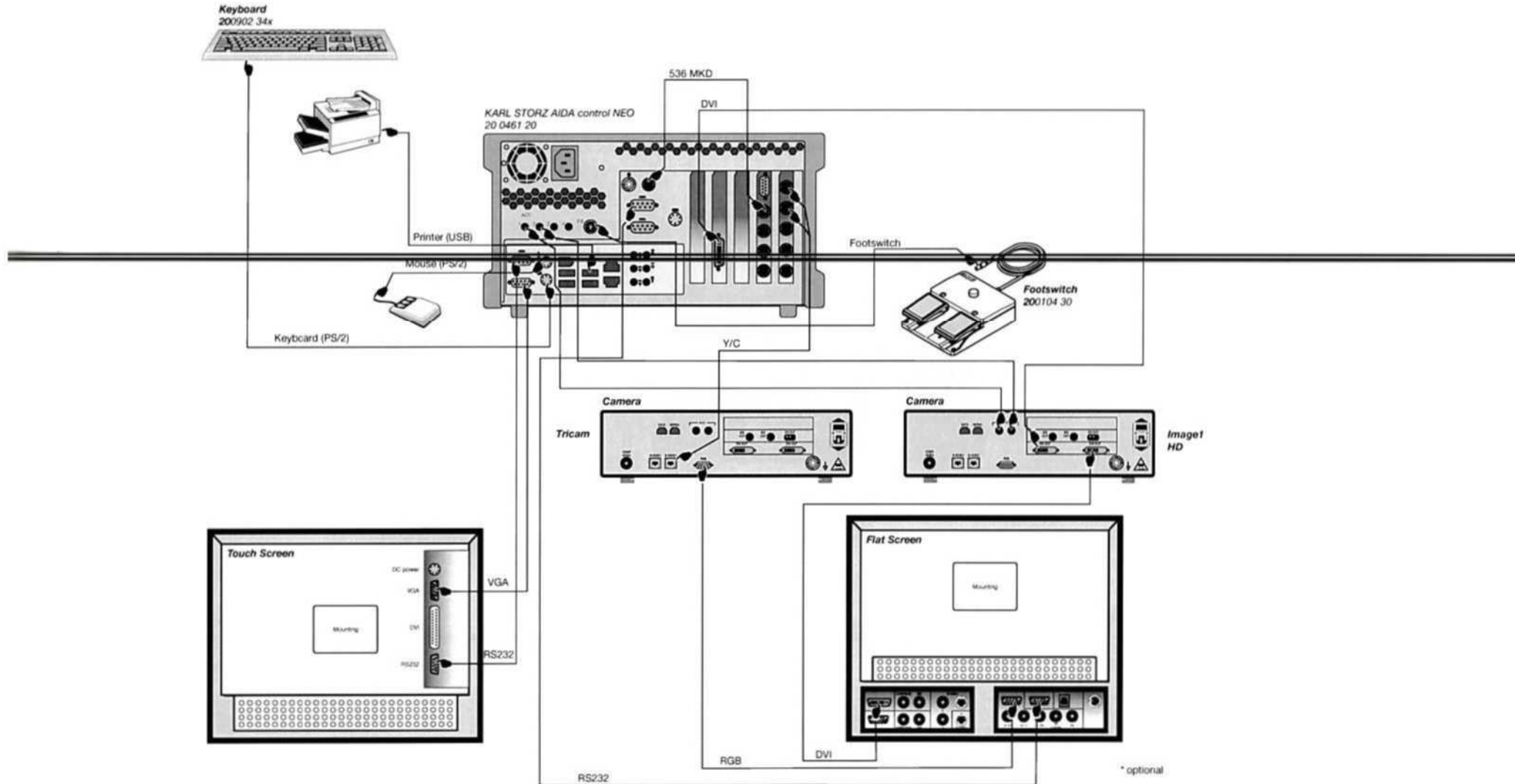
9.2.2 Connection diagram for toggle video inputs with two camera systems

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9.2.3 Setting up the toggle video inputs on surgical screen (DVI/ HD-SDI to VGA)

The following settings are used to switch-over the input signals of a KARL STORZ 9419 / 9523 monitor via a footswitch, camera head buttons or keyboard.

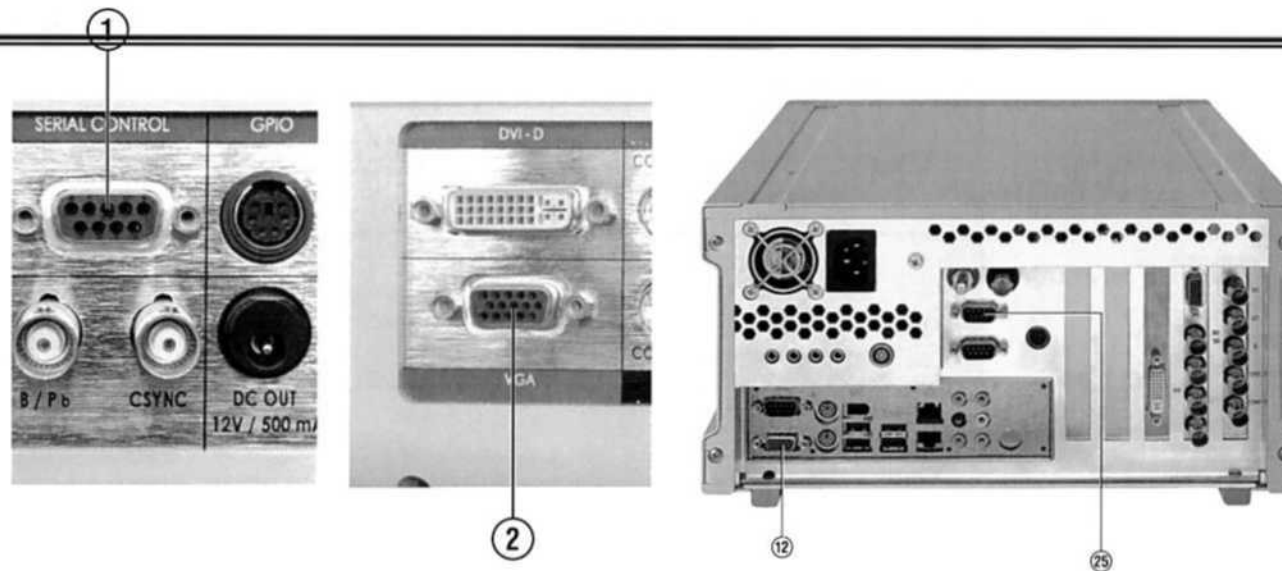
This avoids the need for an additional operator monitor for the KARL STORZ AIDA® compact NEO. Instead, the VGA signal is shown together with the video signal on the main monitor. The toggle commands then allow the operator to switch backward and forward between the AIDA and the live image views.

1. Use an RS232 cable 200400 73 (6 m) / 200403 73 (1.8 m) to connect the serial connection of the 9523N/ 9523NB ① monitor to the COM 2 connection of the KARL STORZ AIDA® control ⑳.
2. Use a VGA cable 200402 72 (6 m) / 200403 72 (2 m) to connect the VGA connection of the 9523N / 9523NB monitor ② to the VGA connection ⑫ on the KARL STORZ AIDA® control.

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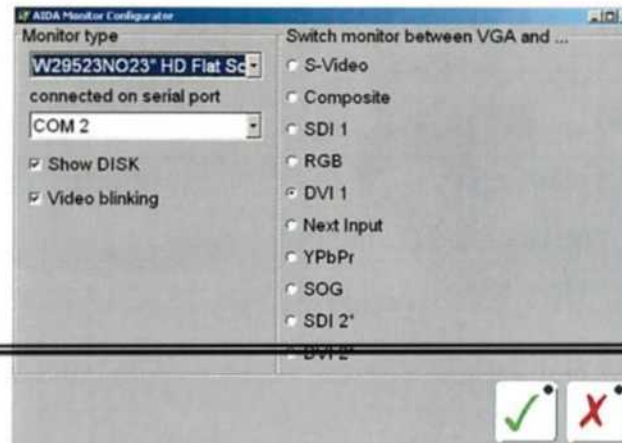
3. Start the program National.exe and select the type of monitor and the video signal used with it.

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10 Technical description

10.1 Technical data AIDA control NEO 200461 20

- Input voltage: 90 VAC ... 264 VAC
- Input frequency: 47 Hz ... 63 Hz
- Current consumption: 6 A (115 VAC) / 3 A (230 VAC)
- Output power: 350 W
- Fuse type: N/A, no user serviceable fuses - contained in power supply
- Dimensions (mm): 304 x 163 x 355 (w x h x d)
- Weight: 6.0 kg
- Operating temperature: 5 °C ... 45 °C / (41 °F ... 113 °F)
- Storage/transport conditions:
 - Air humidity: 10 % ... 90 % (relative humidity, non-condensing)
 - Temperature: -10 °C ... 60 °C / (14 °F ... 140 °F)
- Atmospheric pressure: 500 hPa ... 1080 hPa
- Power supply: Custom Medical Grade power supply
- Motherboard type: Embedded FLEX-ATX
- Hard drive: 500 GB
- DVD drive: Slim SATA DVD±RW/±R CD-RW/R or Slim SATA Blu-ray (±R only)
- RAM: 2 GB DDR2 PC800
- Microprocessor: Intel® Core™ 2 DUO T7400
- Operating system: Windows XP Professional for embedded systems

10.2 Standard compliance (for 200461 20)

According to IEC 60601-1, UL 60601-1, CAN/CSA C22.2 No. 601.1:

Type of protection against electric shocks:
Protection Class I

According to IEC 60601-1-2:

Please read the Electromagnetic Compatibility Information in the appendix.

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10.3 Directive compliance (for 20046120)

According to Medical Device Directive (MDD):

These medical products belong to Class I

These medical products bear the CE mark in accordance with MDD 93/42/EEC.

A code number after the CE mark indicates the responsible notified body.

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10.4 Technical documentation (AIDA control NEO)

On request, the manufacturer will provide those circuit diagrams, itemized parts listings, descriptions, sets of adjustment instructions, and other items of available documentation to suitably qualified

user personnel duly authorized by the manufacturer for their use in repairing those components of the unit that have been designated by their respective manufacturers as repairable.

Supply of such technical documentation relating to the unit shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their level of technical training, to open or repair the unit.

Explicitly exempted herefrom are those maintenance and repair operations described in this installation manual.

NOTE: We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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11 Appendix

11.1 Electromagnetic Compatibility (EMC) Information

WARNING: Medical electrical equipment needs special precautions regarding Electromagnetic Compatibility (EMC). Observe the EMC instructions in this appendix, as well as the instruction manual, during installation and commissioning.

The KARL STORZ AIDA® control model 20046120 corresponds to EN/IEC 60601-1-2 :2001 [CISPR 11 Class A] and therefore meets the EMC requirements of the Medical Device Directive (MDD) 93/42/EEC.

These limits are designed to provide reasonable protection against the typical electromagnetic interference to be expected in a medical environment. The KARL STORZ AIDA® control model 20046120 is a Group 1 unit (as per CISPR 11).

Group 1 contains all the 'equipment and systems which generate or use RF energy only for their internal functioning'.

NOTE: The tables and guidelines that are included in this appendix provide information to the customer or user that is essential in determining the suitability of the equipment or system for the electromagnetic environment of use, and in managing the electromagnetic environment of use to permit the equipment or system to perform its intended use without disturbing other equipment and systems or non-medical electrical equipment. If this equipment does cause harmful interference with other devices, the user is encouraged to try to correct the interference by one or more of the following measures:

- reorient or relocate the receiving device
- increase the separation between the equipment
- connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.

If you have any further questions, please contact your local representative or our service department.

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WARNING: The KARL STORZ AIDA® control model 20046120 should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.

WARNING: The KARL STORZ AIDA® control model 20046120 is intended for use by healthcare professionals only. This is a CISPR 11 Class A medical system (RF emissions) and is suitable for use in all establishments other than domestic and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes. In a domestic environment the KARL STORZ AIDA® control model 20046120 may cause EMC interference, in which case it may be necessary to take adequate mitigation measures, such as re-orienting, relocating, or shielding the KARL STORZ AIDA® control model 20046120 or filtering the connection to the public mains network.

WARNING: The use of portable and mobile RF equipment may have an impact on this and other pieces of medical equipment.

WARNING: The use of cables other than those specified in the KARL STORZ instruction manual may result in increased emissions or decreased immunity of the KARL STORZ AIDA® control model 20046120. The cables listed below have been shown to comply with the requirements of EN/IEC 60601-1-2. When using cables other than those specified here, it is the responsibility of the user to ensure that they comply with EN/IEC 60601-1-2.

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Table 200

Cables which have been shown to comply with EN/IEC 60601-1-2:

Cable type	Shielded	Length [m]	Ferrite	Used for
VGA	Yes	1.8	Yes	Connection to the Monitor
RS232	Yes	1.8	No	Connection to the Touch Screen
4 Pin Mini DIN	Yes	5.0	No	Connection of SCB signals
Y/C [S-video]	Yes	1.8	No	Connection of S-video signals
DVI cable	Yes	1.5	Yes	Connection of AIDA control NEO with IMAGE 1™
AC Power Cord	No	3	No	Power supply

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Table 201		
Guidance and manufacturer's declaration – electromagnetic emissions		
KARL STORZ AIDA control NEO model 20046120 is intended for use in the electromagnetic environment specified below. The customer or user of the KARL STORZ AIDA control NEO model 20046120 should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	KARL STORZ OR1™ control model 20 0461 20 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The KARL STORZ OR1™ control model 20 0461 20 is suitable for use in all establishments other than domestic and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Anhang**KARL STORZ AIDA[®] compact NEO****Appendix****KARL STORZ AIDA[®] compact NEO****Anexo****KARL STORZ AIDA[®] compact NEO****Tabla 201****Guía y declaración del fabricante – emisiones electromagnéticas**

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Table 202

Guidance and manufacturer's declaration – electromagnetic immunity

KARL STORZ AIDA control NEO model 20 0461 20 is intended for use in the electromagnetic environment specified below. The customer or user of the KARL STORZ AIDA control NEO model 20 0461 20 should ensure that it is used in such an environment.

Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidelines
Electrostatic Discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	Complies ± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Complies ± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	Complies ± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<8% U_T (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	Complies <8% U_T (>95% dip in U_T) for 0.5 cycle Complies 40% U_T (60% dip in U_T) for 5 cycles Complies 70% U_T (30% dip in U_T) for 25 cycles Complies <5% U_T (>95% dip in U_T) for 5 sec	Commercial or hospital environment. If the user of the equipment or system requires continued operation during power mains interruptions, it is recommended that the equipment or system be powered from an uninterruptible power supply.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	Complies 3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

* Note: U_T is the a.c. mains voltage prior to application of the test level.

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Table 204

**Guidance and manufacturer's declaration – electromagnetic immunity –
for equipment and systems that are not life-supporting**

KARL STORZ AIDA control NEO model 20 0461 20 is intended for use in the electromagnetic environment specified below. The customer or user of the KARL STORZ AIDA control NEO model 20 0461 20 should ensure that it is used in such an environment.

Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 V _{rms} 150 kHz to < 80 MHz	3 V _{rms}	<p>Portable and mobile RF communications equipment should be used no closer to any part of the KARL STORZ AIDA control NEO model 20 0461 20, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d = [3.5/3] \sqrt{P} \quad 150 \text{ kHz to } < 80 \text{ MHz}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = [3.5/3] \sqrt{P} \quad 80 \text{ MHz to } < 800 \text{ MHz}$ $d = [7/3] \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts [W] according to the transmitter manufacturer and d is the recommended separation in meters [m].</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey a, should be less than the compliance level in each frequency range b.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection by structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio [cellular/cordless] telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the unit is used exceeds the applicable RF compliance level above, the unit should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the unit.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

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Table 206

**Recommended separation distances between portable and mobile RF communications equipment and the
KARL STORZ AIDA control NEO model 200461 20**

The KARL STORZ AIDA control NEO model 200461 20 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the unit can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the unit as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter [W]	Separation distance d [m] according to frequency of transmitter		
	150 kHz to < 80 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	80 MHz to < 800 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{3} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance **d** in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where **P** is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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KARL STORZ Endoscopy South Africa, (Pty) Ltd.
8th Floor Convention Tower
Cnr Heerengracht & Coen Steytler, Foreshore
Cape Town 8001, South Africa
P.O.Box 6061, Roggebaai Cape Town,
8012, South Africa
Phone: +27 21 417 2600, Fax: +27 21 421 5103
E-Mail: info@karlstorz.co.za

TOO KARL STORZ ENDOSCOPY Kazakhstan
Khodjanova 17
050060 Almaty, Kazakhstan
Phone/Fax: +7 72 72 49 43 63, +7 72 72 49 41 00
e-mail: KarlStorzKazakhstan@ttte.kz

KARL STORZ Endoscopy India Private Ltd.
C-126, Okhla Industrial Area, Phase-1

New Delhi 110020, India
Phone: +91 11 26815445-51,
Fax: +91 11 268129 86
E-Mail: karlstorz@vsnl.com

KARL STORZ GmbH & Co. KG
Resident Representative Office
80/33 (44/19) Dang Van Ngu,
F.10-Q. Phu Nhuan
Ho Chi Minh City, Vietnam
Phone: +848 991 8442, Fax: +848 844 0320

KARL STORZ Endoscopy (S) Pte. Ltd.
#05 - 08 San Centre, 171 Chin Swee Road
Singapore 0316, Singapore
Phone: +65 6376-1066, Fax: +65 6376-1068
E-Mail: karlstorz@pacific.net.sg

Karl Storz Endoscopy Australia Pty Ltd
15 Orion Road Lane Cove NSW 2066
P O Box 50 Lane Cove NSW 1595
Phone: +61 (0)2 9490 6700, Fax: +61 (0)2 9420 0695
E-Mail: info@karlstorz.au

KARL STORZ Endoscopy China Ltd.
Hong Kong Representative Office
Unit 1601, Chinachem Exchange Square
1 Hoi Wan Street, Quarry Bay,
Hong Kong, People's Republic of China
Phone: +8 52 28 65 2411, Fax: +8 52 28 65 4114
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Beijing Representative Office
Room 610, China Life Tower
No. 6, Chaowai Street
Beijing, 100020, People's Republic of China
Phone: +86 10 8525 3725, Fax: +86 10 8525 3728
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Unit 3901-3904, Tower 1 Grand Gateway
No. 1 Hong Qiao Road
Shanghai, 200030, People's Republic of China
Phone: +86 21 6113-1188, Fax: +86 21 6113-1199
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Chengdu Representative Office
F-5, 24/F., Chuanxing Mansion,
No. 18 Renming Road South
Chengdu, Sichuan, 610016, People's Rep. of China
Phone: +86 28 8620-0175, Fax: +86 28 8620-0177
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Guangzhou Representative Office
Room 1119-20, Dongshan Plaza
69 Xianlie Road Middle, Dongshan District,
Guangzhou, Guangdong, 510095,
People's Rep. of China

Phone: +86 20 8732-1281, Fax: +86 20 8732-1286
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy Asia Marketing Pte Ltd
3791 Jalan Bukit Merah
06-11 e-Centre @ Redhill
Singapore 159471, Singapore
Tel. No. +65 63761066, Fax. No. +65 63761068
Email: infoasia@karlstorz.com.sg
Email: serviceasia@karlstorz.com.sg

KARL STORZ Endoscopy Singapore Sales Pte Ltd
3791 Jalan Bukit Merah
#06-07 e-Centre @ Redhill
Singapore 159471
Tel. No. +65 65325548, Fax No. +65 65323832
Email: infoosg@karlstorz.com.sg
Email: servicesgp@karlstorz.com.sg

KARL STORZ Endoscopy Japan K. K.
Bois Hongo Building 6F1, 3-42-5 Hongo
Bunkyo-ku, Tokyo 113-0033, Japan
Phone: +81 3 5802-3966, Fax: +81 3 5802-3988
E-Mail: info@karlstorz.co.jp

www.karlstorz.com