CHROMOPHARE® C 450

Service-Manual (D, E)

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The Berchtold CHROMOPHARE® C 450 is a modern surgical light with essential, technical innovations.

Due to the use of modern electronics, a computer calculated polygon-reflektor, and a free-moving, circular mounting track with cardanic suspension of the light head, this system offers many advantages:

- An operating area free from shadows
- high illumination with daylight-type colour temperature
- safe hygiene conditions due to a smooth outer surfaces without projecting operational controls and air-tight covering of the lamp head
- large height adjustment range
- Defective halogen bulbs can easily be replaced by the nursing staff

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2. Technical Data

Berchtold CHROMOPHARE® Surgical Light C 450

Electrical connection data	110-120/220-240 Volt
Illumination measured at 1 m distance with colour temperature 4.500 K	55.000 Lux
with colour temperature 3.600 K	70.000 Lux
Diameter of polygon reflector	410 mm (16.1 in)
Diameter of lamp housing	450 mm (17.7 in)
Light emission surface	78% of the lamp underside
Diameter of small light field	165 mm (6,5 in)
Diameter of large light field	220 mm (8,6 in)
Infinite light intensity adjustment / light field adjustment	by rotating the sterilizable hand grip
Constant light field diameter without refocusing	constant up to 500 mm difference in height
Main lighting source, 1 halogen bulb	100 Watt / 24 V, article no. CZ 905-24
Reserve lighting source, 1 halogen bulb	100 Watt / 24 V, article no. CZ 905-24
<i><u>valid from serial no. 75.252</u></i>	
Main lighting source, 1 halogen bulb	110 Watt / 22,8 V, article no. CZ 905-22
Reserve lighting source, 1 halogen bulb	110 Watt / 22,8 V, article no. CZ 905-22
Automatic switching to reserve lighting in case of defective main light source	with visual display
Average lifetime of halogen bulb	approx. 1000 hours
Swivel radius of the lamp housing as single lamp	1660 mm (5.4 ft)
as double lamp	1610 / 1760 mm (5,2 ft / 5.7 ft)
Height adjustment of the lamp head	1190 mm (3.9 ft)
Electrical system in accordance with VDE and IEC safety standards	

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Transformer and Emergency Power Change-Over

The mains voltage is switched over to the T 1 transformer by means of a main switch and the F 17 and 18 fuses. Voltage from the secondary winding of the transformer is then present at the K 9 relay. This circuit is protected by the F 1 fuse. The K 9 relay picks up and connects the transformer secondary operating voltage and the operating lamps to the relay contacts.

In case of a power failure, the relay contacts of the K 9 relay go into the offposition and switch the lamps to emergency power supply. This circuit is protected by the F 19 and 20 fuses (see page 29).

Circuit Arrangement of the Lamps

In case of ac operating voltage, the system will switch to the H 1 main lamp by means of the S 1 switch and the K 1 current relay. In case of defective operation of the main lamp, the K 1 relay will drop out, as current no longer flows through its winding.

The H 2 reserve lamp is activated by means of the break contact. The H 3 error indication lamp remains lit until the defective halogen bulb is exchanged for a new one.

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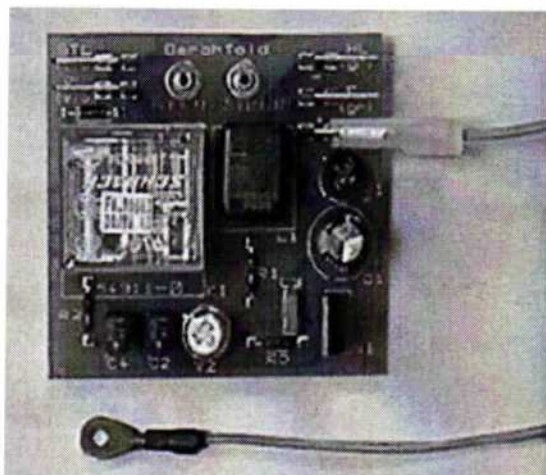
Graphic spare part description - valid for all C 450 SG/SO and valid for C 450 wall/ceiling mounted up to serial no. 70750

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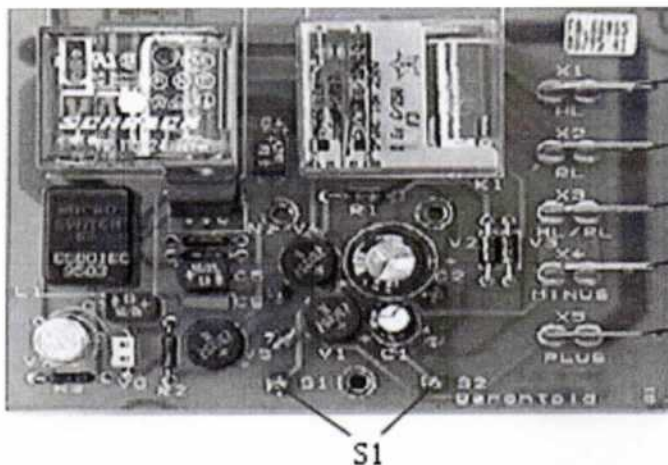
Control board for OR-lights C 450 ceiling and wall mounted
 part no. 54910

- X1 zur Hauptlampe/grau
to main bulb/grey
- X2 zur Reservelampe/lila
to reserve bulb/lila
- X3 zur Reservelampe/lila
to reserve bulb/lila
- X4 Eingang, - Minus/grau
input, - negativ/grey
- X5 Eingang, + Plus/orange
input, + positiv/orange
- S1 Mikroschalter auf der Rückseite
montiert,
Teile Nr. 37801
micro switch, mounted on the back
side,
part no. 37801

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Attention!

Pay attention to the activation of the main halogen bulb at direct current supply. If a switch over from main bulb to reserve bulb is caused, check the polarity at X4 (-) and X5 (+) and change if necessary.

Tastaturfolie, selbstklebend, für OP-Leuchte C 450,
 Teile Nr. 47091
 key board foil, self adhesive, for OP-lights C 450,
 part no. 47091

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
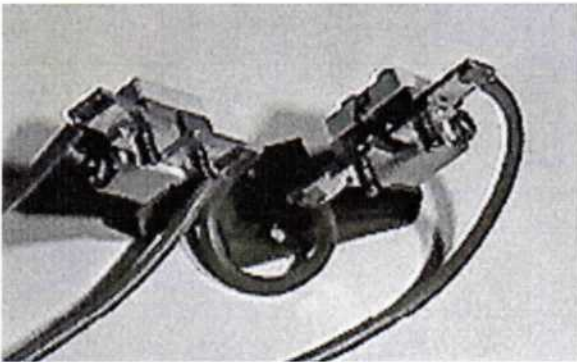
Only in combination lights

friction srew, 340 mm lenght, part no. 38821
friction srew, 260 mm lenght, part no. 38827

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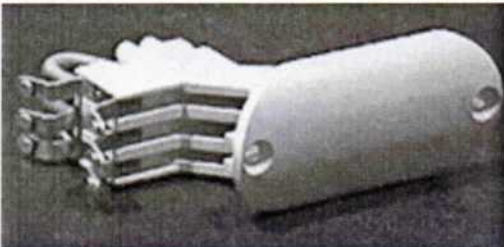Bulb holder with mounted main and reserve bulb holder,
with cable connections and flat plugs
part no. 59881

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
Only in combination lights

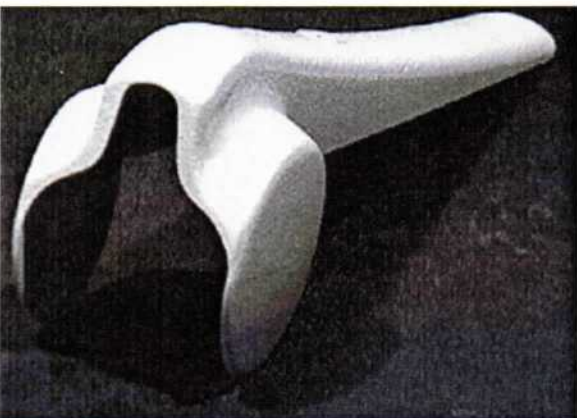contact block for sliding contacts at the central column to
horizontal suspension arm
part no. 39693

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
Cover for joint at spring balanced arm
part no. 20012

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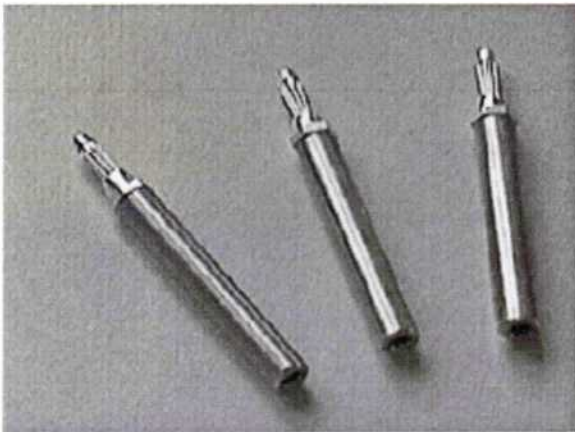
Connector socket for banana plug
part no. 38580

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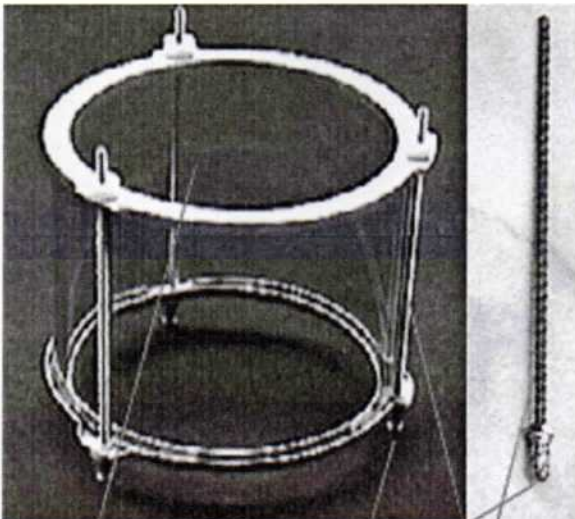Testadapter for check of the halogen bulb supply voltage and
check of the switch over function to reserve bulb
part no. 50574

1 Filterglasgruppe, 4500 Kelvin ohne ThermoSorb,
Teile Nr. 43041
filterglas assembly, 4500 Kelvin, without ThermoSorb,
part no. 43041

1 Filterglasgruppe, 3600 Kelvin, Teile Nr. 48893
filterglas assembly, 3600 Kelvin, part no. 48893

1 Filterglasgruppe, ThermoSorb, Teile Nr. 48893
filterglas assembly, ThermoSorb, part no. 48893

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
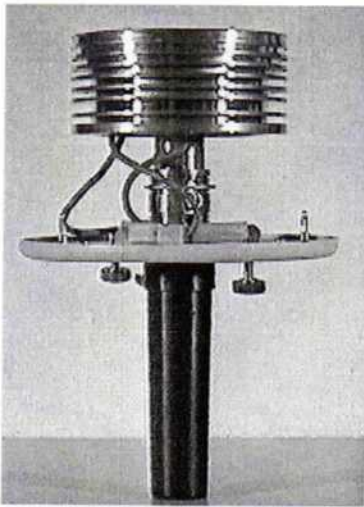
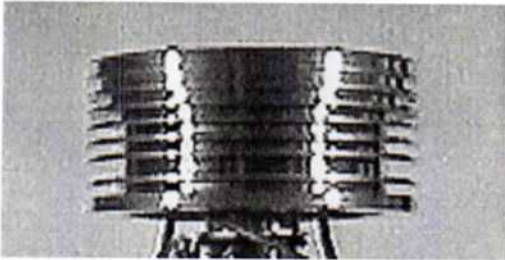
Fokus mechanism for C 450 standard, without halogen bulbs
part no. 54404

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
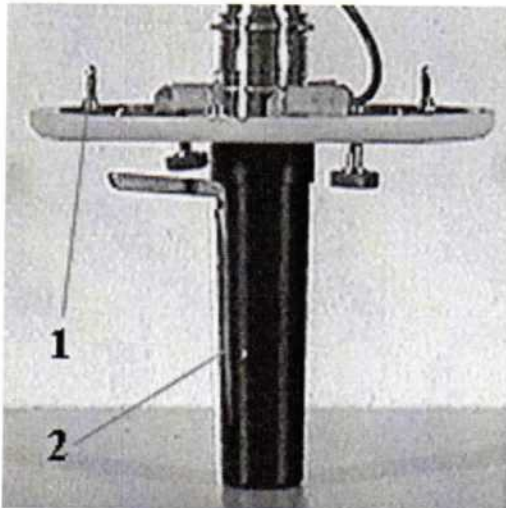
Halogen bulb reflector without bulb holders, part no. 54468
bulb holder see page 14

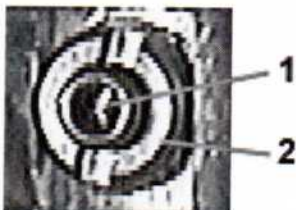- 1 Bündelstecker, Teile Nr. 38598
Banana plug, part no. 38598
- 2 Handgriffaufnahme, Teile Nr. 40769
Fixation mechanism for sterilizable handle, part no. 40769

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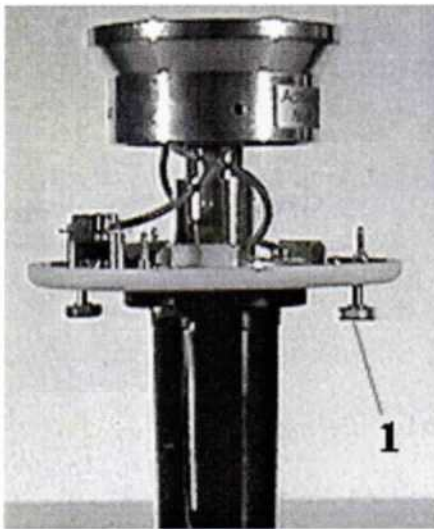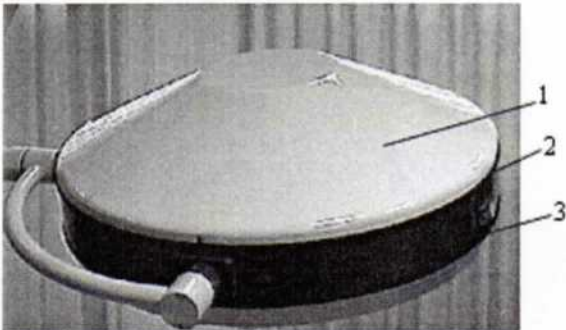
fixation for sterilizable handle and camera for C 450, focus not changable, only for cameras of the productline ChromoVision with fixed focus lens, without halogen bulbs, part no 54467

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
Security lock – spring balanced arm, part no. 20010

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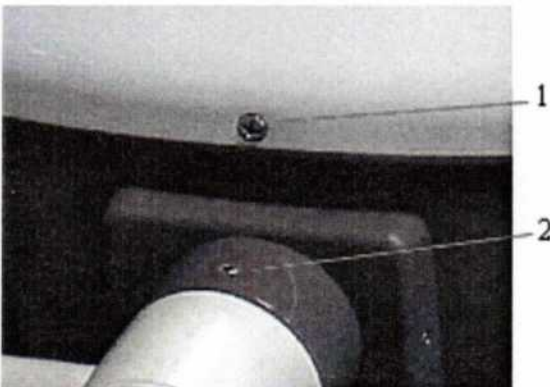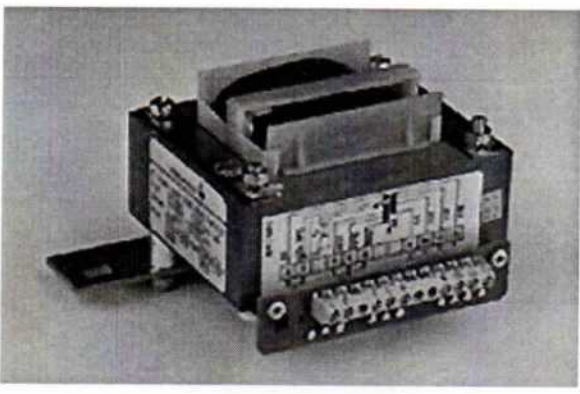
Fixation ring for ceiling cover for ceiling tube ø 65 mm
part no. 50175

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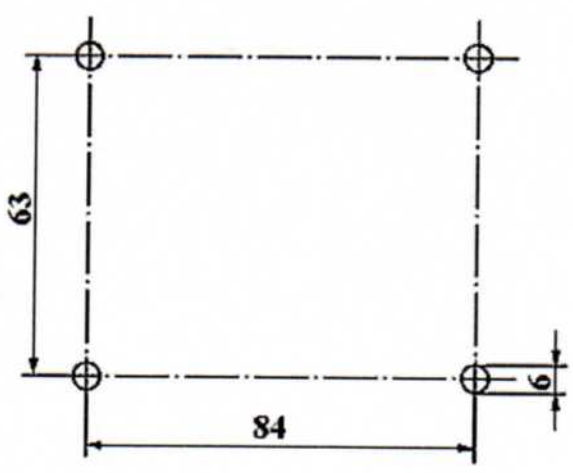
Trafo 110/120/130 Volt, 140 VA separat montiert
Teile Nr. CB 506-13

Transformer 220/240 Volt, 140 VA mounted on the flange
tube
part no. CB 502-13

Transformer 220/240 Volt, 140 VA mounted separately
part no. CB 505-13

Transformer 110/120/130 Volt, 140 VA mounted on the
flange tube, part no. CB 503-13

Transformer 110/120/130 Volt, 140 VA mounted separately
part no. CB 506-13

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
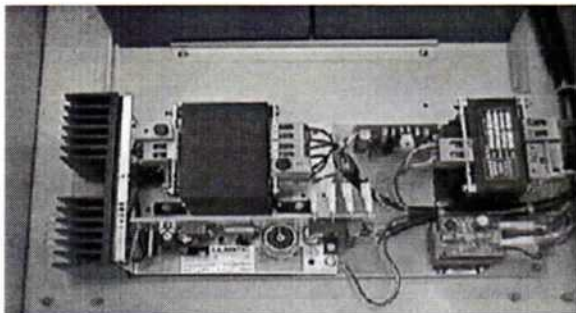
Cover for male sliding contact, part no. 44306
Only for sigle CHROMOPHARE® C 450

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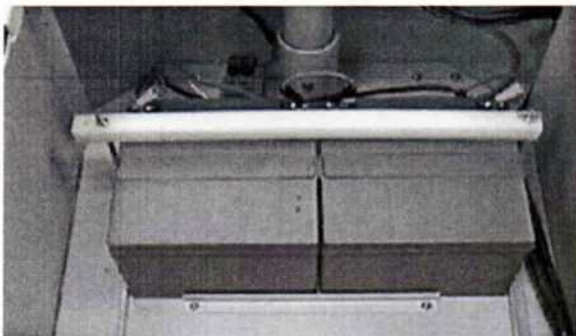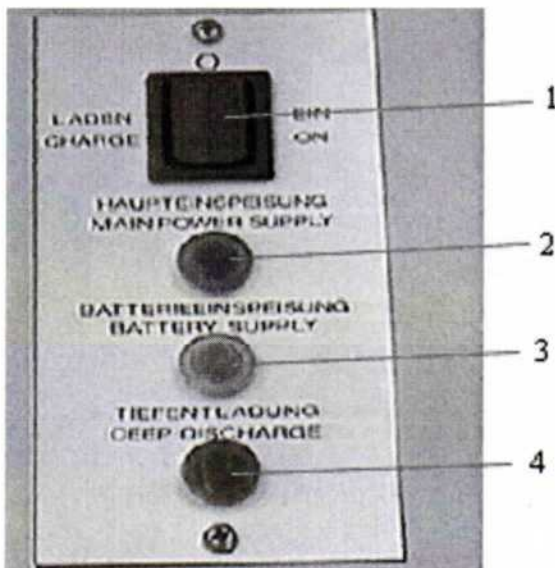
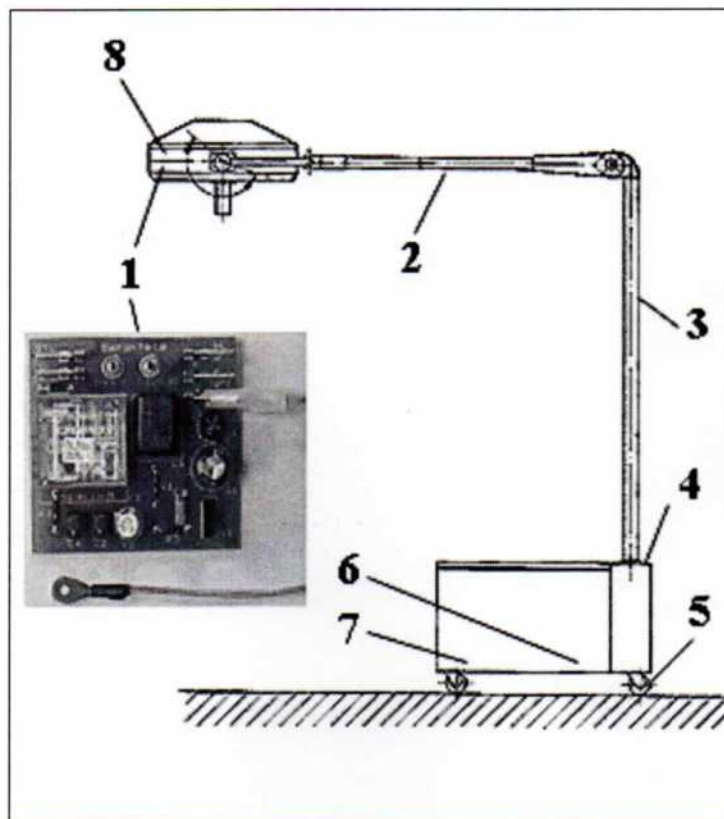
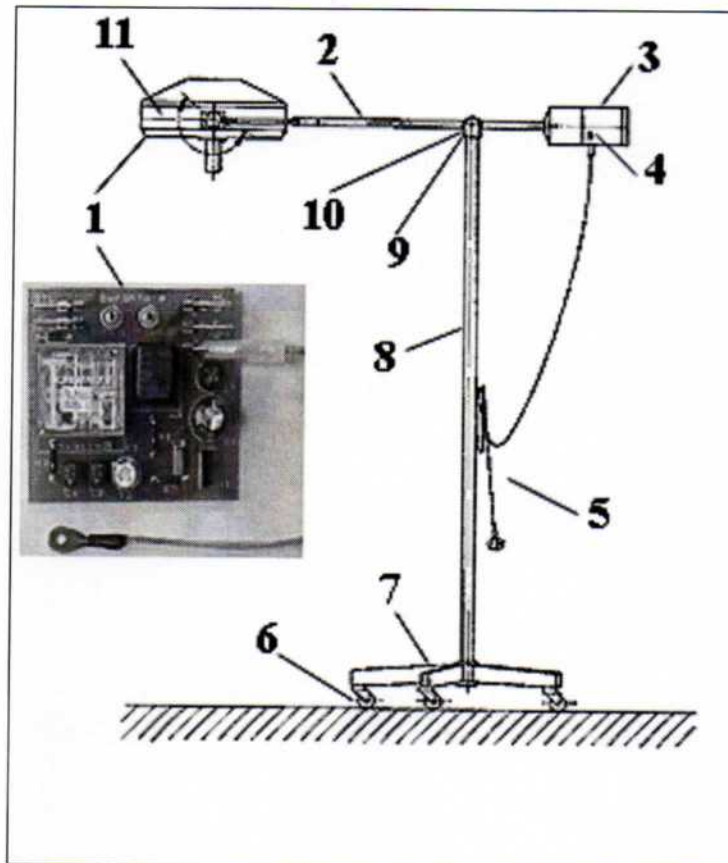
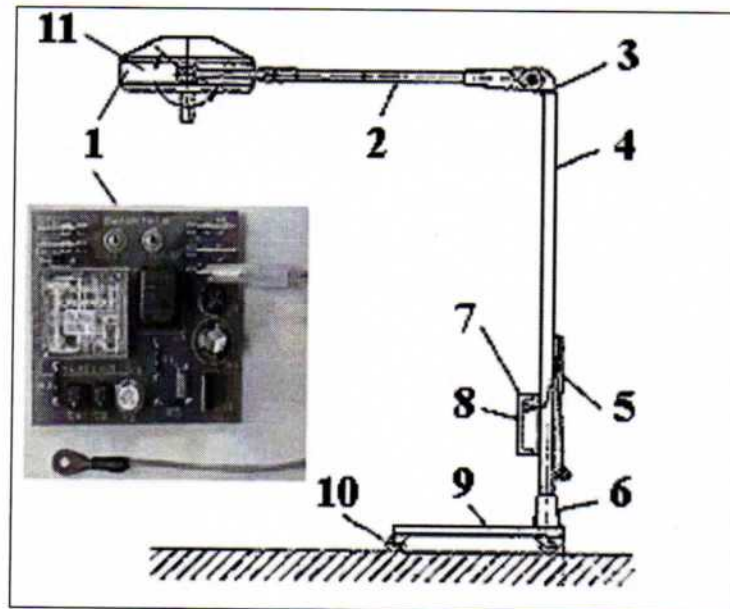
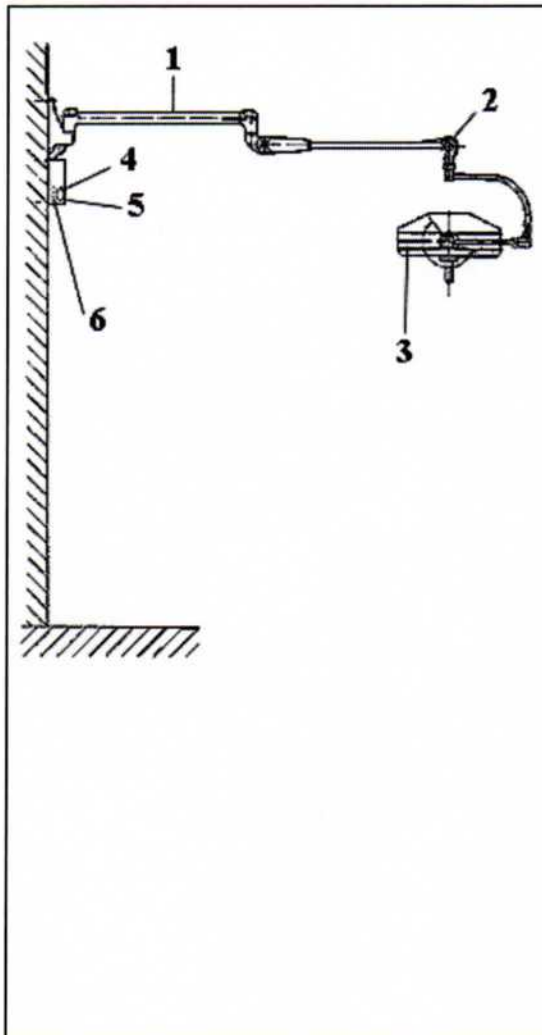
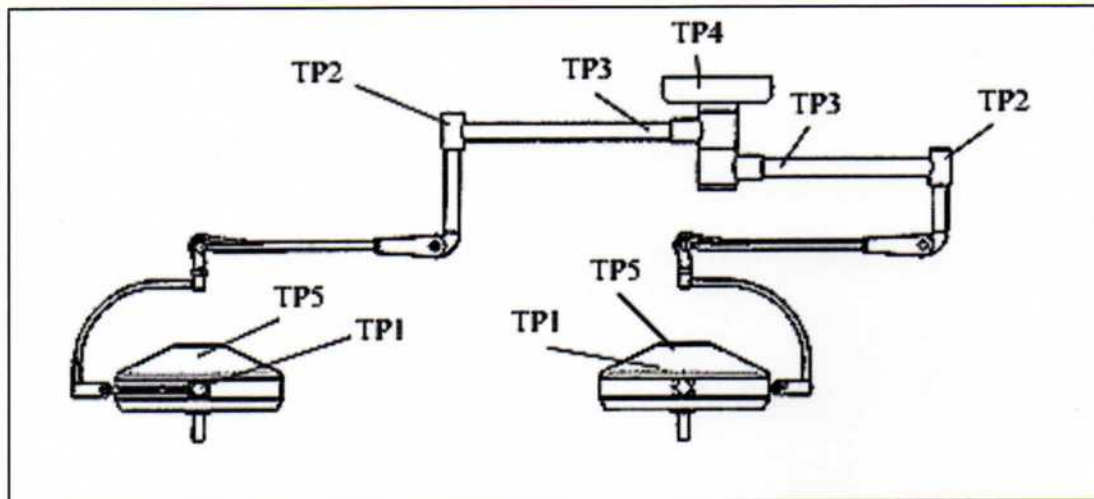
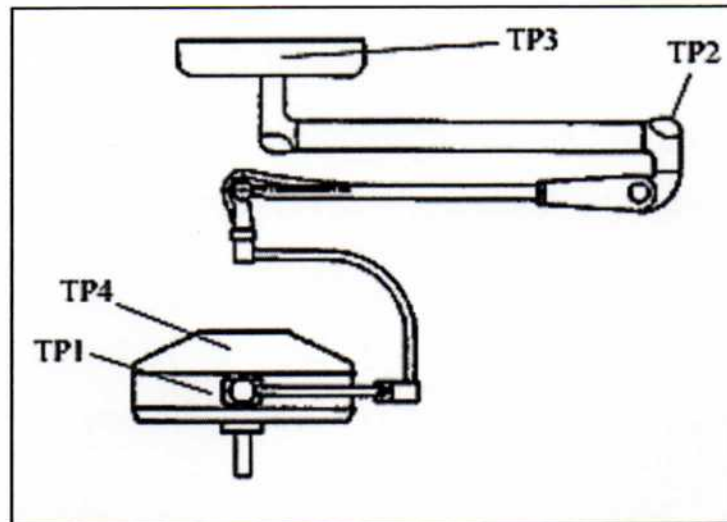
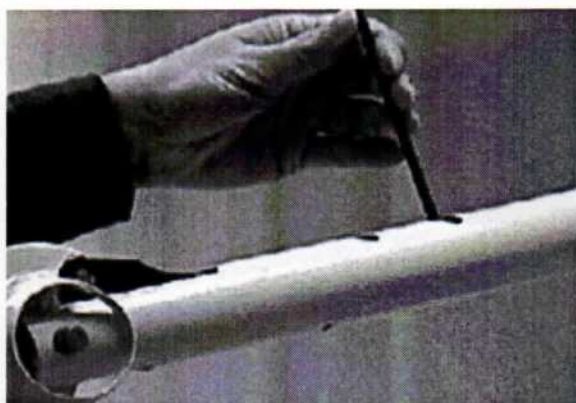
By pulling the round side caps slightly apart the main covering hull can be lifted off of the spring arm

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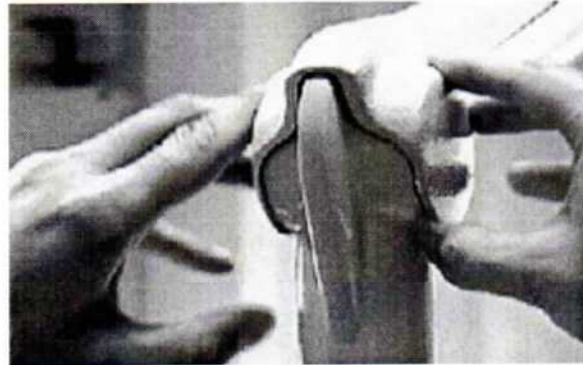
Adjustment:

Light head drifts upward too easily:
Move the acrobat arm upward or downward as far as the circular nut becomes visible in one of the two windows. Turn this circular nut with a steel pin, diameter 4 mm, in the direction of the minus symbol.


Light head moves downward:
Turn the circular nut in the direction of the plus symbol.

8. Einstellung des Höhenanschlags am Federarm Adjustment of the height movement on the acrobat arm

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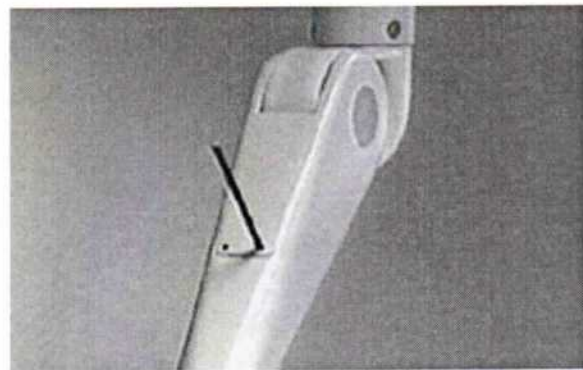Remove the cover plate by using the steel pin out of the covering hull.

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Move the acrobat arm upward or downward as far as the circular nut becomes visible. Turn this circular nut with the steel pin to limit the height movement of the acrobat arm.

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