6.3 ML 700 - Version 3

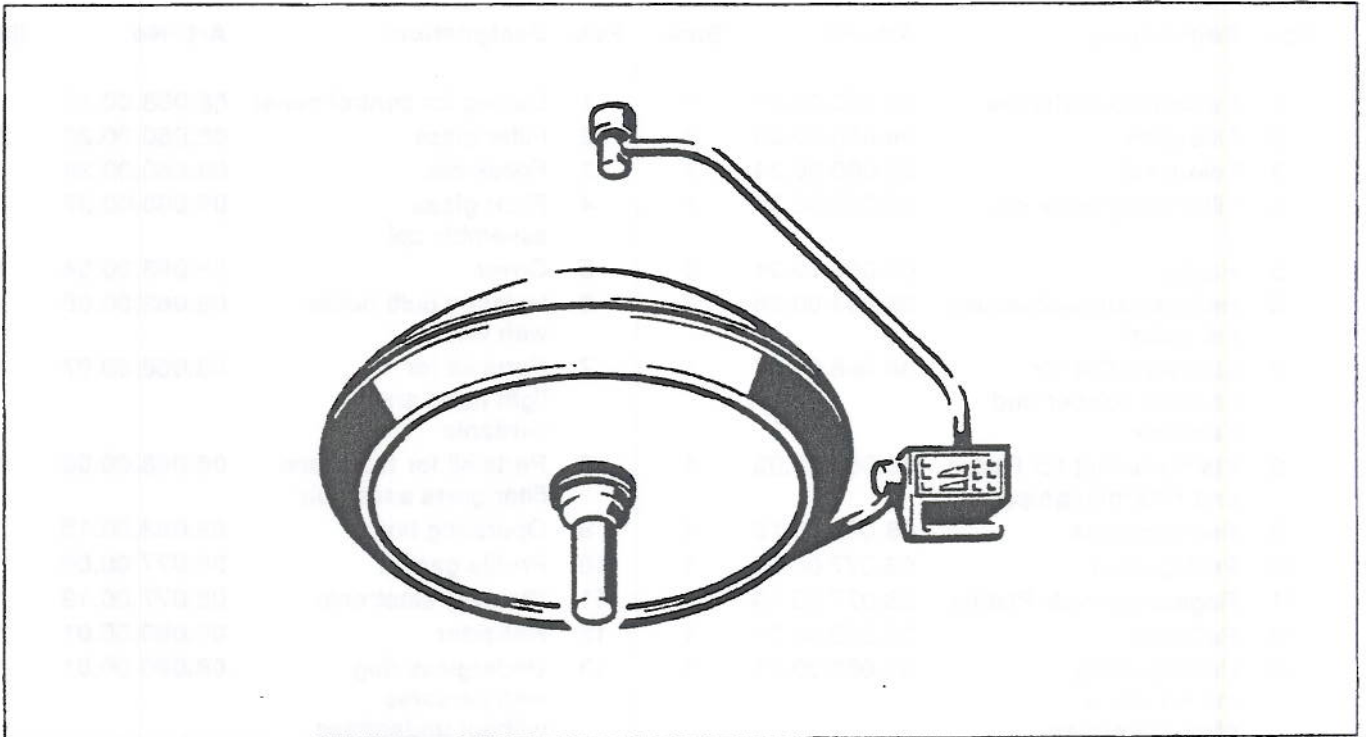
6.3 ML 700 - Version 3

6. Operationsleuchte ML 700

6. Operating Light ML 700

**6.3 ML 700 - Version 3
(mit Elektronik F)**

**6.3 ML 700 - Version 3
(with F electronics)**

6.3 ML 700 - Version 3
(mit Elektronik F)

- Stückliste

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6.3 ML 700 - Version 3
(with F electronics)

- Parts List

Pos.	Designation	Art.-No.	Item
1	Casing for control panel	08.053.00.10	1
2	Filter glass	08.060.00.20	3
3	Focus cpl.	08.060.00.34	1
4	Filter glass assembly cpl.	08.060.00.37	1
5	Cover	08.063.00.04	1
6	Halogen bulb holder with wiring	08.063.00.05	2
7	Parts kit for light head and cardanic	08.068.00.07	1
8	Parts kit for focus and filter glass assembly	08.068.00.08	1
9	Operating board	08.053.00.15	1
10	Profile gasket	08.077.00.06	1
11	Steering electronic	08.077.00.13	1
12	Reflector	08.080.00.01	1
13	Underglass ring with cardanic without underglass without filter glass assembly	08.090.00.01	1
14	Underglass	08.090.00.04	1
15	Halogen bulb 24 V/150 W	86.907.00	2
16	Sterilisable handle	86.990.00	1

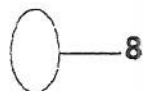
Note

The parts of the parts kits are designated by symbols in the following dismounting/mounting diagrams.

Parts kit for the **light head** and the **cardanic**:

Parts kit for the **focus** and the **filter glass assembly**:

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
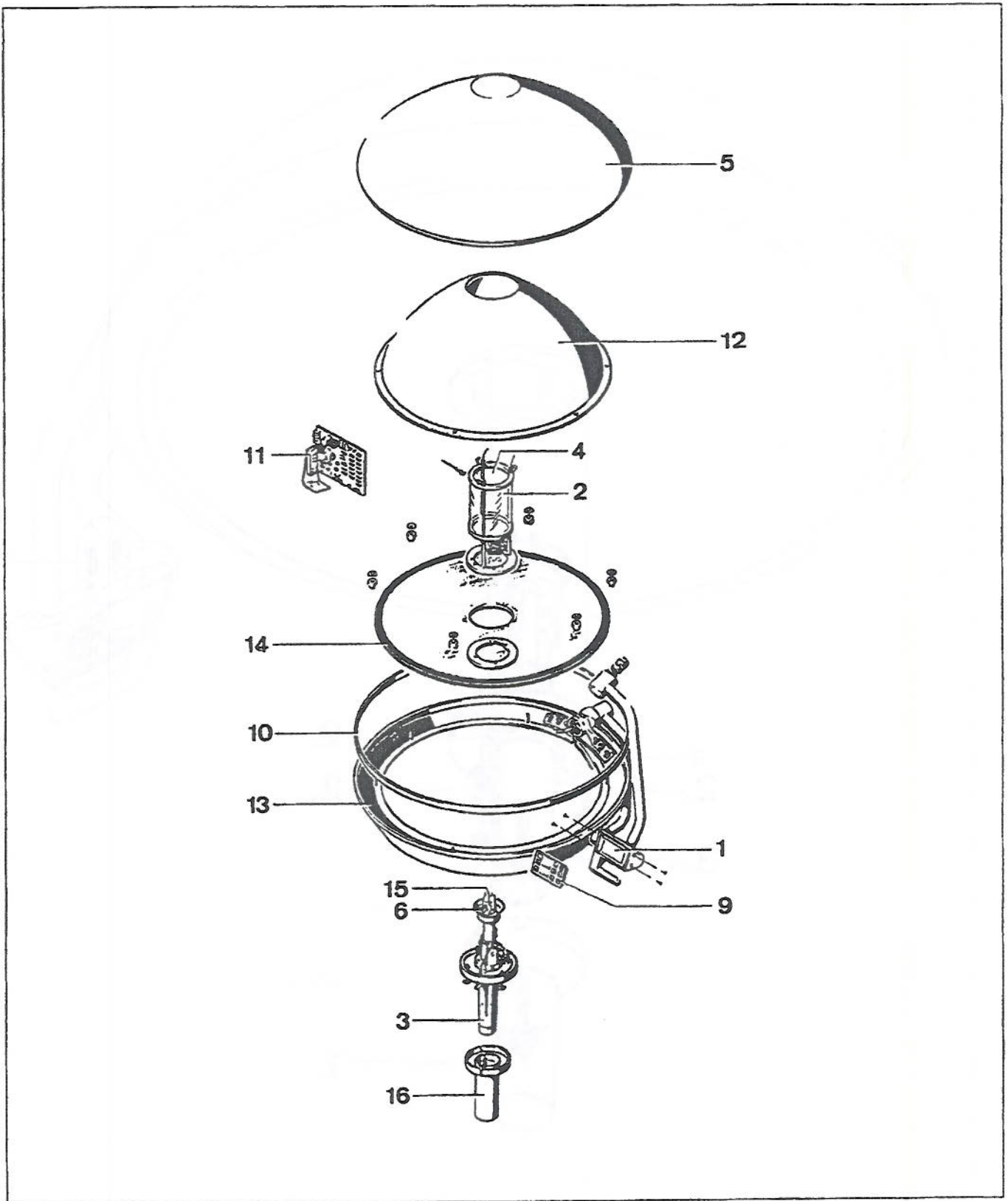
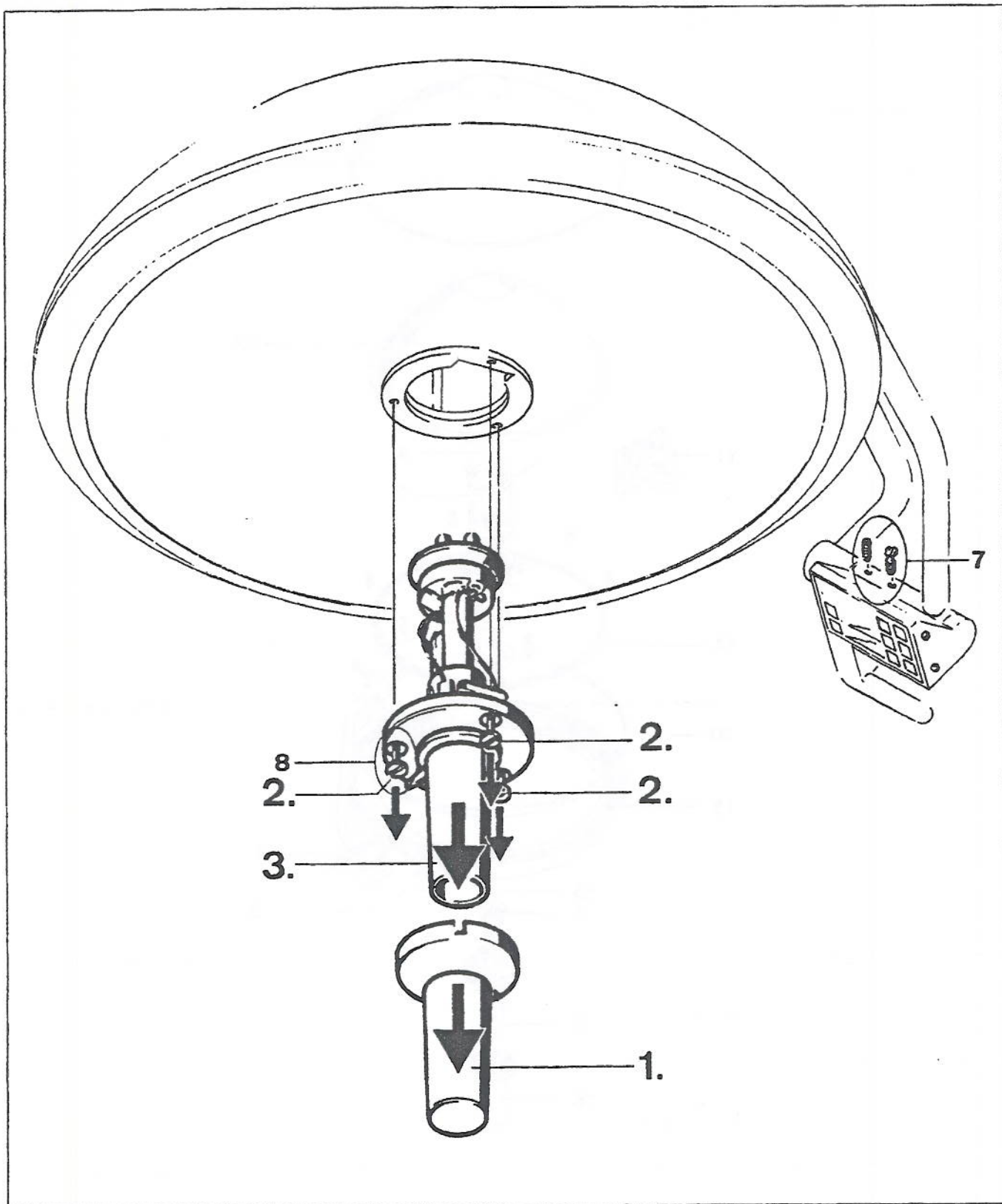
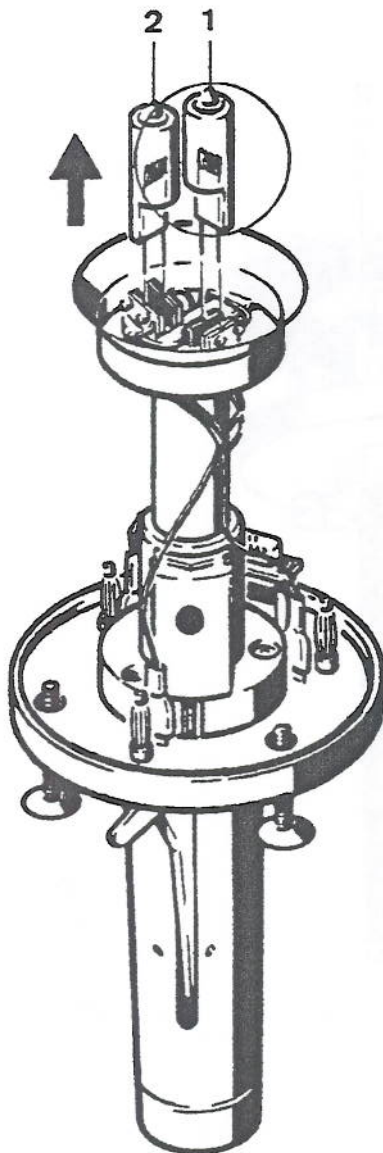
Note

Do not touch the halogen bulbs with bare hands! Use protective clothing when installing!

For the ML 700, use only halogen bulbs with 24 V/150 W!

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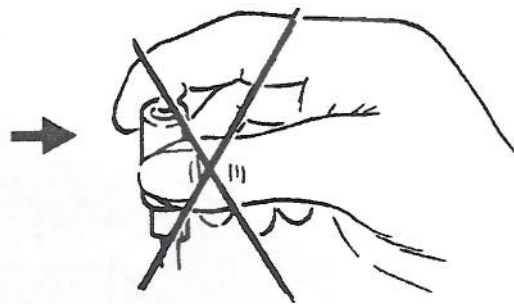Replacement of the halogen bulbs in the focus

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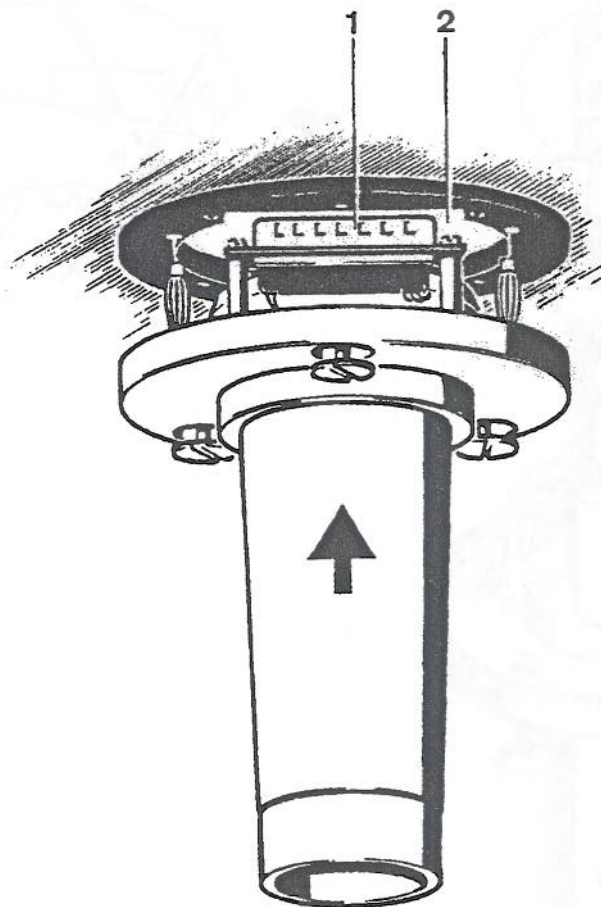
Note

When mounting the focus, be sure that the male plug (1) is installed in the right-angled recess (2) of the focus holder.

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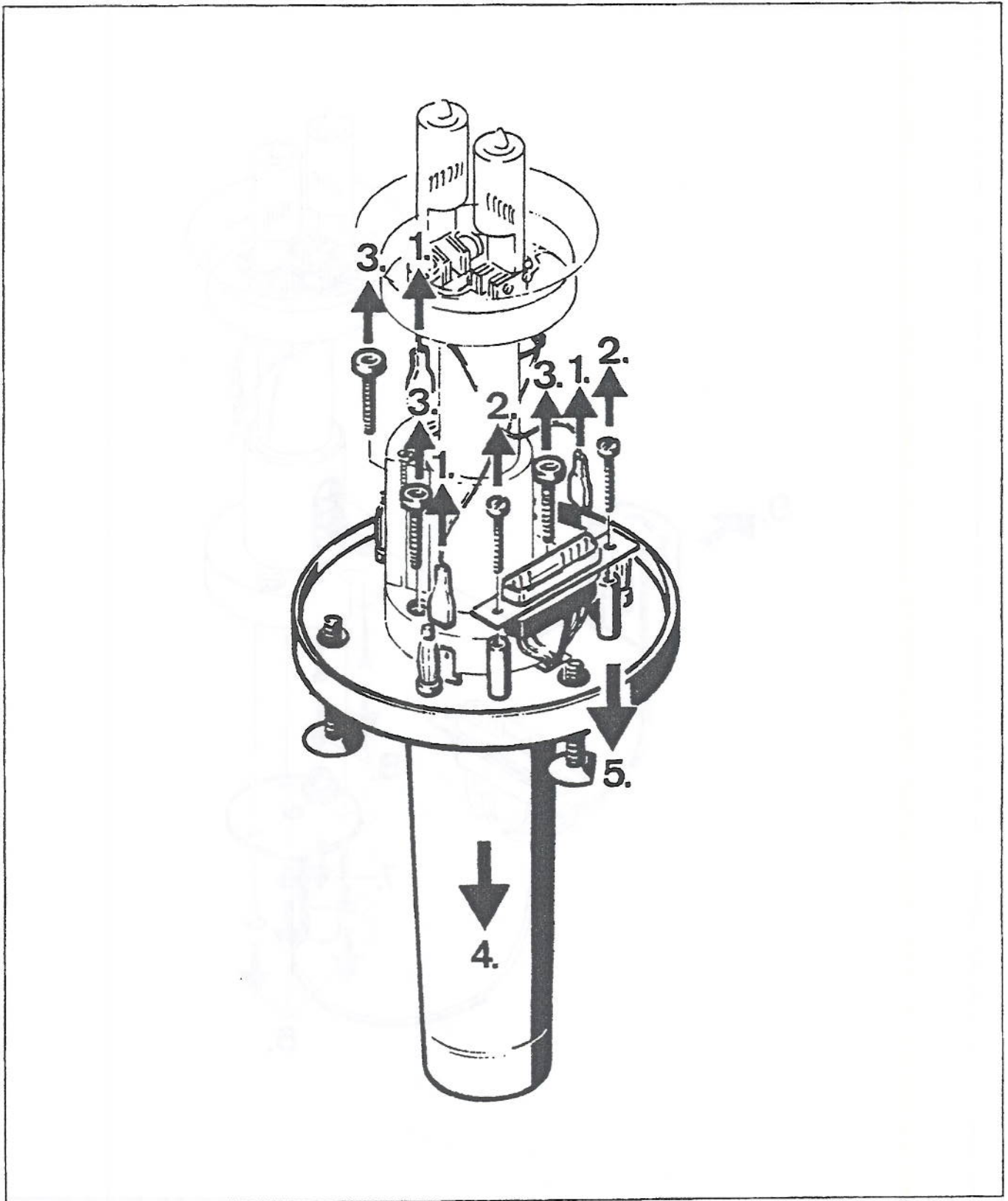
Installation of the focus
- only when mounting

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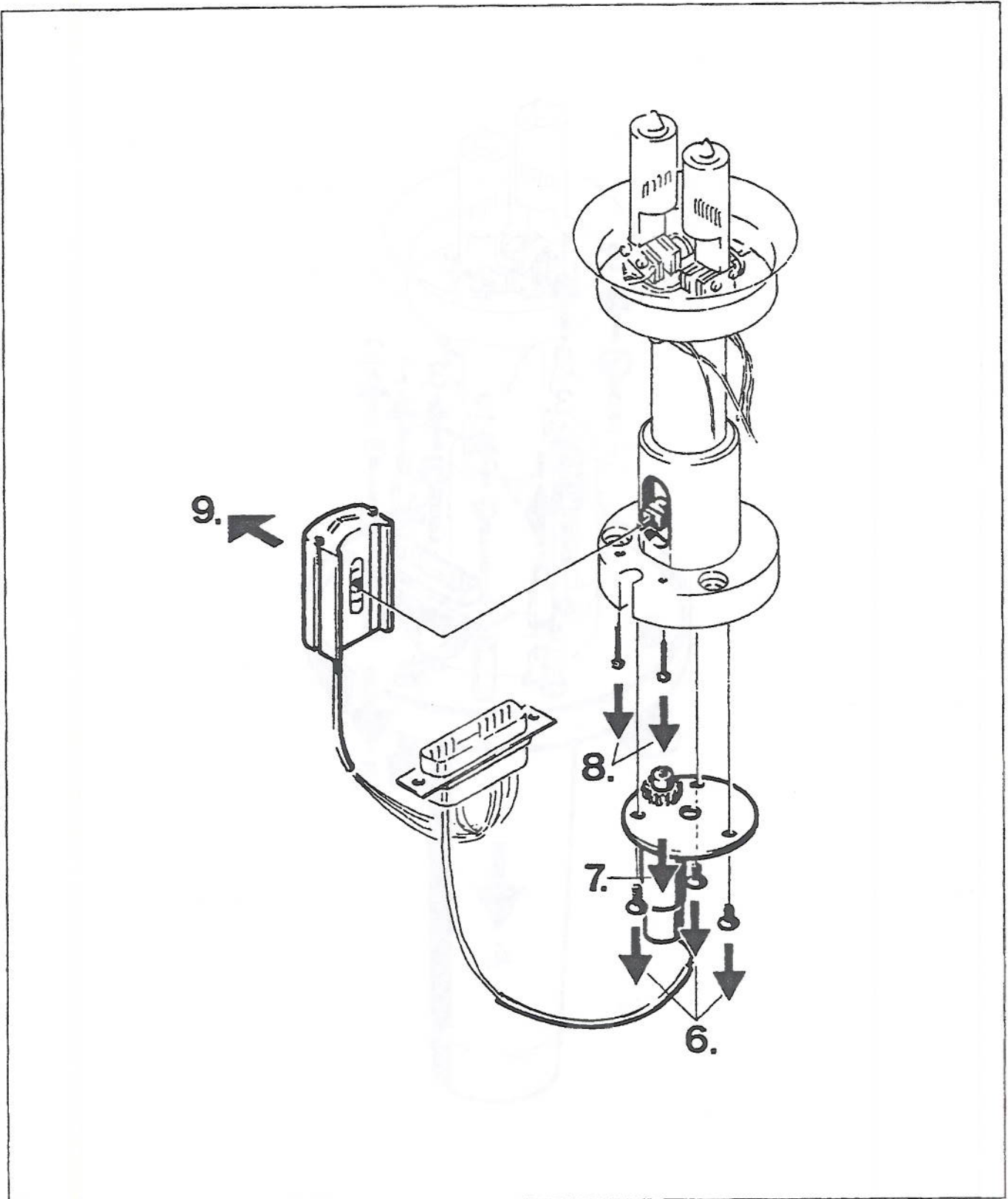
Dismounting of position transmitter cpl. from focus

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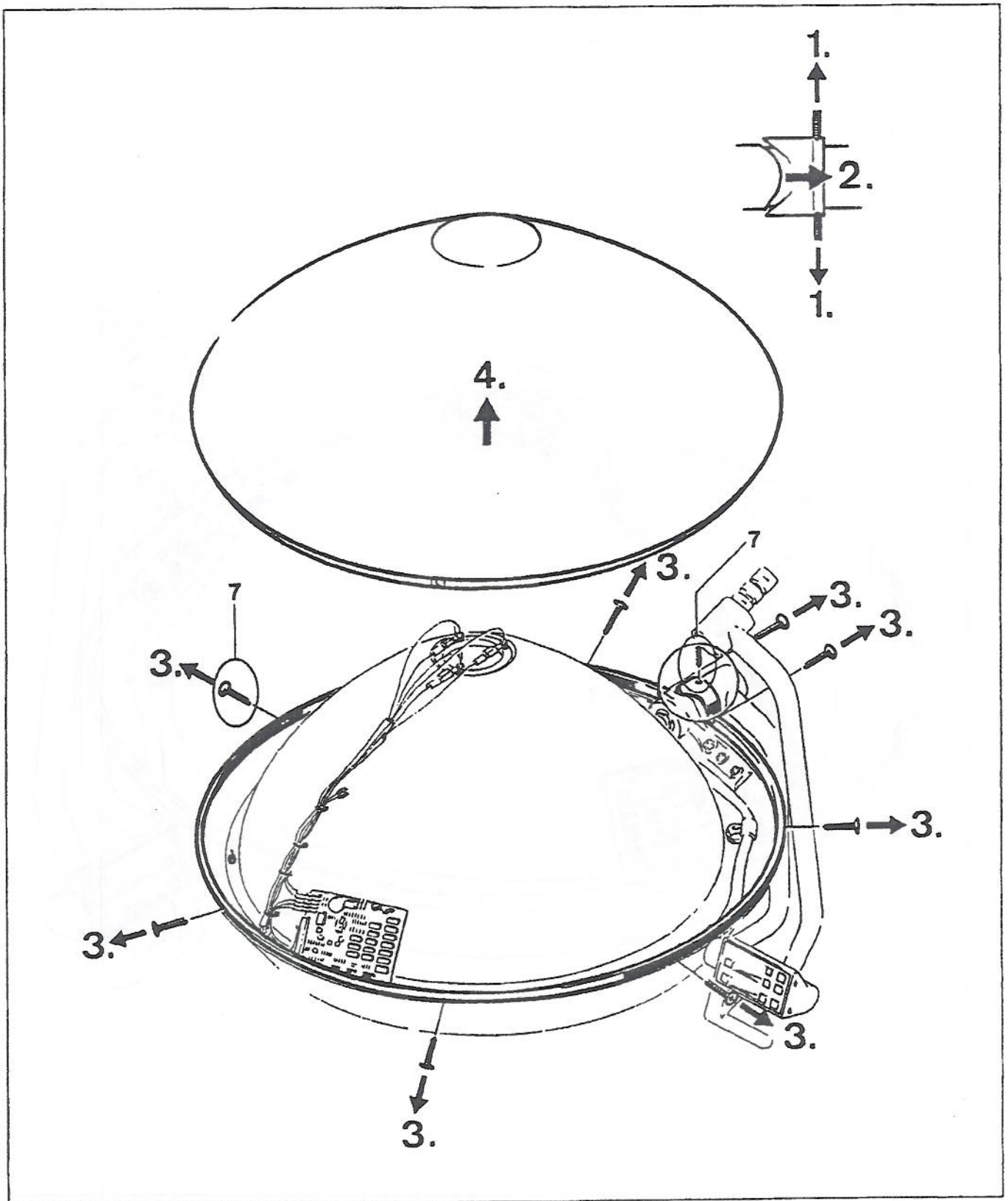
Dismounting of position transmitter cpl. from focus
(Continuation)

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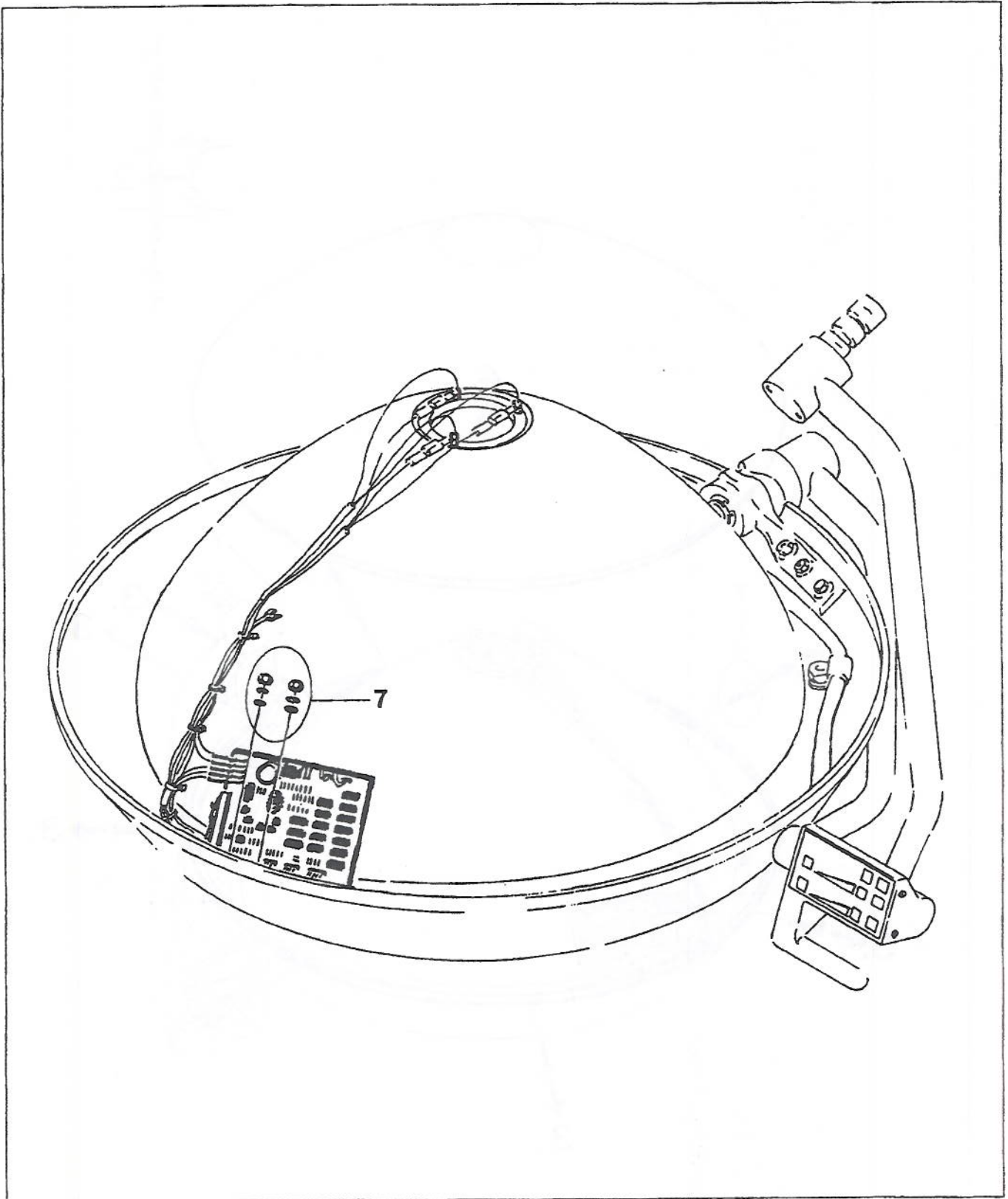
Wiring and dismounting the printed circuit board

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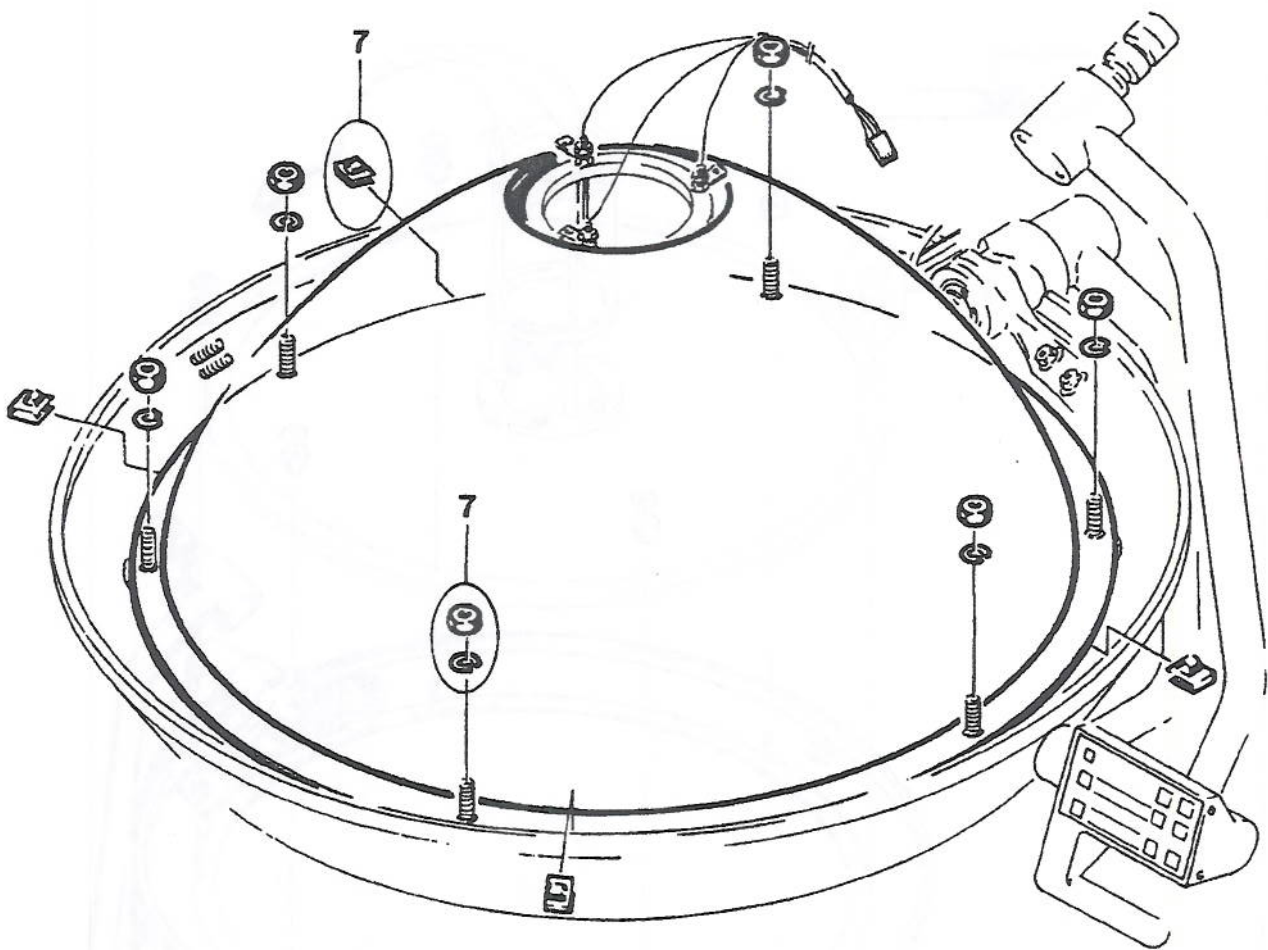
Note

When reinstalling the reflector, be sure that the inside of the underglass, the reflector and the filter glasses are clean.

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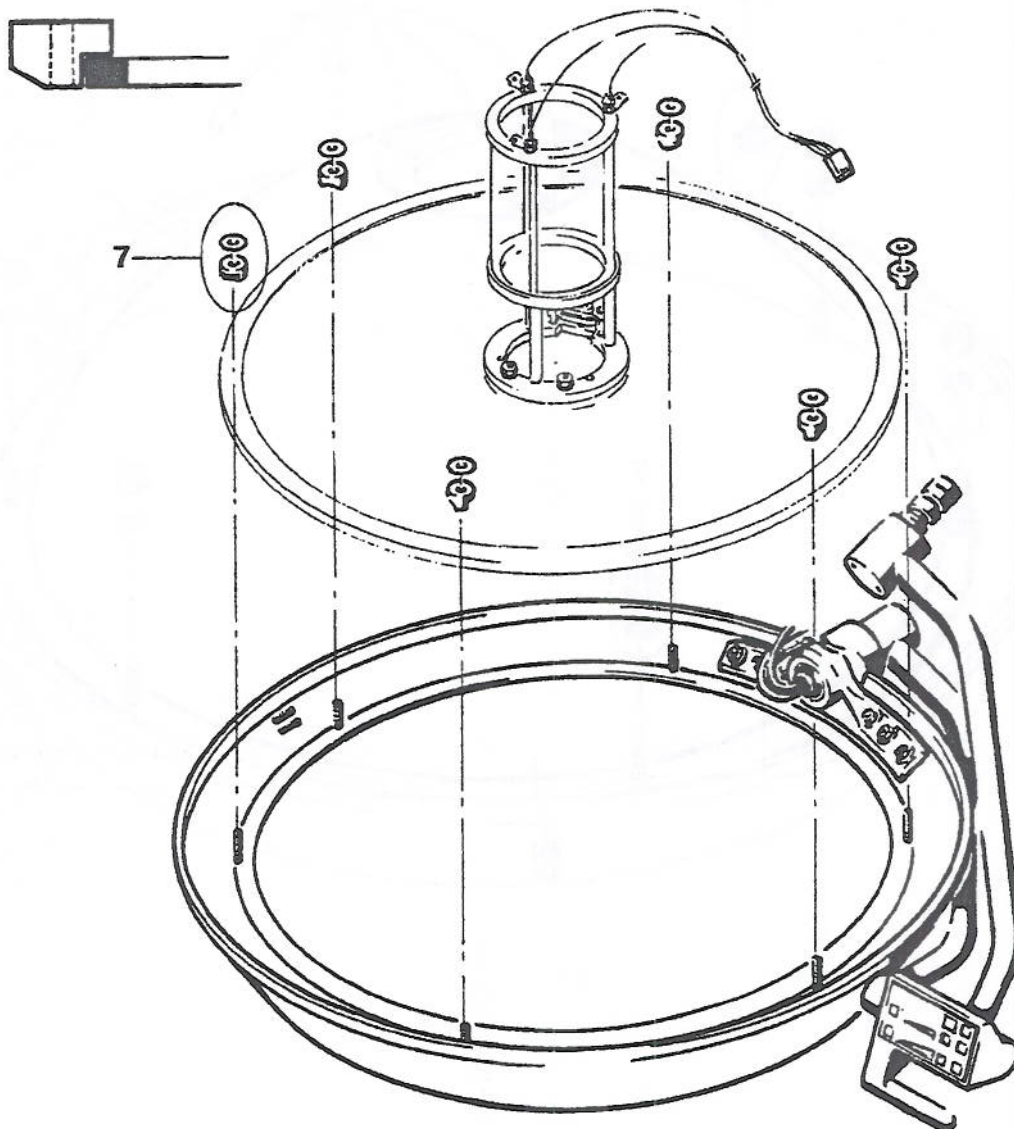
Attention
For reasons of safety, installation of the underglass can be carried out only by Martin personnel! The exchange of this glass by personnel of other services is not allowed!
In this case, the light head is to be sent back to the manufacturer.

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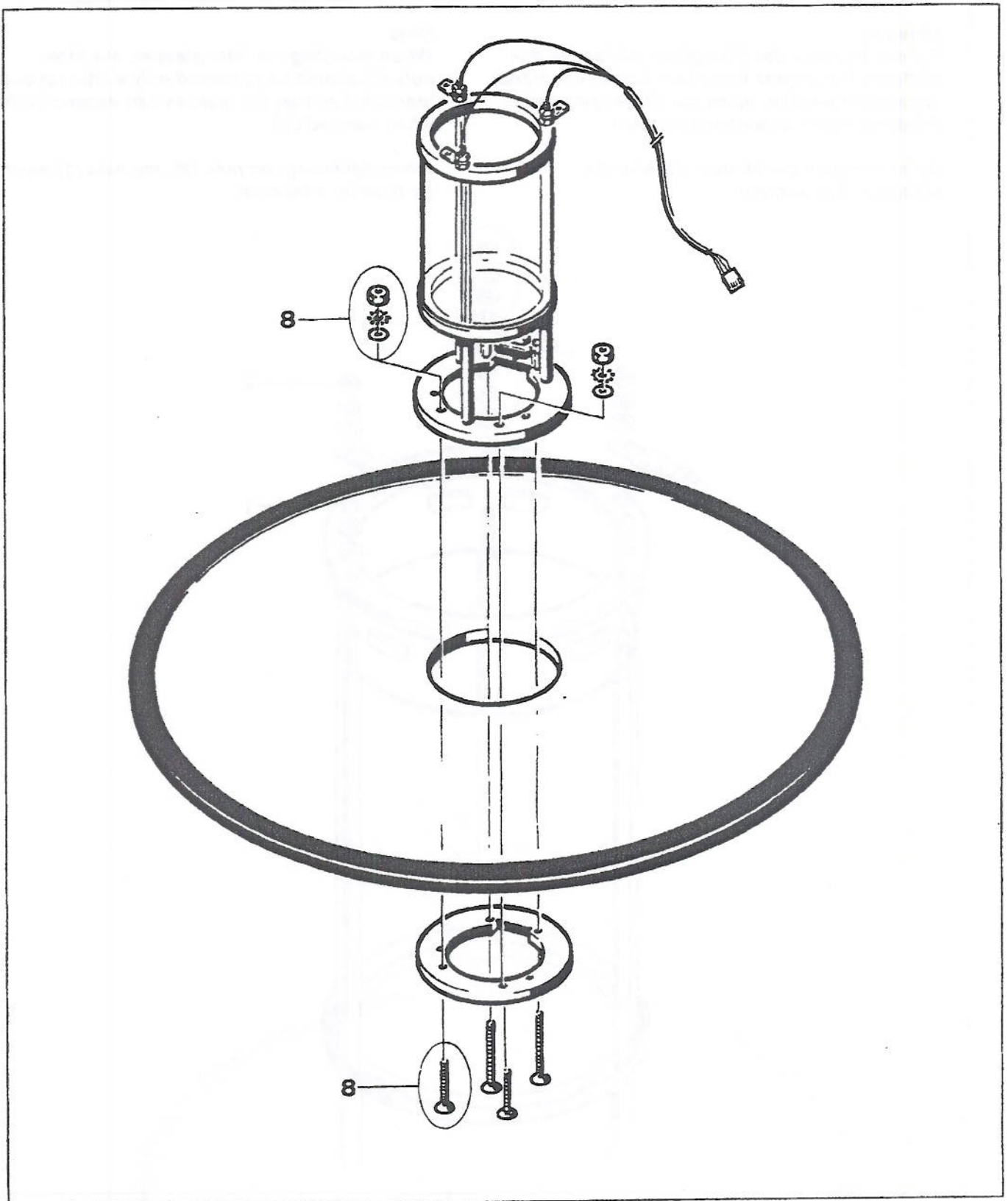
Removal of the spacer bushings and of the underglass

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Removal of the filter glass assembly from the underglass

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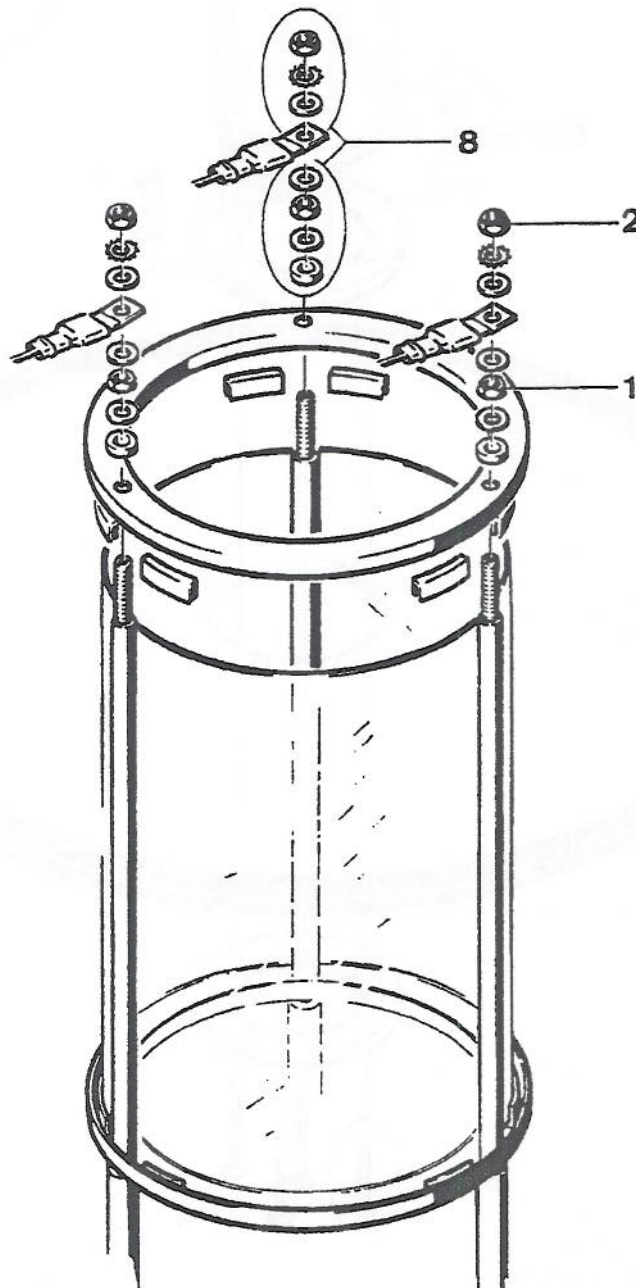
Note

When mounting the filter glasses, the three nuts (1) should be tightened only a little (at best manually) so that the glasses can expand further when warmed up!

When tightening the nuts (2), the nuts (1) must be fixed by a locknut!

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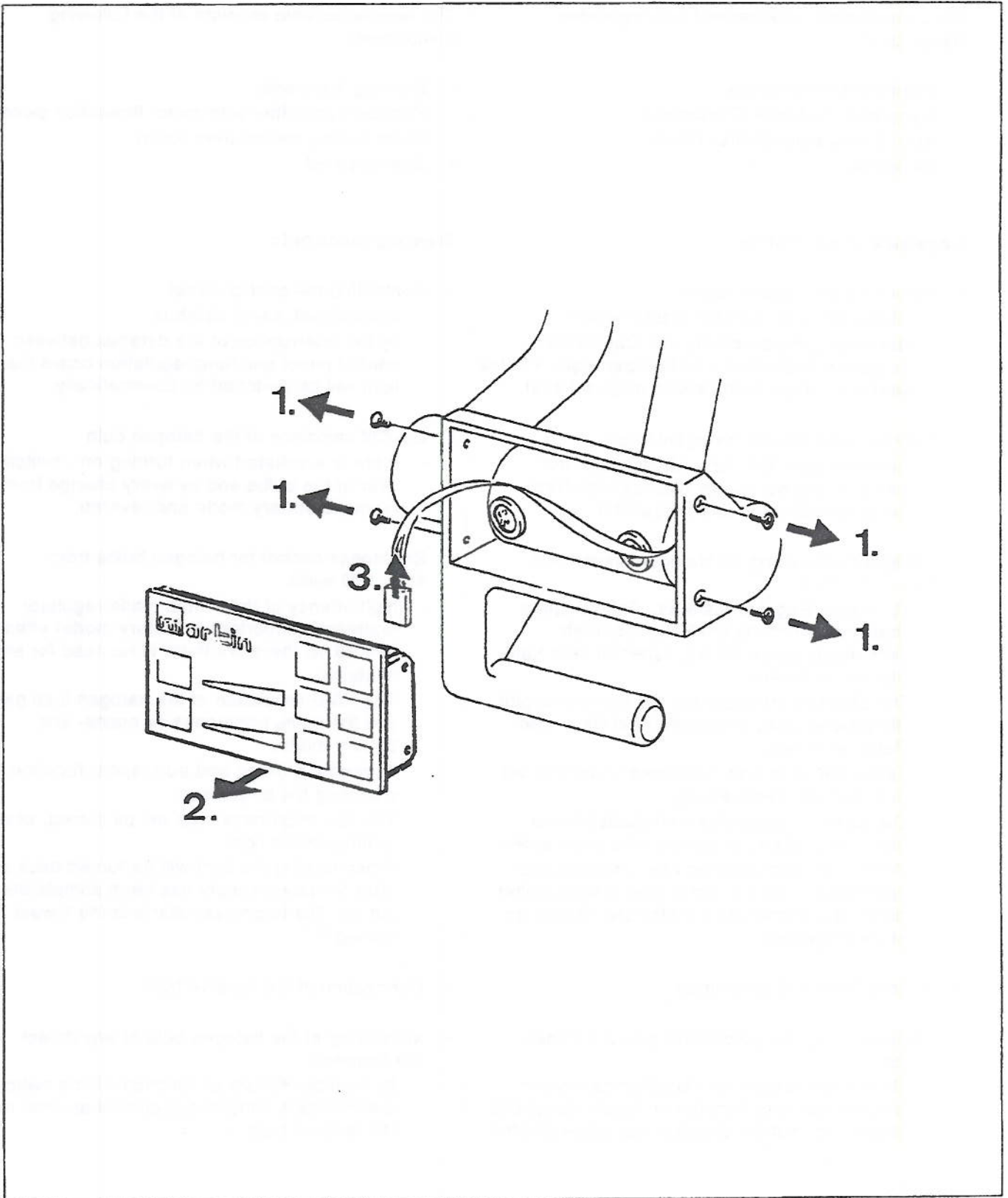
Removal of the service part with Electronics F

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6.3 ML 700 - Version 3 (with F electronics)

- Description of function

The lamp-electronic consists of the following components:

- Steering electronic
- Position transmitter with motor limitation switch
- Mains battery switch over board
- Control panel

Steering electronic

- Controlling the control panel
 - bidirectional, serial databus
 - by the interruption of the databus between control panel and lampregulation board the light will be switched on automatically.
- On-/Off switching of the halogen bulb
 - there is a softstart when turning on - switch over of the bulbs and by every change from mains to battery mode and reverse.
- Brightness control for halogen bulbs from 10 ... 150 watts
 - high efficiency of the switch made regulator (extremely important in battery mode) efficiency rate 99 %, therefore there is no need for any heatsink.
 - The RMS-regulation of the halogen bulb gives you the same brightness by mains- and battery mode.
 - Keyboard storage and autorepeat-function for changing the brightness.
 - The last brightness-step will be stored, when turning off the light.
 - When turning the light will be turned back on, after the powersupply has been completely cut off. The brightness starts in the lowest setting.
- Connection of the reserve bulb
- Monitoring of the halogen bulb of any defect (i.e filament)
 - by mainbulbfailure all function will be switched automatically (brightness-control as well) to the reserve bulb.

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6.3 ML 700 - Version 3 (with F electronics)

- Description of function

- Lightfield adjustment
 - connection for the motor (lightfield)
 - currentlimitation of the motorcurrent (shortcircuit proofed)
 - keyboard storage and autorepeat-function for lightfield adjustment
 - no calibration necessary when turning on, since the information is present as gray-code.
 - Position transmitter limitation thru motor limitation switches.

Position transmitter with motor limitation switch

- The position transmitter is mounted in a profilchassis, which consists of the gray code and two limitation switches for the motor (12 V/70 mA).
- There is no calibration necessary when turning on, since the information is present as gray-code.
- There is only a minimal force necessary, caused by the parallel guided teflon-shider.
- The position transmitter is maintenance free.

In case of servicing, the position transmitter must be changed completeley (Art.-No. see parts list).

Main battery switch over board

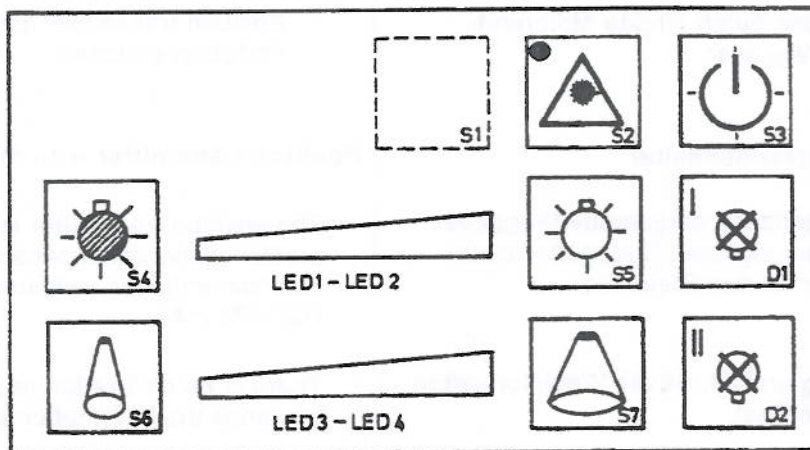
- Supply of lamp regulation board.
- Supervision of the rectified alternating current and switch over to battery current when circuit breakdown.
- Buffering the battery current through electrolyte-condenser.
- Suppression of interferences of wires to main circuit and battery with symmetrical throttle valve.
- Protection of polarity at the output of lamp electronic.
- Load impedance to improve the synchronicity of steering electronic on main supply.

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- S1 not active in this version
- S2 not active in this version
- S3 Control ON/OFF
(Main bulb LA 1 ON, 18 V)
- S4 Main bulb LA 1 darker (to min. 18 V)
- S5 Main bulb LA 1 lighter (up to max. 24 V)
- S6 Narrower focussing
- S7 Wider focussing
- D1 when red LED is on, the main bulb (LA 1) is defective
- D2 when red LED is on, reserve bulb (LA 2) is defective
- LED1-LED2 Indication of light intensity of bulb (continuous)
- LED3-LED4 Indication of focus setting (continuous)

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6.3 ML 700 - Version 3
(with F electronics)

- Installation Remarks

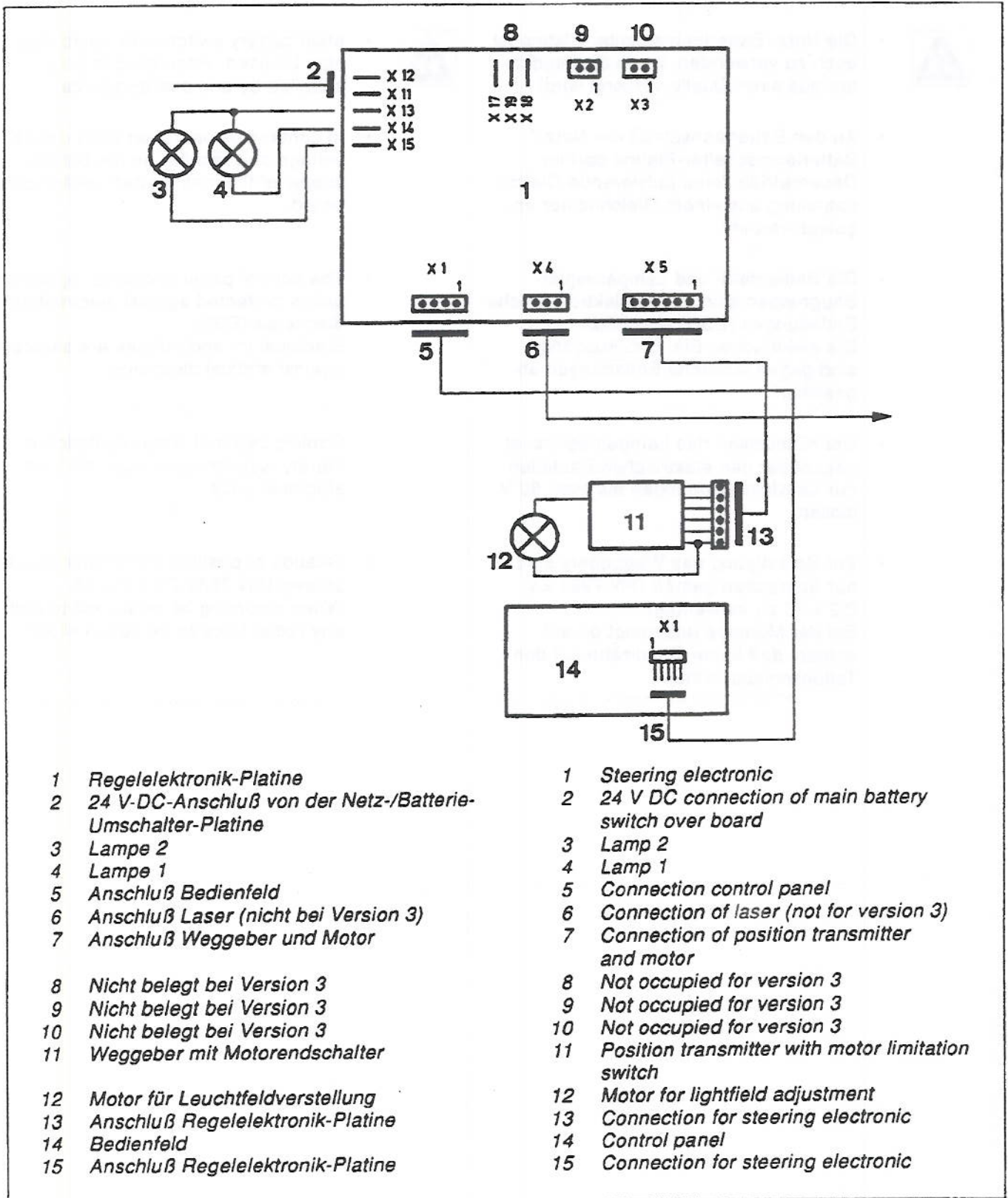
- Main battery switch over board may also be used, when lamp is only supplied by one energy source.
- In continuous operation don't use DC voltage of a rectifier on the battery supply of the main battery switch over board.
- The control panel and lamp regulation unit is protected against electrostatic discharge (ESD). Electrical in- and outputs are secured against statical discharge.
- Cooling beam of lamp regulation is merely isolated up to max. 60 V of electrical units.
- Fixation of position transmitter only with screws DIN 7981 ZV 2,2 x 13. When mounting be aware not to add any radial force to the teflon slider.

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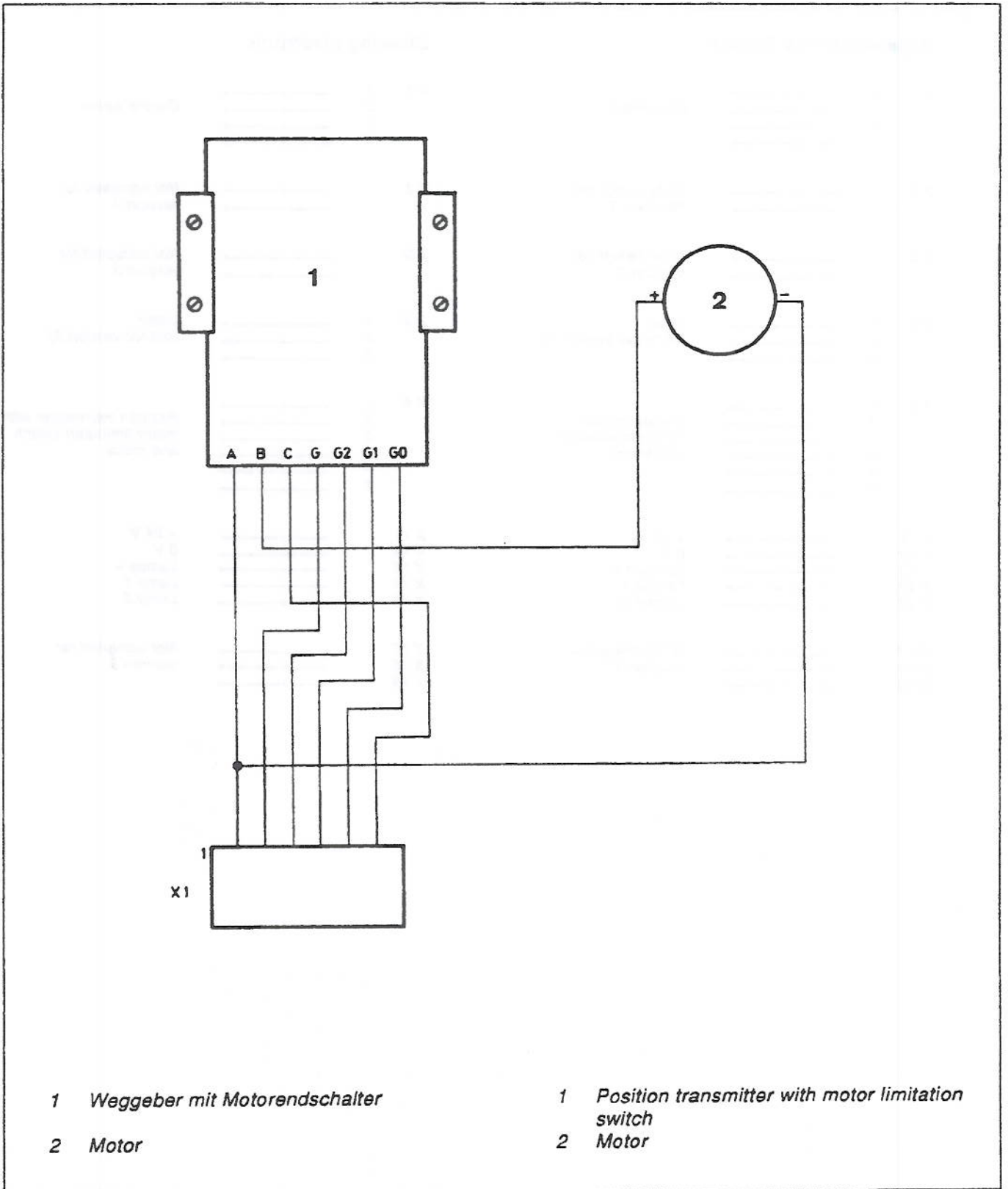
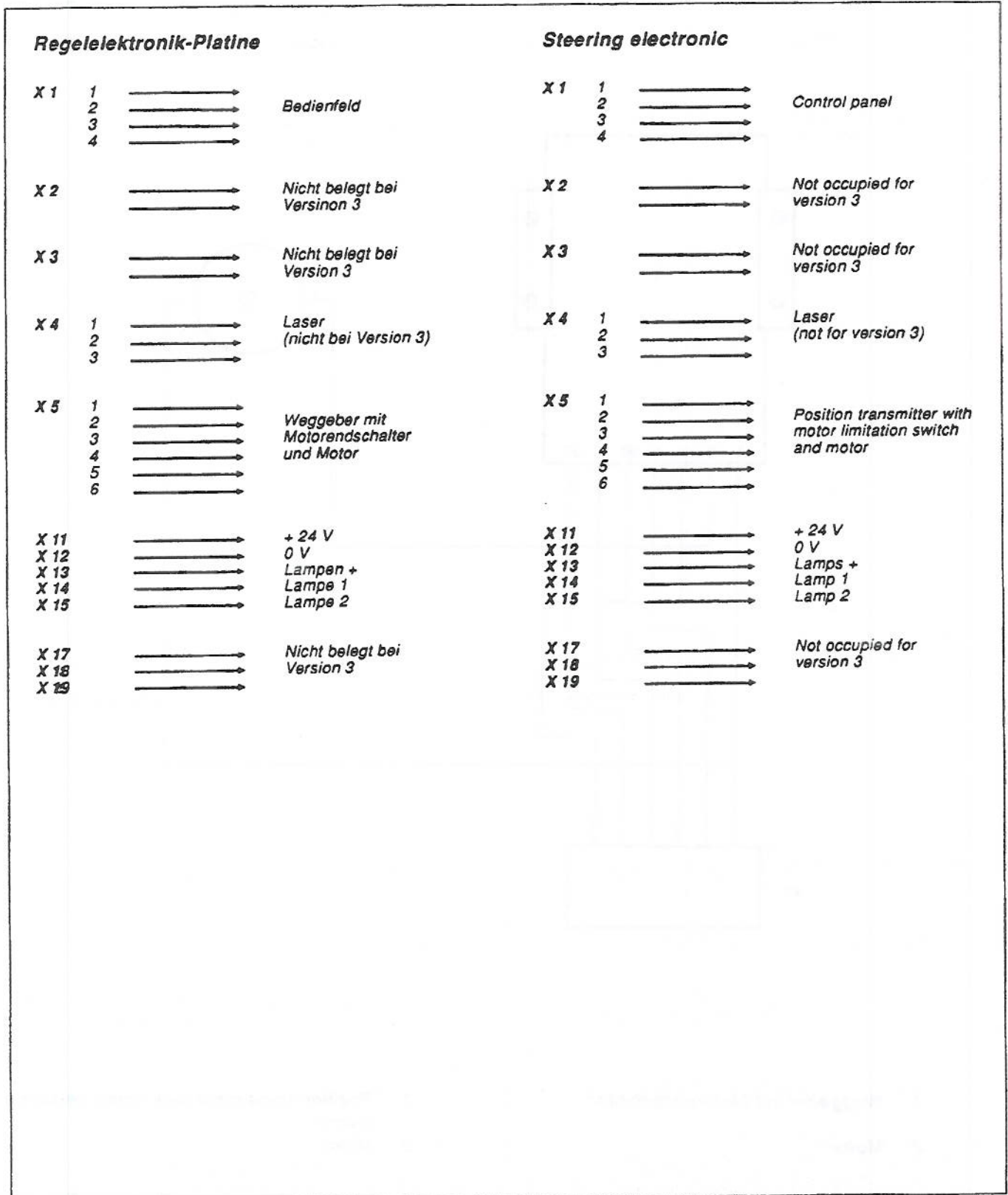
Wiring of position transmitter with motor limitation switch

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
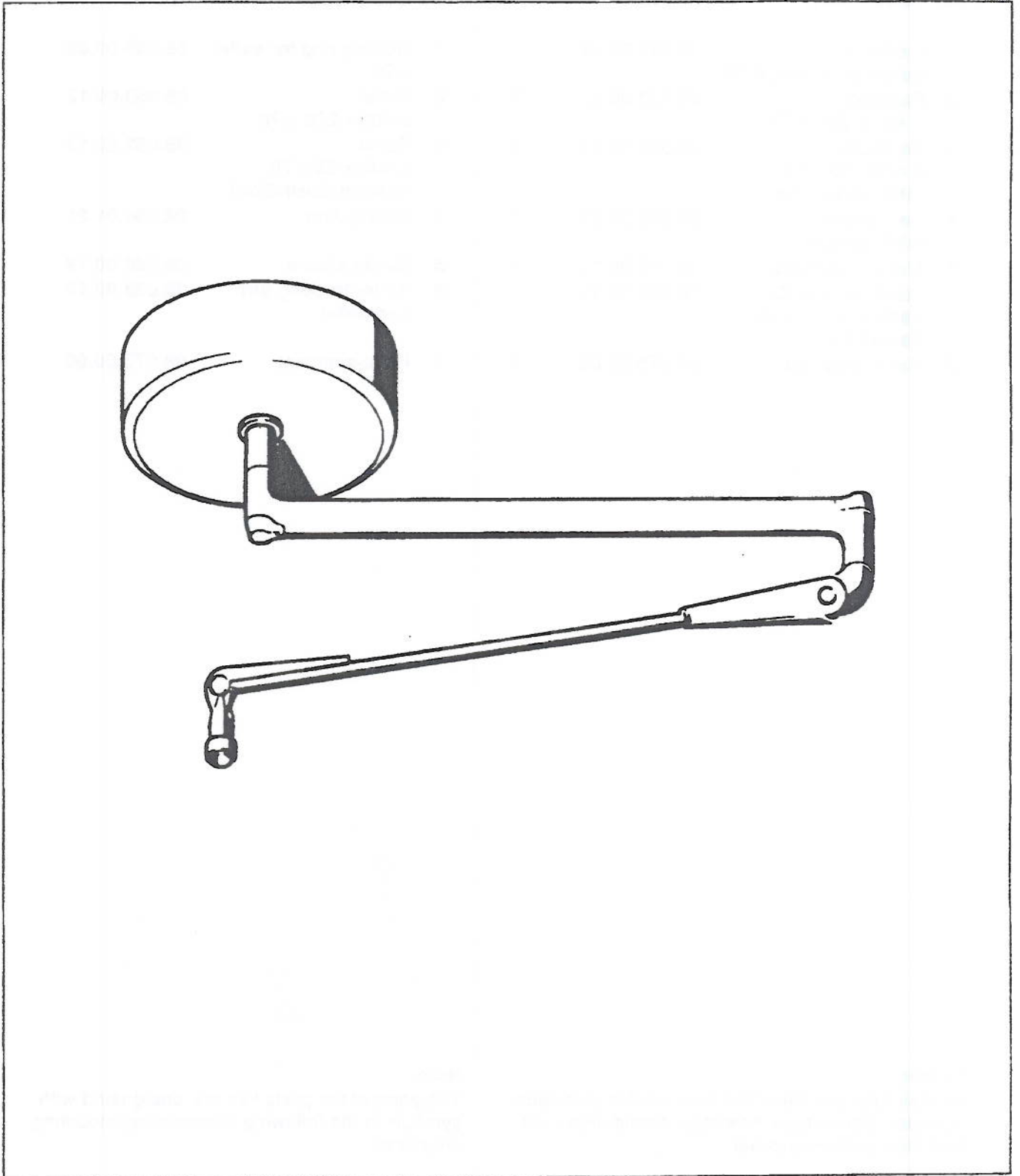
Note

The parts of the parts kits are designated with symbols in the following dismounting/mounting diagrams.

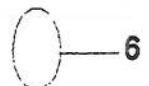Parts for the tracking arm and the soffet:

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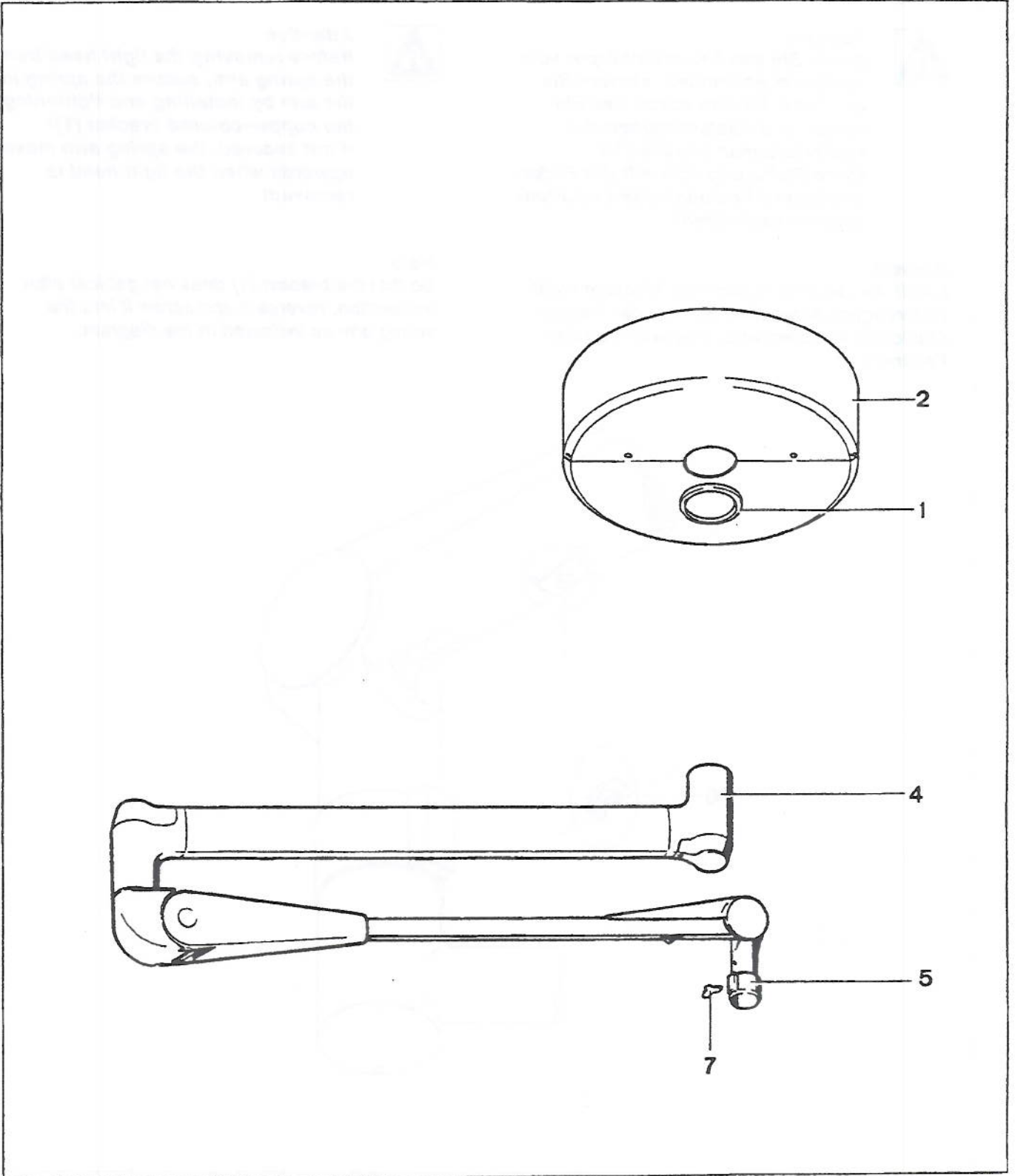
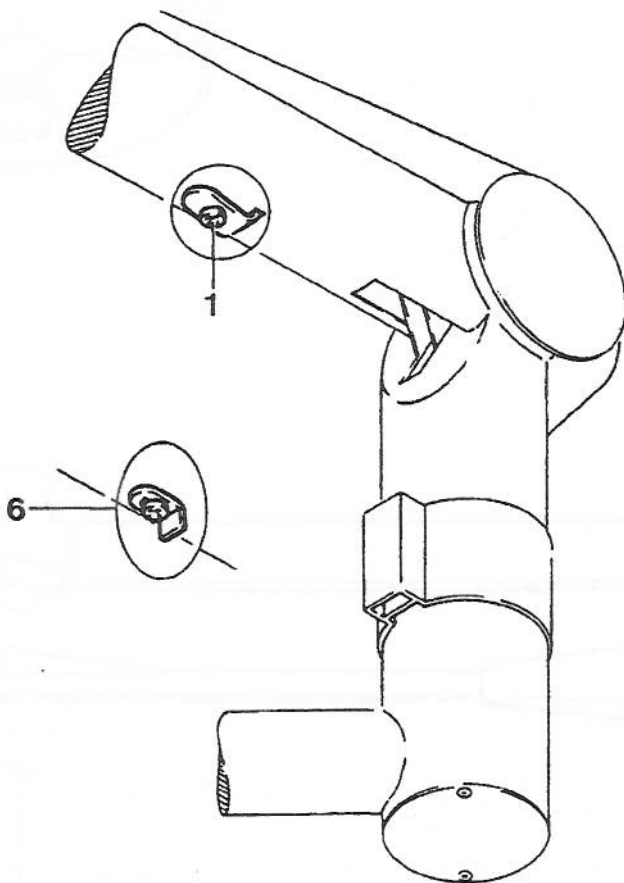
Attention
Before removing the light head from the spring arm, secure the spring in the arm by installing and tightening the copper-colored bracket (1)!
If not secured, the spring arm moves upwards when the light head is removed!

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Note

So that the bracket (1) does not get lost after installation, reverse it and screw it into the spring arm as indicated in the diagram.

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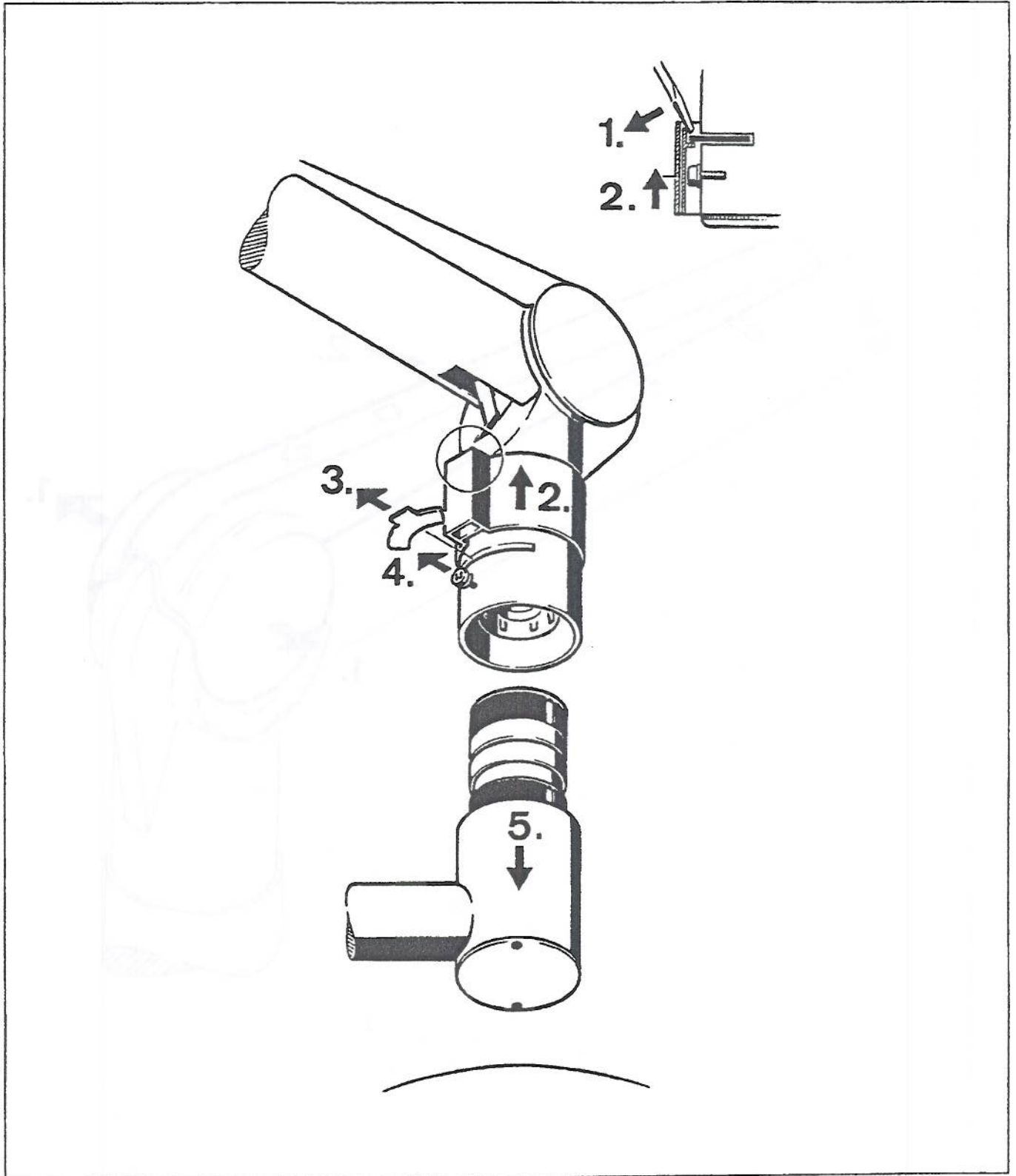
Securing the spring in the spring arm

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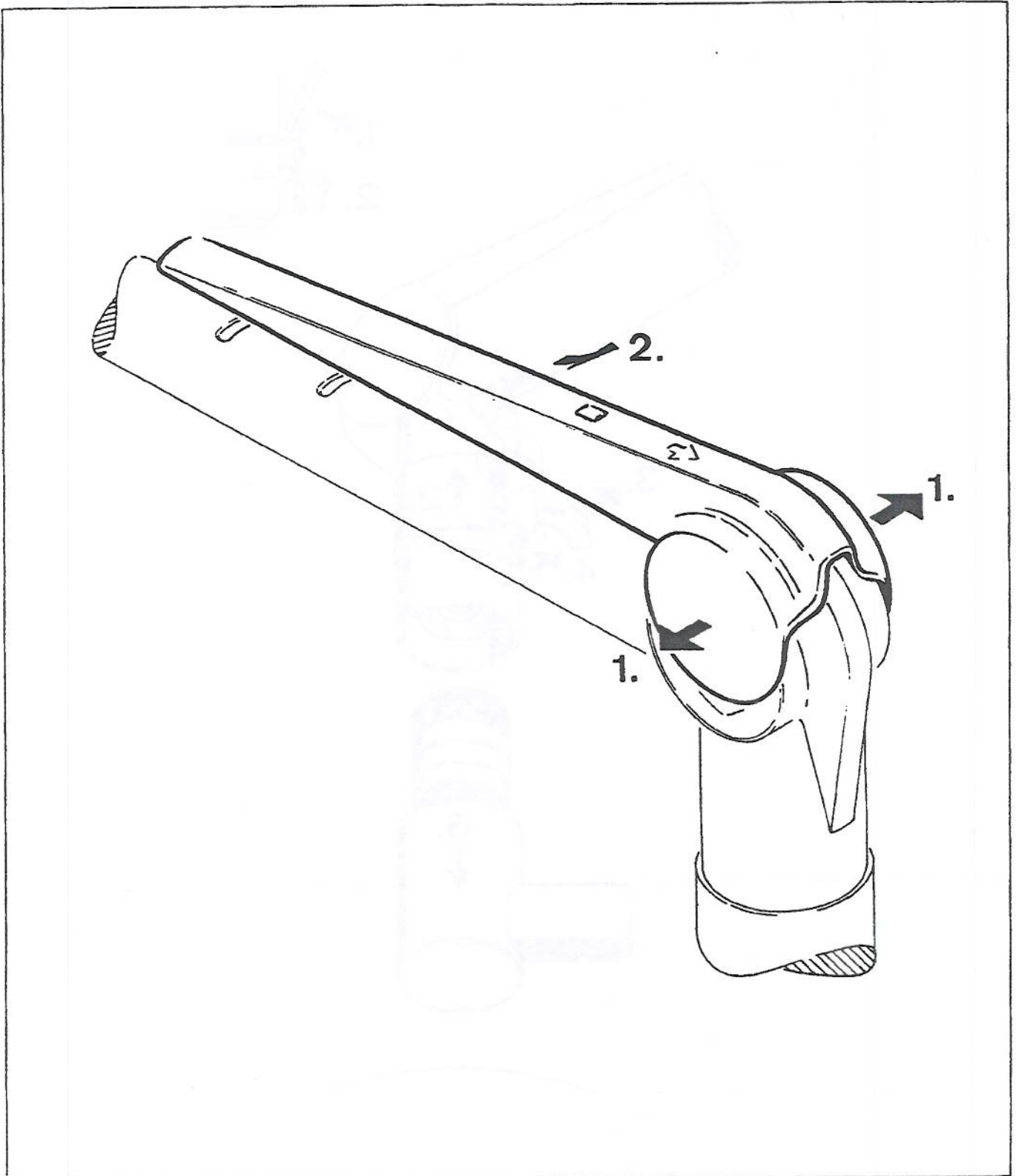
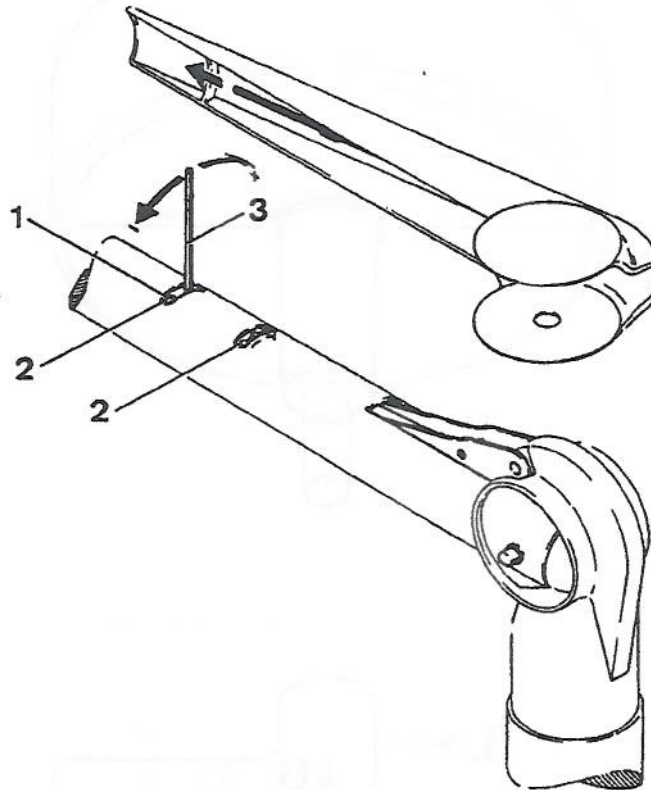
Note

Set the spring arm such that the light head remains securely in any vertically possible position!

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Adjusting

1. Spring arm moves too easily upwards:
 - Move the spring arm vertically until the round nut (1) is visible in one of the two slots (2). Now turn the nut with the wrench (3) to " - ", until the spring arm can be moved equally upwards and downwards.
2. Spring arm moves too easily downwards:
 - Move the spring arm vertically until the round nut (1) is visible in one of the two slots (2). Now turn the nut with the wrench (3) to " + ", until the spring arm can be moved equally upwards and downwards.

Put the wrench back into the cover after making this adjustment. Place the cover onto the spring arm and lock it into place.

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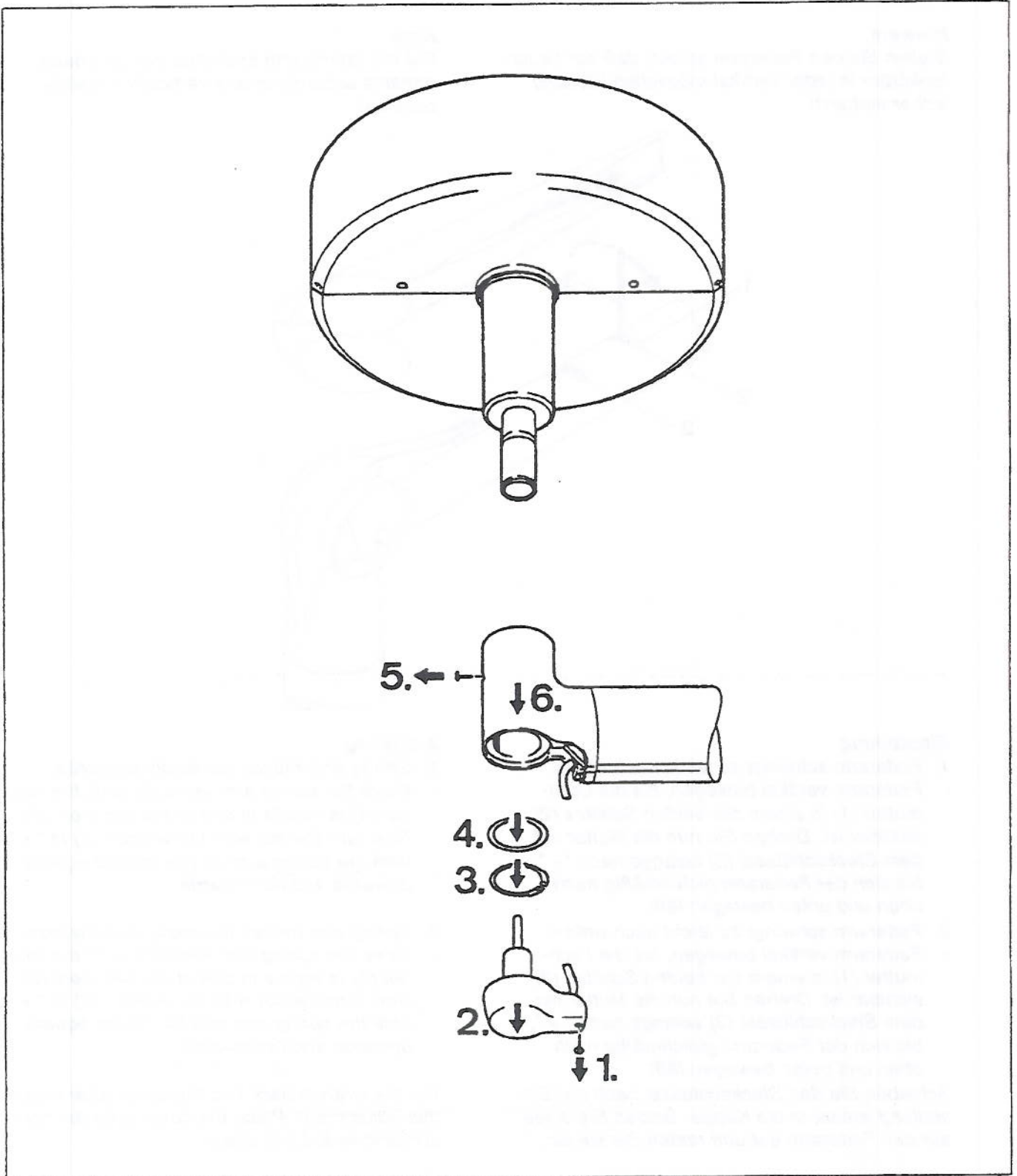
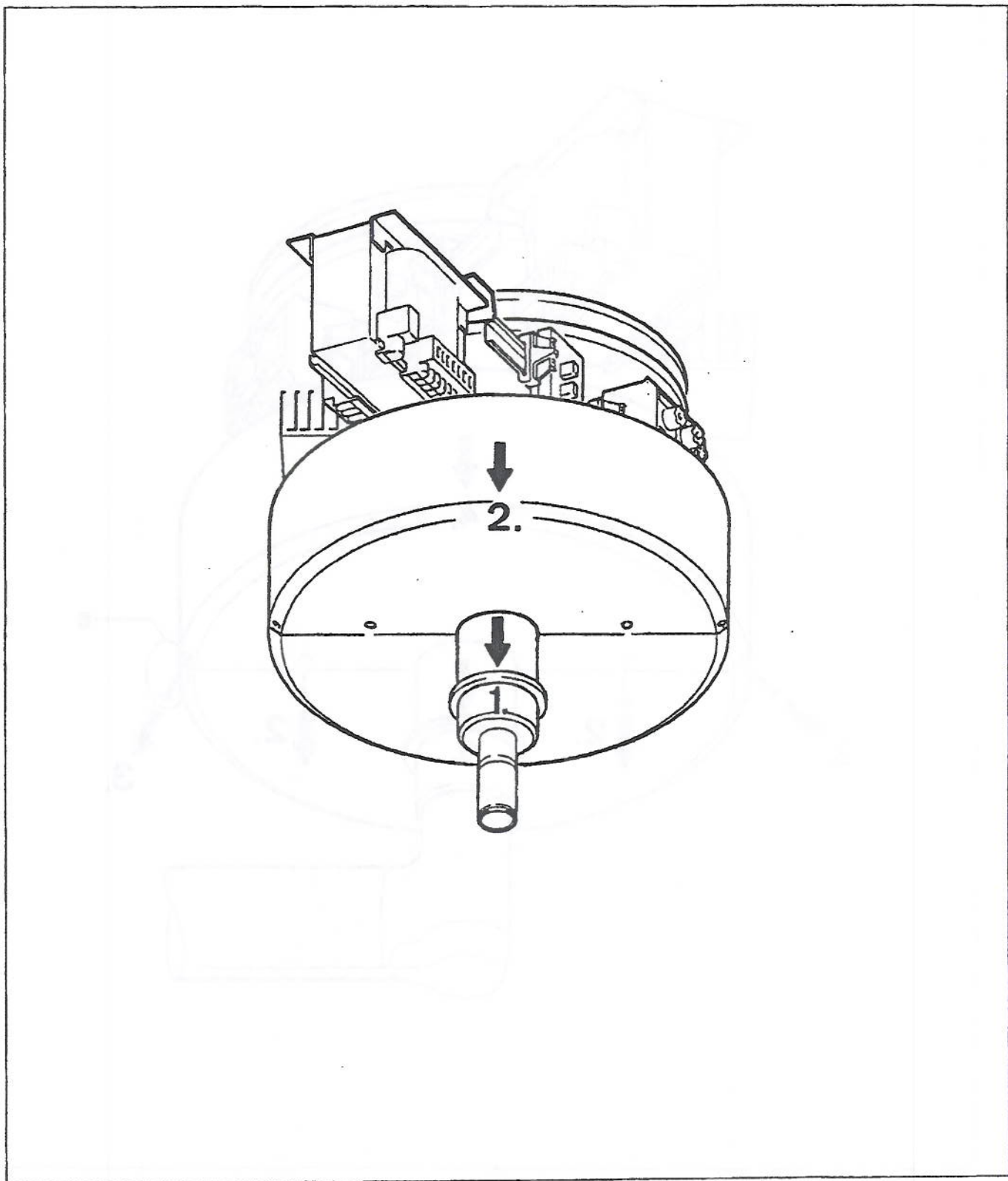
Removing the ceiling arm from the ceiling tube

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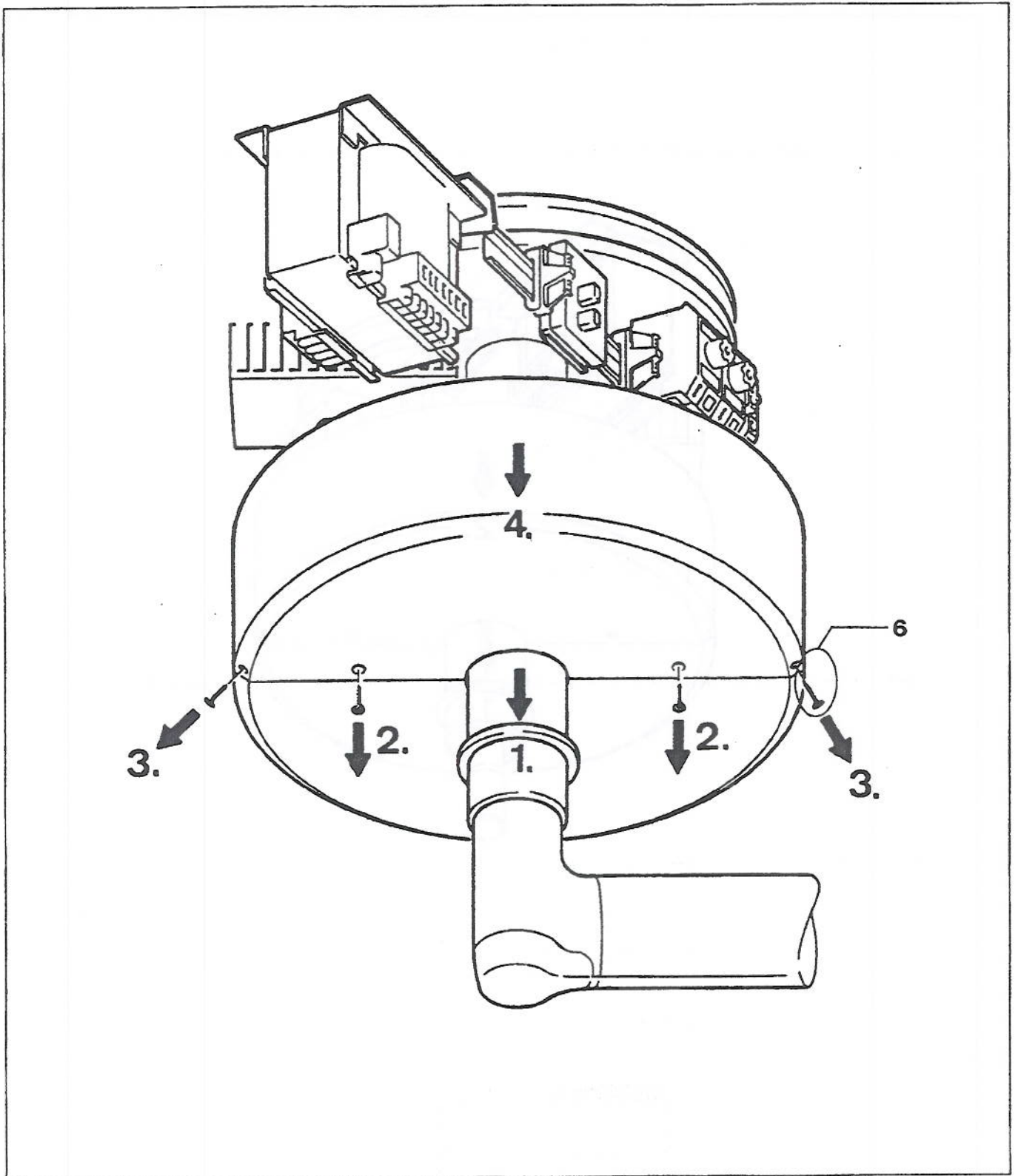
Removing the soffit when ceiling arm is dismantled

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Removing the soffit when ceiling arm is mounted

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
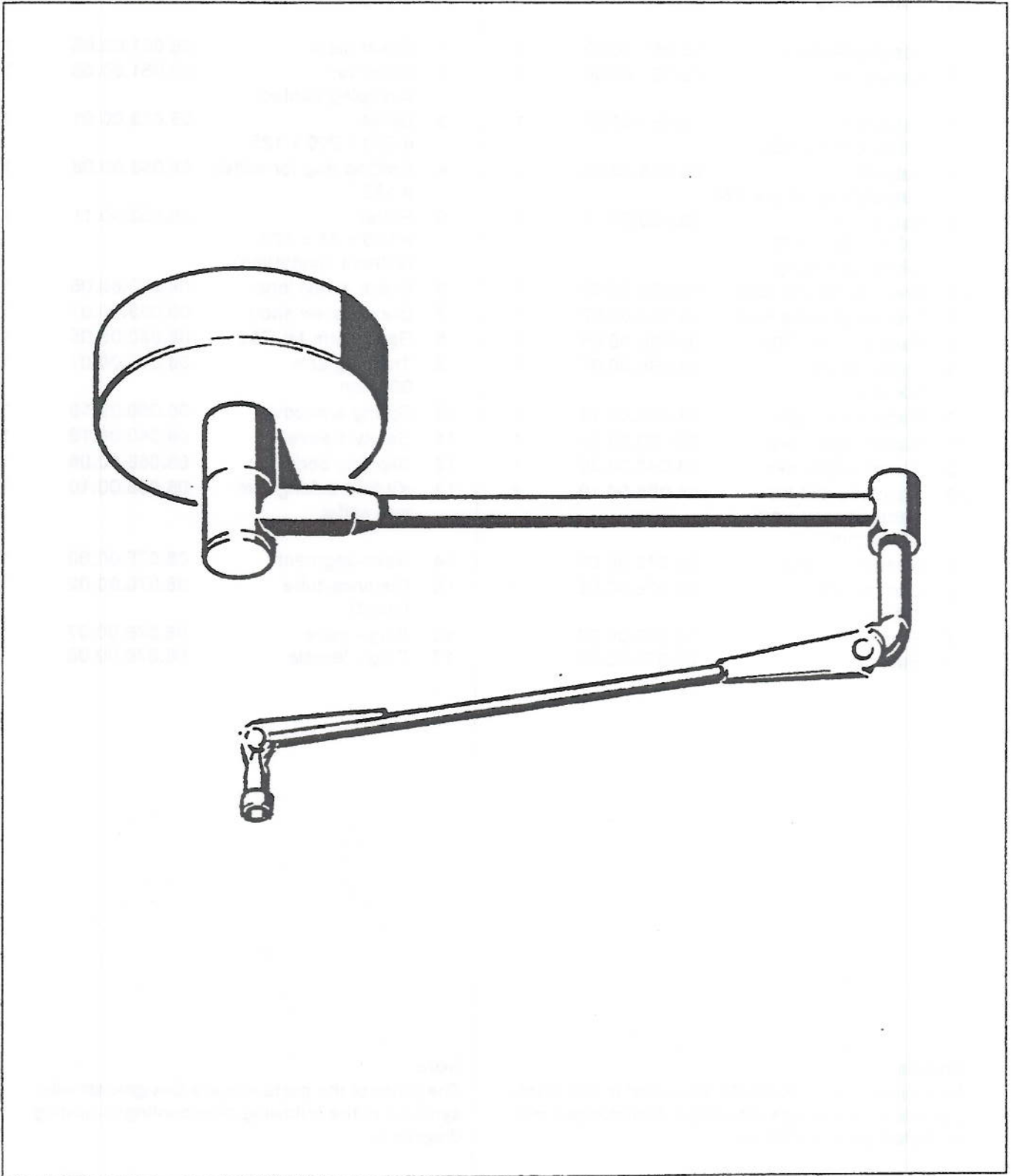
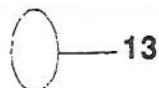
Note

The parts of the parts kits are designated with symbols in the following dismounting/mounting diagrams.

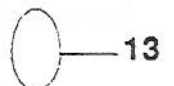Parts for the tracking arm
and the soffet:

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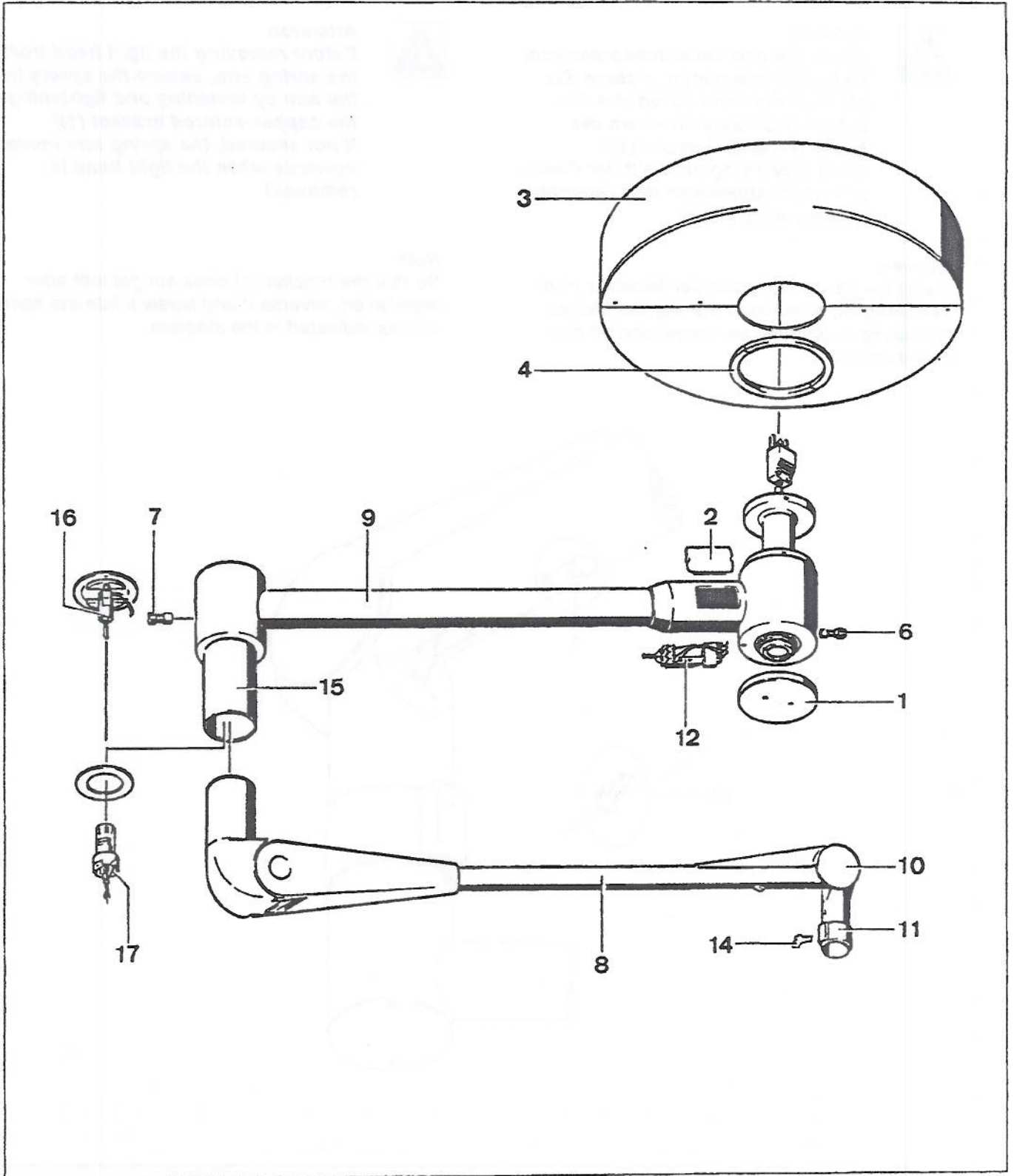
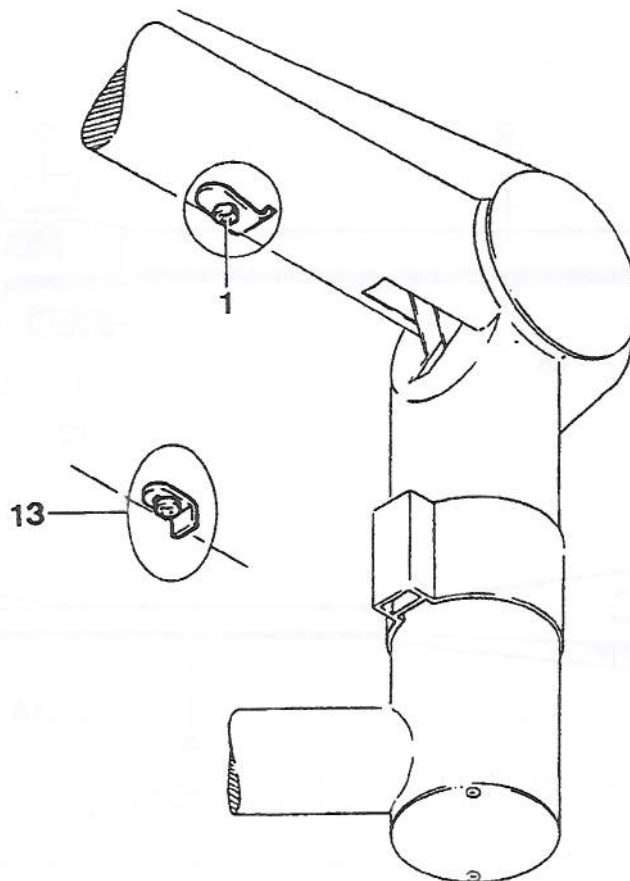
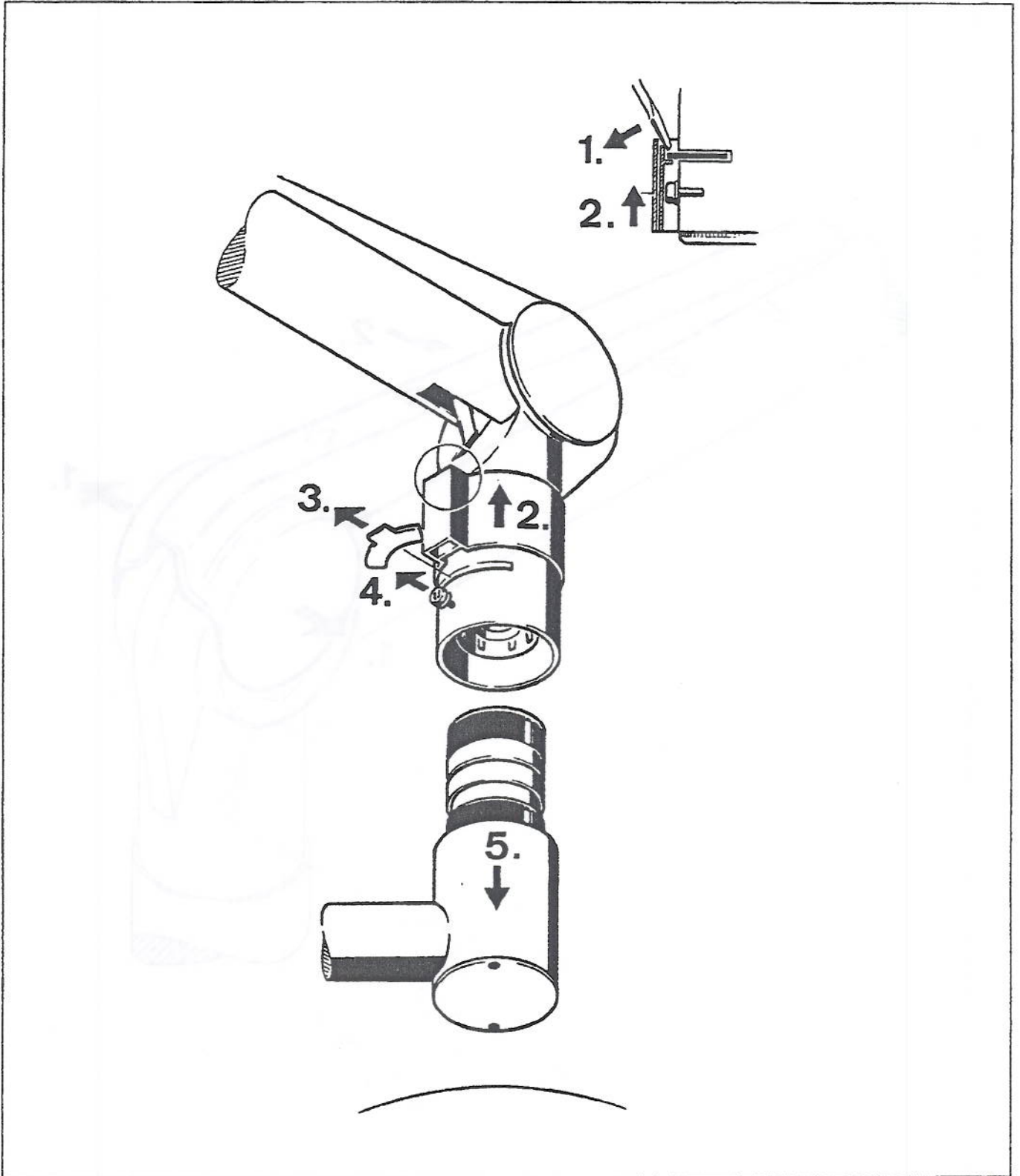
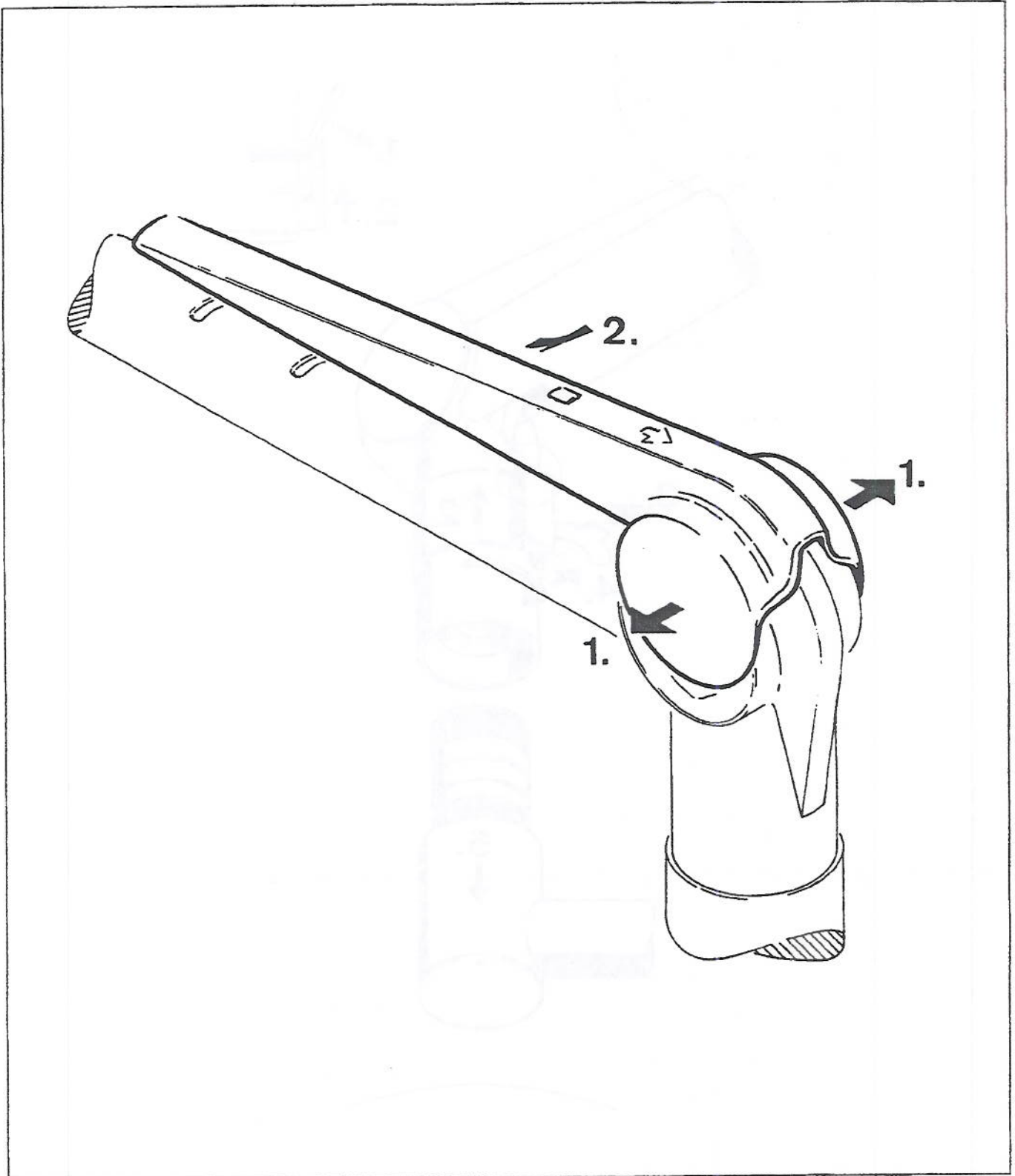
Removing the light head from the spring arm

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- Demontage/Montage

7.2 SA/TA - Version 2
(for one ML 700)

- Dismounting/Mounting

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Removing the cover from the spring arm

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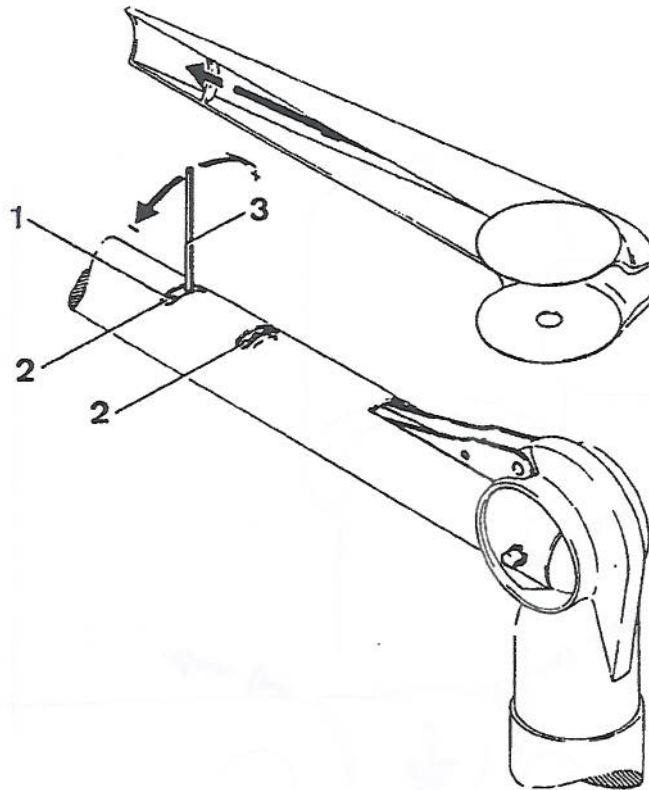
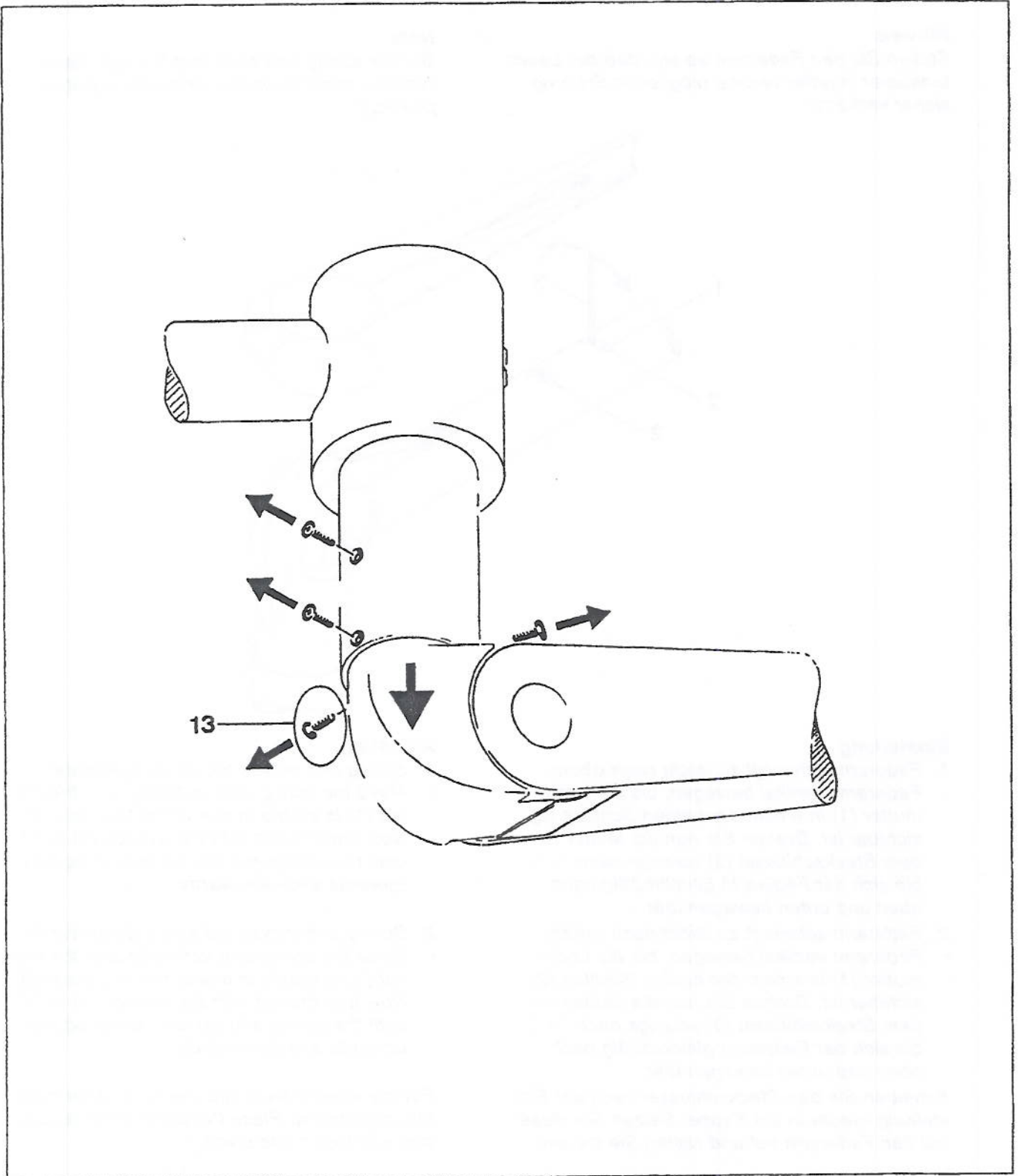
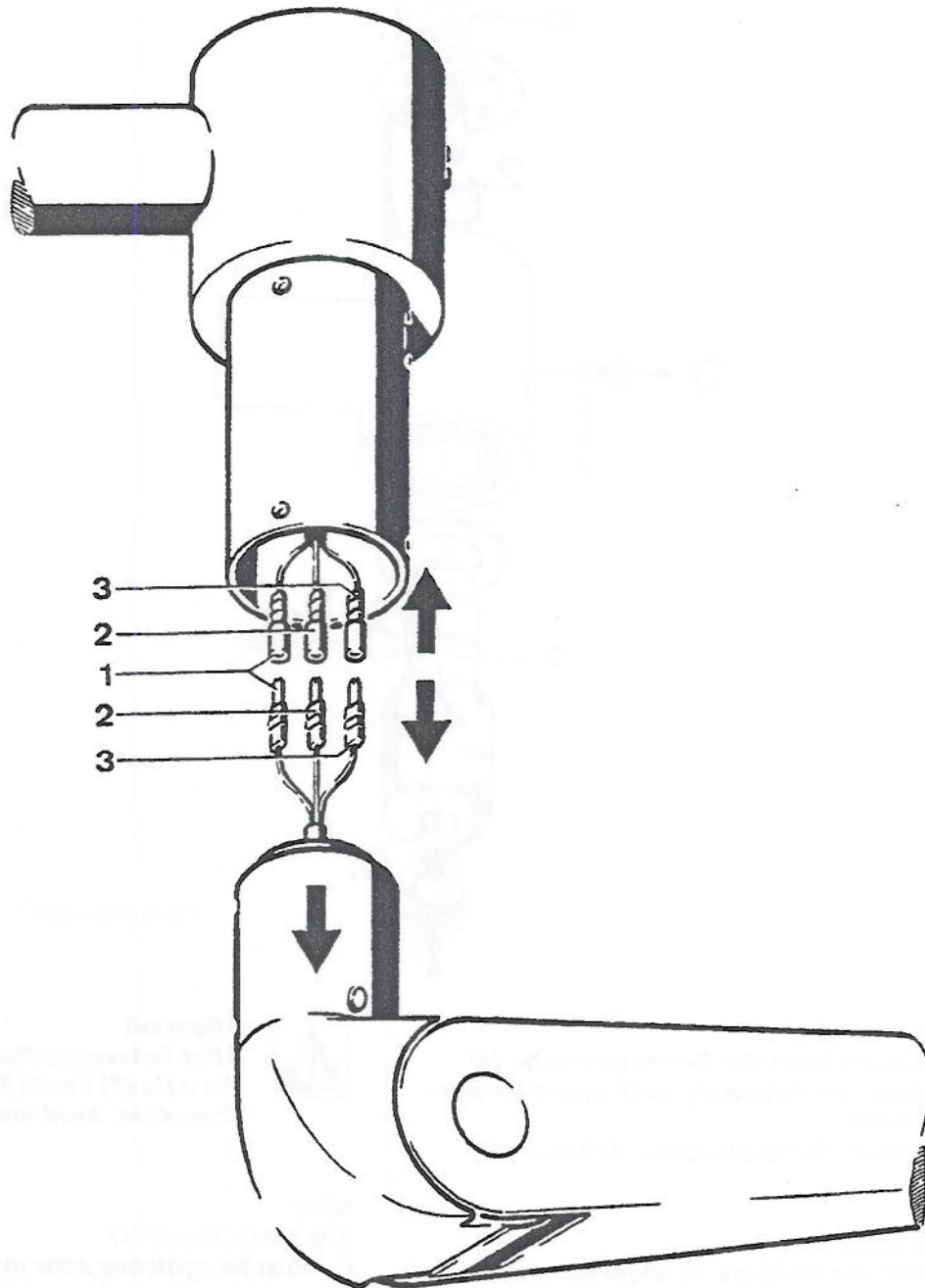
Removing the spring arm from the tracking arm

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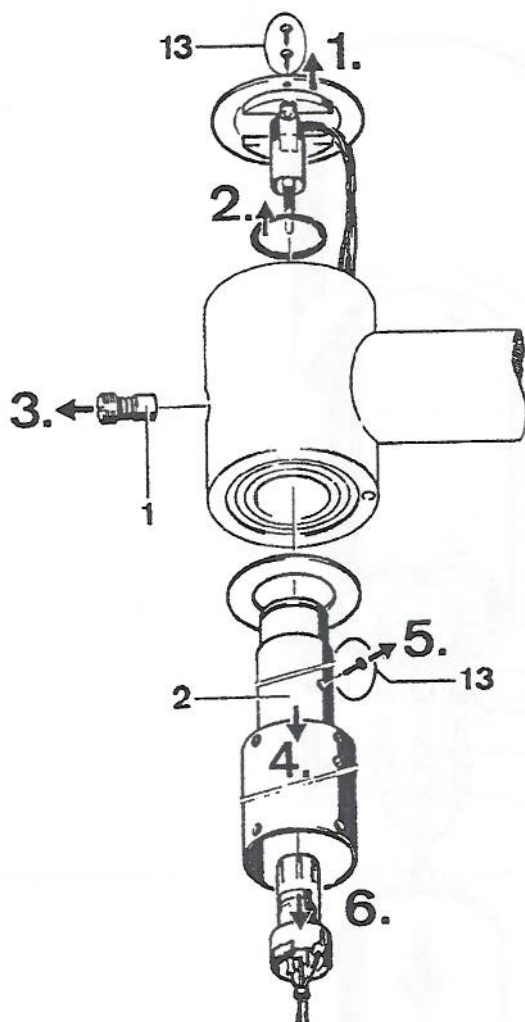
Releasing electrical connections between spring
arm and tracking arm

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Attention
After loosening the brake screw (1),
the axis (2) could fall out downwards!
Therefore, hold the axis in position!

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Note

The brake screw (1)

- must be tightened after mounting so that the spring arm (with light head) can easily be moved, but does not move on its own!
- should be checked for correct functioning approx. once a year!

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Removing the plugs and the brake screw

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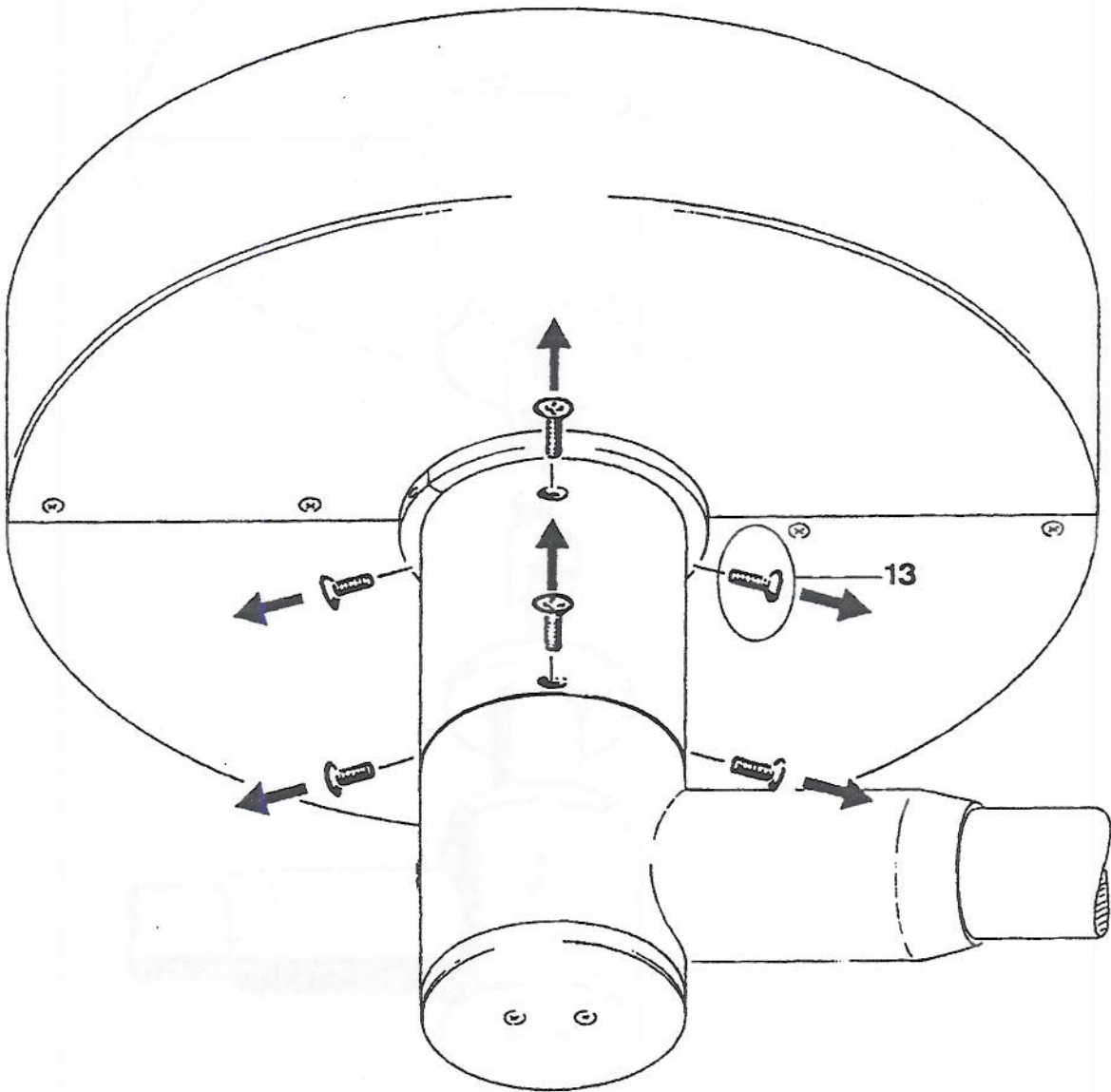
Attention
The tracking arm is heavy!
Therefore, secure the tracking arm with a lifting device before loosening the screws!

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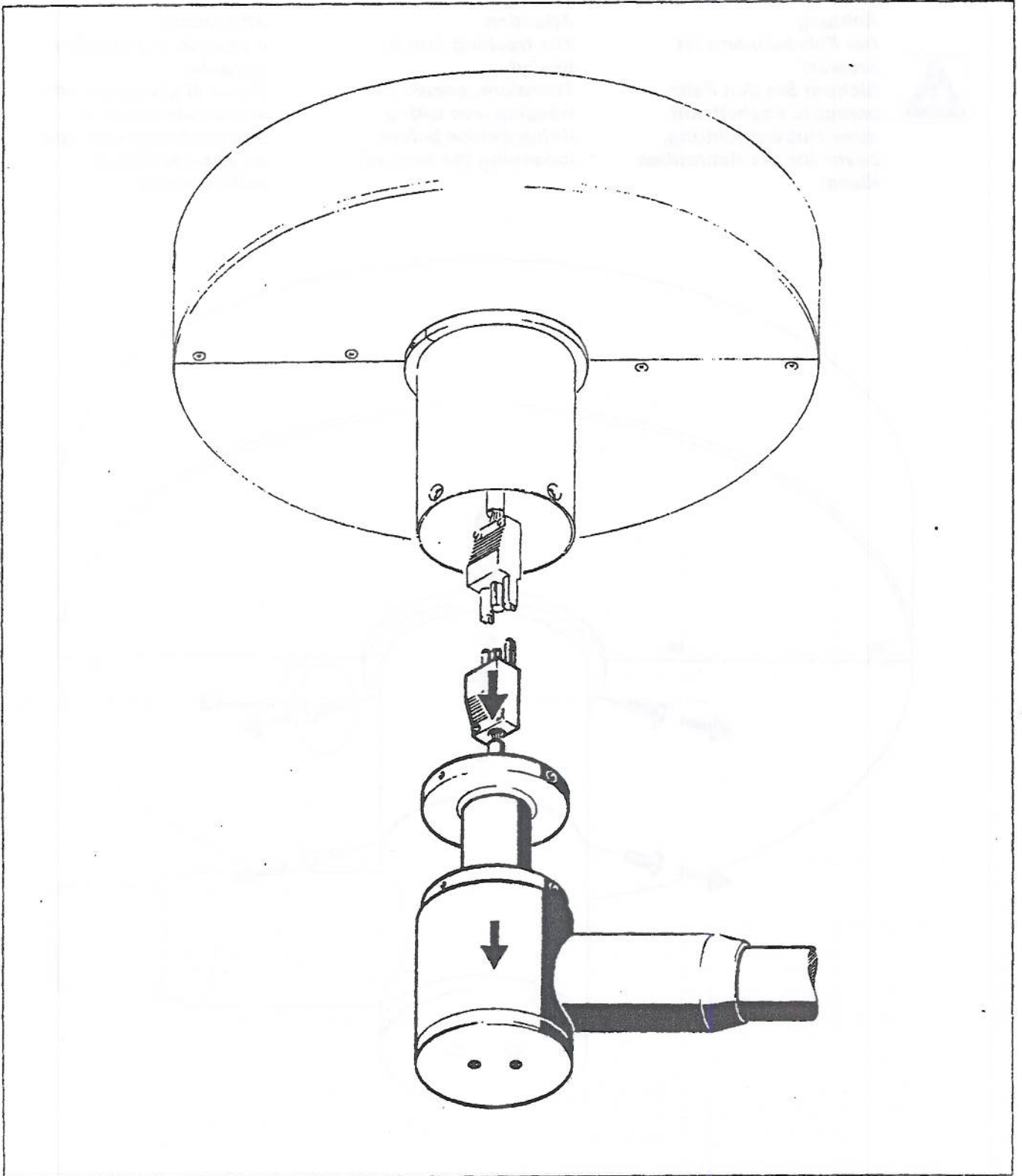
Removing the tracking arm from the ceiling tube

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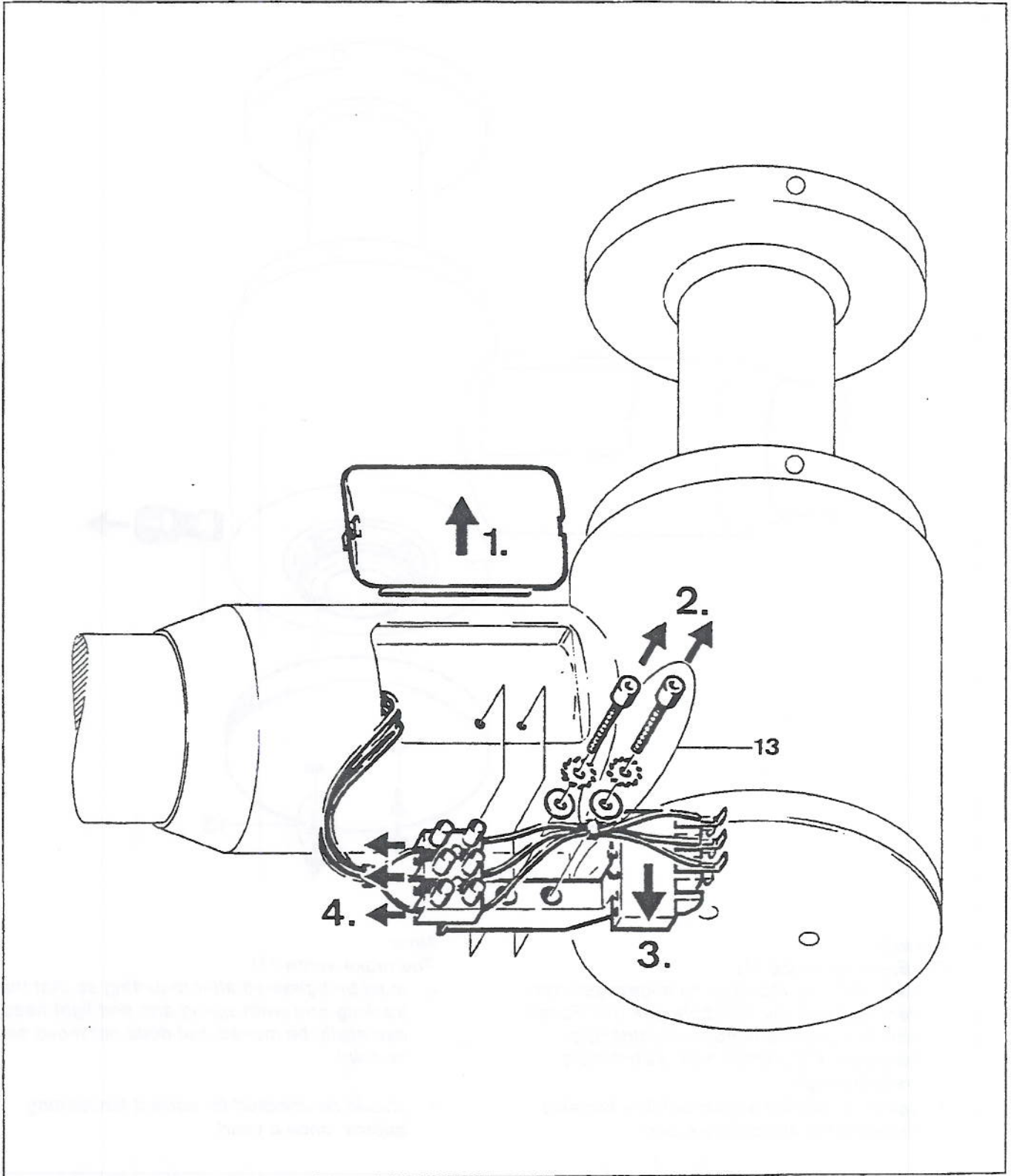
Releasing electrical connections between tracking
arm and ceiling tube

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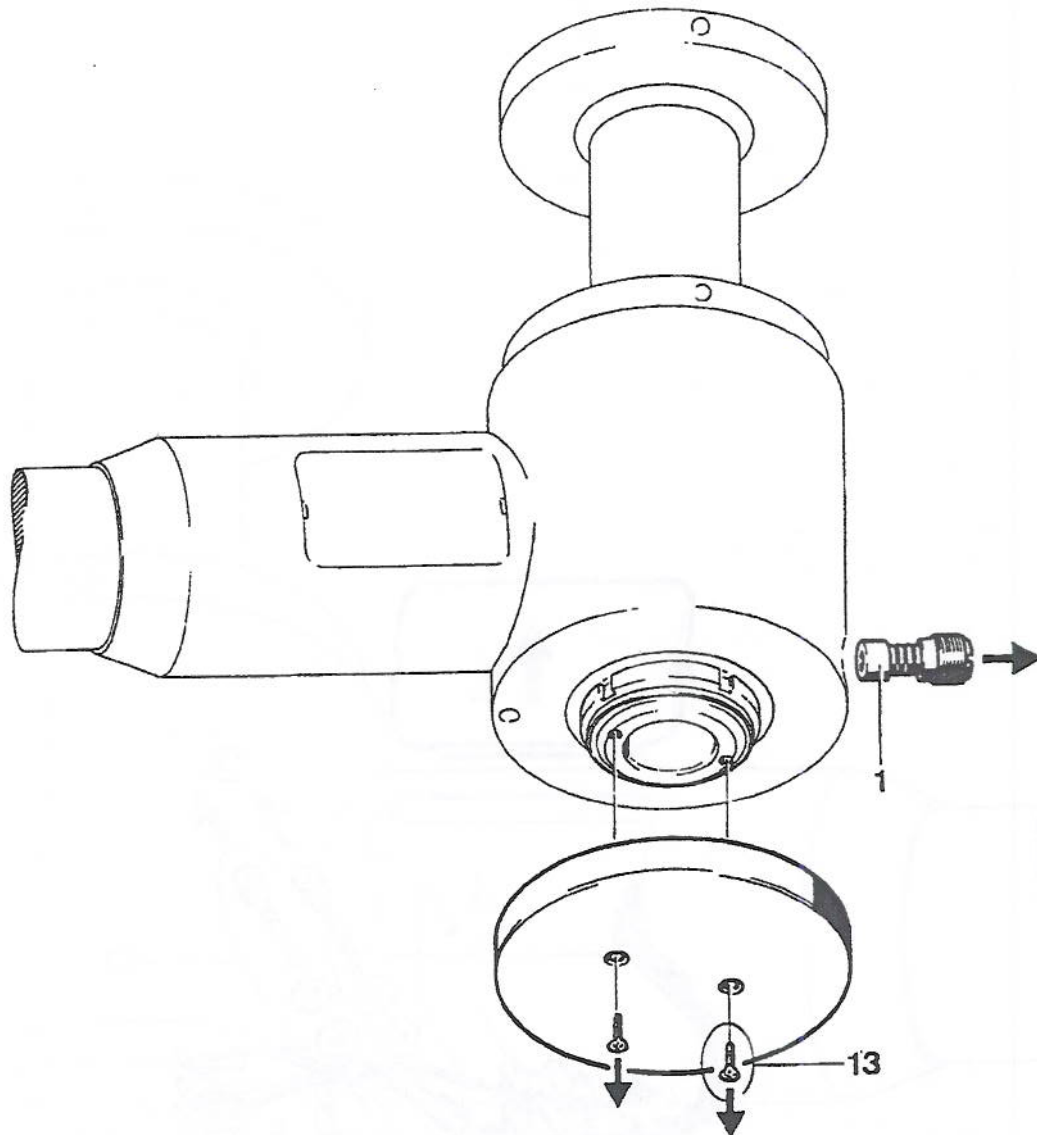
Note

The brake screw (1)

- must be tightened after mounting so that the tracking arm (with spring arm and light head) can easily be moved, but does not move on its own!
- should be checked for correct functioning approx. once a year!

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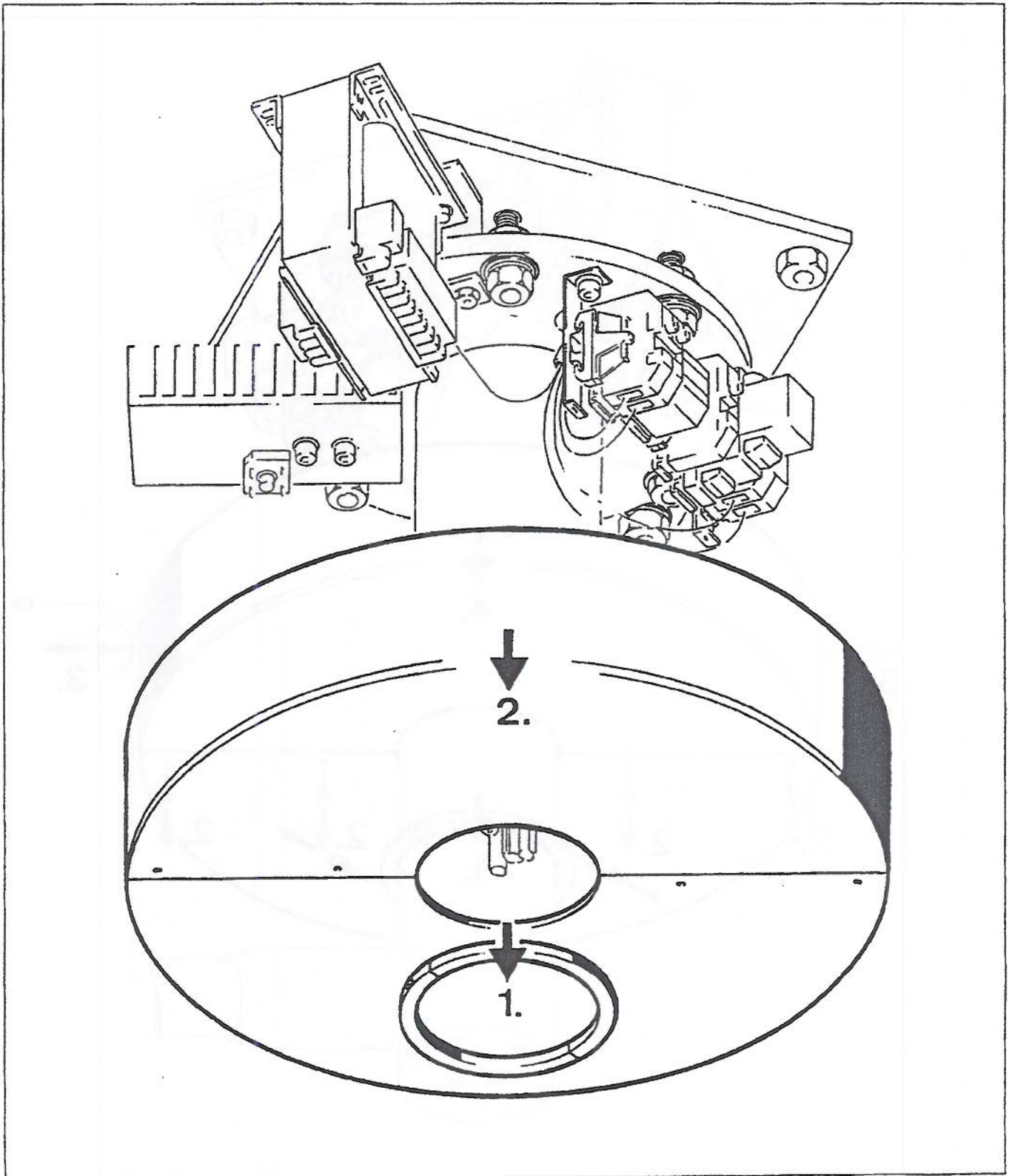
Removing the cover plate and the brake screw

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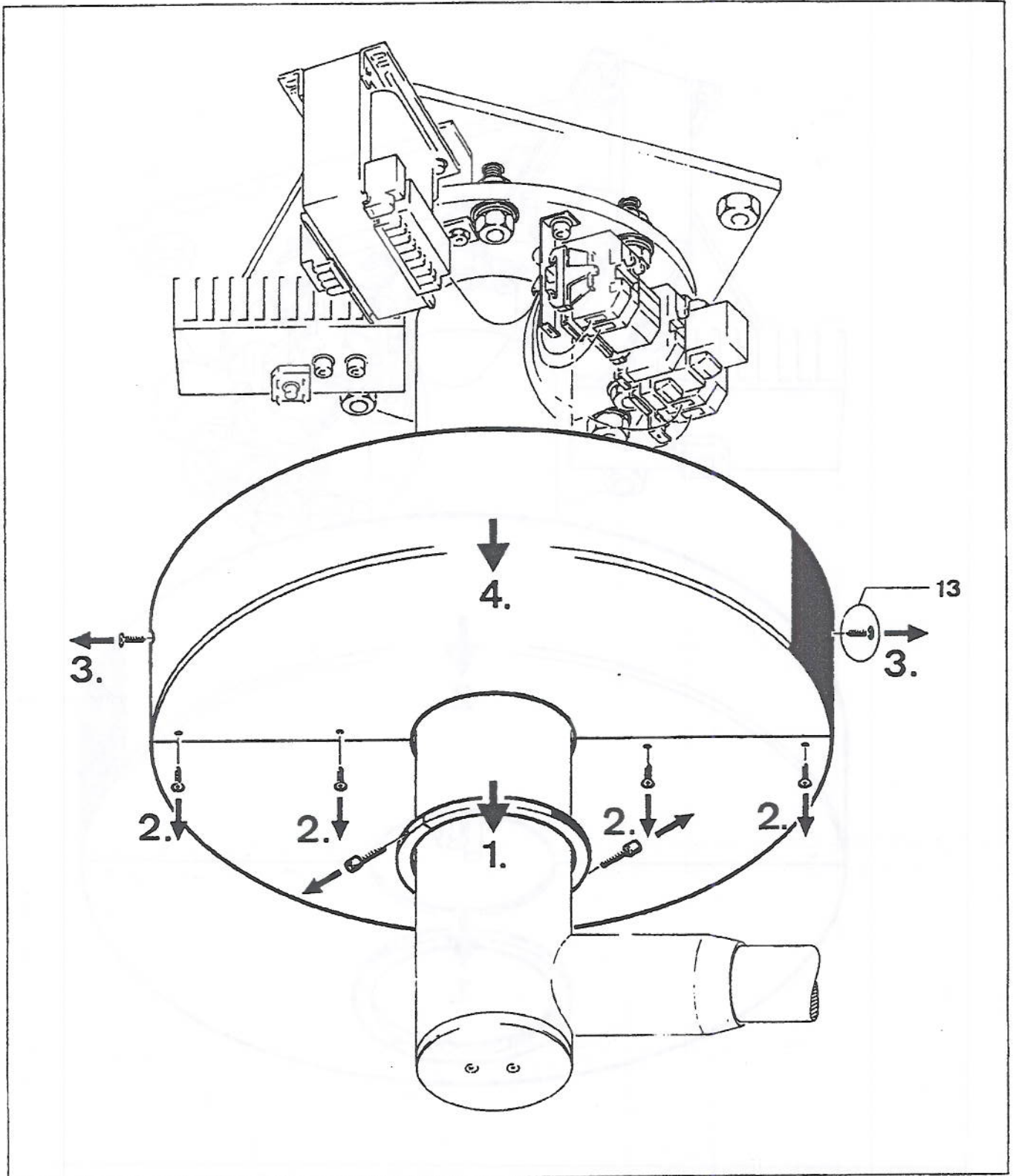
Removing the soffit when tracking arm is dismounted

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Removing the soffet when tracking arm is mounted

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**8. Ceiling Tubes/Mains Supply Units
(CT/MSU) for ML 500, ML 700**

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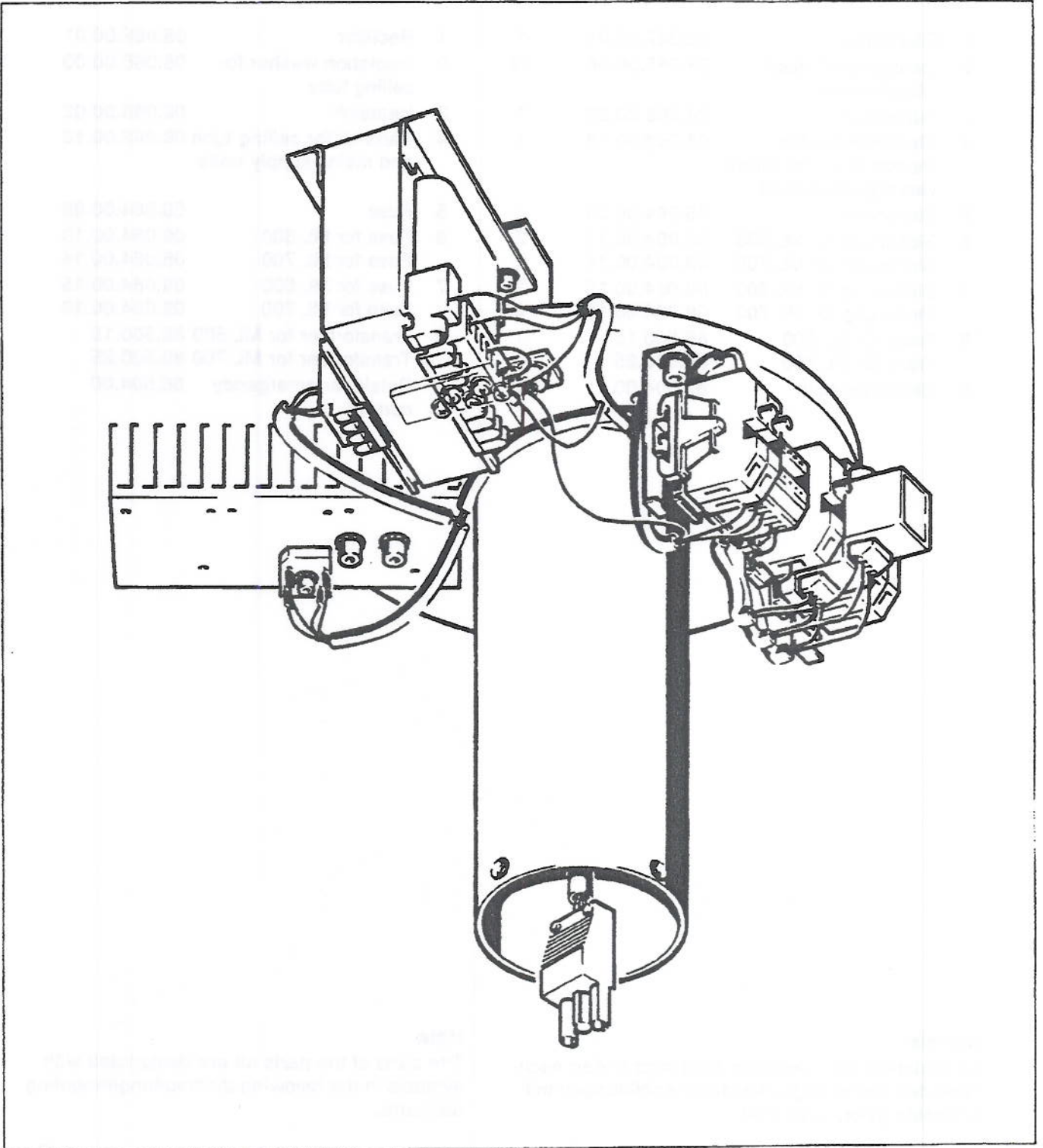
**8. Ceiling Tubes/
Mains Supply Units (CT/MSU)**

8.1 CT/MSU - Version 1
(for one ML 500/ML 700
ceiling mounting
with emergency current switch over)

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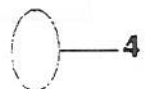
Note

The parts of the parts kit are designated with symbols in the following dismounting/mounting diagrams.

Parts for ceiling tube and mains supply unit:

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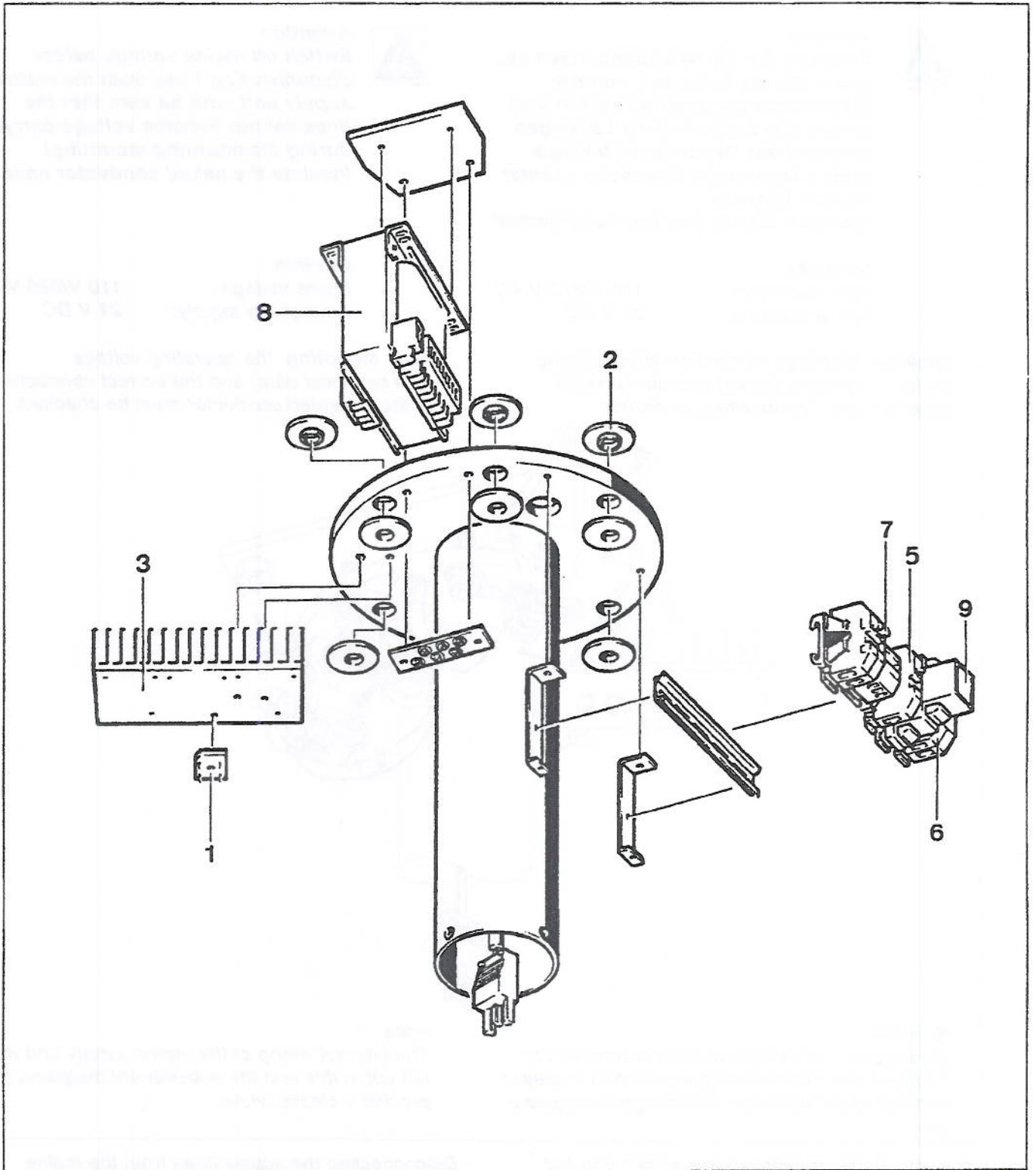
8.1 CT/MSU - Version 1
(for one ML 500/ML 700
ceiling mounting
with emergency current switch over)

- Dismounting/Mounting
(Ceiling tube with mains supply unit)

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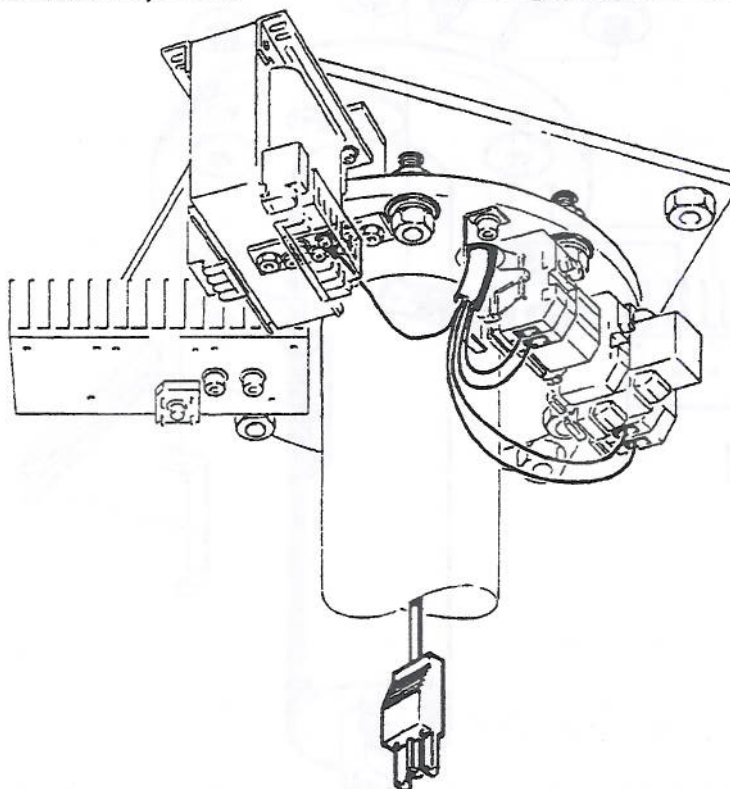
Attention
Switch off mains voltage before disconnecting lines from the mains supply unit, and be sure that the lines cannot become voltage-carrying during dismounting/mounting!
Insulate the naked conductor ends.

Caution
Mains voltage: 110 V/220 V AC
Emergency supply: 24 V DC

After mounting, the operating voltage (see technical data) and the correct connection of the grounded conductor must be checked.

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Note
The internal wiring of the mains supply unit was left out in this and the subsequent diagrams to provide a clearer view.

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Disconnecting the supply lines from the mains supply unit

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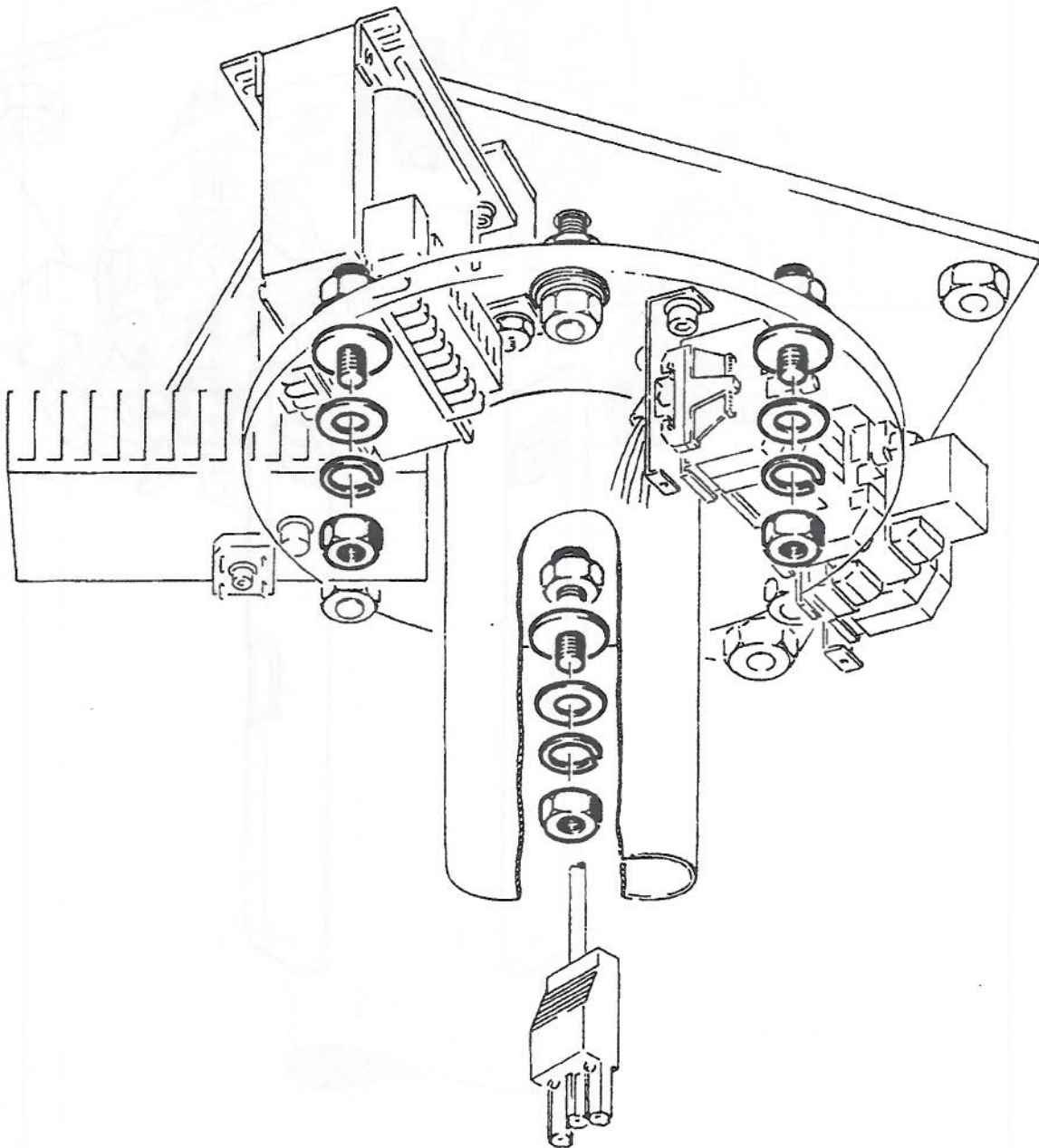
Attention
The ceiling tube with the mounted
mains supply unit is heavy!
Therefore, secure the ceiling tube
with a lifting device before loosening
the nuts!

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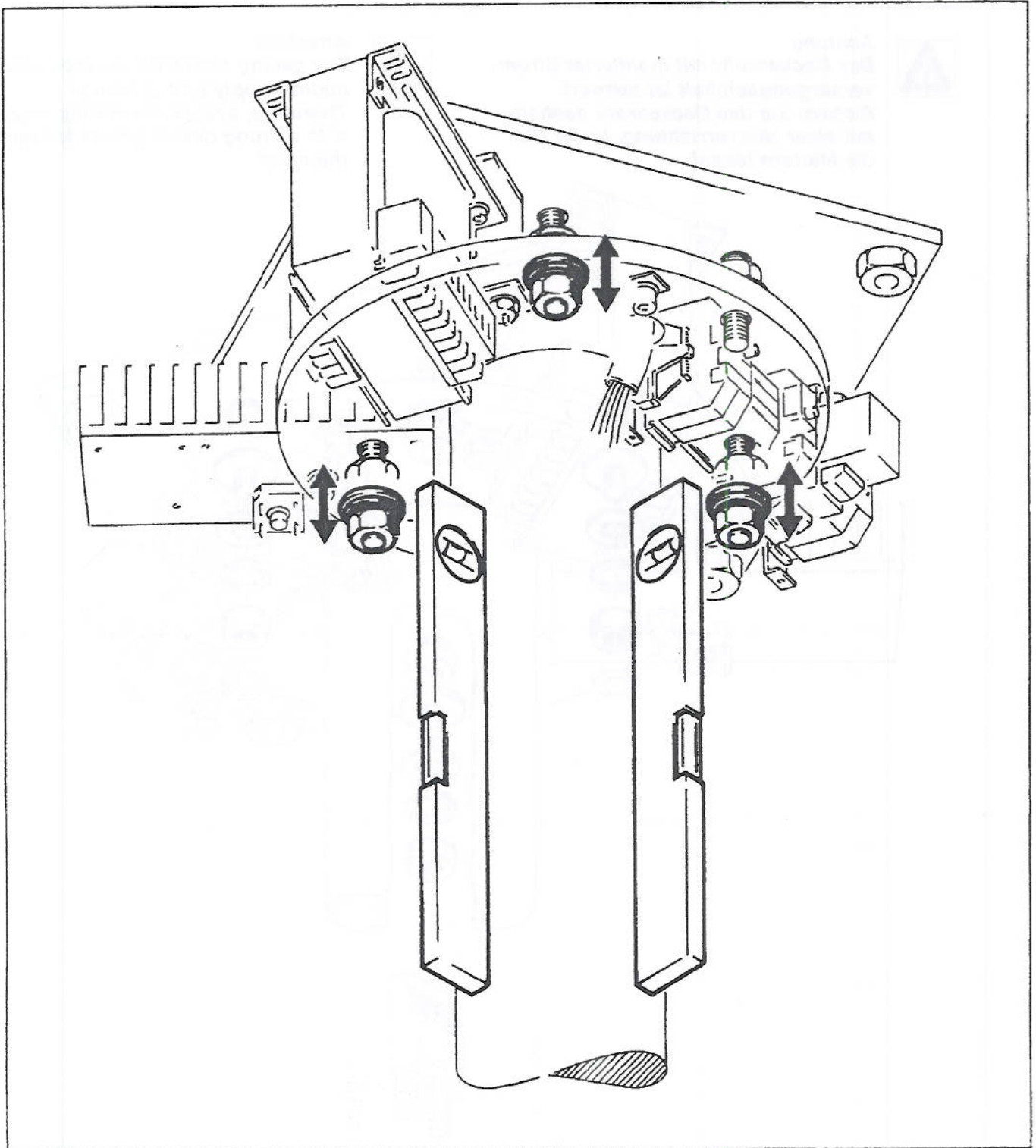
Vertical adjustment of the ceiling tube
- only when installing!

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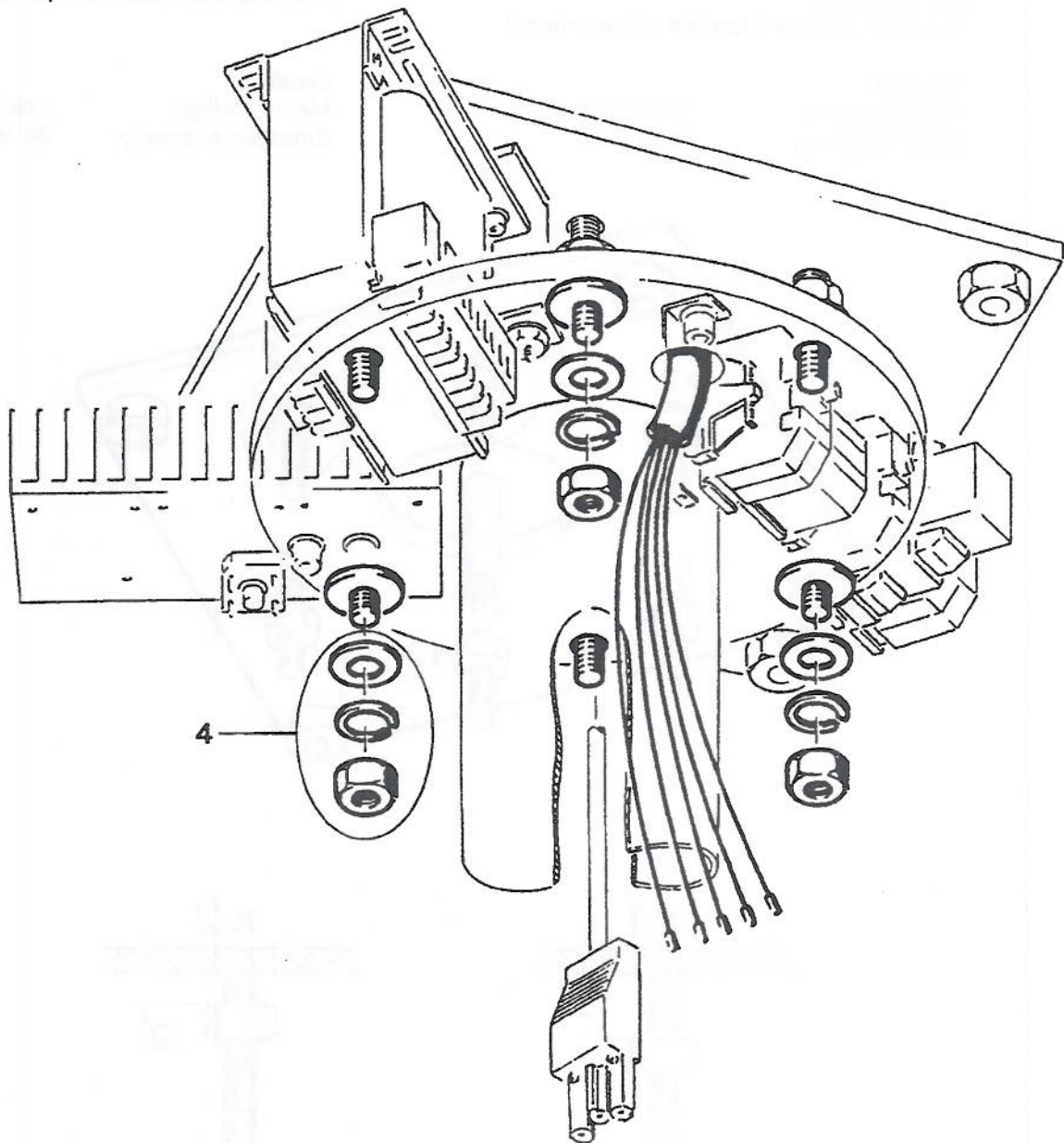
Dismounting the ceiling tube from the mounting ring
or from the intermediate ceiling construction

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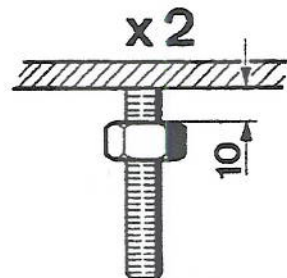
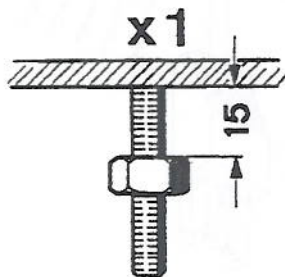
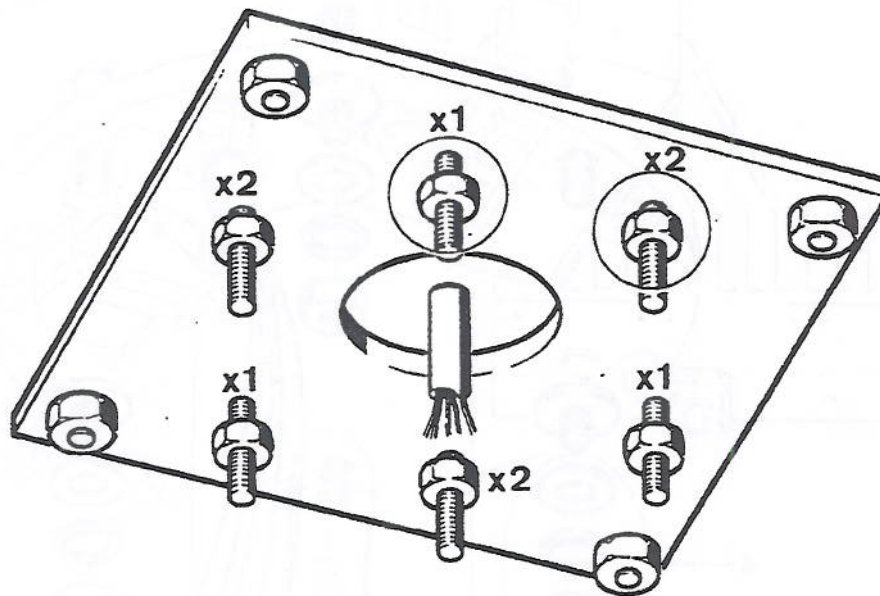
Attention
Switch off the mains voltage before
carrying out dismantling/mounting
work, and be sure that the lines
cannot become voltage-carrying
during this work!
Insulate the naked conductor ends!

Caution
Mains voltage: 110 V/220 V AC
Emergency supply: 24 V DC

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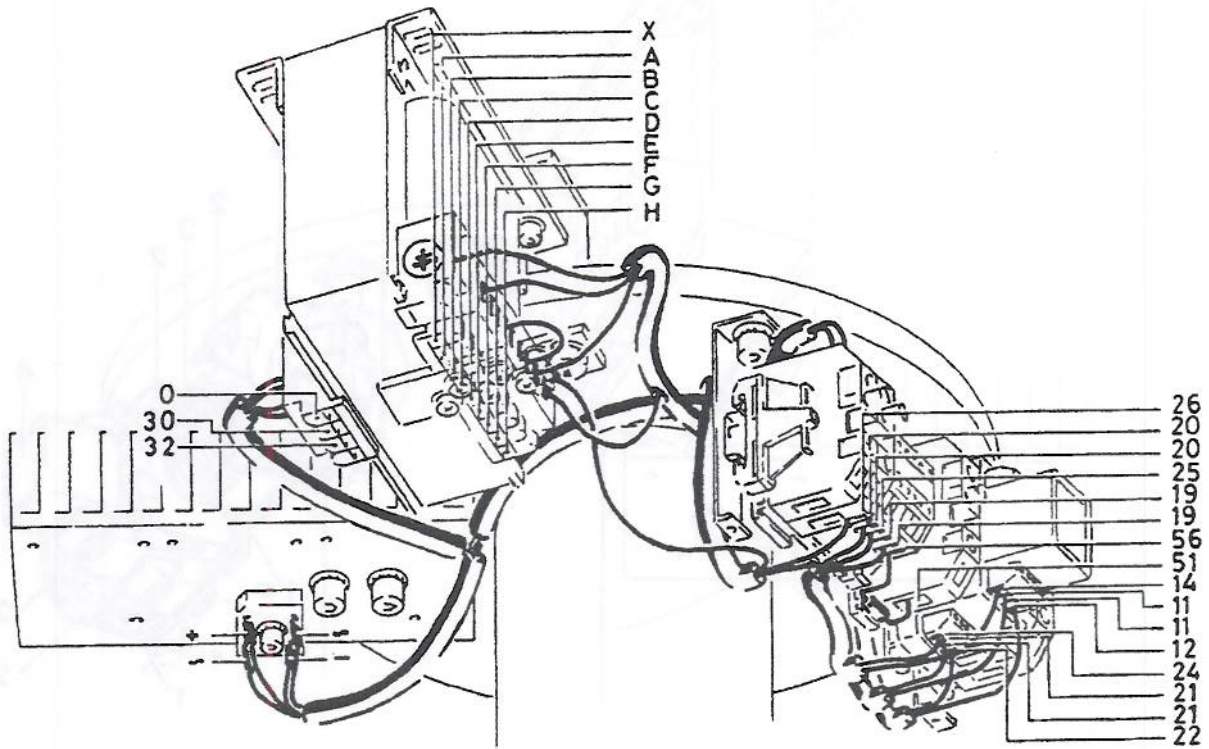
Positioning the nuts of the mounting ring or the
intermediate ceiling construction for mounting the
ceiling tube - only when installing

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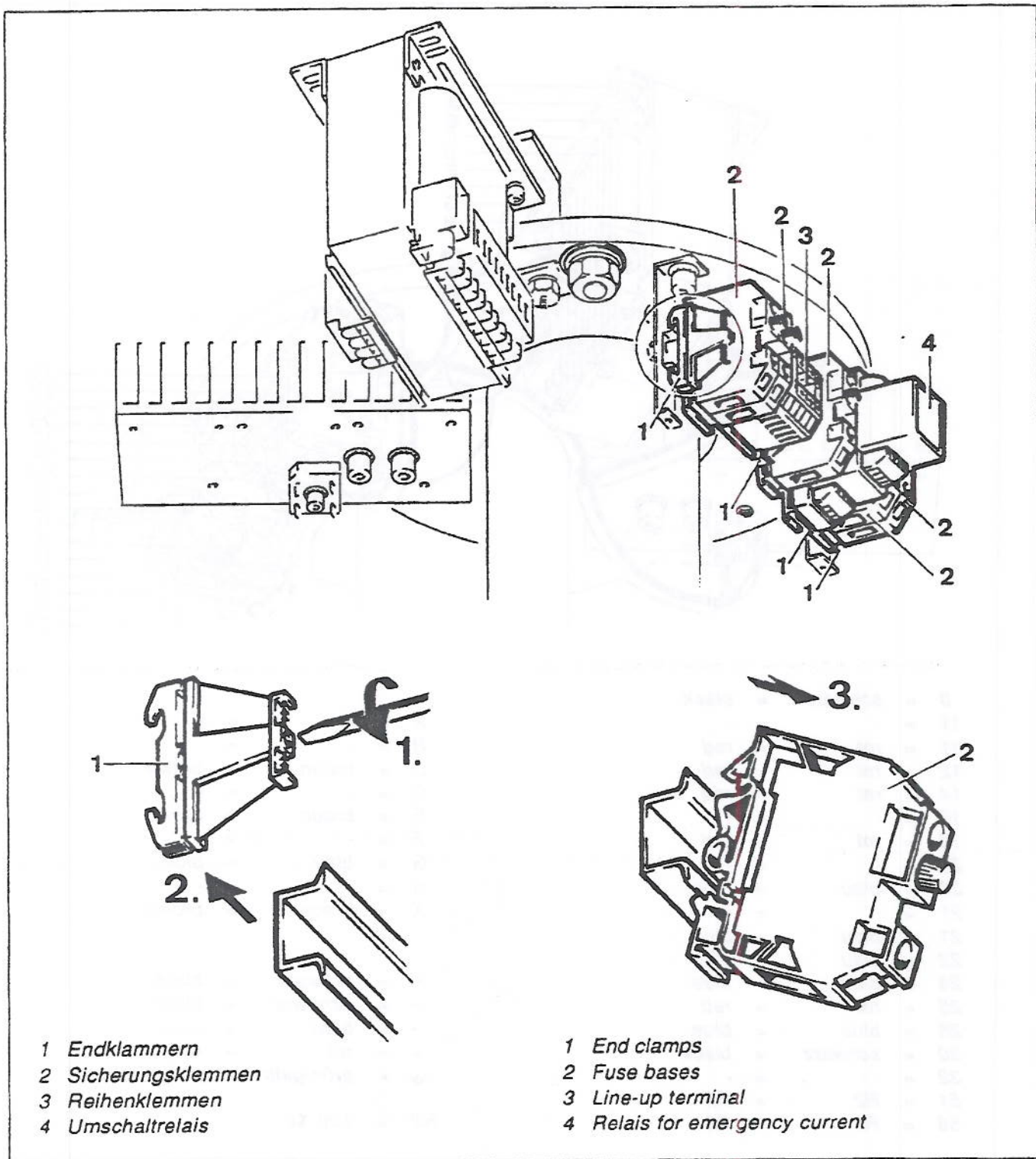
8.1 CT/MSU - Version 1
(for one ML 500/ML 700
ceiling mounting
with emergency current switch over)

- **Dismounting/Mounting**
(parts of the mains supply unit)

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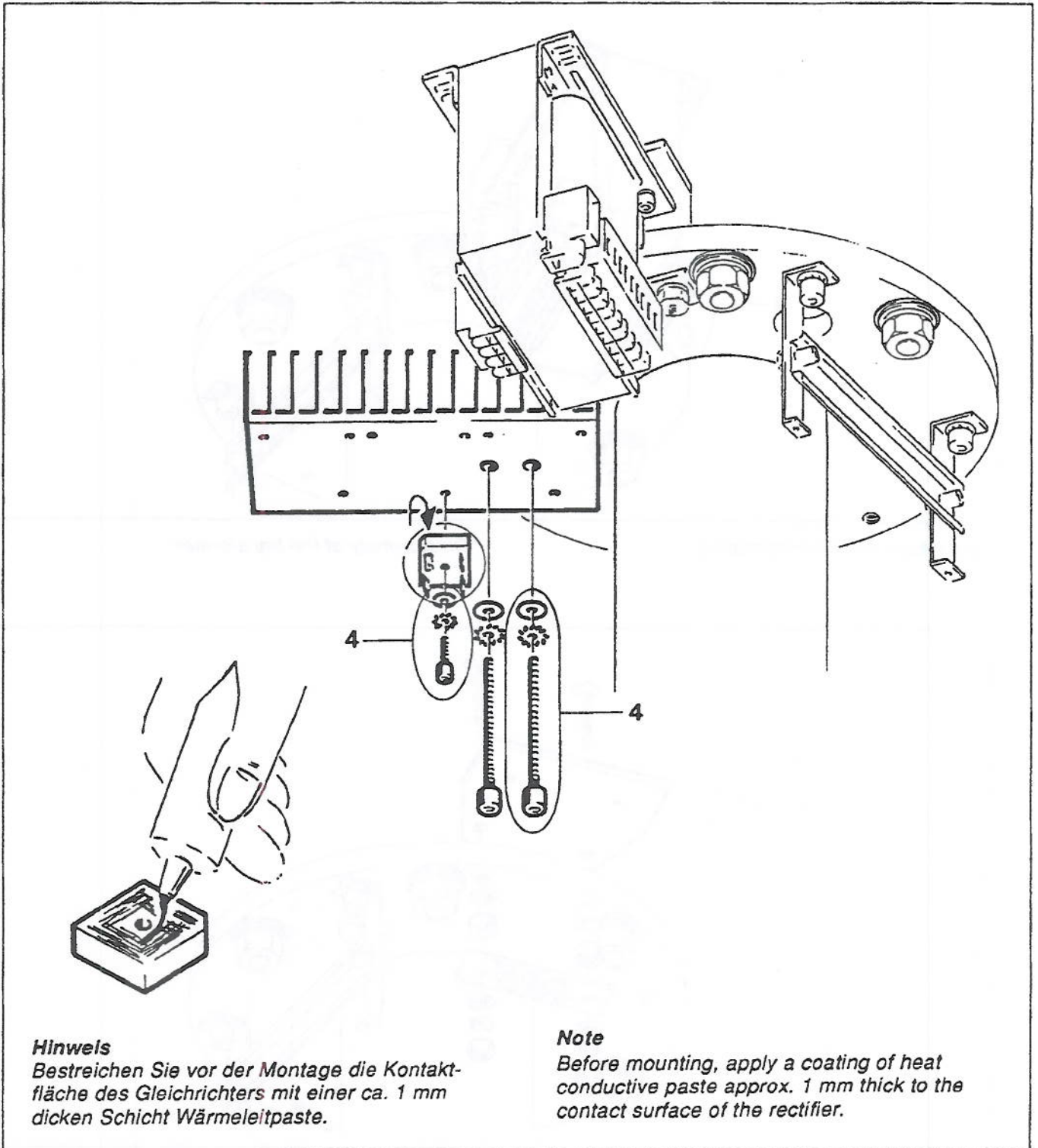
Disassembly of end clamps, fuse bases, line-up terminals and relays for emergency current

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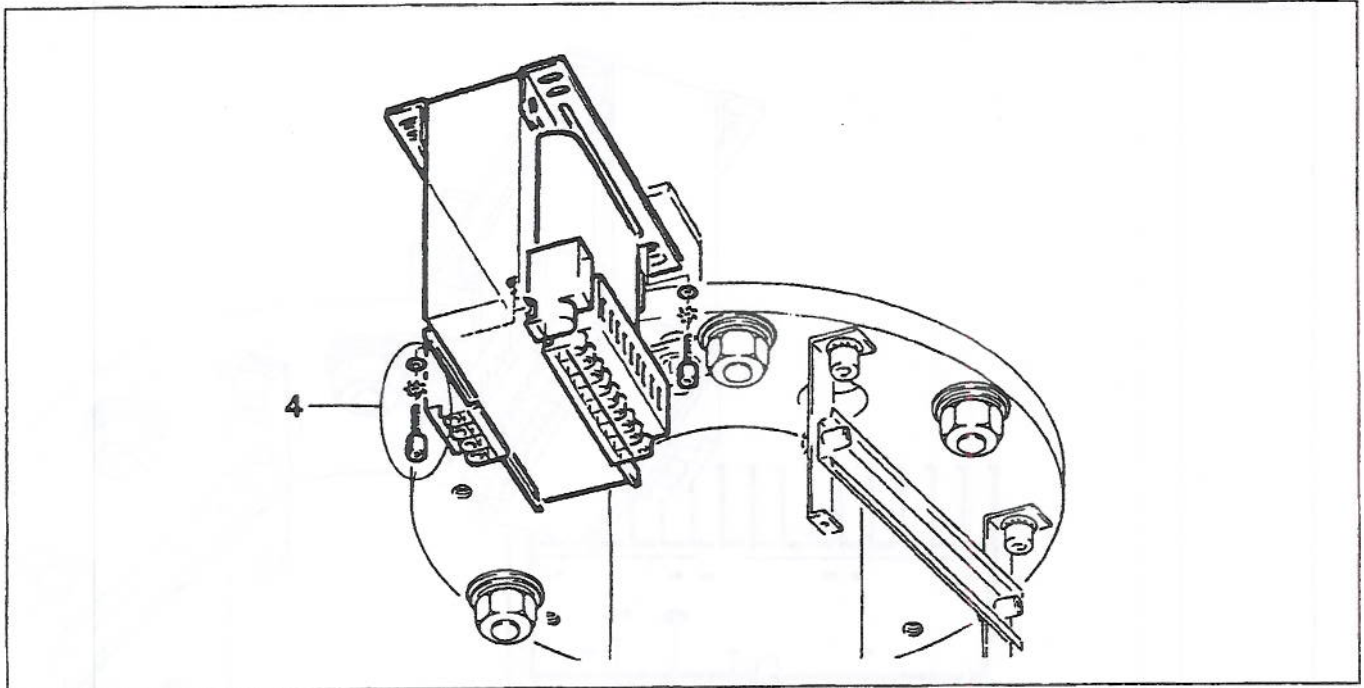
Disassembly of the rectifier and the heatsink

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
8.1 CT/MSU - Version 1
(for one ML 500/ML 700
ceiling mounting
with emergency current switch over)

- Dismounting/Mounting
(parts of the mains supply unit)

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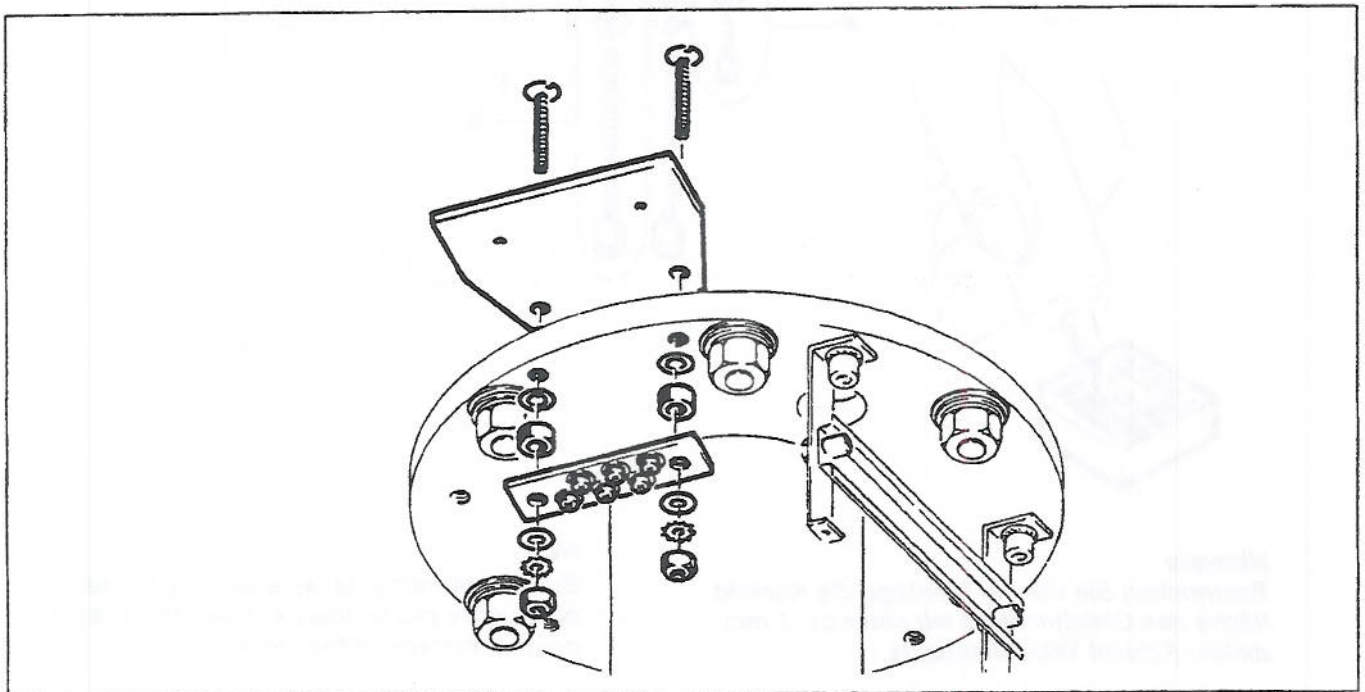*Disassembly of the grounding plate and the
transformer plate*

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**Illumination at 1 m distance with
colour temperature 4.500 K**
light field diameter 130 mm, 75.000 Lux

Diameter

- of poly-parabolic-reflector 410 mm
- of lamp housing 500 mm
- of small light field 130 mm
- of large light field 250 mm

Light field adjustment

through turning on the sterilizable handle

**Constant light field diameter
without refocusing**
up to 500 mm

Main light source
120 W, 24 V DC

Reserve light source
120 W, 24 V DC

**By main bulb failure automatic switch over
to reserve bulb (with optical warning)**

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**9.1 Technical Data
ML 500**

- General

Operating elements
on a control panel on the light head

Finish (bi-colour)
RAL 7035/RAL 7001

Electrical system
in acc. with VDE 0750 and IEC 601.1

Weight (complete with suspension)
31 kg

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Illumination at 1 m distance with colour temperature 4.500 K

light field diameter 170 mm, 105.000 Lux

Diameter

- of poly-parabolic-reflector 560 mm
- of lamp housing 700 mm
- of small light field 170 mm
- of large light field 300 mm

Light field adjustment

step-lessly adjustable via servo-motor

Brightness adjustment

step-less

Constant light field diameter without refocusing

up to 500 mm

Main light source

150 W, 24 V DC

Reserve light source

150 W, 24 V DC

By main bulb failure automatic switch over to reserve bulb (with optical warning)

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**9.2 Technical Data
ML 700**

- General

Operating elements
on a touch pad control panel

Finish (bi-colour)
RAL 7035/RAL 7001

Electrical system
in acc. with VDE 0750 and IEC 601.1

Weight (complete with suspension)
60 kg

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- Operating light - electronics

Limiting values:

Signal voltage, power supply transformer
44 V_{eff}

Signal voltage, battery
40 V

Characteristic data:

Signal voltage, power supply transformer
25 V_{eff} ... 35 V_{eff}

Signal voltage, battery
25 V ... 30 V

Power Input U_E = 30 V:

Static current

Lamp off < 60 mA
Main lamp defect < 100 mA

Power consumption

Main and reserve lamp defect
Light field max.
Brightness max. < 200 mA

Lamp current

max. 12 A

Motor current in case of short circuit

< 100 mA

Gray-Code current in case of short circuit

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- Operating light - electronics

Voltages:

Motor voltage in operation
approx. 9 V

Lampvoltage, max. brightness*
22,7 V +/- 0,5 V_{eff}

Lampvoltage, min. brightness*
16,7 V +/- 0,7 V_{eff}

* Measurements to be conducted only with RMS
responsive meters (preferably ITT METRIX MX 47).

Martin Medizin-Technik · Ludwigstaler Straße 132
Postfach 60 · D-7200 Tuttlingen · Germany (Fed. Rep.)
Telefon (074 61) 706-0 · Telex 762 696 gema d.
Telegramme · Gema Tuttlingen
Telefax (074 61) 706 193
Teletex 74 61 406