LED Slit Lamp

XL-1

Service Manual

This service manual is for Ajinomoto's Service engineers and their distributor's service person.

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→Open the slit ring.

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Illumination control volume is set to low position.

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→Set to high position to get more light power.

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Please contact to Sales representative if above action does not help.

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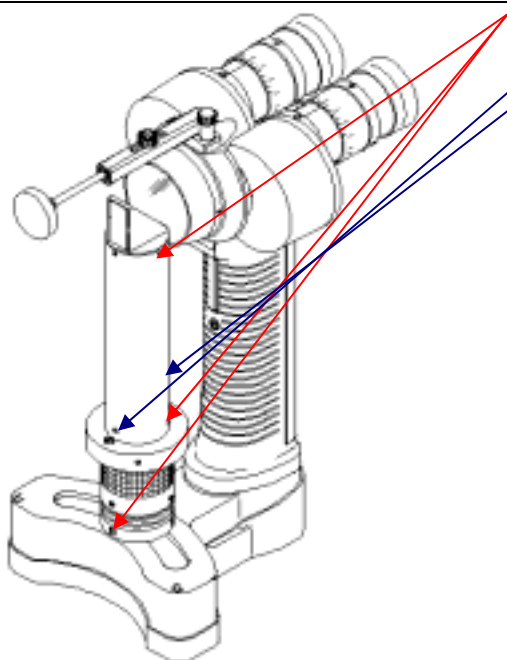
2. **Slit light is not at center position.**

Slit light is shifting to right or left.

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A) Turn the Illumination tube by observing the illumination light. You need to un-loosen the screw and rotate then tighten the screws when you adjusting the light position.

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B) Please take a next step if the above A) does not work well.

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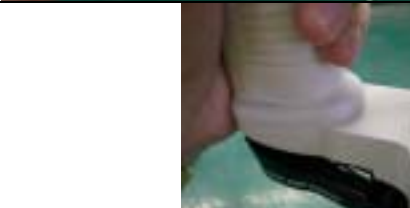
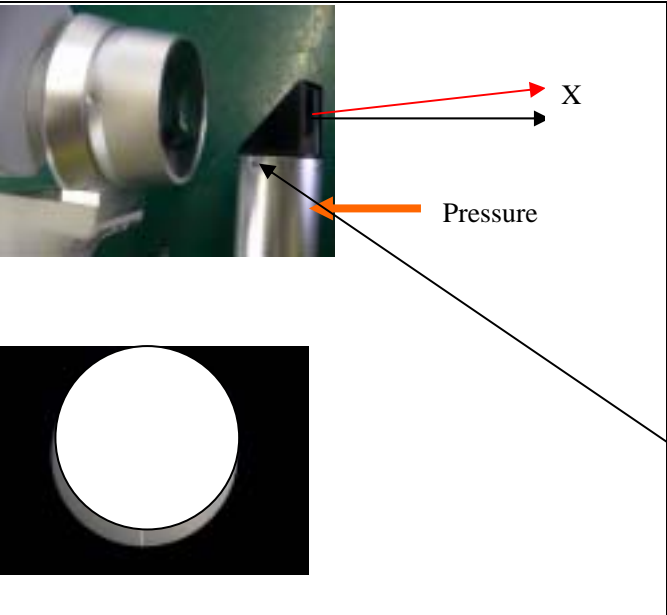
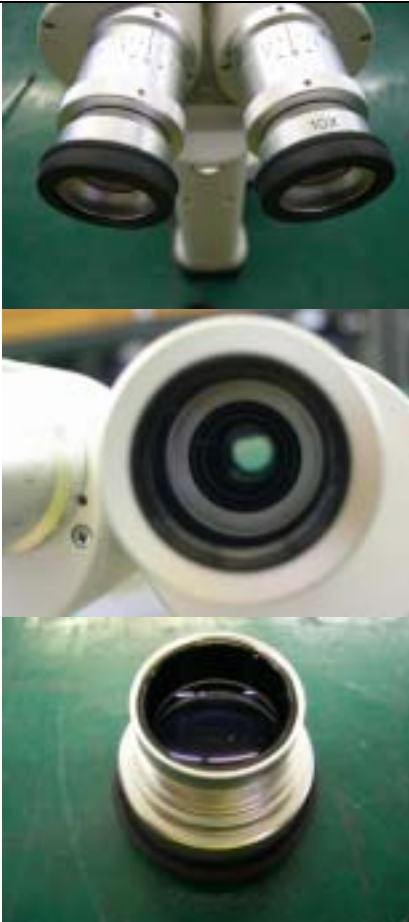
2. Please ask sales distributor if the objective lenses are damaged. You can not replace the objective lenses without special calibration instruments.

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5. Illumination Sliding is heavy
(照明のスライドが重い)

Please add some silicon oil or grease.
Grease lightly to inside of sliding position.
Do not grease too much on the top of the sliding place.

EU Representative

FAUSTO GUZZETTI (SHIN-NIPPON TECHNICAL CENTER)

VIA SOLFERINO,5 21057 OLGiate OLONA(VA), Italy

Supplied by

AJINOMOTO®

AJINOMOTO TRADING, INC.

SHIN-NIPPON Medical & Ophthalmic Instruments Dept.

East Wing 7F, TFT Building, 3-1-22 Ariake, Koto-Ku, Tokyo 135-8071, JAPAN

TEL:813-3528-4416 FAX: 813-3528-4426

<http://www.shin-nippon.jp> <http://www.ajitrade.com>

Manufactured by

Ohira Co., Ltd

258 Nishi Izumida, Minami-Uonuma, Niigata 949-6615, JAPAN

Approval: 15B2X00003