



Ambulatory Blood Pressure Report Generator/Printer

90239A

Service Manual

070-0398-00 Rev. C

Copyright 2003 Datex-Ohmeda, Inc.

All rights reserved. Contents of this publication may not be reproduced in any form without the written permission of Datex-Ohmeda, Inc. Products of Spacelabs Medical, a Division of Instrumentarium, Datex-Ohmeda, Inc. ("Spacelabs Medical") are covered by U.S. and foreign patents and/or pending patents. Printed in U.S.A. Specifications and price change privileges are reserved.

Spacelabs Medical considers itself responsible for the effects on safety, reliability and performance of the equipment only if:

- assembly operations, re-adjustments, modifications or repairs are carried out by persons authorized by Spacelabs Medical, and
- the electrical installation of the relevant room complies with the requirements of the standard in force, and
- the equipment is used in accordance with the operations manual.

Spacelabs Medical will make available, on request, such circuit diagrams, component part lists, descriptions, calibration instructions or other information which will assist appropriately qualified technical personnel to repair those parts of the equipment which are classified by Spacelabs Medical as field repairable.

Spacelabs Medical is committed to providing comprehensive customer support beginning with your initial inquiry through purchase, training, and service for the life of your Spacelabs Medical equipment.

CORPORATE OFFICES

U.S.A.

Spacelabs Medical
5150 220th Ave SE
Issaquah, WA 98029
Telephone: 425-657-7200
Telephone: 800-345-2700
Fax: 425-657-7212

AUSTRALIA

Datex-Ohmeda Pty Ltd
Unit 1, 149 Arthur Street
Locked Bag 356
Homebush, NSW 2140
Telephone: 61-2-9735-7222
Fax: 61-2-9764-2354

AUSTRIA & BELGIUM

Meda n.v.
Oeyvaersbosch 12
B-2630 Aartselaar
Belgium
Telephone: 32-3-870-1111
Fax: 32-3-870-1112

CANADA

Datex-Ohmeda Inc.
1093 Meyerside Drive, Unit 2
Mississauga, Ontario L5T 1J6
Telephone: 905-565-8572
Fax: 905-565-8592

CHINA

Datex-Ohmeda Pte Ltd
Shanghai Representative Office
Room 2509 Lippo Plaza
No. 222 Huaihai Road (M)
Shanghai 200021
Telephone: 86-21-5382-5657
Fax: 86-21-5382-1691

FRANCE

Spacelabs Medical
ZAC de Sans-Souci
1211 Chemin de la Bruyère
69760 Limonest
Telephone: 33 4 78 666 210
Fax: 33 4 78 432 658

GERMANY

Datex-Ohmeda GmbH
Dr. -Alfred-Herrhausen-Allee 24
D-47228 Duisburg
Telephone: 49-2065-691-0
Fax: 49-2065-691-236

HONG KONG

Spacelabs Medical Limited
Suite 901 Tower 1
China Hong Kong City
33 Canton Road, Tsimshatsui
Kowloon
Telephone: 852-2376-1370
Fax: 852-2376-2502

INDIA

Datex-Ohmeda India Pvt. Ltd.
International Trade Tower
S 3 Level, Block E
Nehru Place
New Delhi 110019
Telephone: 91-11-621-6060
Fax: 91-11-621-3003

ITALY

Datex-Ohmeda S.p.A.
Via Cassanese, 100
20090 Segrate (MI)
Telephone: 39-02-216-931
Fax: 39-02-2692-6226

SPAIN

Datex-Ohmeda SL
Manuel Tovar, 26
28034 Madrid
Telephone: 34-91-3342600
Fax: 34-91-3581284

TAIWAN

Datex-Ohmeda Pte Ltd
Taiwan Representative Office
2/FI No. 85 Sec. 2
Chien-Kuo N. RD.
Telephone: 8862-2515-0457
Fax: 8862-2501-9136

THE NETHERLANDS

Datex-Ohmeda B.V.
De Wel 18 3871 MV
Hoevelaken
Telephone: 31-33-25-41-222
Fax: 31-33-25-41-223

Authorized EC Representative UNITED KINGDOM

Datex-Ohmeda Ltd
71 Great North Road, Hatfield
Herts AL9 5EN
Telephone: 44-1707-263-570
Fax: 44-1707-260-065



CAUTION:

- US Federal law restricts the devices documented herein to sale by, or on the order of, a physician.

Contents

Chapter	Page
Introduction	
Overview	1-1
Specifications	1-2
Control Panel Keys	1-3
Installation and Set-up	
Overview	2-1
Loading Paper	2-1
Removing Paper	2-2
Power ON Self Test	2-2
Theory of Operation	
Input/Output	3-1
Central Processing Unit (CPU)	3-3
Block Diagram	3-4
Routine Maintenance	3-4
Troubleshooting	
Main Board/Printer Unit	4-1
Control Panel	4-1
Error Messages	4-2
Additional Error Messages	4-5
Repair	4-6
Assembly/Disassembly Procedures	4-6
Replacing the Control Panel	4-7
Replacing the Printer Unit	4-7
Replacing the Main Board	4-7
Top Level Assembly	4-8
Symbols	

Introduction

Contents

Overview 1
Specifications 2
Control Panel Keys 3

Overview

The 90239A Ambulatory blood Pressure (ABP) Report Generator assembles raw blood pressure data collected by the 90207 ABP monitor into tabular and/or graphical reports. This information can be printed immediately or transferred to data cards for storage and later printing. A battery-backed real-time clock/calendar and memory store the configuration set-ups.

A card slot in the front of the Report Generator (refer to *Figure 1-1*) accepts the following:

- a program update card (for changing the program contained on an internal Flash ROM), or;
- a data card (for storing patient data retrieved from the 90207 ABP Monitor).

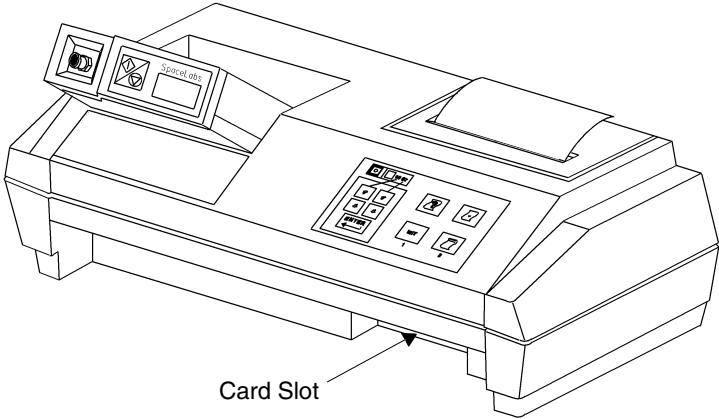


Figure 1-1: 90239A ABP Report Generator

Ambulatory Blood Pressure Report Generator/Printer

Specifications

Physical

Height:	3 inches (7.6 cm)
Width:	10 inches (25.4 cm)
Depth:	6 inches (15.2 cm)
Weight:	1 lb 6.2 oz (0.63 Kg)

Power

120 Volts AC/9 VDC, 800mA adapter (P/N 334-0264-01)

240 Volts AC/9 VDC, 800mA adapter (P/N 119-0125-00)

Operating Environment

Temperature:	+50° F to +104° F (+10° C to +40° C)
Relative Humidity:	0% to 95% non-condensing
Altitude:	-500 to 10,000 feet (-200 to 3900 meters)
Vibration:	0.025-inch (0.635mm) displacement in all three axes from 10 to 60 Hz

Non-Operating Environment

Temperature:	-29° F to +149° F (-34 C to +65 C)
Relative Humidity:	0% to 95% non-condensing
Altitude:	-500 to 10,000 feet (-200 to 3900 meters)
Vibration:	0.025-inch (0.635mm) displacement in all three axes from 10 to 60 Hz

Radio Frequency Interference

Standards

VDE:	0871, Class B
FCC:	Class B

Control Panel Keys

The user enters requests to the Report Generator via two groups of keys located on the front of the unit, refer to *Figure 1-2*. These keys are fully explained in the *90239A ABP Report Generator Operations Manual (P/N 070-0399-xx)*.

- The group of four keys on the right cause the immediate execution of a function.
- The group of five keys on the left are used in programming the ABP Monitor.

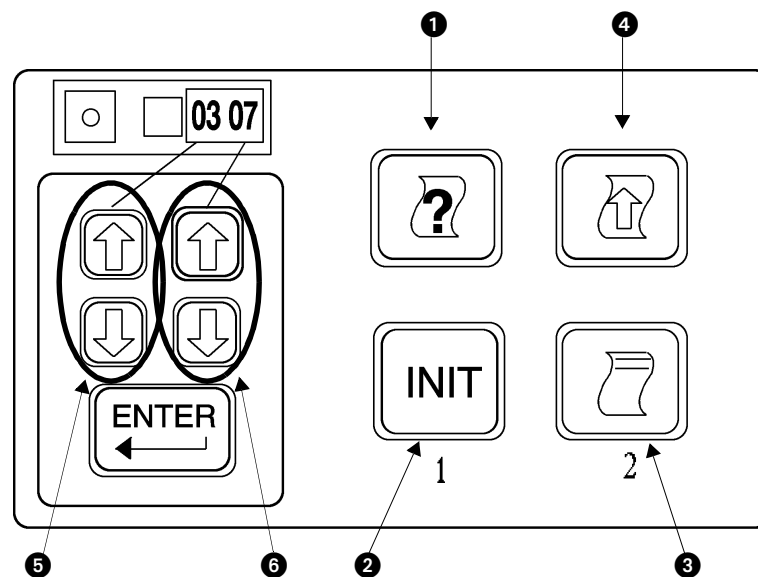


Figure 1-2: Control panel keys

- ❶ HELP (symbolized by a question mark on a sheet of paper)
- ❷ INIT (pressing this key initialized the ABP Monitor)
- ❸ PRINT (symbolized by a sheet of paper containing two lines of text)
- ❹ PAPER ADVANCE (symbolized by a sheet of paper with an UP arrow)
- ❺ The left pair of UP/DOWN arrows affect the two left-most digits on the ABP Monitor display
- ❻ The right pair of UP/DOWN arrows affect the two right-most digits on the ABP Monitor display

Installation and Set-up

Contents

Overview	1
Loading Paper	1
Removing Paper	2
Power ON Self Test	2

Overview

The Report Generator is shipped from the factory with the required software and hardware already installed. The unit requires only an ABP Monitor and electrical power to function.

Loading Paper



CAUTION:

- **Use only Spacelabs Medical thermal paper (P/N 006-0210-02) in the printer; the print mechanism may be permanently damaged if other brands of paper are used.**

1. Orient the paper roll so that it feeds from the bottom, and fold the left side of the paper over so that the right side forms a point.
2. Remove the plastic cover over the cradle, and feed the point through the slot in the bottom of the paper roll cradle until it emerges in the print area (the paper must feed from the bottom of the roll).
3. Pull the paper through until the fold clears the print area. (A pair of tweezers may assist in performing this step.)
4. Feed the paper up through the slot in the printer cover.
5. Replace the plastic cover. This cover **MUST** be in place during printer operation.

Ambulatory Blood Pressure Report Generator/Printer

Removing Paper

To remove paper from the Report Generator, pull the paper in the direction of normal paper movement.



CAUTION:

- **DO NOT** pull the paper backwards through the printer. Reversing the normal direction of paper movement may permanently damage the paper-feed mechanism. Refer to the *90239A ABP Report Generator Operations Manual (P/N 070-0399-xx)* for further information and instructions.

Power ON Self Test

Each time the Report Generator is powered ON, it tests its RAM and ROM areas. A successful self-test results in the Report Generator printing a message similar to the following:

```
Spacelabs 90239A
ABP Report Generator
Vxx.xx.xx
Copyright (c)
2003
Spacelabs

90207 V xx.xx.xx

Date: 03-Feb-03
Time: 14:37
```

If an error message prints out, refer to *Troubleshooting* on page 4-1.

Theory of Operation

Contents

Input/Output	1
Central Processing Unit (CPU)	3
Block Diagram	4
Routine Maintenance	4

Input/Output

Report Generator

The Report Generator communicates with the ABP Monitor over an infrared (IR) serial port. Two lines are used: one to transmit and one to receive. The port is activated by an internal magnet in the bottom of the chute on the Report Generator. This port is capable of baud rates up to 9600 bps.

Control Panel Key Logic

The processor periodically and sequentially scans the control panel keys to determine if any are being pressed. This is determined by whether a key is logic high (+5V) or low (OV) when the key line is read.

Internal Thermal Printer

The CPU controls the thermal printer by sending information to a buffer that feeds data lines to the hot dot drivers. The motor control comes directly from the processor and goes to a motor drive circuit that is capable of providing the power required to drive the motor. Motor rotation and hot dot head position signals from the printer are conditioned and sent on to the processor for use in controlling the printer. The signals from the printer are conditioned and sent on to the processor for use in controlling the printer. The motor rotation signal is a crude sine wave that is turned into a square wave for use by the processor for synchronization.

Power Input

Power is provided from a wall transformer that provides unregulated +9 volts DC at 800mA into the rear of the Report Generator. This unregulated +9VDC input voltage is filtered, and part of it is stepped up to +12VDC. It is then sent to the internal circuitry via three regulated power supplies:

- +5V provides power to analog and digital circuitry;
- +5PV supplies power to high-current sources such as the printer;
- +12V used to program the Flash ROM.

Ambulatory Blood Pressure Report Generator/Printer

RAM/ROM

Program execution is dictated by the software on the Flash ROM and the microprocessor ROM. The RAM performs two functions:

- holds patient data and variables during operations;
- holds graphic images and other information used during the creation and printing of a report.

All functional modes are programmed by means of an internal 256 Kbyte Flash ROM, which performs the following functions:

- holds the software necessary for each operational mode;
- holds 8 Kbytes of internal microprocessor code;
- contains the software for the 90207 ABP Monitor;
- contains phrases in 5 languages.

The software for the Monitor can be downloaded from the Report Generator if the code in the Monitor's RAM needs to be updated.

ROM Card/Data Card

Two types of cards can be inserted into the card slot on the front of the Report Generator:

- A SRAM data card that holds patient data retrieved from ABP Monitors is used for storage and/or later printing. This type of card can be write-protected to prevent accidental erasure of data.
- A Program updated card is used to update the Report Generator program in the Flash ROM and also the most current ABP Monitor code.

Real Time Clock

The Real Time clock is used for the following functions:

- storing the current time and date used in resetting the ABP MOnitor during initialization and for the printed report;
- storing settings used in initialization.

The Real Time clock is battery backed-up so that the time or program settings are not lost after the programming. The clock also stores calibration information for control of the hot dot intensity during printing.

Central Processing Unit (CPU)

The CPU (comprised of a microprocessor used in conjunction with the ROM and RAM and the addressing logic) performs the following functions:

- executes the software stored in the flash ROM memory or microprocessor ROM;
- stores, retrieves, and processes data;
- controls the thermal printer for printing ABP reports and user messages;
- scans the front panel keys and takes action when a key is pressed;
- communicates with the ABP Monitor via the infrared serial port in the monitor chute, with the Real Time Clock, with the serial port and the data key parallel I/O ports;
- downloads software from the ROM program card and programs it into the Flash ROM;
- transfers data from the ABP MONitor to the optional data card.

The processor initially executes its instructions from the ROM internal to the microprocessor. If the self-tests are passed, code can then be executed out of the Flash ROM. The processor can write to the I/O devices by means of latches and buffers tied to the processor's bus and with the microprocessor's I/O ports. The processor interfaces with the following devices:

- memory (microprocessor internal ROM, Flash ROM, plug-in data card RAM or Program update card ROM, and main board RAM);
- printer;
- ABP Monitor (via the IR serial port);
- Real Time Clock (located on the main board).

Ambulatory Blood Pressure Report Generator/Printer

Block Diagram

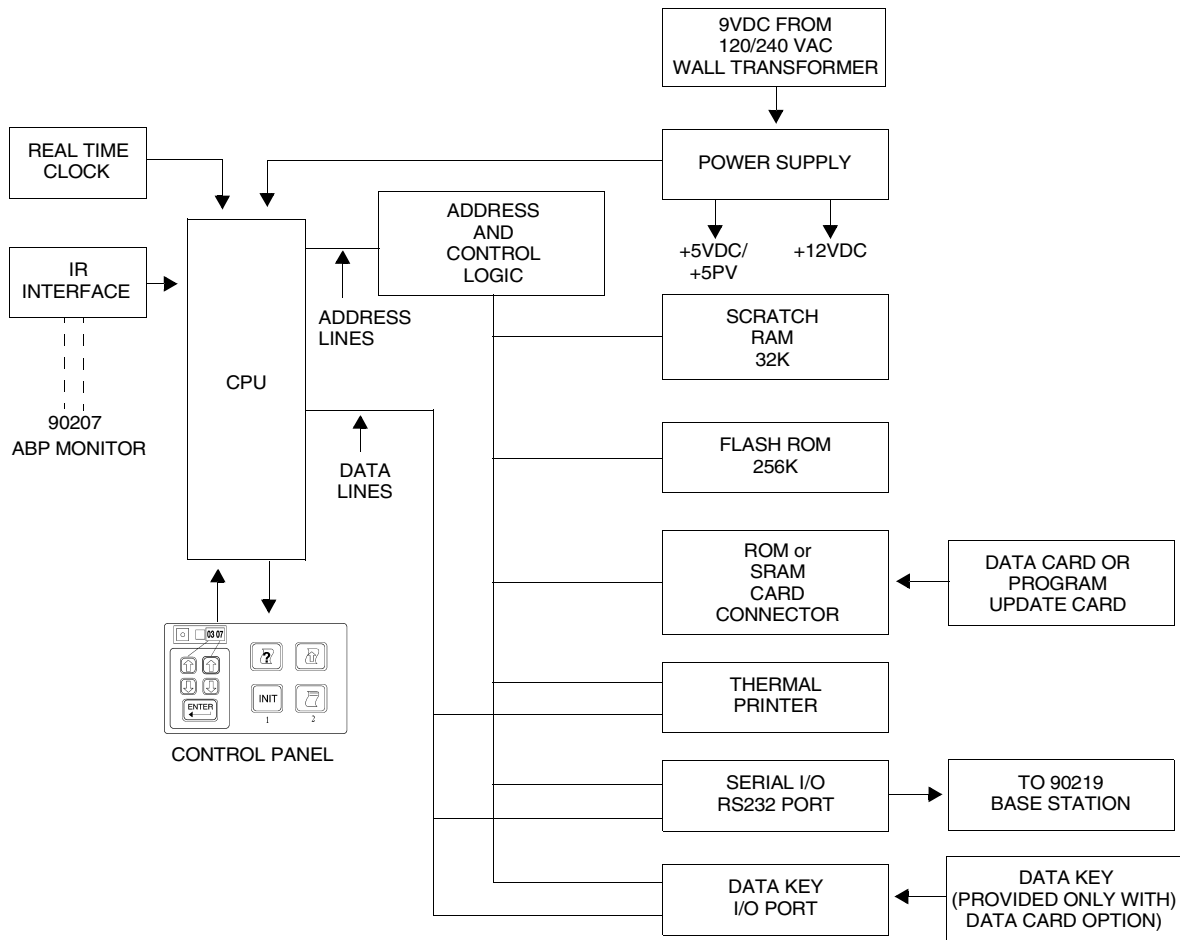


Figure 3-1: 90239A Report Generator

Routine Maintenance

Cleaning

The following steps should be performed on an as-needed basis:

- Wipe all external surfaces with a damp cloth. Ensure that no dust, lint, etc., obstructs the infrared window in the bottom of the chute.
- Remove the printer cover and vacuum out accumulated dirt and paper fiber from the print head area. Use tweezers to remove paper scraps.

Troubleshooting

Contents

Main Board/Printer Unit	1
Control Panel	1
Error Messages	2
Additional Error Messages	5
Repair	6
Assembly/Disassembly Procedures	6
Replacing the Control Panel	7
Replacing the Printer Unit	7
Replacing the Main Board	7
Top Level Assembly	8

Main Board/Printer Unit

The Report Generator should print out header information (software version, the current date, etc.) at each power-up to indicate a functioning main board and printer unit.

In case the header information doesn't print at power-up, replace the printer unit first, then the main board.

Control Panel

If the Report Generator does not respond to certain control panel commands, continuity for the keys can be checked using an ohmmeter across the J2 plug pins. Pin 1 is indicated on the plug by the small triangle molded into the plastic.

<u>Control Panel Key</u>		<u>Continuity Between:</u>
Left UP arrow		Pins 1 and 2
Left DOWN arrow		Pins 1 and 5
Right UP arrow		Pins 2 and 4
Right DOWN arrow		Pins 4 and 5
ENTER key		Pins 1 and 7
HELP key		Pins 2 and 3
INIT key		Pins 5 and 6
LINEFEED key		Pins 6 and 7
PRINT REPORT key		Pins 7 and 8

Ambulatory Blood Pressure Report Generator/Printer

Error Messages

Error messages print whenever a fault condition occurs during operation of the Report Generator. These messages take one of the following forms:

- Interface Fault “n”
- Hardware Interface Fault “n”
- Error “n”

“n” represents one of the error numbers in the following table:

Error Number	Explanation and Possible Causes	Printed Message
2	Serial time-out (the 90207 ABP Monitor and the Report Generator failed to interface). Low batteries in 90207 ABP Monitor. 90207 monitor removed before data was fully read out.	None
3	The 90207 ABP Monitor failed to understand instructions from the Report Generator. Low batteries in 90207 ABP Monitor or bad LEDs in Report Generator. 90207 ABP Monitor misaligned in chute.	None
4	The 90207 ABP Monitor failed to give any response. Low batteries or bad LEDs in Report Generator. 90207 ABP Monitor misaligned in chute.	None
5	Maximum retries exceeded. Defective 90207 ABP Monitor. Defective infrared LEDs in Report Generator.	None
6	Bad 90207 ABP Monitor RAM. Refer to the <i>90207/90217 Ambulatory Blood Pressure Monitors Operations Manual (P/N 070-0137-xx)</i> for instructions.	Low batteries. Monitor RAM fault.
7	The Report Generator does not detect the 90207 ABP Monitor in the chute. Try reinserting the monitor into the chute. Bad lithium battery in 90207 ABP Monitor. Infrared LEDs on main board may be misaligned. Missing magnet on main board. Bad main board.	None

Troubleshooting

Error Number	Explanation and Possible Causes	Printed Message
8	Low battery or bad clock chip on main board. Try resetting the clock. Bad lithium battery on the main board. Bad main board.	90239A Hardware Fault 8.
9	Aborted printing the report. Bad control panel. Bad main board.	Print report aborted.
10	Aborted printing the Help menu. Bad control panel. Bad main board.	Print help aborted.
11	Aborted printing the data card summary. Bad control panel. Bad main board.	Print card summary aborted.
12	Power was removed while updating the Report Generator software or the update card was pulled out prematurely. Bad main board.	None
13	The 90207 ABP Monitor contains no data. Bad 90207 ABP Monitor.	Monitor contains no data.
14	Setup data checksum error. Bad control panel. Bad main board.	Setup data checksum error. Resetting all menu item values to factory defaults. Following is the current date and time. Verify and correct if necessary.
15	Invalid time received from clock. Bad control panel. Bad main board.	Invalid 90239A time or date detected. Please reset time and date.
16	Switch on data card set to write-protect. Bad data card. Bad main board.	None
17	The data card is full. Bad data card. Bad main board.	None

Ambulatory Blood Pressure Report Generator/Printer

Error Number	Explanation and Possible Causes	Printed Message
18	<p>The data card is not formatted. Refer to the <i>90239A Ambulatory Blood Pressure Report Generator Operations Manual (P/N 070-0399-xx)</i> for formatting instructions.</p> <p>Bad data card.</p> <p>Bad main board.</p>	None
19	<p>The data card has suffered a failure while printing data.</p> <p>Bad scan report on data card. Reformat the card.</p> <p>Bad main board.</p>	None
20	<p>Unknown or write-protected card.</p> <p>Turn write protection OFF.</p> <p>Bad data card.</p> <p>Bad main board.</p>	Card is unrecognized or write protected.
21	<p>Bad password.</p> <p>Try removing batteries from the 90207 ABP Monitor, then replacing them. Then re initialize the Monitor if using the 90219 ABP PC Direct/Base Station. This erases any data currently stored in the 90207 ABP Monitor.</p>	<p>Monitor has a different password.</p> <p>Cannot unlock monitor.</p>
22	<p>No lithium battery.</p> <p>Bad lithium battery in 90207 ABP Monitor.</p>	Missing lithium battery.
23	<p>Lithium battery low.</p> <p>Bad lithium battery in 90207 ABP Monitor.</p>	<p>Lithium battery measures low.</p> <p>Replace lithium battery.</p>
24	<p>Paper is not feeding through the print unit. If the 90207 ABP Monitor is placed in the chute, it starts beeping to indicate a printer jam problem.</p> <p>After the problem is resolved, press the line feed key to print the error message.</p>	Aborted due to printer fault. Try again.
25	<p>The data card is not present in the card slot.</p> <p>Bad data card.</p> <p>Bad main board.</p>	No data card present.
26	<p>Low data card battery.</p> <p>Improperly installed data card battery, or bad battery.</p> <p>Defective data card.</p> <p>Bad main board.</p>	The data card battery is low.

Additional Error Messages


The following error messages can also print. Because these errors are not identified in the normal program flow, they do not have error numbers.

Error Number	Printed Message	Possible Causes
None	Monitor software fault. Loading monitor software. 90207 V xx.xx.xx. Monitor software successfully installed in monitor.	Possible problem with 90207 ABP Monitor. Bad main board.
None	AA batteries measure low. Replace batteries if they are alkaline. Verify they have been fully charged if they are NiCads.	Bad batteries in 90207 ABP Monitor. 90207 ABP Monitor erroneously reading battery charge as low.
None	Software on update card is incompatible with system software in 90239A. Update not performed.	Bad update card. Bad main board.
None	Data checksum error.	Bad control panel. Bad main board.
None	Update card failed verification test. Update not performed.	Perform software update with another update card. Bad main board.

Ambulatory Blood Pressure Report Generator/Printer

Repair

Refer to the board layout diagram below for component locations.

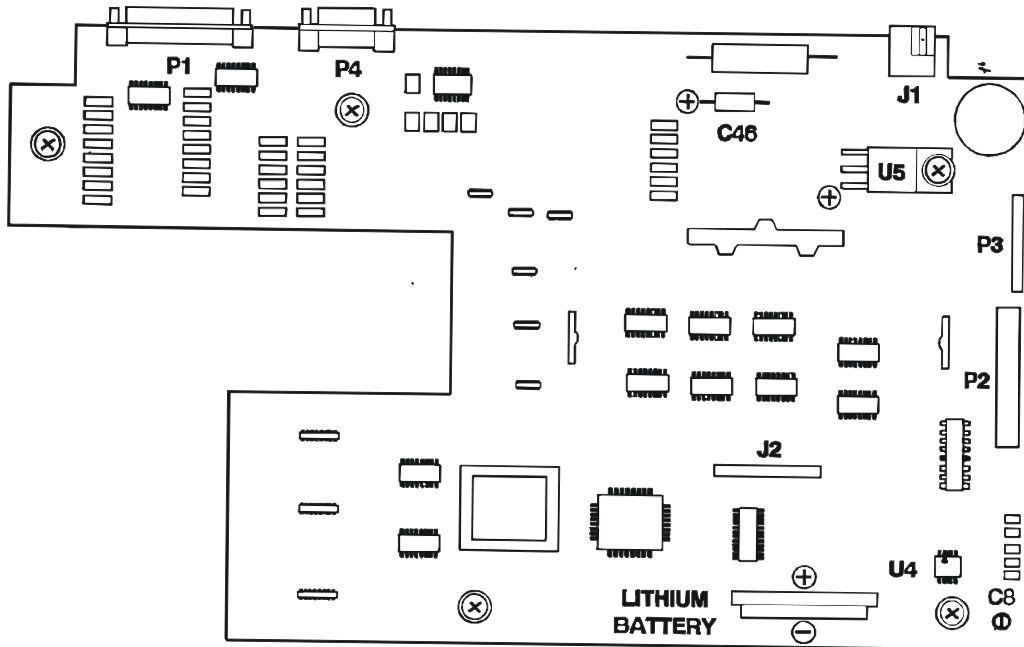

Assembly/Disassembly Procedures

A cross-tip (Phillips) screwdriver will complete most assembly -disassembly procedures. Externally 4 Phillips screws connect the upper and lower halves of the Report Generator.

- The upper half contains the front control panel and the thermal printer.
- The lower half contains the main board and the infrared LEDs that interface with the 90207 ABP Monitor.

Internally, three cables also connect the upper and lower halves:

- One cable connects the control panel to the main board.
- Two cables connect the thermal printer unit to the main board.

To disassemble the Report Generator:

1. Remove the 4 Phillips screws located on the bottom of the unit.
2. Separate the unit into an upper and lower half (they will still be connected together by the cables).

Replacing the Control Panel

The control panel is glued onto the Report Generator via an adhesive backing, so the entire panel can be simply peeled off. A small, flat-blade screwdriver will assist in this procedure. After disassembling the Report Generator:

1. Disconnect the panel ribbon cable from the main board at the J2 connection.
2. Use the small, flat-blade screwdriver (or a similar tool) to pry up one corner of the panel switch, then peel it off.
3. Remove the paper backing from the new control panel, and press it into place.
4. Reconnect the cable to J2 on the main board, and reassemble the Report Generator.

Replacing the Printer Unit

This procedure requires a working 90207 ABP Monitor. After disassembling the Report Generator:

1. Disconnect the two printer cables from the P2 and P3 connections at the main board.
2. Remove the 2 Phillips screws that secure the printer mechanism to the upper cover of the unit.
3. Secure the new printer unit to the upper cover using the Phillips screws previously removed.

The Report Generator must now be adjusted to match the resistance of the new printer unit as follows; this adjustment directly affects print density:

1. Place the 90207 ABP Monitor into the chute, and reprogram the Monitor via Menu Items 12 - 14 to display the date 4/24/47.
2. Press the left set of UP/DOWN arrow keys on the control panel in the following order: left DOWN arrow, left UP arrow, right DOWN arrow, right UP arrows.
3. Press the left DOWN arrow key repeatedly until the right-hand pair of digits on the Monitor goes one step below 00.
4. The resistance of each printer unit is indicated by the letter A, B, or C stamped on the flexible cable where it inserts into the printer. Press the right UP or DOWN arrow key until the number corresponding to the resistance letter is displayed on the Monitor:
1 = C
2 = B
3 = A
5. Press ENTER to select the displayed letter.
6. Reset the date on the ABP Monitor to the current date.

Replacing the Main Board

After disassembling the Report Generator:


1. Disconnect the control panel cable at the J2 connection and the two printer cables at the P2 and P3 connections.
2. Remove the 5 Phillips screws that secure the board to the lower half of the Report Generator. The main board can now be removed.
3. Secure the new main board using the 5 Phillips screws previously removed.

Ambulatory Blood Pressure Report Generator/Printer

Reconnect the cables to the J2, P2, and P3 connections, and reassemble the Report Generator. Perform the "Voltage Check and Clock Adjustment" procedure.





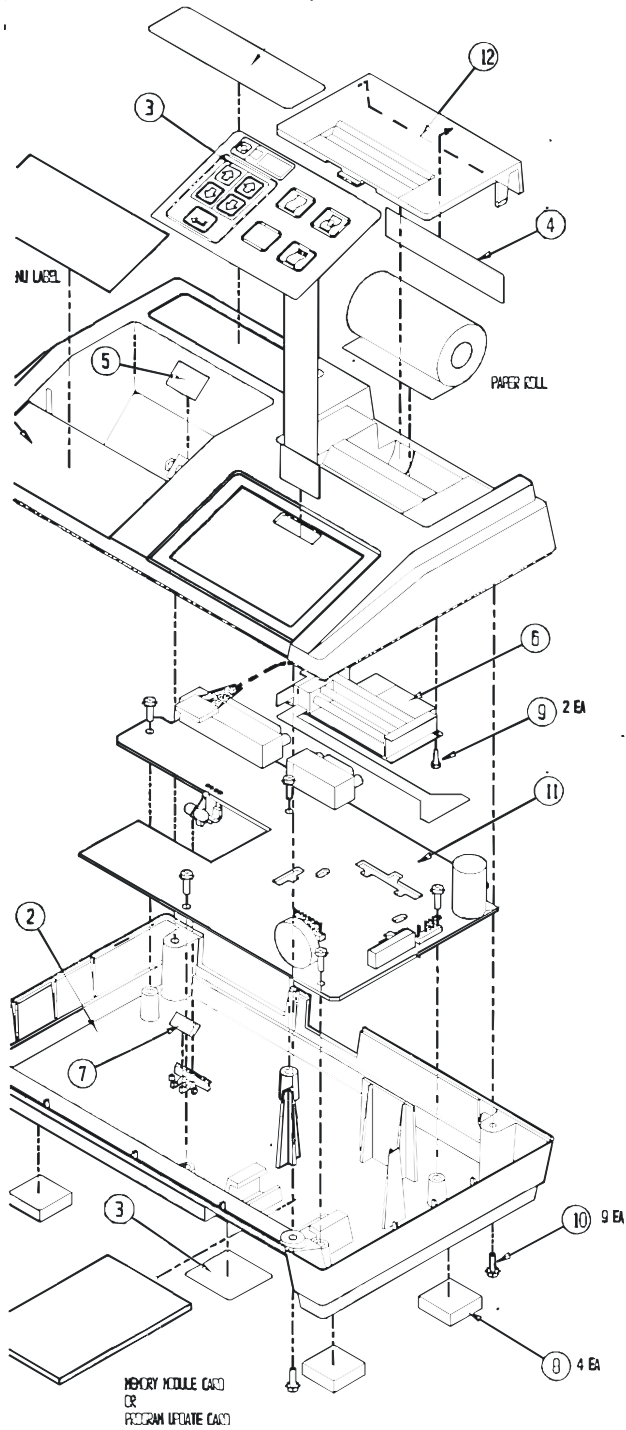
















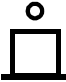


Top Level Assembly

The exploded view below shows the top level assemblies for the Report Generator.













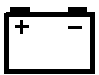
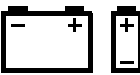








Symbols



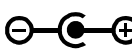




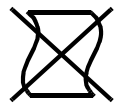


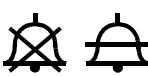











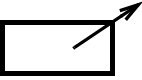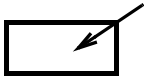

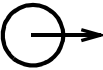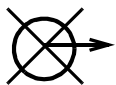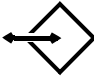

The following list of international and safety symbols describes all symbols used on Spacelabs Medical products. No one product contains every symbol.

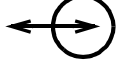Symbol	Description	Symbol	Description
	UCW or Ultraview 1700 HELP Key		UCW or Ultraview 1700 MONITOR SETUP Key
	UCW or Ultraview 1700 SPECIAL FUNCTIONS Key		UCW or Ultraview 1700 ALARMS Key
	UCW or Ultraview 1700 RECORD Key		UCW or Ultraview 1700 PREVIOUS MENU Key
	UCW or Ultraview 1700 NORMAL SCREEN Key		UCW or Ultraview 1700 mouse connection
	UCW or Ultraview 1700 Keyboard Connection		ON — Power Connection to Mains
	OFF — Power Disconnection from Mains		On Position for Push button Power Switch
	Off Position for Push button Power Switch		STOP or CANCEL Key
	CONTINUE Key		START/STOP Key
	START/STOP		START (NIBP) Key
	On Direction		ON/OFF
	Television; Video Display		Recycle
	Protective Earth Ground		Functional Earth Ground













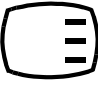





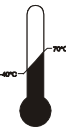

Ambulatory Blood Pressure Report Generator/Printer





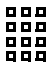


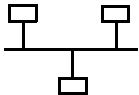
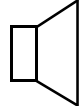

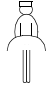
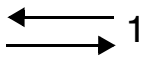
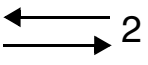

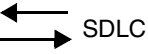
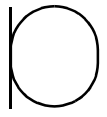
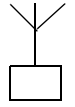
Symbol	Description	Symbol	Description
	ON — Part of the Instrument Only		OFF — Part of the Instrument Only
	Partial ON/OFF		STAND-BY Key
	All batteries should be disposed of properly to protect the environment. Lithium batteries should be fully discharged before disposal. Batteries such as lead-acid (Pb) and nickel-cadmium (Ni-Cd) must be recycled. Please follow your internal procedures and or local (provincial) laws regarding disposal or recycling.		Caution - hazardous voltages. To reduce risk of electric shock, do not remove the cover or back. Refer servicing to a qualified service personnel (U.S.A.). DANGER - High Voltage (International)
	PAUSE or INTERRUPT		Slow Run
	Replace Fuse Only as Marked		Fuse
	Power supply jack polarity. (+ / - Signs May be Reversed)		Equipotentiality Terminal
	Battery Replace only with the appropriate battery.		Replace only with the appropriate battery. (+ / - Signs May be Reversed)
	Alternating Current		Direct Current
	Both Direct and Alternating Current		AC/DC Input
A	Amperes	Hz	Hertz
V	Volts	W	Watts
	Temporary Shut Off of Alarm Tone or Screen Indicators		Alarm

Symbols


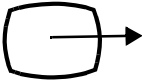


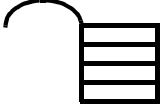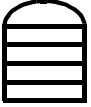
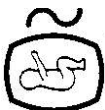
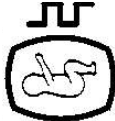











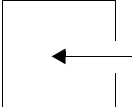
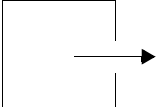
Symbol	Description	Symbol	Description
	ENTER Key		PRINT REPORT Key
	Attention - Consult Operations or Service Manual for Description		Risk of Explosion if Used in the Presence of Flammable Anesthetics
	Indicator — Remote control		Indicator — Local Control
	Return Unit to Monitor Mode		Indicator — Out of Paper
	Activate Recorder for Graphics		Recorder Paper
	Indoor Use Only		Auto Mode (NIBP)
	Output		No Output (Terminated)
	Data Input/Output		HELP (Explain Prior Screen) Key
	Clock/Time Setting Key		Input/Output
	Monitor Setup Select Program Options		Set Initial Conditions Menu
	Access Special Function Menu		Normal Screen
	Return to Prior Menu		TREND/TIMER Key
	Gas Exhaust		Electrocardiograph or Defibrillator Synchronization












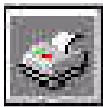
Ambulatory Blood Pressure Report Generator/Printer

Symbol	Description	Symbol	Description
	Arterial Pulse		IEC 601-1 Type BF equipment. The unit displaying this symbol contains an F-type isolated (floating) patient-applied part providing an adequate degree of protection against electric shock.
	IEC 601-1 Type BF equipment which is defibrillator-proof. The unit displaying this symbol contains an F-type isolated (floating) patient-applied part which contains an adequate degree of protection against electric shock, and is defibrillator-proof.		IEC 601-1 Type CF equipment. The unit displaying this symbol contains an F-type isolated (floating) patient-applied part providing a high degree of protection against electric shock.
	IEC 601-1 Type CF equipment. The unit displaying this symbol contains an F-type isolated (floating) patient-applied part providing a high degree of protection against electric shock, and is defibrillator-proof.		ETL Laboratory Approved
	IEC 601-1 Type B equipment. The unit displaying this symbol contains an adequate degree of protection against electric shock.		Canadian Standards Association Approved
	Keypad		Enlarge, Zoom
	Menu Keys		Delete
	Waveform/Parameter Keys		PCMCIA Card
	Keep Dry		Fragile; handle with care
	Foot Switch		This Way Up
	Environmental Shipping/Storage Temperature Limitations		Environmental Shipping/Storage Humidity Limitations

Symbol	Description	Symbol	Description
	Open Padlock		Closed Padlock
	Down Arrow		Up Arrow
	Event	TEMP temp	Temperature
	Antenna		Environmental Shipping/Storage Altitude Limitations
	Network Connection		Audio Output, Speaker
	Remote Alarm; Nurse Alert		Nurse Call
	Serial Port 1		Serial Port 2
	External marker push button connection		SDLC Port
	Microphone		Mermaid Connector

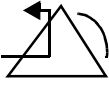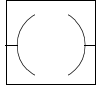Ambulatory Blood Pressure Report Generator/Printer

Symbol	Description	Symbol	Description
	Note		Video Output
	Warning About Potential Danger to Human Beings		Caution About Potential Danger to Equipment
	Non-Invasive Blood Pressure (NIBP), Neonate		Fetal Monitor Connection (Analog)
	Fetal Monitor Connection RS232 (Digital)		Physiological Monitor Connection RS232 (Digital)
	Input		Reset
	Hard Drive		Power Indicator LED
	Activate Telemetry Recorder		Omnidirectional Microphone
	Battery Status		Universal Serial Bus
	Stand-by		Low Battery
	Gas Sampling Port		Gas Return Port

Symbol	Description	Symbol	Description
	Operates on Non-Harmonized Radio Frequencies in Europe		Service Message
	Happy Face		Sad Face
	Reset		Power Indicator LED
	Magnifying Glass		Compression
	File Cabinet		List of Rooms
	Arrows		Printer

Ambulatory Blood Pressure Report Generator/Printer

Abbreviations used as symbols are shown below.

Symbol	Description	Symbol	Description
1 - 32	Access Codes 1 Through 32	AIR	Air
ANT 1 ANT 2	Diversity Antenna System 1 Diversity Antenna System 2	Arr1 ArrNet2	Arrhythmia Net 1 Arrhythmia Net 2
CH ch	EEG, EMG, or ECG Channel EEG Channels - CH1, CH2, CH3, CH4 EMG Channel - CH5	cmH₂O	Centimeters of Water
CMV	Controlled Mechanical Ventilation	C.O. CO co	Cardiac Output
DIA dia	Diastolic	ECG ecg	Electrocardiogram
EEG eeg	Electroencephalogram	EMG emg	Electromyogram
ESIS	Electrosurgical Interference Suppression	EXT	External
FECG	Fetal Electrocardiogram	FHR1 FHR2	Fetal Heart Rate, Channel 1 Fetal Heart Rate, Channel 2
GND gnd	Patient Isolated Ground	HLO hlo	High-Level Output
I:E	Inspiration Expiration Ratio	Multiview	Multi-Lead Electrocardiogram
NIBP nibp	Non-Invasive Blood Pressure	N₂O	Nitrous Oxide
O₂	Oxygen	PEEP	Positive End Expiratory Pressure
PRESS press PRS	Pressure	Pmin	Minimum Inspiratory Pressure

Symbol	Description	Symbol	Description
Ppeak	Peak Inspiratory Pressure	RESP resp	Respiration
SDLC	Synchronous Data Link Control	SPO2 SpO2 SpO₂ SaO₂	Arterial Oxygen Saturation as Measured by Pulse Oximetry
SVO2 SvO2 SvO₂	Mixed Venous Oxygen Saturation	SYS sys	Systolic
T1 T2 T3 T4	Temperature 1 Temperature 2 Temperature 3 Temperature 4	UA	Uterine Activity or Umbilical Artery
VAC	Vacuum connection		

BirthNet, CVScan, Data Shuttle, FT1000, FT3000, Flexchart, Flexform, Flexport, Flextable, Flextools, Flexview, Global Participant Index, Intesys, Multiview, Neoscan, PCIS, PCMS, PrintMaster, Quicknet, Sensorwatch, Spaceview, TRU-CAP, TRU-CUFF, TRU-LINK, UCW, Ultralite, Ultraview, Ultraview Clinical Messenger, Uni-Pouch, Universal Flexport, Varitrend, Web Source and WinDNA are trademarks of Datex-Ohmeda, Inc.

Other brands and product names are trademarks of their respective owners.