SFIGMOMANOMETRI ANEROIDI
ANEROID SPHYGMOMANOMETERS
TENSIOMETRES ANEROIDES
ESFIGMOMANÓMETROS ANEROIDES
ESFIGMOMANÓMETROS ANERÓIDES

Istruzioni per l'uso
Instructions for use
Mode d'emploi
Manual de instrucciones
Instruções de uso

0123

I

GB

F

E

P

FAZZINI

Your health is our aim

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EC Certificate**No.: G3M 01 10 44963 004**

Decision according to Annex VI of Council Directive 93/42/EEC concerning medical devices.

The Certification Body of TÜV PRODUCT SERVICE certifies that

FAZZINI s.r.l.
s.s. Padana Superiore 317
20090 Vimodrone Milano
Italy

in the facility(ies)

FAZZINI s.r.l.
s.s. Padana Superiore 317
20090 Vimodrone Milano
Italy

for the product(s)/product category(ies)

Aneroid sphygmomanometers
Mercury sphygmomanometers

maintains quality assurance which ensures the conformity with the type described in the technical documentation related to the measuring function.

Reasoned assessment see audit report no.: ITA 116141.

Provided the agreed periodical surveillance is carried out, this certificate is valid until 2006-10-21.

Released with the above mentioned certificate number by the Certification Body of TÜV PRODUCT SERVICE.

Department: MHS ITA / rg-tgu
 Date: 2001-10-22

TÜV PRODUCT SERVICE GMBH is Notified Body according to Council Directive 93/42/EEC concerning medical devices with identification no. 0123.

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ENGLISH

Contents	Page
Information on the various models of Fazzini aneroid sphygmomanometers and, where applicable, mounting of the same	13
Selection of suitable cuff sizes	13
How to put on the cuffs	14
How to measure blood pressure	14
Instructions for setting up the left-handed & right-handed mode	15
How to care for the aneroid sphygmomanometer	17
Test of precision	18
Maintenance	18
Technical Data	18

Key to symbols on the manometer scale:

Important: please read the documentation

Information on the various models of Fazzini aneroid sphygmomanometers

Thanks for purchasing a high-quality Fazzini precision aneroid sphygmomanometer manufactured in conformity with UNI-EN standard 1060/2: “Non-invasive blood pressure instruments”.

Sphygmomanometers with 2-tube cuffs, since the manometer is not connected directly to the bulb.

Sphygmomanometers with 1-tube cuffs. In these palm-style models, the bulb is connected directly to the manometer.

Selection of suitable cuff sizes

Cuffs

Our cuffs have a soft strip on one side and a snap on the other. This ensures that the cuffs can be easily opened and closed quickly and repeatedly. On all the models the cuffs are calibrated, i.e. provided with measurement lines. To make sure that you have chosen the right cuff size, check to see whether the white index line is in the range between the arrows after the cuff has been put on. If the index line fails to reach this range, the cuff is too small. If it is beyond the range, the cuff is too large. Precise blood pressure readings can be obtained only if the correct cuff size is used.

Cuff sizes with twin tubes available for all models:

cm.22x4,5 art.080.117.10	Infant
cm.40x5 art.080.118.10	Child
cm.50x10 art.080.119.10	Boy
cm.50x14 art.080.120.10	Adult
cm.60x17 art.080.123.10	Obese

How to put on the cuffs

Cuffs

Place the cuff 2-3 cm above the bend of the elbow on the naked left upper arm; make sure that the symbol "ø" is above the artery. Close the cuff by means of the velcro closure.

In units intended for home use, the diaphragm of the chestpiece integrated into the cuff must be placed above the artery.

To put on the cuff, pull the free end of the cuff through the metal D-shaped ring and close the cuff by means of the velcro closure.

How to measure blood pressure

After you have put on the cuff, pump it up by pressing the bulb until the pressure in the cuff is about 20 mm/Hg above the expected systolic blood pressure (upper value).

It is very important that the person whose blood pressure is being taken is relaxed and sitting comfortably. Furthermore, the lower arm should be placed at the level of the heart with the inside of the arm facing upwards. Place the chestpiece of the stethoscope on the artery under the cuff.

To be able to measure blood pressure, open the air-release screw on the bulb by turning it counter-clockwise. Ideally, the rate of air release should be 2 - 3 mm Hg/s. This rate can be adjusted by turning the screw very carefully. Perform a visual check of the air-release rate by looking at the scale: the pointer must move 1 - 1.5 graduations on the scale per second.

Once the upper blood pressure value i.e. (systolic pressure) has been reached, a rhythmic thumping sound is heard.

When the lower blood pressure value (diastolic pressure) is reached, the thumping sound stops.

Now open the air-release valve completely so that air can escape from the cuff immediately.

The blood pressure measurement procedure has now been completed.

We expressly call your attention to the fact that measuring your blood pressure at home is not a substitute for consulting a physician regularly; only a physician is capable of precisely analyzing the blood pressure readings you have taken.

INSTRUCTIONS FOR SETTING UP THE LEFT-HANDED & RIGHT-HANDED MODE

You are using a palm-type sphygmomanometer designed for the convenient use of both left-handed and right-handed people.

To properly use this instrument, please carefully read the following instruction for the switch between left-handed & right-handed mode.

1. The default factory mode is right-handed (fig. 1)

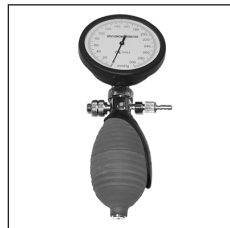
Fig. n1

2. To set up left-handed mode:

- a) Loosen completely the adjustment knob by turning it counter-clockwise (fig. 2-1);
- b) Turn the handle together with the connected valve counter-clockwise 180 degree (fig. 2-2);
- c) Turn the adjustment knob clockwise to tighten the knob (fig. 2-3)

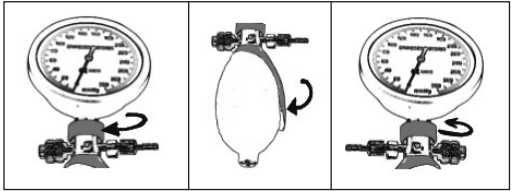
Fig. 2-1

Fig. 2-2

Fig. 2-3

3. To set up right-handed mode when it is in the left-handed mode

- a) Loosen completely the adjustment knob by turning it counter-clockwise (fig. 3-1);
- b) Turn the handle together with the connected valve clockwise 180 degree (fig. 3-2);
- c) Turn the adjustment knob clockwise to tighten the knob (fig. 3-3)

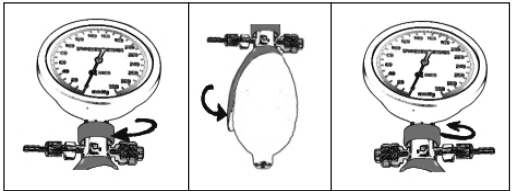
Fig. 3-1

Fig. 3-2

Fig. 3-3

IMPORTANT:

- 1. You must operate the instrument according to above instructions to avoid air-leakage.

2. If the adjustment knob is too tight, you may loosen it together with the handle.

Cuff sizes with one tube available for all models:

cm.22x4,5 art.080.117.00	Infant
cm.40x5 art.080.118.00	Child
cm.50x10 art.080.119.00	Boy
cm.50x14 art.080.120.00	Adult
cm.60x17 art.080.123.00	Obese

How to care for the aneroid sphygmomanometer

Manometer and monoball

Cleaning:

Wipe off the manometer and bulb with a damp cloth.

Sterilisation is non necessary, since the parts do not come into direct contact with the patient's body.

Cuffs

Cleaning:

After removing the bladder, wipe the velcro covers with a damp cloth. Alternatively, these can be washed with soap and cold water like all the other cuffs. If you decide on the latter course, rinse the cuffs with clear water afterwards and let them air dry. Wipe the bladder and tubes with a damp cloth.

IMPORTANT!

Do not iron cuffs!

Never expose the cuffs and the monoball to intensive solar radiation!
Never touch the cuff or bladders with a sharp instrument, since this could cause damage!

Test of precision

Remove the tube from the manometer and hold the manometer in a vertical position. When the pointer stands still at 0 on the scale, the instrument has been adjusted properly. If the pointer is below or above 0, the instrument must be recalibrated. Send the instrument to your specialized dealer.

Maintenance

In accordance with the testing requirements set down in the Medical Product Law (MPG), a maintenance period of two years should be adhered to.

Technical Data

Minimal environmental conditions under which the error tolerance of ± 3 mm/Hg must be satisfied in accordance with the UNI-EN 1060/2 standard:

Measurement conditions:	10°C to 40°C at a relative air humidity of 85 % (non-condensing).
Storage conditions:	-20°C to 70°C at a relative air humidity of 85 % (non-condensing).
Versions:	Desk, wall, stand, and anaesthetic models.
Type of indication:	Round scale.
Scale graduation	Increments of 2 mm/Hg.
Range shown on scale:	0 to 300 mm/Hg.
Measurement range:	0 to 300 mm/Hg.
Pointer movement	No stop pin.
Tube adapters:	"1 or 2" or "1 and 2" depending on the particular model.
Pressure generation:	Bulb.
Pressure reduction:	Air-release valve that can be regulated.

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