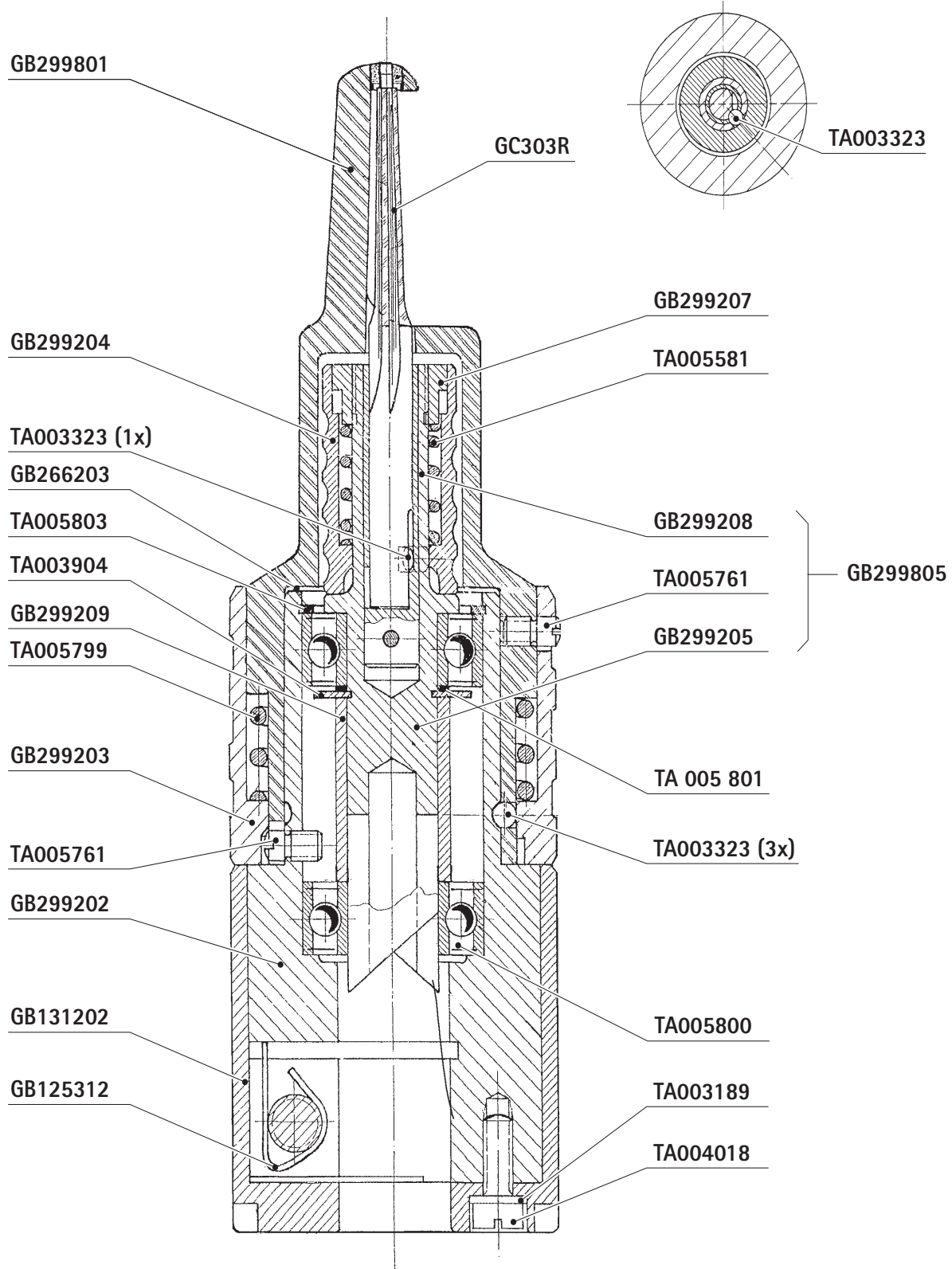
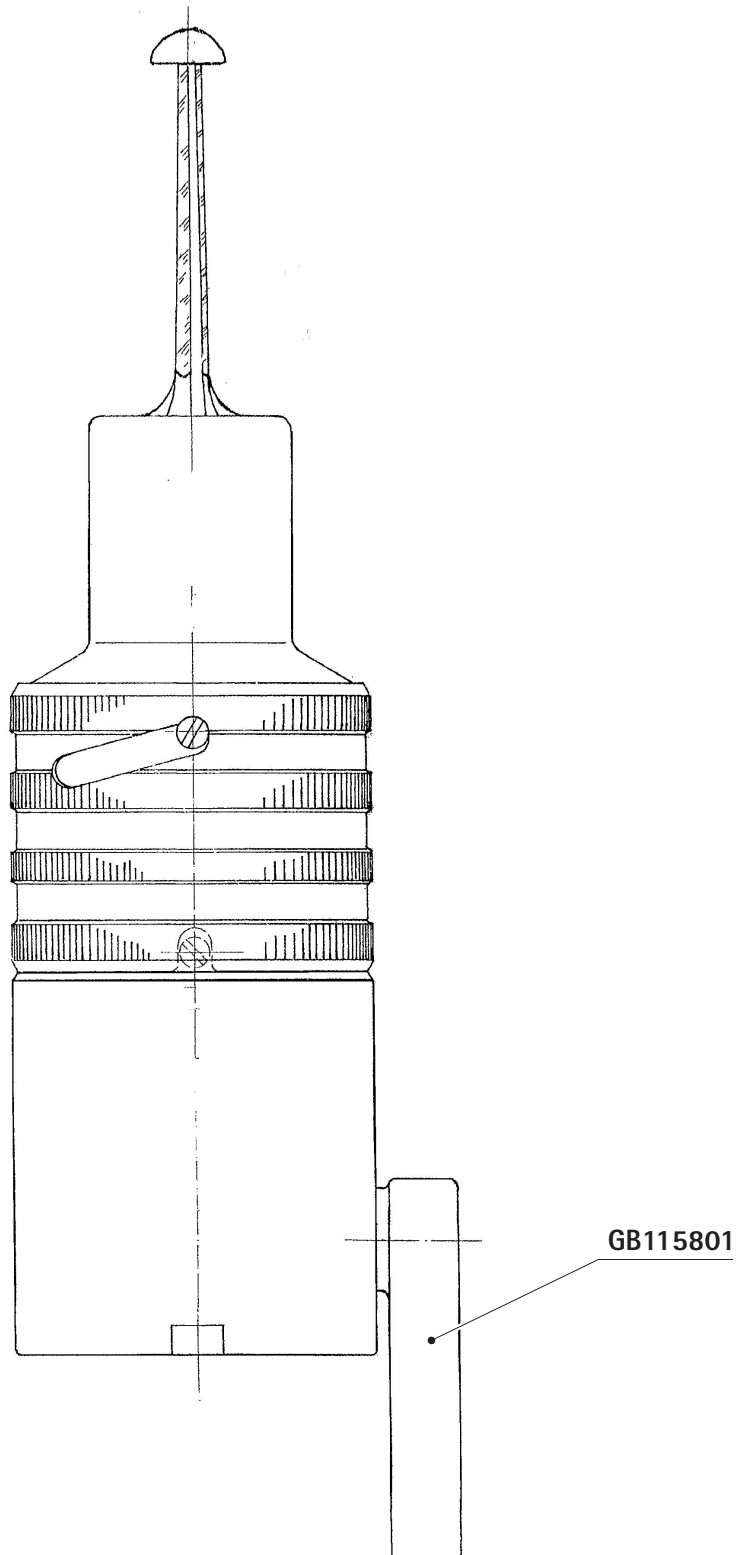
Aesculap Power Systems

-
- GB** Service Manual
Craniotome GB265R, GB268R and Macro Craniotome GB299
 - D** Service-Manual
Craniotom GB265R, GB268R und Makro-Craniotom GB299

Craniotome GB265R, GB268R and Macro Craniotome GB299

Copyright information

Copyright© 2006
 Aesculap AG & Co. KG®
 All rights reserved 08/06

This service information is copyrighted. This service information must not be copied or reproduced by other means either in whole or in part without the express permission of Aesculap AG & Co. KG.

This manual is intended for informational purposes only. Ownership of this manual alone does not constitute or imply authorization to service the product.

The technical information, illustrations and dimensions contained in this manual are non-binding. No claims may be made on the basis of the information contained herein. We reserve the right to make improvements without altering this documentation. We reserve the right to make technical changes without prior notice.

Main address for servicing:

Aesculap Technischer Service (ATS)
 Am Aesculap-Platz
 78532 Tuttlingen / Germany
 Phone: +49 7461 95-2700
 Fax: +49 7461 16-2887
 E-mail: ats@aesculap.de

Other service addresses can be obtained through the address indicated above.

Manufacturer's liability

We expressly point out that we can accept responsibility for any effects on the safety, reliability, and performance of our medical products, if and only if:

- any assembly, extensions, readjustments, modifications, or repairs are performed by technically experienced, knowledgeable and trained personnel and
- the medical products are used as set forth in the instructions for use.

It is possible to learn to service the products through an appropriate course of instruction given on the premises of Aesculap about the relevant medical products. To arrange for such a course, contact Aesculap Technical Service (ATS).

To ensure that your Aesculap warranty remains valid, we recommend that only spare parts that have been factory tested by Aesculap be used for repairs. Spare parts, as well as the relevant tools, can be ordered from Aesculap Technical Service (ATS). Any unauthorized opening and/or alterations of the medical product by third parties lead to the exclusion of our liability, as far as a fault is attributable to such unauthorized opening and/or alteration of the product. Aesculap cannot accept responsibility for the use of unsuitable spare parts, tools or devices. After repairs, dropping, severe damage or misuse, the product should be inspected by a qualified person.

This manual refers to the product as it was when the manual was prepared. Technical changes may be made at any time, particularly in software.

General advisory

This service manual contains illustrations and explanations. Each explanation covers the following: modes of operation, spare parts list, assembly, disassembly, functionality test, and the section tools/devices (if applicable).

For further information about a specific product (e.g. troubleshooting list, accessories) consult the relevant user instructions.

Literature	Art. no.
Instructions for use	TA005834

It is crucial that extreme cleanliness be maintained when the products described below are repaired. All products must undergo a functionality test after being repaired. If, in the course of a repair, a labeled component is replaced, the labeling must be transferred to the spare part. Either the original label should be used or the spare part bearing the relevant label should be ordered. To this end, be sure to include the applicable information when ordering.

Unless otherwise indicated, all medical products should be inspected annually.

General information about motors and handpieces

It is critically important that ball bearings be inspected annually. Even the slightest defect or soiling can lead to overheating during use, which would render the product unusable. If for the repair of, e.g., a motor a ball bearing has to be replaced, it is best in most cases to replace all other ball bearings as well.

If it is necessary to heat up adhesive joints during disassembly, a hot-air blower is the appropriate tool. The smallest possible nozzle should be used so as to avoid damaging other components. When gluing parts together, make certain that the surfaces to be joined are absolutely clean and free of grease. We recommend using Loctite Rapid Cleaner 7063 for this purpose (WS. no. 520001750). Only use adhesives listed in this manual. Follow the glue manufacturer's recommendations.

Lubricate components with the recommended Aesculap product only.

Following disassembly, clean all components thoroughly and inspect them to ensure that they are undamaged. Do not install any component of whose status you are uncertain. Products must always be test-run for several minutes after being repaired.

Contents

1. Safe handling.....	3
2. Tools, auxiliary materials, supplies	3
2.1 Tools.....	3
2.2 Auxiliary materials.....	3
2.3 Supplies	3
3. Expendable parts/Replacement parts.....	4
4. Designation of the craniotome elements	4
5. Basic preparations and maintenance.....	4
5.1 Preparations	4
Surface inspection.....	4
Checking the connections between elements.....	4
Function tests	4
5.2 Maintenance	4
6. Troubleshooting list.....	5
7. Disassembling	6
7.1 Disassembling the support foot	6
Disassembling support foot GB266	6
Disassembling support foot GB298	6
7.2 Disassembling the housing (GB267)	6
7.3 Disassembling the spindle (GB267)	6
7.4 Disassembling the housing (GB299)	6
7.5 Disassembling the spindle (GB299)	6
8. Assembling.....	10
8.1 Preliminary work	7
8.2 Pre-assembling the spindle (GB267)	7
8.3 Assembling the housing (GB267)	7
8.4 Pre-assembling the spindle (GB299)	7
8.5 Assembling the housing (GB299)	7
8.6 Assembling the support foot	7
Assembling support foot GB266.....	7
Assembling support foot GB298.....	7
8. Postmaintenance	11
9.1 Function tests	8
10. Parts list.....	8
10.1 GB265R	8
10.2 GB268R	9
10.3 GB299	9

1. Safe handling

- Operate craniotome GB265R, GB268R and GB299 with authentic Aesculap accessories only.
- To avoid damage to the product, do not knock craniotome GB265R, GB268R and GB299 against hard objects.

2. Tools, auxiliary materials, supplies

2.1 Tools

Designation	Art. no.
Slotted screwdriver	without
Clamping jaws	without

2.2 Auxiliary materials

Designation	Art. no.
Instructions for use	TA005834

2.3 Supplies

Designation	WS. no.
Special grease MI-setral-FKR 2	537001602
Loctite 222	560001813
Loctite 640	560001820
Eloxal Cleaner	JG601
Oil spray	GB149

Craniotome GB265R, GB268R and Macro Craniotome GB299

3. Expendable parts/Replacement parts

Designation	Art. no.
Radial ball bearing 8 x 16 x 6 ALSI (x2)	TA005800
Grooved pin DIN 1473 1.2 x 5 NR	TA005387
Locking washer	TA003904

4. Designation of the craniotome elements

Micro craniotomes

GB267	Handpiece only	Combined GB265R
GB266/ GB266M	Pendular support foot	

GB267	Handpiece only	Combined GB268R
GB298	Support foot (fixed)	

Macro craniotome

GB299	Handpiece (Macro) only, with fixed support foot GB298R
GB295	Pendular support foot craniotome compl. (screwed together), with pendular support foot GB266

5. Basic preparations and maintenance

5.1 Preparations

- Rethread every thread, using a taper.
- Clean all components, remove all silicone and Loctite residues, and degrease.
- Clean the surfaces of craniotome GB265R, GB268R and GB299 with Eloxal Cleaner JG601.
- Lubricate open radial ball bearing TA005800 with special grease MI-setral-FKR 2, WS.-no. 537001602.

Surface inspection

- Check surfaces for:
 - dents
 - stains
 - sharp edges
 - glue residues

Checking the connections between elements

- Connect flexible shaft GA176.
- Turn flexible shaft GA176 by hand. Check that the gear turns smoothly and without resistance.

Risk of injury while handling the cutter (sharp cutting edges)!

- Be careful when handling the cutter.

- Connect cutter GC303R.
- Check the locking and unlocking mechanism of cutter GC303R.
- Turn cutter GC303R by hand. Check that the cutter turns smoothly and without resistance.

Function tests

- Check that the support foot turns freely and without slack under spring pressure.
- Check craniotome GB265R, GB268R and GB299 for heat-up. After 3 minutes of running, the temperature must not exceed 41 °C.
- Check craniotome GB265R, GB268R and GB299 for atypical running noise, which would indicate defective bearings or couplings.
- Check all glued joints for firm seating.
- Spray through craniotome GB265R, GB268R and GB299 with oil spray GB149.

5.2 Maintenance

In order to ensure reliable operation, Aesculap recommends maintenance after every 300 reprocessing cycles or at least once a year.

6. Troubleshooting list

Malfunction	Finding	Cause	Remedy
Cutter not turning	Drive power not transferred to cutter	Driver broken	Replace driver TA005387
Support foot cannot be coupled	Cutter tip does not hit counter support	Support foot bent	Replace support foot Do not bend – risk of breakage!
	Marks on surface or borehole	Adapter surface damaged	Level the surfaces
Tool cannot be coupled	Visual check	Dirt in adapter	Clean the adapter
	Incorrect shape of cutter coupling	Incorrect cutter	Insert a correct cutter
Cutter not turning	Cutter cannot be turned by hand	Bearing blocked	Replace ball bearing TA003916
	No power transmission	Motor drive defective	Check the motor drive

Craniotome GB265R, GB268R and Macro Craniotome GB299

7. Disassembling

7.1 Disassembling the support foot

Disassembling support foot GB266/GB266M

- Heat up and remove headless screw TA005761.
- Remove sleeve GB299303.
- Remove compression spring TA005799 from housing GB266301.
- Remove 3 steel balls TA003323.
- Heat up threaded bushing GB266202 and remove it using a fitting piece of sheet metal.
- Slide off sliding rings GB266203 and, if applicable, shim rings TA008768 (GB266M).

This completes the disassembling of the support foot.

Disassembling support foot GB298

- Heat up and remove headless screw TA005761.
- Remove sleeve GB299303.
- Remove compression spring TA005799 from housing GB299801.
- Remove 3 steel balls TA003323.

This completes the disassembling of the support foot.

7.2 Disassembling the housing (GB267)

- Heat up housing GB267201 if necessary and unscrew it from sleeve GB267200.
- Unscrew headless screw TA005761 from housing GB267201.
- Remove borehole circlip TA005803 from housing GB267201.
- Remove spindle complete from housing GB267201.

This completes the disassembling of the housing.

7.3 Disassembling the spindle (GB267)

- Expel grooved pin TA005387 from spindle GB267800.
- Remove coupling piece TA005386 and coupling spring TA005385 from spindle GB267800.
- Remove locking ring TA003904 and shim ring TA005801 from the spindle.
- Press off 2 radial ball bearings TA011543 from spindle GB267800.
- Remove spacer tube GB299209.
- Unscrew slotted nut GB299207, using the slotted screwdriver.
- Remove compression spring TA005581.
- Remove clamping sleeve GB299204.
- Remove steel ball TA003323.

This completes the disassembling of the spindle.

7.4 Disassembling the housing (GB299)

- Remove borehole circlip TA005803.
- Unscrew headless screw TA005761 from housing GB299202.
- Remove spindle complete from housing GB299202.
- Unscrew clamping screw GB115801 from housing GB299202.
- Unscrew screw TA004018 from housing GB299202.
- Slide off sleeve GB131202 from housing GB299202.
- Remove spring washer TA003189.
- Remove spring clamp GB125312 from housing GB299202.

This completes the disassembling of the housing.

7.5 Disassembling the spindle (GB299)

- Remove locking ring TA003904.
- Press off first radial ball bearing TA005800, spacer tube GB299209 and shim ring(s) TA005801 from spindle GB299805.
- Remove spacer tube GB299209.
- Remove shim ring(s) TA005801.
- Press off second radial ball bearing TA005800 from spindle GB299805.
- Clamp in spindle GB299805 between clamping jaws.
- Unscrew slotted nut GB299207 from spindle GB299805, using the slotted screwdriver.
- Remove compression spring TA005581.
- Remove clamping sleeve GB299204.
- Remove steel ball TA003323 from spindle GB299805.

This completes the disassembling of the spindle.

8. Assembling

8.1 Preparations

- Rethread every thread, using a taper.
- Clean all components, remove Loctite residues, and degrease.

8.2 Pre-assembling the spindle (GB267)

- Insert steel ball TA003323.
- Install clamping sleeve GB299204.
- Install compression spring TA005581.
- Apply Loctite 640 on slotted nut GB299207 and screw in the nut, using the slotted screwdriver.
- Couple a tool and tighten slotted nut GB299207 so that the tool can still be removed.
- Press first radial ball bearing TA005800 onto spindle GB299805.
- Install spacer tube GB299209.
- Install locking ring TA003904.
- Install shim ring(s) TA005801 until the axial slack is minimal.
- Press second radial ball bearing TA005800 onto spindle GB299805.
- Install coupling piece TA005386 and coupling spring TA005385 on spindle GB267800.
- Impel grooved pin TA005387 into spindle GB267800.
- Check coupling piece TA005386 for smooth movement.
This completes the pre-assembling of the spindle.

8.3 Assembling the housing (GB267)

- Slide spindle complete into housing GB267201 and secure it with borehole circlip TA005803.
- Check for axial slack and, if necessary, install additional shim rings TA005801.
- Apply Loctite 640 on housing GB267201 and screw it into sleeve GB267200.
This completes the assembling of the housing.

8.4 Pre-assembling the spindle (GB299)

- Install steel ball TA003323 in spindle GB299805.
- Position clamping sleeve GB299204 on spindle GB299805.
- Install compression spring TA005581 on spindle GB299805.
- Apply Loctite 640 on slotted nut GB299207 and screw the nut into spindle GB299805, using the slotted screwdriver.
- Couple a tool and tighten slotted nut GB299207 so that the tool can still be removed.
- Press first radial ball bearing TA005800 onto spindle GB299805.
- Install shim ring TA005801.
- Install locking ring TA003904.
- Install spacer tube GB299209.
- Press second radial ball bearing TA005800 onto spindle GB299805.
This completes the pre-assembling of the spindle.

8.5 Assembling the housing (GB299)

- Press spindle complete into housing GB299202.
- Check for axial slack, install additional shim rings TA005801, if necessary, and secure the spindle with borehole circlip TA005803.
- Install spring clamp GB125312 in housing GB299202.
- Install sleeve GB131202 on housing GB299202.
- Screw in screw TA004018 with spring washer TA003189 in to housing GB299202.
- Screw in clamping screw GB115801 in to housing GB299202.
- Apply Loctite 222 on headless screw TA005761 and tighten the screw in housing GB267201.
This completes the assembling of the housing.

8.6 Assembling the support foot

Assembling support foot GB266

- Insert 3 steel balls TA003323.
- Insert compression spring TA005799 and sleeve GB299303 in housing GB299801.
- Apply Loctite 222 on headless screw TA005761 and install the screw.
- Slide sliding rings GB266203 on threaded bushing GB266202.
- Insert threaded bushing GB266202 in housing GB266201.
- Apply Loctite 640 on Pendular foot complete GB266800 and install it on threaded bushing GB266202 down to the bedstop.
- Turn back Pendular foot complete GB266800 by 1/4 turn.
Pendular foot complete GB266800 does not have any slack and can be turned easily.
This completes the assembling of the support foot.

Craniotome GB265R, GB268R and Macro Craniotome GB299

Assembling support foot GB266M

- Insert 3 steel balls TA003323.
 - Insert compression spring TA005799 and sleeve GB299303 in housing GB299801.
 - Apply Loctite 222 on headless screw TA005761 and install the screw.
 - Slide sliding rings GB266203 and shim rings TA008768 on threaded bushing GB266302.
 - Screw on support foot GB266810 and screw threaded bushing GB266302 into support foot sleeve GB266301.
 - Choose the number of sliding rings GB266203 and shim rings TA008768 in order to achieve minimal axial slack.
 - Apply Loctite 640 on threaded bushing GB266302 and screw the bushing into support foot sleeve GB266301.
- This completes the assembling of the support foot.

Assembling support foot GB298

- Insert 3 steel balls TA003323.
 - Insert compression spring TA005799 and sleeve GB299303 in housing GB299801.
 - Apply Loctite 222 on headless screw TA005761 and install the screw.
- This completes the assembling of the support foot.

9. Postmaintenance

9.1 Function tests

- Function checks, see Function tests.

10. Parts list

10.1 GB265R

Art. no.	Designation
GB266R	Turnable Dura guard for GB265R and GB299
GB267R	Craniotome Handpiece only, for GB265 and GB268
GB266201	Support foot sleeve
GB266202	Threaded bushing
GB266203	Sliding ring
GB266301	Support foot sleeve
GB266302	Threaded bushing
GB266800	Pendular foot complete
GB266810	Support foot
GB267200	Sleeve
GB267201	Housing
GB267203	Spindle
GB267800	Spindle complete
GB299203	Locking sleeve
GB299204	Clamping sleeve
GB299206	Bearing bush Z-45767
GB299207	Slottednut, Special M 6.5 x 0.50
GB299208	Adapter sleeve
GB299209	Spacer tube
GC303R	Bone cutter for Craniotome GB299
TA003323	Steel ball, 2 mm NR (3 x)
TA003904	Locking washer
TA005385	Couplingspring for GB130
TA005386	Coupling piece
TA005387	Grooved pin DIN 1473 1.2 x 5 NR
TA005581	Compression spring
TA005761	Headless screw DIN 427 M 2.5 x 4.3
TA005799	Compression spring
TA005800	Radial ball bearing 8 x 16 x 6 ALSI (x2)
TA005801	Shimring 8.0 x 10 x 0.2
TA005803	Borehole circlip
TA007960	Parallel pin
TA008768	Shimring 16 x 19 x 0.15

10.2 GB268R

Art. no.	Designation
GB266R	Turnable Dura guard for GB265R and GB299
GB298R	Dura guard sleeve for Craniotome GB299
GB266201	Support foot sleeve
GB266202	Threaded bushing
GB266203	Sliding ring
GB266800	Pendular foot complete
GB266810	Support foot
GB267200	Sleeve
GB267203	Spindle
GB267800	Spindle complete
GB299203	Locking sleeve
GB299204	Clamping sleeve
GB299206	Bearing bush Z-45767
GB299207	Slottednut, Special M 6.5 x 0.50
GB299208	Adapter sleeve
GB299209	Spacer tube
GB299303	Locking sleeve
GB299801	Support foot sleeve, assembled
GC303R	Bone cutter for Craniotome GB299
TA003323	Steel ball, 2 mm NR (3 x)
TA003904	Locking washer
TA005385	Couplingspring for GB130
TA005386	Coupling piece
TA005387	Grooved pin DIN 1473 1.2 x 5 NR
TA005581	Compression spring
TA005761	Headless screw DIN 427 M 2.5 x 4.3
TA005799	Compression spring
TA005800	Radial ball bearing 8 x 16 x 6 ALSI (x2)
TA005801	Shimring 8.0 x 10 x 0.2
TA005803	Borehole circlip
TA007960	Parallel pin

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