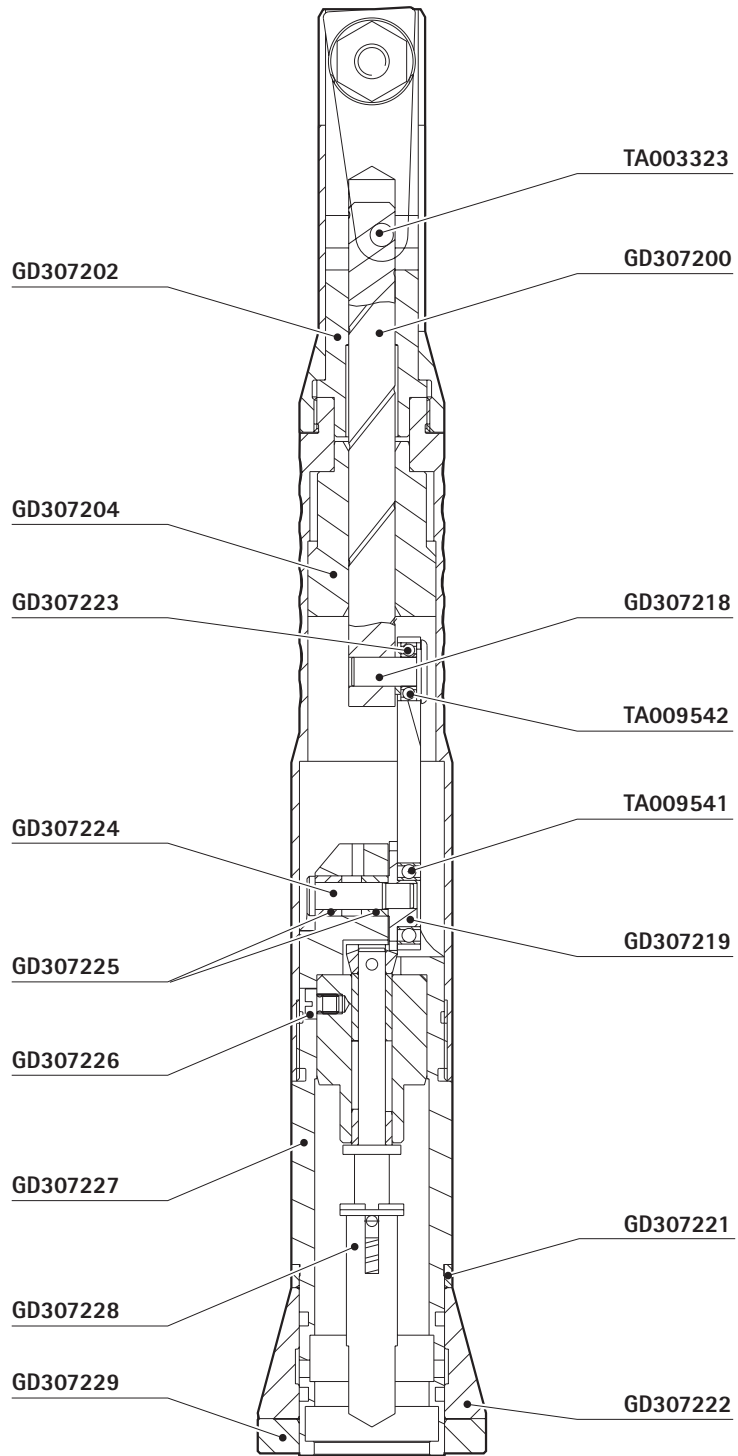
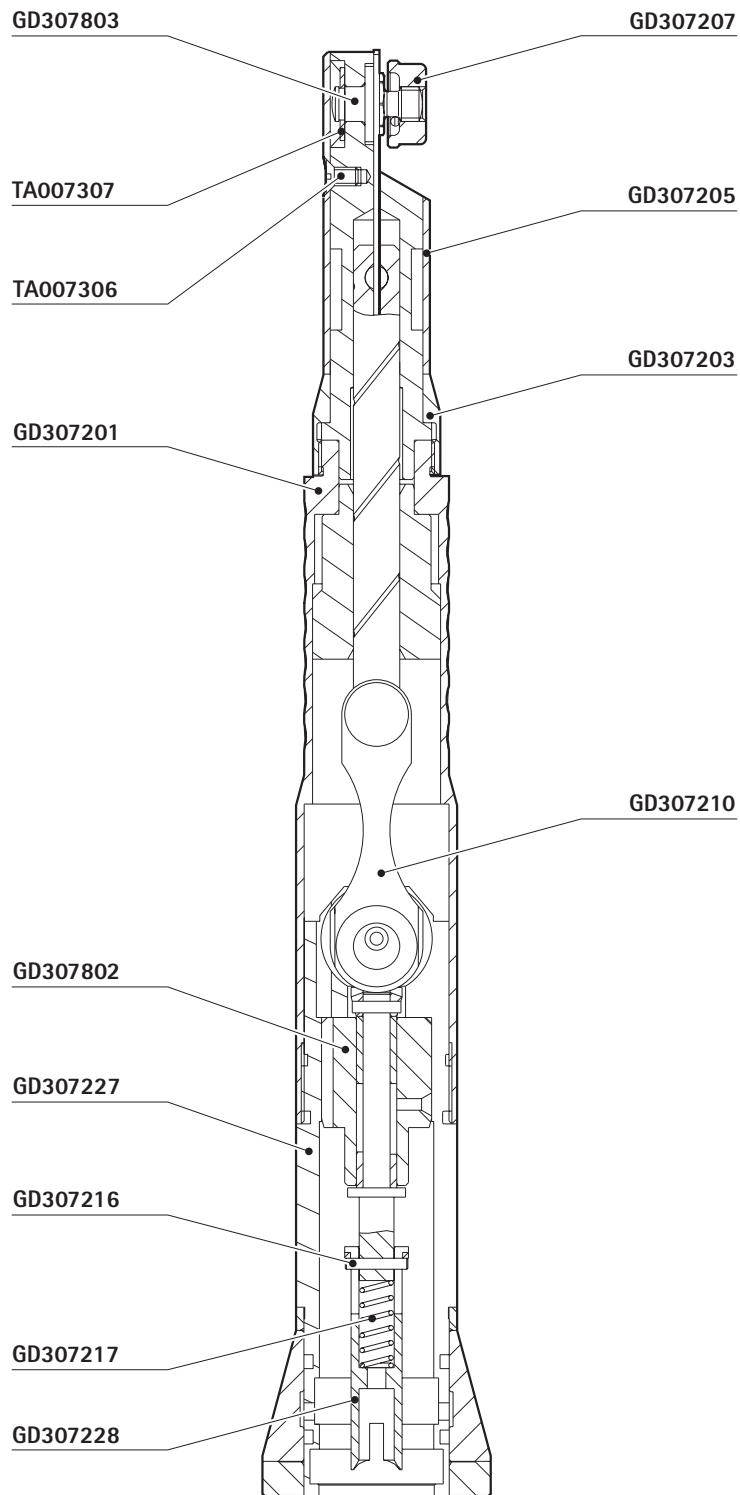
Aesculap Power Systems

-
- GB** **Service Manual**
micro-Line sagittal saw GD307
 - D** **Service-Manual**
micro-Line Sagittalsäge GD307

micro-Line sagittal saw GD307

Copyright information

Copyright© 2006

Aesculap AG & Co. KG®

All rights reserved 11/06

This service information is copyrighted. This service information must not be copied or reproduced by any means either in whole or in part without the express permission of Aesculap AG & CO. KG.

This manual is intended for informational purposes only. Ownership of this manual alone does not constitute or imply authorization to service the product.

The technical information, illustrations and dimensions contained in this manual are non-binding. No claims may be made on the basis of the information contained herein. We reserve the right to make improvements without altering this documentation. We reserve the right to make technical changes without prior notice.

Main address for servicing:

Aesculap Technischer Service (ATS)

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78532 Tuttlingen / Germany

Phone: +49 7461 95-2700

Fax: +49 7461 16-2887

E-mail: ats@aesculap.de

Other service addresses can be obtained through the address indicated above.

Manufacturer's liability

We expressly point out that we can accept responsibility for any effects on the safety, reliability, and performance of our medical products, if and only if:

- any assembly, extensions, readjustments, modifications, or repairs are performed by technically experienced, knowledgeable and trained personnel and
- the medical products are used as set forth in the instructions for use.

It is possible to learn servicing the respective medical products through appropriate courses of instruction offered by Aesculap. To arrange for such a course, please contact Aesculap Technical Service (ATS).

To ensure that your Aesculap warranty remains valid, we recommend using for repairs only Aesculap factory-tested spare parts. Spare parts, as well as the relevant tools, can be ordered from Aesculap Technical Service (ATS). Any unauthorized opening and/or alterations of the medical product by third parties lead to the exclusion of our liability, as far as a fault is attributable to such unauthorized opening and/or alteration of the product. Aesculap cannot accept responsibility for the use of unsuitable spare parts, tools or devices. After repairs, dropping, severe damage or misuse, the product should be inspected by a qualified person.

This manual refers to the product as it was when the manual was prepared. Technical changes, particularly of software, may be made at any time.

General advisory

This service manual contains illustrations and explanations. Each explanation covers the following: modes of operation, parts list, assembly, disassembly, functionality test, and a tools/devices section (if applicable).

For further information about a specific product (e.g., troubleshooting list, accessories) please refer to the relevant user instructions.

Documentation	Art. no.
Instructions for use	TA009539

It is crucial to maintain extreme cleanliness during repairs of the products described below. All products must undergo thorough function tests after any repairs. If, in the course of a repair, a labeled component is replaced, the labeling must be transferred to the spare part. Either the original label should be used or the spare part bearing the relevant label should be ordered. Any such order should include the relevant information.

Unless otherwise indicated, all medical products should be inspected annually.

General information about motors and handpieces

It is critically important that ball bearings be inspected annually. Even the slightest defect or dirt can lead to overheating during use, which would render the product unusable. If for the repair of e.g. a motor a ball bearing has to be replaced, it is best in most cases to also replace all other ball bearings.

If it is necessary to heat up adhesive joints during disassembly, a hot-air heat gun is the appropriate tool. The smallest possible nozzle should be used so as to avoid damaging other components. For glue joints, make certain that the surfaces to be joined are absolutely clean and free of grease. We recommend using Loctite Rapid Cleaner 7063 for this purpose (WS. no. 520001750). Only use adhesives listed in this manual. Follow the glue manufacturer's recommendations.

Lubricate components with the recommended Aesculap product only.

Following disassembly, clean all components thoroughly and inspect them for any damage. Do not install any component of uncertain condition. Products must always be test-run for several minutes after any repair.

Contents

1. Safe handling	3
2. Tools, auxiliary materials, supplies	3
2.1 Tools	3
2.2 Auxiliary materials	3
2.3 Supplies	3
3. Expendable parts/Spare parts	3
4. Basic preparations and maintenance	4
4.1 Function tests	4
4.2 Maintenance	4
5. Troubleshooting list	5
6. Disassembling	6
6.1 Disassembling the handle part	6
6.2 Disassembling the rear housing	6
6.3 Disassembling the drive unit complete	6
7. Assembling	6
7.1 Preparations	6
7.2 Assembling the drive unit complete	6
7.3 Assembling the rear housing	6
7.4 Assembling the handle part	7
8. Postmaintenance	7
8.1 Function tests	7
8.2 Surface inspection	7
9. Parts list	7

1. Safe handling

- Operate the micro-Line sagittal saw GD307 only with authentic Aesculap accessories.

2. Tools, auxiliary materials, supplies

2.1 Tools

Designation	Art. no.
Special device (disassembly)	W-90000167
Special device (assembly)	W-90000168
Special device (disassembly)	W-90000169

2.2 Auxiliary materials

Designation	Art. no.
Instructions for use	TA009539

2.3 Supplies

Designation	WS. no.
Loctite 572	560001817
Lubricant	GB600
Uhu Plus "Schnellfest"	560001162-N

3. Expendable parts/Spare parts

Designation	Art. no.
Fastening nut	GD307207
Connecting rod	GD307210
Shim	GD307223
Steel ball	TA003323
Countersunk screw DIN 936 M 1.6 x 3.0	TA007306
Retaining ring DIN 471 3.0 x 0.4	TA007307
Radial ball bearing 4 x 7 x 2 NR	TA009541
Radial ball bearing 2.5 x 5 x 1.5 NR	TA009542

4. Basic preparations and maintenance

4.1 Function tests

- Clamp in the saw blade and test the micro-Line sagittal saw with a suitable drive unit (e.g., flexible shaft).
- Cautiously saw into a block of wood.
- Check the micro-Line sagittal saw for atypical running noises that might be a sign of defective bearings, gear wheels or connections.
- Check the micro-Line sagittal saw for excessive heat-up.
The temperature must not exceed 41 °C after 30 seconds of running.

4.2 Maintenance

To ensure reliable operation, Aesculap recommends servicing the product every 2 years (depending on frequency of use).

5. Troubleshooting list

Malfunction	Finding	Cause	Remedy
Loud noise	Saw handpiece heating up	Worn gearing	Replace defective gearing components
	Excessive slack at the saw blade adapter	—	Replace rocker complete GD307803
Saw blade not moving	Coupling not engaged; broken coupling components	Coupling defective	Replace pin GD307216
	Rocker lever not moving	Steel ball jammed	Clean or lubricate steel ball 2 mm NR TA003323
	Motor running, but not driving the saw	Gearing defective	Replace drive unit complete GD307802
Insufficient cutting power of saw blades	Saw blade worn	Saw blade blunt	Insert new saw blade
	Saw blade heating up	Inadequate chip removal	Move micro-Line sagittal saw back and forth when working with it
Lack of power of saw handpiece	Loud noise	Worn saw handpiece	Replace defective gearing components
	Saw handpiece heating up	Friction	Spray through saw handpiece with GB600 lubricant

micro-Line sagittal saw GD307

6. Disassembling

6.1 Disassembling the handle part

- Unscrew fastening nut GB307207.
 - Unscrew and remove countersunk screw TA007306.
 - Set rocker complete GD307803 to central position.
 - Remove sleeve GD307205.
 - Remove retaining ring TA007307.
 - Unscrew locknut GD307203. If necessary, warm up the locknut prior to removal.
 - Laterally turn rocker complete GD307803 out of its guide.
 - Remove rocker complete GD307803, observing the ball component.
 - Remove sheath GD307202.
 - Unscrew handle part GD307201 from rear housing GD307227. Warm up components, if necessary.
- This completes disassembling of the handle part.

6.2 Disassembling the rear housing

- Pull connecting rod GD307210 with plunger GD307200 from gear wheel GD307219.
 - Press connecting shaft GD307218 from connecting rod GD307210 or plunger GD307200, respectively.
 - Remove shim GD307223.
 - Pull radial ball bearing TA009542 off from connecting shaft GB307218.
 - Remove radial ball bearing TA009541 from connecting rod GD307210.
 - Unscrew screw GD307226.
Drive unit complete drops out of the rear housing.
 - Press out fastening shaft GD307224 from gear wheel GD307219, using special device W-90000167 and special device W-90000169.
 - Press out 2 bearings GD307225 from the rear housing.
- This completes disassembling of the rear housing.

6.3 Disassembling the drive unit complete

- Expel pin GD307216 from drive unit complete GD307802.
 - Take off coupling piece GD307228 and compression spring GD307217.
- This completes disassembling of the drive unit complete.

7. Assembling

7.1 Preparations

- Clean all components, remove Loctite residues, and degrease.

7.2 Assembling the drive unit complete

- Insert compression spring GD307217 in coupling piece GD307228.
 - Mount coupling piece GD307228 on drive unit complete GD307802 and secure with pin GD307216.
- This completes assembling of the drive unit complete.

7.3 Assembling the rear housing

- Press 2 bearings GD307225 into rear housing GD307227.
 - Insert fastening shaft GD307224 through the rear housing and press it onto gear wheel GD307219, using special device W-90000168.
 - Slide the drive unit complete in its correct position into the rear housing and fasten it with special screw GD307226.
 - Mount radial ball bearing TA009541 and radial ball bearing TA009542 on connecting rod GD307210.
 - Slide in connecting shaft GD307218 from the chamfer side of connecting rod GD307210.
 - Install shim GD307223.
 - Press plunger GD307200 onto the connecting shaft. Note the slanted drill hole for steel ball TA003323 in rocker complete GD307803.
 - Check for smooth running of the assembly.
 - Press connecting rod GD307210 complete onto gear wheel GD307219.
 - Check for smooth running of the assembly.
- This completes assembling of the rear housing.

7.4 Assembling the handle part

- Insert from the rear the plunger with rear housing GD307227 into handle part GD307201.
- Apply Loctite 572 on handle part GD307201 and tighten the screws connecting the handle part to rear housing GD307227.
- To tack sheath GD307202, apply Uhu Plus "Schnellfest" on connecting rod GD307210.
- Mount sheath GD307202 in its correct position on the plunger.
- Slide rocker complete GD307803 into the slot (with steel ball TA003323 inserted), mount it on sheath GD307202 and swivel it in.
- Apply Loctite 572 on locknut GD307203 and screw locknut onto handle part GD307201.
- Mount retaining ring TA007307.
- Mount sleeve GD307205.
- Screw in countersunk screw TA007306.
- Screw on fastening nut GD307207, secure with wire WS-111870080 (length: 7.5 mm) and spot-weld with a laser beam.
- Check for smooth running of the assembly.
This completes assembling of the handle part.

8. Postmaintenance

8.1 Function tests

- Clamp in the saw blade and test the micro-Line saggital saw with a suitable drive unit (e.g., flexible shaft).
- Cautiously saw into a block of wood.
- Check the micro-Line saggital saw for atypical running noises which would be a sign of defective bearings, gear wheels or connections.
- Check the micro-Line saggital saw for heat-up.
The temperature must not exceed 41 °C after 30 seconds of running.

8.2 Surface inspection

- Check the surface of the micro-Line saggital saw for sharp-edged damage and residues.

9. Parts list

Art. no.	Designation
GD307200	Plunger
GD307201	Handle part
GD307202	Sheath
GD307203	Locknut
GD307205	Sleeve
GD307207	Fastening nut
GD307210	Connecting rod
GD307211	Bearing
GD307212	Bearing
GD307213	Beveled pinion
GD307215	Spindle
GD307216	Pin
GD307217	Compression spring
GD307218	Connecting shaft
GD307219	Gear wheel
GD307221	Colored ring green
GD307222	Cone
GD307223	Shim
GD307224	Fastening shaft
GD307225	Bearing (x2)
GD307226	Special screw
GD307227	Rear housing
GD307228	Coupling piece
GD307229	Ring, end
GD307802	Drive unit, complete
GD307803	Rocker, complete
TA003323	Steel ball 2 mm NR
TA006311	Spiral pin DIN 7343 1.0 x 5 1430
TA007306	Countersunk screw DIN 936 M 1.6 x 3.0
TA007307	Retaining ring DIN 471 3.0 x 0.4
TA009541	Radial ball bearing 4 x 7 x 2 NR
TA009542	Radial ball bearing 2.5 x 5 x 1.5 NR

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