

Pro Focus Ultrasound System



LEGAL MANUFACTURER

BK MEDICAL

Mileparken 34

2730 Herlev

Denmark

Tel.:+45 4452 8100 / Fax:+45 4452 8199

www.bkultrasound.com

Email: info@bkultrasound.com

If you have comments about the user documentation, please write to us at the email address above. We would like to hear from you.

BK Medical Customer Satisfaction

Input from our customers helps us improve our products and services. As part of our customer satisfaction program, we contact a sample of our customers a few months after they receive their orders. If you receive an email message from us asking for your feedback, we hope you will be willing to answer some questions about your experience buying and using our products. Your opinions are important to us. You are of course always welcome to contact us via your BK Medical representative or by contacting us directly.

Scanner Software

- **NOT FAULT TOLERANT.** THE SOFTWARE IS NOT FAULT TOLERANT. BK Medical HAS INDEPENDENTLY DETERMINED HOW TO USE THE SOFTWARE IN THE DEVICE, AND MS HAS RELIED UPON BK Medical TO CONDUCT SUFFICIENT TESTING TO DETERMINE THAT THE SOFTWARE IS SUITABLE FOR USE.
- **EXPORT RESTRICTIONS.** You acknowledge that Windows XP Embedded is of US-origin. You agree to comply with all applicable international and national laws that apply to Windows XP Embedded, including the U.S. Export Administration Regulations, as well as end-user, end-use and country destination restrictions issued by U.S. and other governments. For additional information on exporting Windows XP Embedded, see <http://www.microsoft.com/exporting/>
- The Pro Focus 2202 Ultrasound Scanner is closed. Any modification of or installation of software to the system may compromise safety and function of the system. Any modification of or installation of software without written permission from B-K Medical will immediately void any warranty supplied by BK Medical. Such changes will also void any service contract and result in charges to the customer for restoration of the original Pro Focus 2202 system.

Trademarks:

DICOM is the registered trademark of the National Electrical Manufacturers Association for its standards publications relating to digital communications of medical information.

Microsoft and Windows are registered trademarks of Microsoft Corporation in the United States and other countries.

SonoVue is a registered trademark of Bracco S.p.A.

DEFINITY is a registered trademark of Bristol-Myers Squibb Medical Imaging

FireWire is a trademark of Apple Computer, Inc.

CIV-Flex is a trademark of CIVCO Medical Instruments Co., Inc.

Contents

Chapter 1	General and Safety Information	7
	User Documentation	7
	Safety Information	8
	Typographical Conventions	8
	Terminology	8
	About the System	9
	Imaging Modes	9
	Pro Packages	9
	Safety Symbols on the System	10
	General Safety Precautions	12
	Explosion Hazards	13
	Electrical Safety	13
	ESD Training	13
	Interference	14
	Installation	15
	Transducers	15
	Remote Control	16
	Other Safety Considerations	16
	Service and Repair	19
	Connecting Other Equipment	19
	Network Connection	20
	Network Security	21
	Network Printing	21
	Computer Security	21
	Wireless Networks	21
	Acoustic Output	24
	General	24
	Monitor Display	25
	Thermal and Mechanical Indices	25
	Acoustic Output Measurement	26
	Functions Affecting Acoustic Output	26
	Default Acoustic Output	27
	Clinical Measurements: Ranges and Accuracies	27
	Geometric Measurements	29
	Time Measurements	29
	Doppler Measurements	30
	References	30
Chapter 2	Getting Started	33
	General Information	33
	Adjusting the Keyboard Panel and Monitor	34
	Docking the Flat Screen on Arm	34
	Turning the System On and Off	35
	System Busy	35
	Checking the Date	35

Settings on the Monitor	36
Brightness	36
Factory Defaults	37
Menu Window on LCD Flat Screen	37
Adjusting the Reading Lights	37
Setting up a New Patient	37
Opening the Patient Window	38
Changing Pro Package or Preset in the Patient Window	38
New Patient Information from a DICOM Worklist	39
Entering Dates	41
Keeping Image Settings	41
Keeping Patient-Related Data	42
Pausing and Later Continuing an Examination	42
Discontinue Examination with MPPS Server Configured	43
Ending an Examination	43
Connecting and Selecting Transducers	43
Connecting a Transducer	43
Selecting a Transducer	44
Selecting the Imaging Plane	45
Pro Packages and Presets	45
Selecting a Pro Package or Preset	46
Saving a Preset	46
Selecting or Changing the Imaging Mode	47
Performing a Biopsy or Puncture Procedure (including Brachytherapy)	47
Remote Control	50
Cleaning and Disinfection	50

Chapter 3 The User Interface 51

Keyboard Panel Controls	51
The Control Panel	52
The Trackball and Trackball Keys	53
Basic Keys	55
B-Mode Keys	56
B-Mode Adjustment Keys	56
Power and Color Mode Keys	58
Doppler and M-Mode Keys	58
Doppler Adjustment Keys	59
User-Defined Keys	60
Image Annotation Keys	61
Miscellaneous Keys	62
TGC Sliders	63
Keyboard	63
Controls on the Monitor	64
Point and Click	65
Foot Switch	66
Remote Control	66
The Remote Control and Its Functions	67
Pairing the Remote Control with the System	67
Sleep	68

	Mouse Function	68
	Replacing Batteries	68
	Cleaning and Disinfection	68
	Transducers	69
Chapter 4	Technical Guide	71
	Operating Environment	71
	The Rear of the System	71
	Cables Types and Lengths	73
	Monitors	74
	Audio Input and Output	74
	Documentation Accessories	75
	Foot-Switch Connector	76
	Electrical Connections	76
	Power Supply Cord	76
	Connecting Other Equipment	77
	Dismantling the System for Transportation	79
	Keyboard Base and Panel	79
	Monitor	80
	Monitor Base	80
	HistoScanning	81
	Using the System with a Lithotripter	82
	References	82
Appendix 1	Specifications and Indications for Use	83
	Indications for Use	83
	Contraindications	83
Appendix A	Glossary	87

Chapter 1

General and Safety Information

This guide is an abridged version of the Extended User Guide. It contains basic information about the Pro Focus 2202 Ultrasound System (in the following referred to as the Pro Focus) from BK Medical:

- safety information
- turning on the system and getting started
- the user interface
- technical guide – what to connect where

Intended use The system is a 2D and 3D ultrasound echo and flow imaging system for diagnosis, data processing and transfer, and guidance of puncture and biopsy.

NOTE: *Some of the functionality and options described in this guide may not be available with your version of the system.*

User Documentation

Before using the system, please make yourself familiar with the information in the accompanying user information documents. Some documents are printed, and you will find the remaining documents on the accompanying *User Documentation CD (BB1984)*. Make sure that you also read the transducer user guide and specifications for each transducer that you use.

Document	Information
System User Guide	Introductory information, safety information, getting started.
Getting Started	User interface, basic operating instructions.
System Extended User Guide	Information about advanced functions, glossary.
Product Data for system	Specifications for the system, including disinfection methods that can be used. Indications for use for each transducer that can be used with the system.
Technical Data (BZ2100)	Acoustic output data, clinical measurements (ranges and accuracies), factory default power levels and data about EMC (electromagnetic compatibility) for all transducers. Pro Package calculation formulas.
Care, Cleaning and Safety	Cleaning, disinfection, sterilization, checking, storing and disposing of BK Medical equipment. Includes environmental limits.
Transducer User Guide	Specific instructions for the transducer and puncture attachments
Product Data for each transducer	Specifications for the transducer, including disinfection methods that can be used.

Table 1-1. User information documentation that accompanies the equipment.

Improper use Failure to follow safety instructions or use for purposes other than those described in the user guides constitutes improper use.

Safety Information

This user guide contains cautions, warnings and other information about what you must do to ensure the safe and proper performance of the ultrasound system. You must also follow local government rules and guidelines at all times.

Important safety information is indicated in the user guide by means of special formatting.



WARNING Warnings contain information that is important for avoiding personal injury.



Caution: Cautions contain information and instructions that must be followed to avoid damaging equipment, data or software.

NOTE: *Notes contain information that you should be aware of.*

Typographical Conventions

The following typographical conventions are used in this manual:

FREEZE – a function, something you can do by using the keyboard panel or the graphical user interface on the monitor

Freq. – text that can be selected or clicked on the monitor

[Power] – a key on the keyboard panel

Book Titles – names of books and scientific journals

[1] References. Literature references are found at the end of the chapter.

Terminology

Instructions

You control the system by using the graphical user interface on the monitor, pressing keys on the control panel, or typing on the keyboard. Instructions in this user guide tell you what sort of action to take.



When the instructions say	It means
Click Power	Use the trackball to point at the word "Power" on the monitor, and then press [] (Select) on the control panel.
Press [Power]	Press the Power key on the control panel.
Press [Sound ]	Press the Sound key on the control panel.
Type Power	Type the word "Power" on the keyboard.

Table 1-2. Click, Press and Type.

About the System

The system can be used for continuous operation, but imaging duration for individual patients must not exceed 60 minutes. We recommend, however, that you turn off the system at the end of each workday.

The Pro Focus 2202 Ultrasound System is easy to use. You can perform most system operations using just the trackball and the trackball keys to interact with controls on the monitor. Dedicated keys on the control panel make other operations quick and easy.

Imaging Modes

The Pro Focus lets you use various imaging modes:

- B-mode (brightness mode) for real-time imaging of soft tissues: includes harmonic imaging and IQPAC™ with Enhanced Tissue Definition (ETD) and Angular Compound Imaging (ACI)
- Color mode (CFM, color flow mapping, color Doppler mode) for information about flow direction and velocity
- Power mode (power Doppler mode) for higher sensitivity to flow information: includes directional Power Doppler
- Doppler mode (spectral Doppler mode) for information about the spectrum of flow velocities as a function of time
- CW Doppler mode (continuous wave Doppler) for information about high-velocity flow
- M-mode for information about tissue motion as a function of time

NOTE: *Not all imaging modes may be available with your version of the system.*

Pro Packages

Pro Packages contain default presets and calculation formulas that make it quick and convenient to use the system for specific applications.

Safety Symbols on the System

Important safety information is indicated in the user guide and on the system itself by means of special symbols and formatting.
















Table 1-3 contains brief explanations of the symbols used to label the system. (Some labels in the table may appear on the transducer rather than the system itself.)

NOTE *especially the following:*



When you encounter this sign on the system, consult the user guide for important safety information.

BK Medical disclaims all responsibility for the operating safety, reliability and performance of the equipment if these symbols and warnings are disregarded in any way.

Symbol	Name	Description
	Caution or Warning	Consult accompanying user guide (BB1279) when you encounter this sign on the instrument, to avoid reducing its safety.
	Consult instructions for use	Consult user guide or other instructions.
	Manufacturer	Legal manufacturer.
	UL Classification for Canada and US	UL approval of medical equipment with respect to electrical shock, fire and mechanical hazards.
	Potential Equalization	Terminal connected to the chassis. Should be connected to corresponding terminals on other equipment to eliminate potential differences.
	Ground (earth)	Additional protective ground (earth).
	Type BF	BF: Isolated from ground. Maximum patient leakage current under <ul style="list-style-type: none"> • Normal condition $\leq 100\mu\text{A}$ • Single-fault condition $\leq 500\mu\text{A}$
	Type BF	BF, defibrillator-proof
	Type B	B: Maximum patient leakage current under <ul style="list-style-type: none"> • Normal condition $\leq 100\mu\text{A}$ • Single-fault condition $\leq 500\mu\text{A}$
IP	Sealing	Dust- and immersion-protected according to EN 60529 [1].
	Standby	Standby switch on front of system – used to turn system on and off.
	Non-ionizing radiation	Ultrasound system emits acoustic radiation.
	STERILE	Device is in a sterile condition.
	Not watertight	Plug may not be immersed (unless it is covered with a special watertight plug cover).
	ESD (electrostatic discharge)	Do not touch pins in connectors with this symbol unless you follow ESD precautionary procedures.
	WEEE waste	Within the EU, when you discard the equipment, you must send it to appropriate facilities for recovery and recycling.



Symbol	Name	Description
	China ROHS 25 Years Lifetime	Environmentally Friendly Use Period for ROHS is 25 years.
	Specified Radio Equipment	(On remote control UA1237.) This equipment conforms to Japanese Radio Law regulations concerning frequency and power.

Table 1-3. Symbols on the system.

General Safety Precautions

The ultrasound system is designed and tested in accordance with EN/IEC 60601-1 [2] and EN 60601-2-37 [3]. It complies with requirements for Class 1 (protective earth) devices of EN/IEC 60601-1 [2]. It also complies with UL 60601-1 [4] and CSA C22.2 No. 601.1-M90 [5]. It fulfills the requirements for dust protection (IP20) for ordinary equipment specified in EN 60529 [1].



Caution: Federal law in North America restricts this device to sale to, or on the order of, a physician.



Equipment failure

WARNING If at any time the system malfunctions, or the image is severely distorted or degraded, or you suspect in any way that the system is not functioning correctly:

- Remove all transducers from contact with the patient.
- Turn off the system. Unplug the system from the wall and make sure it cannot be used until it has been checked.
- Do not try to repair the system yourself.
- Contact your BK Medical representative or hospital technician.



Isolating the system

WARNING The power supply cord connects the system to the line voltage. To isolate the system, you must unplug the power supply cord from the wall outlet. Do this before you try to make any repairs to the system.



Condensation

Caution: Large variations in temperature or humidity may cause water to condense inside the system. If this happens, the system may fail to operate properly. Always let the system come to room temperature before you plug it in.

- Wait at least 2 hours after the system has been subjected to major changes in temperature or humidity.
- If there is visible evidence of condensation, wait at least 8 hours.

Before you use the system, make sure that all the safety requirements described in this chapter have been satisfied.

Explosion Hazards




WARNING The ultrasound system is not designed to be used in potentially explosive environments. It should not be operated in the presence of flammable liquids or gases, or in oxygen-enriched atmospheres.

There is a possible explosion hazard if the system is used in the presence of flammable anesthetic. The system should be placed at least 25 cm (10 inches) from the patient.

The ultrasound system contains a lithium battery. Never remove or replace this battery. The lithium battery must not be removed except by a BK Medical service representative.

Electrical Safety

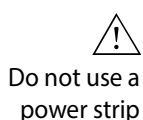

WARNING Do not plug the system into an ordinary power strip. If the ground connection fails, this is dangerous because

- the total leakage current for all the connected equipment can exceed the limits specified in EN/IEC 60601-1 [2].
- the impedance of the ground connection will probably also exceed the limits specified in EN/IEC 60601-1.




WARNING When the equipment is used with 230V (and you believe the leakage current would be within the UL limit if you were using 120V), power to the equipment must come from an installation or supply unit with a center-tapped, 240V single-phase circuit. This will make sure that chassis leakage current during single fault condition fulfills the requirements specified in UL 60601-1 [4] (limit of 300 μ A). If power is not supplied in the way specified, the leakage current can be as high as 500 μ A, the limit specified in EN/IEC 60601-1 [2].




WARNING Never remove the cover to get access to the inside of the scanner. You risk electrical shock if you do so. Do not allow anyone but qualified service personnel to service the scanner.

ESD Training

The ESD Symbol

Anyone using the equipment must be able to recognize the ESD symbol  and understand how to take the necessary precautionary procedures, as described in the caution below.




Caution: Do not touch pins in connectors that have the ESD symbol . Do not connect anything to them unless you follow these ESD (electrostatic discharge) precautionary procedures:

- Discharge your body to ground before you touch the pins with your hand or a tool. For example, touch an unpainted metal part of the system cover.
- You can use a wrist strap connected to the additional protective ground or potential equalization terminal on the scanner if that is more convenient.

Interference

The Pro Focus 2202 Ultrasound System is suitable for use in all establishments, other than domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

Electrical Noise

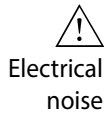


WARNING Electrical noise from nearby devices such as electrosurgical devices – or from devices that can transmit electrical noise to the AC line – may cause disturbances in ultrasound images. This could increase the risk during diagnostic or interventional procedures.

Electromagnetic Interference

Medical electrical equipment requires special precautions regarding EMC (electromagnetic compatibility) [6]. You must follow the instructions in this chapter when you install the system and put it into service.

If the image is distorted, it may be necessary to position the system further from sources of electromagnetic interference or to install magnetic shielding.

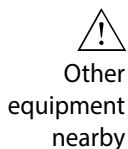

WARNING Do not use this equipment adjacent to other equipment. If you must place it next to or stacked with other equipment, verify that it operates normally there and neither causes nor is affected by electromagnetic interference.

EMC noise can reduce the usable image depth. Therefore, in order to avoid having to repeat an ultrasound examination, you must make sure beforehand that the ultrasound system can be used for the examination. Repeating an examination can be regarded as a potential risk that should be avoided, especially if the examination involves transducers used intracorporeally or transducers used for puncture.

RF (Radio Frequency) Interference

Portable and mobile RF (radio frequency) communication equipment can affect the system, but the system will remain safe and meet essential performance requirements.

An ultrasound system intentionally receives RF electromagnetic energy for the purpose of its operation. The transducers are very sensitive to frequencies within their signal frequency range (0.3MHz to 50MHz). Therefore RF equipment operating in this frequency range can affect the ultrasound image. However, if disturbances occur, they will appear as white lines in the ultrasound picture and cannot be confused with physiological signals.



Caution: Other equipment may interfere with the system, even if that other equipment complies with CISPR (International Special Committee on Radio Interference) emission requirements.



Caution: If you use accessories, transducers or cables with the system, other than those specified, increased emission or decreased immunity of the system may result.

Installation



Installation
safety
requirements

WARNING To ensure safe performance, a qualified electrical engineer or hospital safety personnel must verify that the ultrasound system is correctly installed and that it complies with the safety requirements described below:

- Use only the original power supply cord. This must be fitted with a hospital-approved three-prong grounded power plug. See “Power Supply Cord” on page 76.
- Use only the original monitor power cord to connect the monitor to the system.
- The equipment must only be connected to a grounded AC power supply (or wall outlet) that meets EN/IEC/NEC requirements or applicable local regulations. The examination room’s grounding system should be checked regularly by a qualified electronics engineer or hospital safety personnel.
- Never use extension cords. The increased length of the cord will increase the resistance of the protective ground conductor beyond an acceptable level.
- Keep power cords, sockets and plugs clean and dry at all times.

Original
power cords

If the original power cords are missing or damaged, you must order new ones from your local BK Medical representative.



Secure the
power
supply cord

WARNING Make sure that the power supply cord cannot be accidentally disconnected from the wall or the system.

Transducers



Electrical
shock

WARNING The transducer sockets contain terminals with 5V. Do not touch the patient while you are touching an uncovered socket.



Type B
transducers


WARNING When using Type B (non-isolated) transducers, carefully check all electrical equipment within the patient area. Also, consider using additional protective grounding.




Electrical
burns

WARNING Do not leave transducers in contact with the patient when using HF electrosurgical equipment.

BK Medical transducers fulfill EMC requirements when they are outside as well as inside the patient’s body.


Surface
temperature
on array

WARNING Do not turn the transducer on and allow it to scan into mid-air without ultrasound gel applied to the surface of the array. Doing so may cause the surface temperature on the array to heat up to 27 °C above room temperature (measured according to EN60601–2–37 [3]). To avoid this, freeze the image when the transducer is not used for imaging.


Creutzfeldt-
Jakob
disease

WARNING C-J-1

Do not use a transducer for neurosurgical applications if the patient is suspected of having Creutzfeldt-Jakob disease. If a neurosurgical transducer has been used on a patient suspected of or diagnosed as being Creutzfeldt-Jakob positive, the transducer must be destroyed, following approved procedures for your hospital.

Care

Please refer to the *Care, Cleaning & Safety* manual for information about caring for transducers.

Remote Control

If you use a wireless remote control UA1237 with the system, be sure to read about how to use it in Chapter 3, “The User Interface”.

The remote control uses short wave radio waves to communicate with the system.


Active at a
distance

WARNING The remote control is active at a distance of at least 10 meters from the system, even if the system is in a different room.



Caution: The remote control can be disrupted by other equipment operating at the same frequency of 2.5 GHz.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1 This device may not cause harmful interference, and
- 2 This device must accept any interference received, including interference that may cause undesired operation.

Other Safety Considerations

Mechanical failure or unintended use of ultrasound equipment can result in physical injury to patients or operators.




WARNING Be careful to avoid the following potential sources of injury:

- Parts of the body can be pinched by moveable parts of the system, such as the keyboard panel and the monitor.
- Tilting the system can cause it to be unstable and injure someone.
- Do not lean or sit on the keyboard or any other part of the system. The keyboard panel or monitor can break if subjected to heavy weights or impact.

Measurements


Pay careful attention when you position cursors to make measurements on an image or on a Doppler curve.

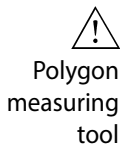

WARNING When you use the polygon measuring tool, if the sides of the polygon intersect (as in forming a curve like a figure eight, for example), the area calculation is incorrect. In this case, the calculated area of the polygon is the area of the bigger loop minus the area of the smaller loop.



WARNING Drawings of Doppler curves, manual and automatic, are meant as tools for positioning cursors so that measurements based on the curves can be calculated automatically. The system has no facilities for checking whether the automatic measurements are reasonable. Curves drawn on very noisy spectra may lead to misplacement of measurement cursors. Make sure that measurement cursors are positioned so that the results are reasonable. If they are not, you must adjust the position of the cursors manually.

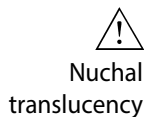

Caution: You must be adequately trained before you attempt to make nuchal translucency measurements.

Vector Flow Imaging¹



WARNING

Before you turn on VFI, check the B-mode image to make sure there are no artifacts visible in the blood vessel. If there are strong artifacts in the B-mode image, the arrows in VFI may be pulled to point in a more axial direction (toward or away from the transducer), especially in low flow situations with correspondingly low PRF. These artifacts will not affect the color mode (CFM) image, so it is important to check in B-mode.



WARNING


Check to make sure the VFI arrows are not aliasing before you activate the assisted Doppler gate placement. Otherwise, the Doppler gate will not be positioned correctly.

1. VFI is not licensed by Health Canada.




WARNING

Check to make sure that the diameter markers correspond to the inner vessel wall and that the connecting line between the markers is perpendicular to the direction of the vessel. Otherwise, the real-time volume flow measurement may not be precise.

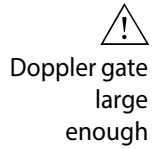


WARNING

Check to make sure that the Doppler gate covers the entire vessel. Otherwise, the real-time volume flow measurement may not be precise.

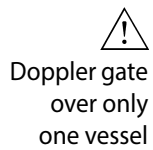

WARNING

Check to make sure that the Doppler gate only covers one vessel. Otherwise, the real-time volume flow measurement may not be precise.

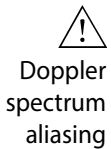

WARNING

Check to make sure that the Doppler spectrum does not alias. Otherwise, the real-time volume flow measurement may not be precise.

Puncture and Brachytherapy



3D Safety Considerations


Pay particular care to the following safety issues when operating the 3D system. For more detailed 3D operating instructions see the Extended User Guide.




WARNING You cannot make accurate measurements on a 3D data set acquired using the untracked freehand method.

If you start to make a measurement on a 3D data set acquired using the untracked freehand method, the following warning appears on the monitor to remind you that the measurement will not be accurate.

Warning on monitor

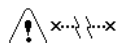



3D measurements

WARNING Measurements obtained with the 3D system and used in diagnosis must be carefully and thoughtfully performed to ensure accurate quantitative assessment. Before you perform a calculation, make sure that all necessary calibrations and measurements are made.


If you suspect that the 3D system's calibration is inaccurate (that is, the measurements are not as expected), contact your local BK Medical service representative to check and confirm the system's proper operation.

Picture in Picture


Do not cover patient ID or other important information


WARNING When you use Picture in Picture, do not cover critical information (such as TI or MI) on the monitor. Make sure that all important information will appear if you print or save the ultrasound image.

Contrast Imaging


High acoustic output during Bubble Burst


Caution: When you turn on Bubble Burst, the acoustic output limits are increased to 1.9 (MI). This change overrides any limits you have set. During Bubble Burst, the acoustic output will exceed normal contrast imaging values and may approach the higher (Bubble Burst) limits. (See "Thermal and Mechanical Indices" on page 25.)

Quality of Printed Images


Print quality

Caution: The quality of a printed ultrasound Image may vary, depending on the printer.


Service and Repair


Authorized personnel

WARNING Service and repair of BK Medical electromedical equipment must be carried out only by the manufacturer or its authorized representatives. BK Medical reserves the right to disclaim all responsibility, including but not limited to responsibility for the operating safety, reliability and performance of equipment serviced or repaired by other parties. After service or repairs have been carried out, a qualified electrical engineer or hospital technician should verify the safety of all equipment.


Connecting Other Equipment




WARNING Consult this user guide before connecting other equipment to terminals marked with .

To fulfill EMC requirements, do not attach cables to the system unless they are the same type as listed in Table 4-2 on page 74 and do not exceed the maximum length given in the table. Do not attach transducers and other accessories unless the user guide for the transducer or accessory states that it can be used with this system. Attaching other equipment may cause an increase in electromagnetic emissions or may cause the system to be more sensitive to electromagnetic interference.

The system must not be galvanically connected to a computer network (DICOM) that has not been isolated. If the network is not isolated, the system must be connected via a network isolator DP0925 (see Accessories in Product Data sheet).

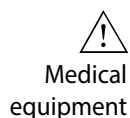

WARNING Equipment that complies with the requirements of EN/IEC 60601-1 [2], UL 60601-1 [4] or CSA C22.2 No. 601.1-M90 [5] can be connected to the system, but the power for the equipment must come from the auxiliary power output on the system or from an independent wall power outlet. You can use the isolated auxiliary power outlets on the system to connect equipment such as a monitor or video printer requiring a total of 350VA or less. Otherwise, you can plug the system and other equipment into an external common isolation transformer in order to control the leakage current during a ground connection fault. Follow the guidelines in EN60601-1-1 [7]. If in doubt, contact your local BK Medical representative.

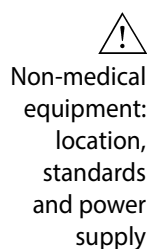

WARNING If you connect non-medical equipment (instruments that do not comply with safety requirements for medical equipment), this equipment must be placed outside the patient environment (1.5m from the bed, for example). The equipment must fulfill the relevant EN standard or other applicable national or international standard.

The power for the equipment must come from the auxiliary power output on the system. You can use the isolated auxiliary power outlets on the system to connect equipment such as a monitor or video printer requiring a total of 350VA or less. Otherwise, you can plug the system and other equipment into an external common isolation transformer in order to control the leakage current during a ground connection fault. Follow the guidelines in EN60601-1-1 [7]. If in doubt, contact your local BK Medical representative.

Network Connection

BK Medical's range of ultrasound systems comply with the DICOM standard for handling, storing, printing and transmitting information in medical imaging. DICOM includes a file format definition and a network communication protocol which facilitates the exchange of data between electronic medical systems.

For detailed information about:

- network requirements
- network configuration
- workflow between devices
- technical specifications
- safety specifications

see BK Medical's DICOM conformance statement at http://www.bkmed.com/DICOM_en.htm

Network Security

It is the responsibility of the on-site personnel or technician to maintain the IT-network and identify, analyze, evaluate and control new risks caused by a change in the network configuration.

If the applicable network connection does not meet the required characteristics of the IT-network, the following hazardous situations may occur:

- Corrupt patient data due to network errors, see patient ID warning on page 37.
- System is unable to use the network due to faulty or overloaded network, see training requirements warning on page 24.
- System overloads the network causing other equipment to fail.

Network
guidelines


NOTE: *If your system interacts with other equipment directly or indirectly you must ensure that your network is properly dimensioned and that critical equipment is placed on a separate network. Otherwise you could risk overloading the network and your equipment failing.*

Network Printing

For printing on network printers, BK Medical support protocols PCL 5, PCL 6 and PS (Post Script).

Computer Security

When the Pro Focus is connected to a hospital network, BK Medical does not take any responsibility for computer viruses from the network that may infect the Pro Focus. We recommend that the system is protected by a network firewall.

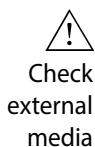

Caution: You must perform a virus check on any external storage medium (USB device or CD) to make sure that it is virus-free before you connect it to the system.

Wireless Networks¹

The system can be connected to a wireless network for printing and archiving data. This requires the optional wireless license for the system, and a Wi-Fi adapter that is supplied by BK Medical when you buy this option. The Wi-Fi adapter is inserted into one of the USB connectors located on the rear of the system (see Fig 4-2 in Chapter 4, “Technical Guide” on page 71).



Caution: Do not use a USB extension cable to connect a Wi-Fi adapter to the system. If you do, the wireless network may become unstable due to electromagnetic interference.


For information on how to install a license on the system, see the Setting Up and Customizing Your System chapter in the Pro Focus 2202 Advanced User Guide (BB1946).

1. Wi-Fi on Pro Focus 2202 UltraView systems has not been licensed by Health Canada.


Connecting to a Wireless Network

You must establish a secure wireless network at your hospital, clinic or institution, including a password for the network, before you can use the system's Wi-Fi for printing and archiving.

To use Wi-Fi on the system you need to install a license. When you have installed the license, a Wi-Fi logo is visible in the lower middle part of the screen of the main screen.

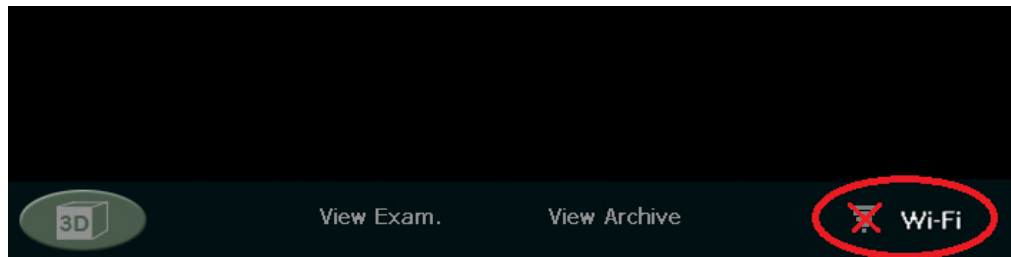

To connect to a wireless network, click the Wi-Fi logo, choose the appropriate network, and type in the password.

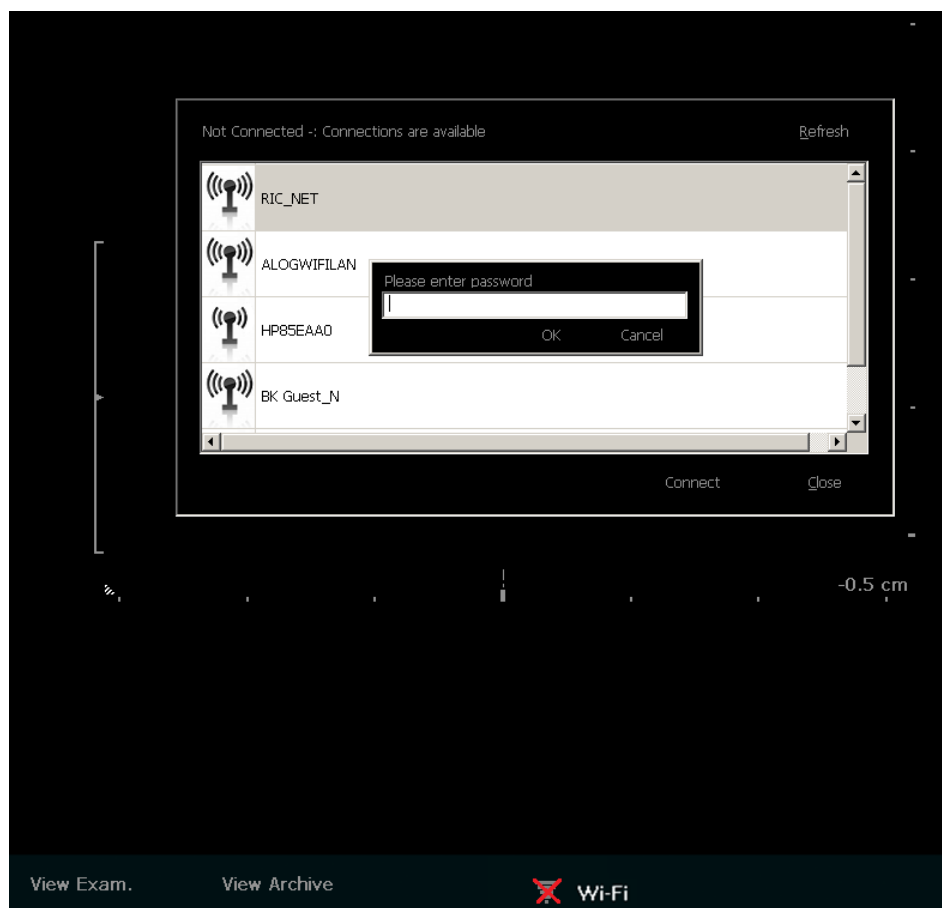

You only have to type in a password the first time you connect to a specific network. When you turn on the system an attempt is made to reconnect to the network that the system was connected to before it was turned off.



Weak vs.
strong signal

The bars in the Wi-Fi logo represents the signal strength. The more lit bars in the logo, the stronger the signal.



No wireless
connection

When the system is not connected to a wireless network, the Wi-Fi logo has a red 'x' over it.

An advanced Wi-Fi setting option is available on the system using Windows configurations. Always follow the security procedures that have been established for your hospital, clinic or institution, as well as national guidelines. Contact your BK Medical service representative for more information.

Additional Characteristics

Frequency band	2.412~2.4835 GHz
Data throughput	Max. 150 Mbps
Latency	Depends on network setup
Integrity	Full integrity of archiving operations
Security characteristics	Support for WEP, WPA and WPA2 encryption. Enterprise encryption (802.1x) requires assistance from an authorized BK Medical technician.
Spectrum management	None required

When you transmit data over a wireless network, some special considerations apply. In particular, the network connections must be set up correctly. See training requirements warning on page 24.

NOTE: *The system only supports one network at a time.*

If the system loses connection with the network while transmitting (for example, because it is moved out of range of the network during a transmission), the pending data is stored temporarily and re-transmitted when the connection is re-established.

For information on saving and printing using the DICOM protocol, see the Documentation chapter in Pro Focus 2202 Extended User Guide (BB1279).



Caution: The network must be set up correctly so that data is sent to the correct location. Otherwise data can be lost or accessed by unauthorized people.



Caution: A safe encrypted protocol for data transmission, approved by the hospital, must be used. This is to prevent unauthorized people from getting access to the data.

Acoustic Output

General


Medical research has yet to prove whether or not ultrasound causes biological effects. Therefore, prudent use considerations require you to follow certain guidelines [3].

Prudent Use



WARNING Always keep the exposure level (the acoustic output level and the exposure time) as low as possible.

- Image patients only when clinical reasons make it necessary.
- Keep exposure time as short as possible.
- Be careful to prepare the patient correctly so that you get the best possible image.
- Start imaging at a low acoustic output level (see “Thermal and Mechanical Indices” on page 25) and increase the level only as much as necessary to obtain a satisfactory image.
- If you switch from an application requiring high acoustic output levels (see “Functions Affecting Acoustic Output” on page 26), to one that requires lower levels (fetal imaging, for example), be sure to reset the levels before you image. (For example, start in B-mode.)
- Take into account all the types of tissue that may be affected. For example, when imaging a breast, it may be appropriate to monitor the TI in bone rather than in soft tissue because the ribs will be subjected to ultrasound.

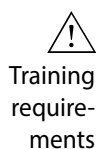

WARNING Before you attempt to use BK Medical equipment, you should be trained in ultrasonography or be under the supervision of someone who is trained in ultrasonography. You should also be thoroughly familiar with the safe operation of your ultrasound system: read all the user documentation that accompanies it. Always use the transducer best suited to the examination.

In addition, if your system interacts with other equipment directly or indirectly, you need to be trained in making sure the interactions are both safe and secure.

No further training is required, but BK Medical offers training in how to use the system. Consult your BK Medical representative for information.

Acoustic output data for transducers used with the system are given in Technical Data (BZ2100). The uncertainty level for each parameter is also listed. For definitions of the parameters, refer to the Food and Drug Administration (FDA) Guide [8] as well as EN 60601-2-37 [3] and AIUM/NEMA standards [9,10].

In North America, the FDA requires all ultrasound equipment to be cleared before it is marketed in the United States.

The routes (or tracks) available for clearance by the FDA are well-defined. Track 3 is for diagnostic ultrasound systems that follow the Output Display Standard. Under Track 3, acoustic output will not be evaluated on an application-specific basis, but the maximum derated Spatial Peak–Temporal Average Intensity (I_{SPTA}) must be $\leq 720 \text{ mW/cm}^2$, the maximum Mechanical Index (MI) must be ≤ 1.9 , and the maximum Thermal Index (TI) must be ≤ 6 . All BK Medical transducers for use with the Pro Focus 2202 Ultrasound System are Track 3.

Monitor Display

The Mechanical Index (MI) and Thermal Index (TI) can be viewed in all imaging modes.

Thermal and Mechanical Indices

The MI and TI indices are intended to allow users to implement the ALARA [11, 12] (As Low As Reasonably Achievable) principle using an indicator related to a potential bioeffect.

The full details of the indices are given in references [3] and [10], but the formulas are given below.

MI Formula

$$\text{MI} = \frac{P_{r0.3}(z_{sp})}{\sqrt{f_c}}$$

where the variables are defined in the table below.

Variable	Definition
$P_{r0.3}(z_{sp})$	Peak Rarefactional Pressure (MPa), derated by 0.3 dB/cm-MHz, measured at z_{sp} , the point on the beam axis where pulse intensity integral ($PII_{0.3}$) is maximum
f_c	measured center frequency (in MHz)

TI Formula

$$\text{TI} = \frac{W_0}{W_{deg}}$$

where the variables are defined in the table below.

Variable	Definition
W_0	time-averaged acoustic power of the source or other power parameter (W)
W_{deg}	estimated power necessary to raise the temperature of the target tissue one degree Celsius ($W/^\circ\text{C}$)

Blood
perfusion and
TI

As a rule of thumb, the Thermal Index (TI) indicates the highest expected temperature increase in degrees Celsius. It is based on an average level of blood perfusion. The displayed TI may underestimate the temperature rise in poorly

perfused tissues; you must take this into account when deciding on the maximum TI you will allow. Conversely, in areas with a rich perfusion of blood the temperature increase will be less than the displayed TI indicates.

Fever A temperature increase of one degree Celsius increase in a patient with fever may cause complications in certain circumstances; it may be safer to delay the investigation.

Acoustic Output Measurement

All values are measured in water according to the EN and AIUM/NEMA display standards [3, 9, 10]. For the some of the acoustic parameters, an estimated in situ derated value is given. This is derived assuming a tissue attenuation of 0.3 dB/cm·MHz when the estimated in situ derated value (I) is described by the following equation:

I formula $I = I_w \exp(-0,069fz)$

where the variables are defined in the table below.

Variable	Definition
I_w	Intensity in water at the position where I is maximum
f	transducer frequency (in MHz)
z	distance (in cm) from the transducer face to the position where I is maximum

It should be stressed that the in situ values given are only applicable when there is attenuating tissue between the transducer face and the focal point.

Possibility of Adverse Effects

Although it is believed that diagnostic ultrasound causes no significant biological effects in mammalian tissue [3, 10], the user should be aware of the hypothetical possibilities of adverse effects [11, 13, 14].

Fetal imaging Current scientific and clinical concern over possible adverse effects is particularly focused on fetal ultrasound imaging. It is due to the increased sensitivity of mammalian cells and organs at this phase of their development and the fact that such a risk could have profound implications on public health. If you use high acoustic output levels for some reason (see “Functions Affecting Acoustic Output”, below), be sure to return to B-mode alone and turn down the power level before you do any fetal imaging.

Functions Affecting Acoustic Output

The system has a control function that ensures that neither the I_{SPTA} nor MI nor TI value exceeds the maximum allowable value. When necessary, the system will reduce the output voltage and/or PRF (pulse repetition frequency) to the transducer in order to comply with requirements.

Some of the system functions can affect the acoustic output, as listed here. (Instructions for using these functions are given in the relevant sections of this user guide.)

- Sizing functions such as ROI (region of interest) – in general, smaller size results in higher acoustic intensity because the pulse repetition frequency (PRF) is higher or the ultrasound beam is more strongly focused.
- Focus – in general, strongly focusing the beam makes the acoustic intensity higher.
- Frame rate – higher frame rate results in higher acoustic intensity.
- Range – increasing the Doppler range increases the acoustic intensity by increasing the PRF.
- CFM Resolution – higher resolution increases the acoustic output.
- Color box size – narrowing the color box generally increases the acoustic output within it.

The user can set a Thermal Index limit. This will provide an upper limit for acoustic output.

Default Acoustic Output

After the system has been turned off, the transducers will start in the default setup when the system is turned on again. The default setup may be factory-defined or defined by the user.

The factory-defined default setup values of acoustic output for each transducer are listed in Technical Data (BZ2100).

These setups have been optimized to give the best compromise between low acoustic output and enough power to obtain the image features as quickly as possible. This is part of the ALARA principle. The factory default setup for all transducers is B-Mode to ensure the lowest acoustic output when you start imaging.

When you enter a new patient ID, the transducer setup will be reset to the default setup. You can prevent this from happening – see “Keeping Image Settings” on page 41.

Fetal imaging

When you use transducers intended for fetal imaging, it is important to make sure that the default settings are appropriate and to reset to the default setting before imaging a new patient.

Clinical Measurements: Ranges and Accuracies

This section states the accuracies for measurements made using the BK Medical range of ultrasound systems. A table containing accuracies for specific transducers can be found in Technical Data (BZ2100).

The measurement accuracies are based on the assumption of “ideal” tissue, that is, a tissue characterized by a sound velocity of 1540 m/s. When making clinical measurements with ultrasound, errors may arise which are not taken into account in this section. For example:

- the sound velocity may vary from approximately 1450 m/s in fatty tissue to 1585 m/s in muscle. This can, in simple cases, give rise to errors of up to 6% for linear measurements. This inaccuracy may be further increased by refraction occurring at tissue boundaries.
- the user can introduce errors when using approximate formulas, when positioning the system’s calipers with respect to the ultrasound image and when outlining structures in the image.
- the ellipsoid volume approximation, described in this user guide, is only applicable when the cross section of the structure being studied approximates an ellipse (the circle being a special ellipse), and when the structure is roughly symmetrical about the selected axis of rotation

NOTE: *The choice of the axis of rotation is important for the calculation of the volume. A vertical axis gives a different volume than a horizontal axis.*

The minimum requirement is that the cross section outlined by the user should be convex. If the user draws a non-convex outline, an inaccuracy is introduced which is not taken into account in this section. In this case, the system calculates and displays the convex hull of the figure, that is, the smallest convex figure containing the non-convex figure outlined by the user (see Fig 1-1). The system bases the calculation of the ellipsoid volume on the convex hull.

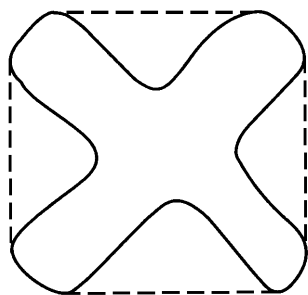


Figure 1-1. The dotted line indicates the convex hull of the non-convex figure.

Measuring volume using a stepping method produces an approximation caused by the finite number of steps in the measurement. The user must always try to assess how large an inaccuracy is introduced by the selected step size, that is, the distance between organ cross sections.

Geometric Measurements

2D Measurements

The geometric measurements performed by BK Medical ultrasound systems are distance, perimeter, area and ellipsoid volume. The accuracy of these measurements is influenced by the following factors:

- transducer geometry
- rounding of results
- resolution of digital image memory

3D Measurements

The 3D volume is found by summing the marked area in the individual slices and multiplying by a factor that includes the distance between the slices and their relative orientation.

To obtain the accuracies listed in the table in Technical Data (BZ2100), you must ensure that the calculation is based on contributions from at least 10 slices for very regular shapes and more for irregular shapes. It must also be possible to discriminate the boundary of the object from the surrounding tissue.

Volume accuracy

The volume accuracy in the table is given as a percentage of the captured 3D volume starting with the first slice and ending with the last slice that intersects the object of interest.

If the 2D ROI (region of interest) is set to be much larger than the object, the accuracy as a percentage of the object volume can be much worse (higher percent).

NOTE: *To ensure that the accuracy of your volume measurement is as high as possible, make sure that the object you are interested in fills the region of interest as much as possible.*

Distance and area accuracy

The accuracy of a distance measurement on a 3D image will never be better than 6%; the accuracy of an area measurement on a 3D image will never be better than 6%.

In the table in Technical Data (BZ2100), the overall measurement accuracy for a full range measurement is given in the right-hand column for each measurement. The footnote below the table states the digital image resolution.

Time Measurements

In M-mode and spectral Doppler mode, data is displayed along a time axis. It is possible to measure time differences. The accuracy for a time difference measurement is:

- rounded to the nearest: 0.01 s
- accuracy: $0.01 \times t$
where t is the full time scale of the image field.

Doppler Measurements

In measuring blood flow velocity it is assumed that the measured power spectral distribution of the Doppler signal equals the blood cell velocity distribution.

The measurement accuracy of blood flow velocity is heavily dependent on the angle θ between the ultrasound beam and the velocity vector of the blood cells. The velocity accuracies given in Technical Data (BZ2100) are valid for $\theta = 55^\circ$. To find the percentage accuracy for other angles, multiply the stated accuracy by

$$\left(\frac{\cos\theta - \cos(\theta + 1,8^\circ)}{\cos\theta} \right) \times 100 + 0,5$$

If the blood velocity exceeds the selected velocity range, aliasing occurs, corresponding to an overload condition of the measurement system.

References

- [1] EN 60529:1991+A1:2000. *Specification for degrees of protection provided by enclosures (IP code)*.
- [2] EN/IEC 60601-1:1990+A1:1993+A2:1995+A13:1996 *Medical electrical equipment. Part 1: General requirements for safety*.
EN/IEC 60601-1:2006 3rd Ed. *Medical electrical equipment. Part 1: General requirements for basic safety and essential performance*.
- [3] EN 60601-2-37:2001+A1:2005+A2:2005 *Medical electrical equipment – Part 2-37: Particular requirements for the safety of ultrasonic medical diagnostic and monitoring equipment*.
EN 60601-2-37:2008 *Medical electrical equipment – Part 2-37: Particular requirements for the basic safety and essential performance of ultrasonic medical diagnostic and monitoring equipment*.
- [4] UL 60601-1:2003 *Medical Electrical Equipment – Part 1. General Requirements for Safety*.
- [5] CSA C22.2 No. 601.1–M90 *Medical Electrical Equipment – Part 1. General Requirements for safety*.
- [6] EN 60601-1-2:2001 + A1:2006 *Medical electrical equipment - Part 1-2: General requirements for safety. Collateral standard: Electromagnetic Compatibility-Requirements and tests*.
EN 60601-1-2:2007 *Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance. Collateral standard: Electromagnetic Compatibility-Requirements and tests*.
- [7] EN 60601–1–1:2001 *Medical electrical equipment –Part 1-1: General requirements for safety. Collateral standard: Safety requirements for medical electrical systems*.
- [8] *Information for Manufacturers Seeking Marketing Clearance of Diagnostic Ultrasound Systems and Transducers*. FDA Center for Devices and Radiological Health. September 2008.
- [9] *Acoustic Output Measurement Standard for Diagnostic Ultrasound Equipment*. AIUM/NEMA. 2004.
- [10] *Standard for Real-Time Display of Thermal and Mechanical Acoustic Output Indices on Diagnostic Ultrasound Equipment*. AIUM/NEMA. 2004.
- [11] *Medical Ultrasound Safety*. AIUM. 2009.
- [12] *ALARA Training Program*. Ultrasound III Training. B-K Medical. 1993.
- [13] Bioeffects considerations for the safety of diagnostic ultrasound. *Journal of Ultrasound in Medicine*, Vol. 7, No. 9 (supplement). September 1988.

[14] *The safety of diagnostic ultrasound*. The British Institute of Radiology. 1987.

AIUM: American Institute of Ultrasound in Medicine

CSA: Canadian Standards Association

EN: European Standards

FDA: Food and Drug Administration (USA)

IEC: International Electrotechnical Commission

NEC: National Electrical Code

NEMA: National Electrical Manufacturers Association

UL: Underwriters Laboratories Inc., USA

Chapter 2 Getting Started

General Information

The system and the monitor menus shown in this user guide may look different from yours.

Before you turn on the system, make sure that the installation has been approved by a qualified electrical engineer or hospital safety personnel. See “Installation” on page 15.

The controls you need to operate the system are located on the front.

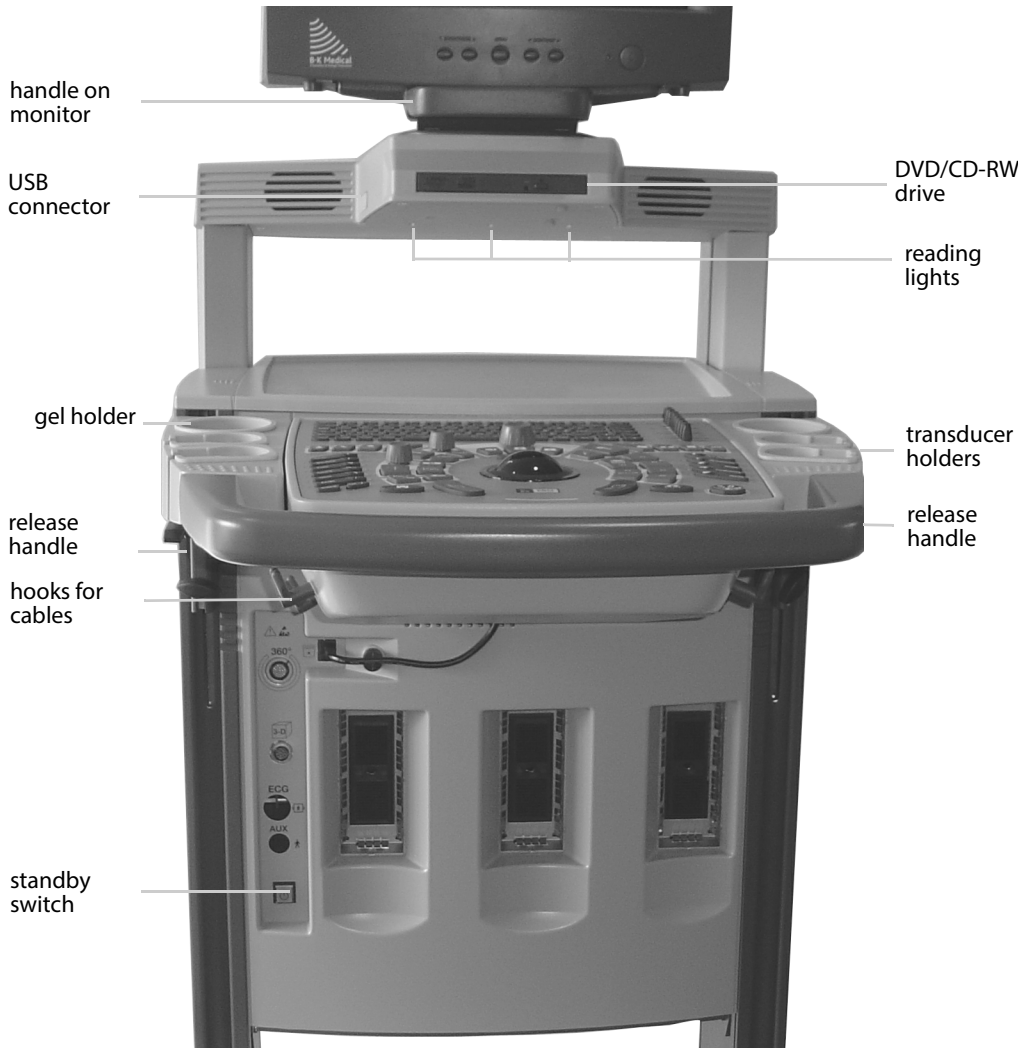


Figure 2-1. The front of the system.

Adjusting the Keyboard Panel and Monitor

You can easily adjust the position of the keyboard panel and monitor to make imaging comfortable for the operator. Instructions vary depending on the type of monitor and keyboard you have.

Up and down

To adjust the height of the keyboard panel and monitor:

- 1 Grab the panel on both sides and pull up on the two release handles (see Fig 2-1). Move the panel up or down.
- 2 Release the handles.

Angle

To change the viewing angle of the monitor:

- Use the handle on the front of the monitor to swivel it from side to side and tilt the monitor up or down.

Adjusting
resistance of
the flat
screen

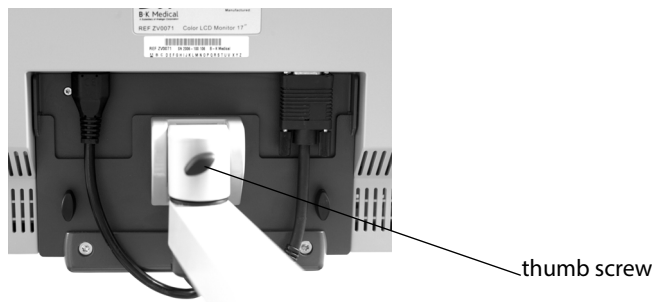


Figure 2-2. Thumb screw for adjusting resistance on the back of the LCD flat screen monitor.

To adjust the force required to change the viewing angle of the flat screen monitor:

- Use the thumb screw on the back of the LCD flat screen to increase or decrease the monitor's sensitivity to angle adjustments.

Hand Rest

The removable hand rest (UA0967) can be used to make working with the keyboard panel more comfortable. See *Care, Cleaning & Safety*.

Docking the Flat Screen on Arm

The flat screen monitor on the articulating arm can be docked. This prevents it from moving around, which is particularly important when the system is being moved to another location.

To dock the LCD flat screen:

- 1 Standing in front of the system, maneuver the flat screen on the arm so that the front of the monitor is over the front of the system body.
- 2 Holding the monitor handle, guide the monitor back.
- 3 Still holding the handle, press so that the monitor clicks into place.



Caution: Avoid using excessive force when docking the flat screen monitor.



Caution: We recommend that you dock the flat screen monitor before you wheel it around the hospital.



Caution: Do not put any weight on the arm nor use it as a lever. The arm is only designed to support the weight of the flat screen monitor.

Turning the System On and Off



To turn the system on and off, use the green standby switch on the front of the system.

To turn the system on:

- Press the standby switch ONCE. A startup screen with progress bar is displayed. Wait until the startup screen disappears and the display on the monitor is stable, indicating that the system is ready to use. Do not try to use the system until it is fully ready.

To turn the system off:

- Make sure the system is completely up and running. Then press the standby switch ONCE.



Wait after you press standby switch

Caution: Whenever you turn the system on or off, you must give the operating system of the system PC enough time to save and recover open files and unsaved data. Otherwise, a serious system failure may occur that requires technical support.

Never unplug the system from the wall without turning off the standby switch and waiting for its light to go out.

Wait to unplug

If you turn off the system without following the recommended procedure (for example, if you unplug it while it is running), the next time you turn the system on, it may take a longer time to start, and the system may be permanently damaged.

System Busy

When the system is busy, that is, performing an operation that takes some time, a spinning disk appears above the **Freeze** control on the monitor.



Caution: Wait until the spinning disk disappears before you press any keys on the system or buttons on the transducer.

Checking the Date

Check that the date and time displayed on the monitor are correct before you start imaging a new patient (or at least at the beginning of each day).



WARNING An incorrect date or time will make documentation of the image incorrect and may also cause some calculated values to be incorrect.

One cause of an incorrect date is battery failure. The battery fails very rarely. If it does fail, the system will usually not operate at all, so you will know to call a technician. However, under certain unusual circumstances the system will operate, but the date will be incorrectly set to a default date.

Settings on the Monitor

Instructions for the monitor vary depending on whether your system is fitted with an LCD flat screen monitor or an LCD flat screen monitor mounted on the articulating arm.

Brightness

Your system may be used with different levels of room lighting. Whenever the lighting changes significantly, you should adjust the monitor brightness.

NOTE: Do not press the [+] button on the 19" LCD flat screen monitor or the [●/▶] button on the 17" LCD flat screen monitor. These activate the contrast menu. The contrast settings should only be adjusted by the hospital technician or by your BK Medical service representative.



Figure 2-3. 19" flat screen monitor buttons.

To adjust the monitor brightness on the 19" LCD flat screen monitor:

- 1 Freeze the image.
- 2 Press [**Alt+Shift+G**] on the keyboard.
A series of gray bars appears on a black background.
- 3 Press [-] once to enable brightness.
- 4 Then press [+] to increase or press [-] to decrease brightness.
Subsequent presses either increase or decrease the brightness.

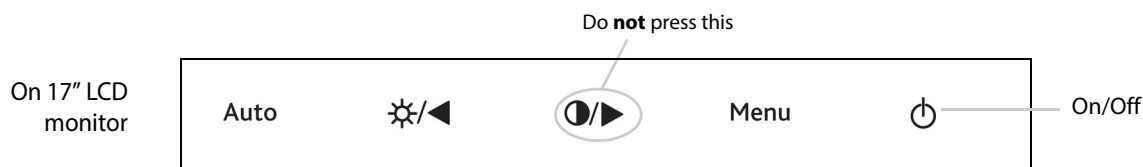





Figure 2-4. 17" flat screen monitor buttons.

To adjust the monitor brightness on the 17" LCD flat screen monitor:

- 1 Freeze the image.
- 2 Press [**Alt+Shift+G**] on the keyboard.
A series of gray bars appears on a black background.
- 3 Press [☀/◀] once to enable brightness.

- 4 Then press [] to increase or press [ / ] to decrease brightness. Subsequent presses either increase or decrease the brightness.

Factory Defaults

If the monitor image appears skew or too large, or the brightness and contrast settings appear incorrect, you can restore the factory default settings.

On LCD monitor

To restore the factory default settings on the LCD flat screen monitor:

- 1 Click **View Archive** at the bottom left of the monitor.
- 2 Point at **USB Eject** (do NOT click) so the frame around the monitor control becomes visible.
- 3 Press [**AUTO**] twice.

Menu Window on LCD Flat Screen

Advanced options

On the flat screen monitor, advanced options (which enable you to finely tune the appearance of images on the monitor) are available via the **Menu** window.

For most users, it is not necessary to adjust these settings because the default settings provide the optimal ultrasound image in most cases. Contact the hospital technician or a BK Medical service representative to adjust the gamma settings or the aspect-ratio controls.

Adjusting the Reading Lights

After you have turned on the system, you can adjust the brightness of the 3 reading lights under the DVD drive. (See Fig 2-1.)

To adjust the reading light brightness:

- Click **Light** under **Image** on the right side of the monitor and drag the slider ((or point at **Light** and press [+/-]).

Setting up a New Patient

Before you image a new patient, you must open the **Patient** window and enter some information about the patient, including the patient ID. The fields in the **Patient** window contain information about the patient and the examination.


Patient ID required

WARNING You must enter a new patient ID before you scan a new patient. Otherwise the documentation will not contain the correct patient identification, and you will not be able to capture images and clips. We recommend that you enter the complete name of the patient.


Verify patient ID

WARNING Verify that the patient name and ID are correct.

Opening the Patient Window

There are 3 ways to open the **Patient** window.

Opening the
Patient
window

To open the Patient window:

- Press [ **ID**].


or

- Click **Enter Patient....** or the name of the previous patient in the identification area on the monitor.

or

- Click **New Examination** in the **Archive** window.

The **Patient** window opens.

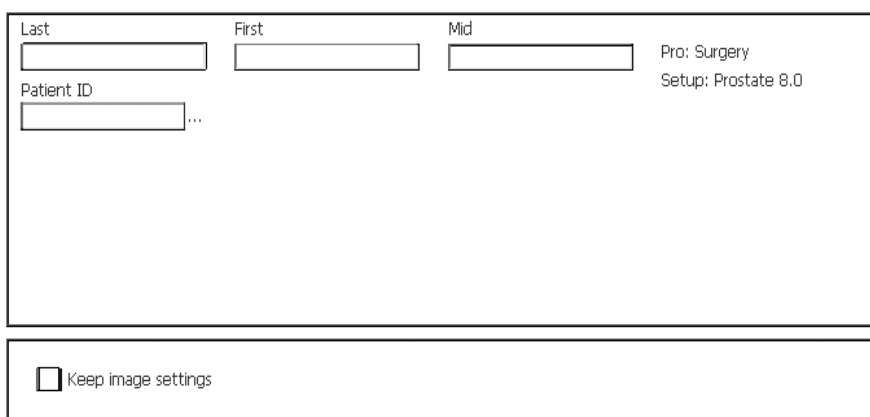

Last	First	Mid	Pro: Surgery
<input type="text"/>	<input type="text"/>	<input type="text"/>	Setup: Prostate 8.0
Patient ID			
<input type="text"/>			
<input type="checkbox"/> Keep image settings			

Figure 2-5. The Patient window.

Change the
Pro Package
first

Some fields in the **Patient** window are different, depending on which Pro Package is selected. Therefore, if you want to change the Pro Package, make sure you do it before you enter information about the patient. See the next section for how to do this.

If DICOM® is installed on your system, the **Patient** window may contain a worklist, as in Fig 2-6.

Auto-
complete

When you start to type the patient's name or ID, the system searches the patient database for patient names that match the entered data. A list of possible matches appears, and you can select the one you want and click it or use the arrow keys on the keyboard to select it and press [**Enter**].

To select the patient from the patient archiving system:

- Click ... next to the **Patient ID** field.

The **Archive** window opens.

Changing Pro Package or Preset in the Patient Window

When the **Patient** window opens, a default Pro Package and a default preset are displayed in the window.

To change Pro Package or preset in the Patient window:

- 1 Point at the name of the Pro Package or preset.
- 2 Press [+/-] to toggle through the available choices.

Click to save patient data and go to Pro Package menu window

If you click one of the default setups (instead of toggling through the choices), the data in the **Patient** window are saved and the **Pro Package** menu window appears. You can select a different Pro Package and preset from the menu. See “Selecting a Pro Package or Preset” on page 46. After you select something from the menu, the **Pro Package** menu window closes. If you have selected a new preset, imaging begins. If you have selected a new Pro Package, the image is frozen when the Patient window closes.

Using the Patient Window

TAB to move between fields

After you have entered information in a field, press [**Tab**] to move from field to field. (You can also move the cursor to a field by clicking the field.)

ENTER closes window

Do not press [**Enter**] (**Return**) unless you want to close the window. [**Enter**] on the keyboard is the same as **OK** in the window.)

Re-opening Patient window

If you decide to enter more patient data after you have started the examination (or if you have closed the **Patient** window by mistake before you have finished entering data), you can re-open the window. To re-open the **Patient** window, click the patient name at the top of the monitor. When the (blank) **Patient** window opens, click **Edit** to restore the previously entered patient data to the fields

NOTE: *Any information you enter after you re-open the **Patient** window will not be transferred to the patient archiving system.*

New Patient Information from a DICOM Worklist

DICOM® is not installed as a default on the system. If DICOM is installed, the system may be set up so that you can retrieve a worklist of patients and then select a patient from the worklist.



Caution: Changing the DICOM setup can cause your system not to work properly. For example, you may be unable to print to a DICOM printer. All changes to the DICOM setup should be made by qualified service personnel only. Do not try to change the DICOM setup yourself.

Worklist Update +/- 3 Days

Date	Time	Name	Birth Date	ID	Sex	Accession Nu...	Req. Proc.

Last: First: Mid:

Patient ID: ...

Pro: General
Setup: Abd-Res-1

Req. Proc. Code Mean: Req. Proc. Code Val.: Sch. Prot. Code Mean: Sch. Prot. Code Val.:

Keep image settings

Edit Discontinue Examination OK Cancel

Figure 2-6. The Patient window with worklist.

Depending on how your DICOM system is set up, the worklist may appear as soon as you open the **Patient** window. If the worklist is blank, you can retrieve the information.

To retrieve a worklist:

- 1 Use the drop-down window in the upper right corner to select the dates or the patients you want the list to include.

If you choose the **Patient** option in the drop-down window, you can filter the information so that only certain patients are displayed by entering patient name, patient ID, accession number and procedure ID in the appropriate fields.

NOTE: *The Accession Number and Req. Procedure ID fields do not appear by default. You must activate the fields in the **Patient Setup** window (click **Customize...** under **Image** on the right side of the monitor. Then click the **Patient** tab).*

- 2 Click **Update**.

The worklist appears in the window. If there are more patients than can fit in the window, you can scroll down to see the rest of the list.

Sorting To sort the worklist by name, or any other column in the worklist, click the column heading.

To select a patient from the worklist:

- 1 Click the row that contains the patient.
Now that patient is shown in the fields below the worklist.
- 2 If required, enter additional information in the fields in the window.
- 3 Press **OK** to start the examination.

Entering Dates

The **Patient** window contains one or more date fields. Dates in these fields can be displayed in various formats.

Format	Example
dd-MM-yyyy	18-11-1944
dd MMM yyyy	18 Nov 1944
MM/dd/yyyy	11/18/1944
MMM/dd/yyyy	Nov/18/1944

Table 2-1. Possible date formats.

The date format is set up in the **General Setup** window, but you probably will not need to change it.

Use number
keys

Use the number keys on the keyboard to enter dates.

NOTE: You cannot type letters in a date field, even when the month is displayed in letters. In the example in Table 2-1, you must type *11* for the month, even if it is displayed as “Nov”.

To enter dates in date fields:

- 1 Put the cursor at the beginning of the date field.
- 2 Type the date (or number of the month, if your system is set to display month first).
- 3 Type a separator or press → to move to the next part of the date. You can use any of the following separators: period (.), comma (,), hyphen (-) and slash (/). (The separator you use when you enter a date does not affect the way the date is displayed in the field.)
- 4 Type the appropriate number.
- 5 Repeat steps 3 and 4.

Keeping Image Settings

When you enter a new patient ID, the transducer setup is reset to the default for the transducer. If for some reason you want to keep the current settings when you start imaging the new patient, you can prevent the settings from being changed back to the default ones. However, this does not work if you change transducer or Pro Package.

To keep transducer settings:

- Check **Keep image settings** in the **Patient** window.
The system uses the image settings from the previous patient.

Keeping Patient-Related Data

When you enter a new patient ID, the patient-related data are reset to the default values. Patient-related data include the following:

- preset
- Bodymark
- Measurements
- Labels

You can start a new examination with the patient-related information from the current patient.

To keep patient-related data:

- 1 At the bottom of the **Patient** window, click **Edit**.
- 2 A new checkbox, **Keep patient-related data**, appears. Make sure that it contains a checkmark.

The system uses the patient-related data from the previous patient. You can edit the information, including changing the patient name.

NOTE: *You must check both **Keep image settings** and **Keep patient-related data** if you want to return to the previous frozen image when you close the **Patient** window. If you do not check both boxes, the system starts imaging when you close the window, and measurements you have made are lost.*

Pausing and Later Continuing an Examination

It is possible to pause an exam and then resume it later.

To pause an exam:

- 1 Start an exam. Capture some images. (This is not essential.)
- 2 In the **Archive** window, click **Pause/Continue Examination**.

To resume a paused exam:

- 1 In the **Archive** window, select a paused examination from the list.
- 2 Click **Pause/Continue Examination**.

To pause an exam and simultaneously resume one that was previously paused:

- 1 Start an exam. Capture some images. (This is not essential.)
- 2 In the **Archive** window, select the paused examination you wish to resume.
- 3 Click **Pause/Continue Examination**.
- 4 The current exam is paused and the selected one is resumed.

Discontinue Examination with MPPS Server Configured

If an MPPS server is configured, you can discontinue the current examination. Click **Discontinue Examination** to end the examination; the system will send a DISCONTINUE message back to the MPPS server. You will be prompted to confirm that the examination must be discontinued. Discontinuing the examination clears the current patient data and closes the **Patient** Window.

Discontinued examinations can be retrieved from the Patient List for later completion; see the Documentation chapter in the Extended User Guide for more details. The *2202 Service Manual* (BI2201) contains instructions for service personnel to use when configuring a MPPS server.


If you try to start or resume examinations on patients with discontinued examinations, you will be prompted to choose between these options:

- Continue the existing examination
- Delete existing examination and start a new
- Cancel

Ending an Examination

An examination is automatically ended if you start a new one.

To end an examination without starting a new one:

- Press (long) [ ID].

Connecting and Selecting Transducers

Connecting a Transducer

On the front of the system body are 3 sockets for connecting array transducers. There can be a socket for connecting mechanical transducers.

Keep cables
tidy

Under the keyboard panel are a number of hooks that can help keep transducer cables tidy and out of the way. (See Fig 2-1.)

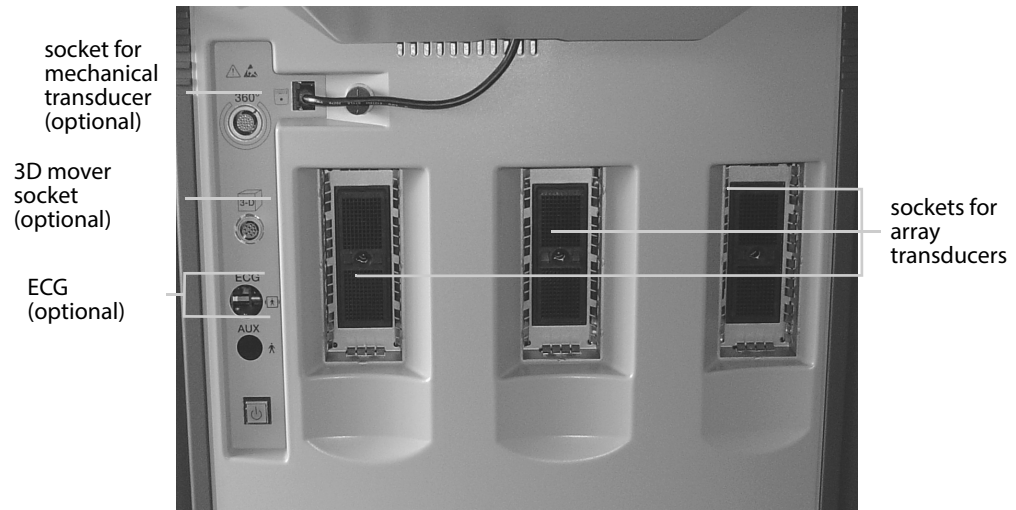


Figure 2-7. The transducer sockets.

To connect a transducer

- 1 Insert the transducer plug into the socket.
- 2 Turn the locking lever on the transducer plug clockwise.

To disconnect a transducer:

- 1 Turn the locking lever on the plug counterclockwise.
- 2 Remove the plug from the socket.

Transducer Holders

The 4 transducer holders beside the keyboard panel keep the transducers you need safe and ready for use. Transducer-holder inserts (DZ9756) are available to hold the smaller transducers in the BK Medical range. See the 2202 Product Data sheet for a list of accessories that can be ordered to use with the system.

Gel Holders

The two gel holders behind the transducer holders hold gel containers ready for use.

Selecting a Transducer

You can choose to image with any of the transducers that are connected to the system. The type number of the transducer currently selected is displayed, together with the imaging frequency, in the Identification area at the top of the monitor. See page 64.

⚠
Type
number
displayed
must match
number on
transducer

WARNING Before you start to image, make sure that the type number of the transducer you have selected matches the number displayed on the monitor. If they do not match, the puncture line on the monitor may not correspond to the true puncture path in the tissue. In case of any inconsistency, stop imaging, turn off the system, and contact your local BK Medical representative.

To select a different transducer that can be used with the same Pro Package:

- Point at the transducer type number on the monitor and press [+/-] to select the transducer you want.

or

- Press the button on the transducer itself (if it has a button).

To change Pro Package as well as transducer, or to see all connected transducers and the Pro Packages that support them, see “Selecting a Pro Package or Preset” on page 46.

Selecting the Imaging Plane

If you are using a transducer with more than one plane, you can select the imaging plane (sagittal, transverse or endfire). The current imaging plane (**T**, **S** or **E**) is displayed on the at the top of the monitor after the transducer type number.

To select the imaging plane:

- Press the appropriate button on the transducer. For details, see the user guide for the transducer.

or

- Click **T**, **S** or **E** (whichever is displayed) on the monitor and select the value you want, or point at **T**, **S** or **E** and press [+/-].

The imaging plane in the selected view is updated to the one you select.

Pro Packages and Presets

Each Pro Package contains all the measurement and calculation tools you need for a particular application area. It also contains appropriate presets for the various transducers that can be used with the Pro Package. Each transducer may have more than one preset, but one setup will be the default one.

The following Pro Packages are available on the system:

- Abdomen
- Brachy
- Cardiac
- Colorectal*
- MSK
- Neuro
- OB/Gyn
- Pediatric*
- Pelvic Floor
- Regional Anesthesia (UGRA Anes)*
- Small Parts
- Surgery
- Urology
- Vascular

* These Pro Packages are available on the Pro Focus 2202 UltraView 800 system only.


A preset contains information about the imaging modes that are being used and all the imaging information for those modes.

Selecting a Pro Package or Preset

The system starts imaging with the default preset for the transducer and Pro Package you are using.

The Pro Package and preset currently selected are displayed in the Identification area of the monitor. See page 64.

To select a Pro Package or preset:

- 1 Press  or click the Pro Package or preset displayed in the identification area of the monitor.

The **Pro Package Menu** window appears. It contains a column for each connected transducer. The column lists the Pro Packages and presets that you can use with that transducer.

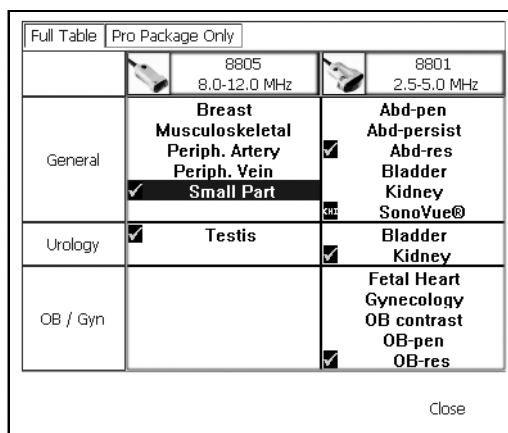


Figure 2-8. The Pro Package Menu window.

- 2 Select the Pro Package and preset you want.
- 3 Click **Close**.

Saving a Preset

When you have changed the setup, you can save it as a new preset.

To save a preset:

- 1 Click **Save** under **Image** on the right side of the monitor.

The **Save Preset** window appears.

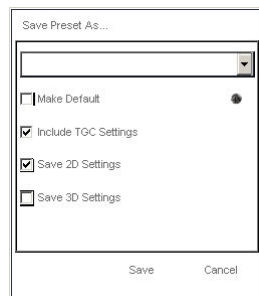


Figure 2-9. The Save Preset window.

- 2 Enter a name for the preset and select the options you want, for example, whether you want the setup to be the default for the transducer with the current Pro Package. (Puncture guide or brachy matrix settings will be saved as part of the setup.)
- 3 Click **Save**.

Selecting or Changing the Imaging Mode

The system is set up to start imaging in B-mode, unless something else is specified in the Pro Package you are using.

To add modes to B-mode, press the key for the mode you want to add.

Performing a Biopsy or Puncture Procedure (including Brachytherapy)

When you use a puncture guide to perform a biopsy or puncture procedure, a puncture line is superimposed on the image.

Brachy matrix
or
transperineal
matrix

When you use a transducer for brachytherapy or prostate transperineal biopsy, a needle guide matrix is superimposed on the image.



Check
puncture
guide type
number

WARNING Always check the type number of the puncture guide displayed on the monitor to make sure that it corresponds to the puncture guide that you are actually using. If the number is incorrect, the puncture line on the monitor may not correspond to the true puncture path in the tissue.



Watch the
needle tip

WARNING The puncture line on the image is an indication of the expected needle path. The needle tip echo should be monitored at all times so any deviation from the desired path can be corrected.

NOTE: *If the image depth is set very low (in order to see tissue close to the transducer with high magnification), depending on your settings, the needle tip echo can be outside the displayed image area. In order to see the needle tip in this case, zoom out so the full needle path is visible or pan the image to the side (to keep the high magnification).*



Verify matrix
type and
coordinates

WARNING For brachytherapy and prostate transperineal biopsy, make sure that the matrix type and coordinates indicated on the monitor agree with the actual matrix template you are using.




Verify matrix
alignment

WARNING Before you use the matrix for seed implantation or transperineal biopsy, check the matrix offset value to verify that it corresponds with the chosen matrix. Then check the matrix alignment. See *Care, Cleaning & Safety*.

Activate
guide or
matrix

Set default

To superimpose a puncture line or matrix on an ultrasound image:

- Press [].
The default puncture guide or matrix appears.

To set a different puncture line or matrix to be the default:

- After you have selected the puncture line or matrix you want (and made any other setup changes you want), save your settings as a new preset (see page 46). Specify that you want the new setup to be your default.

To highlight the dot on a matrix where you expect to see the needle:

- Click one of the dots on the matrix.
A blue square appears in its place.
- To highlight a different dot, click it. The previously highlighted dot returns to normal and the new one is highlighted.

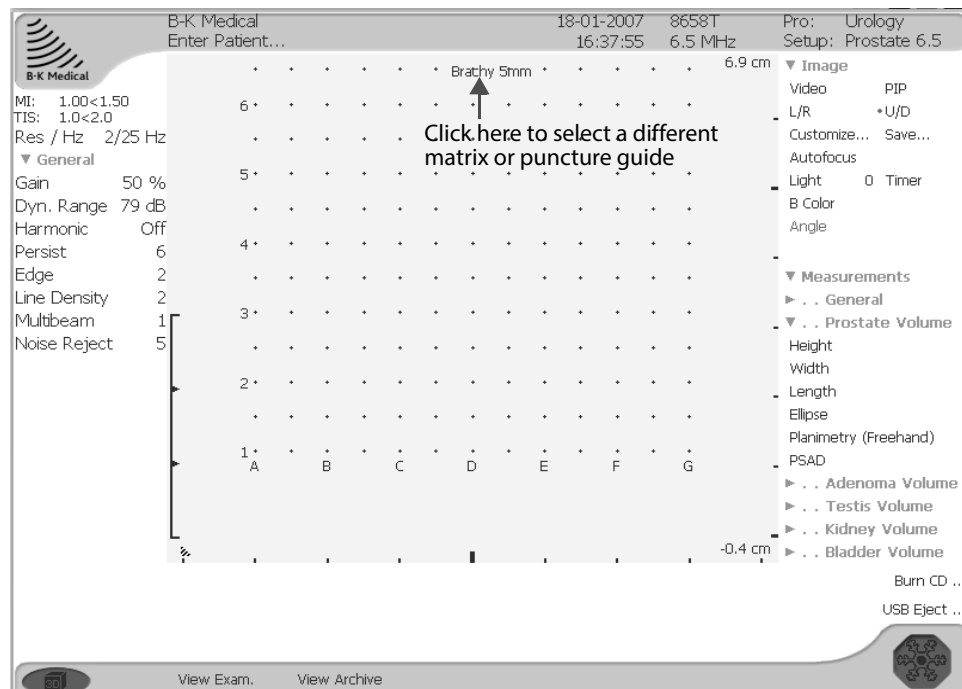


Figure 2-10. A matrix on the image area.

To select a different puncture guide or matrix:

- 1 Click the Type number or matrix name displayed at the top of the image area.
The **Puncture Guide** menu appears. It contains the puncture guides and matrices that are available for the selected transducer.



Figure 2-11. The Puncture Guide menu.

- 2 Click the puncture guide or matrix you want.

The menu disappears and the markings on the image area are updated.

To remove the puncture line or matrix from the monitor:

- Press [].

Programmable Puncture Guide

If you are using a programmable puncture guide, you can change the setup to move it to the left or right or in or out.

Brachy Ruler with Sagittal Plane Imaging

You can set up the system so that a brachy ruler is displayed when you image in the sagittal plane with the 8848 transducer in situations where a brachy matrix appears in the transverse imaging view.

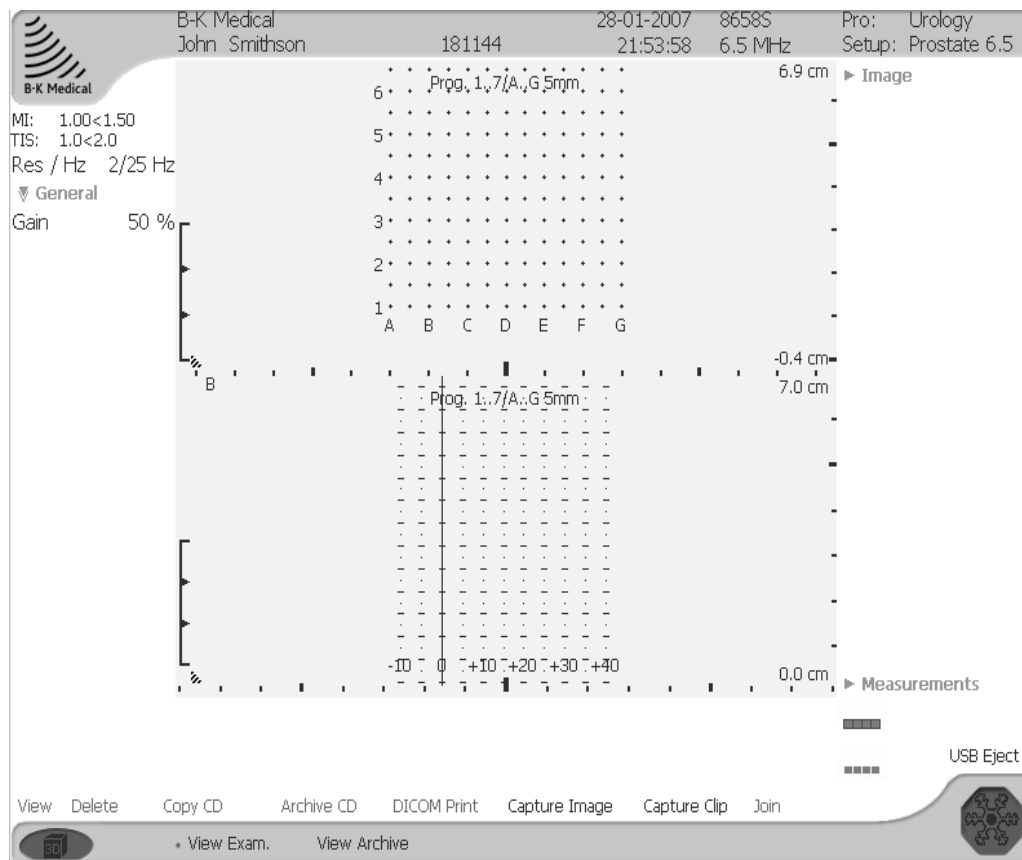


Figure 2-12. A split-screen view with transverse brachy matrix above and sagittal brachy ruler below. Ruler has vertical line to mark 0.

The sagittal brachy ruler is sometimes displayed with a 2-dimensional grid to make it easier for you to see the horizontal position of the needle no matter where it is in the vertical direction.

To move the ruler to the left or right:

- 1 Click the zero position on the horizontal (bottom) axis.
- 2 Drag the vertical line that replaces the ruler markers.

- 3 Click when the line is positioned where you want the new zero position of the ruler.

The vertical line disappears and the ruler markers appear in their new positions.

NOTE: *You can set up the system so that a vertical 0 line is always visible, along with the markers (as shown in Fig 2-12).*

User-Definable Matrix and Ruler

You can also define your own matrix and ruler.

Remote Control

For a description of the controls on the wireless remote control UA1237 and how to pair it with the system, see Chapter 3, “The User Interface”.

The remote control uses short wave radio waves to communicate with the system. Before you use it, be sure to read the warning and other safety information in the section “Remote Control” on page 16.

Cleaning and Disinfection

For details of cleaning and disinfecting the remote control, see *Care, Cleaning & Safety*.

Chapter 3

The User Interface

The Pro Focus system has 4 types of controls, in addition to buttons on the transducers:

- keyboard panel controls (keys)
- controls on the monitor
- foot switch (optional)
- keys on the optional wireless remote control

This chapter gives an overview of the various controls and how they are used in general. Detailed information about using individual controls is contained in the *Pro Focus 2202 Extended User Guide* (BB1279). The monitor menus shown in this user guide may look different from the version on your system.

Keyboard Panel Controls

The Pro Focus keyboard panel contains:

- an alphanumeric keyboard
- a control panel with special system control keys.

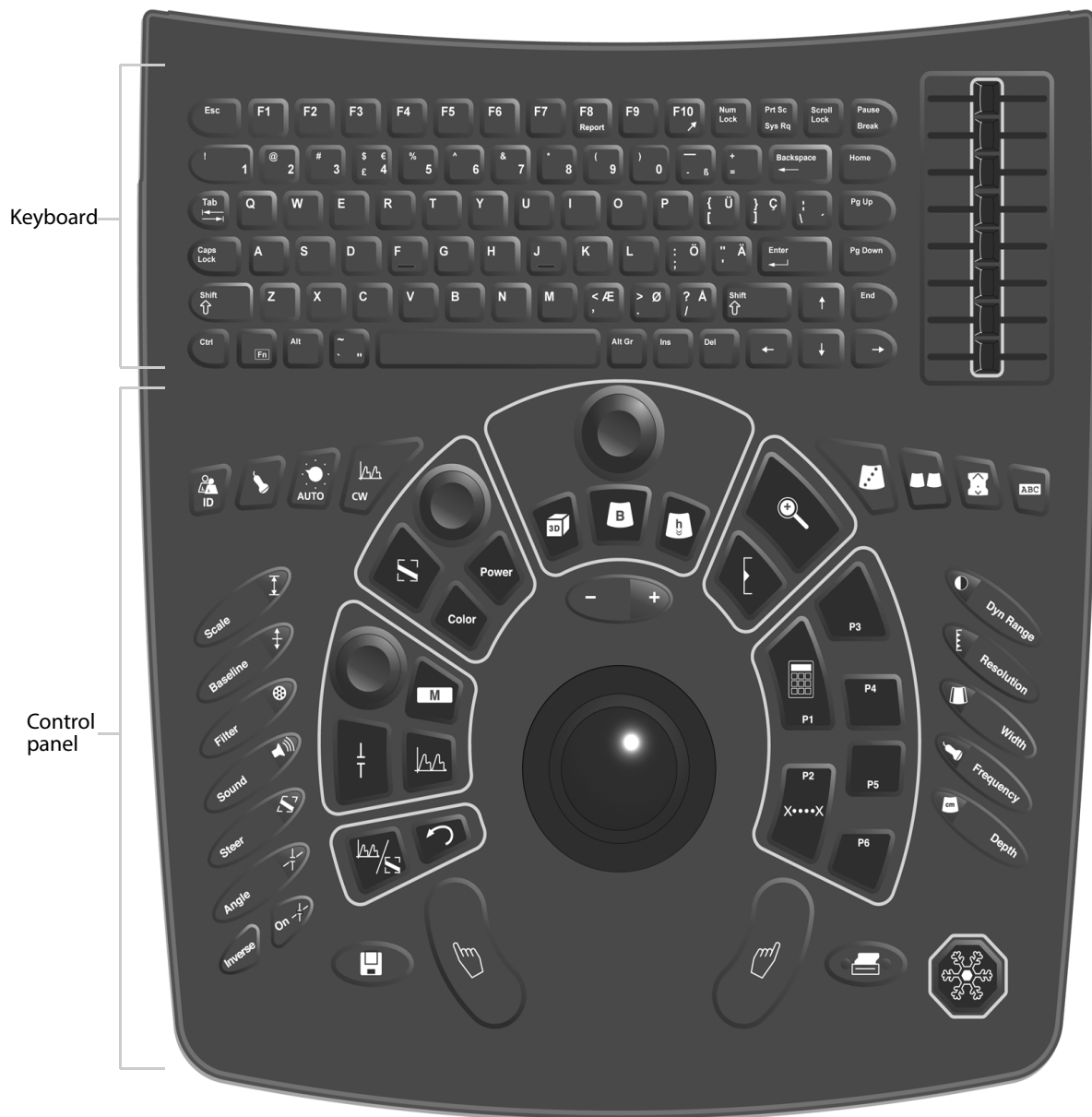


Figure 3-1. The keyboard panel.

The Control Panel

Each control key has a backlit label that is visible when the functionality of the key is available. Keys turn green to indicate an active function.

There are several types of control keys, categorized according to how you use them.

Key types

Key Type	Example	Action
Trackball	Trackball	Rolling the trackball moves a cursor on the monitor.
+/-	[+/-]	Pressing the key adjusts a value or moves a cursor stepwise up or down (pressing the + side increases the value, pressing the - side decreases it). A long press repeats the action.
Press	[Color]	Pressing the key activates a function or turns something on or off. Sometimes a long press (at least one second) is required to turn it off.
Dial	[Gain]	Turning adjusts the gain. Turn clockwise to increase the gain, turn counterclockwise to decrease the gain.
Rocker	[Scale ↑]	Same as for the [+/-] key (the + side being closest to the trackball, the - side furthest away from the trackball).
Slider	TGC controls	Moving the slider adjusts a value continuously.

Table 3-1. Control key types.

Press (long) In some situations, pressing a key normally and pressing it for a longer time (at least one second) have different effects. The longer presses are indicated in this user guide by *press (long)*. A long press usually turns off a function.

The Trackball and Trackball Keys

The trackball, together with the **[Select]** keys and the **[+/-]** key, can be used to operate the on-screen controls.

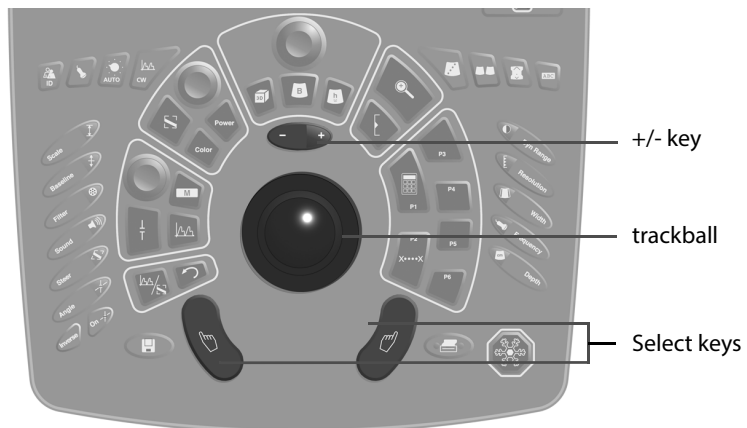



Figure 3-2. The trackball and trackball keys.

Trackball with Select Keys

Like a mouse The trackball and **[Select]** keys are used together like a PC mouse.

- Rolling the trackball moves the cursor and other objects on the monitor.
- Pressing one of the **[Select]** keys is like clicking a left mouse button.

There is no difference between the [] keys – use the one that is most convenient for you.

Adjusting the Trackball The trackball should move smoothly. If it is too tight or too loose, use the trackball adjustment tool (QA0228) to adjust it.









Figure 3-3. Using the trackball adjustment tool.

The tool is stored in the keyboard base below the keyboard panel. To adjust the trackball movement, place both ends of the tool into the small holes in the black ring of the trackball housing. Turn clockwise to tighten and counterclockwise to loosen the trackball. The ball can be removed completely for cleaning.

NOTE: Systems built or serviced after August 2016 may not require tool.

Select Keys

- Click When this user guide tells you to “click” something on the monitor, point at it and press [].
- The [] key can be used to select items from lists or menus on the monitor.
 - If the pointer points at a graphical control (such as the color box or one corner of it), clicking (pressing []) selects the control so that the trackball can move it. Click [] again to deselect the control or selection. Usually this means that the trackball can move the pointer again.
 - If the pointer points at an On/Off type on-screen control (such as **B Color**), pressing [] toggles the control off and on.

Click (long) Some operations require you to point at something on the monitor and then press (long) []. See “Press (long)” on page 53. This is indicated in this user guide as *click (long)*.

+/- Key

+/- key When the pointer points at a control on the monitor, pressing [+/-] changes settings such as values or sizes. A single press increases or decreases values stepwise. A continuous press causes the value to keep changing. Pressing the right side of the key increases a value; pressing the left side decreases it. If the key is pressed for more than ½ second, the action starts repeating.

When the cursor points at	Pressing the +/- key
a softkey whose value can be changed	increases or decreases the value in the field.
a graphical control (such as the color box)	increases or decreases the size of the control.
an item on a list	moves the cursor up or down the list (so you can select a value) Note: this does not apply to the setup menu.

Table 3-2. Effects of pressing the +/- key.

For more details about using the on-screen controls, see “Controls on the Monitor” on page 64.

Basic Keys

The Basic keys are arranged around the [] keys for easy access. These keys work in different imaging modes. For detailed information about using them, see the *Pro Focus 2202 Extended User Guide* (BB1279).

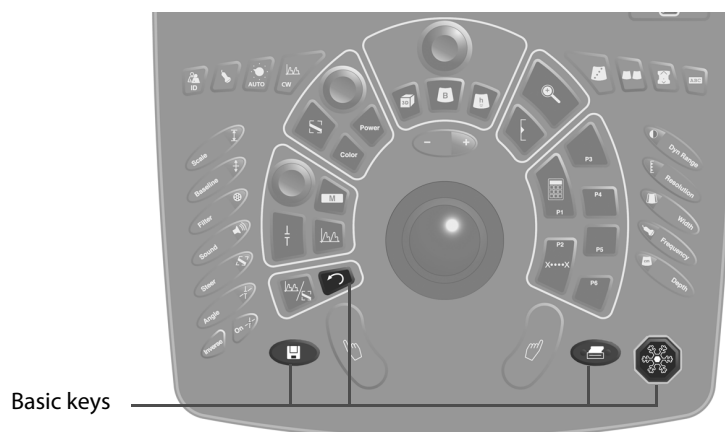


Figure 3-4. The Basic keys on the control panel.

Basic keys

Key	Name	Key Type	Action
[]	Freeze	Press	Freezes and unfreezes the image.
[]	Print	Press	Starts one of the user-defined documentation functions.
[]	Save	Press	Starts one of the user-defined documentation functions.
[]	Cancel	Press	Cancels the present procedure or action. Works in all imaging modes.

Please note that some of the documentation functions are only available after a patient ID has been entered.

Table 3-3. Function of the Basic keys on the control panel.

B-Mode Keys

The B-mode keys are centered above the trackball. For detailed information about using them, see the *Pro Focus 2202 Extended User Guide* (BB1279).

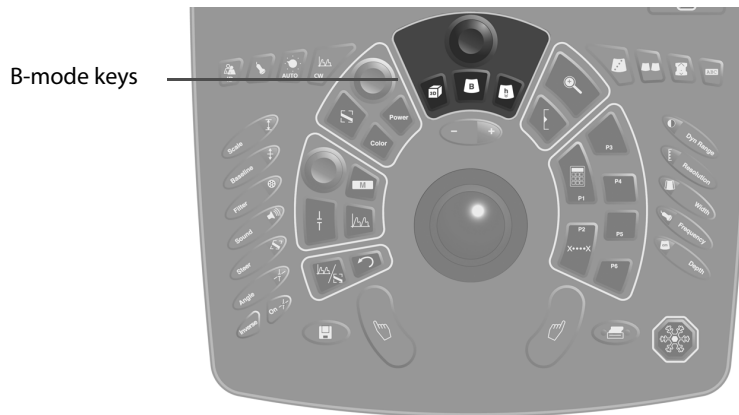


Figure 3-5. The B-mode keys on the control panel.

B-mode keys

Key	Name	Key Type	Action
[Gain]	B Gain/Cine	Dial	Adjusts the overall gain. If the image is frozen, turning activates the Cine scroll, but you can toggle it back to controlling the gain.
[B]	B-mode	Press	If other modes than B are active, return to B-mode only. If only B-mode is active, return to last mode combination.
[3D]	3D	Press	Turns 3D imaging on (if the 3D Pro Package is installed). If 3D is already active, it steps through the different 3D states. A long press turns 3D imaging off.
[h]	Harmonic	Press	Turns harmonic imaging on and off.

Table 3-4. Function of the B-mode keys on the control panel.

B-Mode Adjustment Keys

The B-Mode Adjustment keys are grouped on the right side of the control panel. For detailed information about using them, see the *Pro Focus 2202 Extended User Guide* (BB1279).

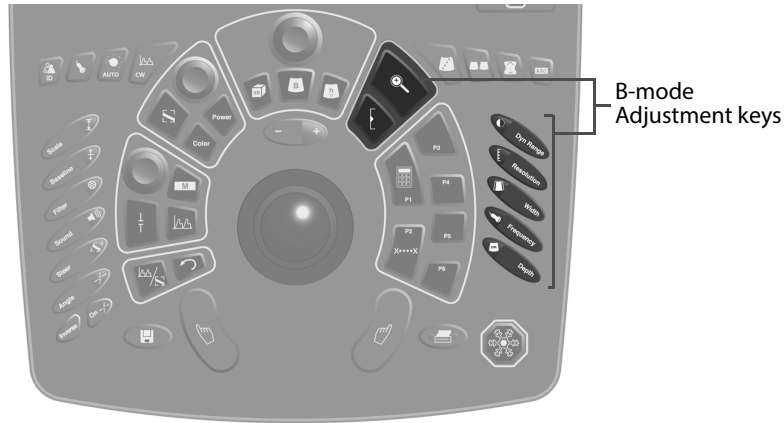


Figure 3-6. The B-mode Adjustment keys on the control panel.

B-mode
Adjustment
keys







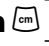
Key	Name	Key Type	Action
[]	Focus	Press	Selects the Focus indicator on the monitor so you can use the trackball and [+/-] keys to adjust the focus.
[]	Zoom	Press	Turns on the zoom box. Press again to zoom in. A third press resets the zoom area. A long press turns zoom off if it is on.
[Dyn Range ]	Dynamic Range	Rocker	Adjusts the B-mode dynamic range of the image.
[Resolution ]	Resolution	Rocker	Increases the resolution (decreases the imaging frame rate) or vice versa when you are imaging in B-mode or Color mode.
[Width ]	Width	Rocker	Adjusts the sector width. Decreasing the width increases the frame rate.
[Frequency ]	Frequency	Rocker	Adjusts the main (B-mode) center frequency when you are imaging in B-mode only. If you are imaging in Color mode, adjusts the center frequency for those modes.
[Depth ]	Depth	Rocker	Changes the maximum depth (distance from the transducer) of tissue shown in the image. Pushing the + side <i>decreases</i> the depth (increases the magnification).

Table 3-5. Function of the B-mode Adjustment keys on the control panel.

Power and Color Mode Keys

The Power and Color Mode keys are placed to the left above the trackball. For detailed information about using them, see the *Pro Focus 2202 Extended User Guide* (BB1279).

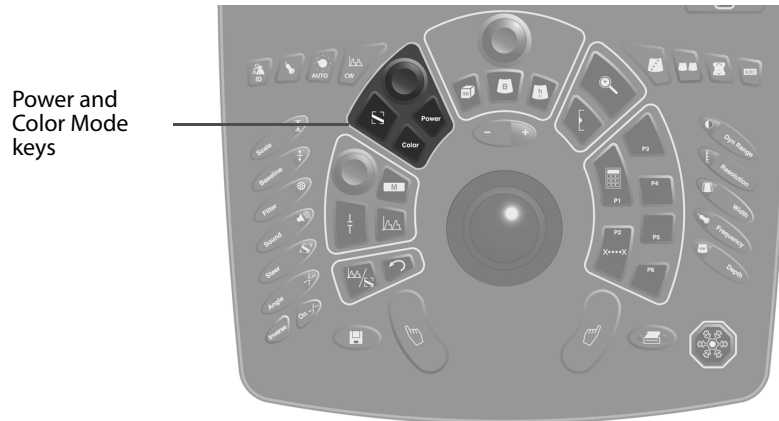


Figure 3-7. The Power and Color Mode keys on the control panel.

Power and
Color mode
keys

Key	Name	Key Type	Action
[Gain]	Color/Power Gain	Dial	Adjusts CFM (Color) gain if Color mode is active or Power gain if Power mode is active.
[Power]	Power Mode	Press	Turns Power mode (power Doppler) on and off. The trackball and [+/-] key are assigned to the color box when Power mode is turned on.
[Color]	Color Mode	Press	Turns Color mode (color Doppler, CFM) on and off. The trackball and [+/-] key are assigned to the color box when Color mode is turned on.
[S]	Color Box	Press	Selects the color box (the same as clicking inside the color box on the monitor). If the color box is already selected, pressing the [S] key cancels the selection. The trackball and [+/-] key are assigned to the color box when color box is selected.

Table 3-6. Function of the Power and Color mode keys on the control panel.

Doppler and M-Mode Keys

The Doppler and M-Mode keys are placed to the left of the trackball. For detailed information about using them, see the *Pro Focus 2202 Extended User Guide* (BB1279).

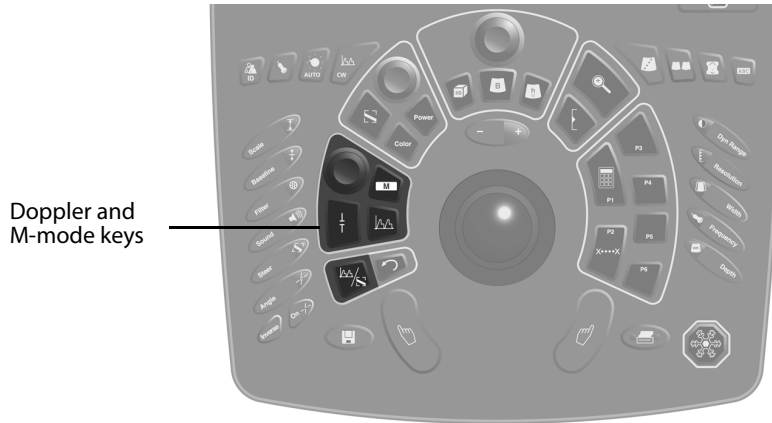


Figure 3-8. The Doppler and M-mode keys on the control panel.

Doppler and M-mode keys



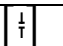
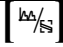
Key	Name	Key Type	Action
[Gain]	M/Doppler Gain	Dial	Adjusts M gain if M-mode is active or Doppler gain if Doppler mode (spectral Doppler) is active.
[]	Doppler Mode	Press	Turns Doppler mode on and off. The trackball and [+/-] key are assigned to the Doppler gate when Doppler mode is turned on.
[]	M-Mode	Press	Turns M-mode on and off. The trackball and [+/-] key are assigned to the M-line when M-mode is turned on.
[]	D/M Cursor	Press	In B-mode, turns on the Doppler gate so that you can position it before you activate Doppler mode. Press again to deselect the gate. A long press removes the gate.
[]	Update	Press	In combination modes, with one mode frozen, changes which mode is imaging and which is frozen. A long press unfreezes any frozen modes in the selected view.

Table 3-7. Function of the Doppler and M-mode keys on the control panel.

Doppler Adjustment Keys

The Doppler Adjustment keys are grouped on the left side of the control panel. For detailed information about using them, see the *Pro Focus 2202 Extended User Guide* (BB1279).

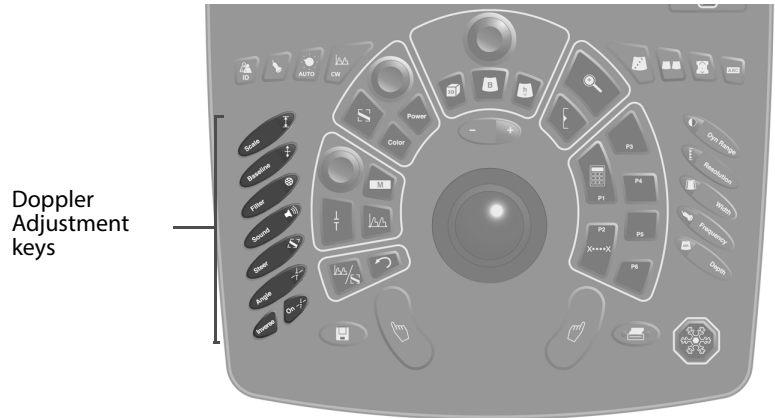


Figure 3-9. The Doppler Adjustment keys on the control panel.

Doppler Adjustment keys

Key	Name	Key Type	Action
[Scale ↕]	Scale	Rocker	Changes the scale/range/PRF in PW Doppler, Color mode and/or Power mode. If PW is active and Color mode or Power mode is active, the [Scale ↕] key adjusts the PW mode. In this case, use the on-screen controls to change the scale for Color mode/Power mode.
[Baseline ⇄]	Baseline	Rocker	Moves the baseline of Doppler mode and/or Color mode. If both modes are active, moves only the Doppler mode spectrum baseline.
[Filter 🌀]	Wall Filter	Rocker	Adjusts the wall filter. If both Doppler and Color modes are active, moves only the Doppler mode spectrum baseline.
[Sound 🔊]	Sound	Rocker	Adjusts the Doppler audio (speaker) volume and the video (speaker) volume during playback.
[Steer ↻]	Steer	Rocker	Changes the beam angle in Color mode, Power mode and PW Doppler.
[Angle ↻]	Angle	Rocker	Changes the Doppler correction angle.
[↻ On]	Angle On	Press	Turns the correction angle on or off.
[Inverse]	Inverse	Press	Inverts the spectrum (Doppler mode) or color scale (Color mode). If both Doppler mode and Color mode are active, inverts the spectrum only.

Table 3-8. Function of the Doppler Adjustment keys on the control panel.

User-Defined Keys

The user-defined keys are placed on the right side of the trackball. By default, the [P1] key is set up to open the **Calculations** menu, and the [P2] key is set up to open the **Measurements** menu.

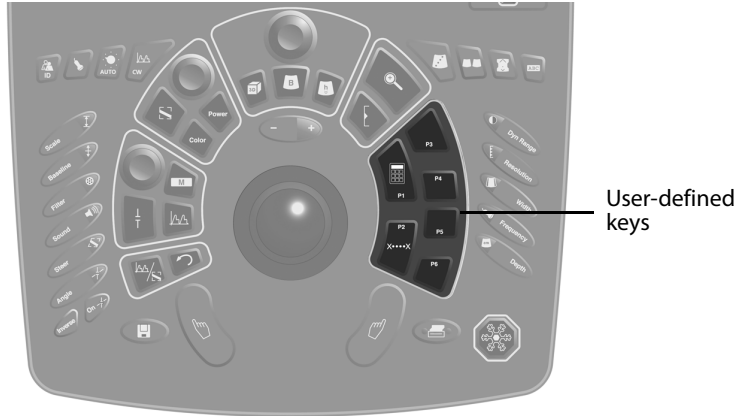


Figure 3-10. The user-defined keys on the control panel.

User-defined keys

Key	Name	Key Type	Action
	Calculation	Press	Opens the Calculations menu on the monitor.
	Measure	Press	Opens the Measurement menu on the monitor. If the menu is already open, starts the measurement at the top of the first open measurement group.
	User-defined	Press	The function of each of the 4 user-defined keys is defined within each Pro Package. Their function depends on which Pro Package is being used. For information about assigning functions to these keys, see the <i>Pro Focus 2202 Extended User Guide</i> (BB1279).

Table 3-9. Function of the user-defined keys on the control panel.

Image Annotation Keys

The Image Annotation keys are grouped on the top right side of the control panel. For detailed information about using them, see the *Pro Focus 2202 Extended User Guide* (BB1279).

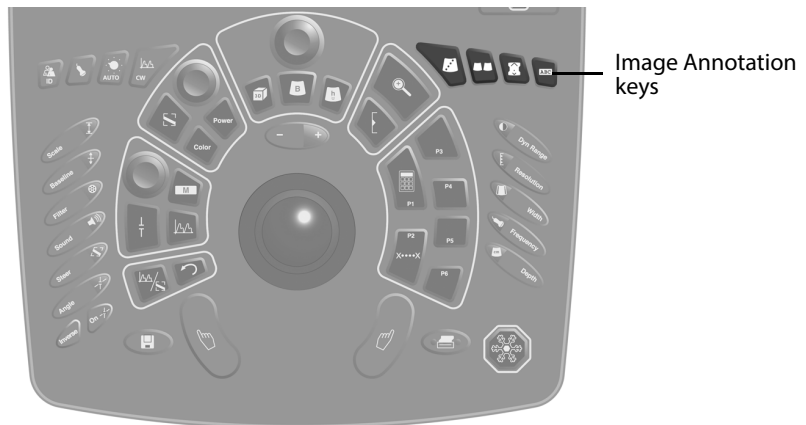


Figure 3-11. The Image Annotation keys on the control panel.

Image Annotation keys

Key	Name	Key Type	Action
	Puncture	Press	Turns a puncture or biopsy line on or off.
	Split Screen	Press	Splits the screen so two views are displayed. If the screen is already split, pressing shifts between which view is active. A long press turns split screen off if it is on.
	Bodymark	Press	Opens the Bodymark window so you can place a bodymark on the image. Press to close the Bodymark window again. A long press removes bodymarks.
	Label	Press	Opens the Label window so you can add text or arrows to the image. A long press removes all labels and arrows.

Table 3-10. Function of the Image Annotation keys on the control panel.

Miscellaneous Keys

The keys on the top left side of the control panel have varied functions.

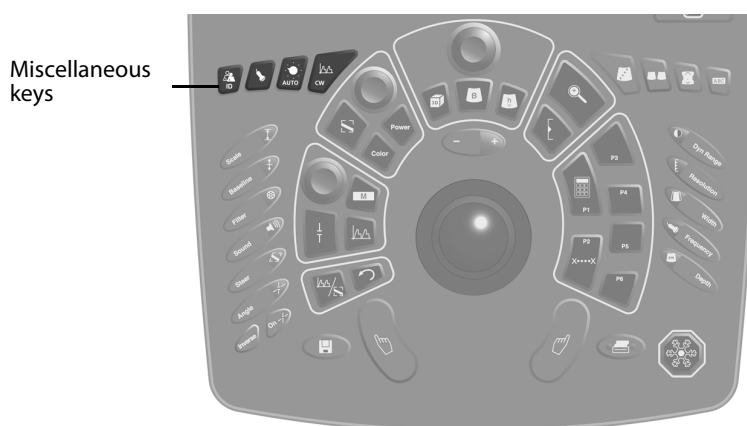


Figure 3-12. The Miscellaneous keys on the control panel.

Miscellaneous keys

Key	Name	Key Type	Action
	ID	Press	Opens the Patient window so you can enter data about a new patient. A long press ends the current examination.
	Transducer/Pro Package	Press	Opens the Pro Package menu on the monitor so you can choose a transducer, a Pro Package or preset.
	Auto	Press	Adjusts parameters or resets parameters to factory default.
	CW Doppler	Press	Turns CW Doppler on and off. The trackball is assigned to the CW Doppler line when CW Doppler mode is turned on.

Table 3-11. Function of the Miscellaneous Control keys.

TGC Sliders

The 8 TGC sliders to the upper right of the keyboard are used to adjust the Time Gain Control at various depths in the image.



Figure 3-13. The TGC sliders on the keyboard panel.

Each slider controls 1/8th of the depth of the displayed imaged area. The top slider affects the top of the image and the bottom slider the bottom. (With 360° transducers, the top slider adjusts the part of the image that is most central – that is, closest to the transducer. The bottom slider adjusts the part of the image farthest from the transducer.)

Move the sliders to the right to increase gain. Move the sliders to the left to decrease gain.

Each slider is related to a part of the monitor rather than a part of the tissue being imaged. Therefore, if you move, resize or invert the image area, you may have to use the sliders to readjust the TGC in order to get the best image.

Keyboard

In addition to the control keys, the keyboard panel contains an alphanumeric keyboard for entering information into the system.

The [F1] to [F10] keys are user-definable keys. The [F8] and [F10] keys have default functions assigned:

F8 and F10

Key	Name	Default Function
[F8]	F8	Displays a report. A report is defined for each Pro Package. The content of the report will vary, depending on the Pro Package and the measurements you have made.
[F10]	F10	Displays an arrow on the image and allows you to add text to the image.

Table 3-12. Default function of the F8 and F10 keys.

Controls on the Monitor

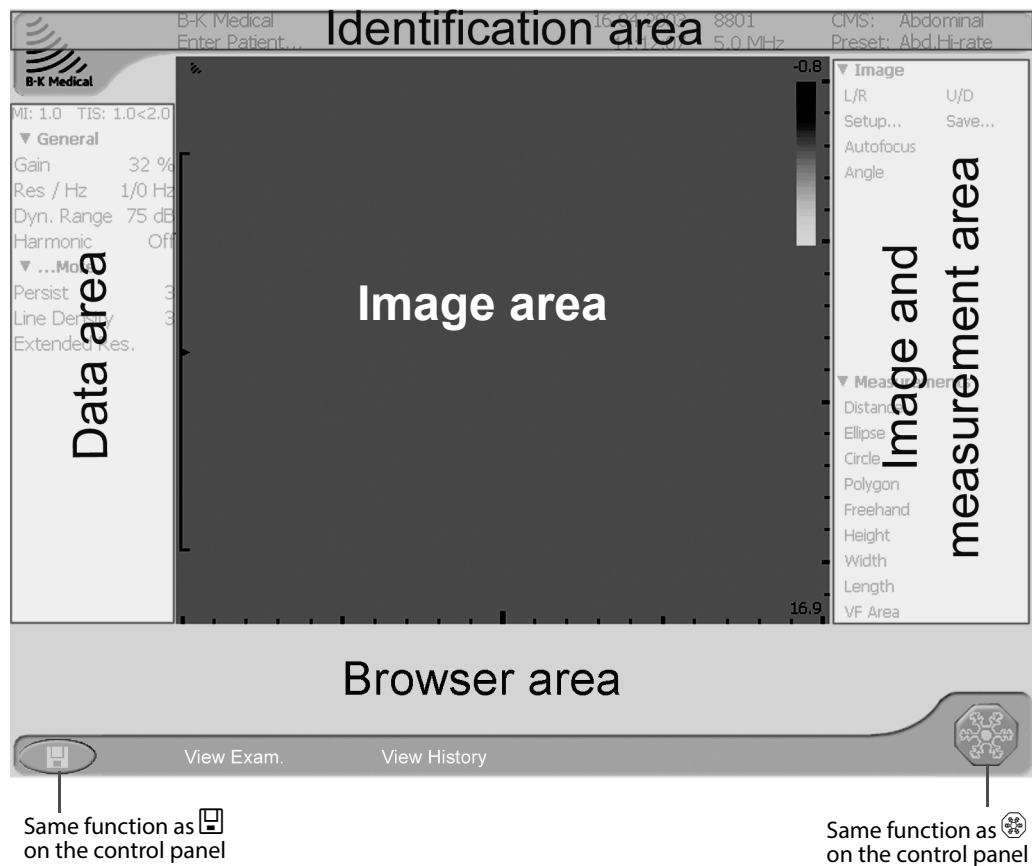


Figure 3-14. The parts of the display on the monitor. The layout shown here may be different from the one on your monitor.



- | | |
|----------------------------|--|
| Image area | <ul style="list-style-type: none"> The Ultrasound Image area is in the center. This area displays the current imaging and/or frozen views. Graphical on-screen controls such as the zoom box, color box and focus indicator are located in this area. |
| Identification area | <ul style="list-style-type: none"> The Identification area is at the top. This contains identifying information about the hospital and patient, the date and time, the transducer type and imaging frequency (and imaging plane, for a biplane transducer), and the name of the Pro Package and preset. |
| Data area | <ul style="list-style-type: none"> The Data area is on the left side. It displays data that must be documented with the image. Information displayed in this area is automatically updated. The Data area has a data section at the top that contains the relevant settings for image parameters. It has a measurement section at the bottom. The measurement section displays the results of current measurements you have made. |
| Image Documentation area | <ul style="list-style-type: none"> The Image area, the Identification area and the Data area make up the Image Documentation area and will appear in documentation (except for the Pro Package and preset names). |
| Image and Measurement area | <ul style="list-style-type: none"> The Image and Measurement area is to the right. You may need to click Image to see the image tools or click Measurements to see the measurement tools. This area will not appear in documentation. |

- Browser area • The Browser area is at the bottom. Small pictures (thumbnails) of the clips and images stored in the system are displayed across the bottom of the monitor. This area will not appear in documentation.

Point and Click

Using the trackball and trackball keys to interact with the controls on the monitor is intuitive and easy. In this user guide, certain terms are used consistently to tell you what to do.

Terminology

- Point at Use the trackball to move the pointer to a control. When a control is *pointed at*, you can adjust it with the [+/-] key but you cannot move it. To move a graphical control, you must select it.
- Click (select) Point at an on-screen control and press . When a control is *selected*, moving the trackball resizes or moves it. (Pressing  again cancels the selection, making it possible to use the trackball to move the pointer.)
- Drag Use the trackball to move a cursor or graphical control.

When an on-screen control has the color	It is	You can
White	Available	Point at it or select it.
Orange	Pointed at	Adjust it with [+/-]. Select it.
Blue	Selected	Move it with the trackball. Adjust it with the pointer or [+/-]
Gray	Unavailable	(No action possible).

Table 3-13. Color coding of controls on the monitor.

For some actions, you do not need to click. You can just point and press the [+/-] key (see the following section).

- Moving **To move a graphical control on the monitor:**
 - 1 Click the control to select it.
 - 2 Drag the selected control to the position you want.
 - 3 Click to release the trackball for other uses.
- Changing size **To resize a graphical control on the monitor:**
 - Select or point at the control and press [+/-].

or

 - 1 Click a boundary of the control (such as a corner of a box or a boundary of the focus indicator) to select it.
 - 2 Drag the selected control part until the control is the size you want.
 - 3 Click to release the trackball for other uses.


Softkeys


The monitor contains many softkeys for functions whose value can be adjusted or changed. These softkeys appear as words or abbreviations on the monitor.

Softkeys with values

To select a value for a softkey:

- 1 Click the softkey.
A list of possible values appears.
- 2 Click the value you want.

You can also use [+/-] to scroll up and down in the list and then press [].

NOTE: *Not all softkeys may be visible. Click the expand menu symbol  to see all items in a menu. To change the setup for how menus appear, see Pro Focus 2202 Extended User Guide (BB1279).*

Slider softkeys

Some softkeys have a slider that you move with the trackball.

On/Off softkeys

Some softkeys have only two values, On and Off.

To change the setting of an On/Off softkey:

- Click it.
- or
- Point at it and press [+/-] to toggle the value.

Small green light

A small green light next to the key indicates that it is set to On.

Foot Switch

The optional foot switch has 2 or 3 pedals. You can assign a function to each pedal, and the same pedal can have different functions for different Pro Packages. For information about assigning functions to the pedals, see *Pro Focus 2202 Extended User Guide* (BB1279).

To use the foot switch:

- Press the appropriate pedal.

NOTE: *Releasing the pedal does not operate or reverse the function.*

Remote Control

The wireless remote control UA1237 is an option available for the Pro Focus UltraView. It uses short wave radio waves to communicate with the system and control many of its functions.

The Remote Control and Its Functions

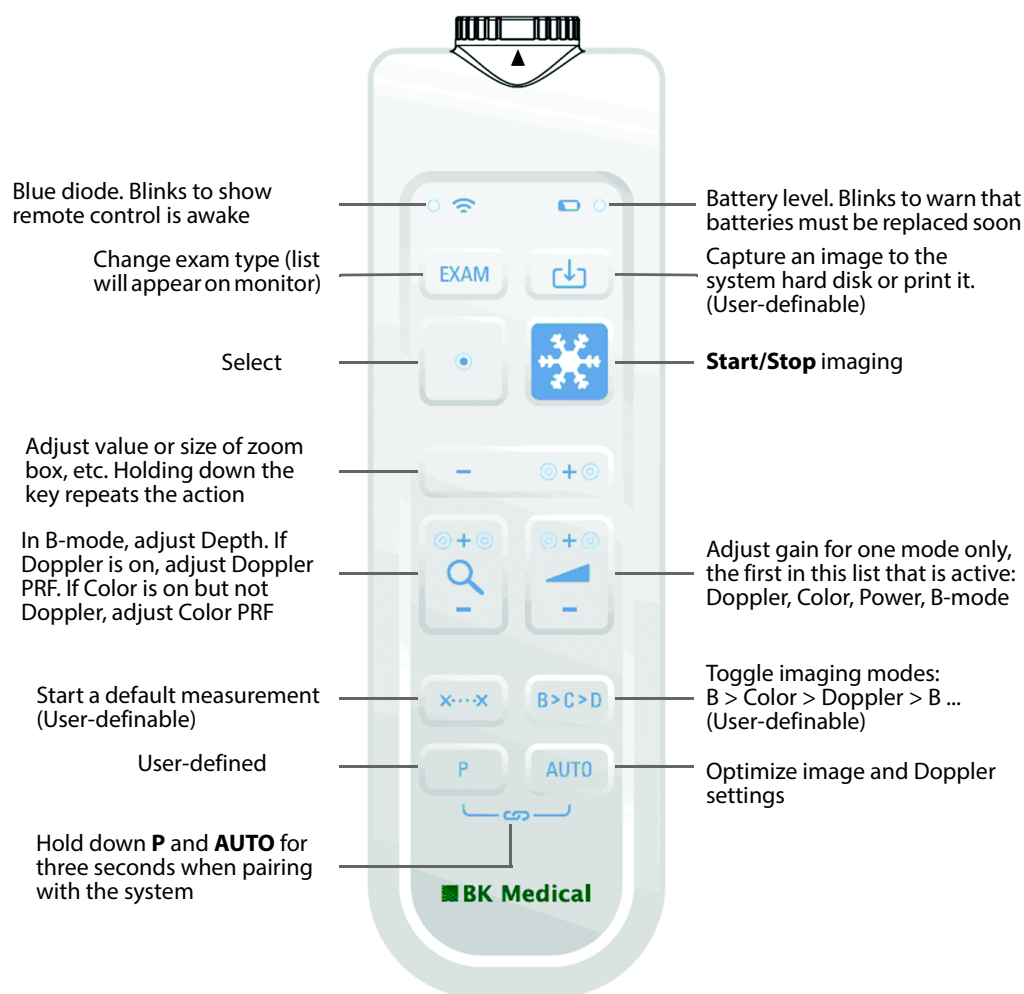


Figure 3-15. The remote control, with the location of its various controls.

The remote control uses short wave radio waves to communicate with the system.

Pairing the Remote Control with the System

- 1 Plug the Mini Bluetooth Adapter into a USB connector on the system. (Use the one that came with the remote control.)
- 2 Press any key on the remote control to wake it up.
- 3 Hold down the **P** and **AUTO** keys on the remote control for three seconds. The remote control emits two beeps.
- 4 Release the keys after the second beep. The blue diode starts blinking fast. A dialog box appears on the monitor. This may take up to two minutes.
- 5 Click **Accept** to pair the system with the remote control (indicated by its serial no.). The pairing process takes up to one minute.

When pairing is done, the remote control emits a beep and the blue diode blinks to show the remote control is awake.

NOTE: *The pairing process may time out. Try again if this happens.*

NOTE: *If the cursor starts drifting on the monitor, place the remote control in its holder and leave it for 30 seconds to recalibrate.*

If you have more than one remote control, be sure you know which system each is paired with; the system and remote control remain paired even if the remote control is taken away to be disinfected. Pairing lasts until you pair the remote control with another system or pair another remote control with the system. Therefore, it is important to keep track of which remote control is paired with which system.

Before each use, verify that the remote control is working correctly.

Sleep

If the remote control is not used or moved for 5 minutes, it goes to sleep to save battery power. To wake it up, press any key.

Mouse Function

You can use the remote control as an air-tracking mouse to operate controls on the system monitor.

To move the cursor, tilt the remote control; that is, move the front end up, down, or side-to-side. The mouse will not respond if you keep it level.

NOTE: *You do not need to point it at the monitor. The cursor responds to changes in the position of the remote control, not to the position itself.*


To select an object on the monitor, move the remote control so that the cursor is on the object, then press **Select**.

Replacing Batteries

To replace the batteries in the remote control:

- 1** Remove the battery cap.
- 2** Insert 2 batteries LR6 size AA 1.5 volt.
- 3** Screw the battery cap on tight until the arrow points to the area of the battery cap with a large gap between the ridges.

When the battery cap is screwed on tight, the remote control is watertight and can be immersed.

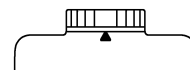

Cleaning and Disinfection

For details of cleaning and disinfecting the remote control, see *Care, Cleaning & Safety*.

Transducers

- Control button** Some transducers have a control button that you can press to *Start* or *Stop* imaging (freeze frame).
- Long press** You can set up your system so that a long press (more than one second) starts one of the user-defined documentation functions, such as capturing an image. See the *Pro Focus 2202 Extended User Guide* (BB1279) for information about setting up user-defined keys.
- If the transducer has more than one control button, you can set up a user-defined long press function for each control button.
- Each time the control button is pressed, a ‘beep’ is emitted.

Chapter 4

Technical Guide

This chapter contains information about the electrical requirements and operating conditions for the system as well as important information about attaching other equipment and so on. Before you turn on the system, make sure that the installation has been approved by a qualified service technician or by hospital safety personnel. See Safety Precautions in Chapter 1, “General and Safety Information”.

Operating Environment

The Pro Focus can be operated safely within the following operating condition limits:

Condition	Limits
Ambient operating temperature	10°C to 40°C (50°F to 104°F)
Humidity	max. 85% RH non-condensing
Atmospheric pressure	between 700 hPa and 1060 hPa

Table 4-1. Operating condition limits.



WARNING Follow the guidelines in EN60601-1-1[1] when you connect the system to other equipment. See “General Safety Precautions” on page 12.



Explosion hazard

WARNING During use, the system must be at least 25 cm (10in) from the patient to minimize the risk of igniting explosive gases.

The system is not intended for use in potentially explosive environments and should be kept well away from flammable gases and liquids. It should not be used in oxygen-enriched atmospheres. Avoid excessive heat, dust and direct sunlight.

The Rear of the System

The rear of the system body (see Fig 4-1 and Fig 4-2) contains a large number of connectors for connecting different equipment.

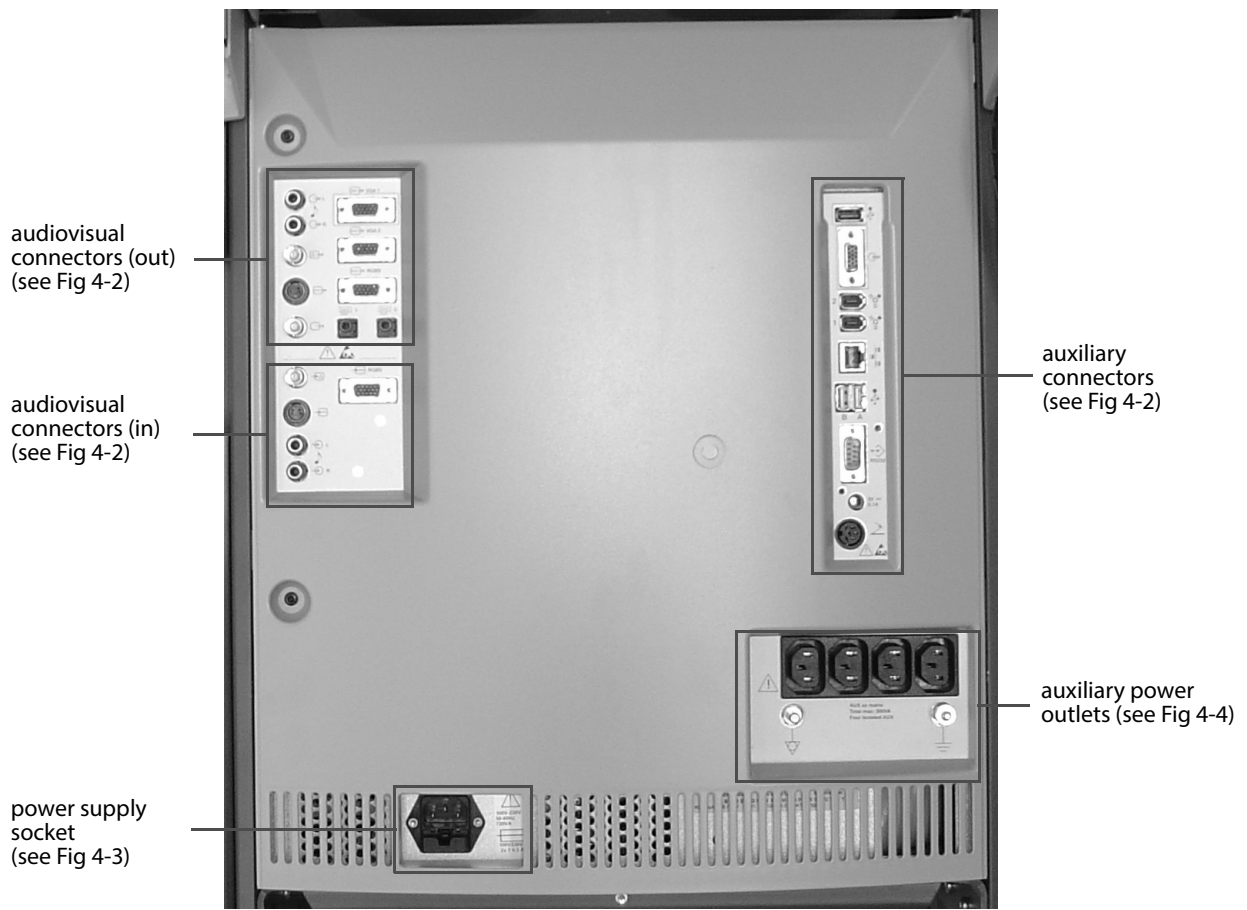


Figure 4-1. The system's rear panel.

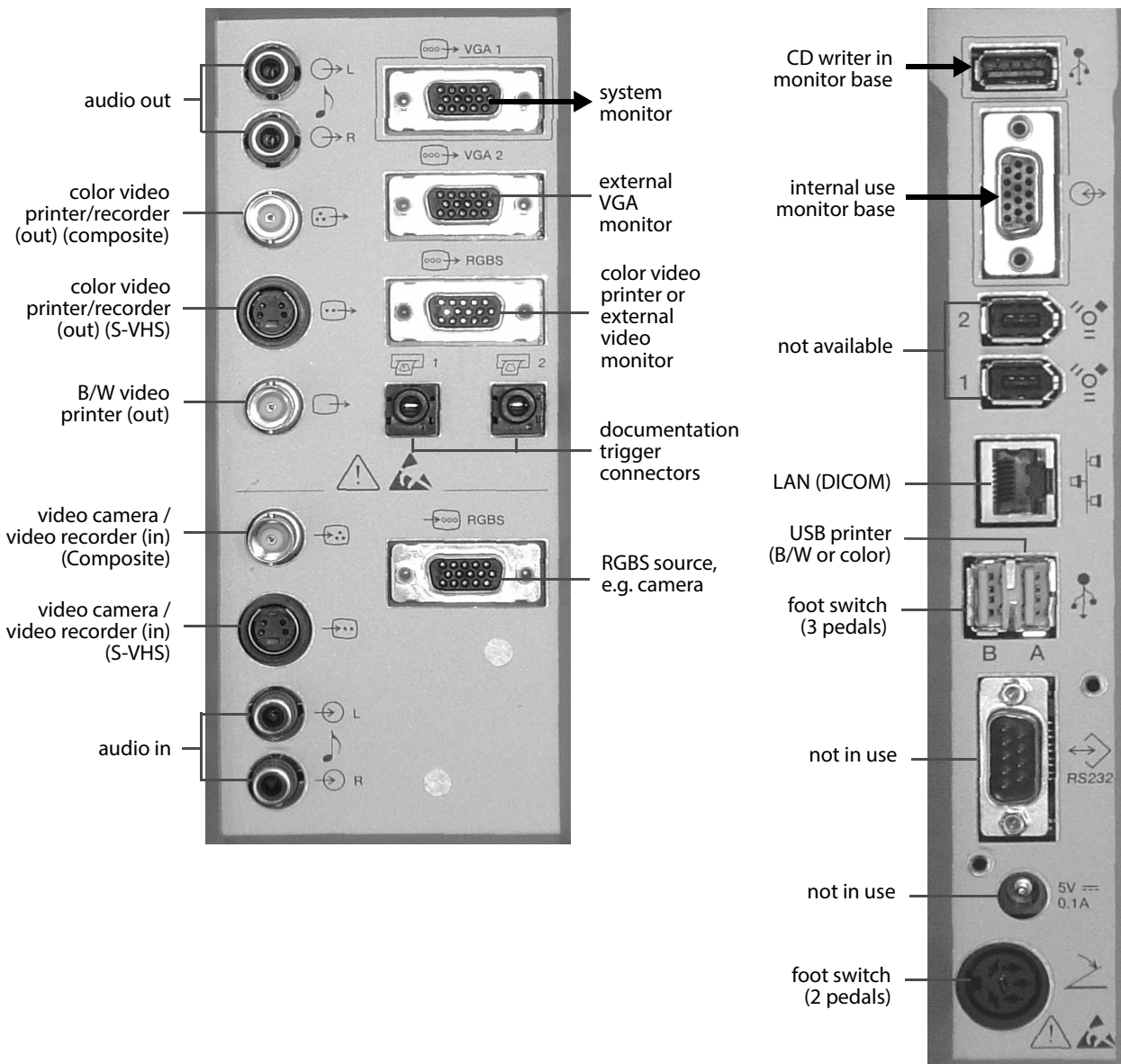


Figure 4-2. The peripheral connectors on the rear of the system.

Bold arrows indicate connections installed at the factory. Do not change these connections.

Cables Types and Lengths

Table 4-2 specifies the cables that were used to test the electromagnetic compatibility (EMC) of the system. EMC data for all transducers used with this system are in the Technical Data (BZ2100) section on the CD that accompanies this user guide. The CD also contains a list of accessories used in testing for EMC compliance.

To fulfill EMC requirements, do not attach cables to the system unless they are the same type as listed in the table and do not exceed the maximum length given in the table.


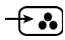

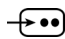





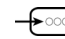










Symbol	Connector	Part number	Type and Maximum Length
	Color Composite Out	-	Shielded, 3 m
	Color Composite In	-	Shielded, 3 m
	B/W Composite Out	-	Shielded, 3 m
	S-VHS In	-	Shielded, 2 m
	S-VHS Out	-	Shielded, 2 m
	Audio In	-	Shielded, 2 m
	Audio Out	-	Shielded, 2 m
	VGA 2 Out	AO 1382	Shielded, 2 m
	RGSB Out	AO 1382	Shielded, 2 m
	RGSB In	AO 1382	Shielded, 2 m
	Documentation Trigger 1	EL4020	Shielded, 1 m
	Documentation Trigger 2	EL4020	Shielded, 1 m
	DICOM	Included in DP0925	Shielded, 3 m
	USB A	-	Shielded, 3 m
	USB B	-	Shielded, 3 m
	RS232	-	Shielded, 3 m
	ECG wire (on front panel)	AO0366	Shielded, 3.5 m

Table 4-2. List of cables used in testing for EMC compliance.

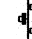
Monitors

The system monitor is connected to the  **VGA1** connector. The power to the monitor comes from one of the auxiliary power outlets. See page 78. You can use the  **VGA2** connector for an external VGA monitor.



Audio Input and Output

An audio Doppler signal is available from the  (**L** and **R**) connectors.


Speakers

The speakers are connected to an internal amplifier (built into the monitor base) and connected to the  connector at the top right of the rear panel via a multi cable.

Recording

To record an audio signal, connect the  connectors to a video recorder. To play a recorded signal back through the system speakers, connect the video recorder output to the  connectors.

Documentation Accessories

The optional digital B/W printer (EQ4100) which should be connected to the  port **A**.

If you install other documentation accessories, always follow the installation and verification instructions in the user guide that accompanies them. See “Connecting Other Equipment” on page 19.

Always use the cables specified in the 2202 Product Data sheet.

PAL/NTSC

The format for the video output that is sent to a video printer or video recorder can be either PAL or NTSC.

To set the format for video output from the system:

- 1 Make sure that the PAL/NTSC switch on the rear of the printer is set to the appropriate format.
- 2 Open the **General Setup** window.
- 3 Select the video output setting you want, and close the window.

Video Documentation Equipment

Table 4-3 gives an overview of video documentation connections. See also Table 4-2.


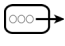
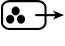













Equipment	Signal type	Connector	Comments
Video printer, B/W	Composite Out		
Video printer, Color	RGBS Out	 RGBS	The R, G, and B signal levels are 0.7Vpp/75Ω. The S (Sync.) signal level is TTL.
Video printer, Color	Composite Out		
Video recorder	Composite Out		
Video recorder	Composite In		For playback
Video recorder	S-VHS Out		
Video recorder	S-VHS In		For playback
Video monitor, B/W	Composite Out		
Video monitor, Color	RGBS Out	 RGBS	Only one piece of external equipment can be connected to the  RGBS connector.
Video camera	RGBS In	RGBS 	
Video camera	Composite In		
Video camera	S-VHS In		

Table 4-3. Overview of video documentation connections.

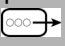Documentation Trigger Connectors

In order to use the foot switch or [] key to print the image on the monitor, connect  1 or  2 to the trigger input of a video printer.

USB Devices

USB
connector
under
monitor

Images can be saved to a USB storage device through the USB connector on the left side of the DVD drive.


External hard
drive



The power available from the USB connector on the left side of the DVD drive is limited. If you are using an external hard drive, attach only a data transfer cable to this connector. Do not use it for supplying power to the hard drive.



Caution: Do not attach 2 USB storage devices (for example a memory stick and an external hard drive) to the system at the same time.

Consult your BK Medical service representative if you have questions about connecting an external hard drive.

The DVD writer built into the monitor base is connected to the  connector at the top right of the rear panel.

 connector **A** on the back of the system is reserved for use with USB printers (black and white or color).  connector **B** on the back of the system is reserved for use with the optional 3-pedal foot switch.

NOTE: *The USB ports on the system are USB 2.0 ports. However, in order to get high-speed performance, the equipment you connect to them must be labeled “certified Hi-Speed USB”.*

Foot-Switch Connector

There are two connectors for the optional foot switches (see page 66), one for a 2-pedal and one for a 3-pedal foot switch (see Fig 4-2 on page 73).

Electrical Connections

Power Supply Cord

The system comes with a power supply cord that is specific to your region. The connector (see Fig 4-3) is on the lower left side of the rear panel.

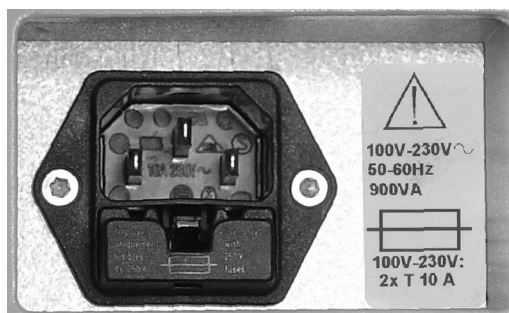


Figure 4-3. The connector for the power supply cord.

⚠
Use only
original
power
supply cord

WARNING Use only the power supply cord that comes with the system. Never use extension cords. The increased length of the cord will increase the resistance of the protective ground conductor and may increase the equipment's leakage current beyond an acceptable level.

The power supply cord for the USA has a hospital-grade 3-prong grounded power plug. Contact your local BK Medical representative if you need a different plug or if the cord is missing or damaged.

⚠
Don't
change
power plug

WARNING Never try to remove or change the plug on the power supply cord.

Cord Retaining Kit UA2204

The optional cord retaining kit UA2204 is attached to the power supply socket to ensure that the power supply cord cannot be accidentally disconnected from the system. The kit consists of two mounting screws and two brackets, each fitting a different type of power plug.

To use the cord retainer, lift the bracket up when you plug in the power supply cord. Then push it down over the plug so it holds the cord.

Connecting Other Equipment

For important information about connecting other equipment safely, see “Connecting Other Equipment” on page 19.

Lithotripter


Special considerations apply if you are using a lithotripter with the system. See “Using the System with a Lithotripter” on page 82.

⚠
Equipment
that uses line
voltage

WARNING When you connect other equipment that uses the line voltage (such as a video printer, video recorder, endoscopic camera control unit or other documentation device) to the system, the connections must follow the guidelines given in EN 60601-1-1 [1]. If in doubt, contact your local BK Medical representative.

Using the RS232 and LAN Connectors


You can connect other equipment to the RS232 and LAN connectors on the back of the system (see Fig 4-2). You can connect a HistoScanning¹ workstation to the LAN connector.

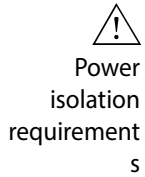

WARNING Electrical equipment that is connected to the RS232 and LAN connectors must get its power from the system's isolated auxiliary power outlets (see below) if it does not comply with EN/IEC 60601-1 [2]. In particular, if you connect an IEC 60950 apparatus, such as a PC, it must be outside the patient area and get its power from the system.

Auxiliary Power Outlets

To minimize leakage currents, plug all other equipment into the isolated auxiliary power outlets on the bottom right of the rear panel. There are 4 auxiliary outlets, as shown in Fig 4-4.

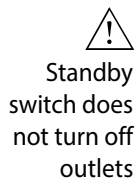

Caution: The 4 auxiliary power outlets are still "live" even when the standby switch on the front is turned off. In order to remove voltage from these outlets, you must unplug the power supply cord from the wall outlet.




Figure 4-4. Auxiliary power outlets.

The voltage and frequency of these outlets is the same as from the power supply cord. The thermal fuses limit the power that can be drawn from the outlets to a safe level, but the available total output from the auxiliary outlets must not exceed 80 VA if the black and white printer EQ4071 is attached to the system. (Actually, the total output is limited to 350 VA, but the system monitor, using 150 VA, is permanently plugged into one of the outlets, and the USB black and white printer uses 120 VA. This leaves 80 VA for other accessories.)

Additional Protective Ground




An additional protective ground can be connected to the  terminal below the auxiliary power outlets (see Fig 4-4).

1. HistoScanning is not market cleared by the FDA for sale in the USA and not licensed by Health Canada.

Potential Equalization

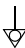

The potential equalization terminal  below the auxiliary power outlets (see Fig 4-4) is connected to the system chassis. It can be connected to corresponding terminals on other equipment to eliminate potential differences.

Dismantling the System for Transportation

The keyboard panel, keyboard base, monitor and monitor base can be removed to make it easier to transport the system.



Figure 4-5. Removing the keyboard base.

Keyboard Base and Panel

To remove the keyboard base and panel:

- 1 Move the keyboard base to the top position.
- 2 Unplug the keyboard panel from the front of the system.
- 3 Lift and remove the keyboard panel (see Fig 4-5).
- 4 Remove the 2 sliding covers from the system.
- 5 Loosen the black screws under each side of the keyboard base.
- 6 Press the black release handles and lift the keyboard base out of the groove.
- 7 Replace the sliding covers.

To replace the keyboard base and panel:

- 1 Remove the 2 sliding covers from the system.
- 2 Slide the keyboard base back into the grooves.
- 3 Tighten the black screws.
- 4 Reconnect the keyboard panel to the front of the system.
- 5 Replace the keyboard panel.

Monitor

Removing and Replacing an LCD Flat Screen Monitor

To remove the LCD flat screen monitor from the base:

- 1 Make sure that the system wheels are locked.
- 2 Unplug the power cord and VGA connector from the back of the monitor.
- 3 Loosen the 2 thumb screws on the bracket to expose approximately 5 mm of thread. If metal screws have replaced the thumb screws (in order to secure the flat screen monitor from theft), use an Allan key to loosen these screws.



Figure 4-6. Loosen the thumb screws to remove the monitor from the base.

- 4 Slide the monitor off the base.

To replace the LCD flat screen monitor:

- 1 Make sure that the system wheels are locked and that the sliding covers are removed.
- 2 Make sure that the two thumb screws on the bracket are loosened exposing approximately 5 mm of thread.
- 3 Slide the monitor onto the base, making sure that it is securely and completely on the base and is not crooked.
- 4 Tighten the thumb screws until the flat screen is secured, or tighten the metal screws using an Allan key.



WARNING If the thumb screws are not tightened correctly so that they secure the monitor, the monitor can fall off and injure you or others when you move the system.

- 5 Plug the power cord and VGA connector into the back of the monitor.

Monitor Base

Removing and Replacing the Monitor Base

To remove the monitor base:

- 1 Make sure that the system wheels are locked.
- 2 Move the monitor base to the top position.

- 3 Disconnect 2 monitor connections on the rear panel of the system: the **VGA1** connector and the power cord (see Fig 4-2).
- 4 Disconnect 2 monitor base connections (internal connection and USB for DVD writer) at the top right of the rear panel (see Fig 4-2).
- 5 Remove the 2 sliding covers from the system (see Fig 4-5).
- 6 Use a TORX 30 screwdriver or an NV4 Allen key to remove the 2 screws on the hook bars (see Fig 4-7) that are attached to the back of the monitor base.
- 7 Loosen the 2 black screws on the hook bars. (Do not remove them.)
- 8 Hold the monitor base under the DVD drive, and lift the hook bars carefully out of the grooves.

To replace the monitor base:

- 1 Make sure that the system wheels are locked and that the sliding covers are removed.
- 2 Slide the hook bars of the monitor base into the grooves (see Fig 4-7). Make sure that the hooks are in place at the top of each groove so that the monitor base is supported correctly.
- 3 Use a TORX 30 screwdriver or an NV4 Allen key to screw the 2 screws into the hook bars.

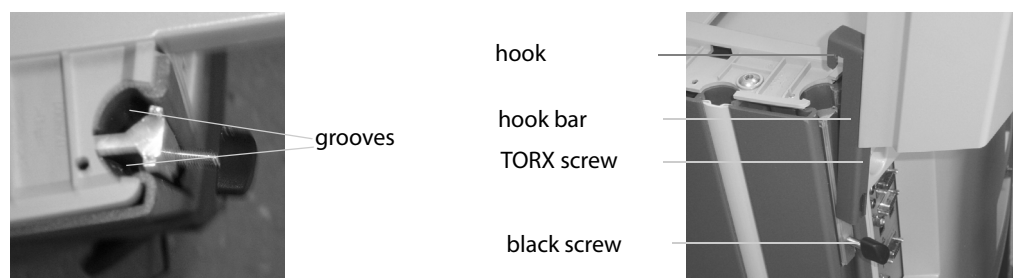


Figure 4-7. Hooks and grooves for holding monitor base.

- 4 Tighten the black screws on the hook bars.
- 5 Replace the sliding covers.
- 6 On the rear of the system, reconnect the 2 connections to the monitor (**VGA1** and auxiliary power outlet) and the 2 connections to the monitor base (internal connector and top USB connector). (See Fig 4-2.)

HistoScanning¹

The scanner units to be used with HistoScanning™ systems from AMD have an extra connector on the back of the system, on the lower left side. It is used to take raw data, before it has been converted to an image on the monitor, out of the system.

To use the Pro Focus for HistoScanning, you must have a HistoScanning license installed.

1. HistoScanning is not market cleared by the FDA for sale in the USA and not licensed by Health Canada.

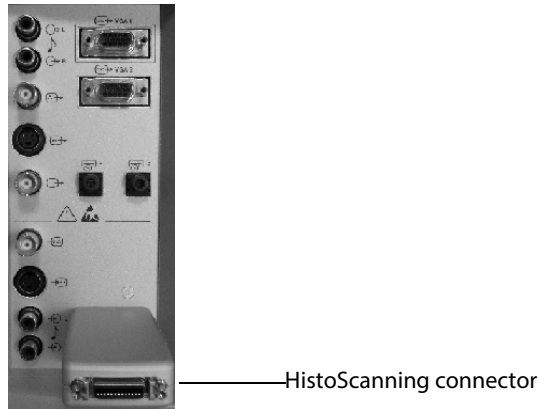


Figure 4-8. Connector for HistoScanning.

Using the System with a Lithotripter

You must have a license installed in order to use a lithotripter with the system.

You must always follow instructions in the manufacturer's user guide for the lithotripter.

Power Connections

If you are using the system with a lithotripter, make sure that the lithotripter complies with the guidelines in EN/IEC 60601-1 [2]. Connect both the system and the lithotripter directly to independent wall outlets.

Connecting the System to the Lithotripter

You can use the RS232 connector on the back of the system (see Fig 4-2) to connect a lithotripter.

You must follow the connection instructions in the manufacturer's user guide for the lithotripter system. The connections must follow the guidelines given in EN 60601-1-1 [1].

References

- [1] EN 60601-1-1:2001 *Medical electrical equipment – Part 1-1: General requirements for safety. Collateral standard: Safety requirements for medical electrical systems.*
- [2] EN/IEC 60601-1:1990+A1:1993+A2:1995+A13:1996 *Medical electrical equipment. Part 1: General requirements for safety.*
EN/IEC 60601-1:2006 3rd Ed. *Medical electrical equipment. Part 1: General requirements for basic safety and essential performance.*

Appendix 1

Specifications and Indications for Use

Specifications for the system are included in the Product Data sheet for the system, which forms part of this user guide.

Indications for Use

The system is intended for use by qualified physicians for ultrasound evaluation. Specific clinical applications and exam types include the following types of imaging:

- Abdominal
- Adult cephalic
- Cardiac (not for direct use on the heart)
- Fetal
- Intraoperative
- Intraoperative neurological
- Musculoskeletal conventional
- Musculoskeletal superficial
- Neonatal Cephalic¹
- Obstetrics
- Pediatric
- Peripheral vascular
- Small organ
- Transrectal
- Transurethral
- Transvaginal

Indicated uses are different for different transducers. The Product Data sheet for the system contains a table listing the indicated uses for each transducer that can be used with the system.

Contraindications

The Pro Focus 2202 ultrasound system is not intended for ophthalmic use or any use causing the acoustic beam to pass through the eye.

1. Neonatal cephalic imaging on the Pro Focus 2202 UltraView has not been CE approved or licensed by Health Canada.

 BK Medical ApS, Mileparken 34, 2730 Herlev, Denmark. T +45 4452 8100 F +45 4452 8199



powered by analogic 

BK Ultrasound
8 Centennial Drive
Peabody
MA 01960
USA
T +1 978-326-1300
bkultrasound.com

**North America
Sales & Service**
BK Ultrasound
8 Centennial Drive
Peabody
MA 01960
USA
T +1 978-326-1300
F +1 978-326-1399
bkultrasound.com

**Europe and Rest of World
Sales, Service & Design Center**
BK Ultrasound
Mileparken 34
2730 Herlev
Denmark
T +45 4452 8100
F +45 4452 8199
bkultrasound.com

**Asia
Sales & Service**
Analogic Medical Equipment
(Shanghai) Co., Ltd.
1377, Lan Dian Road
Pu Dong New District
Shanghai
China 201132
T +86 21 2089 0333
bkultrasound.com

**Canada
Design Center**
Ultrasonix Medical Corporation
130-4311 Viking Way
Richmond
BC V6V 2K9
Canada
T +1 866-437-9508
F +1 604-279-8559
bkultrasound.com