

Reproducibility Test Chart

1. Tare the instrument to zero.
2. Place the test weight on the pan. Record the reading in the column 'Full Scale' .
3. Remove the weight. Do not reset to zero! Record the reading under 'Zero'.
4. Repeat steps 2 and 3 until all lines are filled in.
5. In the last row the calculated standard deviation can be put in. Therefore use a computer program or an on-line calculator like: http://frankshospitalworkshop.com/equipment/documents/weighing_scales/equipment/standard_deviation_calculator.html

Measurement	Zero	Full Scale
1	X	
2		
3		
4		
5		
6		
7		
8		
9		
10		
11		X
Standard Deviation (SD)		

Manufacturer Serial number

Date Technician

Signature